



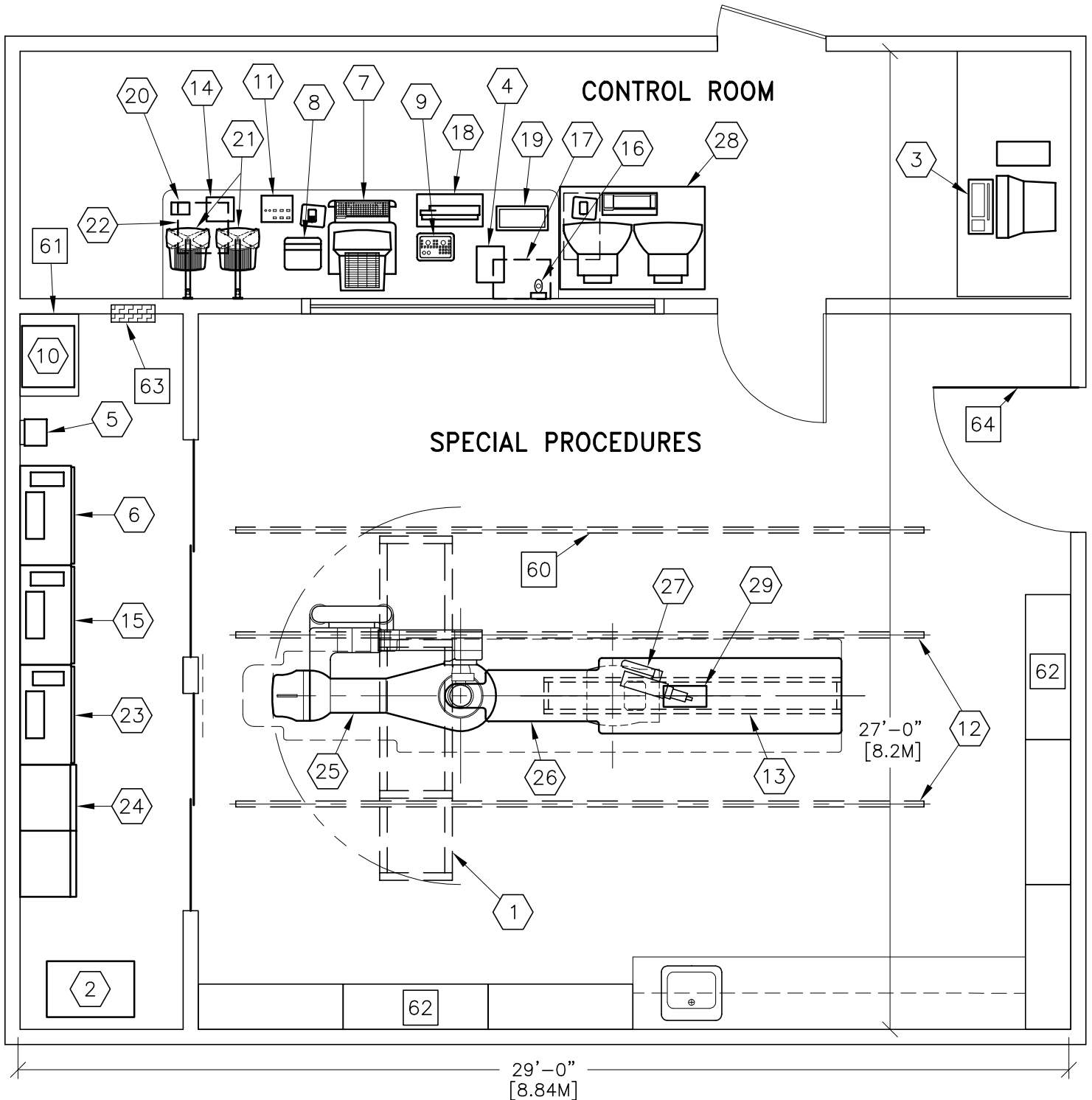

ADVANTX VMP/LCV+/OMEGA IV/DLX/GEMNET/DRYVIEW

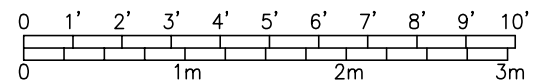

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	26'-0" x 20'-6" [7.92m x 6.24m]	9'-6" [2.89m]



SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 4-49

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	TWO TV MONITORS ON DUAL BOOM SUSPENSION WITH XT INBOARD MOUNTING	352 lbs	600 btu
②	1	WATER CHILLER	229 lbs	6143 btu
③	1	CARDIAC REVIEW STATION	110 lbs	750 btu
④	1	8mm DIGITAL TAPE ARCHIVE	17 lbs	51 btu
⑤	1	STEP DOWN TRANSFORMER	44 lbs	682 btu
⑥	1	DLX ELECTRONICS CABINET	590 lbs	5119 btu
⑦	1	DLX DIGITAL CONSOLE AND KEYBOARD	41 lbs	767 btu
⑧	1	DLX DIALOG DISPLAY	15 lbs	
⑨	1	DLX KEYPAD	4 lbs	
⑩	1	IDF ELECTRONICS	50 lbs	1706 btu
⑪	1	IDF CONTROL	2 lbs	
⑫	2	LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	68 lbs	
⑬	1	COUNTERBALANCED EYE AND THYROID SHIELD WITH LAMP	143 lbs	
⑭	1	DIAMENTOR CONTROL	8 lbs	
⑮	1	ADVANTX-E SYSTEM CABINET	590 lbs	3276 btu
⑯	1	BOLUS CHASE HANDSWITCH	2 lbs	
⑰	1	INJECTOR ELECTRONICS	37 lbs	320 btu
⑱	1	OPERATORS CONSOLE	15 lbs	194 btu
⑲	1	REMOTE CONTROL FOR INJECTOR	4 lbs	
⑳	1	DIAMENTOR LABEL PRINTER	4 lbs	
㉑	2	15 in. MONITOR ON WALL SUPPORT	35 lbs	204 btu
㉒	1	XR-70 VIDIO TAPE RECORDER UNDER COUNTER	24 lbs	
㉓	1	POSITIONER CABINET	859 lbs	3276 btu
㉔	1	ADVANTX VMP 100 GENERATOR	1358 lbs	5119 btu
㉕	1	LCV+ POSITIONER WITH IMAGE INTENSIFIER. (REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION)	1543 lbs	
㉖	1	OMEGA IV/V TABLE WITH ROTATING TOP	1300 lbs	614 btu
㉗	1	INJECTOR HEAD ON TABLE RAIL	15 lbs	
㉘	1	MAC-LAB CONSOLE, INCLUDES MONITORS AND KEYBOARD	341 lbs	3750 btu
㉙	1	REMOTE ACQUISITION UNIT	15 lbs	

ENVIRONMENT

Ambient Operation Temperature: 55-75° (F)
 Allowable Temperature Change: 15° (F) per hour
 Humidity: 20-80 percent non condensing
 Allowable Humidity Change: 10 percent per hour
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE 4-49

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
		<p>***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****</p>		
60		CABLE DRAPE RAIL.		
61		CUSTOMER SUPPLIED SHELF		
62		CATHETER CABINETS		
63		MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION <14.5 W X 24 H X 5.75 IN. D>		
64		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W X 83 IN. H [1118MM X 2108MM], CONTINGENT ON A 96 IN. [2438MM] CORRIDOR WIDTH		