

## Compatibility to Microsoft Security Patches

### Introduction

ViewPoint products run on Windows operating systems. Microsoft releases security patches at least every month to correct vulnerabilities. We test our products against these monthly releases to ensure compatibility. Starting from the month of February 2007, test results will be documented on this sheet. Patches and patch test results are listed according to the month of release. Test outcome can be 'Passed', 'Failed' or 'NA' – Not Applicable.

- Passed:* GE ViewPoint verified the compatibility of the patch against the ViewPoint software.
- NA:* The patch does not affect the ViewPoint software. These are patches for e.g. the Microsoft Office products.
- Failed:* The patch was tested and found not to be compatible with ViewPoint. The user must not install those patches.

The ViewPoint system could be infected by a computer virus. This infection can result in a loss of data, incorrect data or an inoperable ViewPoint system. The user has to ensure that regular updates of the operation system will be done. The installation of patches is in the responsibility of the user. We recommend that the user installs all patches which are marked as 'Passed' or 'NA'.

### Document Version

Version: 1.0                      November 09, 2009

### Test Platforms

The following operating systems were tested:

- Windows 2000 Professional (with Service Pack 4) /Win-2k-Prof-SP4
- Windows XP Professional (with Service Pack 3)/Win-XP-Prof-SP3
- Windows 2003 Server (with Service Pack 2)/Win-2k3-Srv-SP2
- Windows Vista Business (with Service Pack 2)/Win-Vista-SP2
- Windows 2008 Server (with Service Pack 2)/Win-2k8-Srv-SP2
- 

### Products Covered

Our tests proved that the following ViewPoint products are compatible with Microsoft patches:

- ViewPoint 5.6
- ViewPoint 5.5

Please refer to Tables 1 and 2 below for detailed test results.



Table 1: Test results for ViewPoint 5.6\*

		ViewPoint 5.6				
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP3	Win-2k3-Srv-SP2	Win-Vista Business Sp2	Win-2k8-Srv-SP2
October 2009	<a href="#">MS09-062</a>	Passed	Passed	Passed	Passed	NA
	<a href="#">MS09-061</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-060</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-059</a>	NA	NA	NA	Passed	Passed
	<a href="#">MS09-058</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-057</a>	Passed	Passed	Passed	NA	NA
	<a href="#">MS09-056</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-055</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-054</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-053</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-052</a>	Passed	Passed	Passed	NA	NA
	<a href="#">MS09-051</a>	NA	Passed	Passed	Passed	Passed
	<a href="#">MS09-050</a>	NA	NA	NA	Passed	Passed
September 2009	<a href="#">MS09-049</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-048</a>	NA	NA	Passed	Passed	Passed
	<a href="#">MS09-047</a>	NA	Passed	Passed	Passed	NA
	<a href="#">MS09-046</a>	Passed	Passed	Passed	NA	NA
	<a href="#">MS09-045</a>	Passed	Passed	Passed	Passed	Passed
August 2009	<a href="#">MS09-044</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-043</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-042</a>	Passed	Passed	Passed	NA	NA
	<a href="#">MS09-041</a>	NA	Passed	Passed	Passed	Passed
	<a href="#">MS09-040</a>	NA	NA	Passed	NA	NA
	<a href="#">MS09-039</a>	Passed	NA	NA	NA	NA
	<a href="#">MS09-038</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-037</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-036</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-035</a>	NA	NA	NA	NA	NA
<a href="#">MS09-034</a>	Passed	Passed	Passed	Passed	Passed	
July 2009	<a href="#">MS09-033</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-032</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-031</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-030</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-029</a>		Passed		Passed	Passed
	<a href="#">MS09-028</a>	Passed	Passed	Passed	Passed	NA
June 2009	<a href="#">MS09-027</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-026</a>	Passed	Passed	Passed	Passed	Passed



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

	<a href="#">MS09-025</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-024</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-023</a>	NA	NA	NA	NA	NA
	<a href="#">MS09-022</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-021</a>	NA	NA	NA	Passed	NA
	<a href="#">MS09-020</a>	Passed	NA	NA	NA	NA
	<a href="#">MS09-019</a>	Passed	Passed	Passed	Passed	Passed
	<a href="#">MS09-018</a>	NA	NA	NA	NA	NA

		ViewPoint 5.6				
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP3	Win-2k3-Srv-SP2	Win-Vista Business Sp1	Win-2k8-Srv-SP1
May 2009	<a href="#">MS09-017</a>	NA	NA	NA	NA	NA

		ViewPoint 5.6			
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP3	Win-2k3-Srv-SP2	Win-Vista Business Sp1
April 2009	<a href="#">MS09-016</a>	NA	NA	NA	NA
	<a href="#">MS09-015</a>	Passed	Passed	Passed	Passed
	<a href="#">MS09-014</a>	Passed	Passed	Passed	Passed
	<a href="#">MS09-013</a>	Passed	Passed	Passed	Passed
	<a href="#">MS09-012</a>	Passed	Passed	Passed	Passed
	<a href="#">MS09-011</a>	Passed	Passed	Passed	NA
	<a href="#">MS09-010</a>	Passed	Passed	Passed	NA
	<a href="#">MS09-009</a>	NA	NA	NA	Passed
	March 2009	<a href="#">MS09-008</a>	Passed	NA	NA
<a href="#">MS09-007</a>		Passed	Passed	Passed	Passed
<a href="#">MS09-006</a>		Passed	Passed	Passed	Passed
February 2009	<a href="#">MS09-005</a>	NA	NA	NA	NA
	<a href="#">MS09-004</a>	NA	NA	NA	NA
	<a href="#">MS09-003</a>	NA	NA	NA	NA
	<a href="#">MS09-002</a>	NA	Passed	Passed	Passed
January 2009	<a href="#">MS09-001</a>	Passed	Passed	Passed	Passed
December 2008	<a href="#">MS08-076</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-075</a>	NA	NA	NA	Passed
	<a href="#">MS08-074</a>	NA	NA	NA	Passed
	<a href="#">MS08-073</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-072</a>	NA	NA	NA	Passed
	<a href="#">MS08-071</a>	Passed	Passed	Passed	Passed



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

		ViewPoint 5.6				
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP3	Win-2k3-Srv-SP2	Win-Vista Business Sp1	
November 2008	<a href="#">MS08-070</a>	NA	NA	NA	NA	
	<a href="#">MS08-069</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-068</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-067</a>	NA	Passed	Passed	Passed	
	<a href="#">MS08-066</a>	NA	Passed	Passed	NA	
October 2008	<a href="#">MS08-065</a>	NA	NA	NA	NA	
	<a href="#">MS08-064</a>	NA	Passed	Passed	Passed	
	<a href="#">MS08-063</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-062</a>	Passed	NA	NA	NA	
	<a href="#">MS08-061</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-060</a>	NA	NA	NA	NA	
	<a href="#">MS08-059</a>	NA	NA	NA	NA	
	<a href="#">MS08-058</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-057</a>	NA	NA	NA	NA	
	<a href="#">MS08-056</a>	NA	NA	NA	NA	
	<a href="#">MS08-055</a>	NA	NA	NA	Passed	
	September 2008	<a href="#">MS08-054</a>	NA	NA	NA	Passed
		<a href="#">MS08-053</a>	NA	NA	NA	NA
<a href="#">MS08-052</a>		Passed	Passed	Passed	Passed	
<a href="#">MS08-051</a>		NA	NA	NA	NA	
August 2008	<a href="#">MS08-050</a>	NA	NA	NA	NA	
	<a href="#">MS08-049</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-048</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-047</a>	NA	NA	NA	Passed	
	<a href="#">MS08-046</a>	Passed	Passed	Passed	NA	
	<a href="#">MS08-045</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-044</a>	NA	NA	NA	NA	
	<a href="#">MS08-043</a>	NA	NA	NA	NA	
	<a href="#">MS08-042</a>	NA	NA	NA	NA	
	<a href="#">MS08-041</a>	NA	NA	NA	NA	
	<a href="#">MS08-040</a>	NA	NA	NA	NA	
July 2008	<a href="#">MS08-039</a>	NA	NA	NA	NA	
	<a href="#">MS08-038</a>	NA	NA	NA	Passed	
	<a href="#">MS08-037</a>	Passed	Passed	Passed	NA	
	<a href="#">MS08-036</a>	NA	Passed	Passed	Passed	
June 2008	<a href="#">MS08-035</a>	NA	NA	NA	NA	
	<a href="#">MS08-034</a>	Passed	NA	NA	NA	
	<a href="#">MS08-033</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-032</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-031</a>	Passed	Passed	Passed	Passed	
	<a href="#">MS08-030</a>	NA	Passed	NA	Passed	



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

		ViewPoint 5.6			
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP3	Win-2k3-Srv-SP2	Win-Vista Business Sp1
May 2008	<a href="#">MS08-029</a>	NA	NA	NA	NA
	<a href="#">MS08-028</a>	Passed	NA	NA	NA
	<a href="#">MS08-027</a>	NA	NA	NA	Passed
	<a href="#">MS08-026</a>	NA	NA	NA	Passed

		ViewPoint 5.6			
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP2	Win-2k3-Srv-SP2	Win-Vista Business
April 2008	<a href="#">MS08-025</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-024</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-023</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-022</a>	Passed	NA	NA	NA
	<a href="#">MS08-021</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-020</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-019</a>	NA	NA	NA	Passed
	<a href="#">MS08-018</a>	NA	NA	NA	NA
March 2008	<a href="#">MS08-017</a>	NA	NA	NA	NA
	<a href="#">MS08-016</a>	NA	NA	NA	NA
	<a href="#">MS08-015</a>	NA	NA	NA	Passed
	<a href="#">MS08-014</a>	NA	NA	NA	Passed
February 2008	<a href="#">MS08-013</a>	NA	NA	NA	NA
	<a href="#">MS08-012</a>	NA	NA	NA	NA
	<a href="#">MS08-011</a>	NA	NA	NA	NA
	<a href="#">MS08-010</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-009</a>	NA	NA	NA	NA
	<a href="#">MS08-008</a>	Passed	Passed	Passed	Passed
	<a href="#">MS08-007</a>	NA	Passed	Passed	Passed
	<a href="#">MS08-006</a>	NA	NA	NA	NA
	<a href="#">MS08-005</a>	Passed	NA	NA	NA
	<a href="#">MS08-004</a>	NA	NA	NA	Passed
	<a href="#">MS08-003</a>	NA	NA	NA	NA
January 2008	<a href="#">MS08-002</a>	Passed	Passed	Passed	NA
	<a href="#">MS08-001</a>	Passed	Passed	Passed	Passed
December 2007	<a href="#">MS07-069</a>	Passed	Passed	Passed	Passed
	<a href="#">MS07-068</a>	NA	Passed	Passed	Passed
	<a href="#">MS07-067</a>	NA	Passed	Passed	NA
	<a href="#">MS07-066</a>	NA	NA	NA	Passed
	<a href="#">MS07-065</a>	Passed	Passed	NA	NA



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

		ViewPoint 5.6			
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP2	Win-2k3-Srv-SP2	Win-Vista Business
	<a href="#">MS07-064</a>	Passed	Passed	Passed	Passed
	<a href="#">MS07-063</a>	NA	NA	NA	Passed
November 2007	<a href="#">MS07-062</a>	Passed	NA	NA	NA
	<a href="#">MS07-061</a>	NA	Passed	Passed	NA
October 2007	<a href="#">MS07-060</a>	NA	NA	NA	NA
	<a href="#">MS07-059</a>	NA	NA	NA	NA
	<a href="#">MS07-058</a>	Passed	Passed	Passed	Passed
	<a href="#">MS07-057</a>	Passed	Passed	Passed	Passed
	<a href="#">MS07-056</a>	Passed	Passed	Passed	Passed
September 2007	<a href="#">MS07-055</a>	Passed	Passed	Passed	Passed
	<a href="#">MS07-054</a>	NA	NA	NA	NA
	<a href="#">MS07-053</a>	NA	NA	NA	NA
	<a href="#">MS07-052</a>	NA	NA	NA	NA
	<a href="#">MS07-051</a>	Passed	NA	NA	NA

\*) All the patches listed in Table 2 for ViewPoint 5.5 have also been applied for ViewPoint 5.6



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

Table 2: Test results for ViewPoint 5.5

Release Month	Patch Bulletin ID	ViewPoint 5.5		
		Win-2k-Prof-SP4	Win-XP-Prof-SP2	Win-2k3-Srv-SP2
August 2007	<a href="#">MS07-050</a>	Passed	Passed	Passed
	<a href="#">MS07-049</a>	NA	NA	NA
	<a href="#">MS07-048</a>	NA	NA	NA
	<a href="#">MS07-047</a>	Passed	Passed	Passed
	<a href="#">MS07-046</a>	Passed	Passed	NA
	<a href="#">MS07-045</a>	Passed	Passed	Passed
	<a href="#">MS07-044</a>	NA	NA	NA
	<a href="#">MS07-043</a>	Passed	Passed	Passed
	<a href="#">MS07-042</a>	Passed	Passed	Passed
July 2007	<a href="#">MS07-041</a>	NA	Passed	NA
	<a href="#">MS07-040</a>	Passed	Passed	Passed
	<a href="#">MS07-039</a>	Passed	NA	Passed
	<a href="#">MS07-038</a>	NA	NA	NA
	<a href="#">MS07-037</a>	NA	NA	NA
	<a href="#">MS07-036</a>	NA	NA	NA
June 2007	<a href="#">MS07-035</a>	Passed	Passed	Passed
	<a href="#">MS07-034</a>	NA	Passed	Passed
	<a href="#">MS07-033</a>	Passed	Passed	Passed
	<a href="#">MS07-032</a>	NA	NA	NA
	<a href="#">MS07-031</a>	Passed	Passed	Passed
	<a href="#">MS07-030</a>	NA	NA	NA
May 2007	<a href="#">MS07-029</a>	Passed	NA	Passed
	<a href="#">MS07-028</a>	NA	NA	NA
	<a href="#">MS07-027</a>	Passed	Passed	Passed
	<a href="#">MS07-026</a>	NA	NA	NA
	<a href="#">MS07-025</a>	NA	NA	NA
	<a href="#">MS07-024</a>	NA	NA	NA
	<a href="#">MS07-023</a>	NA	NA	NA
April 2007	<a href="#">MS07-022</a>	Passed	Passed	Passed
	<a href="#">MS07-021</a>	Passed	Passed	Passed
	<a href="#">MS07-020</a>	Passed	Passed	Passed
	<a href="#">MS07-019</a>	NA	Passed	NA
	<a href="#">MS07-018</a>	NA	NA	NA
	<a href="#">MS07-017</a>	Passed	Passed	Passed
March 2007	<a href="#">No security patch was released</a>			
February 2007	<a href="#">MS07-016</a>	Passed	Passed	Passed
	<a href="#">MS07-015</a>	NA	NA	NA
	<a href="#">MS07-014</a>	NA	NA	NA



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
[www.gehealthcare.com/viewpoint](http://www.gehealthcare.com/viewpoint)

ExtReport\_MSSecurityPatch\_Oct-2009\_V1.0.doc\_Date\_2009\_11\_09

		ViewPoint 5.5		
Release Month	Patch Bulletin ID	Win-2k-Prof-SP4	Win-XP-Prof-SP2	Win-2k3-Srv-SP2
	<a href="#">MS07-013</a>	Passed	Passed	Passed
	<a href="#">MS07-012</a>	Passed	Passed	Passed
	<a href="#">MS07-011</a>	Passed	Passed	Passed
	<a href="#">MS07-010</a>	NA	NA	NA
	<a href="#">MS07-009</a>	Passed	Passed	Passed
	<a href="#">MS07-008</a>	Passed	Passed	Passed
	<a href="#">MS07-007</a>	NA	Passed	NA
	<a href="#">MS07-006</a>	NA	Passed	Passed
	<a href="#">MS07-005</a>	NA	NA	NA
January 2007	<a href="#">MS07-004</a>	Passed	Passed	Passed
	<a href="#">MS07-003</a>	NA	NA	NA
	<a href="#">MS07-002</a>	NA	NA	NA
	<a href="#">MS07-001</a>	NA	NA	NA
Older patches	<a href="#">Until MS06-078</a>	Passed	Passed	Passed

**Extras**

Windows Vista is being included as part of these tests starting from September 2007; Windows Server 2008 is being included as part of these tests starting from May 2009.

**Glossary**

*Patch Bulletin ID:* This is a number used to uniquely identify a bulletin that describes the released patch. Bulletins are released by Microsoft to describe patches, affected systems and other intricacies.

**Conclusion**

All Microsoft Patches that were released as at October 13, 2009 are compatible with ViewPoint products.

**Contact**

ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
D 82234 Wessling, GERMANY  
Tel + 49 (0) 8153 / 9311-0  
Fax + 49 (0) 8153/ 9311-30



ViewPoint Bildverarbeitung GmbH  
Argelsrieder Feld 12  
82234 Wessling, Germany  
www.gehealthcare.com/viewpoint