

Pro 1000 to Unity Network® via 1928

WHAT 1928 (Part # 001928 for \$750.00) The 1928 allows Unity Network connection to:

- Send Pro 1000 data to the CIC Pro
- Send Pro 1000 data to OTHER monitors for viewing (e.g. Solar, Dash)
NOTE: Other GE monitors CANNOT be viewed on the Pro 1000 (does not receive)
- Send alarms to the Alarm Display Unit (ADU)
- Interface with StatView

WHY Pro 1000, like other GE monitoring products, can communicate with the Unity Network

WHO Step-Down, Telemetry, ED, Outpatient Surgery ... or anyplace networking is handy

HOW The 1928 is a physical box that plugs into the back of the Pro 1000 and then plugs into a Unity port in the room, so this is a hardwired solution. The box contains software that converts the RS-232 serial data to Unity. The 1928 also provides a "pass through" of the PRO 1000 serial host communication to allow the downloading of configuration files as well as software upgrades for the PRO 1000.

OTHER Compatible with Pro 1000 V3 and V2, but NOT V1

