			3Sp		
			3Sp-D		
			5Sp		
			M5S-D		
	Aquasonics Gel	Propylene Glycol	Parker Laboratory Inc.	OK	—
	Natural Image	Propylene Glycol	Sonotech Inc.	OK	—
	Anios	Dedecyl Dimethyl Ammonium		OK	< 20 minutes
	Transeptic Spray	IPA	Parker Laboratory Inc.	OK	< 20 minutes
	T-Spray	Quaternary Ammoniums	Pharmaceutical Innovations	OK	< 20 minutes
	Sani-Cloth HB	Quaternary Ammoniums	PDI	OK	< 30 minutes
	Super Sani-Cloth	IPA	K & K Chemical Co	OK	< 30 minutes
	Gidex OPA	Ortho-phthlaldehyde	Johnson & Johnson	OK	< 15
	Gidex Plus	Gutaraldehyde	Johnson & Johnson	OK	< 25 minutes
	Enzol	Proteolytic enzymes	Johnson & Johnson	OK	< 25 minutes
	Gigasept FF	Succindialdehyde	Schülke & Mayr		< 30 minutes
	KlenZyme	Proteolytic Enzymes	Steris	OK	< 25minutes
	MetriZyme	Proteinase substilisin	Metrex Research Corporation	OK	< 25 minutes
	Milton	Sodium Hypochlorite	Milton Pharmaceutical Pty Ltd	OK	< 25 minutes
	Sporox	Hydrogen Peroxyde	DSHealthcare Inc.	OK	< 25 minutes
	Sporox II	Hydrogen Peroxyde	DSHealthcare Inc.	OK	< 25 minutes
	Virkon	Potassium Peroxymonosulfate	Antec International	OK	< 25 minutes
Wavicide-01	Gutaraldehyde	MCC	OK	< 50 minutes	

**CAUTION** : Refer to the ultrasound system user manual for important probe care and cleaning instructions.

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