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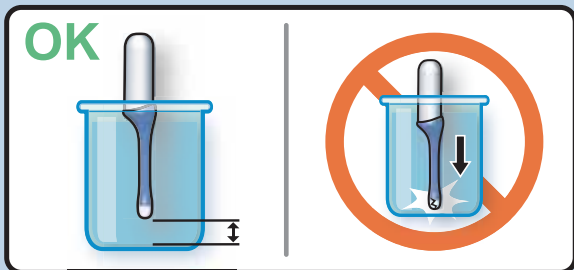
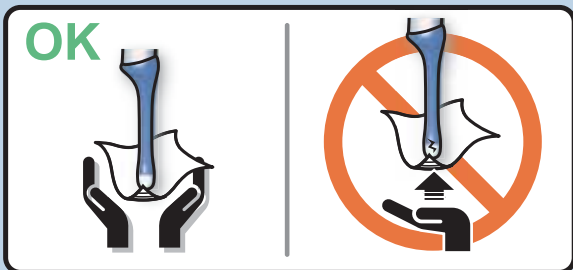
			4D10L 4D16L	4DE7C	4D3C-L	4D8C		
	GE Ultrasound Contact Gel	Gel	GE Accessories	OK	OK	OK	OK	—
	<b>Aquasonics 100</b>	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	—
	<b>Projelly</b>	Gel	JEX Inc.	⊘	⊘	⊘	⊘	—
	<b>Sonogel</b>	Gel	Sonogel Vertriebs GmbH	OK	OK	OK	OK	—
	<b>Scan</b>	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	—
	<b>Shanghai A</b>	Gel	Shanghai She Li Yiliao Qixie Co.Ltd	⊘	⊘	⊘	⊘	—
	<b>Shanghai B</b>	Gel	Shanghai She Li Yiliao Qixie Co.Ltd	⊘	⊘	⊘	⊘	—
	<b>Wavelength</b>	Gel	National Therapy Products Inc.	⊘	⊘	⊘	⊘	—
	<b>Incidin Foam</b>	20%IPA, 10%Ethyl Alcohol	Ecolab	⊘	OK	⊘	⊘	< 5 minutes
	<b>T-Spray</b>	Quaternary Ammoniums	Pharmaceutical Innovations	OK	OK	OK	OK	< 15 minutes
	<b>T-Spray II</b>	Quaternary Ammoniums	Pharmaceutical Innovations	OK	OK	OK	OK	< 15 minutes
	<b>Transept Spray</b>	70% IPA	Parker Laboratory Inc.	⊘	OK	OK	⊘	< 20 minutes
	<b>Asepti-Wipes</b>	8%IPA, <1%Quat	Ecolab Inc.	⊘	OK	OK	⊘	< 15 minutes
	<b>Asepti-Wipes II</b>	14.85%IPA	Ecolab Inc.	⊘	OK	OK	⊘	< 15 minutes
	<b>Septiwipes</b>	Quaternary Ammoniums	Dr.Schumacher GmbH	OK	⊘	⊘	⊘	< 15 minutes
	<b>CaviWipes</b>	14.85%IPA	Metrex	⊘	⊘	⊘	⊘	< 15 minutes
	<b>Cleanisept</b>	Quaternary Ammoniums	Dr.Schumacher GmbH	OK	⊘	⊘	⊘	< 15 minutes
	<b>MetriWipes</b>	8%IPA, <1%Quat	Metrex	⊘	OK	OK	⊘	< 15 minutes
	<b>NuClean II</b>	Isopropanol / Ethanol	Alkapharm	⊘	⊘	⊘	⊘	< 15 minutes
	<b>Sani-Cloth Germicidal</b>	8%IPA, <1%Quat	PDI	⊘	OK	OK	⊘	< 15 minutes
	<b>Sani-Cloth active</b>	Quaternary Ammoniums	PDI	⊘	⊘	⊘	⊘	< 15 minutes
	<b>Sani-Cloth HB</b>	Quaternary Ammoniums	PDI	OK	OK	OK	OK	< 30 minutes
	<b>PeraSafe</b>	Peracetic Acid	Antec International	OK	OK	OK	OK	< 15 minutes
	<b>Alkazyme</b>	Proteolytic Enzymes	Alkapharm	⊘	⊘	⊘	⊘	< 10 minutes
	<b>Cidex</b>	Glutaraldehyde	Johnson & Johnson	OK	OK	OK	OK	< 50 minutes
	<b>Cidex OPA</b>	Ortho-phthalaldehyde	Johnson & Johnson	OK	OK	OK	OK	< 15 minutes
	<b>Cidex Plus</b>	Glutaraldehyde	Johnson & Johnson	⊘	OK	OK	⊘	< 25 minutes
	<b>Gigasept</b>	Succindialdehyde, formaldehyde	Schülke & Mayr	OK	OK	⊘	OK	< 30 minutes
	<b>Gigasept AF</b>	Quaternary Ammoniums	Schülke & Mayr	⊘	OK	OK	⊘	< 30minutes
	<b>Gigasept FF</b>	Succindialdehyde	Schülke & Mayr	OK	OK	OK	OK	< 30minutes
	<b>Klenzyme</b>	Proteolytic Enzymes	STERIS	⊘	OK	OK	⊘	< 10 minutes
	<b>Metricide 14</b>	Glutaraldehyde	Metrex	OK	OK	OK	OK	< 50 minutes
	<b>Metricide 28</b>	Glutaraldehyde	Metrex	⊘	⊘	⊘	⊘	< 50 minutes
	<b>Metricide 30</b>	Glutaraldehyde	Metrex	⊘	⊘	⊘	⊘	< 25 minutes
	<b>Sekusept Plus</b>	Nonionic surfactant	Henkel-Ecolab	⊘	OK	⊘	⊘	< 30 minutes
	<b>Ster-Bac Blu</b>	Quaternary Ammoniums	Ecolab Inc.	OK	OK	OK	OK	< 50 minutes
	<b>Wavicide-01</b>	Glutaraldehyde	MCC	OK	OK	OK	OK	< 50 minutes