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			M5S-D		
	<b>Aquasonics Gel</b>	Propylene Glycol	Parker Laboratory Inc.	OK	—
	<b>Natural Image</b>	Propylene Glycol	Sonotech Inc.	OK	—
	<b>Anios</b>	Dedecyl Dimethyl Ammonium	Anios Laboratory	OK	< 20 minutes
	<b>Transeptic Spray</b>	IPA	Parker Laboratory Inc.	OK	< 20 minutes
	<b>T-Spray</b>	Quaternary Ammoniums	Pharmaceutical Innovations	OK	< 20 minutes
	<b>Sani-Cloth HB</b>	Quaternary Ammoniums	PDI	OK	< 30 minutes
	<b>Super Sani-Cloth</b>	IPA	K & K Chemical Co	OK	< 30 minutes
	<b>Cidex OPA</b>	Ortho-phthlaldehyde	Johnson & Johnson	OK	< 15 minutes
	<b>Cidex Plus</b>	Glutaraldehyde	Johnson & Johnson	OK	< 25 minutes
	<b>Enzol</b>	Proteolytic enzymes	Johnson & Johnson	OK	< 25 minutes
	<b>Gigasept FF</b>	Succindialdehyde	Schülke & Mayr	OK	< 30 minutes
	<b>KlenZyme</b>	Proteolytic Enzymes	STERIS	OK	< 25minutes
	<b>MetriZyme</b>	Proteinase substilisin	Metrex Research Corporation	OK	< 25 minutes
	<b>Milton</b>	Sodium Hypochlorite	Milton Pharmaceutical Pty Ltd	OK	< 25 minutes
	<b>Sporox</b>	Hydrogen Peroxyde	DSHealthcare Inc.	OK	< 25 minutes
	<b>Sporox II</b>	Hydrogen Peroxyde	DSHealthcare Inc.	OK	< 25 minutes
	<b>Virkon</b>	Potassium Peroxymonosulfate	Antec International	OK	< 25 minutes
<b>Wavicide-01</b>	Glutaraldehyde	MCC	OK	< 50 minutes	