



CAUTION : Refer to the ultrasound system user manual for important probe care and cleaning instructions.

ACHTUNG : Wichtige Hinweise zur Pflege und Reinigung von Schallköpfen finden Sie in der Gebrauchsanweisung Ihres Ultraschallgeräts.

ATTENTION : Se reporter au manuel de l'utilisateur de l'échographe pour des instructions concernant la maintenance et le nettoyage de la sonde.

CUIDADO : Consultar el Manual del Operador del sistema de ultrasonido para obtener las instrucciones importantes relativas al mantenimiento y limpieza de la sonda.

CUIDADO : Consultar o Manual do Usuário do sistema de ultra-sonografia para obter as instruções importantes relativas à manutenção e limpeza do transdutor.

CAUTELA : Per le istruzioni relative alla manutenzione e pulizia delle sonde fare riferimento al manuale per l'utente del sistema ultrasuoni.

小心 : 请参考超声波系统用户手册，了解详细的有关探头维护和清洗方面的指导信息。

注意 : 重要なプローブの手入れ並びにクリーニング手順に関して超音波システムのユーザーマニュアルを参照して下さい。

		C358 3Cb	C386	C364 CBF C36 CK	C551 CAE C55	546L	739L	LA39	LB L39	LE	L764 LH L76 LP	I739 T739	LI LT	LD	S222	S317	S611	S220 W	S316 UC	C721 CZB	CS C31	E721 MTZ E72 MZ	618E	618C		
Aquasonics 100	Gel	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
Sonogel	Gel	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
Scan	Gel	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
T-Spray	Quaternary Ammoniums	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	⊘	OK	OK	OK	OK	OK	⊘	OK	OK	OK	OK	OK	< 20 minutes
T-Spray II	Quaternary Ammoniums	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 20 minutes
Transeptic Spray	Isopropyl Alcohol	⊘	⊘	OK	OK	⊘	⊘	⊘	OK	OK	OK	⊘	OK	⊘	⊘	⊘	⊘	OK	OK	⊘	OK	⊘	⊘	⊘	⊘	< 15 minutes
Sani-Cloth HB	Quaternary Ammoniums	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 30 minutes
PeraSafe	Peracetic Acid	⊘	⊘	⊘	⊘	OK	OK	OK	⊘	⊘	⊘	OK	⊘	⊘	OK	OK	OK	⊘	⊘	⊘	⊘	OK	⊘	⊘	< 15 minutes	
Cidex	Glutaraldehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
Cidex OPA	Ortho-phthlaldehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 15 minutes
Cidex Plus	Glutaraldehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 25 minutes
Gigasept AF	Quaternary Ammoniums	⊘	⊘	⊘	⊘	OK	OK	OK	⊘	⊘	⊘	OK	⊘	⊘	OK	OK	OK	⊘	⊘	⊘	⊘	⊘	⊘	⊘	⊘	< 30 minutes
Gigasept FF	Succindialdehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	⊘	OK	OK	OK	OK	OK	⊘	OK	⊘	⊘	⊘	⊘	< 30 minutes
Ster-Bac Blu	Quaternary Ammoniums	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
Metricide 14	Glutaraldehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
Wavicide-01	Glutaraldehyde	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
Hibitane	Chlorhexidine	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 30 minutes

