




DISCOVERY RF180

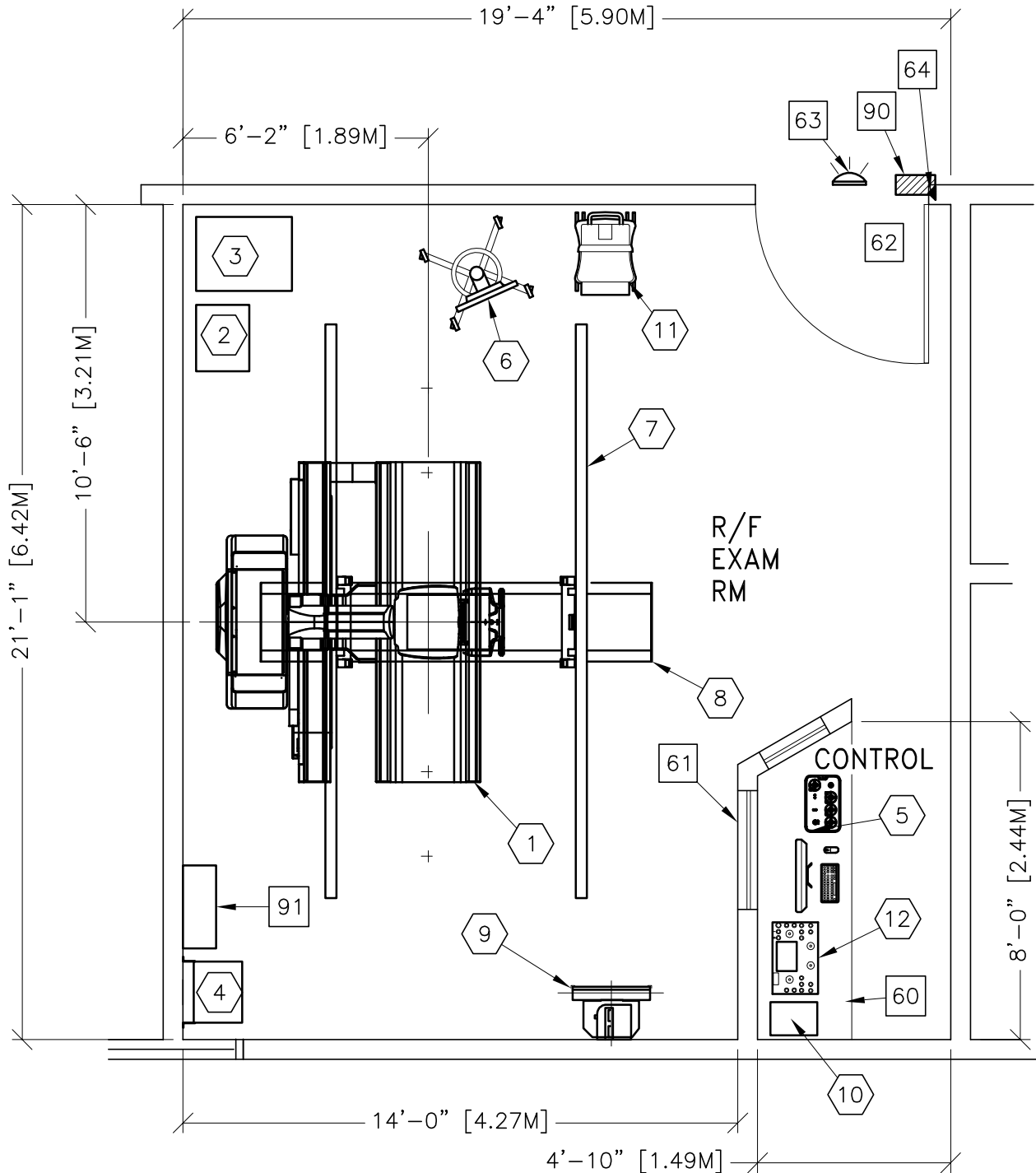

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	20'-6" x 16'-0" [6.25m x 4.88m]	8'-6" [2.6m]

ROOM PLAN LOCATED ON PAGE 2-70

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	TABLE <WITH OVER-FLOOR PLATE>	3060 lbs	2388 btu
②	1	GENERATOR	200 lbs	3500 btu
③	1	DIGITAL SYSTEM CABINET	242 lbs	1092 btu
④	1	15 KVA TRANSFORMER	210 lbs	
⑤	1	OPERATORS CONSOLE	17 lbs	149 btu
⑥	1	MONITOR CART	37 lbs	149 btu
⑦	1	4410m KALOS RAILS	661 lbs	
⑧	1	OTS WITH 3M BRIDGE	171 lbs	1190 btu
⑨	1	NON-TILTING WALL STAND	376 lbs	
⑩	1	PARTIAL SYSTEM UPS	74 lbs	171 btu
⑪	1	TIMMS 2000 ON CART <TPC>	200 lbs	
⑫	1	CONTROL CONSOLE	44 lbs	

ENVIRONMENT

Ambient Operation Temperature:

Allowable Temperature Change: 50° TO 95° (F), 10° TO 35° (C)

Humidity:

Allowable Humidity Change: 30% to 85% (no condensation)

Max - Altitude: 700 to 1060 hectopascals

ROOM PLAN LOCATED ON PAGE 2-70

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
60		COUNTER TOP FOR EQUIPMENT - MINIMUM DEPTH 24" AND ADDITIONAL SHELVING MAY BE REQUIRED BELOW COUNTER TOP FOR PC TOWER. PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE CABLES.		
61		CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING WINDOW.		
62		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 47.2" W x 78.7" H [1200mm x 2000mm], CONTINGENT ON A 62.9" [1600mm] CORRIDOR WIDTH		
63		X-RAY ON WARNING LIGHT - AVAILABLE FROM GEXPRO CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
64		DOOR LIMIT SWITCH (NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)		
90		X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL CAT. NO. E4502SS		
91		MAIN DISCONNECT,		