



GE Healthcare

## ***Invasive Cardiology Security Website***

### ***Interventional - Invasive Cardiology***

<b>Product Group:</b>	Interventional Invasive Products
<b>Products:</b>	Mac-Lab IT/XT/XTi, CardioLab IT/XT/XTi, SpecialsLab and ComboLab IT/XT/XTi Recording Systems, Centricity Cardiology Data Management Systems
<b>Version</b>	6.9.6 Release 1
<b>Subject:</b>	Security Information
<b>Date:</b>	14 October 2020

#### **Summary**

The following information is provided to GE Healthcare Technologies customers in regards to known technical security vulnerabilities associated with Mac-Lab<sup>®</sup> Hemodynamic, CardioLab<sup>®</sup> Electrophysiology, SpecialsLab and ComboLab IT Recording Systems for Cath Lab, EP Lab and other interventional labs as well as the Centricity<sup>®</sup> Cardiology Data Management Systems.

#### **Security Patch Base Configuration**

The security patch base configuration of the Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi product at release is listed within the MLCL Base Configuration under the Hemodynamic, Electrophysiology and Cardiovascular Information Technologies section of the [http://www3.gehealthcare.com/en/Support/Invasive\\_Cardiology\\_Product\\_Security](http://www3.gehealthcare.com/en/Support/Invasive_Cardiology_Product_Security) website.

#### **Process**

The following actions are taken whenever Microsoft/OEMs releases new security patches:

- The Invasive Cardiology Engineering Team performs a security analysis process for supported Mac-Lab IT/XT/XTi, CardioLab IT/XT/XTi, GE Client Review and INW Server hardware/software.
- If a vulnerability meets Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi validation criteria, the vulnerability is communicated through the GEHC Product Security Database and Invasive Cardiology Security Website within Three weeks of the patch release.



## GE Healthcare

- Upon validation of the Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi vulnerability, the GEHC Product Security Database and Invasive Cardiology Security Website and affected Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi Security Patch Installation Instructions are updated.

The Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi vulnerability validation criteria are as follows: Any vulnerability that allows malware to alter or deny Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi functionality and/or infect and propagate through normal system use.

Customers are responsible to stay informed with Microsoft vulnerability notifications and to visit the Invasive Cardiology websites to understand the Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi impact. Once a security patch is validated, customers are responsible for the installation of security patches. All Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi Security Patch Installation Instructions are available on the Invasive Cardiology Security Website below the Validated Patches table.

Vulnerabilities exposed after the Mac-Lab IT/XT/XTi and CardioLab IT/XT/XTi product release which do not meet the criteria to be validated are not listed within the GEHC Product Security Database and Invasive Cardiology Security Website. These vulnerabilities are deemed to be non-critical and/or outside normal clinical workflow of the Mac-Lab IT/XT/XTi, CardioLab IT/XT/XTi and Centricity INW systems and will not be validated. Unlisted patches should not be installed on the products in order to eliminate malfunction and breakdown risks.



**CONTENTS**

**Revision History**..... 5

**Installation of the Security Patches on MLCL systems** ..... 8

    Requirements:..... 8

    How to Log On to Acquisition and Review Systems..... 9

    How to Log On to the Centricity Cardiology INW Server ..... 9

    How to Log On to MLCL Software Only Systems ..... 9

    How to Install Printer Firmware..... 10

    How to Install HP NIC Driver on the INW Server (HP ProLiant ML350 G8)..... 10

    How to Install HP NIC Driver on the INW Server (HP ProLiant DL370 G6)..... 10

    ML350p Gen8 (P72) BIOS Update to v 2015.07.01 (B) Instructions: ..... 10

    ML350p Gen8 (P72) BIOS Update to v2018.01.22 Instructions:..... 11

    ML350p Gen8 BIOS Update to 5/21/2018 Instructions:..... 12

    DL370 Gen6 (P63) BIOS Update to v2018.02.22 Instructions:..... 12

    DL370 Gen6 BIOS Update to 5/21/2018 Instructions:..... 13

    Z600 BIOS Update to v3.61 Instructions: ..... 13

    CVE-2020-1350 | Workaround via Registry update..... 14

    OPTIONAL – How to Install INW Server Performance Enhancement ..... 15

    OPTIONAL – How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498..... 16

    OPTIONAL – How to Install Plugin 35291 –Weak Hashing..... 17

    OPTIONAL – How to Install Plugin 65821 –SSL RC4 Cipher Suites Supported ..... 17



OPTIONAL – How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration..... 17

OPTIONAL – How to Disable the SMB1 Protocol ..... 18

OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required ..... 18

OPTIONAL - CDrive Disk Space Cleanup..... 19

OPTIONAL – Removal of Authenticated Users from Share on INW server..... 20

OPTIONAL - Remote Desktop Services Remote Code Execution Vulnerability ..... 21

OPTIONAL – Disabling Questra Agent Service ..... 23

**Patch Links ..... 24**

**6.9.6 R1 Patch Installation Paths ..... 25**

    Unqualified Patches MLCL v6.9.6 R1..... 26

    MLCL v6.9.6 R1 (New Procedure) ..... 26

    MLCL v6.9.6 R1 only (Old Procedure) ..... 42

    MLCL V6.9.6 2017 Patch Updates 1 ..... 53

    MLCL V6.9.6 2017 Patch Updates 2 ..... 66

**Patches post Old and New procedure..... 70**

    MLCL V6.9.6 2017 Patch Updates 3 ..... 70

    MLCL V6.9.6 2018 Patch Updates 4 ..... 72

    MLCL V6.9.6 2018 Patch Updates 5 ..... 76

    MLCL V6.9.6 2018 Patch Updates 6 ..... 83

    MLCL V6.9.6 2019 Patch Updates 7 ..... 89

    MLCL V6.9.6 2019 Patch Updates 8 ..... 103



MLCL V6.9.6 2020 Patch Updates 9 ..... 108

MLCL V6.9.6 2020 Patch Updates 10 ..... 112

MLCL V6.9.6 2020 Patch Updates 11 ..... 114

MLCL v6.9.6 Optional Security Updates..... 119

**Contact Information ..... 123**

## Revision History

Revision	Date	Comments
1.0	25 September 2017	<ul style="list-style-type: none"> <li>• 6.9.6 document separation (old and new procedures sections for patches update)</li> <li>• Qualified KB4034664 August Monthly Rollup</li> <li>• Added 6.9.6 Installation paths for simplifying patches installation</li> <li>• September Unqualified Patches</li> </ul>
2.0	27 October 2017	<ul style="list-style-type: none"> <li>• Updated document with October Unqualified Patches</li> </ul>
3.0	20 November 2017	<ul style="list-style-type: none"> <li>• Updated document with October qualified Patches</li> </ul>
4.0	11 December 2017	<ul style="list-style-type: none"> <li>• November Unqualified Patches</li> </ul>
5.0	20 December 2017	<ul style="list-style-type: none"> <li>• Updated document for Patch installation paths and statement included to uninstall previous monthly patches prior to installation of October monthly patch on the server</li> </ul>
6.0	31 January 2018	<ul style="list-style-type: none"> <li>• Qualified November and December monthly rollups with other patches. Also added January Unqualified patches</li> </ul>
7.0	9 March 2018	<ul style="list-style-type: none"> <li>• Added Unqualified February Patch</li> <li>• Added missing qualified patches</li> <li>• Qualified minimum password length changes</li> <li>• Changed verbiage from “Patch Refresh” to “Patch Updates”</li> </ul>



## GE Healthcare

		<ul style="list-style-type: none"> <li>• Further MLCL System Security Recommendation</li> </ul>
8.0	20 April 2018	<ul style="list-style-type: none"> <li>• Added January, February, March and April Qualified Patches</li> <li>• Updated "Further MLCL Systems Security Recommendations" section and moved the section Optional Security Updates</li> </ul>
9.0	30 April 2018	<ul style="list-style-type: none"> <li>• Added "Optionally Remove Abode Reader on INW Server" section to the optional section</li> <li>• Added Unqualified April Patch</li> </ul>
10.0	17 May 2018	<ul style="list-style-type: none"> <li>• Added May Qualified Patches</li> <li>• Added .NET patches for versions 3.5 SP1, 4.5.2</li> <li>• Added Microsoft Office Patches</li> </ul>
11.0	29 June 2018	<ul style="list-style-type: none"> <li>• Added Unqualified June Patch</li> </ul>
12.0	5 September 2018	<ul style="list-style-type: none"> <li>• Added Unqualified August Patch</li> </ul>
13.0	12 September 2018	<ul style="list-style-type: none"> <li>• Qualified June, July, August and September Patches</li> <li>• Qualified .Net Framework Patches</li> <li>• Qualified Microsoft Office 2010 Patches</li> <li>• Qualified Z600 BIOS update</li> <li>• Qualified ML350p Gen8 BIOS update</li> <li>• Qualified DL370 Gen6 BIOS update</li> </ul>
14.0	5 December 2018	<ul style="list-style-type: none"> <li>• Added Unqualified October Patch</li> </ul>
15.0	10 May 2019	<ul style="list-style-type: none"> <li>• Qualified October 2018, November 2018, December 2018, January 2019, February 2019 and March 2019</li> <li>• Qualified .Net Framework Patches</li> <li>• Qualified Microsoft Office 2010 Patches</li> <li>• Added instructions to disable 3DES to the optional section</li> <li>• Added instructions to disable SMBv2 Signing to the optional section</li> <li>• Added instructions to clean up C:\ Drive</li> <li>• Added instructions to remove Authenticated Users from INW server</li> <li>• Added Adobe Reader DC MUI Installation and Configuration</li> </ul>
16.0	24 May 2019	<ul style="list-style-type: none"> <li>• Qualified April 2019 and May 2019 security patches</li> <li>• Added instructions to optionally Disable Remote Desktop Services</li> </ul>



## GE Healthcare

17.0	20 Jun 2019	<ul style="list-style-type: none"> <li>• Updates as part of SPR HCSDM00555264 – Renamed section MLCL V6.9.6 2018 Patch Updates 7 to MLCL V6.9.6 2019 Patch Updates 7</li> </ul>
18.0	27 Jun 2019	<ul style="list-style-type: none"> <li>• Added Unqualified June 2019 patches</li> <li>• Removed instructions to disable 3DES protocol</li> </ul>
19.0	5 August 2019	<ul style="list-style-type: none"> <li>• Added Unqualified July 2019 patches</li> </ul>
20.0	19 August 2019	<ul style="list-style-type: none"> <li>• Added instructions to disable Remote Desktop Services per CVE-2019-1181/1182</li> </ul>
21.0	30 August 2019	<ul style="list-style-type: none"> <li>• Added Unqualified August 2019 patches</li> </ul>
22.0	05 November 2019	<ul style="list-style-type: none"> <li>• Qualified June and July 2019 Patches</li> <li>• Added Unqualified September 2019 Patches</li> <li>• Updated Installation Paths section with details regarding patch updates.</li> <li>• Added Monthly Rollup patches for June, July, August 2019.</li> <li>• Added .NET patches for versions 3.5.1, 4.5.2</li> <li>• Added Internet Explorer 11 patches.</li> <li>• Added SHA-2 code signing support update patch.</li> <li>• Added Microsoft Office 2010 Patches.</li> <li>• Added Adobe 12 Patch Installation Instructions.</li> </ul>
23.0	18 March 2020	<ul style="list-style-type: none"> <li>○ Added section MLCL V6.9.6 2020 Patch Updates 9 <ul style="list-style-type: none"> <li>▪ Qualified SHA-2 code signing support update (Windows Server R2 2008 only)</li> <li>▪ Qualified IE11 Cumulative Updates October 2019</li> <li>▪ Qualified November 2019, December 2019 Monthly Rollups</li> <li>▪ Qualified November 2019, December 2019 &amp; January 2020 Service Stack Updates.</li> <li>▪ Qualified Extended Security Updates (ESU) Licensing Preparation Package</li> </ul> </li> <li>○ Updated Unqualified Patches MLCL v6.9.6 R1 table</li> <li>○ Remediation <ul style="list-style-type: none"> <li>▪ Added additional instruction to CDrive Disk Space Cleanup section</li> <li>▪ Marked Abode updates as Superseded in MLCL V6.9.6 2017 R1 (New Procedure), MLCL V6.9.6 2017 Patch Updates 2, MLCL V6.9.6 2017 Patch Updates 3</li> <li>▪ “OPTIONAL – Removal of Authenticated Users from Share on INW server” – updated section with new instruction.</li> <li>▪ MLCL V6.9.6 2019 Patch Updates 7 – Removed NOTE for clarification</li> </ul> </li> </ul>



## GE Healthcare

24.0	25 June 2020	<ul style="list-style-type: none"><li>• Added OPTIONAL - Disabling Qestra Agent Service section</li><li>• Added section MLCL V6.9.6 2020 Patch Updates 10</li><li>• Added January 2020 Security Only Update and KB4539602</li><li>• Updated MLCL v6.9.6 Optional Security Updates section</li></ul>
25.0	14 October 2020	<ul style="list-style-type: none"><li>• Added section MLCL V6.9.6 2020 Patch Updates 11</li><li>• Updated Unqualified Patches MLCL v6.9.6 R1 section</li><li>• Added section CVE-2020-1350   Workaround via Registry update</li><li>• Removed section CVE-2019-1181/1182   Remote Desktop Services Remote Code Execution Vulnerability</li></ul>

## Installation of the Security Patches on MLCL systems

### Requirements:

- Updates may be applied at any time other than while the Mac-Lab IT/XT/XTi, CardioLab IT/XT/XTi or SpecialsLab application is open.
- Updates must be re-applied if the system is re-imaged.
- Updates apply to both networked and standalone systems.
- Best practice is to update all applicable MLCL systems at the site.

**This document applies to 6.9.6R1 only. Please verify that you are running 6.9.6 using the following procedure before proceeding:**

1. Launch the Mac-Lab CardioLab application.
2. Select **Help > About Mac-Lab** (or **CardioLab**, as applicable).
3. Verify the version number is **6.9.6 Release 1**.
4. Click **Close**.
5. Close the application.

**Recommendation: Use Internet Explorer (IE) for Catalog download.** If you are using the cart feature to download patches, to see the cart it requires opening another tab or new window for <http://catalog.update.microsoft.com>





GE Healthcare

REFER TO [6.9.6 R1 PATCH INSTALLATION PATHS](#) SECTION FOR INSTRUCTIONS ON HOW TO APPLY THE NECESSARY PATCHES ACCORDING TO YOUR SYSTEM. THE FOLLOWING INSTRUCTIONS ARE A HOW TO AND ARE OPTIONAL UP TO THE PATCH INSTALLATION SECTION.

## How to Log On to Acquisition and Review Systems

When starting up an Mac-Lab, CardioLab or SpecialsLab Acquisition or Review system, an auto-logon sequence starts and automatically logs on to the operating system. To install a security patch, the user must be logged on as **mlcltechuser**.

**NOTE:** Password information is contained within the Security Guide Manual. Otherwise, contact the system administrator or GE Technical Support for current password information.

1. Power on the Acquisition system.
2. The system boots up to the **Custom Shell** screen.
3. Press **Ctrl + Action + Del**.
4. Click **Logoff**. On Windows XP, click **Logoff** again.
5. Click **OK**.
6. Immediately hold down the **Shift** key until the login window is displayed.
7. Log on to the operating system locally as **mlcltechuser**.
8. Log on to the **Custom Shell** locally as **mlcltechuser**.

## How to Log On to the Centricity Cardiology INW Server

Password information is contained within the Security Guide Manual. Otherwise, contact the system administrator or GE Technical Support for current password information. Logon to the INW Server as **administrator**

## How to Log On to MLCL Software Only Systems

Since Software Only systems are supported by the customer, the system needs to be logged into with an **administrator** account.



## How to Install Printer Firmware

The system which will apply the firmware to the printer should be provided by the customer.

**NOTE:** Mac-Lab CardioLab system should not be used to download and/or apply the Printer Firmware.

- Follow the download link in the table
- Select the appropriate printer
- Select English and the applicable MLCL operating system
- Select English and under the Firmware category select the applicable Firmware Update Utility and Click Download
- Launch the firmware installer and follow the instructions to complete the firmware update

## How to Install HP NIC Driver on the INW Server (HP ProLiant ML350 G8)

1. Using a non-MLCL system, visit the URL and download **cp025039.exe**.
2. Place **cp025039.exe** on removable media.
3. Ensure the INW Server is the HP ProLiant ML350 G8, log on to the server as **administrator**.
4. Insert the removable media and launch the **cp025039.exe** application.
5. Follow the installation instructions to complete the installation.

## How to Install HP NIC Driver on the INW Server (HP ProLiant DL370 G6)

1. Using a non-MLCL system, visit the URL and download **cp024564.exe**.
2. Place **cp024564.exe** on removable media.
3. Ensure the INW Server is the HP ProLiant DL370 G6, log on to the server as **administrator**.
4. Insert the removable media and launch the **cp024564.exe** application.
5. Follow the installation instructions to complete the installation.

## ML350p Gen8 (P72) BIOS Update to v 2015.07.01 (B) Instructions:

1. Go to the HP Customer Support - Software and Driver Downloads website:



## GE Healthcare

[https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX\\_13ce4f3fc79b4433873342345b](https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_13ce4f3fc79b4433873342345b)

2. Select **Download** for HP ML350p Gen8 Server BIOS 2015.07.01 (B)
3. Log on to the ML350p Gen8 Server as **administrator**.
4. Run the downloaded **cp031045.exe** file.
5. Click on **Run**
6. Click on **Install**
7. Follow the onscreen instructions, select **close** after installation is complete.
8. Select **Yes** to reboot.
9. The BIOS update will take a few minutes, do not remove power during update.
10. After the update, on boot screen before Windows launches, verify the **ProLiant System BIOS – P72 (01/07/2015)** on the top of the screen appears.

### ML350p Gen8 (P72) BIOS Update to v2018.01.22 Instructions:

11. Go to the HP Customer Support - Software and Driver Downloads website:  
[https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&swItemId=MTX\\_58f2f298203c4664a6e5c9402d&swEnvOid=4184#tab1](https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&swItemId=MTX_58f2f298203c4664a6e5c9402d&swEnvOid=4184#tab1)
12. Click on **Download** for System ROMPaq Firmware Upgrade for HP ProLiant ML350p Gen8 (P72) Servers (For USB Key-Media)
13. Obtain a formatted USB Key media
14. Download the SoftPaq to a directory on a system running Microsoft Windows 2000, Microsoft Windows XP, Microsoft Windows Vista, Microsoft Windows 7, Microsoft Windows Server 2003, Microsoft Windows Server 2008, or Microsoft Windows Server 2008 R2 and change to that directory
15. From that drive and directory, execute the downloaded SoftPaq file: Simply double click on the SP99236.exe file and follow the installation wizard to complete the SoftPaq installation process. At the end of a successful installation of the SoftPaq a **web page** will automatically appear to provide you with the different methods for restoring and/or upgrading the firmware on the system
16. Click on **Create a bootable USB Key**
17. Click on **Launch HPE USB key Setup Creation Utility**
18. Click on **Run**
19. Click on **Start**
20. Click on **OK** when complete
21. Insert this USB Key into the USB Key port of the system to be updated and power the system on
22. Using your keyboard use **F11** to select **Boot Menu**
23. Using your keyboard select **#3 One Time Boot to USB DriveKey**
24. **Ready To Flash** window will appear
25. Click on **Continue** or Use the Keyboard **Enter** Key to Continue
26. After successful installation another window will appear



27. Click on **Exit** or use the keyboard **Esc** key to exit
28. Use the Power button to restart the system.
29. After the update, on boot screen before Windows launches, verify the **ProLiant System BIOS – P72 (01/22/2018)** on the Top of the screen appears.

### ML350p Gen8 BIOS Update to 5/21/2018 Instructions:

1. Go to the HP Customer Support - Software and Driver Downloads website:  
[https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&swItemId=MTX\\_3a5fbdd653f94fac9e2a12e8dc&swEnvOid=4184](https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&swItemId=MTX_3a5fbdd653f94fac9e2a12e8dc&swEnvOid=4184)
2. Select **Download** for HP ML350p Gen8 Server BIOS 5/21/2018
3. Log on to the ML350p Gen8 Server as **administrator**.
4. Run the downloaded **cp036431.exe** file.
5. Click on **Run**
6. Click on **Install**
7. Follow the onscreen instructions, select **close** after installation is complete.
8. Select **Yes** to reboot.
9. The BIOS update will take a few minutes, do not remove power during update.
10. After the update, on boot screen before Windows launches, verify the BIOS version 5/21/2018 on the top of the screen appears.

### DL370 Gen6 (P63) BIOS Update to v2018.02.22 Instructions:

1. Go to the HP Customer Support - Software and Driver Downloads website:  
[https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX\\_df78827f61164b6f952314da7c](https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_df78827f61164b6f952314da7c)
2. Select **Download** for HP DL370p Gen6 Server BIOS 2018.02.22
3. Log on to the DL370p Gen6 Server as **administrator**.
4. Run the downloaded **cp035397.exe** file.
5. Click on **Run**
6. Click on **Install**
7. Follow the onscreen instructions, select **close** after installation is complete.
8. Select **Yes** to reboot.
9. The BIOS update will take a few minutes, do not remove power during update.



After the update, on boot screen before Windows launches, verify the **ProLiant System BIOS – P63 (02/22/2018)** on the top of the screen appears

## DL370 Gen6 BIOS Update to 5/21/2018 Instructions:

1. Go to the HP Customer Support - Software and Driver Downloads website:  
[https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=3890183&swItemId=MTX\\_37660617800b4f3db17cb11599&swEnvOid=4184](https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=3890183&swItemId=MTX_37660617800b4f3db17cb11599&swEnvOid=4184)
2. Select **Download** for HP DL370 Gen6 Server BIOS 5/21/2018
3. Log on to the DL370 Gen6 Server as **administrator**.
4. Run the downloaded **SP99324.exe** file.
5. Click on **Run**
6. Click on Next
7. Select **I accept the terms in the license agreement**
8. Click on Next to accept the default path
9. Web page opens
10. Click on **Create a bootable USB Key**
11. Click on **Launch HPE USB key Setup Creation Utility**
12. Click on **Run**
13. Click on **Start**
14. Click on **OK** when complete
15. Insert this USB Key into the USB Key port of the system to be updated and power the system on
16. Using your keyboard use **F11** to select **Boot Menu**
17. Using your keyboard select **#3 One Time Boot to USB DriveKey**
18. **Ready To Flash** window will appear
19. Click on **Continue** or Use the Keyboard **Enter** Key to Continue
20. After successful installation another window will appear
21. Click on **Exit** or use the keyboard **Esc** key to exit
22. Use the Power button to restart the system.
23. After the update, on boot screen before Windows launches, verify the **ProLiant System BIOS – P72 (05/21/2018)** on the Top of the screen appears.

## Z600 BIOS Update to v3.61 Instructions:



1. Go to the HP Customer Support - Software and Driver Downloads website:
2. <https://support.hp.com/us-en/drivers/selfservice/hp-z600-workstation/3718663>
3. Select **BIOS**.
4. Select **Download** for HP Z600 Windows Workstation System BIOS 3.61 Rev.A
5. Log on to the z600 computer as **administrator**.
6. Run the downloaded **sp84160.exe** file.
7. Select **Yes** to allow.
8. Click on Next
9. Select **I accept the terms in the license agreement**.
10. Click on Next
11. Select **View Contents of the HPQFlash folder**. This opens folder:  
  
`C:\swsetup\SP84160\HPQFlash`
12. Run **HPQFlash.exe**.
13. Select **Yes** to allow.
14. Click on Next and Next again
15. Follow the onscreen instructions, select **Continue** to restart
16. The BIOS update will take a few minutes, do not remove power during update. The computer will Reboot during this update.
17. After the update, on the first boot screen before Windows launches, verify the BIOS version 3.61 on the bottom left of the screen appears.

## CVE-2020-1350 | Workaround via Registry update

Note: Apply this workaround only if the INW is a domain controller.

Below are the steps to apply workaround via registry update,

1. Log on to the INW Server as Administrator.
2. Select Windows Start -> Run and enter Regedit.
3. In Regedit make the following registry change to restrict the size of the largest inbound TCP-based DNS response packet that's allowed:  
Subkey: HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Services\DNS\Parameters  
Value: TcpReceivePacketSize  
Type: DWORD  
Value data: 0xFF00



4. Close the Registry Editor
5. Restart the System.

**Note:**

It is the customer's responsibility to ensure compatibility of the workaround with the hospital network. At a minimum, it is recommended to test DICOM, CVIT and HL7 communication (as applicable).

As noted within the following Microsoft support article below, serious problems might occur if you modify the registry incorrectly. Before you modify it, back up the registry for restoration in case problems occur.

<https://support.microsoft.com/en-us/help/4569509/windows-dns-server-remote-code-execution-vulnerability>

For additional information regarding this workaround, refer to the Microsoft support article.

## OPTIONAL – How to Install INW Server Performance Enhancement

The following patches do not resolve security vulnerabilities and are optional. These patches may improve network performance. The installation procedure below must be followed and all listed patches deployed together. This deployment may take up to 12 hours, the large percentage within the KB2775511 installation.

1. Using a non-MLCL system, visit and download the following patches to removable media.  
Visit <http://catalog.update.microsoft.com/> and enter the below KB numbers to access the patches.  
KB2775511 - <http://support.microsoft.com/kb/2775511>  
KB2732673 - <http://support.microsoft.com/kb/2732673>  
KB2728738 - <http://support.microsoft.com/kb/2728738>  
KB2878378 - <http://support.microsoft.com/kb/2878378>

The following patches are summarized at KB2473205 - <https://support.microsoft.com/en-us/kb/2473205>  
KB2535094 - <http://support.microsoft.com/kb/2535094> Download at - <https://support.microsoft.com/en-us/hotfix/kbhotfix?kbnm=2535094&kbln=en-us>  
KB2914677 - <http://support.microsoft.com/kb/2914677> Download at - <https://support.microsoft.com/en-us/hotfix/kbhotfix?kbnm=2914677&kbln=en-us>  
KB2831013 - <http://support.microsoft.com/kb/2831013> Download at - <https://support.microsoft.com/en-us/hotfix/kbhotfix?kbnm=2831013&kbln=en-us>  
KB3000483 - <http://support.microsoft.com/kb/3000483> Download at - <http://catalog.update.microsoft.com/>  
KB3080140 - <http://support.microsoft.com/kb/3080140> Download at - <http://catalog.update.microsoft.com/>



## GE Healthcare

KB3044428 – <http://support.microsoft.com/kb/3044428> Download at - <https://support.microsoft.com/en-us/hotfix/kbhotfix?kbnm=3044428&kbIn=en-us>

2. Log on to the INW Server as **Administrator**.
3. Insert the removable media and install the patches in the order listed above.
4. Follow the Microsoft installation instructions to complete the installation of the patches.
5. Select **Windows Start -> Run** and enter **Regedit** and Enter.
6. In **Regedit** window, navigate to **HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\services\Tcpip**
7. In the dialog Menu, select **File -> Export**. Name the file **MLCLRegSave.reg** and place in the **C:\Temp** directory.
8. In Regedit windows, from **Tcpip** navigate to **Parameters**.
9. In the dialog Menu, select **Edit -> New -> DWORD (32-bit) Value**. A new entry is created and name it '**MaxUserPort**'.
10. Right click on '**MaxUserPort**', select **Modify** and enter the value **65534** with a base of **Decimal**.
11. Follow the same procedure above and create a new entry named '**TcpTimedWaitDelay**'. Enter the value **60** with a base of **Decimal**.
12. Exit the **Regedit** dialog.
13. Reboot the INW Server.

### OPTIONAL – How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498

1. Log on to Windows as **Administrator** or a member of that group.
2. Open a command prompt and enter the following commands:
3. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 2.0" /f`
4. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 2.0\Client" /v DisabledByDefault /t REG_DWORD /d 00000001 /f`
5. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 2.0\Server" /v DisabledByDefault /t REG_DWORD /d 00000001 /f`
6. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 2.0\Server" /v Enabled /t REG_DWORD /d 00000000 /f`
7. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 3.0" /f`
8. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 3.0\Server" /f`
9. `REG ADD "HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 3.0\Server" /v DisabledByDefault /t REG_DWORD /d 00000001 /f`





## GE Healthcare

10. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Protocols\SSL 3.0\Server" /v Enabled /t REG\_DWORD /d 00000000 /f
11. Close the command prompt.

### OPTIONAL – How to Install Plugin 35291 –Weak Hashing

- 1) Load your security certificate in SQL Server on each ML/CL system in the network (server, acquisitions, reviews and virtual reviews) or the ML/CL standalone acquisition.
- 2) Disable RDP on each member of the network.
  - a) My Computer>Properties>Remote settings>Remote
  - b) Check "Don't allow connections to this computer".
  - c) Click ok and reboot.

### OPTIONAL – How to Install Plugin 65821 –SSL RC4 Cipher Suites Supported

1. Log on to Windows as **Administrator** or a member of that group.
2. Open a command prompt and enter the following commands:
3. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 128/128" /f
4. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 128/128" /v Enabled /t REG\_DWORD /d 00000000 /f
5. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 40/128" /f
6. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 40/128" /v Enabled /t REG\_DWORD /d 00000000 /f
7. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 56/128" /f
8. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Control\SecurityProviders\SCHANNEL\Ciphers\RC4 56/128" /v Enabled /t REG\_DWORD /d 00000000 /f
9. Close the command prompt.

### OPTIONAL – How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration

1. Log on to Windows as Administrator or a member of that group
2. Open Regedit do the following



## GE Healthcare

- a. On Windows 7
  - i. Navigate to HKLM\System\CurrentControlSet\Services\RtkAudioService
  - ii. Change the imagepath key value from:  
**C:\Program Files\Realtek\Audio\HDA\RtkAudioService.exe**  
To:  
**"C:\Program Files\Realtek\Audio\HDA\RtkAudioService.exe"**  
Note: The leading and trailing Quote marks are part of the key value. The quotes are what removes the vulnerability
  
- b. On Windows 2008R2
  - i. Navigate to HKLM\System\CurrentControlSet\Services\Gems Task Scheduler
  - ii. Change the imagepath key value from:  
**C:\Program Files (x86)\GE Healthcare\MLCL\Bin\ArchiveUtility\GEMS\_TaskSvc.exe**  
To:  
**"C:\Program Files (x86)\GE Healthcare\MLCL\Bin\ArchiveUtility\GEMS\_TaskSvc.exe"**  
Note: The leading and trailing Quote marks are part of the key value. The quotes are what removes the vulnerability

### OPTIONAL – How to Disable the SMB1 Protocol

**NOTE: SMBV2 Signing might result in nonfunctional VIVID interface with Acquisition system**

1. Log on to Windows as **Administrator** or a member of that group.
2. Open a command prompt and enter the following commands:
3. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Services\LanmanServer\Parameters" /f
4. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Services\LanmanServer\Parameters" /v SMB1 /t REG\_DWORD /d 00000000 /f
5. sc.exe config lanmanworkstation depend= bowser/mrxsmb20/lsi
6. sc.exe config mrxsmb10 start= disabled

### OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required

**NOTE: SMBV2 Signing might result in nonfunctional VIVID interface with Acquisition system**



## GE Healthcare

1. Log on to Windows as **Administrator** or a member of that group.
2. Open a command prompt and enter the following commands:
3. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Services\LanmanServer\Parameters" /f
4. REG ADD "HKEY\_LOCAL\_MACHINE\SYSTEM\CurrentControlSet\Services\LanmanServer\Parameters" /v RequireSecuritySignature /t REG\_DWORD /d 1 /f
5. Close the command prompt.

### OPTIONAL - CDrive Disk Space Cleanup

Follow the below instructions to cleanup CDrive disk space on 6.9.6 R1 Review workstations and Virtual Review systems only.

- 1) Note the following login requirements:
  - On Review workstations and Virtual Review systems, you must be logged in as member of **domain** administrator group to Windows and Custom Shell.
- 2) Open **Windows Explorer**.
- 3) Click **Organize > Folder and search options > View**.
- 4) Select **Show hidden files, folders, and drivers**.
- 5) Deselect **Hide protected operating system files (Recommended)**.
- 6) If prompted, click **Yes**.
- 7) Click **OK**.
- 8) If present, permanently delete the contents of the following folders, if prompted click **Continue**:
  - C:\Windows\SoftwareDistribution\Download
  - C:\Users\MLCLLogonUser\AppData\Local\Temp
  - C:\Users\MLCLLogonUser.<domainname>\AppData\Local\Temp
  - C:\Users\MLCLTechUser\AppData\Local\Temp
  - C:\Users\MLCLTechUser.<domainname>\AppData\Local\Temp
  - C:\Users\MLCLUser\AppData\Local\Temp
  - C:\Users\MLCLUser.<domainname>\AppData\Local\Temp
  - C:\Temp

**Note:** On Virtual Review system do not delete **C:\Temp\VirtualReviewBanner.reg** and **C:\Temp\Tools** folder.

  - C:\Windows\Temp
  - C:\Windows\Logs
  - C:\Windows\Installer\\$\PatchCahce\$\Managed
- 9) Click **Organize > Folder and search options > View**.



- 10) Select **Don't show hidden files, folders, or drivers**.
- 11) Select **Hide protected operating system files (Recommended)**.
- 12) Click **OK**.
- 13) Right click **Computer** and select **Manage**.
- 14) Expand and click **Storage > Disk Management**.
- 15) Right click on the **C:** partition and select **Properties**.
- 16) Click **Tools > Defragment now**.
- 17) Select **C:** partition and click **Defragment disk**.
- 18) Allow the Defragmentation to complete.
- 19) Close Disk Defragmenter window.
- 20) Close C: Properties window.
- 21) Open **Windows Explorer**.
- 22) Right click on **C: Drive**
- 23) Click **Properties->Disk Cleanup->Clean up system files**.
- 24) If prompted, click **Yes**
- 25) In Files to delete section, there are checkboxes listed. Select all the checkboxes, ensure there is a tick mark against all checkboxes.
- 26) Click on **OK** button.
- 27) If prompted, Are you sure you want to permanently delete these files? Click on **Delete Files** button.
- 28) Once disk cleanup process is complete. **Restart the system**.

## **OPTIONAL – Removal of Authenticated Users from Share on INW server**

### **Pre-requisites:**

1. Authenticated Users on INW systems must be removed only on INW servers.
2. Authenticated Users on INW systems must be removed only on **Studies** directory located in **D:\GEData\Studies** path.
3. The following MLCL systems Groups should be added to the **Permissions** of **D:\GEData\Studies** directory:
  - a. **MLCLAdmGrp**
  - b. **MLCLSystemGrp**
  - c. **MLCLUsrGrp**
  - d. **SYSTEM (Member Server only)**



### Steps to remove Authenticated user

1. On the INW Server, run Windows Explorer.
2. Browse to **D:\GEData**.
3. Right-click on the **Studies** directory and select **Properties**.
4. Select the **Sharing** tab.
5. Click the **Advanced Sharing** button.
6. Click the **Permissions** button.
7. In the **Group or user names** list, select **Authenticated Users** and click **Remove**.
8. Click **Add**.
9. Enter the following group names and then click **OK**: **MLCLAdmGrp**; **MLCLSystemGrp**; **MLCLUsrGrp**.
10. In a Member Server environment *only*, do the following:
  - a. Click **Add**.
  - b. Click **Locations**, select the name of the INW Server in the **Location** tree, and click **OK**.
  - c. Enter the name **SYSTEM** and then click **OK**.
11. Select the group **MLCLAdmGrp** and, in the **Allow** column, check the **Change** box.
12. Select the group **MLCLSystemGrp** and, in the **Allow** column, check the **Change** box.
13. Select the group **MLCLUsrGrp** and, in the **Allow** column, check the **Change** box.
14. In a Member Server environment *only*, do the following:
  - a. Select the group **SYSTEM** and, in the **Allow** column, check the **Change** box.
15. Click **OK** to close the **Permissions** window.
16. Click **OK** to close the **Advanced Sharing** window.
17. Click **Close** to close the **Properties** window.
18. Reboot the server when there are no cases in progress.

## OPTIONAL - Remote Desktop Services Remote Code Execution Vulnerability

We recommend that the RDS is disabled on all ML/CL systems if not in use. Refer to the instructions below on disabling the service.

### Disable Remote Desktop Services on Windows 7

On Windows 7, do the following to disable Remote Desktop Services:



1. Right-click on **Computer** and select **Properties**
2. On the **System** window, click **Remote settings**
3. Uncheck the following option if it is checked:  
**Allow Remote Assistance connections to this computer**
4. Select the following option if it is not already selected:  
**Don't allow connections to this computer**
5. Click **OK** to close the **System Properties** window
6. Click **Start > Control Panel > Administrative Tools > Services**
7. Double-click on the entry for **Remote Desktop Services**
8. Set the **Startup type** to **Disabled**
9. Click **OK** to close the **Remote Desktop Services Properties** window
10. Reboot the system

### **Disable Remote Desktop Services on Windows 2008 R2**

On Windows Server 2008 R2, do the following to disable Remote Desktop Services:

1. Right-click on **Computer** and select **Properties**
2. On the **System** window, click **Remote settings**
3. Uncheck the following option if it is checked:  
**Allow Remote Assistance connections to this computer**
4. Select the following option if it is not already selected:  
**Don't allow connections to this computer**



5. Click **OK** to close the **System Properties** window
6. Click **Start > Administrative Tools > Services**
7. Double-click on the entry for **Remote Desktop Services**
8. Set the **Startup type** to **Disabled**
9. Click **OK** to close the **Remote Desktop Services Properties** window
10. Reboot the system

## OPTIONAL – Disabling Questra Agent Service

If InSite functionality is not used on the system, we recommend disabling Questra Agent Service and TightVNC Server Service on all ML/CL systems. Refer to the instructions below on disabling the services.

1. Click **Start > Control Panel > Administrative Tools > Services**
2. Double-click on the entry for **QUESTRA SERVICE AGENT**
3. Set the **Startup type** to **Disabled**
4. Click **OK** to close the **QUESTRA SERVICE AGENT Properties** window
5. Double-click on the entry for **TightVNC Server**
6. Set the **Startup type** to **Disabled**
7. Click **OK** to close the **TightVNC Server Properties** window
8. Reboot the system



## Patch Links

The patches displayed below are qualified on an independent basis and can be installed on a one-by-one basis, although it is recommended that all qualified patches are installed. There are dependencies within the qualified patch list. In the table below, it is recommended the patches are installed in order from top to bottom to ensure all pre-requisites are met for all patches. On occasion the patch dependencies require system reboots which are identified in the table below.

**NOTE:** Due to site configurations, system patch set, qualified patches which have been installed previously or patch dependencies, some patches could fail to install due to the functionality is already installed. The Microsoft patch installer will alert you to this issue. If this occurs, please continue with the next patch installation.

**Alternate Patch locations:** In early 2016 Microsoft announced that some patches would no longer be available on the Microsoft Download Center <https://blogs.technet.microsoft.com/msrc/2016/04/29/changes-to-security-update-links/>. Therefore, some of the links provided below may not work. Microsoft may move/remove these links at any time without notice. However, if the links do not work, there are two alternate methods for downloading patches. The first is the Microsoft Catalog <http://catalog.update.microsoft.com>. Most fixes not on Microsoft Download Center will be available from the Microsoft Catalog. If a fix is not available from the Microsoft Catalog, Microsoft has monthly ISO files of the security updates available at <https://support.microsoft.com/en-us/kb/913086>. To use the ISOs, determine the month of the patch, download the applicable ISO and extract the patch. If after exhausting all three methods, you are still unable to obtain a patch, please contact GE Technical Support for further assistance.

**Recommendation: Use Internet Explorer (IE) for Catalog download. If you are using the cart feature to download patches, to see the cart it requires opening another tab or new window for <http://catalog.update.microsoft.com>**





## 6.9.6 R1 Patch Installation Paths

There are two installation paths

- Old Procedure
- New Procedure

The New procedure was developed to remove patch redundancy and streamline the patch installation for 6.9.6R1. To provide a transition both the Old and the New procedures will be supported for patch releases. Either procedure can be used and the results will be the same. However, it is extremely important that you fully complete one of these procedures to ensure you are at a common baseline to apply any future patch updates. If you have been following the old procedure then most likely will want to continue to follow it for this patch refresh. But if you are applying patches for the first time or are reimaging the system from a disaster recovery, the use of the New Procedure will be your quicker and preferred path.

The following sample scenarios are provided for reference.

- (1) New Setup/Reimage a machine for disaster recovery.
  - (a) Apply the updates from the following sections
    - (i) MLCL v6.9.6 R1 only (New Procedure)
- (2) The machine was setup and patched. Need to patch recently qualified patches.
  - (a) Apply the updates from the following sections
    - (i) Verify that all patches from section **MLCL v6.9.6 R1 only (Old Procedure)** have been applied. Install any unapplied patches
    - (ii) MLCL V6.9.6 2017 Patch Updates 1 (if patches from this section were previously not applied)
    - (iii) MLCL V6.9.6 2017 Patch Updates 2 (if patches from this section were previously not applied)

Post installation of patches using the scenarios (old or new procedure) described above patches from following sections should be installed:

- (i) MLCL V6.9.6 2017 Patch Updates 3 (if patches from this section were previously not applied) and latest Patch Updates sections
- (ii) Patches from subsequent patch update sections starting section MLCL V6.9.6 2018 Patch Updates 4

Note: The section MLCL Optional Security Updates applies to both scenarios. The optional updates provide additional security but are not required. You may apply some of the optional patches but choose to bypass others. For example, you may want to disable some of the vulnerable protocols but you may not want to address Weak Hashing due to cost and complexity of certificate management. This will not cause problems. However, ***all other updates are strongly recommended.***



Some updates are documented as superseded. These are left in the document for completeness but can be skipped.

### Unqualified Patches MLCL v6.9.6 R1

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
Operating System Platform	Windows Server 2008 R2 SP1	Windows 7 SP1	Windows 7 SP1	Windows 7 SP1
Current Unqualified Vulnerability	None	None	None	None

### MLCL v6.9.6 R1 (New Procedure)

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) It is expected that some patches listed will already be on the system.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	



## GE Healthcare

	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
KB2901907	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42642">https://www.microsoft.com/en-us/download/details.aspx?id=42642</a>	Right click and Run as Administrator; Verify that .Net 4.5.2 shows in Control Panel\Programs and Features
Subsequent patches depend on installation of KB2901907		
KB2639308	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=28929">https://www.microsoft.com/en-us/download/details.aspx?id=28929</a>	Install only .MSU file
IE11	<a href="http://download.microsoft.com/download/9/2/F/92FC119C-3BCD-476C-B425-038A39625558/IE11-Windows6.1-x86-en-us.exe">http://download.microsoft.com/download/9/2/F/92FC119C-3BCD-476C-B425-038A39625558/IE11-Windows6.1-x86-en-us.exe</a>	
	<b>Reboot Required</b>	
KB2979596	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44278">https://www.microsoft.com/en-us/download/details.aspx?id=44278</a>	Run from cmd prompt with SQLServer2008SP4-KB2979596-x86-ENU.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
KB3020369	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f00e3c36-f5e3-465c-95d2-a84a22425868">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f00e3c36-f5e3-465c-95d2-a84a22425868</a>	
KB3138612	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5</a>	
KB3125574	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ed4c8c3-0f06-4227-99e3-e9f143394687">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ed4c8c3-0f06-4227-99e3-e9f143394687</a>	
	<b>Reboot Required</b>	



## GE Healthcare

Adobe 11.0.19 <b>Superseded</b>	<a href="http://suppo.rtdo.wnlo.ads.adobe.com/detail.jsp?ftpID=6123">http://suppo.rtdo.wnlo.ads.adobe.com/detail.jsp?ftpID=6123</a>	
KB4022719	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b9815359-6aad-467e-8666-2351fad3c45">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b9815359-6aad-467e-8666-2351fad3c45</a>	
KB3205402	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8938e57-3bdd-411e-8bdd-38ebbac1db50">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8938e57-3bdd-411e-8bdd-38ebbac1db50</a>	Right Click and Run as Administrator; Apply KB3210139 and KB3210131 Only
<b>KB4025341</b> <b>July Rollup 2017</b>	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=12c93ad9-ef0e-4ce6-8a1d-84713223d24a">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=12c93ad9-ef0e-4ce6-8a1d-84713223d24a</a>	This patch is required to be installed for the August rollup to successfully install.
KB4019112	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1daeb6d1-b103-4baa-bbde-5326e17e89e4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1daeb6d1-b103-4baa-bbde-5326e17e89e4</a>	<b>Run KB4014514 and KB4014504 only.</b> <b>For KB4014514, right click and Run as Administrator,</b>
	<b><u>Reboot Required</u></b>	
MS09-043 KB947318	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=22505">https://www.microsoft.com/en-us/download/details.aspx?id=22505</a>	<b>Acquisition Only</b>
MS11-049 KB2251487	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>	
MS12-034: KB2596672	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=29730">https://www.microsoft.com/en-us/download/details.aspx?id=29730</a>	<b>Acquisition Only</b>
MS13-075 KB2687413	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40196">https://www.microsoft.com/en-us/download/details.aspx?id=40196</a>	<b>GECW Client Review Only</b>
MS13-081 KB2862330	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40507">https://www.microsoft.com/en-us/download/details.aspx?id=40507</a>	
KB3138612	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5</a>	



## GE Healthcare

KB3172605	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8382aa41-9de1-4bb2-b8b0-4ab89451be64">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8382aa41-9de1-4bb2-b8b0-4ab89451be64</a>	
KB3179573	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb159d73-16f1-4c8a-bc3a-c768f6e2a7ce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb159d73-16f1-4c8a-bc3a-c768f6e2a7ce</a>	
	<b><u>Reboot Required</u></b>	
<b>Patches below can be applied in any order</b>		
kb947318	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ceff8ca0-08db-41d0-b825-fcc2ceba8b4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ceff8ca0-08db-41d0-b825-fcc2ceba8b4</a>	<b>Do not run on Review Systems</b>
KB3192391	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bc4fc430-38b5-4ef5-ba85-fb8254ac9be9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bc4fc430-38b5-4ef5-ba85-fb8254ac9be9</a>	
KB2538242	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb49cc19-8847-4986-aa93-5e905421e55a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb49cc19-8847-4986-aa93-5e905421e55a</a>	Select file title starts with x86
KB2538243	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428</a>	Install 2 files with x86 in title; Select Repair option if required
KB2565063	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=719584bc-2208-4bc9-a650-d3d6347eb32e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=719584bc-2208-4bc9-a650-d3d6347eb32e</a>	Install 2 files with x86 in title; Select Repair option if required
KB2863902	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=73ab9cea-fa20-4d92-9719-f01f13f613c1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=73ab9cea-fa20-4d92-9719-f01f13f613c1</a>	
KB2863926	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df24debf-341b-403b-9e0a-6cd01a025d8d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df24debf-341b-403b-9e0a-6cd01a025d8d</a>	
KB2920748	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ac78292-3a54-4100-837e-140fa85bb0bc">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ac78292-3a54-4100-837e-140fa85bb0bc</a>	
KB2863817	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0ab24c4-9edf-4faa-a69b-078470f6fd40">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0ab24c4-9edf-4faa-a69b-078470f6fd40</a>	Use all-convintl-en-us_.....cab



## GE Healthcare

KB3085528	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7ba6a6e-7884-4042-a2ce-f236c43a0989">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7ba6a6e-7884-4042-a2ce-f236c43a0989</a>	
KB2881029	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b158fbc-892b-4d1b-8c02-91682088ad72">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b158fbc-892b-4d1b-8c02-91682088ad72</a>	
kb2553432	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9badf612-4854-4023-9867-76e3677311af">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9badf612-4854-4023-9867-76e3677311af</a>	
kb3118390	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ecc22c5-46d4-40b2-a640-60e9da447657">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ecc22c5-46d4-40b2-a640-60e9da447657</a>	
kb3118378	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ae54ce3d-e321-4831-a1ba-fcae8eb430a0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ae54ce3d-e321-4831-a1ba-fcae8eb430a0</a>	
kb3127953	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=95499009-678d-4bc4-96d9-384781a18627">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=95499009-678d-4bc4-96d9-384781a18627</a>	
KB3161949	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=25c0fd19-d4e4-4af5-aad9-f308dde496d5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=25c0fd19-d4e4-4af5-aad9-f308dde496d5</a>	
KB2900986	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=82b62134-bafb-4fd3-815e-73534b9d1aa5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=82b62134-bafb-4fd3-815e-73534b9d1aa5</a>	
kb2850016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3188a7d3-91d0-4780-897f-1990c4b3e952">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3188a7d3-91d0-4780-897f-1990c4b3e952</a>	
KB3031432	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ca8e6e8-fc4a-4974-a208-18cdf1d01d86">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ca8e6e8-fc4a-4974-a208-18cdf1d01d86</a>	Apply kb3004375 and KB3031432
KB3059317	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9202ee09-24c9-4ae7-83bd-3b5d4b0e5c22">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9202ee09-24c9-4ae7-83bd-3b5d4b0e5c22</a>	
kb3101521	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=52d30a1b-2f14-4463-9034-92f46b76576d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=52d30a1b-2f14-4463-9034-92f46b76576d</a>	
kb2965313	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a4474e45-3c37-4708-9a5a-31e3538ddd44">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a4474e45-3c37-4708-9a5a-31e3538ddd44</a>	Use all-wordintl-en-us_.....cab
KB3156016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f21d3be8-dd35-4ac7-97b8-b4d06d4ed7f2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f21d3be8-dd35-4ac7-97b8-b4d06d4ed7f2</a>	



## GE Healthcare

KB3156019	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b5b7f2e4-bf3f-4974-ab23-f3a2ab886d31">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b5b7f2e4-bf3f-4974-ab23-f3a2ab886d31</a>	
KB3156017	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f6acb0e-47be-458e-8559-ab9f1179dfba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f6acb0e-47be-458e-8559-ab9f1179dfba</a>	
KB3159398	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2329e2b7-4e31-42e3-86aa-2ebfaa2c6339">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2329e2b7-4e31-42e3-86aa-2ebfaa2c6339</a>	
KB3161958	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fda0cfc-7ca1-4f78-ae19-90977c948ed3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fda0cfc-7ca1-4f78-ae19-90977c948ed3</a>	
KB3170455	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=65b4f1cb-3cd4-4546-8896-dc462d484282">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=65b4f1cb-3cd4-4546-8896-dc462d484282</a>	
KB3125869	<a href="https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015">https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015</a>	Download and install only the “Enable the User32 exception handler hardening feature in Internet Explorer”
KB2880971	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fe594d9e-9828-451f-aa56-2c2cf431ade3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fe594d9e-9828-451f-aa56-2c2cf431ade3</a>	
KB2810073	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=20c03f3c-cf50-4469-9e6a-c4f74622a160">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=20c03f3c-cf50-4469-9e6a-c4f74622a160</a>	Only Run .MSP file
KB3000483	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dbd91e47-4238-4bbe-8c8b-87c2d02c57d2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dbd91e47-4238-4bbe-8c8b-87c2d02c57d2</a>	Only Run 2 .MSU files
KB3054845	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd3cb1c0-dd41-45d0-a60d-794470b83761">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd3cb1c0-dd41-45d0-a60d-794470b83761</a>	
kb3054848	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=79ebf15b-3343-41fb-bf88-9bd6b4253fc0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=79ebf15b-3343-41fb-bf88-9bd6b4253fc0</a>	
kb3054835	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bd726abf-fa50-489d-ab30-fa13ee3e0ba0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bd726abf-fa50-489d-ab30-fa13ee3e0ba0</a>	
kb3054842	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc804f6e-c557-47e0-9f5e-4373b286677d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc804f6e-c557-47e0-9f5e-4373b286677d</a>	



## GE Healthcare

KB3055044	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=58454b4a-dd6b-4902-865b-c57ce21d3b91">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=58454b4a-dd6b-4902-865b-c57ce21d3b91</a>	
KB2553313	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f26e1911-9cf6-4082-8349-456230bc04ec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f26e1911-9cf6-4082-8349-456230bc04ec</a>	
KB2598244	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=aaa34820-0955-45db-b5a6-3048bb56843f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=aaa34820-0955-45db-b5a6-3048bb56843f</a>	
KB3055033	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ef6e2e39-225f-4cad-97bb-200d1fab3e53">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ef6e2e39-225f-4cad-97bb-200d1fab3e53</a>	
KB2965310	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fbe46296-974f-4eaf-ba1f-3f57787e932b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fbe46296-974f-4eaf-ba1f-3f57787e932b</a>	
KB3055039	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d5db1fcb-f98a-4b2d-abbc-003327425628">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d5db1fcb-f98a-4b2d-abbc-003327425628</a>	
kb3085560	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6f15ec3c-4e27-483d-baed-6e17fba8ffc7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6f15ec3c-4e27-483d-baed-6e17fba8ffc7</a>	Use all-convintl-en-us_.....cab
kb3085526	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c9e7d92-d1df-4cc4-8906-ff2429479d19">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c9e7d92-d1df-4cc4-8906-ff2429479d19</a>	
kb3101544	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5dddb0e6-6f66-4ae0-86e3-6a8d3d23a201">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5dddb0e6-6f66-4ae0-86e3-6a8d3d23a201</a>	
kb3101543	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=396a0e18-01af-4f19-a5ea-c5c9e352321f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=396a0e18-01af-4f19-a5ea-c5c9e352321f</a>	
kb3085594	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=74ccaa37-b9c6-4c4c-a65b-da83e3e6341a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=74ccaa37-b9c6-4c4c-a65b-da83e3e6341a</a>	
kb2817478	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b1bbd02b-d4c3-48c3-b648-1852de5d2c8e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b1bbd02b-d4c3-48c3-b648-1852de5d2c8e</a>	
kb3101520	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1e5f0476-1148-43b8-bfc4-10a1a29ddc32">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1e5f0476-1148-43b8-bfc4-10a1a29ddc32</a>	
kb3054984	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=35c6b5dc-c065-4d9a-a71f-9d0a6beb4df9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=35c6b5dc-c065-4d9a-a71f-9d0a6beb4df9</a>	





## GE Healthcare

kb3115123	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e8bd0027-368c-4ecf-bc9b-5c64195e2c53">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e8bd0027-368c-4ecf-bc9b-5c64195e2c53</a>	
kb3114400	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=98015a12-b9cd-48ab-bd7f-7dea92bb8f67">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=98015a12-b9cd-48ab-bd7f-7dea92bb8f67</a>	Use all-convintl-en-us_.....cab
kb3114869	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7efaa3ba-dee4-45ba-84b0-50c7aa10437b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7efaa3ba-dee4-45ba-84b0-50c7aa10437b</a>	
kb3114885	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ca07bc3-8c95-4644-be17-040a3964a02a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ca07bc3-8c95-4644-be17-040a3964a02a</a>	use all-onenoteint-en-us_....cab
kb3115474	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8bc2bfec-84bb-488e-ae91-f4814d409dba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8bc2bfec-84bb-488e-ae91-f4814d409dba</a>	use all-outlookintl-en-us_.....cab
kb3115471	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fd8a17d9-dad0-4f8d-95c4-0677ca0fbb08">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fd8a17d9-dad0-4f8d-95c4-0677ca0fbb08</a>	
kb3118309	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb00cc28-9cb3-437e-bac5-069bc32f8aeb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb00cc28-9cb3-437e-bac5-069bc32f8aeb</a>	
kb3128037	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=46311afb-a6ab-4fad-b6aa-4ec97507beba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=46311afb-a6ab-4fad-b6aa-4ec97507beba</a>	
kb3118380	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84ea23ce-49d8-4684-ac3b-45f0396053cf">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84ea23ce-49d8-4684-ac3b-45f0396053cf</a>	
kb3114395	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df66c1c1-7fab-4a43-886c-5203d1495bd8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df66c1c1-7fab-4a43-886c-5203d1495bd8</a>	
kb2889841	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb220b30-6d01-4e57-8db6-3e492d6b65d3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb220b30-6d01-4e57-8db6-3e492d6b65d3</a>	
kb3128034	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9cd72446-5c11-433a-9589-fd85afcc4eb0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9cd72446-5c11-433a-9589-fd85afcc4eb0</a>	
KB3178687	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=726adfc6-4ac9-4409-bdab-2892b7058e78">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=726adfc6-4ac9-4409-bdab-2892b7058e78</a>	
	<b>Reboot Required</b>	



## GE Healthcare

KB3045311	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11</a>	Only run x86 file; Run from a cmd prompt as Administrator using the following command: AMD64_X86-all-sqlserver2008-kb3045311-x86_30561aef89c6d174fee7b77bed6b3b8539542558.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
Adobe 11.0.20 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191</a>	
KB3178688	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=322c28f5-349c-468a-ac94-901616f52372">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=322c28f5-349c-468a-ac94-901616f52372</a>	
KB3191847	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4b4bbe2b-a25d-4509-a069-f5efc227b4ad">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4b4bbe2b-a25d-4509-a069-f5efc227b4ad</a>	
KB3191907	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c8533f11-51f9-4f84-96d8-c619947cc7c0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c8533f11-51f9-4f84-96d8-c619947cc7c0</a>	
KB3118310	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c59a1bb2-ff1f-427a-a8d7-2cab1cb3e7d1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c59a1bb2-ff1f-427a-a8d7-2cab1cb3e7d1</a>	Use en-us file all-convintl-en-us_....cab
KB3191843	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f715a81d-102d-416a-9a89-e9ebdace0a6d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f715a81d-102d-416a-9a89-e9ebdace0a6d</a>	
KB3191899	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7698c63a-b85f-4647-bcb1-1be0256c3f43">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7698c63a-b85f-4647-bcb1-1be0256c3f43</a>	
KB3203468	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a</a>	Use all-proof-en-us_.....cab
KB3213624	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3658f96e-a521-429d-a9a9-e70e30f5d830">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3658f96e-a521-429d-a9a9-e70e30f5d830</a>	
KB3178690	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=06e2c9fb-65b7-48f5-b6e2-58071f17f9bd">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=06e2c9fb-65b7-48f5-b6e2-58071f17f9bd</a>	



**Reboot Required**

KB4034664 August Rollup 2017 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e0a94bad-5b2c-4611-9066-24491ce9bb4f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e0a94bad-5b2c-4611-9066-24491ce9bb4f</a>	
	Make the Following Registry Change	
	<b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b>	
	<b>State=dword:00010000</b>	

**Windows 7 (6.9.6 R1 GECW Review)**

Link		Notes
<a href="https://downloadcenter.intel.com/download/22165/Intel-Graphics-Media-Accelerator-Driver-Windows-7-Windows-Vista-exe-">https://downloadcenter.intel.com/download/22165/Intel-Graphics-Media-Accelerator-Driver-Windows-7-Windows-Vista-exe-</a>		Display adapter driver update Win7Vista_151719.exe

**Windows 2008 R2 (INW Server)**

KB	Link	Notes
	Make the following Registry Change	
	<b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b>	
	<b>State=dword:00023c00</b>	



## GE Healthcare

KB2979596	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44278">https://www.microsoft.com/en-us/download/details.aspx?id=44278</a>	Run from cmd prompt with SQLServer2008SP4-KB2979596-x64-ENU.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
KB3020369	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=076f5c31-ea9c-41db-8640-081a79c79602">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=076f5c31-ea9c-41db-8640-081a79c79602</a>	
KB3138612	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ae1092c-f211-4873-aabd-9ee1a142acbb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ae1092c-f211-4873-aabd-9ee1a142acbb</a>	
KB2639308	<a href="http://www.microsoft.com/en-ph/download/details.aspx?id=28933">http://www.microsoft.com/en-ph/download/details.aspx?id=28933</a>	
<b>IE11</b>	<a href="http://download.microsoft.com/download/7/1/7/7179A150-F2D2-4502-9D70-4B59EA148EAA/IE11-Windows6.1-x64-en-us.exe">http://download.microsoft.com/download/7/1/7/7179A150-F2D2-4502-9D70-4B59EA148EAA/IE11-Windows6.1-x64-en-us.exe</a>	Install only .MSU file Right click and Run as Administrator. If installation fail with a message requesting to connect to Internet, then install from the command line with these parameter /quiet /closeprograms e.g. "IE11-Windows6.1-x64-en-us.exe /quiet /closeprograms" Note: It could take few retry attempts to install.
	<b><u>Reboot Required</u></b>	
kb3125574	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6147e6c1-663b-41bc-9582-9579343857d9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6147e6c1-663b-41bc-9582-9579343857d9</a>	



## GE Healthcare

	<b><u>Reboot Required</u></b>	
KB3172605	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3363f98b-78a3-44a7-93df-d770b2dd150a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3363f98b-78a3-44a7-93df-d770b2dd150a</a>	
KB3179573	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=366304fa-105b-4f48-a07e-d2e6bb274533">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=366304fa-105b-4f48-a07e-d2e6bb274533</a>	
KB4022719	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4aa2ed70-7c26-4727-bd11-6a98b6dd622c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4aa2ed70-7c26-4727-bd11-6a98b6dd622c</a>	
KB4025341 July Rollup 2017	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b2423c5b-0254-4747-88bb-ec1a785549cb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b2423c5b-0254-4747-88bb-ec1a785549cb</a>	This patch is required to be installed for the August rollup to successfully install.
	<b><u>Reboot Required</u></b>	
<b>OPTIONAL – INW Server Performance Enhancement</b>	Please see section – OPTIONAL - How to Install INW Server Performance Enhancement	
<b>HP INW Server NIC Driver (HP ProLiant ML350 G8)</b>	<a href="http://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_de344f22b2c245ec859d7821e3">http://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_de344f22b2c245ec859d7821e3</a> Please see section - How to Install HP NIC Driver on the INW Server (HP ProLiant ML350 G8)	
<b>HP INW Server NIC Driver (HP ProLiant DL370 G6)</b>	<a href="http://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_109438fbecd6411388820b630c">http://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_109438fbecd6411388820b630c</a> Please see section - How to Install HP NIC Driver on the INW Server (HP ProLiant DL370 G6)	
<b>MS09-043 KB947318</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=22505">https://www.microsoft.com/en-us/download/details.aspx?id=22505</a>	
<b>MS11-025 KB2538242</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26347">https://www.microsoft.com/en-us/download/details.aspx?id=26347</a>	Select file title starts with x86



## GE Healthcare

MS11-049 KB2251487	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>	
KB2539530	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26416">https://www.microsoft.com/en-us/download/details.aspx?id=26416</a>	
MS12-034: KB2596672	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=29730">https://www.microsoft.com/en-us/download/details.aspx?id=29730</a>	
MS13-081 KB2862330	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40372">https://www.microsoft.com/en-us/download/details.aspx?id=40372</a>	
KB967642	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=21751">https://www.microsoft.com/en-us/download/details.aspx?id=21751</a>	
MS13-090 KB2900986	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40921">https://www.microsoft.com/en-us/download/details.aspx?id=40921</a>	
KB2919469	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41785">https://www.microsoft.com/en-us/download/details.aspx?id=41785</a>	
KB2830477	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41730">https://www.microsoft.com/en-us/download/details.aspx?id=41730</a>	
KB2871997	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42765">https://www.microsoft.com/en-us/download/details.aspx?id=42765</a>	
MS14-024 KB2817330	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42862">https://www.microsoft.com/en-us/download/details.aspx?id=42862</a>	
MS14-036 KB2878233	<a href="https://www.microsoft.com/en-US/download/details.aspx?id=43330">https://www.microsoft.com/en-US/download/details.aspx?id=43330</a>	
KB3006137	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45889">https://www.microsoft.com/en-us/download/details.aspx?id=45889</a>	
MS15-037 KB3046269	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46499">https://www.microsoft.com/en-us/download/details.aspx?id=46499</a>	
	<b><u>Reboot Required</u></b>	
KB2901983	<a href="https://www.microsoft.com/en-in/download/details.aspx?id=42642">https://www.microsoft.com/en-in/download/details.aspx?id=42642</a>	



## GE Healthcare

MS14-072 KB2978128	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44686">https://www.microsoft.com/en-us/download/details.aspx?id=44686</a>	
KB2923545	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41984">https://www.microsoft.com/en-us/download/details.aspx?id=41984</a>	
KB2970228	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44060">https://www.microsoft.com/en-us/download/details.aspx?id=44060</a>	
KB2984981	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44482">https://www.microsoft.com/en-us/download/details.aspx?id=44482</a>	
	<b><u>Reboot Required</u></b>	
MS14-057 KB2972107	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44320">https://www.microsoft.com/en-us/download/details.aspx?id=44320</a>	
MS14-053 KB2972216	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44181">https://www.microsoft.com/en-us/download/details.aspx?id=44181</a>	
MS14-057 KB2979578	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44322">https://www.microsoft.com/en-us/download/details.aspx?id=44322</a>	
MS15-041 KB3037581	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46529">https://www.microsoft.com/en-us/download/details.aspx?id=46529</a>	
	<b><u>Reboot Required</u></b>	
KB2994023	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44451">https://www.microsoft.com/en-us/download/details.aspx?id=44451</a>	
	<b><u>Reboot Required</u></b>	



## GE Healthcare

Patches below can be applied in any order		
kb2894844	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=00e0ded8-dc85-4017-8b54-a1456c63a61b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=00e0ded8-dc85-4017-8b54-a1456c63a61b</a>	
KB3074230	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0baf87d-e8ec-4ddc-b8b0-89f668ee6504">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0baf87d-e8ec-4ddc-b8b0-89f668ee6504</a>	
KB3074550	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2f7719fa-8a6d-4b16-9c69-66447f308ee7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2f7719fa-8a6d-4b16-9c69-66447f308ee7</a>	
KB3097996	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=103a0bdd-e71c-4f33-8cf1-7b68198ad536">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=103a0bdd-e71c-4f33-8cf1-7b68198ad536</a>	
KB3098781	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ed6e065-7e37-44bb-8798-245649851dec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ed6e065-7e37-44bb-8798-245649851dec</a>	
KB3122656	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=428c2fef-afe5-46a7-a454-da2fa0e3af27">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=428c2fef-afe5-46a7-a454-da2fa0e3af27</a>	
KB3127229	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f49f137-0741-498e-9f7c-754862054221">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f49f137-0741-498e-9f7c-754862054221</a>	
KB3135996	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=24aeb988-b368-435b-923b-c09b8a2d3fa5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=24aeb988-b368-435b-923b-c09b8a2d3fa5</a>	
KB3142033	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eac1599f-2887-40ce-9cf4-0f364dc37343">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eac1599f-2887-40ce-9cf4-0f364dc37343</a>	
KB3159398	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=56a7a134-5ad6-43c2-aab1-e10d2fef8f7c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=56a7a134-5ad6-43c2-aab1-e10d2fef8f7c</a>	
KB3161949	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5bc0e07b-33ca-4ea2-8253-4f0dcd26783a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5bc0e07b-33ca-4ea2-8253-4f0dcd26783a</a>	
KB3170455	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=45460055-be25-47ad-ad0a-2a4711fcc74">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=45460055-be25-47ad-ad0a-2a4711fcc74</a>	





## GE Healthcare

KB3163251	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b053e5b-04a5-4f47-92c5-74f6caba0de7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b053e5b-04a5-4f47-92c5-74f6caba0de7</a>	
KB3125869	<a href="https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015">https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015</a>	Download and install only the "Enable the User32 exception handler hardening feature in Internet Explorer"
KB2538243	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428</a>	Only run X86-all-vcredist_x86....
KB3156016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=de32fc3d-60a4-4fc9-9588-0c404765940a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=de32fc3d-60a4-4fc9-9588-0c404765940a</a>	
KB3156019	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=05cb4345-49ea-4eca-8ba3-f870466c46c7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=05cb4345-49ea-4eca-8ba3-f870466c46c7</a>	
KB3161958	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=795571cf-bd8d-4602-bb28-11080e614333">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=795571cf-bd8d-4602-bb28-11080e614333</a>	
Adobe 11.0.20 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191</a>	
KB890830	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3ac4f294-f247-4771-813b-8f02b6f33526">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3ac4f294-f247-4771-813b-8f02b6f33526</a>	
HPSBMU03653 rev.1	<a href="https://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_083799d6dad34195bb47cb43c1">https://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_083799d6dad34195bb47cb43c1</a>	
	<b><u>Reboot Required</u></b>	
KB3070738	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=12e7d9ba-d1e3-4631-b83e-6bafd96d5421">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=12e7d9ba-d1e3-4631-b83e-6bafd96d5421</a>	
KB4019112	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dedea6da-e039-487b-8ec6-2729551f7165">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dedea6da-e039-487b-8ec6-2729551f7165</a>	Run KB4014514 and KB4014504 only. For KB4014514, right click and Run as Administrator .
KB3045311	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11</a>	Only run x64 file; Run from a cmd prompt as Administrator using the following command: AMD64-all-



		sqlserver2008-kb3045311-x64_37a197c60990d2e83e98d1090109a4ab3f2abe4b.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
	<b><u>Reboot Required</u></b>	
	<b>Make the following Registry Change</b>	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	<b>Apply only on Domain Controller</b>
	State=dword:00010000	
KB4034664 August Rollup 2017 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80f7899d-451d-4e3f-b54e-d488a06a3c58">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80f7899d-451d-4e3f-b54e-d488a06a3c58</a>	To successfully install this Rollup, you must uninstall July Rollup <b>KB4025341</b> and reboot before installing <b>KB4034664</b>
	[HKEY_LOCAL_MACHINE\System\CurrentControlSet\Services\NTDS\Parameters]  LdapEnforceChannelBinding=DWORD:1	Apply only on Domain Controller. Need starting July Rollup-KB4025341 (CVE-2017-8563)
	<b><u>Reboot Required</u></b>	

## MLCL v6.9.6 R1 only (Old Procedure)

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) It is expected that some patches listed will already be on the system.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
Operating System Platform	Windows Server 2008 R2 SP1	Windows 7 SP1	Windows 7 SP1	Windows 7 SP1
Patch	Download URL	Download URL	Download URL	Download URL
MS15-127 KB3100465 (Domain Controller only)	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=50127">https://www.microsoft.com/en-us/download/details.aspx?id=50127</a>	N/A	N/A	N/A
MS15-067 KB3069762	N/A	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=47833">https://www.microsoft.com/en-us/download/details.aspx?id=47833</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=47833">https://www.microsoft.com/en-us/download/details.aspx?id=47833</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=47833">https://www.microsoft.com/en-us/download/details.aspx?id=47833</a>
HP System Management Homepage HPSBMU03380 Version: 7.5.4.3(1 Apr 2016) <b>Superseded</b>	<a href="https://h20566.www2.hp.com/hpsc/swd/public/detail?idx=&amp;action=driverDocument&amp;itemLocale=&amp;swItemId=MTX_544617581c264c8eaafe6b273a&amp;mode">https://h20566.www2.hp.com/hpsc/swd/public/detail?idx=&amp;action=driverDocument&amp;itemLocale=&amp;swItemId=MTX_544617581c264c8eaafe6b273a&amp;mode</a>  Please see section - How to Install HP System Management Homepage Patch on the INW Server – HPSBMU03380  <b>NOTE:</b> HPSBMU03051 must be installed before HPSBMU03380.	N/A	N/A	N/A
<b>OPTIONAL – INW Server Performance Enhancement</b>	Please see section – OPTIONAL - How to Install INW Server Performance Enhancement	N/A	N/A	N/A



## GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
<b>Adobe Reader APSB15-15 Superseded</b>	<a href="ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/">ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/</a> Download file - AdbeRdrUpd11012_MUI.msp  Please see section – How to Install Adobe Reader APSB15-15	<a href="ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/">ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/</a> Download file - AdbeRdrUpd11012_MUI.msp  Please see section – How to Install Adobe Reader APSB15-15	<a href="ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/">ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/</a> Download file - AdbeRdrUpd11012_MUI.msp  Please see section – How to Install Adobe Reader APSB15-15	<a href="ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/">ftp://ftp.adobe.com/pub/adobe/reader/win/11.x/11.0.12/misc/</a> Download file - AdbeRdrUpd11012_MUI.msp  Please see section – How to Install Adobe Reader APSB15-15
<b>HP INW Server NIC Driver (HP ProLiant ML350 G8)</b>	<a href="http://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_de344f22b2c245ec859d7821e3">http://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_de344f22b2c245ec859d7821e3</a>  Please see section - How to Install HP NIC Driver on the INW Server (HP ProLiant ML350 G8)	N/A	N/A	N/A
<b>HP INW Server NIC Driver (HP ProLiant DL370 G6)</b>	<a href="http://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_109438fbecdc6411388820b630c">http://h20566.www2.hpe.com/hpsc/swd/public/detail?swItemId=MTX_109438fbecdc6411388820b630c</a>  Please see section - How to Install HP NIC Driver on the INW Server (HP ProLiant DL370 G6)	N/A	N/A	N/A
<b>MS14-049 KB2918614</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43764">https://www.microsoft.com/en-us/download/details.aspx?id=43764</a>			
<b>MS09-043 KB947318</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=22505">https://www.microsoft.com/en-us/download/details.aspx?id=22505</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=22505">https://www.microsoft.com/en-us/download/details.aspx?id=22505</a>		
<b>MS11-025 KB2538242</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26347">https://www.microsoft.com/en-us/download/details.aspx?id=26347</a>			Select file title starts with x86
<b>MS11-049 KB2251487</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26419">https://www.microsoft.com/en-us/download/details.aspx?id=26419</a>



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
KB2539530	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=26416">https://www.microsoft.com/en-us/download/details.aspx?id=26416</a>			
MS12-034: KB2596672	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=29730">https://www.microsoft.com/en-us/download/details.aspx?id=29730</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=29730">https://www.microsoft.com/en-us/download/details.aspx?id=29730</a>	N/A	N/A
MS13-075 KB2687413	N/A	N/A	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40196">https://www.microsoft.com/en-us/download/details.aspx?id=40196</a>	N/A
KB2852386	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42739">https://www.microsoft.com/en-us/download/details.aspx?id=42739</a>			
MS13-081 KB2862330	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40372">https://www.microsoft.com/en-us/download/details.aspx?id=40372</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40507">https://www.microsoft.com/en-us/download/details.aspx?id=40507</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40507">https://www.microsoft.com/en-us/download/details.aspx?id=40507</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40507">https://www.microsoft.com/en-us/download/details.aspx?id=40507</a>
KB967642	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=21751">https://www.microsoft.com/en-us/download/details.aspx?id=21751</a>			
KB2862152	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41018">https://www.microsoft.com/en-us/download/details.aspx?id=41018</a>			
KB2893519	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41017">https://www.microsoft.com/en-us/download/details.aspx?id=41017</a>			
MS13-090 KB2900986	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40921">https://www.microsoft.com/en-us/download/details.aspx?id=40921</a>			
MS13-095 KB2868626	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=40930">https://www.microsoft.com/en-us/download/details.aspx?id=40930</a>			
KB2891804	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41268">https://www.microsoft.com/en-us/download/details.aspx?id=41268</a>			
MS13-101 KB2887069	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41276">https://www.microsoft.com/en-us/download/details.aspx?id=41276</a>			
MS13-099 KB2892074	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41226">https://www.microsoft.com/en-us/download/details.aspx?id=41226</a>			
MS13-098 KB2893294	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41217">https://www.microsoft.com/en-us/download/details.aspx?id=41217</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
MS14-009 KB2911501	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41835">https://www.microsoft.com/en-us/download/details.aspx?id=41835</a>			
KB2919469	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41785">https://www.microsoft.com/en-us/download/details.aspx?id=41785</a>			
MS14-007 KB2912390	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41758">https://www.microsoft.com/en-us/download/details.aspx?id=41758</a>			
KB2830477	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41730">https://www.microsoft.com/en-us/download/details.aspx?id=41730</a>			
KB2929733	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42175">https://www.microsoft.com/en-us/download/details.aspx?id=42175</a>			
KB2908783	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42340">https://www.microsoft.com/en-us/download/details.aspx?id=42340</a>			
KB2928562	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42331">https://www.microsoft.com/en-us/download/details.aspx?id=42331</a>			
KB2843630	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41706">https://www.microsoft.com/en-us/download/details.aspx?id=41706</a>			
KB2871997	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42765">https://www.microsoft.com/en-us/download/details.aspx?id=42765</a>			
MS14-026 KB2931356	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42768">https://www.microsoft.com/en-us/download/details.aspx?id=42768</a>			
MS14-024 KB2817330	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42862">https://www.microsoft.com/en-us/download/details.aspx?id=42862</a>			
MS14-036 KB2957509	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43180">https://www.microsoft.com/en-us/download/details.aspx?id=43180</a>			
MS14-036 KB2878233	<a href="https://www.microsoft.com/en-US/download/details.aspx?id=43330">https://www.microsoft.com/en-US/download/details.aspx?id=43330</a>			
MS14-033 KB2939576	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43162">https://www.microsoft.com/en-us/download/details.aspx?id=43162</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
<b>MS14-031 KB2957189</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43143">https://www.microsoft.com/en-us/download/details.aspx?id=43143</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43147">https://www.microsoft.com/en-us/download/details.aspx?id=43147</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43147">https://www.microsoft.com/en-us/download/details.aspx?id=43147</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43147">https://www.microsoft.com/en-us/download/details.aspx?id=43147</a>
<b>HP System Management Homepage HPSBMU03051</b> <b>Version: 7.3.3.1(19 Jun 2014)</b> <b>Superseded</b>	<a href="http://h20564.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_e8076c2a35804685ad65b2b1ba">http://h20564.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_e8076c2a35804685ad65b2b1ba</a>  Please see section - How to Install HP System Management Homepage Patch on the INW Server – HPSBMU03051	N/A	N/A	N/A
<b>MS14-036 KB2957503</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43194">https://www.microsoft.com/en-us/download/details.aspx?id=43194</a>			
<b>KB2800095</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42363">https://www.microsoft.com/en-us/download/details.aspx?id=42363</a>			
<b>MS14-040 KB2961072</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43459">https://www.microsoft.com/en-us/download/details.aspx?id=43459</a>			
<b>MS14-039 KB2973201</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43475">https://www.microsoft.com/en-us/download/details.aspx?id=43475</a>			
<b>KB2973351</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43471">https://www.microsoft.com/en-us/download/details.aspx?id=43471</a>			
<b>MS14-046 KB2937610</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43954">https://www.microsoft.com/en-us/download/details.aspx?id=43954</a>			
<b>MS14-046 KB2943357</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43930">https://www.microsoft.com/en-us/download/details.aspx?id=43930</a>			
<b>MS14-045 KB2976897</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43885">https://www.microsoft.com/en-us/download/details.aspx?id=43885</a>			
<b>MS14-047 KB2978668</b>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43842">https://www.microsoft.com/en-us/download/details.aspx?id=43842</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
KB2966583	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=43470">https://www.microsoft.com/en-us/download/details.aspx?id=43470</a>			
MS14-045 KB2993651	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44040">https://www.microsoft.com/en-us/download/details.aspx?id=44040</a>			
MS14-053 KB2973112	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44184">https://www.microsoft.com/en-us/download/details.aspx?id=44184</a>			
KB2977728	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44086">https://www.microsoft.com/en-us/download/details.aspx?id=44086</a>			
KB2978092	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44131">https://www.microsoft.com/en-us/download/details.aspx?id=44131</a>			
KB2985461	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44100">https://www.microsoft.com/en-us/download/details.aspx?id=44100</a>			
KB2984972	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44336">https://www.microsoft.com/en-us/download/details.aspx?id=44336</a>			
MS14-057 KB2972100	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44312">https://www.microsoft.com/en-us/download/details.aspx?id=44312</a>			
MS14-057 KB2979570	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44327">https://www.microsoft.com/en-us/download/details.aspx?id=44327</a>			
KB2977292	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44335">https://www.microsoft.com/en-us/download/details.aspx?id=44335</a>			
MS14-072 KB2978120	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44603">https://www.microsoft.com/en-us/download/details.aspx?id=44603</a>			
MS14-078 KB2991963	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44708">https://www.microsoft.com/en-us/download/details.aspx?id=44708</a>			
MS14-066 KB2992611	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44618">https://www.microsoft.com/en-us/download/details.aspx?id=44618</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44633">https://www.microsoft.com/en-us/download/details.aspx?id=44633</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44633">https://www.microsoft.com/en-us/download/details.aspx?id=44633</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44633">https://www.microsoft.com/en-us/download/details.aspx?id=44633</a>
MS14-071 KB3005607	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44778">https://www.microsoft.com/en-us/download/details.aspx?id=44778</a>			





## GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
MS14-064 KB3006226	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44770">https://www.microsoft.com/en-us/download/details.aspx?id=44770</a>			
MS14-064 KB3010788	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44861">https://www.microsoft.com/en-us/download/details.aspx?id=44861</a>			
MS14-074 KB3003743	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44852">https://www.microsoft.com/en-us/download/details.aspx?id=44852</a>			
MS14-068 KB3011780	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44966">https://www.microsoft.com/en-us/download/details.aspx?id=44966</a>			
KB3006625	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45043">https://www.microsoft.com/en-us/download/details.aspx?id=45043</a>			
KB3013410	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45160">https://www.microsoft.com/en-us/download/details.aspx?id=45160</a>			
KB3014406	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45191">https://www.microsoft.com/en-us/download/details.aspx?id=45191</a>			
MS15-007 KB3014029	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45351">https://www.microsoft.com/en-us/download/details.aspx?id=45351</a>			
MS15-003 KB3021674	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45417">https://www.microsoft.com/en-us/download/details.aspx?id=45417</a>			
MS15-005 KB3022777	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45479">https://www.microsoft.com/en-us/download/details.aspx?id=45479</a>			
KB3020338	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45714">https://www.microsoft.com/en-us/download/details.aspx?id=45714</a>			
MS15-014 KB3004361	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45565">https://www.microsoft.com/en-us/download/details.aspx?id=45565</a>			
KB3006137	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45889">https://www.microsoft.com/en-us/download/details.aspx?id=45889</a>			
MS15-027 KB3002657	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46200">https://www.microsoft.com/en-us/download/details.aspx?id=46200</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
MS15-028 KB3030377	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46126">https://www.microsoft.com/en-us/download/details.aspx?id=46126</a>			
MS15-021 KB3032323	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45969">https://www.microsoft.com/en-us/download/details.aspx?id=45969</a>			
MS15-020 KB3033889	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46153">https://www.microsoft.com/en-us/download/details.aspx?id=46153</a>			
KB3033929	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46083">https://www.microsoft.com/en-us/download/details.aspx?id=46083</a>			
MS15-023 KB3034344	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45982">https://www.microsoft.com/en-us/download/details.aspx?id=45982</a>			
MS15-029 KB3035126	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=45962">https://www.microsoft.com/en-us/download/details.aspx?id=45962</a>			
MS15-024 KB3035132	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46133">https://www.microsoft.com/en-us/download/details.aspx?id=46133</a>			
MS15-020 KB3039066	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46275">https://www.microsoft.com/en-us/download/details.aspx?id=46275</a>			
MS15-041 KB3037574	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46528">https://www.microsoft.com/en-us/download/details.aspx?id=46528</a>			
MS15-038 KB3045685	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46461">https://www.microsoft.com/en-us/download/details.aspx?id=46461</a>			
MS15-035 KB3046306	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46495">https://www.microsoft.com/en-us/download/details.aspx?id=46495</a>			
MS15-034 KB3042553	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46480">https://www.microsoft.com/en-us/download/details.aspx?id=46480</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46501">https://www.microsoft.com/en-us/download/details.aspx?id=46501</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46501">https://www.microsoft.com/en-us/download/details.aspx?id=46501</a>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46501">https://www.microsoft.com/en-us/download/details.aspx?id=46501</a>
MS15-037 KB3046269	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46499">https://www.microsoft.com/en-us/download/details.aspx?id=46499</a>			
MS15-039 KB3046482	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46479">https://www.microsoft.com/en-us/download/details.aspx?id=46479</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
MS15-038 KB3045999	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46556">https://www.microsoft.com/en-us/download/details.aspx?id=46556</a>			
KB3020370	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46737">https://www.microsoft.com/en-us/download/details.aspx?id=46737</a>			
KB3048761	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46795">https://www.microsoft.com/en-us/download/details.aspx?id=46795</a>			
	<b><u>Reboot Required</u></b>			
	<p>Make the following Registry Change</p> <p><b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b></p> <p><b>State=dword:00023c00</b></p>			
KB2901907	<p><a href="https://www.microsoft.com/en-in/download/details.aspx?id=42642">https://www.microsoft.com/en-in/download/details.aspx?id=42642</a></p> <p><b>Right click and Run as Administrator</b></p>			
	<p>Make the following Registry Change</p> <p><b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b></p> <p><b>State=dword:00010000</b></p>			
MS14-072 KB2978128	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44686">https://www.microsoft.com/en-us/download/details.aspx?id=44686</a>			



GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
KB2923545	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=41984">https://www.microsoft.com/en-us/download/details.aspx?id=41984</a>			
KB2970228	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44060">https://www.microsoft.com/en-us/download/details.aspx?id=44060</a>			
KB2984981	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44482">https://www.microsoft.com/en-us/download/details.aspx?id=44482</a>			
	<b><u>Reboot Required</u></b>			
MS14-057 KB2972107	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44320">https://www.microsoft.com/en-us/download/details.aspx?id=44320</a>			
MS14-053 KB2972216	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44181">https://www.microsoft.com/en-us/download/details.aspx?id=44181</a>			
MS14-057 KB2979578	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44322">https://www.microsoft.com/en-us/download/details.aspx?id=44322</a>			
MS15-041 KB3037581	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46529">https://www.microsoft.com/en-us/download/details.aspx?id=46529</a>			
	<b><u>Reboot Required</u></b>			
KB2994023	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44451">https://www.microsoft.com/en-us/download/details.aspx?id=44451</a>			
KB3008627	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44850">https://www.microsoft.com/en-us/download/details.aspx?id=44850</a>			
KB3020369	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46828">https://www.microsoft.com/en-us/download/details.aspx?id=46828</a>			
KB3045645	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=46806">https://www.microsoft.com/en-us/download/details.aspx?id=46806</a>			
MS14-025 KB2928120	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42683">https://www.microsoft.com/en-us/download/details.aspx?id=42683</a>			



	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
Domain Controller Only				
	<b><u>Reboot Required</u></b>			
KB2577795	<a href="https://support.microsoft.com/en-us/kb/2577795">https://support.microsoft.com/en-us/kb/2577795</a>			

## MLCL V6.9.6 2017 Patch Updates 1

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

1. The above patches are required patches for 6.9.6 and must be applied first.
2. It is expected that some patches listed will already be on the system.
3. Pay attention to the Notes section for special handling instructions
4. Patches must be applied in order except where indicated.
5. Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.
6. Patches will not install if the software component to be patched is not present (such as an IE8 patch on a system that does not have IE8 installed).

**Note:** KB4022719 replaces KB4012215 and KB4019263 for both Windows 7 and Windows Server 2008 R2 to address CVE-2017-8543, CVE-2017-8544, CVE-2017-0277, CVE-2017-0276, CVE-2017-0275, CVE-2017-0274, CVE-2017-0272, CVE-2017-0271, CVE-2017-0270, CVE-2017-0269, CVE-2017-0268, CVE-2017-0171, CVE-2017-0280, CVE-2017-0279 and CVE-2017-0278.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	



## GE Healthcare

	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
KB2901907	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=42642">https://www.microsoft.com/en-us/download/details.aspx?id=42642</a>	Right click and Run as Administrator; Verify that .Net 4.5.2 shows in Control Panel\Programs and Features
Subsequent patches depend on installation of KB2901907 .		
KB2979596	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44278">https://www.microsoft.com/en-us/download/details.aspx?id=44278</a>	Run from cmd prompt with SQLServer2008SP4-KB2979596-x86-ENU.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
KB3020369	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f00e3c36-f5e3-465c-95d2-a84a22425868">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f00e3c36-f5e3-465c-95d2-a84a22425868</a>	
KB3138612	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0bae11c4-626f-4b7e-b539-06c95cb014d5</a>	
KB2639308	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=28929">https://www.microsoft.com/en-us/download/details.aspx?id=28929</a>	Install only .MSU file
IE11	<a href="http://download.microsoft.com/download/9/2/F/92FC119C-3BCD-476C-B425-038A39625558/IE11-Windows6.1-x86-en-us.exe">http://download.microsoft.com/download/9/2/F/92FC119C-3BCD-476C-B425-038A39625558/IE11-Windows6.1-x86-en-us.exe</a>	Right click and Run as Administrator. If installation fail with a message requesting to connect to Internet, then install from the command line with these parameter /quiet /closeprogra



# GE Healthcare

		ms e.g. "IE11-Windows6.1-x86-en-us.exe /quiet /closeprograms" Note: It could take few retry attempts to install.
	<b>Reboot Required</b>	
KB3125574	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ed4c8c3-0f06-4227-99e3-e9f143394687">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ed4c8c3-0f06-4227-99e3-e9f143394687</a>	
	<b>Reboot Required</b>	
KB3172605	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8382aa41-9de1-4bb2-b8b0-4ab89451be64">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8382aa41-9de1-4bb2-b8b0-4ab89451be64</a>	
Adobe 11.0.19 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/detail.jsp?ftpID=6123">http://supportdownloads.adobe.com/detail.jsp?ftpID=6123</a>	
KB3179573	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb159d73-16f1-4c8a-bc3a-c768f6e2a7ce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb159d73-16f1-4c8a-bc3a-c768f6e2a7ce</a>	
KB4022719 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b9815359-6aad-467e-8666-2351fad3c45">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b9815359-6aad-467e-8666-2351fad3c45</a>	
KB3205402 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8938e57-3bdd-411e-8bdd-38ebbac1db50">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8938e57-3bdd-411e-8bdd-38ebbac1db50</a>	Right Click and Run as Administrator; Apply KB3210139 and KB3210131 Only
	<b>Reboot Required</b>	
<b>Patches below can be applied</b>		



## GE Healthcare

in any order		
kb947318	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ceff8ca0-08db-41d0-b825-fcc2cebaf8b4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ceff8ca0-08db-41d0-b825-fcc2cebaf8b4</a>	<b>Do not run on Review Systems</b>
KB3192391	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bc4fc430-38b5-4ef5-ba85-fb8254ac9be9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bc4fc430-38b5-4ef5-ba85-fb8254ac9be9</a>	
KB2538242	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb49cc19-8847-4986-aa93-5e905421e55a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb49cc19-8847-4986-aa93-5e905421e55a</a>	Select file title starts with x86
KB2538243	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428</a>	Install files with x86 in title; Select Repair option if required
KB2565063	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=719584bc-2208-4bc9-a650-d3d6347eb32e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=719584bc-2208-4bc9-a650-d3d6347eb32e</a>	Install files with x86 in title; Select Repair option if required
KB2863902	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=73ab9cea-fa20-4d92-9719-f01f13f613c1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=73ab9cea-fa20-4d92-9719-f01f13f613c1</a>	
KB2863926	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df24debf-341b-403b-9e0a-6cd01a025d8d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df24debf-341b-403b-9e0a-6cd01a025d8d</a>	
KB2920748	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ac78292-3a54-4100-837e-140fa85bb0bc">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7ac78292-3a54-4100-837e-140fa85bb0bc</a>	
KB2863817	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0ab24c4-9edf-4faa-a69b-078470f6fd40">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0ab24c4-9edf-4faa-a69b-078470f6fd40</a>	Use all-convintl-en-us_.....cab
<b>KB3104002</b> <span style="background-color: red; color: white; padding: 2px;">Superseded</span>	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=50346">https://www.microsoft.com/en-us/download/details.aspx?id=50346</a>	
KB3085528	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7ba6a6e-7884-4042-a2ce-f236c43a0989">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7ba6a6e-7884-4042-a2ce-f236c43a0989</a>	
KB2881029	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b158fbc-892b-4d1b-8c02-91682088ad72">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b158fbc-892b-4d1b-8c02-91682088ad72</a>	
kb2553432	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9badf612-4854-4023-9867-76e3677311af">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9badf612-4854-4023-9867-76e3677311af</a>	





## GE Healthcare

kb3118390	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ecc22c5-46d4-40b2-a640-60e9da447657">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ecc22c5-46d4-40b2-a640-60e9da447657</a>	
kb3118378	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ae54ce3d-e321-4831-a1ba-fcae8eb430a0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ae54ce3d-e321-4831-a1ba-fcae8eb430a0</a>	
kb3127953	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=95499009-678d-4bc4-96d9-384781a18627">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=95499009-678d-4bc4-96d9-384781a18627</a>	
KB3161949	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=25c0fd19-d4e4-4af5-aad9-f308dde496d5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=25c0fd19-d4e4-4af5-aad9-f308dde496d5</a>	
KB2900986	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=82b62134-bafb-4fd3-815e-73534b9d1aa5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=82b62134-bafb-4fd3-815e-73534b9d1aa5</a>	
kb2850016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3188a7d3-91d0-4780-897f-1990c4b3e952">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3188a7d3-91d0-4780-897f-1990c4b3e952</a>	
KB3031432	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ca8e6e8-fc4a-4974-a208-18cdf1d01d86">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ca8e6e8-fc4a-4974-a208-18cdf1d01d86</a>	Apply kb3004375 and KB3031432
KB3059317	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9202ee09-24c9-4ae7-83bd-3b5d4b0e5c22">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9202ee09-24c9-4ae7-83bd-3b5d4b0e5c22</a>	
kb3101521	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=52d30a1b-2f14-4463-9034-92f46b76576d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=52d30a1b-2f14-4463-9034-92f46b76576d</a>	
kb2965313	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a4474e45-3c37-4708-9a5a-31e3538ddd44">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a4474e45-3c37-4708-9a5a-31e3538ddd44</a>	Use all-wordintl-en-us_.....cab
KB3156016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f21d3be8-dd35-4ac7-97b8-b4d06d4ed7f2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f21d3be8-dd35-4ac7-97b8-b4d06d4ed7f2</a>	
KB3156019	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b5b7f2e4-bf3f-4974-ab23-f3a2ab886d31">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b5b7f2e4-bf3f-4974-ab23-f3a2ab886d31</a>	
KB3156017	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f6acb0e-47be-458e-8559-ab9f1179dfba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f6acb0e-47be-458e-8559-ab9f1179dfba</a>	
KB3159398	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2329e2b7-4e31-42e3-86aa-2ebfaa2c6339">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2329e2b7-4e31-42e3-86aa-2ebfaa2c6339</a>	



## GE Healthcare

KB3161958	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fda0cfc-7ca1-4f78-ae19-90977c948ed3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fda0cfc-7ca1-4f78-ae19-90977c948ed3</a>	
KB3170455	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=65b4f1cb-3cd4-4546-8896-dc462d484282">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=65b4f1cb-3cd4-4546-8896-dc462d484282</a>	
KB3177186	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=88aaa062-4a7b-4690-ac06-8527d3db830d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=88aaa062-4a7b-4690-ac06-8527d3db830d</a>	
KB3125869	<a href="https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015">https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015</a>	Download and install only the “Enable the User32 exception handler hardening feature in Internet Explorer”
KB2880971	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fe594d9e-9828-451f-aa56-2c2cf431ade3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fe594d9e-9828-451f-aa56-2c2cf431ade3</a>	
KB2810073	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=20c03f3c-cf50-4469-9e6a-c4f74622a160">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=20c03f3c-cf50-4469-9e6a-c4f74622a160</a>	Only Run .MSP file
KB3000483	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dbd91e47-4238-4bbe-8c8b-87c2d02c57d2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dbd91e47-4238-4bbe-8c8b-87c2d02c57d2</a>	Only Run 2 .MSU files
KB3054845	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd3cb1c0-dd41-45d0-a60d-794470b83761">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd3cb1c0-dd41-45d0-a60d-794470b83761</a>	
kb3054848	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=79ebf15b-3343-41fb-bf88-9bd6b4253fc0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=79ebf15b-3343-41fb-bf88-9bd6b4253fc0</a>	
kb3054835	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bd726abf-fa50-489d-ab30-fa13ee3e0ba0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bd726abf-fa50-489d-ab30-fa13ee3e0ba0</a>	
kb3054842	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc804f6e-c557-47e0-9f5e-4373b286677d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc804f6e-c557-47e0-9f5e-4373b286677d</a>	
KB3055044	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=58454b4a-dd6b-4902-865b-c57ce21d3b91">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=58454b4a-dd6b-4902-865b-c57ce21d3b91</a>	
KB2553313	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f26e1911-9cf6-4082-8349-456230bc04ec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f26e1911-9cf6-4082-8349-456230bc04ec</a>	



## GE Healthcare

KB2598244	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=aaa34820-0955-45db-b5a6-3048bb56843f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=aaa34820-0955-45db-b5a6-3048bb56843f</a>	
KB3055033	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ef6e2e39-225f-4cad-97bb-200d1fab3e53">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ef6e2e39-225f-4cad-97bb-200d1fab3e53</a>	
KB2965310	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fbe46296-974f-4eaf-ba1f-3f57787e932b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fbe46296-974f-4eaf-ba1f-3f57787e932b</a>	
KB3055039	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d5db1fcb-f98a-4b2d-abbc-003327425628">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d5db1fcb-f98a-4b2d-abbc-003327425628</a>	
kb3085560	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6f15ec3c-4e27-483d-baed-6e17fba8ffc7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6f15ec3c-4e27-483d-baed-6e17fba8ffc7</a>	Use all-convintl-en-us_.....cab
kb3085526	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c9e7d92-d1df-4cc4-8906-ff2429479d19">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c9e7d92-d1df-4cc4-8906-ff2429479d19</a>	
kb3101544	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ddd0e6-6f66-4ae0-86e3-6a8d3d23a201">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ddd0e6-6f66-4ae0-86e3-6a8d3d23a201</a>	
kb3101543	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=396a0e18-01af-4f19-a5ea-c5c9e352321f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=396a0e18-01af-4f19-a5ea-c5c9e352321f</a>	
kb3085594	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=74ccaa37-b9c6-4c4c-a65b-da83e3e6341a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=74ccaa37-b9c6-4c4c-a65b-da83e3e6341a</a>	
kb2817478	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b1bbd02b-d4c3-48c3-b648-1852de5d2c8e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b1bbd02b-d4c3-48c3-b648-1852de5d2c8e</a>	
kb3101520	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1e5f0476-1148-43b8-bfc4-10a1a29ddc32">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1e5f0476-1148-43b8-bfc4-10a1a29ddc32</a>	
kb3054984	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=35c6b5dc-c065-4d9a-a71f-9d0a6beb4df9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=35c6b5dc-c065-4d9a-a71f-9d0a6beb4df9</a>	
kb3115123	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e8bd0027-368c-4ecf-bc9b-5c64195e2c53">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e8bd0027-368c-4ecf-bc9b-5c64195e2c53</a>	
kb3114400	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=98015a12-b9cd-48ab-bd7f-7dea92bb8f67">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=98015a12-b9cd-48ab-bd7f-7dea92bb8f67</a>	Use all-convintl-en-us_.....cab



## GE Healthcare

kb3114869	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7efaa3ba-dee4-45ba-84b0-50c7aa10437b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7efaa3ba-dee4-45ba-84b0-50c7aa10437b</a>	
kb3114885	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ca07bc3-8c95-4644-be17-040a3964a02a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ca07bc3-8c95-4644-be17-040a3964a02a</a>	use all-onenoteint-en-us_....cab
kb3115474	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8bc2bfec-84bb-488e-ae91-f4814d409dba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8bc2bfec-84bb-488e-ae91-f4814d409dba</a>	use all-outlookintl-en-us_.....cab
kb3115471	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fd8a17d9-dad0-4f8d-95c4-0677ca0fbb08">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fd8a17d9-dad0-4f8d-95c4-0677ca0fbb08</a>	
kb3118309	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb00cc28-9cb3-437e-bac5-069bc32f8aeb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fb00cc28-9cb3-437e-bac5-069bc32f8aeb</a>	
kb3128037	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=46311afb-a6ab-4fad-b6aa-4ec97507beba">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=46311afb-a6ab-4fad-b6aa-4ec97507beba</a>	
kb3118380	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84ea23ce-49d8-4684-ac3b-45f0396053cf">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84ea23ce-49d8-4684-ac3b-45f0396053cf</a>	
kb3114395	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df66c1c1-7fab-4a43-886c-5203d1495bd8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=df66c1c1-7fab-4a43-886c-5203d1495bd8</a>	
kb2889841	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb220b30-6d01-4e57-8db6-3e492d6b65d3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bb220b30-6d01-4e57-8db6-3e492d6b65d3</a>	
kb3128034	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9cd72446-5c11-433a-9589-fd85afcc4eb0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9cd72446-5c11-433a-9589-fd85afcc4eb0</a>	
KB3178687	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=726adfc6-4ac9-4409-bdab-2892b7058e78">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=726adfc6-4ac9-4409-bdab-2892b7058e78</a>	
	<b><u>Reboot Required</u></b>	
KB3045311	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11</a>	Only run x86 file; Run from a cmd prompt as Administrator using the following command: AMD64_X86-all-sqlserver2008-kb3045311-



GE Healthcare

		x86_30561aef89c6d174fee7b77bed6b3b8539542558.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
	<b><u>Reboot Required</u></b>	
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	

**Windows 7 (6.9.6 R1 GECW Review)**

	Link	Notes
	<a href="https://downloadcenter.intel.com/download/22165/Intel-Graphics-Media-Accelerator-Driver-Windows-7-Windows-Vista-exe-">https://downloadcenter.intel.com/download/22165/Intel-Graphics-Media-Accelerator-Driver-Windows-7-Windows-Vista-exe-</a>	Display adapter driver update Win7Vista_151719.exe

**Windows 2008R2 (INW Server)**

KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	



## GE Healthcare

KB2979596	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=44278">https://www.microsoft.com/en-us/download/details.aspx?id=44278</a>	Run from cmd prompt with SQLServer2008SP4-KB2979596- x64-ENU.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
KB3020369	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=076f5c31-ea9c-41db-8640-081a79c79602">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=076f5c31-ea9c-41db-8640-081a79c79602</a>	
KB3138612	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ae1092c-f211-4873-aabd-9ee1a142acbb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5ae1092c-f211-4873-aabd-9ee1a142acbb</a>	
KB2639308	<a href="http://www.microsoft.com/en-ph/download/details.aspx?id=28933">http://www.microsoft.com/en-ph/download/details.aspx?id=28933</a>	Install only .MSU file
IE11	<a href="http://download.microsoft.com/download/7/1/7/7179A150-F2D2-4502-9D70-4B59EA148EAA/IE11-Windows6.1-x64-en-us.exe">http://download.microsoft.com/download/7/1/7/7179A150-F2D2-4502-9D70-4B59EA148EAA/IE11-Windows6.1-x64-en-us.exe</a>	Right click and Run as Administrator. If installation fail with a message requesting to connect to Internet, then install from the command line with these parameter /quiet /closeprograms e.g. "IE11-Windows6.1-x64-en-us.exe /quiet /closeprograms" Note: It could take few retry attempts to install.
	<b><u>Reboot Required</u></b>	
kb3125574	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6147e6c1-663b-41bc-9582-9579343857d9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6147e6c1-663b-41bc-9582-9579343857d9</a>	
	<b><u>Reboot Required</u></b>	



## GE Healthcare

Adobe 11.0.19 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/detail.jsp?ftplD=6123">http://supportdownloads.adobe.com/detail.jsp?ftplD=6123</a>	
KB3172605	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3363f98b-78a3-44a7-93df-d770b2dd150a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3363f98b-78a3-44a7-93df-d770b2dd150a</a>	
KB3179573	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=366304fa-105b-4f48-a07e-d2e6bb274533">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=366304fa-105b-4f48-a07e-d2e6bb274533</a>	
KB4022719	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4aa2ed70-7c26-4727-bd11-6a98b6dd622c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4aa2ed70-7c26-4727-bd11-6a98b6dd622c</a>	
	<b><u>Reboot Required</u></b>	
<b>Patches below can be applied in any order</b>		
kb2894844	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=00e0ded8-dc85-4017-8b54-a1456c63a61b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=00e0ded8-dc85-4017-8b54-a1456c63a61b</a>	
KB3205402 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5036bd76-3251-49ba-ad86-a99b00853ad4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5036bd76-3251-49ba-ad86-a99b00853ad4</a>	Only apply KB3210139 (Right click and run as Administrator) & KB3210131
KB2972216	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e63b4eca-27a1-4fd0-b311-e468da0cd02e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e63b4eca-27a1-4fd0-b311-e468da0cd02e</a>	
KB2972107	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=151e15f7-fa7a-40c4-a58a-721cb88e8071">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=151e15f7-fa7a-40c4-a58a-721cb88e8071</a>	
KB2979578	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d44bd7d1-ec33-4a65-942c-093aec39b02b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d44bd7d1-ec33-4a65-942c-093aec39b02b</a>	
KB2978128	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e257d135-40b9-4361-9e7d-8537d256d54b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e257d135-40b9-4361-9e7d-8537d256d54b</a>	



## GE Healthcare

KB3074230	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0baf87d-e8ec-4ddc-b8b0-89f668ee6504">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d0baf87d-e8ec-4ddc-b8b0-89f668ee6504</a>	
KB3074550	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2f7719fa-8a6d-4b16-9c69-66447f308ee7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2f7719fa-8a6d-4b16-9c69-66447f308ee7</a>	
KB3097996	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=103a0bdd-e71c-4f33-8cf1-7b68198ad536">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=103a0bdd-e71c-4f33-8cf1-7b68198ad536</a>	
KB3098781	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ed6e065-7e37-44bb-8798-245649851dec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4ed6e065-7e37-44bb-8798-245649851dec</a>	
KB3122656	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=428c2fef-afe5-46a7-a454-da2fa0e3af27">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=428c2fef-afe5-46a7-a454-da2fa0e3af27</a>	
KB3127229	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f49f137-0741-498e-9f7c-754862054221">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f49f137-0741-498e-9f7c-754862054221</a>	
KB3135996	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=24aeb988-b368-435b-923b-c09b8a2d3fa5">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=24aeb988-b368-435b-923b-c09b8a2d3fa5</a>	
KB3156017	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a300bcb-8e8a-4e38-a04d-9da62feda917">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a300bcb-8e8a-4e38-a04d-9da62feda917</a>	
KB3142033	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eac1599f-2887-40ce-9cf4-0f364dc37343">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eac1599f-2887-40ce-9cf4-0f364dc37343</a>	
KB3159398	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=56a7a134-5ad6-43c2-aab1-e10d2fef8f7c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=56a7a134-5ad6-43c2-aab1-e10d2fef8f7c</a>	
KB3161949	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5bc0e07b-33ca-4ea2-8253-4f0dcd26783a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5bc0e07b-33ca-4ea2-8253-4f0dcd26783a</a>	
KB3170455	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=45460055-be25-47ad-ad0a-2a4711fcca74">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=45460055-be25-47ad-ad0a-2a4711fcca74</a>	
KB3163251	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b053e5b-04a5-4f47-92c5-74f6caba0de7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b053e5b-04a5-4f47-92c5-74f6caba0de7</a>	
KB3184471	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8b314103-7382-4c65-8679-135d24dc8966">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8b314103-7382-4c65-8679-135d24dc8966</a>	





## GE Healthcare

KB3177186	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=04c3010d-9989-495c-ab1c-dbc011015822">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=04c3010d-9989-495c-ab1c-dbc011015822</a>	
KB3125869	<a href="https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015">https://support.microsoft.com/en-us/help/3125869/ms15-124-vulnerability-in-internet-explorer-could-lead-to-aslr-bypass-december-16,-2015</a>	
KB2538243	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=729a0dcb-df9e-4d02-b603-ed1aee074428</a>	Only run X86-all-vcredist_x86....
KB3156016	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=de32fc3d-60a4-4fc9-9588-0c404765940a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=de32fc3d-60a4-4fc9-9588-0c404765940a</a>	
KB3156019	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=05cb4345-49ea-4eca-8ba3-f870466c46c7">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=05cb4345-49ea-4eca-8ba3-f870466c46c7</a>	
KB3161958	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=795571cf-bd8d-4602-bb28-11080e614333">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=795571cf-bd8d-4602-bb28-11080e614333</a>	
HPSBMU03653 rev.1	<a href="https://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_083799d6dad34195bb47cb43c1">https://h20566.www2.hp.com/hpsc/swd/public/detail?swItemId=MTX_083799d6dad34195bb47cb43c1</a>	
	<b><u>Reboot Required</u></b>	
KB3045311	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2fd51fbf-3f70-4c85-986b-5653b0fb7f11</a>	Only run x64 file; Run from a cmd prompt as Administrator using the following command: AMD64-all-sqlserver2008-kb3045311-x64_37a197c60990d2e83e98d1090109a4ab3f2abe4b.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
	<b>Make the following Registry Change</b>	
	<b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b>	Apply only on Domain Controller



	State=dword:00010000	
	<b>Reboot Required</b>	

## MLCL V6.9.6 2017 Patch Updates 2

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) It is expected that some patches listed will already be on the system.
- 3) Pay attention to the Notes section for special handling instructions
- 4) Patches must be applied in order except where indicated.
- 5) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.
- 6) Patches will not install if the software component to be patched is not present (such as an IE8 patch on a system that does not have IE8 installed).

**Note:** KB4022719 replaces KB4012215 and KB4019263 for both Windows 7 and Windows Server 2008 R2 to address CVE-2017-8543, CVE-2017-8544, CVE-2017-0277, CVE-2017-0276, CVE-2017-0275, CVE-2017-0274, CVE-2017-0272, CVE-2017-0271, CVE-2017-0270, CVE-2017-0269, CVE-2017-0268, CVE-2017-0171, CVE-2017-0280, CVE-2017-0279 and CVE-2017-0278.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
Adobe 11.0.20 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191</a>	
KB4025341 July Rollup 2017	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=12c93ad9-ef0e-4ce6-8a1d-84713223d24a">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=12c93ad9-ef0e-4ce6-8a1d-84713223d24a</a>	This patch is required to be installed for the August rollup to successfully install.



## GE Healthcare

KB4019112	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=1daeb6d1-b103-4baa-bbde-5326e17e89e4">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=1daeb6d1-b103-4baa-bbde-5326e17e89e4</a>	Run KB4014514 and KB4014504 only. For KB4014514, right click and Run as Administrator,
KB3191847	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4b4bbe2b-a25d-4509-a069-f5efc227b4ad">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4b4bbe2b-a25d-4509-a069-f5efc227b4ad</a>	
KB3191907	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c8533f11-51f9-4f84-96d8-c619947cc7c0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c8533f11-51f9-4f84-96d8-c619947cc7c0</a>	
KB3118310	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c59a1bb2-ff1f-427a-a8d7-2cab1cb3e7d1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c59a1bb2-ff1f-427a-a8d7-2cab1cb3e7d1</a>	Use en-us file all-convintl-en-us_....cab
KB3191843	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f715a81d-102d-416a-9a89-e9ebdace0a6d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f715a81d-102d-416a-9a89-e9ebdace0a6d</a>	
KB3191899	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7698c63a-b85f-4647-bcb1-1be0256c3f43">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7698c63a-b85f-4647-bcb1-1be0256c3f43</a>	
KB3203468	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a</a>	Use all-proof-en-us_.....cab
KB3213624	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3658f96e-a521-429d-a9a9-e70e30f5d830">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3658f96e-a521-429d-a9a9-e70e30f5d830</a>	
	<b>Reboot Required</b>	
KB3178688	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=322c28f5-349c-468a-ac94-901616f52372">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=322c28f5-349c-468a-ac94-901616f52372</a>	
KB3178690	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=06e2c9fb-65b7-48f5-b6e2-58071f17f9bd">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=06e2c9fb-65b7-48f5-b6e2-58071f17f9bd</a>	
	<b>Reboot Required</b>	
KB4034664 August Rollup 2017 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e0a94bad-5b2c-4611-9066-24491ce9bb4f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e0a94bad-5b2c-4611-9066-24491ce9bb4f</a>	To successfully install this Rollup, you must uninstall July Rollup <b>KB4025341</b> and <b>reboot before installing KB4034664</b>



## GE Healthcare

	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b><u>Reboot Required</u></b>	

### Windows 2008R2 (INW Server)

KB	Link	Notes
Adobe 11.0.20 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6157&amp;fileID=6191</a>	
KB4025341 July Rollup 2017	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b2423c5b-0254-4747-88bb-ec1a785549cb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b2423c5b-0254-4747-88bb-ec1a785549cb</a>	This patch is required to be installed for the August rollup to successfully install.
KB4019112	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dedea6da-e039-487b-8ec6-2729551f7165">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dedea6da-e039-487b-8ec6-2729551f7165</a>	<b>Run KB4014514 and KB4014504 only. For KB4014514, right click and Run as Administrator,</b>
KB890830	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c860821d-5eef-4d0c-9e9d-ee4638f34dca">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c860821d-5eef-4d0c-9e9d-ee4638f34dca</a>	<b>Run on Domain Controller only.</b>
	<b><u>Reboot Required</u></b>	
KB4034664 August Rollup 2017 <b>Superseded</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80f7899d-451d-4e3f-b54e-d488a06a3c58">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80f7899d-451d-4e3f-b54e-d488a06a3c58</a>	<b>To successfully install this Rollup, you must uninstall July Rollup KB4025341 and reboot before applying KB4034664</b>



## GE Healthcare

	[HKEY_LOCAL_MACHINE\System\CurrentControlSet\Services\NTDS\Parameters] LdapEnforceChannelBinding=DWORD:1	Apply only on Domain Controller. Need starting July Rollup- KB4025341 (CVE-2017-8563)
	<b><u>Reboot Required</u></b>	



## Patches post Old and New procedure

### MLCL V6.9.6 2017 Patch Updates 3

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) It is expected that some patches listed will already be on the system.
- 3) Pay attention to the Notes section for special handling instructions
- 4) Patches must be applied in order except where indicated.
- 5) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.
- 6) Patches will not install if the software component to be patched is not present (such as an IE8 patch on a system that does not have IE8 installed).

**Note:** KB4041681 replaces KB4041678 for both Windows 7 and Windows Server 2008 R2 to address CVE-2017-11771, CVE-2017-11772, CVE-2017-11780, CVE-2017-11781

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
Adobe 11.0.23 <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6279&amp;fileID=6314">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6279&amp;fileID=6314</a>	
KB4041681 October 2017 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8a346e85-6ae3-46aa-a9e1-2e70e760f61c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8a346e85-6ae3-46aa-a9e1-2e70e760f61c</a>	
	<b>Reboot Required</b>	



Windows 2008R2 (INW Server)

KB	Link	Notes
<b>Adobe 11.0.23</b> <b>Superseded</b>	<a href="http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6279&amp;fileID=6314">http://supportdownloads.adobe.com/thankyou.jsp?ftpID=6279&amp;fileID=6314</a>	
<b>KB4041681</b> <b>October 2017</b> <b>Monthly Rollup</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd0388fd-5aca-4a13-8417-c28e1d8b7dda">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd0388fd-5aca-4a13-8417-c28e1d8b7dda</a>	<p><b>To successfully install this Rollup, you must uninstall July Rollup KB4025341, August Rollup KB4034664 and reboot before applying KB4041681</b></p> <p>Make the following Registry Change – Only on Domain Controller if it does not exist:  [HKEY_LOCAL_MACHINE\System\CurrentControlSet\Services\NTDS\Parameters]  LdapEnforceChannelBinding=DWORD:1</p> <p><b>This registry key is needed on Domain controller</b> starting July Rollup- KB4025341 (CVE-2017-8563)</p>
	<b><u>Reboot Required</u></b>	



## MLCL V6.9.6 2018 Patch Updates 4

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.
- 5) Refer to the above links for the following previously qualified patches KB4019112. These patches were previously qualified but not listed. Please apply these patch to the to virtual review system other systems do not require these patches to be installed separately. In case you receive a message like “Not applicable to the computer”, ignore that is an indication that the patch already exist on the system.

**Follow these steps to make the following registry changes to remediate June and September monthly rollup vulnerabilities.**

Reference: <https://portal.msrc.microsoft.com/en-us/security-guidance/advisory/CVE-2017-8529>

### Windows 7 (Acquisition, Review and Virtual Review) & Windows 2008R2 (INW Server):

1. Click **Start**, click **Run**, type **regedt32** or type **regedit**, and then click **OK**.
2. In Registry Editor, locate the following registry folder: **HKEY\_LOCAL\_MACHINE\SOFTWARE\Microsoft\Internet Explorer\Main\FeatureControl\**
3. Right-click **FeatureControl**, point to **New**, and then click **Key**.
4. Type **FEATURE\_ENABLE\_PRINT\_INFO\_DISCLOSURE\_FIX**, and then press Enter to name the new subkey.
5. Right-click **FEATURE\_ENABLE\_PRINT\_INFO\_DISCLOSURE\_FIX**, point to **New**, and then click **DWORD Value**.
6. Type "iexplore.exe" for the new DWORD value.
7. Double-click the new DWORD value named iexplore.exe and change the **Value** data field to **1**.
8. Click **OK** to close.

### Windows 2008R2 (INW Server):

1. Click **Start**, click **Run**, type **regedt32** or type **regedit**, and then click **OK**.
2. In Registry Editor, locate the following registry folder: **HKEY\_LOCAL\_MACHINE\SOFTWARE\WOW6432Node\Microsoft\Internet Explorer\Main\FeatureControl\**





## GE Healthcare

3. Right-click **FeatureControl**, point to **New**, and then click **Key**.
4. Type **FEATURE\_ENABLE\_PRINT\_INFO\_DISCLOSURE\_FIX**, and then press Enter to name the new subkey.
5. Right-click **FEATURE\_ENABLE\_PRINT\_INFO\_DISCLOSURE\_FIX**, point to **New**, and then click **DWORD Value**.
6. Type "iexplore.exe" for the new DWORD value.
7. Double-click the new DWORD value named iexplore.exe and change the **Value** data field to **1**.
8. Click **OK** to close.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
KB4048957 November 2017 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=224b07ab-de98-45f0-8b9c-83551cac66f6">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=224b07ab-de98-45f0-8b9c-83551cac66f6</a>	
	<b><u>Reboot Required</u></b>	
KB4054518 December 2017 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5b48d1cb-83f7-43e1-9308-18872ffe4dce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5b48d1cb-83f7-43e1-9308-18872ffe4dce</a>	
	<b><u>Reboot Required</u></b>	
KB3203468 July 2017 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a</a>	
KB3213626 September 2017 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2bb1487f-b287-41a9-b0ec-01b42aa4759e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2bb1487f-b287-41a9-b0ec-01b42aa4759e</a>	
KB3128027 September 2017 Microsoft	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=474aa90a-7767-4f4f-b3f5-2ffa12fea4e6">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=474aa90a-7767-4f4f-b3f5-2ffa12fea4e6</a>	



## GE Healthcare

<b>PowerPoint 2010</b>		
<b>KB3141537 September 2017 Microsoft Publisher 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0c646d3e-697d-4463-a6ea-afb3493c5cea">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0c646d3e-697d-4463-a6ea-afb3493c5cea</a>	
<b>KB2553338 October 2017 Microsoft Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=14e73852-cbd2-456a-a9a8-7f0c10f1fa40">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=14e73852-cbd2-456a-a9a8-7f0c10f1fa40</a>	Might get error message (The upgrade path cannot be installed...) This error message can be ignored.
<b>KB2837599 October 2017 Microsoft Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=54ccbc02-879e-4aa1-b817-12418ce8dfcd">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=54ccbc02-879e-4aa1-b817-12418ce8dfcd</a>	
<b>KB4011612 December 2017 Microsoft Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8230d598-8ab1-4efc-89b6-d3507a6dfd20">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8230d598-8ab1-4efc-89b6-d3507a6dfd20</a>	Might get error message (The upgrade path cannot be installed...) This error message can be ignored.
<b>KB4011660 January 2018 Microsoft Excel 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7594745-04d5-4631-b2d7-289816f4dd43">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7594745-04d5-4631-b2d7-289816f4dd43</a>	
<b>KB4011659 January 2018 Microsoft Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0b5a1bf0-3043-47fd-afc3-d2fb55a46a96">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0b5a1bf0-3043-47fd-afc3-d2fb55a46a96</a>	
<b>KB4011611 January 2018 Microsoft Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b2c376c-ea57-4925-b81d-3b765d456f2b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3b2c376c-ea57-4925-b81d-3b765d456f2b</a>	Extract to a location and run the extraction to install. Check installed updates for a successful install.



## GE Healthcare

KB4011610 January 2018 Microsoft Office 2010	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=56447">https://www.microsoft.com/en-us/download/details.aspx?id=56447</a>	
KB4054172 January 2018 .NET Framework	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=537fc3ba-4248-40b8-9498-8a671abebfe9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=537fc3ba-4248-40b8-9498-8a671abebfe9</a>	Install the following KB4054172, KB4019990 and KB4054176
KB2719662	Create the following Registry Keys	
	Key=[HKEY_LOCAL_MACHINE\SOFTWARE\Microsoft\Windows\CurrentVersion\Policies\Windows\Sidebar\ Value Name=[TurnOffSidebar] Type=[REG_DWORD] Data=[1]	
KB2269637	Create the following Registry Keys	
	Key=[ HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\Session Manager\ Value Name=[CWDIllegalInDllSearch] Type=[REG_DWORD] Data=[1]	
	<b><u>Reboot Required</u></b>	

### Windows 2008R2 (INW Server)

KB	Link	Notes
KB3177467 Service Stack	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f1b99598-a22d-4fbc-9b63-09724833acc3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f1b99598-a22d-4fbc-9b63-09724833acc3</a>	Required for successful Monthly rollup installation without



		uninstalling previous monthly rollup
	<b>Reboot Required</b>	
KB4048957 November 2017 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=435d3006-04ae-4c27-a5f9-3c36f09e58ed">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=435d3006-04ae-4c27-a5f9-3c36f09e58ed</a>	
	<b>Reboot Required</b>	
KB4054518 December 2017 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=09064e30-6f3e-4c99-8d09-fbc2ba06b436">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=09064e30-6f3e-4c99-8d09-fbc2ba06b436</a>	
	<b>Reboot Required</b>	
KB4054172 January 2018 .NET Framework	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fdecaf44-50a3-4667-a935-f9e7af0bb317">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fdecaf44-50a3-4667-a935-f9e7af0bb317</a>	
KB2269637	Create the following Registry Keys	
	Key=[ HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\Session Manager\ Value Name=[CWDIllegalInDllSearch] Type=[REG_DWORD] Data=[1]	
	<b>Reboot Required</b>	

## MLCL V6.9.6 2018 Patch Updates 5

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions



- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
KB4056894 January 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=63bb3909-e5fe-45a2-8d59-44f9df52317f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=63bb3909-e5fe-45a2-8d59-44f9df52317f</a>	
	<b>Reboot Required</b>	
KB4091290	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c70372c5-bd6c-48f7-b562-c326bc1327a4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c70372c5-bd6c-48f7-b562-c326bc1327a4</a>	
KB4074598 February Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=651e95ab-6e7c-4ea6-9cd2-3cbabd9b76f0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=651e95ab-6e7c-4ea6-9cd2-3cbabd9b76f0</a>	
	<b>Reboot Required</b>	
KB4099950	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0872a60d-385a-4486-8322-9e759802017a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0872a60d-385a-4486-8322-9e759802017a</a>	<b>CAUTION: This patch has to be applied before March Monthly rollup KB4088875. Not applying this before March rollup can affect NIC settings.</b>
KB4088875 March Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3ed75c38-aa36-437e-bf4f-574789591e03">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3ed75c38-aa36-437e-bf4f-574789591e03</a>	
	<b>Reboot Required</b>	



## GE Healthcare

KB4096040	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=be3cdb55-862c-4362-b015-894e381e07f9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=be3cdb55-862c-4362-b015-894e381e07f9</a>	
KB4099467	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f325deb3-28fc-45a3-ab7b-5264f801daf6">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f325deb3-28fc-45a3-ab7b-5264f801daf6</a>	
	<b>Reboot Required</b>	
KB4093118 April Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=647f49ef-0f0a-49dc-9766-dd255cded1af">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=647f49ef-0f0a-49dc-9766-dd255cded1af</a>	
KB4011707	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=969c78ec-cd6a-4295-ade3-a57ca7f8b3b2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=969c78ec-cd6a-4295-ade3-a57ca7f8b3b2</a>	
KB3114874	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9fae99be-ddc3-4e37-b3ee-9b631fd50eca">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9fae99be-ddc3-4e37-b3ee-9b631fd50eca</a>	
KB3114416	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=da77f81d-187b-4cae-a75a-d64766a7713d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=da77f81d-187b-4cae-a75a-d64766a7713d</a>	
KB4057114	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0aa653a1-1459-44cd-be0b-0fcb77e4ef85">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0aa653a1-1459-44cd-be0b-0fcb77e4ef85</a>	Install AMD64_X86-en-sqlserver2008-kb4057114-x86_a9295f99a2ee7c714f540f3697be0fd4aee7a7bf.exe Run from a cmd prompt as Administrator using the following command: AMD64_X86-en-sqlserver2008-kb4057114-x86_a9295f99a2ee7c714f540f3697be0fd4aee7a7bf.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE
	<b>Reboot Required</b>	
KB4103718 May 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f4d0c73-a177-48cf-a3e7-97d1a94cba87">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f4d0c73-a177-48cf-a3e7-97d1a94cba87</a>	



GE Healthcare

	<b><u>Reboot Required</u></b>	
KB4095874 .NET 3.5 SP1 and KB4096495 .NET 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d000d6a2-3321-4381-9a24-3345b2cd0435">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d000d6a2-3321-4381-9a24-3345b2cd0435</a>	KB4099633 is the KB number to use to download and install the patch for KB4095874 and KB4096495. Inside your downloaded file has multiple KBs. Follow these order of installation: <ol style="list-style-type: none"> <li>1) Install KB4019990(You may see already installed message for this KB, ignore and continue)</li> <li>2) KB4095874</li> <li>3) KB4096495</li> </ol>
	<b><u>Reboot Required</u></b>	
KB4022146 Microsoft Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5795d737-4091-4784-a707-007b99d3daef">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5795d737-4091-4784-a707-007b99d3daef</a>	KB4022146 Microsoft Excel 2010
KB2899590 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d8acdbaf-7f56-4c65-a898-9fdcf7a2d83a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d8acdbaf-7f56-4c65-a898-9fdcf7a2d83a</a>	KB2899590 Microsoft Office 2010
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b><u>Reboot Required</u></b>	



**Windows 2008R2 (INW Server)**

KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
<p>CP031045 Firmware Update for ML350p Gen 8 Server</p>	<p><a href="https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_13ce4f3fc79b4433873342345b">https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_13ce4f3fc79b4433873342345b</a></p>	<p>Refer to “ML350p Gen8 (P72) BIOS Update to v 2015.07.01 (B) Instructions” section above for installation instructions. <b>This should be installed before installing “ML350p Gen8 (P72) BIOS Update to v2018.01.22 Instructions”.</b> This should only be installed on ML350p G8 Servers.</p>
<p>CP034981 Firmware Update for ML350p Gen 8 Server</p>	<p><a href="https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&amp;swItemId=MTX_58f2f298203c4664a6e5c9402d&amp;swEnvOid=4184#tab1">https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&amp;swItemId=MTX_58f2f298203c4664a6e5c9402d&amp;swEnvOid=4184#tab1</a></p>	<p>Refer to “ML350p Gen8 (P72) BIOS Update to v2018.01.22 Instructions” section above for installation instructions. <b>Install “ML350p Gen8 (P63) BIOS Update to v 2015.07.01 (B) Instructions” before installing this update.</b> This should only be installed on ML350p G8 Servers.</p>
<p>CP035397 Firmware Update for</p>	<p><a href="https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_df78827f61164b6f952314da7c">https://support.hpe.com/hpsc/swd/public/detail?swItemId=MTX_df78827f61164b6f952314da7c</a></p>	<p>Refer to “DL370 Gen6 BIOS Update to v2018.02.22 Instructions” section above for installation instructions. <b>This</b></p>





## GE Healthcare

DL370 Gen 6 Server		should only be installed on DL370 G6 Servers.
	<b>Reboot Required</b>	
KB4056894 January 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc887fd2-cd35-434b-b6e3-1fef99b2e7ce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=fc887fd2-cd35-434b-b6e3-1fef99b2e7ce</a>	
	<b>Reboot Required</b>	
KB4091290	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c96d43ea-477f-47a1-919f-6936c8d628a3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c96d43ea-477f-47a1-919f-6936c8d628a3</a>	
KB4074598 February Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f3ab18cb-219e-4287-b14c-3a05c8d9479a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f3ab18cb-219e-4287-b14c-3a05c8d9479a</a>	
	<b>Reboot Required</b>	
KB4099950	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=38b41383-c716-488c-a937-163bf04f6956">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=38b41383-c716-488c-a937-163bf04f6956</a>	<b>CAUTION: This patch has to be applied before March Monthly rollup KB4088875. Not applying this before March rollup can affect NIC settings.</b>
KB4096040	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3af42ab1-13ed-42b5-9e4e-a841a71e7f2c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3af42ab1-13ed-42b5-9e4e-a841a71e7f2c</a>	
KB4088875 March Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=03df6731-e0a6-4917-9da3-161a0b7f6b09">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=03df6731-e0a6-4917-9da3-161a0b7f6b09</a>	
KB4099467	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=38800bcc-c954-4822-b864-6ae91cc19bb2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=38800bcc-c954-4822-b864-6ae91cc19bb2</a>	
	<b>Reboot Required</b>	
KB4093118 April Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2c7363c-323f-4e92-892a-90b83027e4aa">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2c7363c-323f-4e92-892a-90b83027e4aa</a>	



GE Healthcare

	<b>Reboot Required</b>	
KB4057114	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0aa653a1-1459-44cd-be0b-0fcb77e4ef85">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0aa653a1-1459-44cd-be0b-0fcb77e4ef85</a>	<p><b>Install AMD64-en-sqlserver2008-kb4057114-x64_9ce0b7c5909d8fcc5b9a12d17f29b7864a9df33a.exe file.</b> Run from a cmd prompt as Administrator using the following command: AMD64-en-sqlserver2008-kb4057114-x64_9ce0b7c5909d8fcc5b9a12d17f29b7864a9df33a.exe /ACTION=Patch /INSTANCENAME=MSSQLSERVER /IGNORESERVICERESTARTSTATE</p>
	<b>Reboot Required</b>	
KB4103718 May 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4fe75106-a2ba-4186-aecd-10424a19225e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4fe75106-a2ba-4186-aecd-10424a19225e</a>	
	<b>Reboot Required</b>	
KB4095874 .NET 3.5 SP1 and KB4096495 .NET 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=62ccd808-b5a5-4be9-8a38-8e2a829e29d1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=62ccd808-b5a5-4be9-8a38-8e2a829e29d1</a>	<p><b>KB4099633 is the KB number to use to download and install the patch for KB4095874 and KB4096495. The downloaded file has multiple KBs. Follow these order of installation:</b></p> <ol style="list-style-type: none"> <li><b>1) Install KB4019990(You may see already installed message for this KB, ignore and continue)</b></li> </ol>



		2) KB4095874 3) KB4096495
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b>Reboot Required</b>	

### MLCL V6.9.6 2018 Patch Updates 6

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
Z600 BIOS Update to v3.61	<a href="https://support.hp.com/us-en/drivers/selfservice/hp-z600-workstation/3718663">https://support.hp.com/us-en/drivers/selfservice/hp-z600-workstation/3718663</a>	Refer to Z600 BIOS Update to v3.61 Instructions above



GE Healthcare

**WARNING: After installing this BIOS version onto the system, prior BIOS versions cannot be installed onto the system.**

	<b>Reboot Required</b>	
KB4284826 June 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=326a9830-3983-402d-b48b-7a35f99c516a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=326a9830-3983-402d-b48b-7a35f99c516a</a>	
	<b>Reboot Required</b>	
KB4338818 July 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ea409dca-1368-48cf-94c1-d510b1690d74">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ea409dca-1368-48cf-94c1-d510b1690d74</a>	
	<b>Reboot Required</b>	
KB4338821	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dafd8f28-09a9-4d17-8a6c-93341a8379ec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dafd8f28-09a9-4d17-8a6c-93341a8379ec</a>	
	<b>Reboot Required</b>	
KB4343900 August 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d785f6dd-d90b-4cb9-838a-8faf5971165f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d785f6dd-d90b-4cb9-838a-8faf5971165f</a>	
	<b>Reboot Required</b>	
KB4343894 IE 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3fc3f78a-19c5-45bc-9f06-6a14cec0f007">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3fc3f78a-19c5-45bc-9f06-6a14cec0f007</a>	
	<b>Reboot Required</b>	
KB4457144	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f5c88de4-720e-4ed1-b95f-6ce4d4d06087">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f5c88de4-720e-4ed1-b95f-6ce4d4d06087</a>	



## GE Healthcare

September 2018 Monthly Rollup		
	<b>Reboot Required</b>	
KB4345590 .NET Framework	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=08968031-142d-4dcb-9fd8-29fbd6ae9960">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=08968031-142d-4dcb-9fd8-29fbd6ae9960</a>	Install KB4344149 first (restart if prompted) followed by KB4344152
	<b>Reboot Required</b>	
KB3203468 Microsoft Office Proof 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7a599998-ca41-4840-90ea-8143724e5c6a</a>	
KB4032223 Microsoft Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd28cd6b-b3c2-4266-b417-d26d53f7d75f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cd28cd6b-b3c2-4266-b417-d26d53f7d75f</a>	
KB4227175 Microsoft Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bba3a281-f10f-45bc-adf8-9e4436d4c39b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bba3a281-f10f-45bc-adf8-9e4436d4c39b</a>	
KB4018311 Microsoft Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f96028c5-b20a-41a6-9963-83261d690a62">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f96028c5-b20a-41a6-9963-83261d690a62</a>	
KB3213636 Microsoft Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7e7049e7-b870-4c87-af60-bc5a0bace002">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7e7049e7-b870-4c87-af60-bc5a0bace002</a>	
KB4022198 Microsoft Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cc4958f4-308d-474c-a655-567e41cf9b1d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cc4958f4-308d-474c-a655-567e41cf9b1d</a>	



## GE Healthcare

<b>KB3115197</b> <b>Microsoft</b> <b>Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=393d97f5-b447-43c5-9e0c-b6707c8d29ce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=393d97f5-b447-43c5-9e0c-b6707c8d29ce</a>	
<b>KB3115248</b> <b>Microsoft</b> <b>Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4cc02a6f-9551-4909-bce1-4a45d41404ec">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4cc02a6f-9551-4909-bce1-4a45d41404ec</a>	
<b>KB4022199</b> <b>Microsoft</b> <b>Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f3c71ef-c743-4824-8951-deb8cdabf2eb">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3f3c71ef-c743-4824-8951-deb8cdabf2eb</a>	
<b>KB4022137</b> <b>Microsoft</b> <b>Office 2010 SP2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9d64be86-d3f2-47a4-a9a3-f8ae05842ed3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9d64be86-d3f2-47a4-a9a3-f8ae05842ed3</a>	
<b>KB4018310</b> <b>Microsoft</b> <b>PowerPoint</b> <b>2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4cc49d2c-fe7e-4a55-987e-4b1d4ccb8e1c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4cc49d2c-fe7e-4a55-987e-4b1d4ccb8e1c</a>	
<b>KB4011186</b> <b>Microsoft</b> <b>Publisher 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4564150c-2ab1-42e4-a135-105b924dc45d">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4564150c-2ab1-42e4-a135-105b924dc45d</a>	
<b>KB4022202</b> <b>Microsoft</b> <b>Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=59253783-a245-4897-993e-57a2ed30c0e0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=59253783-a245-4897-993e-57a2ed30c0e0</a>	
<b>KB4011674</b> <b>Microsoft</b> <b>Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b13107b6-02b9-4f00-a6d9-3bf44a3c1247">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b13107b6-02b9-4f00-a6d9-3bf44a3c1247</a>	
<b>KB4022141</b> <b>Microsoft</b> <b>Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2f428f2-7c02-4a49-8680-4407404cb7a2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2f428f2-7c02-4a49-8680-4407404cb7a2</a>	



## GE Healthcare

	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b><u>Reboot Required</u></b>	

### Windows 2008R2 (INW Server)

KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
HPESBHF03874 rev.1 ML350p Gen8 (CP036431)	<a href="https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&amp;swItemId=MTX_3a5fbdd653f94fac9e2a12e8dc&amp;swEnvOid=4184">https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=5195931&amp;swItemId=MTX_3a5fbdd653f94fac9e2a12e8dc&amp;swEnvOid=4184</a>	Refer to ML350p Gen8 BIOS Update to 5/21/2018 Instructions above
	<b><u>Reboot Required</u></b>	
HPESBHF03874 rev.1 DL370 Gen6 (SP99324)	<a href="https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=3890183&amp;swItemId=MTX_37660617800b4f3db17cb11599&amp;swEnvOid=4184">https://support.hpe.com/hpsc/swd/public/detail?sp4ts.oid=3890183&amp;swItemId=MTX_37660617800b4f3db17cb11599&amp;swEnvOid=4184</a>	Refer to DL370 Gen6 BIOS Update to 5/21/2018 Instructions above
	<b><u>Reboot Required</u></b>	
KB4284826 June 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=999fa80e-c59d-4ff6-8268-c6a8c365f428">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=999fa80e-c59d-4ff6-8268-c6a8c365f428</a>	



GE Healthcare

	<b><u>Reboot Required</u></b>	
KB4338818 July 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c930eab-7b7e-4616-b5b5-d6e4a723bc71">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1c930eab-7b7e-4616-b5b5-d6e4a723bc71</a>	
	<b><u>Reboot Required</u></b>	
KB4338821	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8e951203-5d8e-4f69-9560-ce698c4f7a77">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8e951203-5d8e-4f69-9560-ce698c4f7a77</a>	
	<b><u>Reboot Required</u></b>	
KB4343900 August 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=71600c77-2a56-4ba2-991b-ad477cfc9eb9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=71600c77-2a56-4ba2-991b-ad477cfc9eb9</a>	Create the following registry settings if they do not exist Key=[HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\Session Manager\Memory Management] Value Name=[FeatureSettingsOverride] Type=[REG_DWORD] Data=[0] Key=[HKEY_LOCAL_MACHINE\SYSTEM\CurrentControlSet\Control\Session Manager\Memory Management] Value Name=[FeatureSettingsOverrideMask] Type=[REG_DWORD] Data=[3]
	<b><u>Reboot Required</u></b>	
KB4343894 IE 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6525d333-fafc-4345-ad06-6edc8db84aaf">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=6525d333-fafc-4345-ad06-6edc8db84aaf</a>	
	<b><u>Reboot Required</u></b>	





## GE Healthcare

KB4457144 September 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2986185e-cd31-4f1d-99a5-bea6bd1ef53c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2986185e-cd31-4f1d-99a5-bea6bd1ef53c</a>	
	<b>Reboot Required</b>	
KB4345590 .NET Framework	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=14afae30-094f-4dca-ba5f-e22347edb98f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=14afae30-094f-4dca-ba5f-e22347edb98f</a>	Install KB4344149 first (restart if prompted) followed by KB4344152
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b>Reboot Required</b>	

## MLCL V6.9.6 2019 Patch Updates 7

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

## Adobe Reader DC MUI Installation and Configuration

Follow the below instructions to install and configure the Adobe Reader DC MUI on 6.9.6 R1 Acquisition system, Review workstation and INW server.

### Adobe Reader DC MUI Installer and Patch Download Locations:

- MUI Installer:



## GE Healthcare

- Location: <ftp://ftp.adobe.com/pub/adobe/reader/win/AcrobatDC/1500720033/>
  - File Name: AcroRdrDC1500720033\_MUI.exe
- Patch (**Adobe Acrobat Reader MUI DC (Continuous Track) update - All languages**):
- Location: <https://supportdownloads.adobe.com/detail.jsp?ftpID=6523>

**Note:** v2019.008.20081 is the adobe reader DC patch version tested and qualified for installation.

### Adobe Reader DC MUI Installer and Patch Installation Instructions:

**Note:** The Adobe Reader DC automatically uninstalls the previous version of Adobe Reader XI during installation.

- 1) Note the following login requirements:
  - On standalone Acquisition systems, you must be logged in as member of **local** administrator group to Windows and Custom Shell.
  - On networked Acquisition systems, Review workstations, you must be logged in as member of **domain** administrator group to Windows and Custom Shell.
  - On INW server, you must be logged in as member of **domain** administrator group to Windows.
- 2) Navigate to the adobe installer location, right click **AcroRdrDC1500720033\_MUI.exe** and select **Run as Administrator**.
- 3) Wait for the file extraction to complete.
- 4) Keep the default folder location and click **Install**.
- 5) Wait for the installation to complete.
- 6) Click **Finish**.
- 7) Navigate to the adobe patch location, double click **AcroRdrDCUpd1900820081\_MUI.msp**.
- 8) Click **Update**.
- 9) Wait for the patch installation to complete.
- 10) Click **Finish**.
- 11) Restart the system.

### Adobe Reader DC MUI Configuration:

Adobe Reader DC must be configured under the following windows users in order:

- Workstations:
  - MLCLLogonUser



## GE Healthcare

- MLCLTechUser
- Server:
  - Administrator
  - MLCLTechUser

### **MLCLLogonUser:**

- 1) Note the following login requirements:
  - On standalone Acquisition systems, you must be logged in as **local MLCLLogonUser** to Windows and **local MLCLUser** to Custom Shell.
  - On networked Acquisition systems, Review workstations, you must be logged in as **domain MLCLLogonUser** to Windows and **domain MLCLUser** to Custom Shell.
- 2) Click **Start > All Programs > Acrobat Reader DC**.
- 3) At license agreement window, click **Accept**.
- 4) Close Welcome window.
- 5) Click **Edit > Preferences > General**.
- 6) Deselect **Show me messages when I launch Adobe Acrobat Reader**.
- 7) Click **Internet**.
- 8) Deselect **Allow Speculative downloading in the background**.
- 9) Click **Reviewing**.
- 10) Click **Tracker**.
- 11) Deselect **Show notification icon in system tray**.
- 12) Click **OK**.
- 13) Click **Edit > Preferences > Reviewing**.



## GE Healthcare

- 14) Deselect **Show Welcome dialog when opening file**.
- 15) Deselect **Show server connection warning dialog when opening file**.
- 16) Click **OK**.
- 17) Close **Adobe Reader DC**.
- 18) Navigate to **C:\Program Files\GE Healthcare\MLCL\Doc** folder.
- 19) Open **Mac-Lab Operator's Manual.pdf**.
- 20) Click **Edit > Preferences > Documents**.
- 21) Select **Remember current state of the Tools pane**.
- 22) Click **OK**.
- 23) Click **View > Show/Hide** and deselect **Tools Pane**.
- 24) Close **Adobe Reader DC**.
- 25) If present, delete the Acrobat Reader DC shortcut from the desktop.
- 26) Click **Continue**.
- 27) Enter administrator Username and Password and Click Yes.

### **MLCLTechUser:**

- 1) Note the following login requirements:
  - On standalone Acquisition systems, you must be logged in as **local MLCLTechUser** to Windows and Custom Shell.
  - On networked Acquisition systems, Review workstations, you must be logged in as **domain MLCLTechUser** to Windows and Custom Shell.
  - On INW server, you must be logged in as **domain MLCLTechUser** to Windows.
- 2) Click **Start > All Programs > Acrobat Reader DC**.
- 3) Close Welcome window.
- 4) Click **Edit > Preferences > General**.
- 5) Deselect **Show me messages when I launch Adobe Acrobat Reader**.
- 6) Click **Internet**.
- 7) Deselect **Allow Speculative downloading in the background**.
- 8) Click **Reviewing**.



## GE Healthcare

- 9) Click **Tracker**.
- 10) Deselect **Show notification icon in system tray**.
- 11) Click **OK**.
- 12) Click **Edit > Preferences > Reviewing**.
- 13) Deselect **Show Welcome dialog when opening file**.
- 14) Deselect **Show server connection warning dialog when opening file**.
- 15) Click **OK**.
- 16) Close **Adobe Reader DC**.
- 17) Navigate to **C:\Program Files\GE Healthcare\MLCL\Doc** folder on workstations and **C:\Program Files (x86)\GE Healthcare\MLCL\Doc** folder on server.
- 18) Open **Mac-Lab Operator's Manual.pdf**.
- 19) Click **Edit > Preferences > Documents**.
- 20) Select **Remember current state of the Tools pane**.
- 21) Click **OK**.
- 22) Click **View > Show/Hide** and deselect **Tools Pane**.
- 23) Close **Adobe Reader DC**.
- 24) If present, delete the Acrobat Reader DC shortcut from the desktop.

### Administrator:

- 1) Note the following login requirements:
  - On INW server, you must be logged in as **domain Administrator** to Windows.
- 2) Click **Start > All Programs > Acrobat Reader DC**.
- 3) Close Welcome window.
- 4) Click **Edit > Preferences > General**.
- 5) Deselect **Show me messages when I launch Adobe Acrobat Reader**.
- 6) Click **Internet**.
- 7) Deselect **Allow Speculative downloading in the background**.
- 8) Click **Reviewing**.
- 9) Click **Tracker**.
- 10) Deselect **Show notification icon in system tray**.
- 11) Click **OK**.
- 12) Click **Edit > Preferences > Reviewing**.



## GE Healthcare

- 13) Deselect **Show Welcome dialog when opening file.**
- 14) Deselect **Show server connection warning dialog when opening file.**
- 15) Click **OK.**
- 16) Close **Adobe Reader DC.**
- 17) Navigate to **C:\Program Files\GE Healthcare\MLCL\Doc** folder on workstations and **C:\Program Files (x86)\GE Healthcare\MLCL\Doc** folder on server.
- 18) Open **Mac-Lab Operator's Manual.pdf.**
- 19) Click **Edit > Preferences > Documents.**
- 20) Select **Remember current state of the Tools pane.**
- 21) Click **OK.**
- 22) Click **View > Show/Hide** and deselect **Tools Pane.**
- 23) Close **Adobe Reader DC.**
- 24) If present, delete the Acrobat Reader DC shortcut from the desktop.

### Disable Adobe Reader Update Service:

- 1) Note the following login requirements:
  - On standalone Acquisition systems, you must be logged in as member of **local** administrator group to Windows and Custom Shell.
  - On networked Acquisition systems, Review workstations, you must be logged in as member of **domain** administrator group to Windows and Custom Shell.
  - On INW server, you must be logged in as member of **domain** administrator group to Windows.
- 2) Launch Services.msc.
- 3) Stop and Disable **Adobe Acrobat Update Service.**

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	



## GE Healthcare

KB4490628 Servicing Stack Update	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=71c7172c-f6ea-493d-ab74-65d2548e834b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=71c7172c-f6ea-493d-ab74-65d2548e834b</a>	
KB4462923 October 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2f075cf-7563-40e7-9aa3-8d2562e60cf9">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2f075cf-7563-40e7-9aa3-8d2562e60cf9</a>	
	<b><u>Reboot Required</u></b>	
KB4467107 November 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=81c3ab78-c5a5-403b-b347-0d9421539574">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=81c3ab78-c5a5-403b-b347-0d9421539574</a>	
	<b><u>Reboot Required</u></b>	
KB4471318 December 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=345b59c2-a130-42aa-9b98-a66dd085451c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=345b59c2-a130-42aa-9b98-a66dd085451c</a>	
	<b><u>Reboot Required</u></b>	
KB4480970 January 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4972abbb-3924-406f-8612-4b6f3ad53442">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4972abbb-3924-406f-8612-4b6f3ad53442</a>	
	<b><u>Reboot Required</u></b>	
KB4486563 February 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b29b6d12-dbd3-4d44-b0cf-f6e5e298cab2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b29b6d12-dbd3-4d44-b0cf-f6e5e298cab2</a>	
	<b><u>Reboot Required</u></b>	



## GE Healthcare

KB4489878 March 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=039eb986-8841-4931-88e9-7f429ed74fb4">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=039eb986-8841-4931-88e9-7f429ed74fb4</a>	
	<b><u>Reboot Required</u></b>	
KB4493472 April 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c263402a-d44d-4473-bff6-40e5cc468c82">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c263402a-d44d-4473-bff6-40e5cc468c82</a>	
	<b><u>Reboot Required</u></b>	
KB4499164 May 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=94e0a99a-327b-4a75-a40a-b38f0a4e18cf">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=94e0a99a-327b-4a75-a40a-b38f0a4e18cf</a>	
	<b><u>Reboot Required</u></b>	
KB4470641 .NET Framework 3.5.1 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3ef1b3d-4a92-45b1-b9b6-203d388abe1a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3ef1b3d-4a92-45b1-b9b6-203d388abe1a</a>	
	<b><u>Reboot Required</u></b>	
KB4470637 .NET Framework 4.5.2 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b427c69-99db-4e3d-838f-3d9f5df18a6c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=7b427c69-99db-4e3d-838f-3d9f5df18a6c</a>	Right Click on the file and Run as Administrator. Note: This will be extracted to a temporary location and the update will run. Wait till .Net Update window display and follow onscreen instructions
	<b><u>Reboot Required</u></b>	





## GE Healthcare

KB4480059 .NET Framework 4.5.2 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b8c1b025-fd0a-4c5c-9aa8-3549f8eef894">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b8c1b025-fd0a-4c5c-9aa8-3549f8eef894</a>	
	<b><u>Reboot Required</u></b>	
KB3114565 November 2018 Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8597b0b-75c8-49e9-b352-2c036324dcac">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=f8597b0b-75c8-49e9-b352-2c036324dcac</a>	
KB4461570 December 2018 Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=67f17a1d-98a3-4554-8d26-21e0e32454d8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=67f17a1d-98a3-4554-8d26-21e0e32454d8</a>	
KB2553332 January 2019 Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c23a7e38-0ae6-4565-ac68-fcbc0ba37194">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c23a7e38-0ae6-4565-ac68-fcbc0ba37194</a>	
KB4461614 January 2019 Office 2010 SP2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e21643f5-d332-490b-94f6-666bcf37a48">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e21643f5-d332-490b-94f6-666bcf37a48</a>	
KB4461466 October 2018 Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1914e637-8c9a-49fa-8361-2f621e2cb1cf">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1914e637-8c9a-49fa-8361-2f621e2cb1cf</a>	
KB4461577 December 2018 Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7419ea5-4069-4e51-bd42-d6d0f89b0000">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d7419ea5-4069-4e51-bd42-d6d0f89b0000</a>	



## GE Healthcare

<b>KB4092439</b> <b>October 2018</b> <b>Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2dc153d4-a8a6-461d-bcee-96d16d168ec3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2dc153d4-a8a6-461d-bcee-96d16d168ec3</a>	
<b>KB4461625</b> <b>January 2019</b> <b>Word 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ce188561-f88e-4130-b478-c71e737af957">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=ce188561-f88e-4130-b478-c71e737af957</a>	Install both the language version e.g. all-wordintl-en-us_.... and All-word-x-none_.....
<b>KB4461521</b> <b>December 2018</b> <b>PowerPoint 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e1eedb8e-0c58-4b7d-9da5-b34066d06ad8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e1eedb8e-0c58-4b7d-9da5-b34066d06ad8</a>	
<b>KB4462230</b> <b>Microsoft Excel 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2728f82a-4baf-438c-93bb-0c6ef3ab2628">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2728f82a-4baf-438c-93bb-0c6ef3ab2628</a>	
<b>KB3115120</b> <b>Microsoft Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=57460e64-0e98-46c3-9fdd-5a6b410b9e2e">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=57460e64-0e98-46c3-9fdd-5a6b410b9e2e</a>	
<b>KB3191908</b> <b>Microsoft Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3d5594e2-20dc-4714-8b1f-521061557021">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3d5594e2-20dc-4714-8b1f-521061557021</a>	
<b>KB3213631</b> <b>Microsoft Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9c1e1b48-fc01-4698-824c-4a3767377eae">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9c1e1b48-fc01-4698-824c-4a3767377eae</a>	
<b>KB4022206</b> <b>Microsoft Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bac66ae6-b816-4718-89c2-0e3af0f223a2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=bac66ae6-b816-4718-89c2-0e3af0f223a2</a>	Ignore Windows installer message if they appear
<b>KB4022208</b> <b>Microsoft Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ff1653c-e6e9-41a7-8df7-6ee81ec2dd2c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=2ff1653c-e6e9-41a7-8df7-6ee81ec2dd2c</a>	Ignore Windows installer message if they appear



## GE Healthcare

<b>KB4462177</b> <b>Microsoft</b> <b>Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80edffc4-eab0-475b-a4f3-8b1d531db0dd">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=80edffc4-eab0-475b-a4f3-8b1d531db0dd</a>	
<b>KB4462223</b> <b>Microsoft</b> <b>Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=334be4e6-18ce-4d4c-a133-3c720d8c1262">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=334be4e6-18ce-4d4c-a133-3c720d8c1262</a>	
<b>KB4461623</b> <b>Microsoft</b> <b>Outlook 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cefe4cf0-ac44-44cd-872b-fb6e5c292754">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=cefe4cf0-ac44-44cd-872b-fb6e5c292754</a>	Install both the language version e.g. all-wordintl-en-us_.... and All-word-x-none_.....
<b>KB3128031</b> <b>Microsoft</b> <b>Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=efdfb48c-174b-4b88-9dbe-18431473ece3">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=efdfb48c-174b-4b88-9dbe-18431473ece3</a>	
<b>KB4092436</b> <b>Microsoft</b> <b>Office 2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b8df3741-3ae0-40de-9857-52ef03468f16">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b8df3741-3ae0-40de-9857-52ef03468f16</a>	
<b>KB4022136</b> <b>Microsoft</b> <b>PowerPoint</b> <b>2010</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2e3b299-dd41-4d04-be48-c53abc9705bf">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=d2e3b299-dd41-4d04-be48-c53abc9705bf</a>	Install both the language version e.g. all-wordintl-en-us_.... and All-word-x-none_.....
<b>KB4487078</b> <b>.NET</b> <b>Framework</b> <b>3.5.1, 4.5.2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=30526e5c-503a-4300-8d4c-c215d1a30349">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=30526e5c-503a-4300-8d4c-c215d1a30349</a>	Install KB4483455 and KB4483458
<b>KB4340004</b> <b>.NET</b> <b>Framework</b> <b>3.5.1, 4.5.2</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c52c0560-8129-4649-812d-08473bb26801">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c52c0560-8129-4649-812d-08473bb26801</a>	Install KB4338602
<b>KB4345679</b> <b>.NET</b>	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1b11fddd-9a22-472e-a412-ef225a5de41a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1b11fddd-9a22-472e-a412-ef225a5de41a</a>	Install KB4344173



## GE Healthcare

Framework 3.5.1, 4.5.2		
KB4487121 .NET Framework 3.5.1, 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5b16b18c-5d7c-4ab0-a6cd-366b2553e1b2">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=5b16b18c-5d7c-4ab0-a6cd-366b2553e1b2</a>	Install KB4483474 and KB4483483
KB4493435 Internet Explorer 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4a2f0647-ecd1-4e6c-9c83-c19f37e7b3e0">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4a2f0647-ecd1-4e6c-9c83-c19f37e7b3e0</a>	
	<b><u>Reboot Required</u></b>	
KB3064209 CPU Microcode update	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=47656">https://www.microsoft.com/en-us/download/details.aspx?id=47656</a>	Ignore if you receive already installed on the computer message
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b><u>Reboot Required</u></b>	

### Windows 2008R2 (INW Server)

KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	



## GE Healthcare

	State=dword:00023c00	
KB4490628 Servicing Stack Update	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1d4f0343-a41a-4782-8aed-18a620431171">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1d4f0343-a41a-4782-8aed-18a620431171</a>	
KB4462923 October 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3e23546-1642-4edc-bdd4-932f941f97a1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3e23546-1642-4edc-bdd4-932f941f97a1</a>	
	<b><u>Reboot Required</u></b>	
KB4467107 November 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1f74c5cc-0f12-40e5-a947-81516a1cce12">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=1f74c5cc-0f12-40e5-a947-81516a1cce12</a>	
	<b><u>Reboot Required</u></b>	
KB4471318 December 2018 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3df43753-6cbf-40d1-9249-db8009bbbc7a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3df43753-6cbf-40d1-9249-db8009bbbc7a</a>	
	<b><u>Reboot Required</u></b>	
KB4480970 January 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=baf43ca5-a997-48e2-bb96-375193004bb8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=baf43ca5-a997-48e2-bb96-375193004bb8</a>	
	<b><u>Reboot Required</u></b>	
KB4486563 February 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9ec6ce3a-e6b5-4741-a44b-01d76ce427d1">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9ec6ce3a-e6b5-4741-a44b-01d76ce427d1</a>	
	<b><u>Reboot Required</u></b>	
KB4489878 March 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4f0792e3-3d80-41c3-bc5b-0440b0df9f20">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=4f0792e3-3d80-41c3-bc5b-0440b0df9f20</a>	
	<b><u>Reboot Required</u></b>	



## GE Healthcare

KB4493472 April 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c8630c56-66ba-4f5f-8564-30d73f25beb8">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c8630c56-66ba-4f5f-8564-30d73f25beb8</a>	
<b>Reboot Required</b>		
KB4499164 May 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c832b037-3ac5-4dfb-b43b-cd70c004a803">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c832b037-3ac5-4dfb-b43b-cd70c004a803</a>	
<b>Reboot Required</b>		
KB4470641 .NET Framework 3.5.1 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=437d0478-b891-4d79-b5fb-4b7290d77334">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=437d0478-b891-4d79-b5fb-4b7290d77334</a>	
<b>Reboot Required</b>		
KB4470637 .NET Framework 4.5.2 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fe7139a-9081-4151-9b83-c499dcdb852b">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0fe7139a-9081-4151-9b83-c499dcdb852b</a>	Right Click on the file and Run as Administrator. Note: This will be extracted to a temporary location and the update will run. Wait till .Net Update window display and follow onscreen instructions.
<b>Reboot Required</b>		
KB4480059 .NET Framework 4.5.2 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eab83cf8-4e43-40cc-be52-caaf0fb1381c">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=eab83cf8-4e43-40cc-be52-caaf0fb1381c</a>	
KB4487078 .NET	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e220e2c6-d8a8-4a77-b509-a4db5c3e7736">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=e220e2c6-d8a8-4a77-b509-a4db5c3e7736</a>	<b>Install KB4483455 and KB4483458</b>



## GE Healthcare

Framework 3.5.1, 4.5.2		
KB4487121 .NET Framework 3.5.1, 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f601e2c-ed42-409a-b754-3c5f402f4f9a">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=8f601e2c-ed42-409a-b754-3c5f402f4f9a</a>	Install KB4483474 and KB4483483
KB4493435 Internet Explorer 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=23f66904-5d8d-4e14-b06b-48157e4d9327">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=23f66904-5d8d-4e14-b06b-48157e4d9327</a>	
	<b>Reboot Required</b>	
KB3064209 CPU Microcode update	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=47673">https://www.microsoft.com/en-us/download/details.aspx?id=47673</a>	Ignore if you receive already installed on the computer message
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b>Reboot Required</b>	

### MLCL V6.9.6 2019 Patch Updates 8

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.



### Adobe 12 (19.012.20034) Patch Installation Instructions

Follow the below instructions to install and configure the Adobe Reader DC MUI on 6.9.6 R1 **Acquisition system, Review workstation and INW server.**

- Patch (**Adobe Acrobat Reader MUI DC (Continuous Track) update - All languages**):

Location/Link	Version	Notes
<a href="https://supportdownloads.adobe.com/detail.jsp?ftpID=6693">https://supportdownloads.adobe.com/detail.jsp?ftpID=6693</a>	2019.012.20034	v2019.012.20034 patch is tested and qualified for installation

- Note the following login requirements:
  - On standalone Acquisition systems, you must be logged in as member of **local** administrator group to Windows and Custom Shell.
  - On networked Acquisition systems, Review workstations, you must be logged in as member of **domain** administrator group to Windows and Custom Shell.
  - On INW server, you must be logged in as member of **domain** administrator group to Windows.
- Navigate to the adobe installer location, double click on **AcroRdrDCUpd1901220034\_MUI.msp** and select Open
- Wait for Windows Installer to complete.
- Click **Update**.
- When prompted for User Access Control Click on Yes to continue
- Wait for the patch installation to complete.
- Click **Finish**.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	





## GE Healthcare

KB4503292 June 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3b33c94-4178-4957-a1c4-e6272b16a182">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=c3b33c94-4178-4957-a1c4-e6272b16a182</a>	
	<b><u>Reboot Required</u></b>	
KB4507449 July 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=30651730-f83e-4702-9dd9-feab76438aed">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=30651730-f83e-4702-9dd9-feab76438aed</a>	
	<b><u>Reboot Required</u></b>	
KB4018313 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b10524c1-71cf-4b9f-956a-5dc8d8502919">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=b10524c1-71cf-4b9f-956a-5dc8d8502919</a>	
KB4464567 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dc49e69c-71ca-46ef-8afc-90257dd60246">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=dc49e69c-71ca-46ef-8afc-90257dd60246</a>	
KB4461619 Microsoft Word 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=369edcb8-e930-4f3f-a93c-fefa6b7012dd">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=369edcb8-e930-4f3f-a93c-fefa6b7012dd</a>	
KB4462224 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a9aa79ea-0e8f-4d55-a55c-5e5a82ad5f41">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=a9aa79ea-0e8f-4d55-a55c-5e5a82ad5f41</a>	
KB4464572 Microsoft Excel 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=72bbbf9a-584a-49ed-af5b-03248bc5a9ce">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=72bbbf9a-584a-49ed-af5b-03248bc5a9ce</a>	
KB4462174 Microsoft Office 2010	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0ddf0ad9-d785-4c1e-b472-40f7132015d8">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=0ddf0ad9-d785-4c1e-b472-40f7132015d8</a>	
	<b><u>Reboot Required</u></b>	



## GE Healthcare

<b>KB4507420</b> .NET Framework 3.5.1, 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=85b53f91-8683-4db9-a2c2-c6acac4d96df">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=85b53f91-8683-4db9-a2c2-c6acac4d96df</a>	<b>Only install setup files listed below and ignore other files:</b> 1) ndp45-kb4507001-x86_306e8aa676ccfe1a27a9bbe1fbaa93aeb87e91fb.exe 2) windows6.1-kb4507004-x86_625917bdc06b040d7c312e1b8d805a43ff31cf8b.msu
	<b><u>Reboot Required</u></b>	
<b>KB4507434</b> Internet Explorer 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9f31d69d-d28d-401d-a3b6-9019ce60987f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=9f31d69d-d28d-401d-a3b6-9019ce60987f</a>	
	<b><u>Reboot Required</u></b>	
<b>KB4474419</b> SHA-2 code signing support update	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=77e7077c-a23b-4d6e-ac42-6a120fbd3c37">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=77e7077c-a23b-4d6e-ac42-6a120fbd3c37</a>	
	<b><u>Reboot Required</u></b>	
<b>KB4512506</b> August 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=70fb559c-f9e7-4c56-9d47-68cf0ffe68a2">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=70fb559c-f9e7-4c56-9d47-68cf0ffe68a2</a>	
	Make the Following Registry Change	
	<b>[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]</b>	
	<b>State=dword:00010000</b>	



**Reboot Required**

**Windows 2008R2 (INW Server)**

KB	Link	Notes
	Make the following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00023c00	
KB4503292 June 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84167563-985e-4b02-ae56-ee75fa01e744">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=84167563-985e-4b02-ae56-ee75fa01e744</a>	
	<b>Reboot Required</b>	
KB4507449 July 2019 Monthly Rollup	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3c284887-ea1a-4c2b-9e34-00eaa6e44c0f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=3c284887-ea1a-4c2b-9e34-00eaa6e44c0f</a>	
	<b>Reboot Required</b>	
KB4507420 .NET Framework 3.5.1, 4.5.2	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=63be6ae4-edc2-4100-bf2c-15d3f2d51f30">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=63be6ae4-edc2-4100-bf2c-15d3f2d51f30</a>	<b>Only install setup files listed below and ignore other files:</b> 1) ndp45-kb4507001-x64_be0daee8df411dedfc9fdb0b3eafd20af3afeb0.exe 2) windows6.1-kb4507004-x64_a7d988d65422c8cfbbed141c95aea37a8743167.msu
	<b>Reboot Required</b>	
KB4507434 Internet Explorer 11	<a href="http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=86517a70-4a9b-4551-b46c-2c4d7497b64f">http://catalog.update.microsoft.com/v7/site/ScopedViewInline.aspx?updateid=86517a70-4a9b-4551-b46c-2c4d7497b64f</a>	



	<b>Reboot Required</b>	
KB4474419 SHA-2 code signing support update	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c4b6b6d8-f1a6-4134-9bb5-d739415c2637">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=c4b6b6d8-f1a6-4134-9bb5-d739415c2637</a>	
	<b>Reboot Required</b>	
KB4512506 August 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=82bf6959-36ff-4d3e-a909-b1cf9ffc9880">https://www.catalog.update.microsoft.com/ScopedViewInline.aspx?updateid=82bf6959-36ff-4d3e-a909-b1cf9ffc9880</a>	
	Make the Following Registry Change	
	[HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	
	State=dword:00010000	
	<b>Reboot Required</b>	

### MLCL V6.9.6 2020 Patch Updates 9

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

### Windows 7 (Acquisition, Review and Virtual Review)



## GE Healthcare

KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00023c00	
KB4524135 IE 11 Cumulative Update October 2019	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4524135">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4524135</a>	Install "2019-10 Cumulative Security Update for Internet Explorer 11 for Windows 7 for x86-based systems (KB4524135)"
	<b><u>Reboot Required</u></b>	
KB4519974 IE 11 Cumulative Update October 2019	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4519974">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4519974</a>	Install "2019-10 Cumulative Security Update for Internet Explorer 11 for Windows 7 for x86-based systems (KB4519974)"
	<b><u>Reboot Required</u></b>	
KB4523206 November 2019 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4523206">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4523206</a>	Install "2019-11 Servicing Stack Update for Windows 7 for x86-based Systems (KB4523206)"
	<b><u>Reboot Required</u></b>	
KB4525235 November 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4525235">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4525235</a>	Install "2019-11 Security Monthly Quality Rollup for Windows 7 for x86-based Systems (KB4525235)"
	<b><u>Reboot Required</u></b>	
KB4531786 Dec ember 2019 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4531786">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4531786</a>	Install "2019-12 Servicing Stack Update for Windows 7 for x86-based Systems (KB4531786)"
	<b><u>Reboot Required</u></b>	



GE Healthcare

KB4530734 Dec ember 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4530734">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4530734</a>	Install “2019-12 Security Monthly Quality Rollup for Windows 7 for x86-based Systems (KB4530734)”
<b>Reboot Required</b>		
KB4536952 Jan uary 2020 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4536952">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4536952</a>	Install “2020-01 Servicing Stack Update for Windows 7 for x86-based Systems (KB4536952)”
<b>Reboot Required</b>		
KB4538483 Extended Security Updates (ESU) Licensing Preparation Package	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4538483">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4538483</a>	Install “2020-02 Extended Security Updates (ESU) Licensing Preparation Package for Windows 7 for x86- based Systems (KB4538483)”
	Make the Following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00010000	
<b>Reboot Required</b>		

Windows Server 2008 R2 (INW Server)		
KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00023c00	



## GE Healthcare

KB4474419 SHA-2 code signing support update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4474419">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4474419</a>	Install “2019-09 Security Update for Windows Server 2008 R2 for x64-based Systems (KB4474419)”
<b>Reboot Required</b>		
KB4524135 IE 11 Cumulative Update October 2019	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4524135">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4524135</a>	Install “2019-10 Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based systems (KB4524135)”
<b>Reboot Required</b>		
KB4519974 IE 11 Cumulative Update October 2019	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4519974">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4519974</a>	Install “2019-10 Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based systems (KB4519974)”
<b>Reboot Required</b>		
KB4523206 November 2019 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4523206">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4523206</a>	Install “2019-11 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems (KB4523206)”
<b>Reboot Required</b>		
KB4525235 November 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4525235">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4525235</a>	Install “2019-11 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems (KB4525235)”
<b>Reboot Required</b>		
KB4531786 Dec ember 2019 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4531786">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4531786</a>	Install “2019-12 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems (KB4531786)”



	<b>Reboot Required</b>	
KB4530734 Dec ember 2019 Monthly Rollup	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4530734">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4530734</a>	Install “2019-12 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems (KB4530734)”
	<b>Reboot Required</b>	
KB4536952 Jan uary 2020 SSU Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4536952">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4536952</a>	Install “2020-01 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems (KB4536952)”
	<b>Reboot Required</b>	
KB4538483 Extended Security Updates (ESU) Licensing Preparation Package	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4538483">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4538483</a>	Install “2020-02 Extended Security Updates (ESU) Licensing Preparation Package for Windows Server 2008 R2 for x64-based Systems (KB4538483)”
	Make the Following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00010000	
	<b>Reboot Required</b>	

## MLCL V6.9.6 2020 Patch Updates 10

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions





- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00023c00	
Disabling Questra Agent Service	N/A	Refer Disabling Questra Agent Service section.
KB4534314 Jan 2020 Security Only Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534314">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534314</a>	Install “2020-01 Security Only Quality Update for Windows 7 for x86-based Systems (KB4534314)”
<b>Reboot Required</b>		
KB4539602	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4539602">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4539602</a>	Install “2020-01 Update for Windows 7 for x86-based Systems (KB4539602)”
	Make the Following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00010000	
<b>Reboot Required</b>		

Windows Server 2008 R2 (INW Server)		
KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]	



	State=dword:00023c00	
Disabling Questra Agent Service	N/A	Refer Disabling Questra Agent Service section.
KB4534314 Jan 2020 Security Only Update	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534314">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534314</a>	Install "2020-01 Security Only Quality Update for Windows Server 2008 R2 for x64-based Systems (KB4534314)"
<b>Reboot Required</b>		
KB4539602	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4539602">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4539602</a>	Install "2020-01 Update for Windows Server 2008 R2 for x64-based Systems (KB4539602)"
	Make the Following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00010000	
<b>Reboot Required</b>		

### MLCL V6.9.6 2020 Patch Updates 11

The following patches bring the MLCL system to a more recent patch level and address several security vulnerabilities. The following guidelines apply:

- 1) The above patches are required patches for 6.9.6 and must be applied first.
- 2) Pay attention to the Notes section for special handling instructions
- 3) Patches must be applied in order except where indicated.
- 4) Reboots are only required where indicated. If a Patch requests a reboot at another point, the system can be rebooted but it is not required.

### Adobe Acrobat Reader DC (2020.009.20063) Patch Installation Instructions

Follow the below instructions to install and configure the Adobe Acrobat Reader DC MUI on Acquisition system, Review workstation, Virtual Review and INW server.



## GE Healthcare

- Patch (**Adobe Acrobat Reader MUI DC (Continuous Track) update - All languages**):

Location/Link	Version	Notes
<a href="https://supportdownloads.adobe.com/detail.jsp?ftplD=6921">https://supportdownloads.adobe.com/detail.jsp?ftplD=6921</a>	2020.009.20063	v2020.009.20063 patch is tested and qualified for installation

1) Note the following login requirements:

- On standalone Acquisition systems, you must be logged in as member of **local** administrator group to Windows and Custom Shell.
- On networked Acquisition systems, Review workstations and Virtual Review you must be logged in as member of **domain** administrator group to Windows and Custom Shell.
- On INW server, you must be logged in as member of **domain** administrator group to Windows.

2) Navigate to the adobe installer location, double click on **AcroRdrDCUpd2000920063\_MUI.msp** and select Open

3) Wait for Windows Installer to complete.

4) Click **Update**.

5) When prompted for User Access Control Click on Yes to continue

6) Wait for the patch installation to complete.

7) Click **Finish**.

Windows 7 (Acquisition, Review and Virtual Review)		
KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00023c00	
Adobe Acrobat Reader DC (2020.009.20063)	<a href="https://supportdownloads.adobe.com/detail.jsp?ftplD=6921">https://supportdownloads.adobe.com/detail.jsp?ftplD=6921</a>	Refer to above instructions.
KB4534251 - Security	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534251">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534251</a>	Install "2020-01 Cumulative Security Update for Internet Explorer 11 for



GE Healthcare

Update for IE Jan 2020		Windows 7 for x86-based systems (KB4534251)”
	<b>Reboot Required</b>	
KB4534976 - Security Only Update for .NET Framework Jan 2020	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534976">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534976</a>	Under “2020-01 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 (KB4534976)”, Install the following, 1) windows6.1-kb4532960- x86_ef650860586c2c1b76f15a1143d 778caacafbb69.msu 2) ndp45-kb4532964- x86_9ff5c2ace268f9bc3c1f2684f8bf0 5f2e89261ea.exe
	<b>Reboot Required</b>	
KB4484458 - Security Update for Word 2010: July 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101563">https://www.microsoft.com/en-us/download/details.aspx?id=101563</a>	Install “Security Update for Microsoft Word 2010 (KB4484458) 32-Bit Edition”
KB4484415 - Security Update for Excel 2010: June 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101378">https://www.microsoft.com/en-us/download/details.aspx?id=101378</a>	Install “Security Update for Microsoft Excel 2010 (KB4484415) 32-Bit Edition”
KB4484266 - Security Update for Office 2010: April 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101175">https://www.microsoft.com/en-us/download/details.aspx?id=101175</a>	Install “Security Update for Microsoft Office 2010 (KB4484266) 32-Bit Edition”



## GE Healthcare

KB4484235 - Security Update for PowerPoint 2010: April 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101162">https://www.microsoft.com/en-us/download/details.aspx?id=101162</a>	Install "Security Update for Microsoft PowerPoint 2010 (KB4484235) 32-Bit Edition"
KB4475506 - Security Update for Office 2010: August 2019	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=100096">https://www.microsoft.com/en-us/download/details.aspx?id=100096</a>	Install "Security Update for Microsoft Office 2010 (KB4475506) 32-Bit Edition"
KB4484238 - Security Update for Office 2010: April 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101105">https://www.microsoft.com/en-us/download/details.aspx?id=101105</a>	Install "Security Update for Microsoft Office 2010 (KB4484238) 32-Bit Edition"
KB4484126 - Security Update for Office 2010: April 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101169">https://www.microsoft.com/en-us/download/details.aspx?id=101169</a>	Install "Security Update for Microsoft Office 2010 (KB4484126) 32-Bit Edition"
KB3203462 - Security Update for Office 2010: April 2020	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101181">https://www.microsoft.com/en-us/download/details.aspx?id=101181</a>	Install "Security Update for Microsoft Office 2010 (KB3203462) 32-Bit Edition"
KB3015640 - Update for Microsoft	<a href="https://www.microsoft.com/en-us/download/details.aspx?id=101259">https://www.microsoft.com/en-us/download/details.aspx?id=101259</a>	Install "Update for Microsoft Office 2010 (KB3015640) 32-Bit Edition"



Office 2010: May 2020		
	Make the Following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00010000	
<b>Reboot Required</b>		

<b>Windows Server 2008 R2 (INW Server)</b>		
KB	Link	Notes
	Make the following Registry Change [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing] State=dword:00023c00	
Adobe Acrobat Reader DC (2020.009.20063)	<a href="https://supportdownloads.adobe.com/detail.jsp?ftpID=6921">https://supportdownloads.adobe.com/detail.jsp?ftpID=6921</a>	Refer to above instructions.
KB4534251 - Security Update for IE Jan 2020	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534251">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534251</a>	Install "2020-01 Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based systems (KB4534251)"
<b>Reboot Required</b>		
KB4534976 - Security Only Update for .NET Framework Jan 2020	<a href="https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534976">https://www.catalog.update.microsoft.com/Search.aspx?q=KB4534976</a>	Under "2020-01 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64 (KB4534976)" Install the following,



		<p>1) windows6.1-kb4532960-x64_f99f406b14c6cb4a501cdff53728c8631330fc08.msu                  2) ndp45-kb4532964-x64_4e90c195438094eadc62b08daa62b518087c1cde.exe</p>
	<b>Reboot Required</b>	
CVE-2020-1350	<p><a href="https://support.microsoft.com/en-us/help/4569509/windows-dns-server-remote-code-execution-vulnerability">https://support.microsoft.com/en-us/help/4569509/windows-dns-server-remote-code-execution-vulnerability</a></p>	<p><b>Refer CVE-2020-1350   Workaround via Registry update section.</b></p> <p><u>Note: Apply this workaround only if the INW is a domain controller.</u></p>
	<p>Make the Following Registry Change                  [HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing]                  State=dword:00010000</p>	
	<b>Reboot Required</b>	

## MLCL v6.9.6 Optional Security Updates

The following optional updates may be applied to further enhance the security profile of the MLCL systems. These updates should be evaluated on a site-by-site basis in accordance with local IT policy. The configuration changes in this section are compatible with MLCL product functionality but may introduce site specific IT impact as a result of disabling of legacy SSL protocols, prohibiting remote desktop usage and requiring certificate generation and maintenance.

### Additional Security Setting and Patches

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
Patch	Download URL	Download URL	Download URL	Download URL



## GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
<b>MS16-047</b> <b>KB3149090</b> <b>Superseded</b>	<a href="https://technet.microsoft.com/library/security/MS16-047">https://technet.microsoft.com/library/security/MS16-047</a>	<a href="https://technet.microsoft.com/library/security/MS16-047">https://technet.microsoft.com/library/security/MS16-047</a>	<a href="https://technet.microsoft.com/library/security/MS16-047">https://technet.microsoft.com/library/security/MS16-047</a>	<a href="https://technet.microsoft.com/library/security/MS16-047">https://technet.microsoft.com/library/security/MS16-047</a>
<b>Plugin 20007 – Disable SSL V2/V3 – KB187498</b>	Please see section - How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498	Please see section - How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498	Please see section - How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498	Please see section - How to Install Plugin 20007 – Disable SSL V2/V3 – KB187498
<b>Plugin 78479 - Poodle</b>	No change needed. Step above fixes this.	No change needed. Step above fixes this.	No change needed. Step above fixes this.	No change needed. Step above fixes this.
<b>Plugin 35291 – Weak Hashing</b>	Please see section - How to Install Plugin 35291 – Weak Hashing (Refer to <a href="https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx">https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx</a> for more information)	Please see section - How to Install Plugin 35291 – Weak Hashing (Refer to <a href="https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx">https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx</a> for more information)	Please see section - How to Install Plugin 35291 – Weak Hashing (Refer to <a href="https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx">https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx</a> for more information)	Please see section - How to Install Plugin 35291 – Weak Hashing (Refer to <a href="https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx">https://technet.microsoft.com/en-us/library/ms191192(v=sql.105).aspx</a> for more information)
<b>Plugin 45411</b>	No change needed. Step above fixes this.	No change needed. Step above fixes this.	No change needed. Step above fixes this.	No change needed. Step above fixes this.
<b>Plugin 65821 – SSL RC4 Cipher Suites Supported (Bar Mitzvah)</b>	Please see section - How to Install Plugin 65821 – SSL RC4 Cipher Suites Supported (Bar Mitzvah)	Please see section - How to Install Plugin 65821 – SSL RC4 Cipher Suites Supported (Bar Mitzvah)	Please see section - How to Install Plugin 65821 – SSL RC4 Cipher Suites Supported (Bar Mitzvah)	Please see section - How to Install Plugin 65821 – SSL RC4 Cipher Suites Supported (Bar Mitzvah)
<b>Plugin 63155 - Microsoft Windows Unquoted Service Path Enumeration</b>	Please see section - How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration	Please see section - How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration	Please see section - How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration	Please see section - How Remove Vulnerability for Plugin 63155 – Microsoft Windows Unquoted Service Path Enumeration





## GE Healthcare

	INW Server	Acquisition - Mac-Lab IT/XT/XTi , CardioLab IT/XT/XTi and SpecialsLab	GE Client Review Workstation	Virtual Review
<b>Plugin 59915 – Vulnerabilities in Gadgets Could Allow Remote Code Execution</b>	N/A	Please follow the section titled <b>“Disable the Sidebar in the system Registry”</b> in the following Article: <a href="https://technet.microsoft.com/library/security/2719662">https://technet.microsoft.com/library/security/2719662</a>	Please follow the section titled <b>“Disable the Sidebar in the system Registry”</b> in the following Article: <a href="https://technet.microsoft.com/library/security/2719662">https://technet.microsoft.com/library/security/2719662</a>	Please follow the section titled <b>“Disable the Sidebar in the system Registry”</b> in the following Article: <a href="https://technet.microsoft.com/library/security/2719662">https://technet.microsoft.com/library/security/2719662</a>
<b>Disable SMB1 Protocol</b>	Please see section How to Disable the SMB1 Protocol	Please see section How to Disable the SMB1 Protocol	Please see section How to Disable the SMB1 Protocol	Please see section How to Disable the SMB1 Protocol
<b>Configure Required SMBv2 Signing</b>	Please see section OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required	Please see section OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required	Please see section OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required	Please see section OPTIONAL – How to Configure Required SMBv2 Signing - SMBv2 signing not required
<b>CDrive Disk Space Cleanup</b>	N/A	N/A	Please see section OPTIONAL - CDrive Disk Space Cleanup	Please see section OPTIONAL - CDrive Disk Space Cleanup
<b>Removal of Authenticated Users from Share on INW server</b>	Please see section OPTIONAL – Removal of Authenticated Users from Share on INW server	N/A	N/A	N/A
<b>Disable Remote Desktop Services</b>	Please see section OPTIONAL – Remote Desktop Services Remote Code Execution Vulnerability	Please see section OPTIONAL – Remote Desktop Services Remote Code Execution Vulnerability	Please see section OPTIONAL – Remote Desktop Services Remote Code Execution Vulnerability	Please see section OPTIONAL – Remote Desktop Services Remote Code Execution Vulnerability
<b>Disabling Qestra Agent Service</b>	Please see section OPTIONAL – Disabling Qestra Agent Service	Please see section OPTIONAL – Disabling Qestra Agent Service	Please see section OPTIONAL – Disabling Qestra Agent Service	Please see section OPTIONAL – Disabling Qestra Agent Service

### Password Policy

Password Policy: Minimum Password Length may be changed above the 14 characters limit to meet security requirements. Refer to **Password** section of the Security Guide for further details about changing passwords.

### Optionally Remove Abode Reader on INW Server



## GE Healthcare

We recommend that Adobe Reader be uninstalled on the INW Server. On the server Adobe Reader is only used to review INW operator manual. The manual is available in printed format and through online manual portal.

### Further MLCL Systems Security Recommendations

We strongly recommend following these recommendations:

- Change default password to stronger, more secure and unique for each user account
- Disable RDP in for each system using the following steps:
  - My Computer>Properties>Remote settings>Remote
  - Check "Don't allow connections to this computer".
  - Click ok and reboot.
- If Insite functionality is not used on the system then turn off the VNC service using the following steps:
  - Click on Start button
  - Click on Control Panel
  - Click on Administrative Tools
  - Double click Services
  - Right Click TightVNC Server and select Properties
  - Under Startup type select Disabled from the dropdown
  - Click on Apply and Ok

We also recommend following these general recommendations and the recommendations listed in MLCL Security guide

- Enhanced physical security
- Network firewalls
- Demilitarized Zones and perimeter defenses for site network
- Intrusion detection systems - network intrusion protection system
- Virtual Private Networks
- Network traffic analysis
- Log analysis



GE Healthcare

## Contact Information

If you have any additional questions, please contact our Technical Support Department.