



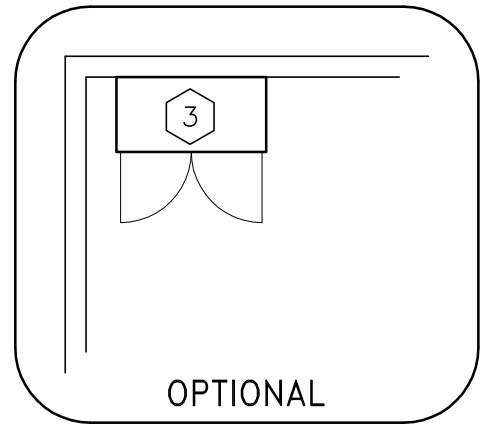


Revolution Frontier with 2000 Table

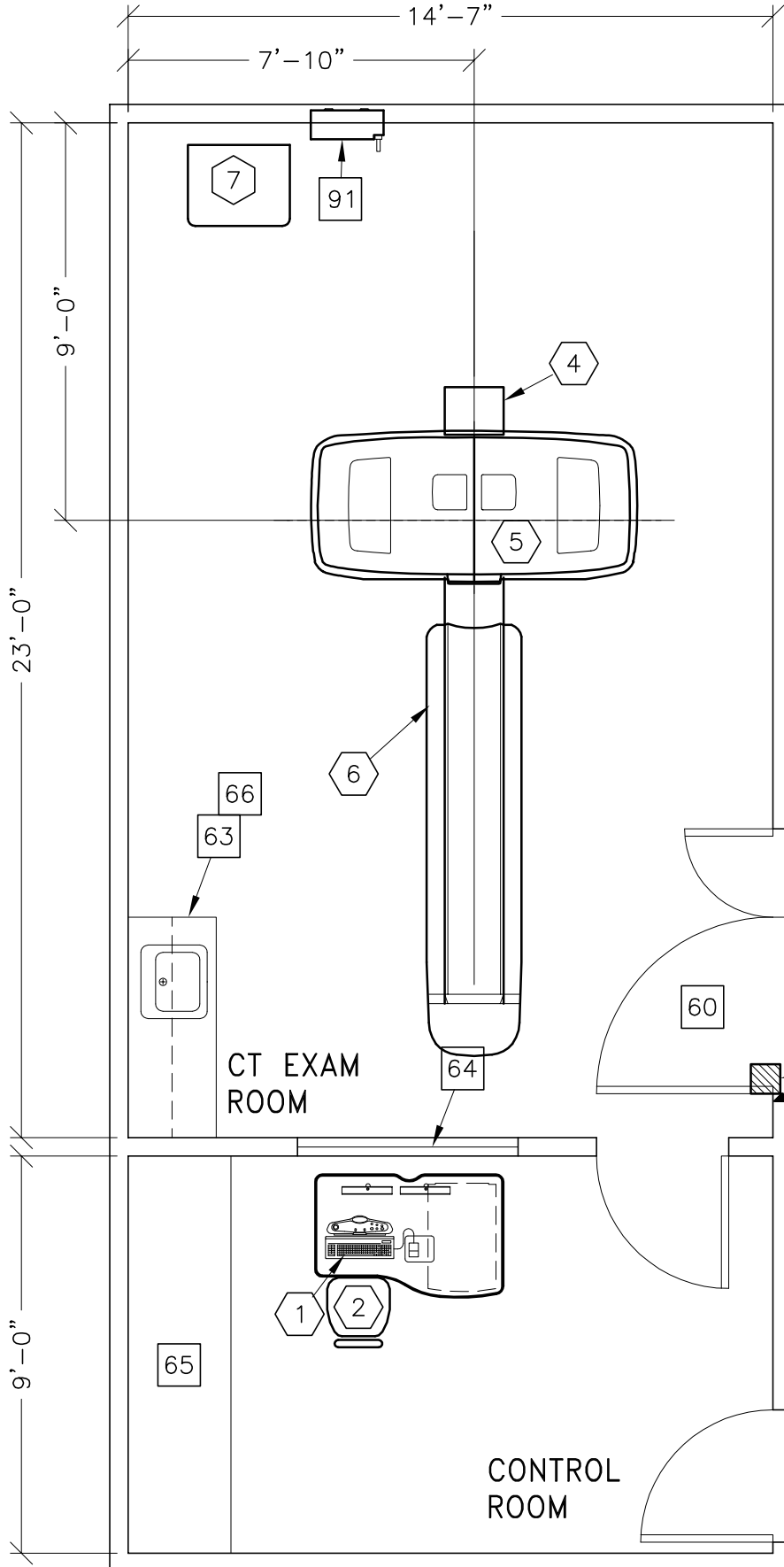

NOTE: THIS IS A SUGGESTED LAYOUT BASED OFF OF GE HEALTHCARE PRE-INSTALL MANUAL. PLEASE CONTACT LOCAL PROJECT MANAGER OF INSTALLATION IF YOUR ROOM LAYOUT OR DIMENSIONS ARE DIFFERENT THAN THE LAYOUT SHOWN.

RECOMMENDED  
SCAN ROOM  
CEILING HEIGHT  
9'-0" [2743mm]

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL:	20-0" x 13'-6" [6.1M x 4.1M]	8'-0" [2.43M]

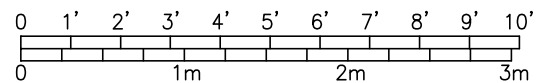

ROOM PLAN LOCATED ON PAGE 6-118

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	OPERATOR'S CONSOLE / COMPUTER	186 lbs	3200 btu
②	1	OPERATOR'S CHAIR		
③	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	99 lbs	
④	1	REAR CABLE COVER		
⑤	1	REVOLUTION FRONTIER GANTRY <BTU'S INCLUDE GANTRY, TABLE>	4161 lbs	35007 btu
⑥	1	2000 PATIENT TABLE WITH EXTENDED TABLE TOP.	1113 lbs	
⑦	1	POWER DISTRIBUTION UNIT	804 lbs	
<p>*****  <b>CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS</b>            *****</p>				
⑥0		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
⑥1		DOOR LIMIT SWITCH <REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
⑥2		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
⑥3		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
⑥4		COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
⑥5		LEAD GLASS WINDOW		
⑥6		CASEWORK REQUIRED WITH A MINIMUM OF 16CF TO STORE ALL SERVICE MATERIALS		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1 FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		
91		MAIN DISCONNECT CONTROL		

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)  
 Allowable Temperature Change: 5° F [3° C] per hour  
 Humidity: 30-60 percent non condensing  
 Allowable Humidity Change: 5 percent per hour  
 Max - Altitude: 7875 [2400M] ft. above sea level