



XR: How to Resume an “In Progress” Exam

Precision 500D/WDR1, Proteus/WDR1, Optima* XR220amx, Optima XR646, Definium* 8000, Discovery* XR650, XR656 and XR656Plus

The following steps can be followed when an exam has been interrupted, thereby allowing the user to retrieve the patient and/or images associated with that exam. The exam status may be displayed as “In Progress” on the worklist.

1. This step is applicable to a P500 with WDR1

On the P500, select “Start Cassette Only Exam”



2. Proceed to the Worklist screen.

Precision 500D/WDR1, Proteus/WDR1, Optima XR220 and Optima XR646
Select the “Worklist” button on the upper left corner of the screen




Definium 8000, Discovery XR650, XR656 and XR656Plus

Worklist Screen



3. Select the “Filter List” button

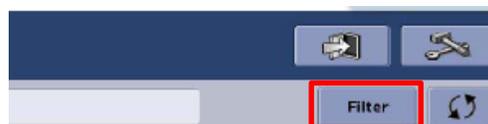

4. The Worklist Filter screen will appear

Filter List

Show list for
 This System CR Modality DX Modality All systems

Include
 Completed Exams Discontinued Exams

Date
All
From (mm/dd/yyyy) To (mm/dd/yyyy)

Patient Info
Last Name: First Name:
Accession#: Patient ID:

OK Cancel

5. Enter search criteria and Select "OK"

Ensure "Completed Exams" and "Discontinued Exams" checkboxes are not selected

6. The Patient Name should display with the status "In Progress"

Date	Patient Name	Patient ID	Accession#	Status	Physician	Modality	Location	Description
07/01/2011	EMW,	61347613	43000328	SCHEDULED		DX		RequestedPro...
07/01/2011	see,	94920644	37592002	SCHEDULED		DX		RequestedPro...
06/29/2011	BobAga,	78210431	01922089	SCHEDULED		OT		RequestedPro...
06/29/2011	AgoDeb96,	24533042	89986786	SCHEDULED		OT		RequestedPro...
07/01/2011	012	NEW ID		SCHEDULED		DX		
06/24/2011	NEW1106241L	NEW PATIENTID 20110624151L		SCHEDULED		DX		
07/01/2011	NEW1107011L	NEW PATIENTID 20110701125L		IN PROGRESS		DX		
06/27/2011	NEW1106271L	NEW PATIENTID 20110627110L		IN PROGRESS		DX		
06/26/2011	NEW PATIENT	NEW ID		IN PROGRESS		DX		
06/24/2011	NEW1106241L	NEW PATIENTID 20110624151L		IN PROGRESS		DX		
06/24/2011	test	NEW ID		IN PROGRESS		DX		

Emergency Cassette Resume

7. Highlight patient exam and Select "Resume"

8. Continue with exam or close patient exam

Any previous images will transfer to the local database/completed exams list

9. The images will automatically push to PACS if auto send is turned on. If auto send is not configured, images will need to be manually sent to PACS