





IGS 5X0 SYSTEM

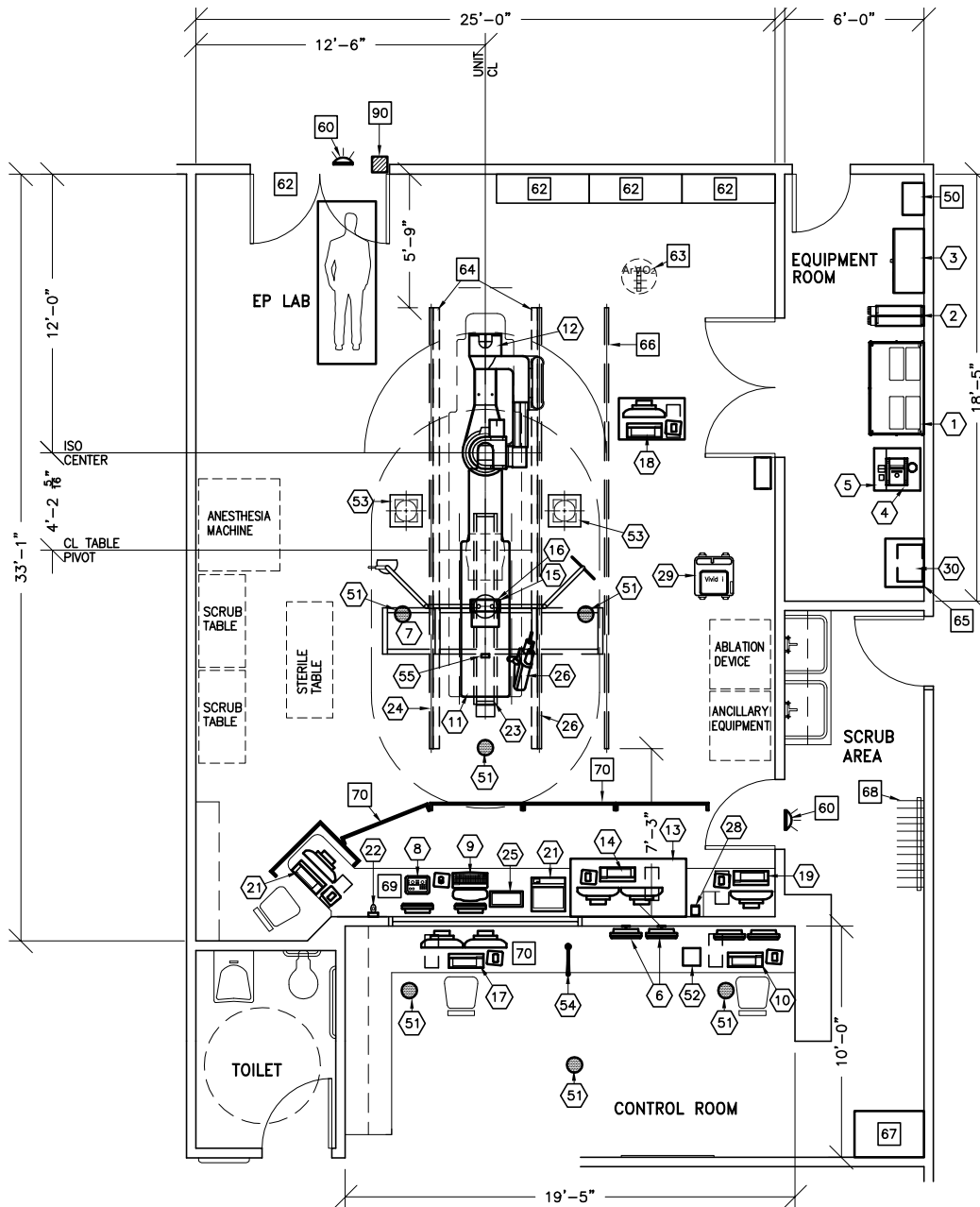

NOTE: THIS IS A SUGGESTED LAYOUT BASED OFF OF GE HEALTHCARE PRE-INSTALL MANUAL. PLEASE CONTACT LOCAL PROJECT MANAGER OF INSTALLATION IF YOUR ROOM LAYOUT OR DIMENSIONS ARE DIFFERENT THAN THE LAYOUT SHOWN.

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
RECOMMENDED:	32'-0" x 25'-0" [9.75m x 7.62m]	9'-6" [2.89m]

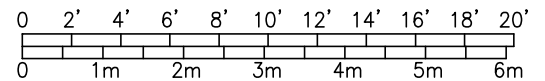

ROOM PLAN LOCATED ON PAGE 5-117

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	C-FRT CABINET	1193 lbs	2388 btu
②	1	8 KVA UPS	185 lbs	1760 btu
③	1	POWER DISTRIBUTION UNIT (PDB)	628 lbs	1365 btu
④	1	DETECTOR CHILLER	33 lbs	706 btu
⑤	1	COOLIX 4100 WATER CHILLER	264 lbs	18730 btu
⑥	2	19 in. MONITOR ON WALL SUPPORT	26 lbs	204 btu
⑦	1	LARGE DISPLAY MONITOR ON SINGLE MONITOR SUSPENSION 9 ft. 6 in. INBOARD BRIDGE (MOUNT TWO GE MONITORS ON BACKSIDE OF LD MONITOR)	784 lbs	1706 btu
⑧	1	CONTROL ROOM MONITOR WITH DL KEYPAD	22 lbs	204 btu
⑨	1	OPERATORS CONSOLE	22 lbs	546 btu
⑩	1	AW WORKSTATION	81 lbs	1201 btu
⑪	1	INNOVA IQ TABLE	2242 lbs	614 btu
⑫	1	INNOVA POSITIONER (REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION)	1653 lbs	2416 btu
⑬	1	WORKSTATION CART		
⑭	1	COMBOLAB CONSOLE INCLUDES MONITORS AND KEYBOARD	566 lbs	2935 btu
⑮	1	TRAM NET RACK	8 lbs	
⑯	1	CLAB 2 PLUS AMPLIFIER	24 lbs	204 btu
⑰	1	REMOTE MONITORING WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑱	1	NURSING NOTES WORKSTATION	46 lbs	682 btu
⑲	1	MAPPING WORKSTATION	46 lbs	682 btu
⑳	1	CLINICAL WORKSTATION	46 lbs	682 btu
㉑	1	COLOR PRINTER		1054 btu
㉒	1	BOLUS CHASE HANDSWITCH	2 lbs	
㉓	1	COUNTERBALANCED EYE AND THYROID SHIELD WITH R96 LAMP	143 lbs	
㉔	2	LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	68 lbs	
㉕	1	REMOTE CONTROL FOR INJECTOR	4 lbs	
㉖	1	INJECTOR HEAD ON TABLE RAIL	15 lbs	
㉗	1	INJECTOR ELECTRONICS	37 lbs	320 btu
㉘	1	XR BUZZER (LOCATED ABOVE CEILING)	2 lbs	
㉙	1	VIVID I ULTRASOUND UNIT (ON SAFELOCK CART)		

ENVIRONMENT

Ambient Operation Temperature: 68°-77° F, (20°-25° C)
 Allowable Temperature Change: Not Available
 Humidity: 30%-75% non condensing
 Allowable Humidity Change: Not Available
 Max - Altitude: 9842 ft. (3000 M) above sea level

ROOM PLAN LOCATED ON PAGE 5-117

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
50	1	DISCOVERY MAIN DISCONNECT, REFERENCE JUNCTION POINT 'MDP' ON SHEET E1 FOR DETAILED DESCRIPTION.	42 lbs	1532 btu
51	6	VITALINQ SPEAKER		
52	1	VITALINQ CONSOLE		
53	2	SKYTRON LIGHTING UNIT	50 lbs	341 btu
54	1	VITALINQ MICROPHONE		
55	1	VITALINQ MICROPHONE (ONE ON MONITOR BRIDGE IN EXAM ROOM)		
<p align="center">***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****</p>				
60		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU		
61		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
62		CATHETER CABINETS		
63		MED GASES IN CEILING		
64		BEARING BLOCK OUTLINE, SEE S1 FOR MORE INFORMATION.		
65		SHELF - CUSTOMER TO PROVIDE ADEQUATE WALL SUPPORT		
66		CABLE DRAPE RAIL.		
67		CUSTOMER SUPPLIED STORAGE CABINET		
68		LEAD APRON RACK		
69		COUNTER TOP FOR EQUIPMENT- MINIMUM DEPTH 30 in. OR ADDITIONAL SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
70		MOBILE RADIATION SHIELD		
90		X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -CAT. NO. E4502SS FOR WARNING LIGHT & ROOM LIGHT CONTROL.		