

## ***Advantage Workstation [Keystrokes] and Accelerator Line Commands***

### **In Viewer, type in the command followed by the [Enter] key**

**?** Help menu  
**7** Selects image 7 (any # can be used)  
**s2** Will bring up series 2.  
**pe** Prior exam  
**ne** Next exam  
**ps** Prior series  
**ns** Next series  
**xr s2** Cross reference all of series 2 (any # can be used)  
**xr s2/1-17** Cross reference images 1- 17 in series 2 (any # can be used)  
**xr s2/1-17:3** Cross reference every third image between 1-17 in series 2  
**xr s2/4-4** Cross reference image 4 in series 2 (any # can be used)  
**noxr** Turns off cross reference lines  
**tpr** Text page ROI's  
**tps** Series text page  
**tpe** Exam text page  
**af** Full annotation  
**ap** Partial annotation  
**an** No annotation  
**prop a** Propagate annotation to all images  
**prop i1-10** Propagate annotation to images 1-10  
**wl 50** Window level of 50 (any # can be used)  
**ww 200** Window width of 200 (any # can be used)  
**zoom 2** Magnifies image with a factor of 2 (any # can be used)  
**no** Display normal

**fh** Flips image(s) horizontally  
**fv** Flips image(s) vertically  
**rr** Rotates image(s) to the right  
**rl** Rotates image(s) to the left  
**invert** Inverts the video, also try the [End] keyboard key when using Viewer  
**tm on** Tick marks on  
**tm off** Tick marks off  
**reset** Returns to original display parameters  
**utp** Will bring up a blank viewport with a user annotation box  
**rshut** Opens a rectangular shutter  
**eshut** Opens an elliptical shutter  
**sshut** Opens a square shutter  
**[Page Up] & [Page Down]** keys to view prior or next images when in Viewer  
**[↓] & [↑]** To adjust the window level when in Viewer  
**[←] & [→]** To adjust the window width when in Viewer

### **Keyboard shortcuts in Volume Analysis**

**[S]** Screen save image  
**[F1]** Send image to scrapbook  
**[F2]** Send screen to scrapbook  
**[F3]** Send multiple image display to scrapbook  
**[F4]** Send image series to the scrapbook with 2 window/levels available  
**[F5]-[F11]** Preset W/L's  
**[◆]** To edit trace lines  
**[Alt] [F5]** To toggle windows front/back