




G	9/Jun/2021	Initial release per PIM revision 18	<div>Typical</div> <div>----</div> <div>----</div>						
F	19/Aug/2020	Initial release per PIM revision 17							
E	05/Dec/2019	Initial release per PIM revision 16							
D	09/Sep/2019	Initial release per PIM revision 15							
C	31/May/2019	Initial release per PIM revision 14							
B	08/Apr/2019	Initial release per PIM revision 13							
A	04/Feb/2019	Initial release per PIM revision 11							
REV	DATE	MODIFICATIONS							
01 - C1 - Cover Sheet 02 - C2 - Disclaimer - Site Readiness Checklist 03 - A1 - General Notes 04 - A2 - Equipment Layout 05 - A3 - Radiation Protection 06 - A4 - Equipment Dimensions (1) 07 - A5 - Equipment Dimensions (2) 08 - A6 - Delivery 09 - S1 - Structural Notes		10 - S2 - Structural Layout 11 - S3 - Structural Details (1) 12 - S4 - Structural Details (2) 13 - M1 - HVAC 14 - E1 - Electrical Notes 15 - E2 - Electrical Layout 16 - E3 - Electrical Elevations 17 - E4 - Power Requirements 18 - E5 - Electrical Details - Interconnect	<div> <div>GE Healthcare</div><div>----</div><div>----</div><div>----</div></div>						
A mandatory component of this drawing set is the GE Healthcare Pre Installation manual. Failure to reference the Pre Installation manual will result in incomplete documentation required for site design and preparation. Pre Installation documents for GE Healthcare products can be accessed on the web at: www.gehealthcare.com/siteplanning			Drawn by		Verified by	Concession	S.O. (GON)	PIM Manual	Rev
			PMM		BAP	-	----	5418654-1EN	18
Format		Scale	File Name				Date	Sheet	
A3		1/4"=1'-0"	EN-CT-TYP-REVOLUTION_CT-WEB.DWG				09/Sep/2021	01/18	

GE does not take responsibility for any damages resulting from changes on drawings made by others. Errors may occur by not referring to the complete set of final issue drawing. GE cannot accept responsibility for any damage due to the partial use of GE final issue drawings, however caused. All dimensions are in millimeters unless otherwise specified. Do not scale from printed pdf files. GE accepts no responsibility or liability for defective work due to scaling from these drawings.

DISCLAIMER

GENERAL SPECIFICATIONS

- GE is not responsible for the installation of developers and associated equipment, lighting, cassette trays and protective screens or derivatives not mentioned in the order.
- The final study contains recommendations for the location of GE equipment and associated devices, electrical wiring and room arrangements. When preparing the study, every effort has been made to consider every aspect of the actual equipment expected to be installed.
- The layout of the equipment offered by GE, the dimensions given for the premises, the details provided for the pre-installation work and electrical power supply are given according to the information noted during on-site study and the wishes expressed by the customer.
- The room dimensions used to create the equipment layout may originate from a previous layout and may not be accurate as they may not have been verified on site. GE cannot take any responsibility for errors due to lack of information.
- Dimensions apply to finished surfaces of the room.
- Actual configuration may differ from options presented in some typical views or tables.
- If this set of final drawings has been approved by the customer, any subsequent modification of the site must be subject to further investigation by GE about the feasibility of installing the equipment. Any reservations must be noted.
- The equipment layout indicates the placement and interconnection of the indicated equipment components. There may be local requirements that could impact the placement of these components. It remains the customer's responsibility to ensure that the site and final equipment placement complies with all applicable local requirements.
- All work required to install GE equipment must be carried out in compliance with the building regulations and the safety standards of legal force in the country concerned.
- These drawings are not to be used for actual construction purposes. The company cannot take responsibility for any damage resulting therefrom.

CUSTOMER RESPONSIBILITIES

- It is the responsibility of the customer to prepare the site in accordance with the specifications stated in the final study. A detailed site readiness checklist is provided by GE. It is the responsibility of the customer to ensure all requirements are fulfilled and that the site conforms to all specifications defined in the checklist and final study. The GE Project Manager of Installation (PMI) will work in cooperation with the customer to follow up and ensure that actions in the checklist are complete, and if necessary, will aid in the rescheduling of the delivery and installation date.
- Prior to installation, a structural engineer of record must ensure that the floor and ceiling is designed in such a way that the loads of the installed system can be securely borne and transferred. The layout of additional structural elements, dimensioning and the selection of appropriate installation methods are the sole responsibility of the structural engineer. Execution of load bearing structures supporting equipment on the ceiling, floor or walls are the customer's responsibility.

RADIO-PROTECTION

- Suitable radiological protection must be determined by a qualified radiological physicist in conformation with local regulations. GE does not take responsibility for the specification or provision of radio-protection.

THE UNDERSIGNED, HEREBY CERTIFIES THAT I HAVE READ AND APPROVED THE PLANS IN THIS DOCUMENT.		
DATE	NAME	SIGNATURE


GLOBAL SITE READINESS CHECKLIST (DI)

DOC1809666 Rev. 7

Site Ready Checks at Installation
EHS Site Requirements
Overall access route to the scan room free from obstruction / high hazards.
Enough space to store tools, equipment, parts, install waste and the general area free from obstruction and trip hazards.
Enough necessary facilities for the GE employees available.
No 3rd parties working in the area that may affect the safety of the installation activity.
Area free from any chemical, gas, dust, welding fume exposure and has painting been completed and dry.
All emergency routes identified, signed and clear from obstruction.
Accessible single source lockable panel that LOTO can be applied to for GE equipment installation (MDP and/or PDU).
There are no other conditions or hazards that you have observed or have been made aware of by the customer or contractors on site.
Required for Mechanical Install start
Room dimensions, including ceiling height, for all Exam, Equipment/Technical & Control rooms meets GE specifications.
Ceiling support structure, if indicated on the GE drawing, is in the correct location and at the correct height according to the Original Equipment Manufacturer specifications.
Levelness and spacing has been measured, and is ready for the installation of any GE supplied components.
Overhead support Structure (unistrut) has been confirmed with customer/contractor to meet required GE provided criteria.
Finished ceiling is installed. If applicable ceiling tiles installed per PMI discretion.
Floor levelness/flatness is measured and within tolerance, and there are no visible defects per GEHC specifications.
Entry door threshold meets PIM requirement.
Rooms that will contain equipment, including staging areas if applicable, are construction debris free. Precautions must be taken to prevent debris from entering rooms containing equipment.
Cable ways (floor/wall/ceiling/Access Flooring) are available for installation of GE cables are of correct length and diameter.
Cable ways routes per GE Final drawings and cable access openings areas installed at a time determined by GEHC PM. Surface floor duct can be installed at time of system installation.
Adequate room illumination installed and working.
Customer supplied countertops where GE equipment will be installed are in place.
Required for Calibration Start
HVAC systems Installed, and the site meets minimum environmental operational system requirements.
System power & grounding (PDB/MDP) is available as per GE specifications.
System power & grounding (PDB/MDP) is installed at point of final connection and ready to use. Lock Out Tag Out is available.
PMI to confirm all feeder wires and breaker are size appropriately. EPO installed if needed.
PMI to confirm with electrician all power and signal cables are well terminated ensuring there are no loose connections.
Network outlets installed.
Computer network available and working.
Lead doors and windows complete or scheduled to be installed. If applicable, radiation protection (shielding) finished & radioprotection regulatory approval for installation obtained.

Note: The details shown here are only an extract from DOC1809666. For the complete document please contact your PMI.

CUSTOMER SITE READINESS REQUIREMENTS	ENVIRONMENT
<ul style="list-style-type: none">Any deviation from these drawings must be communicated in writing to and reviewed by your local GE healthcare installation project manager prior to making changes.Make arrangements for any rigging, special handling, or facility modifications that must be made to deliver the equipment to the installation site. If desired, your local GE healthcare installation project manager can supply a reference list of rigging contractors.New construction requires the following;<ol style="list-style-type: none">Secure area for equipment,Power for drills and other test equipment,Capability for image analysis,Restrooms.Provide for refuse removal and disposal (e.g. crates, cartons, packing)For CT, MR, PET/CT, and SPECT systems it is required to minimize vibrations within the scan room. It is the customer's responsibility to contract a vibration consultant/engineer to implement site design modifications to meet the GE vibration specification. Refer to the system preinstallation manual for vibration specifications.	<p>ALTITUDE</p> <ul style="list-style-type: none">The system shall meet all functional and performance specifications when placed in a room that is at an elevation of -150 m to 2,400 m [-492 ft to 7,875 ft] above sea level. For different altitudes refer to the PIM. <p>MAGNETIC FIELD SPECIFICATIONS</p> <ul style="list-style-type: none">Limit the magnetic interference to guarantee specified imaging performance. <p>GANTRY:</p> <ul style="list-style-type: none">Ambient static magnetic fields less than 1 Gauss.Ambient AC magnetic fields less than 0.01 Gauss. <p>OPERATOR CONSOLE:</p> <ul style="list-style-type: none">Ambient static magnetic fields less than 10 Gauss. <p>SYSTEM CABINET:</p> <ul style="list-style-type: none">Ambient static magnetic fields less than 10 Gauss. <p>SYSTEM NOISE LEVEL</p> <ul style="list-style-type: none">The maximum noise level produced by the system in the scan room is less than 70 dBA at one meter [3.3 ft] from any surface of the system.
Typical	REVOLUTION CT/CT ES/APEX/CT WITH APEX EDITION CT-TYP-REVOLUTION CT-WEB.DWG Rev G Date 09/Sep/2021 General Notes 03/18

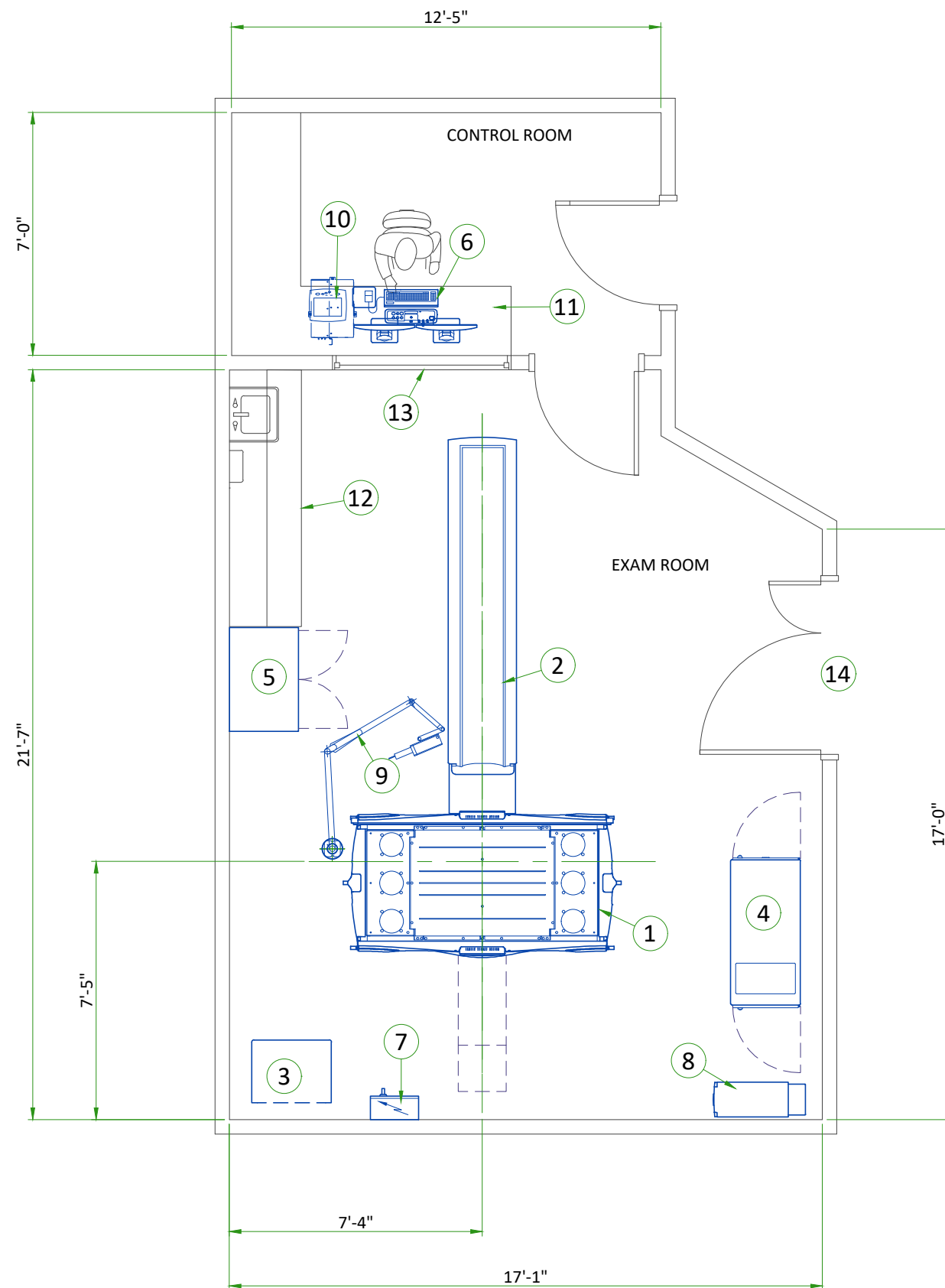

LEGEND

A	GE Supplied	D	Available from GE
B	GE Supplied/contractor installed	E	Equipment existing in room
C	Customer/contractor supplied and installed	*	Item to be reinstalled from another site

BY	ITEM	DESCRIPTION	MAX HEAT OUTPUT (btu)	WEIGHT (lbs)	MAX HEAT OUTPUT (W)	WEIGHT (kg)
A	1	Gantry	27150	6335	7950	2875
A	2	Patient Table (NG-2000V)	-	1474	-	670
A	3	Power Distribution Unit	1200	796	352	361
A	4	System Cabinet IV	10578	736	3100	334
A	5	Service Cabinet	-	90	-	41
A	6	Operators Console	5100	106	1500	48
D/B	7	Main Disconnect Panel	-	55	-	25
D	8	Partial UPS (14 kVA)	3000	620	880	282
D	9	Injector - ceiling mounted	-	79	-	36
D	10	Injector Control and Electronics	320	22	94	10
C	11	Counter top for equipment				
C	12	Counter top with sink, base and wall cabinets				
C	13	Lead glass window				
C	14	Minimum door opening for equipment delivery is 46 in. w x 78 in. h [1168mm x 1980mm], contingent on a 96 in. [2438mm] corridor width				

EXAM ROOM HEIGHT

Finished ceiling height	8'-0"
-------------------------	-------

For Accessory Sales: (866) 281-7545 Options 1, 2, 1, 2 or mail to: gehaccessorysales@ge.com

RADIATION PROTECTION TABLES

A cylindrical PMMA phantom with a diameter of 32 cm [12.60 in] and a length of 25 cm [9.84 in] (for the 80 mm [3.15 in] collimation measurements) or 30 cm [11.81 in] (for the 160 mm [6.30 in] collimation measurements) is centered in the scan plane and scanned. Scatter radiation measurements are made for both the vertical and the horizontal planes which include the axis of rotation. The horizontal plane is 1.03 m [3.38 ft] above the floor. Tables below show scatter radiation for 80 mm [3.15 in] collimation. The air kerma per 100 mAs (μGy / 100 mAs) is provided at 0.5 m [1.6 ft] intervals within each plane. The CT scan technique that results in the maximum scatter radiation per unit mAs is used for all measurements as follows:

- Detector Coverage = 80 mm [3.15 in] (128x0.625 mm [5.04x0.02 in])
- Tube voltage = 140 kV
- SFOV = Medium Body

Typical Scatter Radiation in μGy/100 mAs - Horizontal Plane 80 mm [3.15 in] Collimation													
X-axis (m [ft])	Z-axis (m [ft])												
	3 [9.8]	2.5 [8.2]	2 [6.6]	1.5 [4.9]	1 [3.3]	0.5 [1.6]	0 [0]	-0.5 [-1.6]	-1 [-3.3]	-1.5[-4.9]	-2 [-6.6]	-2.5[-8.2]	-3 [-9.8]
-1.5 [-4.9]	1.9	2.7	3.9	3.5	2.6	0.4	0.1	0.1	0.1	0.2	1.0	1.5	1.3
-1 [-3.3]	1.9	2.9	4.8	8.3	12.3	2.3	ISO		2.1	4.7	3.6	2.4	1.7
-0.5 [-1.6]	1.9	2.7	4.7	8.4	18.2	45.4			16.5	6.3	4.4	2.4	1.3
0 [0]	1.1	2.6	4.1	8.0	21.2	81.2			26.1	7.6	4.3	2.4	1.5
0.5 [1.6]	1.6	2.7	4.5	7.7	18.6	45.4			16.5	6.3	4.4	2.4	1.3
1 [3.3]	1.9	2.9	4.8	8.3	12.3	2.3			2.1	4.7	3.6	2.4	1.7
1.5 [4.9]	1.9	2.7	3.9	3.5	2.6	0.4	0.1	0.1	0.1	0.2	1.0	1.5	1.3

Typical Scatter Radiation in μGy/100 mAs - Vertical Plane 80 mm [3.15 in] Collimation													
Y-axis (m [ft])	Z-axis (m [ft])												
	3 [9.8]	2.5 [8.2]	2 [6.6]	1.5 [4.9]	1 [3.3]	0.5 [1.6]	0 [0]	-0.5 [-1.6]	-1 [-3.3]	-1.5[-4.9]	-2 [-6.6]	-2.5[-8.2]	-3 [-9.8]
1.5 [4.92]	2.2	3.2	4.4	6.3	3.3	0.3	0.3	0.1	0.1	1.4	2.4	2.3	2.1
1 [3.28]	2.1	3.1	4.7	6.7	10.7	2.8	ISO		1.1	4.7	4.3	2.9	1.9
0.5 [1.64]	2.1	3.1	5.1	9.1	18.0	42.4			19.7	7.9	4.4	2.8	1.8
0 [0]	1.1	2.6	4.1	8.0	21.2	81.2			26.1	7.6	4.3	2.4	1.5
-0.5 [-1.64]	Table								14.9	6.2	4.2	2.5	1.5


Tables below show scatter radiation for 160 mm [6.30 in] collimation. The air kerma per 100 mAs (μGy / 100 mAs) is provided at 0.5 m [1.6 ft] intervals within each plane. The CT scan technique that results in the maximum scatter radiation per unit mAs is used for all measurements as follows:

- Detector Coverage = 160 mm [6.30 in] (256x0.625 mm [10.08x0.02 in])
- Tube voltage = 140 kV
- SFOV = Medium Body


Typical Scatter Radiation in μGy/100 mAs - Horizontal Plane 160 mm [6.30 in] Collimation													
X-axis (m [ft])	Z-axis (m [ft])												
	3 [9.8]	2.5 [8.2]	2 [6.6]	1.5 [4.9]	1 [3.3]	0.5 [1.6]	0 [0]	-0.5 [-1.6]	-1 [-3.3]	-1.5[-4.9]	-2 [-6.6]	-2.5[-8.2]	-3 [-9.8]
-1.5 [-4.9]	3.7	4.6	7.1	9.2	5.7	1.8	0.2	0.7	0.7	3.1	7.8	5.2	3.5
-1 [-3.3]	3.6	4.8	7.8	12.8	16.9	8.4	ISO		9.2	11.3	4.6	3.6	3.4
-0.5 [-1.6]	3.9	4.7	8.0	22.0	29.4	68.3			49.8	17.8	8.3	5.0	3.4
0 [0]	3.0	3.4	4.1	25.8	35.4	86.8			52.1	17.0	8.1	4.8	3.2
0.5 [1.6]	3.0	4.0	7.2	14.3	29.5	56.2			50.9	15.1	7.7	5.2	4.4
1 [3.3]	3.3	4.6	7.5	13.2	27.8	6.5			36.3	15.3	9.2	5.1	4.2
1.5 [4.9]	3.1	4.4	6.5	8.3	8.3	2.1	0.6	0.4	0.2	1.8	8.8	3.4	3.0

Typical Scatter Radiation in μGy/100 mAs - Vertical Plane 160 mm [6.30 in] Collimation													
Y-axis (m [ft])	Z-axis (m [ft])												
	3 [9.8]	2.5 [8.2]	2 [6.6]	1.5 [4.9]	1 [3.3]	0.5 [1.6]	0 [0]	-0.5 [-1.6]	-1 [-3.3]	-1.5[-4.9]	-2 [-6.6]	-2.5[-8.2]	-3 [-9.8]
1.5 [4.9]	3.5	5.0	7.3	9.6	7.9	0.7	0.5	0.2	0.3	1.6	4.4	4.2	3.6
1 [3.3]	3.6	5.2	8.4	11.1	15.1	9.7			0.3	14.7	8.7	5.7	4.4
0.5 [1.6]	3.7	5.0	10.3	13.5	15.6	7.8	ISO		1.4	11.6	8.4	5.6	3.9
0 [0]	3.0	3.4	4.1	25.8	35.4	86.8			52.1	17.0	8.1	4.8	3.2
-0.5 [-1.6]	Table								26.2	14.7	9.6	5.3	3.4


GANTRY WITH TABLE

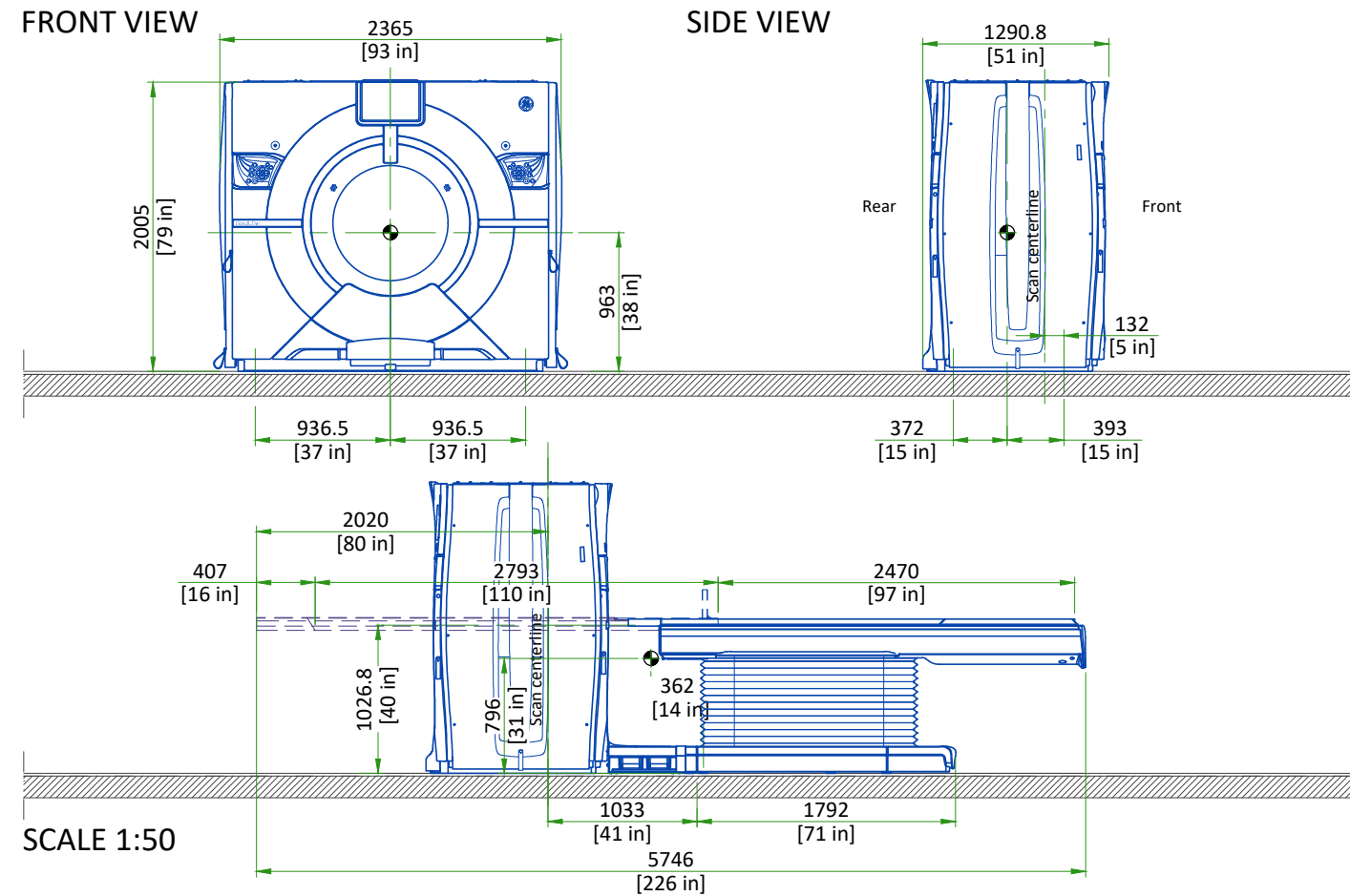


POWER DISTRIBUTION UNIT (PDU)

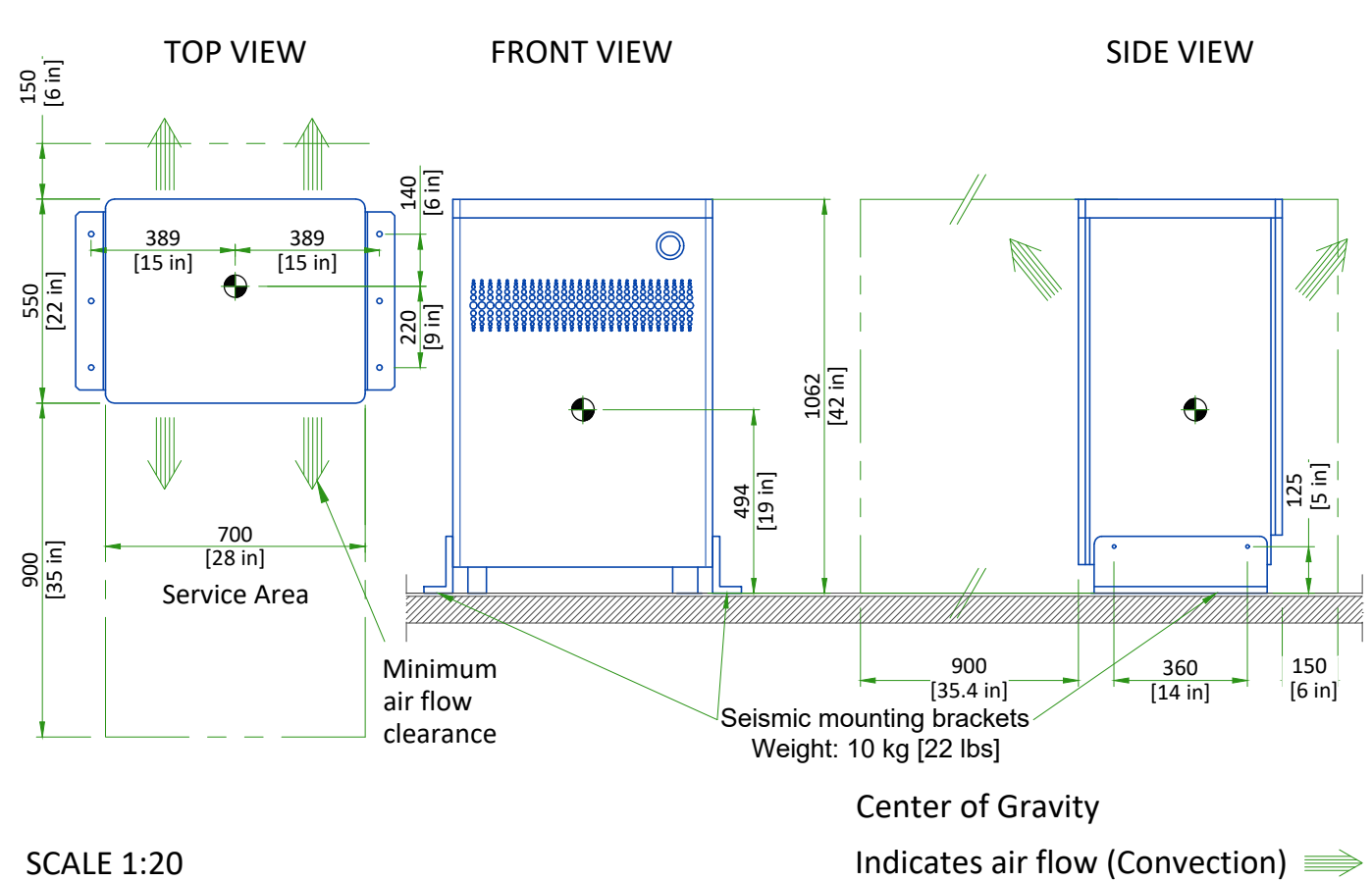

SYSTEM CABINET

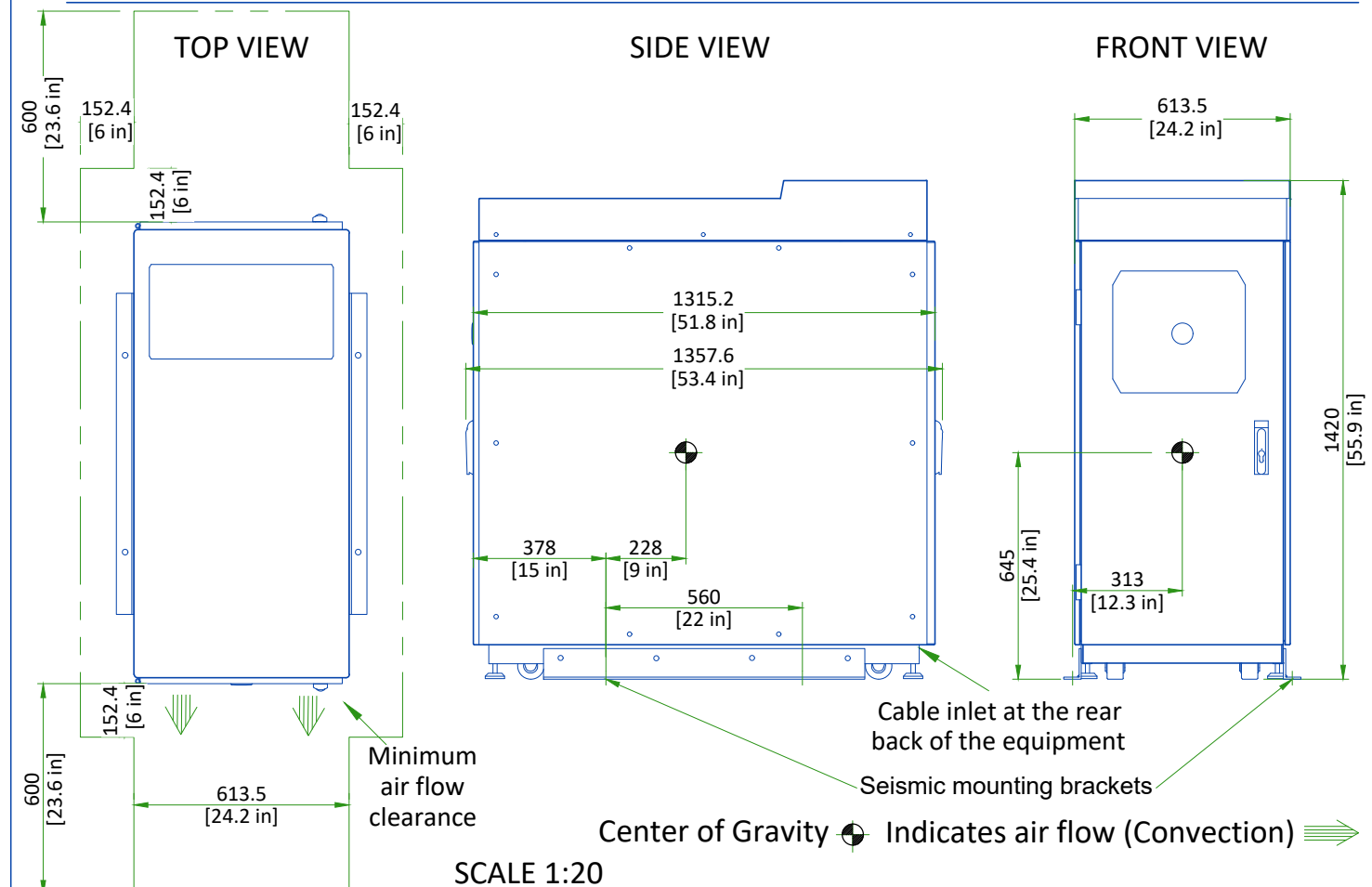


SMART WORKSPACE DESK AND OPEN CONSOLE

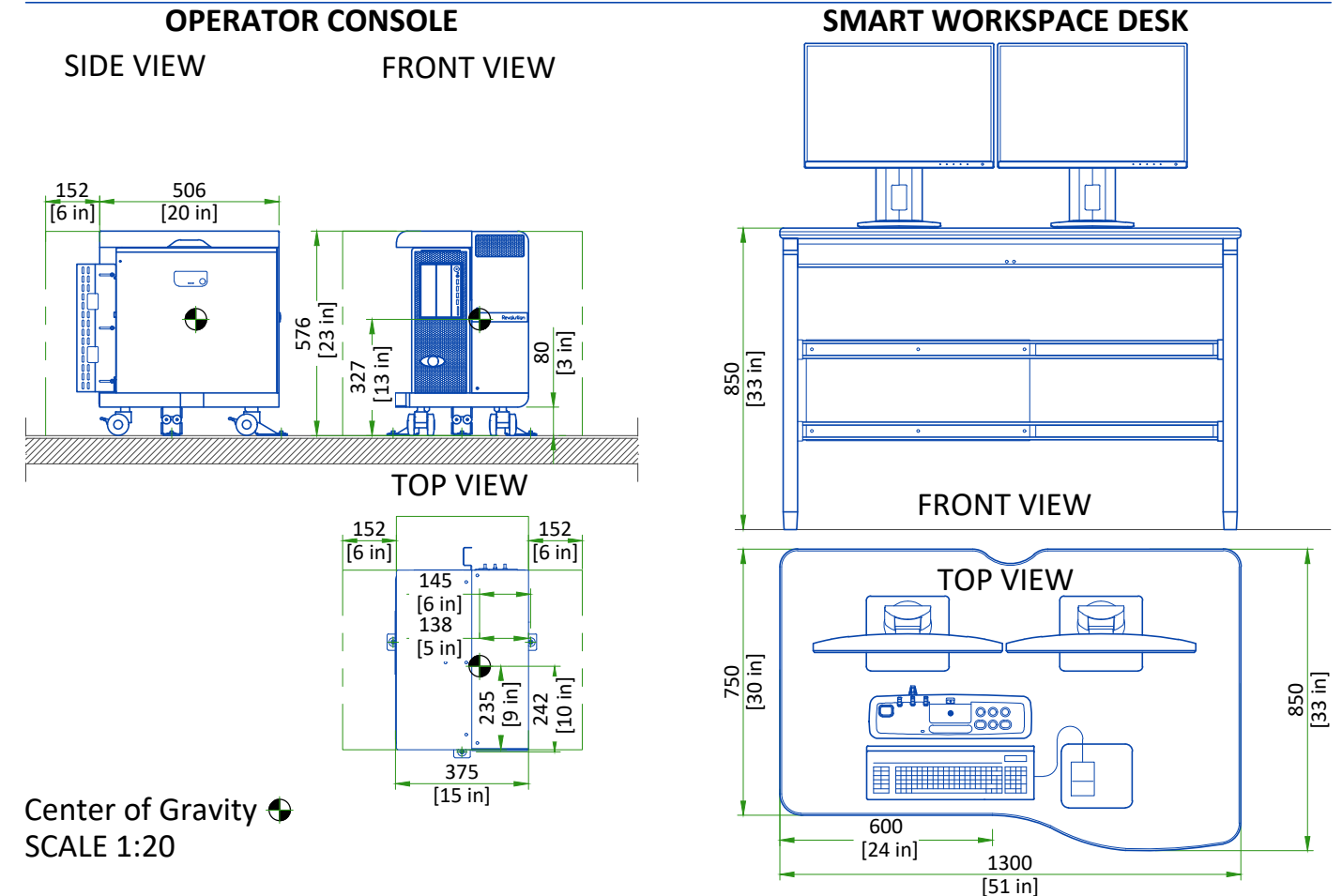


MAIN DISCONNECT PANEL


TOP VIEW

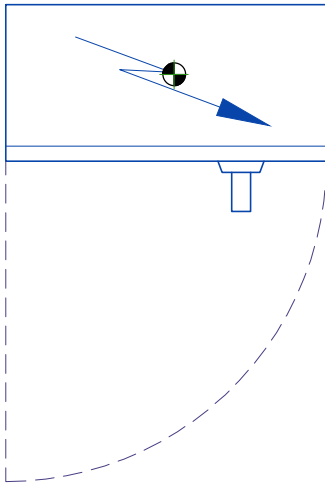


FRONT VIEW

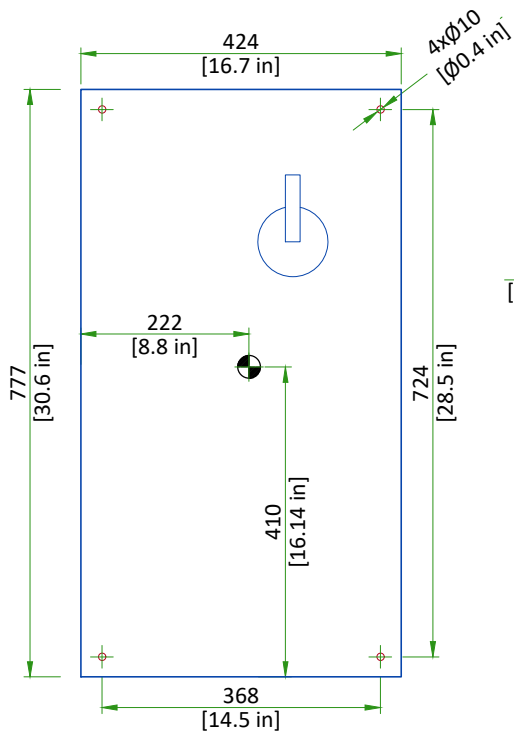

SIDE VIEW

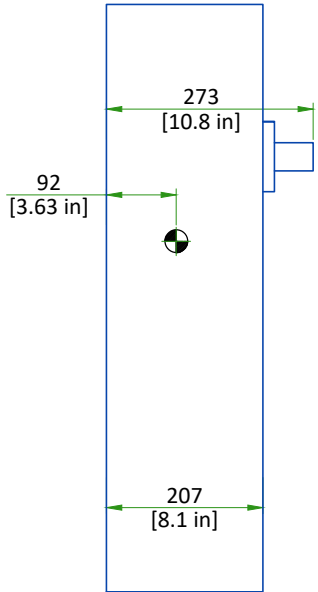


Center of gravity 
SCALE 1:10

DELIVERY

THE CUSTOMER/CONTRACTOR SHOULD:

- Provide an area adjacent to the installation site for delivery and unloading of the GE equipment.
- Ensure that the dimensions of all doors, corridors, ceiling heights are sufficient to accommodate the movement of GE equipment from the delivery area into the definitive installation room.
- Ensure that access routes for equipment will accommodate the weights of the equipment and any transportation, lifting and rigging equipment.
- Ensure that all necessary arrangements for stopping and unloading on public or private property not belonging to the customer have been made.


DIMENSIONS OF DELIVERY WITH DOLLY TRANSPORT EQUIPMENT					
EQUIPMENT		DIMENSIONS		WEIGHT	
GANTRY	STATIONARY ASSEMBLY	LENGTH	2753 mm [108.4 in]	1738.8 kg	3830 lbs
		WIDTH	1149 mm [45.2 in]		
		HEIGHT	2049 mm [80.7 in]		
	ROTATING ASSEMBLY	LENGTH	2878 mm [113.3 in]	1681.6 kg	3704 lbs
		WIDTH	1018 mm [40.1 in]		
		HEIGHT	1905 mm [75 in]		
NG-2000V PATIENT TABLE		LENGTH	2972 mm [117 in]	737.2 kg	1622 lbs
		WIDTH	1120 mm [44.1 in]		
		HEIGHT	1230 mm [48.4 in]		

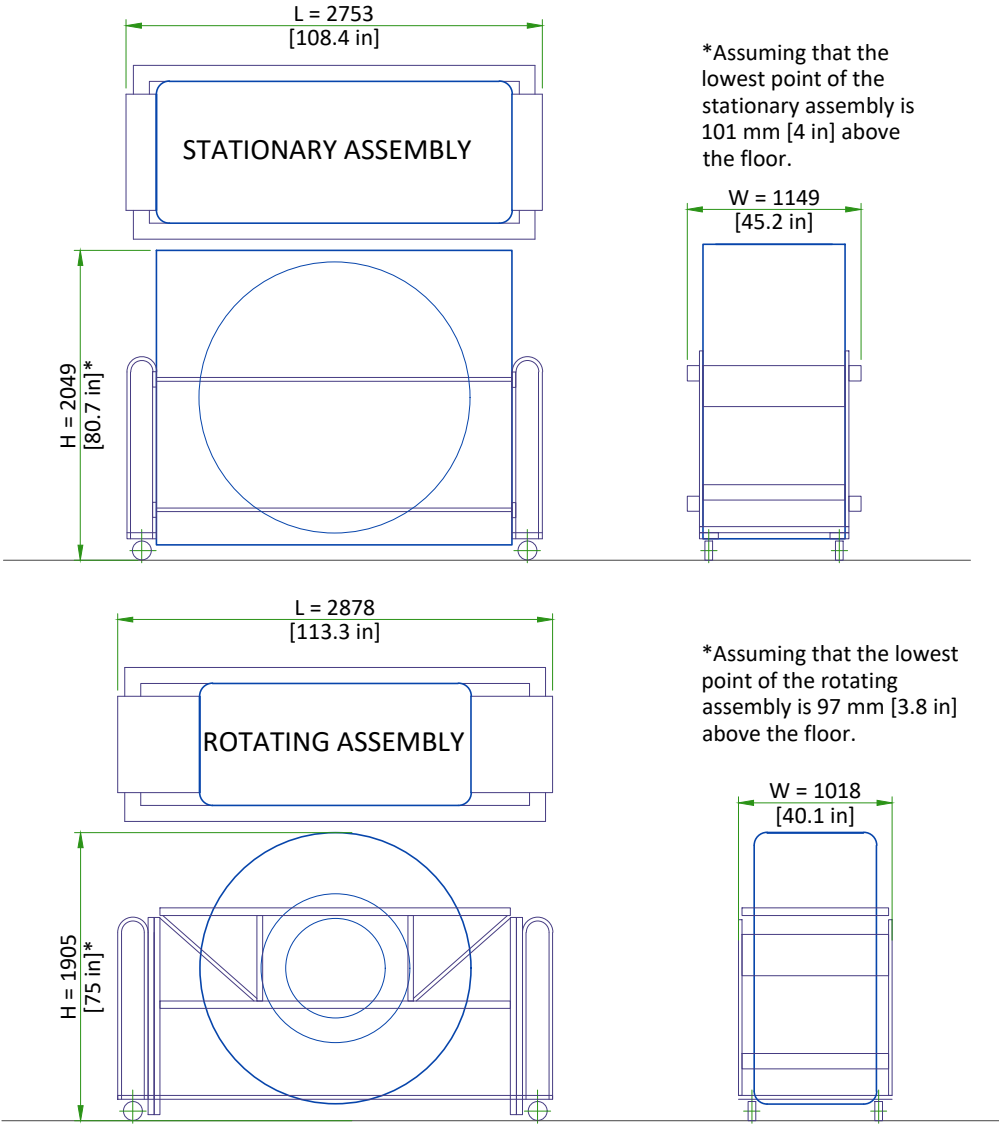

NOT TO SCALE


STRUCTURAL NOTES

- all steel work and parts necessary to support ceiling mounted equipment is to be supplied by the customer or his contractors.
- methods of support for the steelwork that will permit attachment to structural steel or through bolts in concrete construction should be favored. Do not use concrete or masonry anchors in direct tension.
- all units that are wall mounted or wall supported are to be provided with supports where necessary. Wall supports are to be supplied and installed by the customer or his contractors. See plan and detail sheets for suggested locations and mounting hole locations.
- all ceiling mounted fixtures, air vents, sprinklers, etc. To be flush mounted, or shall not extend more than 1/4" below the finished ceiling.
- floor slabs on which equipment is to be installed must be level to 1/4" in 10'-0"
- dimensions are to finished surfaces of room.
- customers contractor must provide all penetrations in post tension floors.
- customers contractor must provide and install any non-standard anchoring. Documents for standard anchoring methods are included with GE equipment drawings for geographic areas that require such documentation.
- customers contractor must provide and install hardware for "through the floor" anchoring and/or any bracing under access floors. This contractor must also provide floor drilling that cannot be completed because of an obstruction encountered while drilling by the GE installer such as rebar etc.
- it is the customer's responsibility to perform any floor or wall penetrations that may be required. The customer is also responsible for ensuring that no subsurface utilities (e.g., electrical or any other form of wiring, conduits, piping, duct work or structural supports (i.e. post tension cables or rebar)) will interfere or come in contact with subsurface penetration operations (e.g. drilling and installation of anchors/screws) performed during the installation process. To ensure worker safety, GE installers will perform surface penetration operations only after the customer's validation and completion of the "GE surface penetration permit"

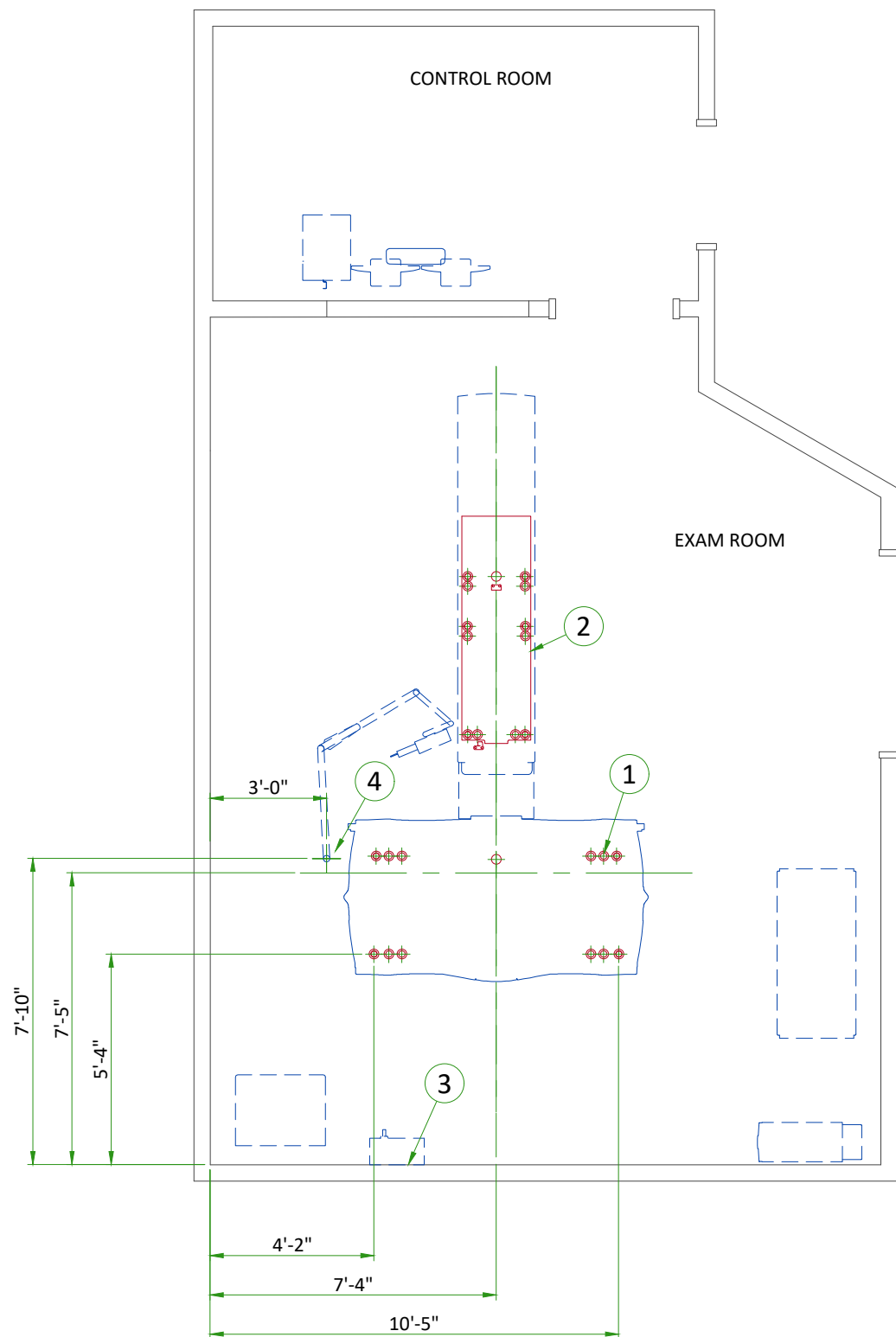


ITEM	DESCRIPTION
(CONTRACTOR SUPPLIED & INSTALLED)	
1	Gantry leveling pads. See Structural Detail
2	Table base plate and leveling pads. See Structural Detail
3	Support Backing, locate as shown
4	Structural supports for fastening the overhead counterpoised suspension. Support should run continuous with no fittings extending below face of channel, be parallel, square, and in the same horizontal plane, above finished ceiling. Ensure mounting surface is installed level or plumb within +/- 1 degree, and is structurally sufficient to maintain a level or plumb condition under 110 lb (50kg) system load and maximum system moment of 4400 in-lb (500n-m). Methods of support that will permit attachment to structural steel or through bolts in concrete construction should be favored. Do not use screw anchors in direct tension. 14" x 14" x 1/2" thick steel plate provided by manufacturer. See detail on structural detail sheets.

ANCHORING/LOADING DISTRIBUTION TO THE FLOOR

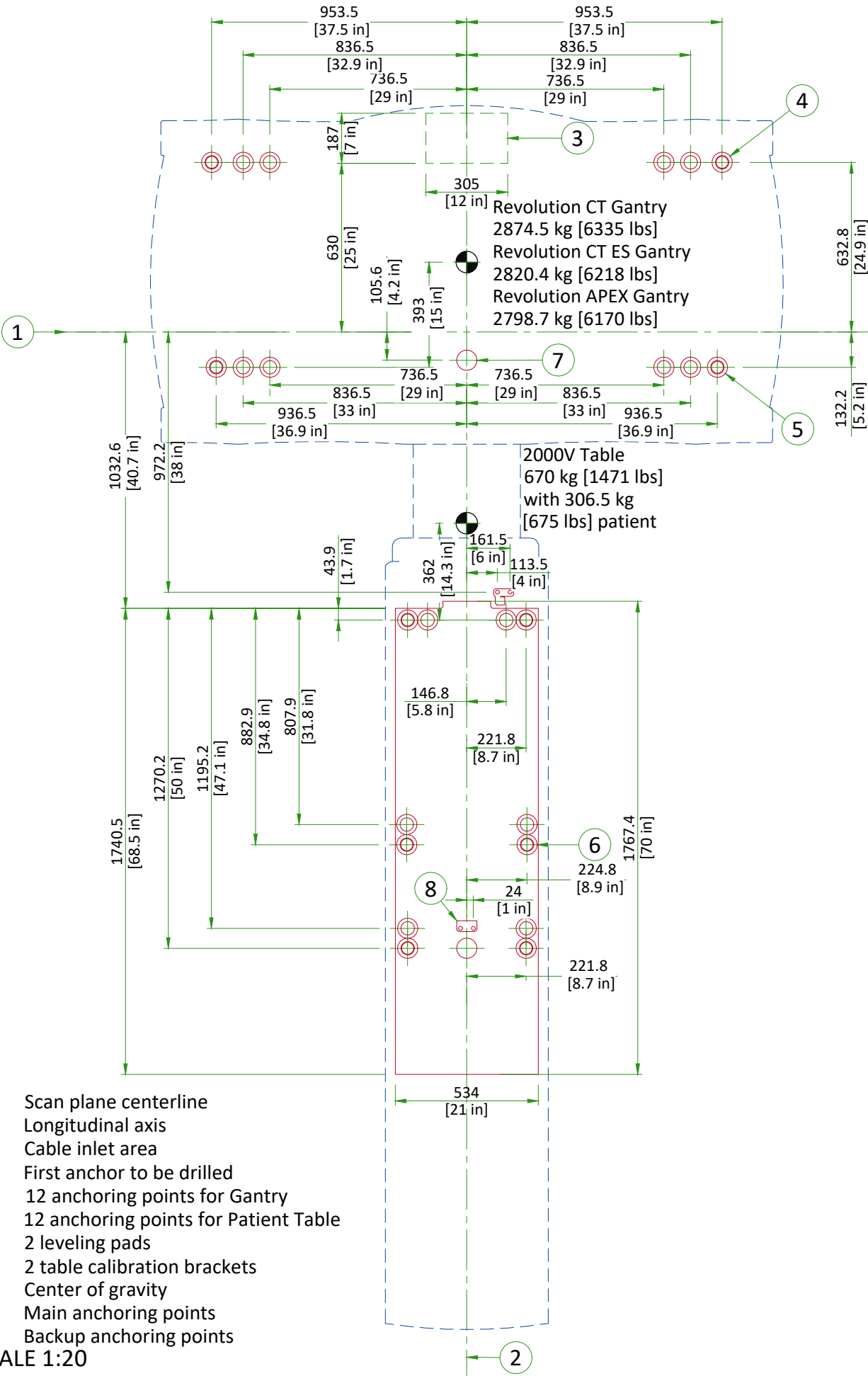

FLOOR SPECIFICATIONS

GE SUPPLIED GANTRY ANCHORS

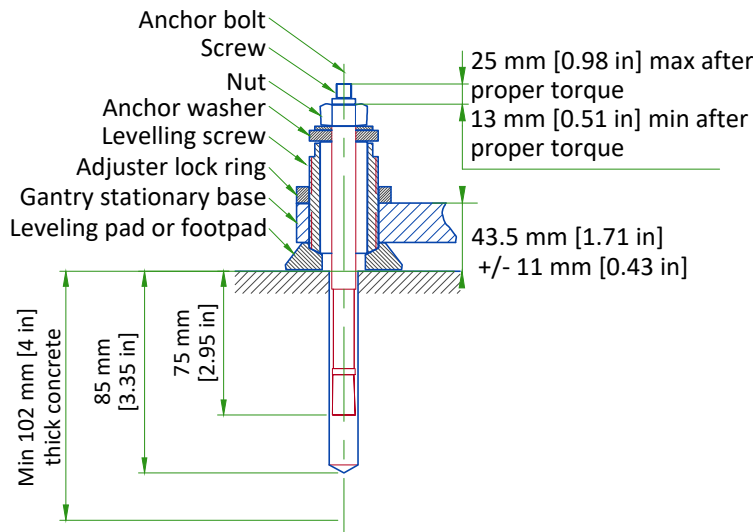

FINISHED FLOOR REQUIREMENTS


- Installation requires a finished floor in the scan and control rooms.
- The floor surface in the scan room directly under the gantry and table must be level.
- The floor levelness tolerance of the floor surface that the gantry and table will rest on is 6 mm [0.25 in] over a 3500 mm [10.0 ft] distance.
- Shims should not be used to compensate for a floor that does not meet this requirement.
- If the concrete floor has a floor covering installed over it (such as floor tile), the foot pad load bearing areas under the gantry and the patient table shall be cut into the flooring to ensure the table and gantry rest on a solid surface. (Openings cut during installation.)
- These floor penetrations can be sealed if required. These requirements apply to all installation types.

Notes:

- Anchors must be embed at least 100 mm [3.9 in] from concrete floor edge or expansion joint

NOT TO SCALE

CUSTOMER/CONTRACTOR SUPPLIED MOUNTING PLATE

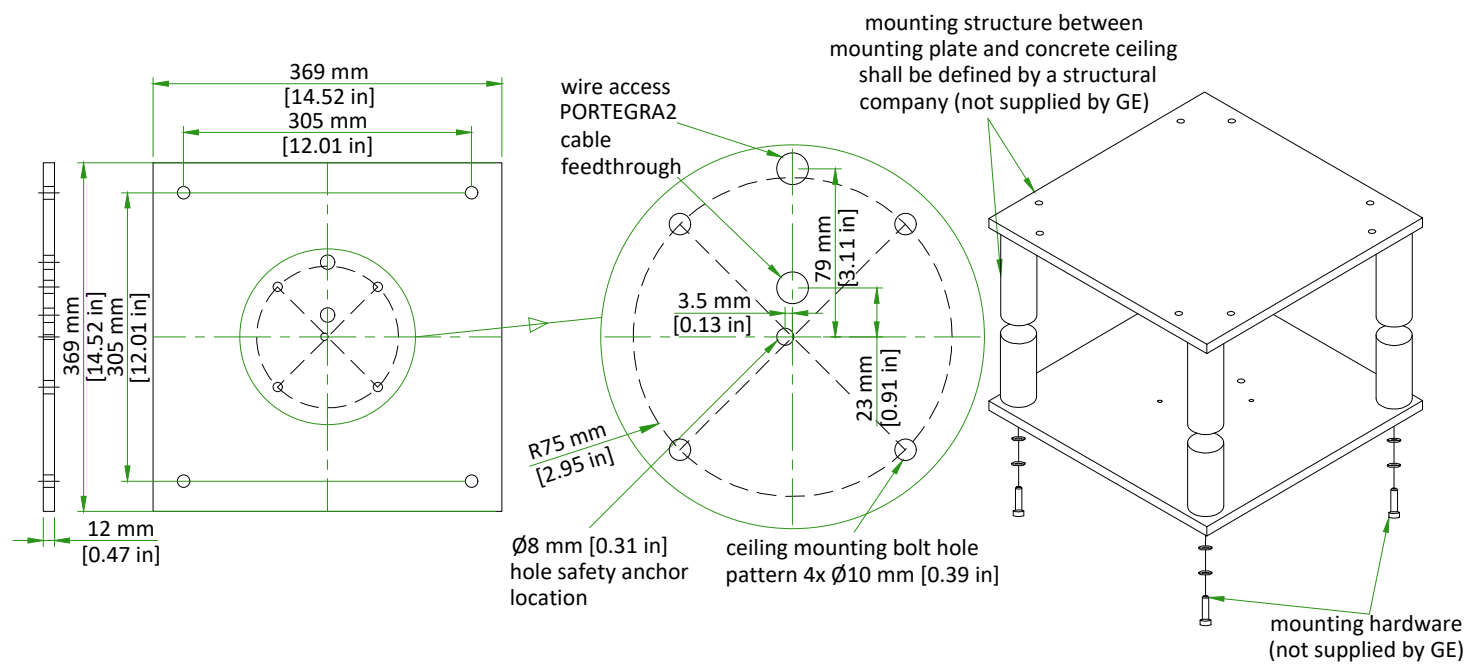

The exact location of all five drill holes for MAVIG column has to be kept, otherwise installation can't be accomplished. Column flange and safety chain fixings to concrete or to structure other than MAVIG anchoring plate or MAVIG bridge shall be defined by a structural company. All design and pre-installation activity must be done in accordance of the MAVIG Installation manual. Contact your GE Project Manager for OEM documentation. Installation of mounting plate performed by GE or a GE sub-contractor.

NOT TO SCALE

CEILING SUSPENSION DISCLAIMER

Safety and precautionary comments:

Only qualified, licensed technicians can perform electrical connections, installation, removal and repair. It is strongly recommended that at least two persons perform the installation.

Installing the system: Prior to installation, a structural engineer must confirm that the mounting structure is strong enough to provide proper support for the entire system and any attached end devices. Installation must be completed according to local building codes.

Determination of required installation hardware and torque values for installation of the ceiling column and ceiling track is the sole responsibility of the structural engineer.

Ceiling mounted systems must be installed properly. Failure to follow the instructions provided may lead to a potentially dangerous and unstable condition of the system.

GE and/or MAVIG is not responsible for unauthorized modifications made to the system or use of the system for unintended purposes. GE and/or MAVIG cannot be held liable for improper operation and modifications. Since improper modifications may impair proper operation, safety or reliability of the system, product modifications require written authorization from MAVIG.


Under GE responsibility or under Customer responsibility, for all pre-installations, whatever is the supporting structure (bridge, chair, Unistrut channel, other channels, direct anchorage in concrete, transversal beam, etc. ...) a certificate must be obtained from a structural engineer.

This certificate shall include the definition of fasteners and of their tightening torque, especially for the non-standard cases described in MAVIG PIM and for which the standard anchoring/screws delivered with product shall not be used but shall be defined (and implemented in most cases) by the structural company.

WARNING:

It is prohibited to alter the length of the ceiling column or remove any securing screws.


SUSPENSION COLUMN LENGTHS AND INSTALLATION DETAILS

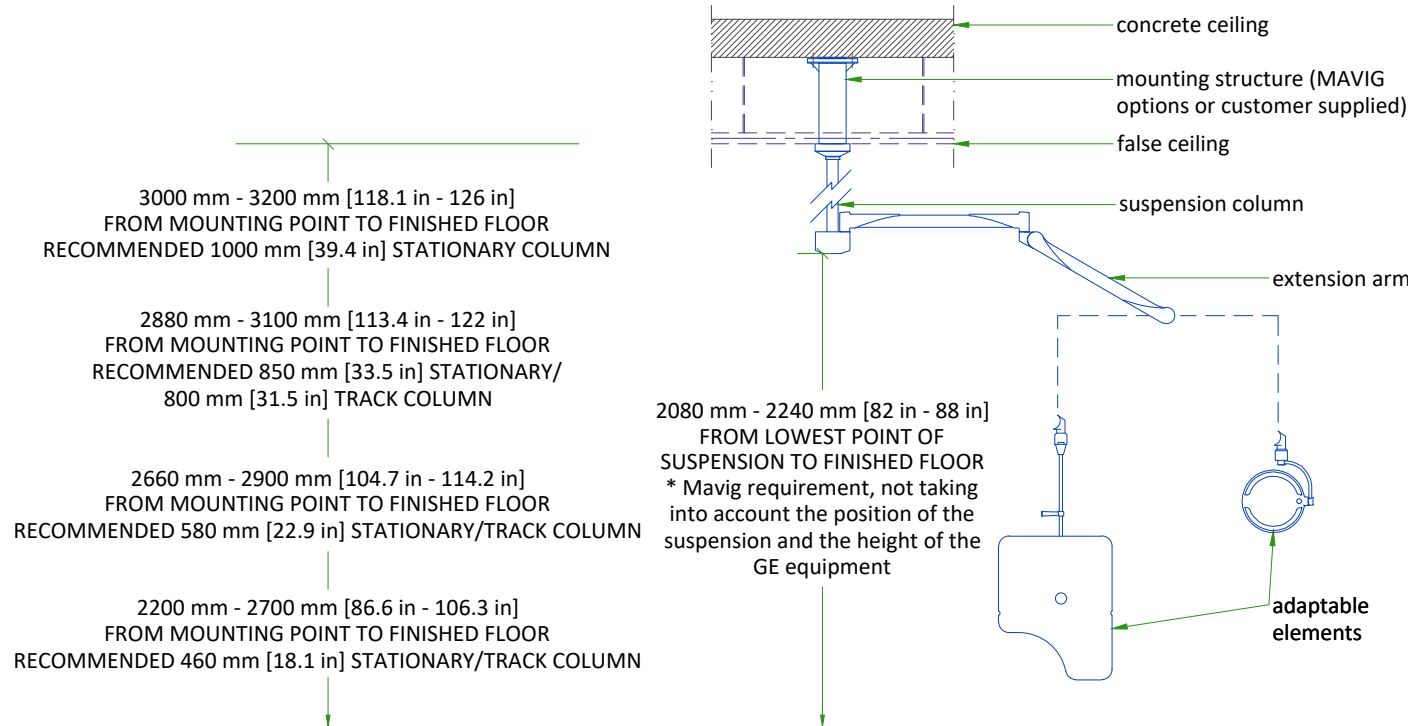

- Available column lengths might differ, please refer to the GE commercial catalog for current selection options
- For rooms with higher mounting point than 3200 mm [126 in], a ceiling construction between structural ceiling and vertical column is suggested which needs to be designed by a structural engineer
- All design and pre-installation activity must be done in accordance of the MAVIG Installation manual
- Contact your GE Project Manager for OEM documentation
- Installation of mounting plate performed by GE or a GE sub-contractor

NOT TO SCALE

PORTEGRA2 COLUMN ASSEMBLY

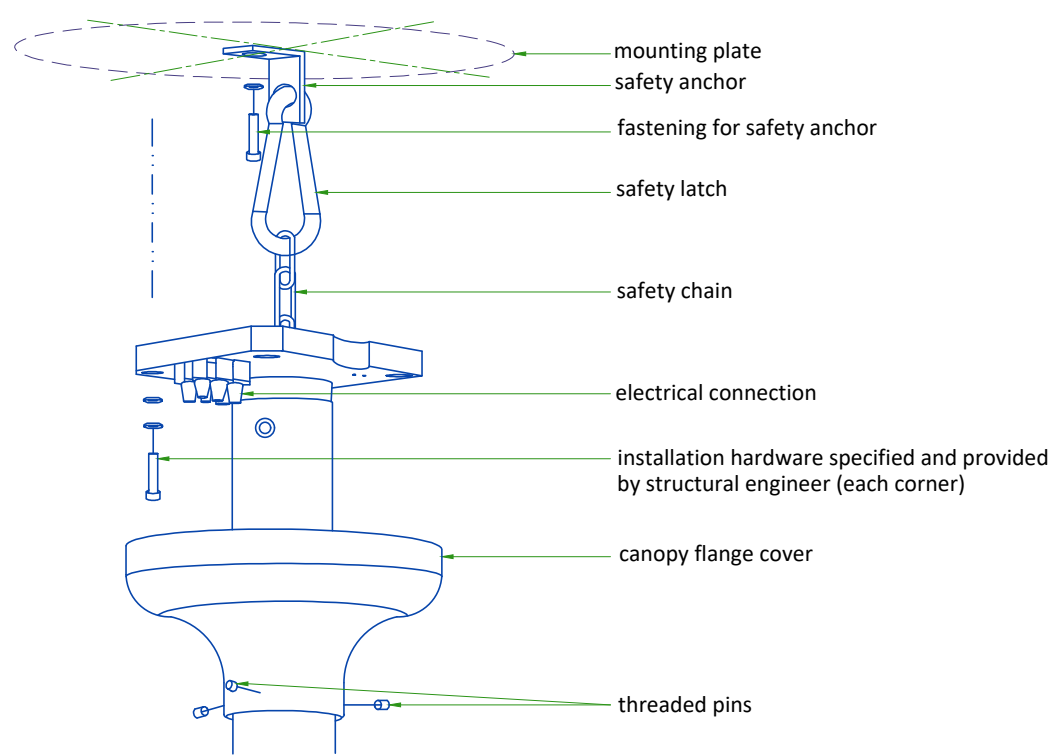

- Safety chain shall be always attached.
- Do not use shims between column and mounting surface.

All design and pre-installation activity must be done in accordance of the MAVIG Installation manual. Contact your GE Project Manager for OEM documentation. Installation of mounting plate performed by GE or a GE sub-contractor.

NOT TO SCALE

TEMPERATURE AND HUMIDITY SPECIFICATIONS

IN-USE CONDITIONS

	EXAM ROOM			CONTROL ROOM		
Temperature (From -150 m [-492 ft] to 1600 m [5249 ft])	Min	Recommended	Max	Min	Recommended	Max
	18°C	22°C	26°C	18°C	22°C	26°C
	64°F	72°F	79°F	64°F	72°F	79°F
Temperature (From 1600 m [5249 ft] to 3000 m [9843 ft])	Min	Recommended	Max	Min	Recommended	Max
	18°C	22°C	25°C	18°C	22°C	25°C
	64°F	72°F	77°F	64°F	72°F	77°F
Relative humidity (1)	30% to 70%			30% to 70%		

STORAGE CONDITIONS

Temperature	+4°C to +27°C
	40°F to +80°F
Temperature gradient	≤ 3°C/h
	≤ 5.4°F/h
Relative humidity (1)	20% to 60%
Humidity gradient	≤ 5%/h

Storage longer than 6 months is not recommended.

(1) Non-condensing

AIR RENEWAL

According to local standards.

NOTE

In case of using air conditioning systems that have a risk of water leakage it is recommended not to install it above electric equipment or to take measures to protect the equipment from dropping water.

HEAT DISSIPATION DETAILS

ROOM	DESCRIPTION	Max (btu)	Max (kW)
Exam Room	Gantry and Patient Table	27150	7.95
	TOTAL	27150	8.0
Exam Room or Technical Room*	Power Distribution Unit	1200	0.352
	System Cabinet IV	10578	3.1
	Partial UPS - Powerware 9355-15-14GE	3000	0.88
	TOTAL	14778	4.3
Control Room	Operator console (including 2 monitors)	5100	1.5
	TOTAL	5100	1.5
*Technical Room is not mandatory, the placements of these elements are recommended in the Exam Room.			

ELECTRICAL NOTES

- 1. All wires specified shall be copper stranded, flexible, thermo-plastic, color coded, cut 10 foot long at outlet boxes, duct termination points or stubbed conduit ends. All conductors, power, signal and ground, must be run in a conduit or duct system. Electrical contractor shall ring out and tag all wires at both ends. Wire runs must be continuous copper stranded and free from splices.
- 1.1. Aluminum or solid wires are not allowed.
- 2. Wire sizes given are for use of equipment. Larger sizes may be required by local codes.
- 3. It is recommended that all wires be color coded, as required in accordance with national and local electrical codes.
- 4. Conduit sizes shall be verified by the architect, electrical engineer or contractor, in accordance with local or national codes.
- 5. Convenience outlets are not illustrated. Their number and location are to be specified by others. Locate at least one convenience outlet close to the system control, the power distribution unit and one on each wall of the procedure room. Use hospital approved outlet or equivalent.
- 6. General room illumination is not illustrated. Caution should be taken to avoid excessive heat from overhead spotlights. Damage can occur to ceiling mounting components and wiring if high wattage bulbs are used. Recommend low wattage bulbs no higher than 75 watts and use dimmer controls (except MR). Do not mount lights directly above areas where ceiling mounted accessories will be parked.
- 7. Routing of cable ductwork, conduits, etc., must run direct as possible otherwise may result in the need for greater than standard cable lengths (refer to the interconnection diagram for maximum usable lengths point to point).
- 8. Conduit turns to have large, sweeping bends with minimum radius in accordance with national and local electrical codes.
- 9. A special grounding system is required in all procedure rooms by some national and local codes. It is recommended in areas where patients might be examined or treated under present, future, or emergency conditions. Consult the governing electrical code and confer with appropriate customer administrative personnel to determine the areas requiring this type of grounding system.
- 10. The maximum point to point distances illustrated on this drawing must not be exceeded.
- 11. Physical connection of primary power to GE equipment is to be made by customers electrical contractor with the supervision of a GE representative. The GE representative would be required to identify the physical connection location, and insure proper handling of GE equipment.
- 12. GEHC conducts power audits to verify quality of power being delivered to the system. The customer's electrical contractor is required to be available to support this activity.


- All junction boxes, conduit, duct, duct dividers, switches, circuit breakers, cable tray, etc., are to be supplied and installed by customers electrical contractor.
- Conduit and duct runs shall have sweep radius bends
- Conduits and duct above ceiling or below finished floor must be installed as near to ceiling or floor as possible to reduce run length.
- Ceiling mounted junction boxes illustrated on this plan must be installed flush with finished ceiling.
- All ductwork must meet the following requirements:
 - 1.Ductwork shall be metal with dividers and have removable, accessible covers.
 - 2.Ductwork shall be certified/rated for electrical power purposes.
 - 3.Ductwork shall be electrically and mechanically bonded together in an approved manner.
 - 4.PVC as a substitute must be used in accordance with all local and national codes.
- All openings in access flooring are to be cut out and finished off with grommet material by the customers contractor.
- General contractor to insert pull cords for all cable run conduits between the equipment room and the operators control room.
- 10 foot pigtails at all junction points.
- Grounding is critical to equipment function and patient safety. Site must conform to wiring specifications shown on this plan.

CONNECTIVITY REQUIREMENTS

Broadband Connections are necessary during the installation process and going forward to ensure full support from the Engineering Teams for the customers system. Maximum performance and availability for the customers system is maintained and closely monitored during the lifetime of the system. Proactive and reactive maintenance is available utilising the wide range of digital tools using the connectivity solutions listed below:

- Site-to-Site VPN/GE Solution
- Site-to-Site VPN/Customer Solution
- Connection through Dedicated Service Network
- Internet Access - connectivity for InSite 2.0


The requirements for these connectivity solutions are explained in the broadband solutions catalogue (separate document).

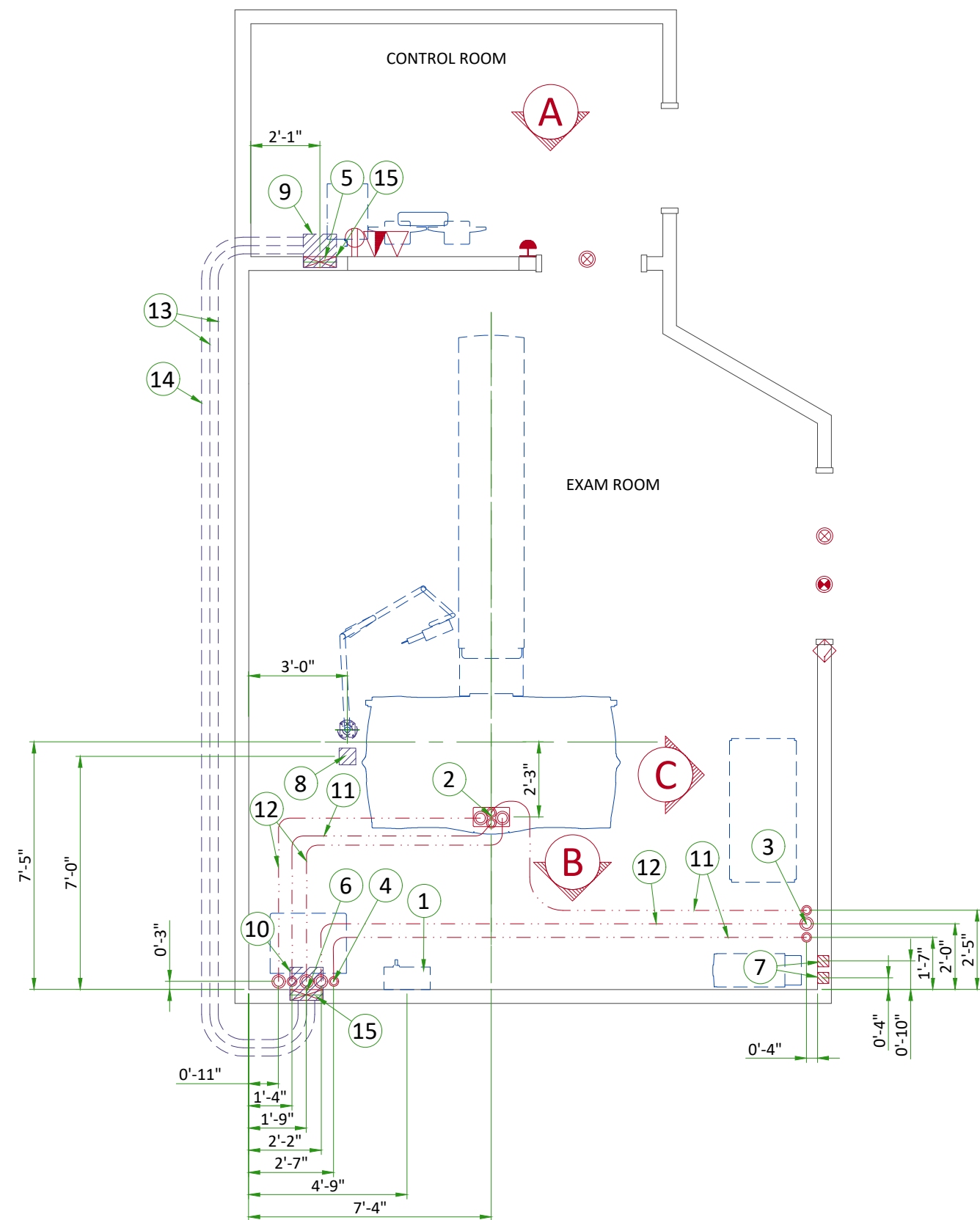


ITEM	DESCRIPTION
1	Main disconnect panel
2	Suitable bushings & lock nuts (Gantry)
3	Suitable bushings & lock nuts (System Cabinet)
4	Suitable bushings & lock nut (Power Distribution Unit)
5	Grommeted opening (Operator's console)
6	Provide suitable length and diameter flexible metal conduit with 2 90 degree connectors
7	4" x 4" x 4" [100 x 100 x 100] box (Partial UPS)
8	6" x 6" x 4" [150 x 150 x 100] box above ceiling (injector)
9	Box above ceiling - size per local code (control room)
10	Box above ceiling - size per local code (exam room)
11	2 1/2" [64] conduit below floor
12	3 1/2" [89] conduit below floor
13	2 1/2" [64] conduit above ceiling
14	3 1/2" [89] conduit above ceiling
15	10" x 3 1/2" [250 x 100] flush wall duct with minimum 2 dividers

ITEM	QTY	Outlet Legend for GE Equipment
△		Dedicated telephone line(s)
▲		Network outlet
⊕		Duplex hospital grade, dedicated wall outlet 120-v, single phase power
⏻		System emergency off (SEO), (recommended height 1.2m [48"] above floor)
⊗		X-Ray room warning light control panel
⊙		X-Ray ON lamp (L1) - 24V
◇		Door interlock switch (needed only if required by state/local codes)


Additional Conduit Runs (Contractor Supplied and Installed)					
From (Bubble # / Item)	To (Bubble # / Item)	Qty	Size		
			In.	mm	
3 Phase Power	1 Main Disconnect Panel	1	As req'd	As req'd	
	Emergency Off	1	1/2	13	
1 Main Disconnect Panel	10 Power Distribution Unit	1	As req'd	As req'd	
10 Power Distribution Unit	Door Switch	1	1/2	13	
	Warning Light Control	1	1/2	13	
		1	1/2	13	
1 Phase Power		1	1/2	13	
1 Main Disconnect Panel	7 Partial UPS	1	1 1/4	30	
10 Power Distribution Unit		1	2	50	
8 Injector	9 Injector Control	1	2 1/2	64	

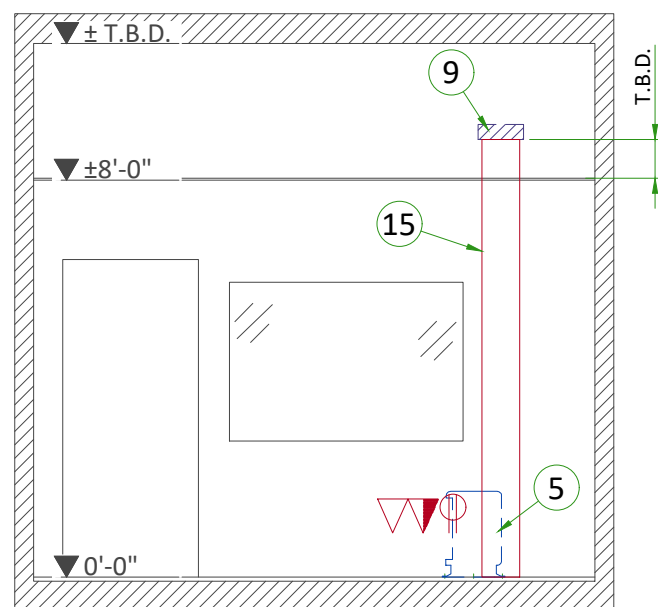

A

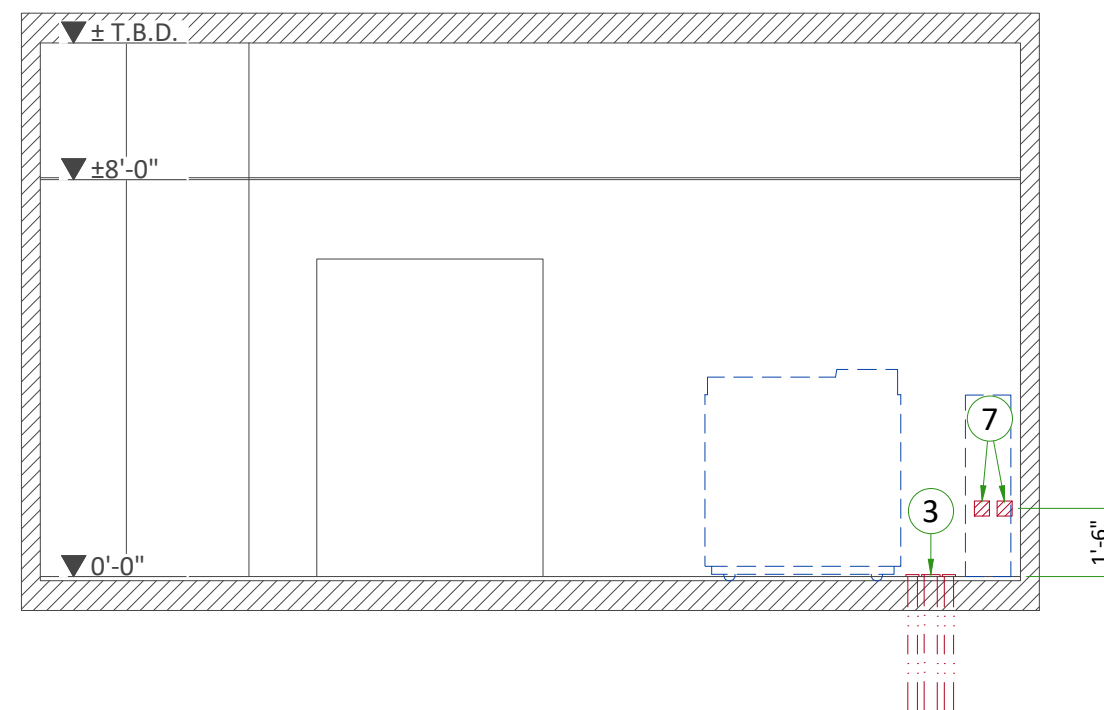

C

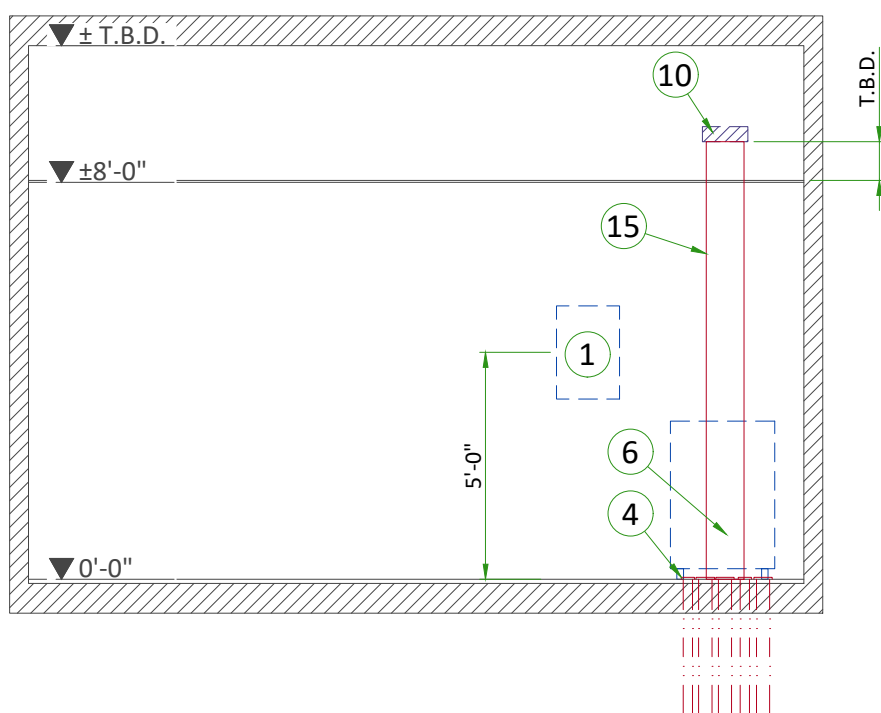

B

POWER REQUIREMENTS

POWER SUPPLY	3 PHASES+G (4-Wire system, neutral not used) 380/400/420/440/460/480 V ± 10%
FREQUENCIES	50/60 Hz ± 3 Hz
MAXIMUM POWER DEMAND	150 kVA
AVERAGE (CONTINUOUS) POWER DEMAND	11 kVA
POWER FACTOR	0.85

- Power supply should come into a main disconnect panel (MDP) containing the protective units and controls.
- The section of the supply cable should be calculated in accordance with its length and the maximum permissible voltage drops.
- There must be discrimination between supply cable protective device at the beginning of the installation (main low-voltage transformer side) and the protective devices in the MDP.
- Potential future upgrade may require a Maximum Power Demand of 200 kVA to be compatible.

SUPPLY CHARACTERISTICS

- Power input must be separate from any others which may generate transients (elevators, air conditioning, radiology rooms equipped with high speed film changers...).
- All equipment (lighting, power outlets, etc...) installed with GE system components must be powered separately.
- Phase imbalance 2% maximum.
- Transients must be less than 1500V peak. (on a 400V line)
- Maximum source regulation allowable is 6%. The combination of daily voltage variation (no load) and source regulation under full load shall not exceed + 10/-13% of nominal.

GROUND SYSTEM

- System of equipotential grounding.
- Equipotential: The equipotential link will be by means of an equipotential bar. This equipotential bar should be connected to the protective earth conductors in the ducts of the non GE cableways and to additional equipotential connections linking up all the conducting units in the rooms where GE system units are located.

CABLES

- Power and cable installation must comply with the distribution diagram.
- All cables must be isolated and flexible, cable color codes must comply with standards for electrical installation.
- The cables from signaling and remote control (Y, SEO, L...) will go to MDP with a pigtail length of 1.5m, and will be connected during installation. Each conductor will be identified and isolated (screw connector).


CABLEWAYS

The general rules for laying cableways should meet the conditions laid down in current standards and regulations, with regard to:

- Protecting cables against water (cableways should be waterproof).
- Protecting cables against abnormal temperatures (proximity to heating pipes or ducts).
- Protecting cables against temperature shocks.
- Replacing cables (cableways should be large enough for cables to be replaced).
- Metal cableways should be grounded.

FEEDER TABLE							
Sub-Feeder length (MDP/PDB to PDU)	MIN. SUB-FEEDER WIRE SIZE, AWG OR MCM (sq. mm)/VAC						
	380 VAC	400 VAC	420 VAC	440 VAC	460 VAC	480 VAC	
15m (50ft)	3/0 (95)	3/0 (95)	3/0 (95)	2/0 (70)	2/0 (70)	2/0 (70)	
GENERAL NOTES							
In all cases qualified personnel must verify that the feeder (at the point of take-off) and the run to the CT system meet all the requirements stated in the PIM							
For a single unit installation, the minimum transformer size is 225KVa, with 2.4% rated regulation at unity power factor. Resultant maximum allowable feeder regulation is 3.6%. Note: Not applicable to 200kVA systems.							
Grounding conductor will be a 1/0 minimum. this ground will run from the equipment back to the power source/main grounding point and always travel in the same conduit with the feeders							

POWER DISTRIBUTION

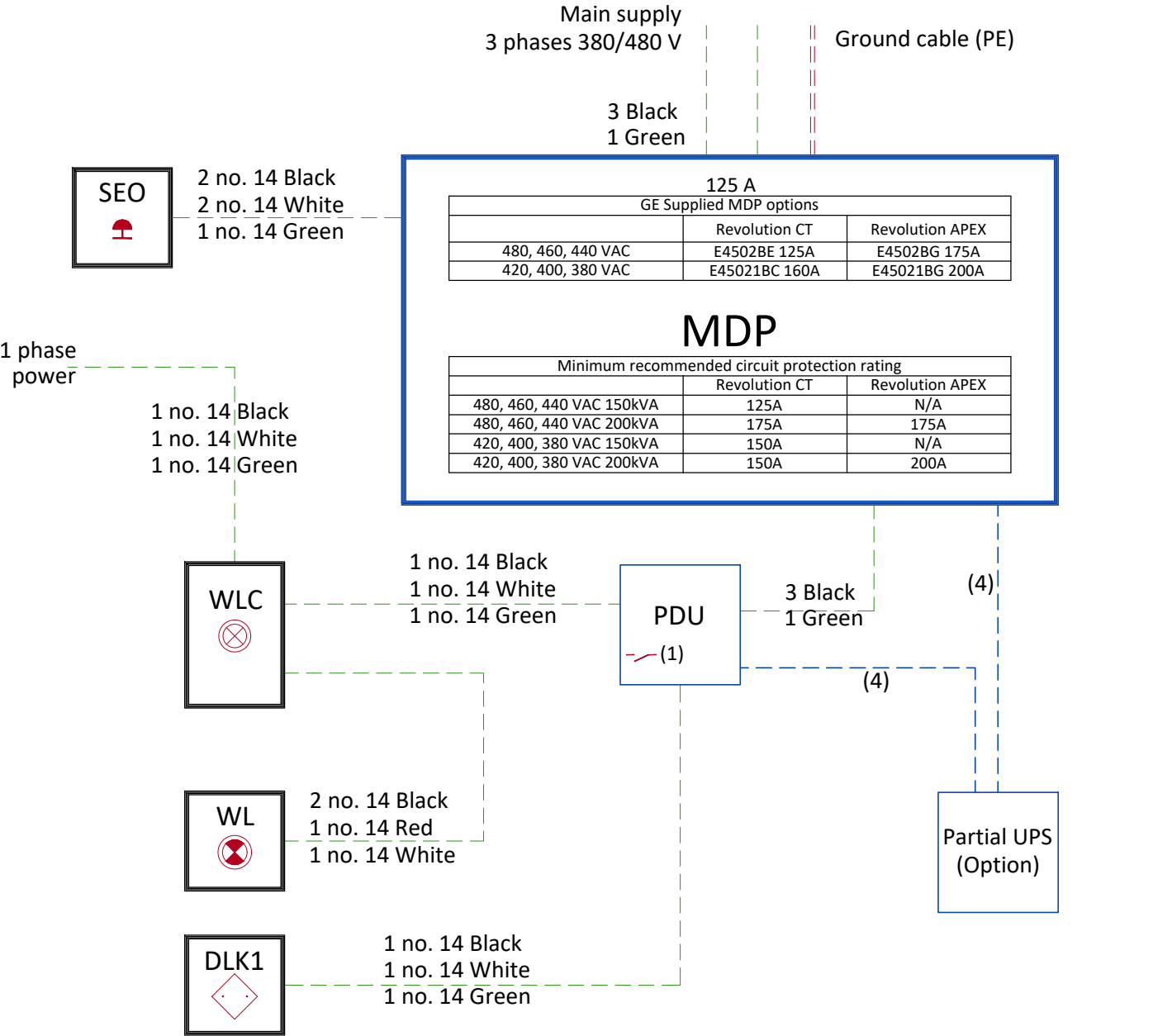


- MDP Main Disconnect Panel
PDU Power distribution unit
SEO Emergency OFF button (Control Room), located 1.50m (4.9') above floor
WLC Warning Light Control
WL Warning Light
DLK1 Door Interlock Switch (needed only if required by state/local codes)

- Notes :
- (1) Two dry contacts: "System ON" and "X-Ray ON", both released by PDU.
Max. voltage = 30 V
- (2) If length < 10 m (32.8')
Cable with 2m (6.6') extra length on the floor behind the back of PDU
- (3) Cable with 2m (6.6') extra length on the floor behind the back of PDU
- (4) Cable delivered with partial UPS installed by GE (Option)

---	Cable SUPPLIED BY CUSTOMER
---	Cable SUPPLIED BY GE
---	Equipment SUPPLIED BY CUSTOMER
---	Equipment SUPPLIED BY GE


TYPICAL CABLE MANAGEMENT

CONDUIT IN THE FLOOR

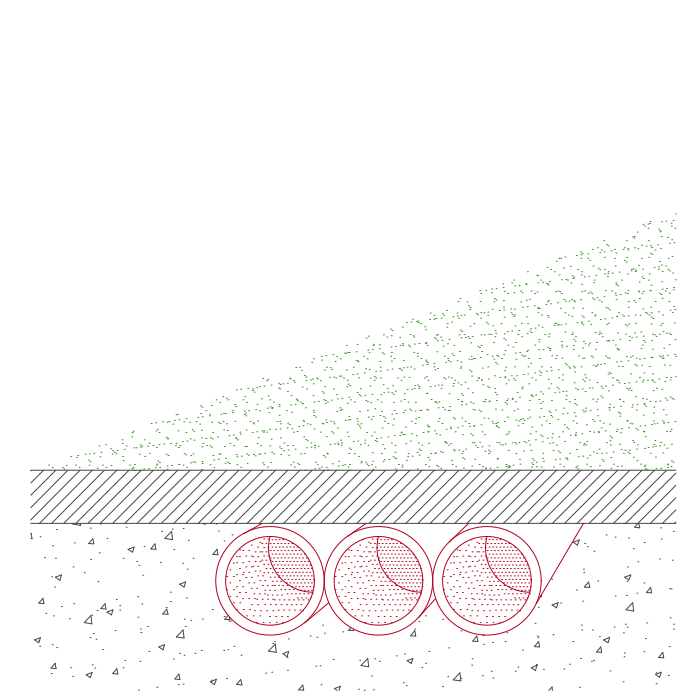

NOT TO SCALE

VERTICAL DUCT ON WALL

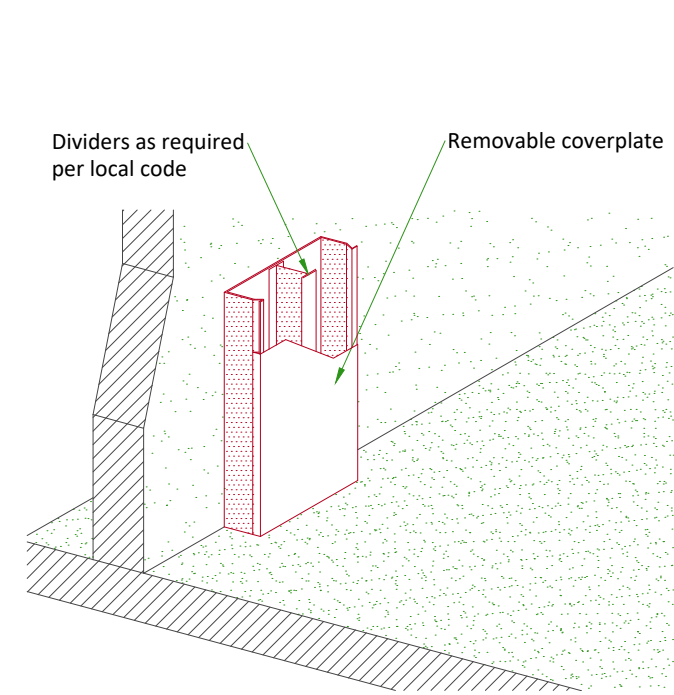

INTERCONNECTIONS

