



DISCLAIMER

GENERAL SPECIFICATIONS

- GE is not responsible for the installation of developers and associated equipment, lighting, cassette trays and protective screens or derivatives not mentioned in the order.
- The final study contains recommendations for the location of GE equipment and associated devices, electrical wiring and room arrangements. When preparing the study, every effort has been made to consider every aspect of the actual equipment expected to be installed.
- The layout of the equipment offered by GE, the dimensions given for the premises, the details provided for the pre-installation work and electrical power supply are given according to the information noted during on-site study and the wishes expressed by the customer.
- The room dimensions used to create the equipment layout may originate from a previous layout and may not be accurate as they may not have been verified on site. GE cannot take any responsibility for errors due to lack of information.
- Dimensions apply to finished surfaces of the room.
- Actual configuration may differ from options presented in some typical views or tables.
- If this set of final drawings has been approved by the customer, any subsequent modification of the site must be subject to further investigation by GE about the feasibility of installing the equipment. Any reservations must be noted.
- The equipment layout indicates the placement and interconnection of the indicated equipment components. There may be local requirements that could impact the placement of these components. It remains the customer's responsibility to ensure that the site and final equipment placement complies with all applicable local requirements.
- All work required to install GE equipment must be carried out in compliance with the building regulations and the safety standards of legal force in the country concerned.
- These drawings are not to be used for actual construction purposes. The company cannot take responsibility for any damage resulting therefrom.

CUSTOMER RESPONSIBILITIES

- It is the responsibility of the customer to prepare the site in accordance with the specifications stated in the final study. A detailed site readiness checklist is provided by GE. It is the responsibility of the customer to ensure all requirements are fulfilled and that the site conforms to all specifications defined in the checklist and final study. The GE Project Manager of Installation (PMI) will work in cooperation with the customer to follow up and ensure that actions in the checklist are complete, and if necessary, will aid in the rescheduling of the delivery and installation date.
- Prior to installation, a structural engineer of record must ensure that the floor and ceiling is designed in such a way that the loads of the installed system can be securely borne and transferred. The layout of additional structural elements, dimensioning and the selection of appropriate installation methods are the sole responsibility of the structural engineer. Execution of load bearing structures supporting equipment on the ceiling, floor or walls are the customer's responsibility.

|   |      |           |
|---|------|-----------|
| THE UNDERSIGNED, HEREBY CERTIFIES THAT I HAVE READ AND APPROVED THE PLANS IN THIS DOCUMENT. |      |           |
| DATE  | NAME | SIGNATURE |
|   |      |           |

GLOBAL SITE READINESS CHECKLIST (DI)

DOC1809666 Rev. 7

|  |
|--|
| Site Ready Checks at Installation  |
| EHS Site Requirements  |
| Overall access route to the scan room free from obstruction / high hazards.  |
| Enough space to store tools, equipment, parts, install waste and the general area free from obstruction and trip hazards.  |
| Enough necessary facilities for the GE employees available.  |
| No 3rd parties working in the area that may affect the safety of the installation activity.  |
| Area free from any chemical, gas, dust, welding fume exposure and has painting been completed and dry.   |
| All emergency routes identified, signed and clear from obstruction.  |
| Accessible single source lockable panel that LOTO can be applied to for GE equipment installation (MDP and/or PDU).  |
| There are no other conditions or hazards that you have observed or have been made aware of by the customer or contractors on site.   |
| Site Ready Checks for MR Magnet delivery if separate from system   |
| Specific for MR Magnet Delivery  |
| All permits, plans and permissions received for rigging and/or delivery.   |
| Adequate delivery route from truck to final place of installation has been reviewed with all stakeholders, all communications/notifications have occurred, arrangements have been made for special handling (rigging, elevator, fork lift, etc.)                       |
| All floors along delivery route will support weight of the equipment, temporary reinforcements arranged if needed.   |
| Chilled water supply for Water Cooled Compressor or Air Cooled Compressor is ready and meets GE specifications.  |
| Water drain available in the equipment room, if applicable.  |
| Power for MR compressor & Chiller is available.  |
| Power is available for magnet monitoring.  |
| Connectivity is available for magnet monitoring.   |
| Ensure cryogen venting system is complete to outside the building & available for magnet connection.   |
| Exhaust fan system is installed per GE requirements and VERIFIED operational by suction at intake.   |
| Required for Mechanical Install start  |
| Room dimensions, including ceiling height, for all Exam, Equipment/Technical & Control rooms meets GE specifications.  |
| Ceiling support structure, if indicated on the GE drawing, is in the correct location and at the correct height according to the Original Equipment Manufacturer specifications.   |
| Levelness and spacing has been measured, and is ready for the installation of any GE supplied components.  |
| Finished ceiling is installed. If applicable ceiling tiles installed per PMI discretion.   |
| Floor levelness/flatness is measured and within tolerance, and there are no visible defects per GEHC specifications.   |
| Entry door threshold meets PIM requirement   |
| Floor Strength and thickness have been discussed with customer/contractor and they have confirmed GE requirements are met.   |
| Rooms that will contain equipment, including staging areas if applicable, are construction debris free. Precautions must be taken to prevent debris from entering rooms containing equipment.  |
| Cable ways (floor/wall/ceiling/Access Flooring) are available for installation of GE cables are of correct length and diameter.  |
| Overhead cable trays must be install exactly as shown in GE supplied installation draings to assure proper Gradient Cable length.  |
| Cable ways routes per GE Final drawings and cable access openings areas installed at a time determined by GEHC PM. Surface floor duct can be installed at time of system installation.   |
| Adequate room illumination installed and working.  |
| Customer supplied countertops where GE equipment will be installed are in place.   |
| RF Shield installed with possible exception of magnet entrance. RF Shield Effectivity and Ground Isolation Test needed. If GE responsible for supplying RF shield, the RF shield Effectivity and Ground Isolation Test data is a Mandatory attachment into MyProjects. |
| Required for Calibration start   |
| HVAC systems Installed, and the site meets minimum environmental operational system requirements.  |
| System power & grounding (PDB/MDP) is available as per GE specifications.  |
| System power & grounding (PDB/MDP) is installed at point of final connection and ready to use. Lock Out Tag Out is available.  |
| PMI to confirm all feeder wires and breaker are size appropriately. EPO installed if needed.   |
| PMI to confirm with electrician all power and signal cables are well terminated ensuring there are no loose connections.   |
| Network outlets installed.   |
| Computer network available and working.  |

Note: The details shown here are only an extract from DOC1809666. For the complete document please contact your PMI.

CUSTOMER SITE READINESS REQUIREMENTS

- Any deviation from these drawings must be communicated in writing to and reviewed by your local GE healthcare installation project manager prior to making changes.
- Make arrangements for any rigging, special handling, or facility modifications that must be made to deliver the equipment to the installation site. If desired, your local GE healthcare installation project manager can supply a reference list of rigging contractors.
- New construction requires the following;
  - Secure area for equipment,
  - Power for drills and other test equipment,
  - Capability for image analysis,
  - Restrooms.
- Provide for refuse removal and disposal (e.g. crates, cartons, packing)
- For CT, MR, PET/CT, and SPECT systems it is required to minimize vibrations within the scan room. It is the customer's responsibility to contract a vibration consultant/engineer to implement site design modifications to meet the GE vibration specification. Refer to the system preinstallation manual for vibration specifications.

MRI SITE PLANNING REMINDERS

Please refer to pre-installation checklist in pre-installation manual listed on the cover sheet for items critical to image quality.

- The layout should be arranged so that the 5g line is contained to the magnet room. If not possible, a barrier is recommended to prevent entry to the 5g field area.
- The spaces around, above, and below the magnet must be reviewed for effects of the 5g, 3g, 1g, and .5g fields. Refer to the proximity limit chart in the MR pre-installation manual referenced on the cover sheet.
- For moving metal, the restriction lines typically extend outside of the MR space. Please confirm there are no moving metal concerns within these areas.
- For vibration, analysis to be completed as required per pre-installation manual.
- For EMI, review the site for the location of the main electrical feeders, AC devices, or distribution systems. An EMI study is recommended if large AC systems are nearby.
- Details of the floor below the magnet must be reviewed. The structural engineer must verify that the quantity of steel in the volume 10ft [3.1m] x 10ft [3.1m] x 1ft [.3m] deep (below the magnet) does not exceed the allowable steel content as given in the MR pre-installation manual referenced on the cover sheet.
- Remove, cover, or fill-in abandoned ducts or troughs from the Equipment and Magnet rooms. Access/computer room flooring in the Equipment room can either be removed or assessed and reinforced to support heavier cabinets.

Responsibility for the coordination, design, engineering, and site preparation resides with the customer and their project architects and contractors. GE does not, by providing reviews and furnishing comments and assistance, accept any responsibility beyond its obligations as defined in the MR system, sale/purchase agreement.

IMAGE QUALITY CONSIDERATIONS

Broadband RF noise is a single transient or continuous series of transient disturbances caused by an electrical discharge. Low humidity environmental conditions will have higher probability of electrical discharge. The electrical discharge can occur due to electrical arcing (micro arcing) or merely static discharge. Some potential sources capable of producing electrical discharge include:

- Loose hardware/fasteners vibration or movement (electrical continuity must always be maintained)
- Flooring material including raised access flooring (panels & support hardware) and carpeting
- Electrical fixtures (i.e. Lighting fixtures, track lighting, emergency lighting, battery chargers, outlets)
- Ducting for HVAC and cable routing
- RF shield seals (walls, doors, windows etc.)


For additional information regarding image quality, refer to the pre-installation manual listed on the cover sheet.

MAGNETIC INTERFERENCE SPECIFICATIONS

- The customer must establish protocols to prevent persons with cardiac pacemakers, neurostimulators, and biostimulation devices from entering magnetic fields of greater than 5 gauss (exclusion zone).
- Main power transformers must remain outside the 3 gauss field.
- Potential exists under fault conditions that the 5 gauss line may expand radially to 14.8 ft. [4.5 m] and axially to 19.7 ft. [6.0 m] for 8 seconds or less. It should be noted that normal rampdowns or magnet rundown unit initiated quenches will not cause the magnetic field to expand.
- It is recommended every site consider the event of a quench and plan accordingly (such as placing 5 gauss warning signs at expanded locations).
- The ferrous metal objects listed below must not move into or inside of the moving metal sensitivity line during scans.


| TYPICAL MOVING MAGNETIC MASS  | DISTANCE RADIALY |        | DISTANCE AXIALLY |        |
|---|------------------|--------|------------------|--------|
| Carts, Gurneys 100-400 lbs [45-182 kg]  | 3 Gauss line     |        | 3 Gauss line     |        |
| Forklifts, small elevator, cars, minivans vans, pickup trucks, ambulances (objects greater than 400 lbs [182 kg]) | 20.0 FT          | 6.05 M | 25.0 FT          | 7.65 M |
| Buses and trucks (dump, tractor trailer, utility, fire trucks)  | 23.2 FT          | 7.10 M | 29.2 FT          | 8.90 M |
| For objects >400 lbs [182 kg], Z-axis EMI disturbances must not exceed 5.6 mG peak-to-peak.                       |                  |        |                  |        |



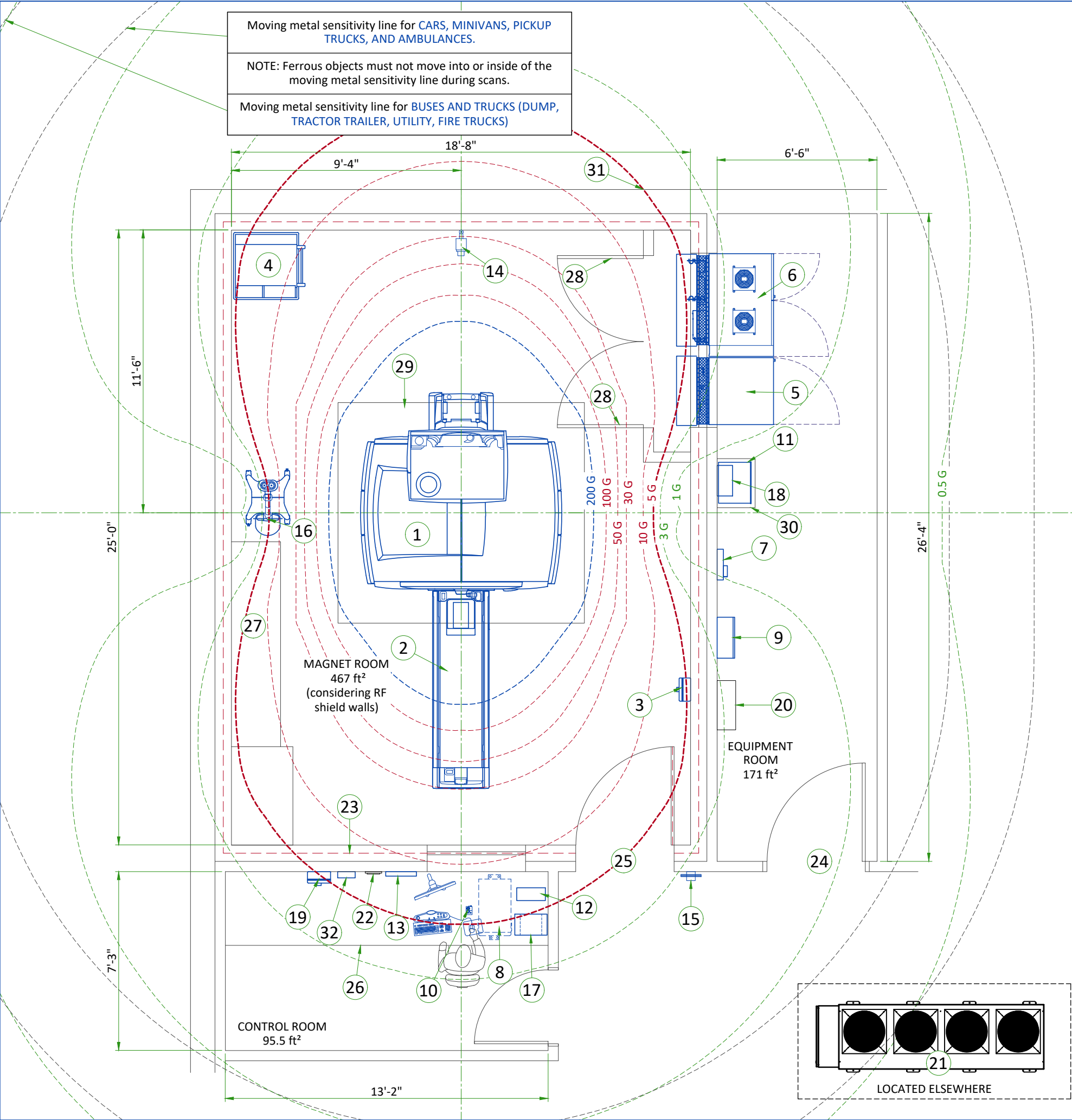


| LEGEND   |  |  |                       |  |                     |             |
|--|--|--|-----------------------|--|---------------------|-------------|
| A  | GE Supplied                                |  | D                     | Available from GE                        |                     |             |
| B  | GE Supplied/contractor installed           |  | E                     | Equipment existing in room               |                     |             |
| C  | Customer/contractor supplied and installed |  | *                     | Item to be reinstalled from another site |                     |             |
| 200 Gauss  |  |  | 5 Gauss               |  |                     |             |
| 100, 50, 30, 10 Gauss  |  |  | 3, 1, 0.5 Gauss       |  |                     |             |
| BY   | ITEM                                       | DESCRIPTION  | MAX HEAT OUTPUT (btu) | WEIGHT (lbs)                             | MAX HEAT OUTPUT (W) | WEIGHT (kg) |
| A  | 1  | 3T Magnet  | 8189                  | 16500                                    | 2400                | 7500        |
| A  | 2  | Patient table  | -                     | 572                                      | -                   | 259.50      |
| A  | 3  | Magnet rundown unit  | -                     | 7  | -                   | 3.2         |
| A  | 4  | Phantom set storage cabinet  | -                     | 350                                      | -                   | 136         |
| A  | 5  | Integrated Cooling Cabinet   | 3412                  | 1034                                     | 1000                | 469         |
| A  | 6  | Integrated System Cabinet  | 24232                 | 2758                                     | 7100                | 1251        |
| A  | 7  | Magnet monitor   | 819                   | 10                                       | 240                 | 4.5         |
| A  | 8  | Operator console computer  | 4947                  | 122.80                                   | 1450                | 55.70       |
| B  | 9  | Main Disconnect Panel  | 901                   | 96                                       | 264                 | 43.5        |
| A  | 10   | Pneumatic patient alert  | -                     | 0.5                                      | -                   | 0.2         |
| A  | 11   | MR Elastography  | 480                   | 53.4                                     | 0.14                | 24.22       |
| A  | 12   | 700 va partial UPS   | -                     | 26.5                                     | -                   | 11.5        |
| A  | 13   | CCTV Monitor   | -                     | 7.7                                      | -                   | 3.50        |
| A  | 14   | CCTV Camera  | -                     | 0.3                                      | -                   | 0.14        |
| D  | 15   | Metal Detector (hand held)   | -                     | -  | -                   | -           |
| D  | 16   | Injector on pedestal   | -                     | 94                                       | -                   | 43          |
| D  | 17   | Injector control   | 675                   | 17                                       | 198                 | 8           |
| D  | 18   | Injector power supply  | 660                   | 6  | 193                 | 3           |
| D  | 19   | Remote magnet rundown unit   | -                     | 7  | -                   | 3.20        |
| B/D  | 20   | Water filter   | -                     | -  | -                   | -           |
| B/D  | 21   | Dimplex Chiller  | 123000                | 2550                                     | 36000               | 1157        |
| B/D  | 22   | Remote graphic display   | -                     | -  | -                   | -           |
| C  | 23   | Define RF shield's inset according to provisions made by the RF Shield vendor                        |                       |  |                     |             |
| C  | 24   | Minimum opening for equipment delivery is 40 in. w x 82 in. h, contingent on a 72 in. corridor width |                       |  |                     |             |
| C  | 25   | Minimum opening for equipment delivery is 43 in. w x 82 in. h, contingent on a 96 in. corridor width |                       |  |                     |             |
| C  | 26   | Counter top for equipment- provide grommets openings as required to route cables                     |                       |  |                     |             |
| C  | 27   | Base cabinet for storage of: surface coils, patient positioning pads, phantoms, etc.                 |                       |  |                     |             |
| C  | 28   | Louvered doors - refer to preinstall for requirements  |                       |  |                     |             |
| C  | 29   | Magnet access 9'-0"x10'-0"   |                       |  |                     |             |
| C  | 30   | Shelf  |                       |  |                     |             |
| C  | 31   | Warning! 5 Gauss line outside the Magnet room limits   |                       |  |                     |             |
|  |  |  |                       |  |                     |             |
| Exam room height   |  |  |                       |  |                     |             |
| Finished floor to slab height  |  |  |                       |  | TBD                 |             |
| Recommended finished ceiling height  |  |  |                       |  | 8'-9"               |             |
| RF SHIELD - 100 dB ATTENUATION   |  |  |                       |  |                     |             |
|  |  |  |                       |  |                     |             |
| For Accessory Sales: (866) 281-7545 Options 1, 2, 1, 2 or mail to: gehcaccessorysales@ge.com |  |  |                       |  |                     |             |

# SIDE VIEW WITH MAGNETIC FIELD

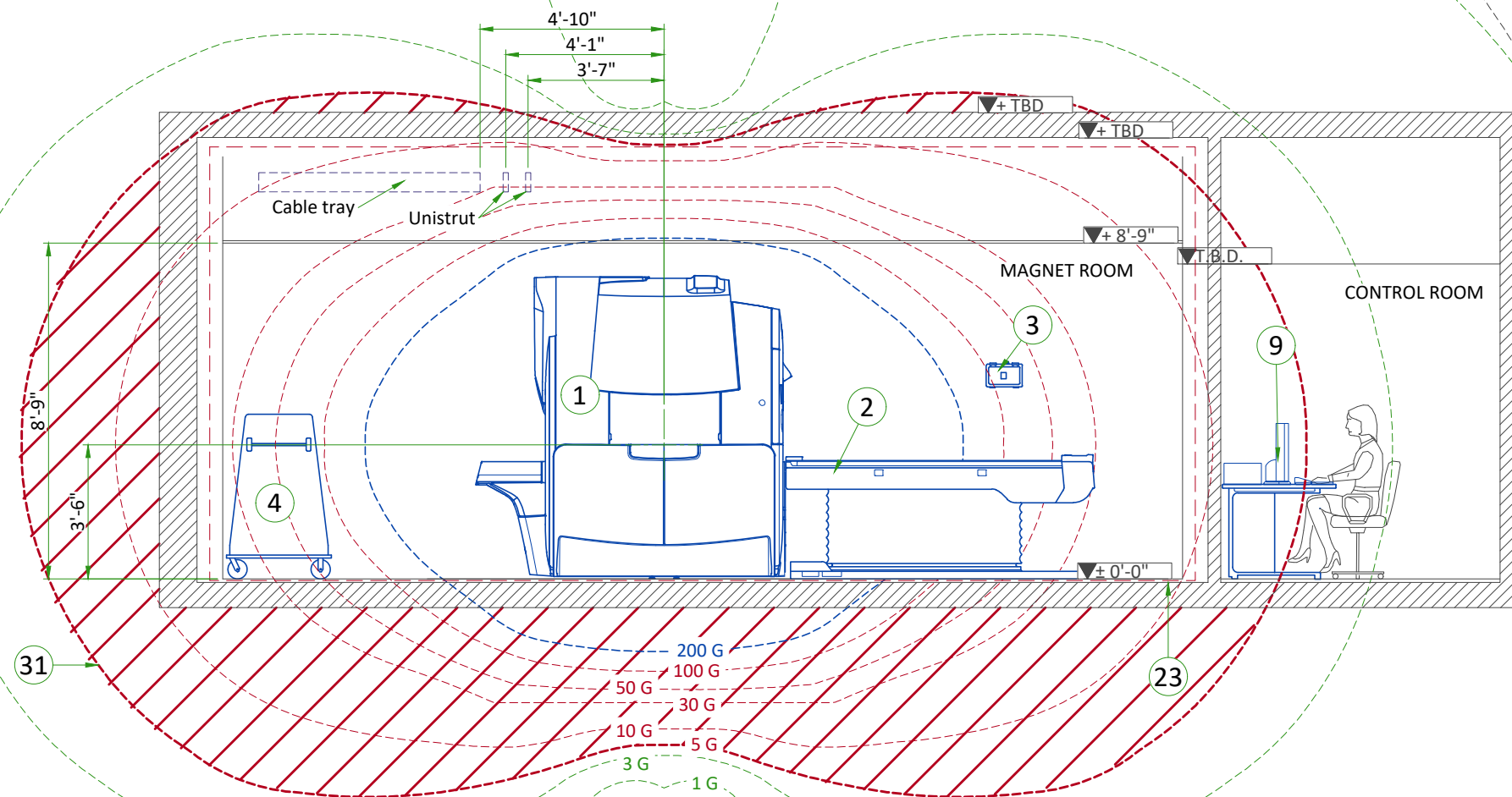

# FRONT VIEW WITH MAGNETIC FIELD

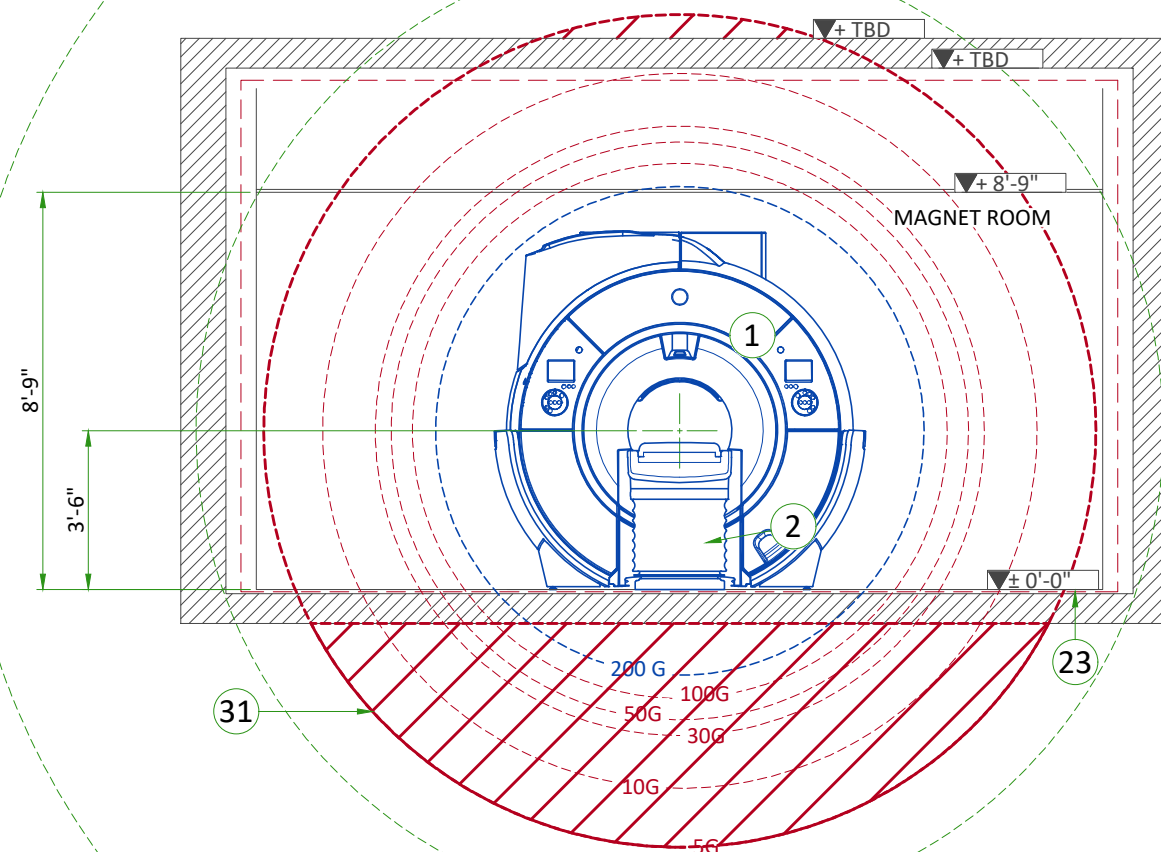


ACOUSTICS SPECIFICATIONS

Acoustic and vibroacoustic information is provided for site planning and architectural design activities. It is the customer's responsibility to hire a qualified acoustic engineer for solutions to further attenuate this transmitted noise and vibration, if required. The actual room noise level may vary based on room design, optional equipment, and usage:

Control Room: 62dBA  
Equipment Room: 80dBA  
Magnet Room: 127dBA\*  
(maximum sound pressure level at magnet bore isocenter)

\* Frequency: 20 Hz to 20kHz


| FREQUENCY (Hz) | AMPLITUDE (m/s²) |
|----------------|------------------|
| 2              | 0.05             |
| 10             | 0.05             |
| 20             | 0.35             |
| 30             | 0.35             |
| 35             | 0.2              |
| 50             | 0.2              |

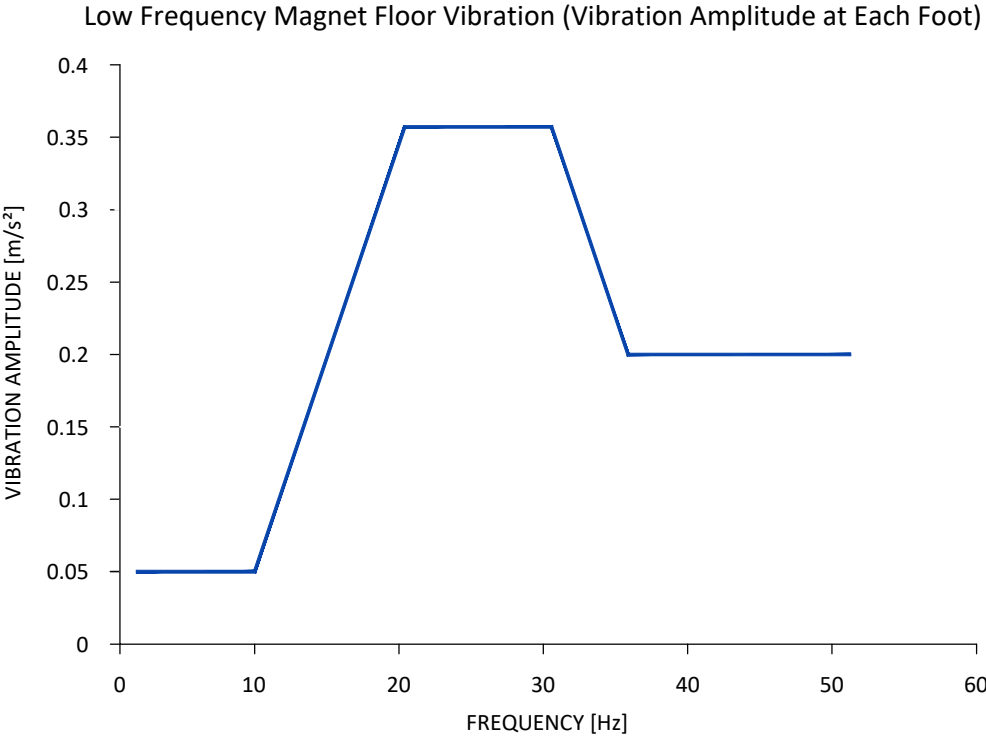

ISOGAUSS PLOTS

\* The isogauss contour plots depicted on this drawing represent magnetic fringe fields resulting from the normal operation of the magnet provided with the MR system. The actual magnetic field intensity at any point in the vicinity of the magnet when installed may vary from the contour plots due to factors such as the concentrating effects of nearby ferrous objects ambient magnetic fields, including the earth's magnetic field. Therefore, the contours shown are only approximations of actual field intensities found at a corresponding distance from the magnet's isocenter.

SOUND PRESSURE SPECTRAL DISTRIBUTION

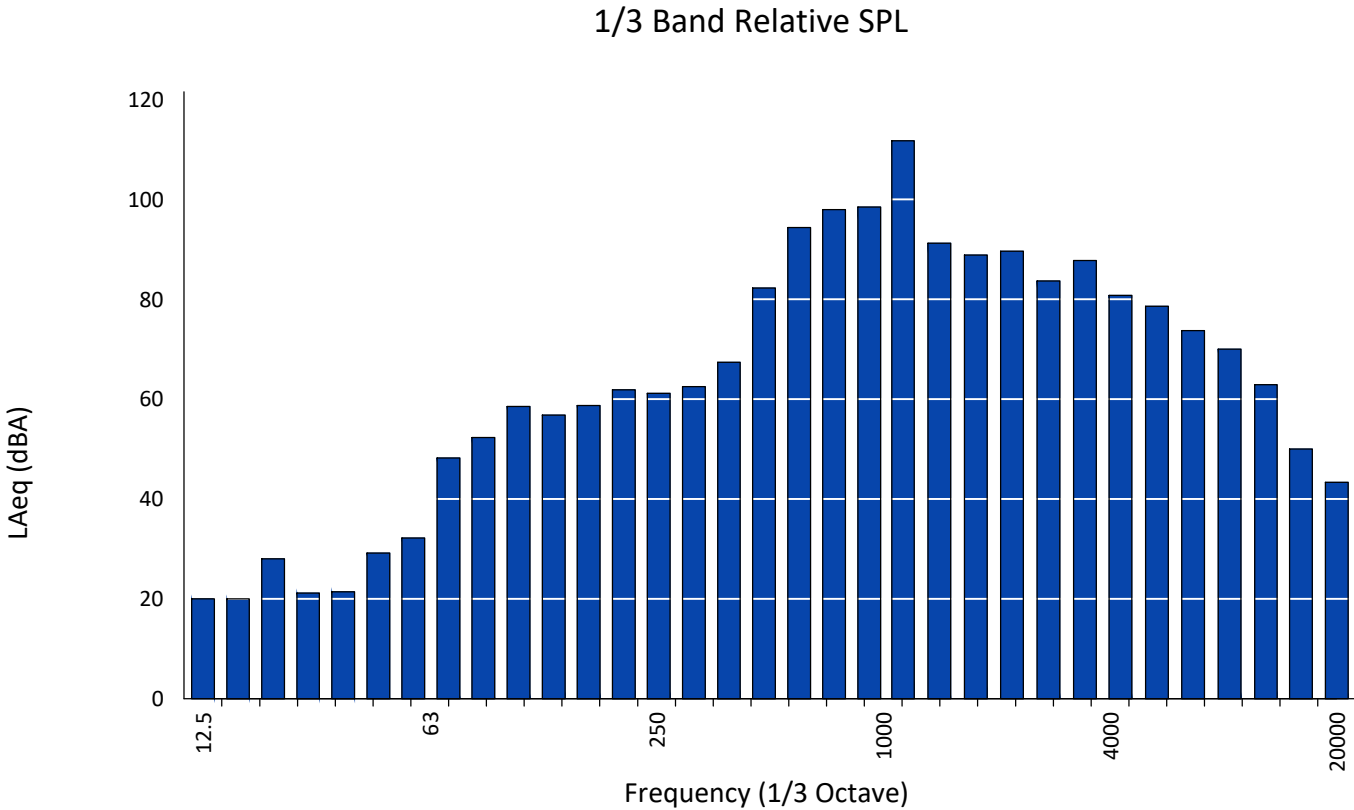

MAGNETIC PROXIMITY LIMITS



| Gauss (mT) Limit   | Equipment   |
|--------------------|---|
| 0.5 gauss (0.05mT) | Nuclear camera  |
| 1 gauss (0.1mT)    | Positron Emission Tomography scanner, Linear Accelerator, Cyclotrons, Accurate measuring scale, Image intensifiers, Bone Densitometers, Video display (tube), CT scanner, Ultrasound, Lithotripter, Electron microscope, Digital X-Ray  |
| 3 gauss (0.3mT)    | Power transformers, Main electrical distribution transformers   |
| 5 gauss (0.5mT)    | Cardiac pacemakers, Neurostimulators, Biostimulation devices  |
| 10 gauss (1mT)     | Magnetic computer media, Line printers, Film processor, X-ray tubes, Emergency generators, Commercial laundry equipment, Food preparation area, Water cooling equipment, HVAC equipment, Major mechanical equipment room, Credit cards, watches, and clocks, Air conditioning equipment, Fuel storage tanks, Motors greater than 5 horsepower |
| 50 gauss (5mT)     | Metal detector for screening, LCD panels, Telephones  |
| No Limit           | Digital Detectors   |

The customer must provide detail defining ferrous material below the magnet to the Project Manager so the GE Healthcare MR Siting and Shielding team can review for compliance.

| STEEL MASS LIMITS TO MAGNET ISOCENTER (3x3 m [10x10 ft] AREA UNDER MAGNET) |         |                                |       |                                     |       |
|--|---------|--------------------------------|-------|-------------------------------------|-------|
| Limits Of Steel Mass   |         | Distance From Magnet Isocenter |       | Distance Below Top Surface Of Floor |       |
| kg/m²  | lbs/ft² | mm                             | in    | mm                                  | in    |
| 0  | 0       | 0 - 1143                       | 0-45  | 0 - 76                              | 0-3   |
| 9.8  | 2       | 1143 - 1194                    | 45-47 | 76 - 127                            | 3-5   |
| 14.7   | 3       | 1194 - 1321                    | 47-52 | 127 - 254                           | 5-10  |
| 39.2   | 8       | 1321 - 1397                    | 52-55 | 254 - 330                           | 10-13 |
| 98.0   | 20      | 1397+                          | 55+   | 330+                                | 13+   |


The actual field strength can be affected by Magnetic shielding, Earth's magnetic field, other magnetic fields and stationary or moving metal. This information must be used to evaluate potential site interaction of GE Healthcare equipment with other non-GE Healthcare equipment. Magnetic shielding can be installed to prevent interaction between the magnet and nearby sensitive devices. The GE Healthcare Project Manager of Installation (PMI) can work with the customer to coordinate the magnetic shielding site evaluation. The customer is responsible for installation of all magnetic shielding.

## PENETRATION PANELS WALL OPENINGS

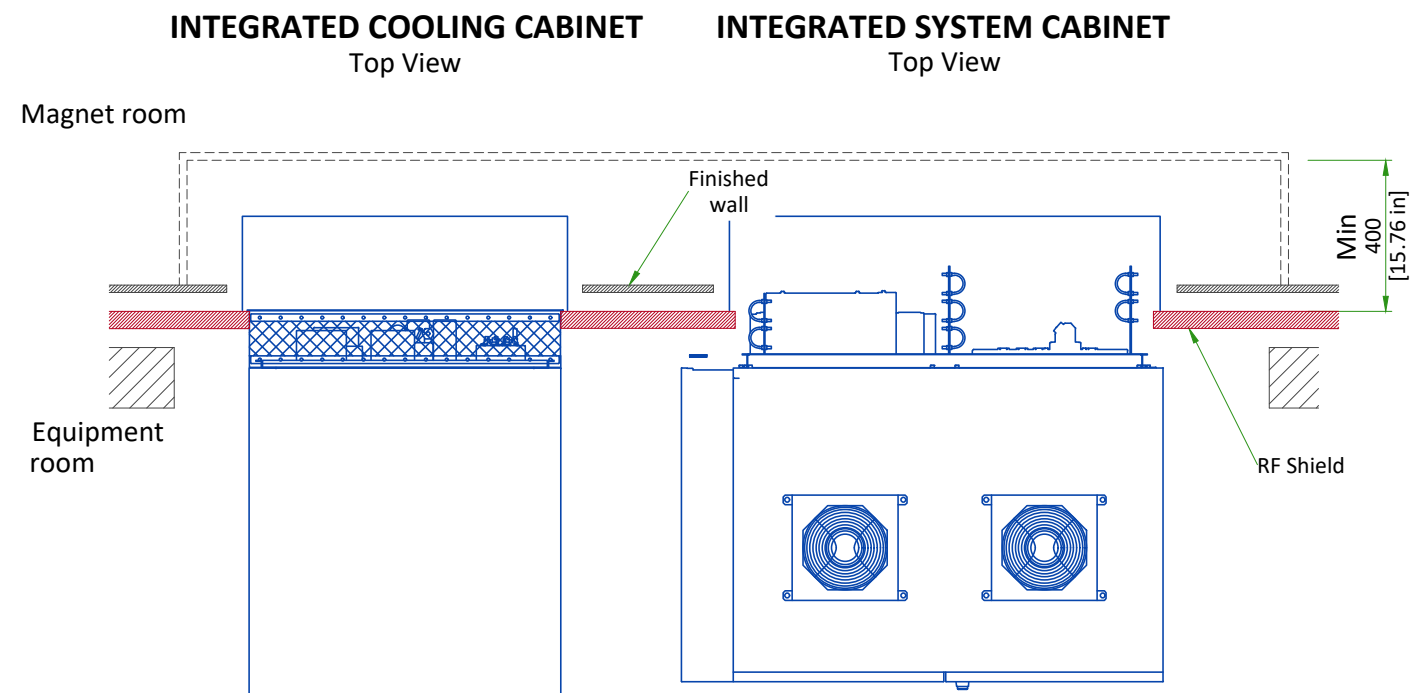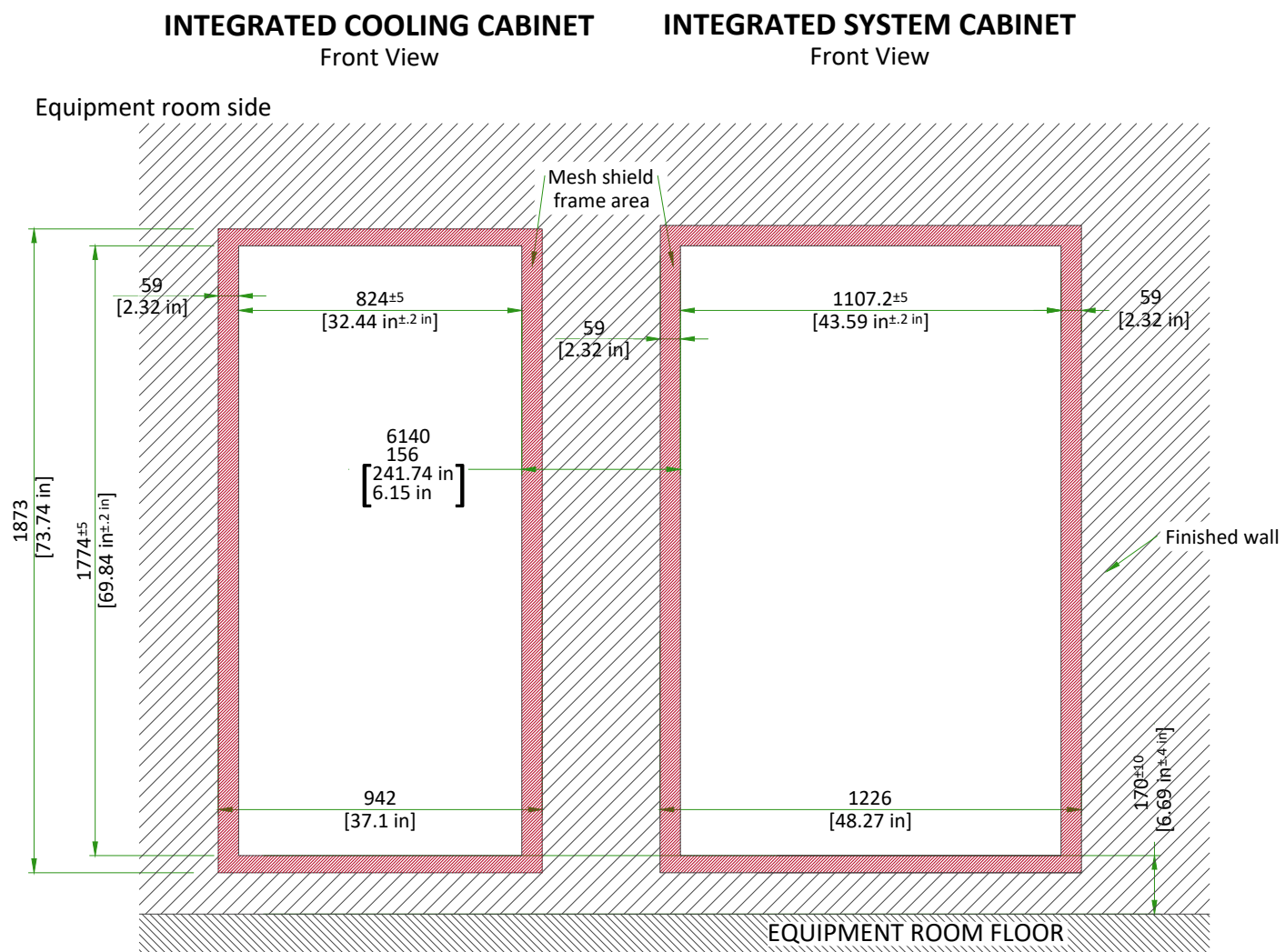

Integrated System Cabinet and Integrated Cooling Cabinet must be covered on both sides for safety.

## PENETRATION PANEL CLOSET

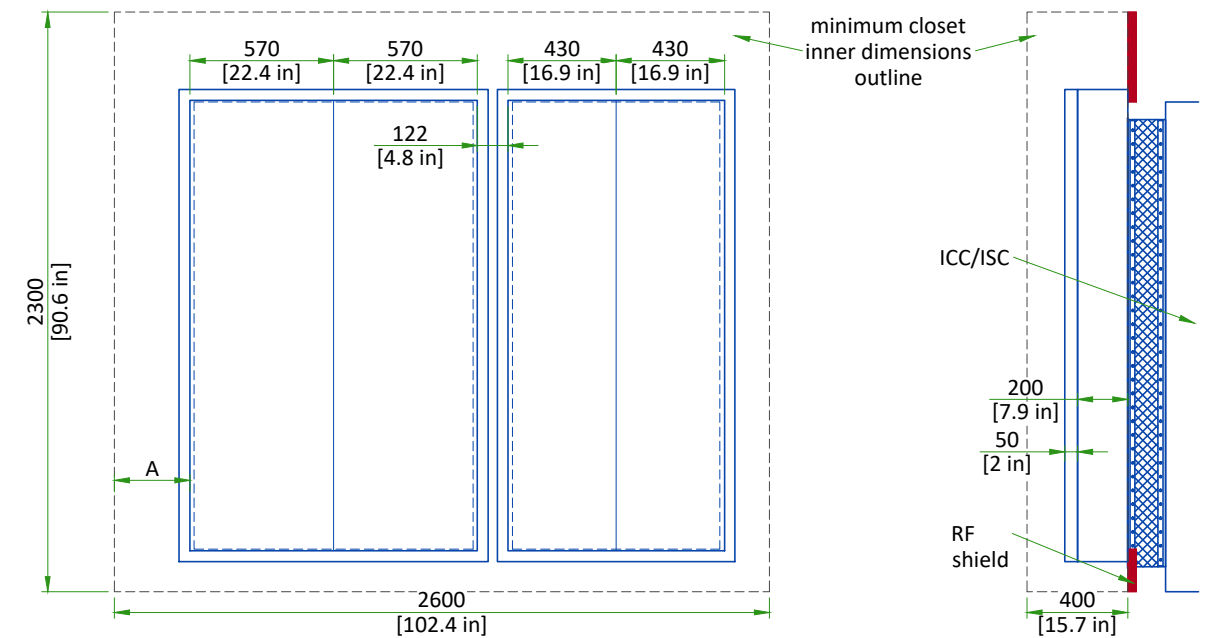


A: 300 mm [11.81in] side space for excess cable storage. Left side space and right side space can be swapped.  
NOTE: All RF mesh Assy Shipping brackets should be kept on frame until after installed and RF test is completed.

An enclosure (i.e. closet) must be provided to restrict access to the PEN panels and for storage of excess interconnections.

- The PEN closet must have a mechanical locking mechanism to restrict access to the PEN panels
- The PEN closet must maintain the minimum service area outside the 200 Gauss in the magnet room.
- PEN closet must allow free air exchange of **400CFM (680 m<sup>3</sup>/hour)** between the Magnet room and PEN closet for MR system blowers. Airflow may be achieved through door louvers or other openings in the PEN closet that meet all other PEN closet requirements

### MINIMUM MAGNET CEILING HEIGHT (TOP VIEW)


Shaded area within solid lines indicates floor to ceiling minimum height of 2500 mm [98.42in]. If ceiling is between 2500 mm [98.42in] and 2667 mm [105in], flexible main lead extension for low ceiling height is required for ramping the magnet. Contact the PMI for preparation.

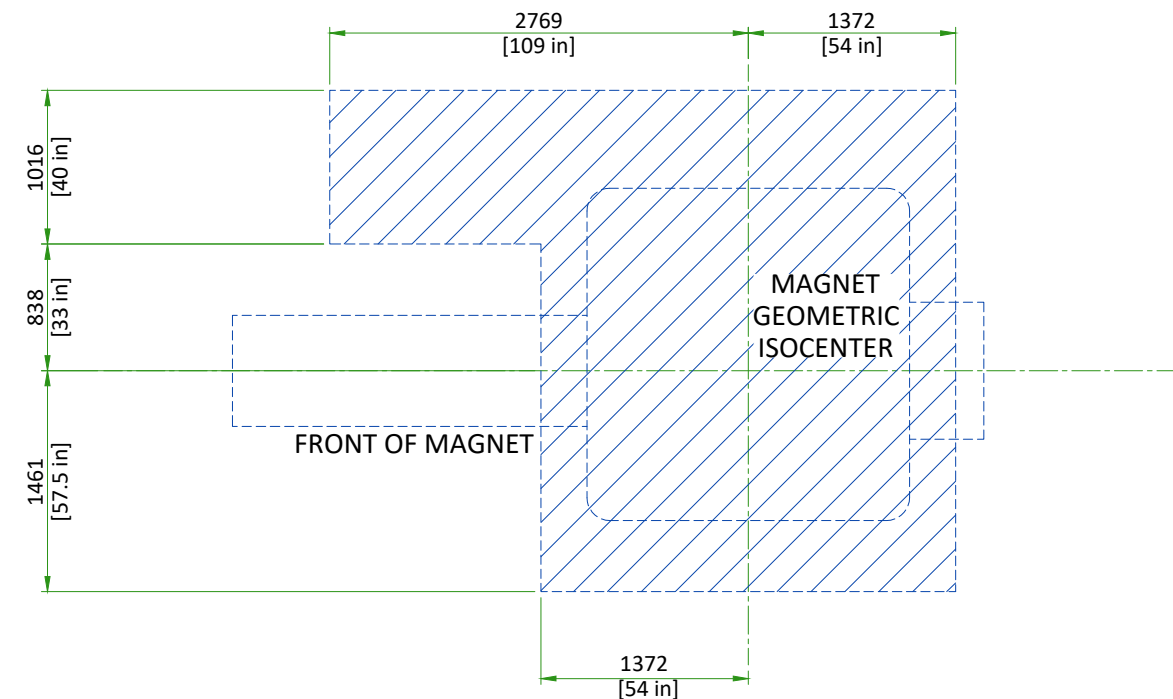

SCALE 1:50




MAGNET ENCLOSURE

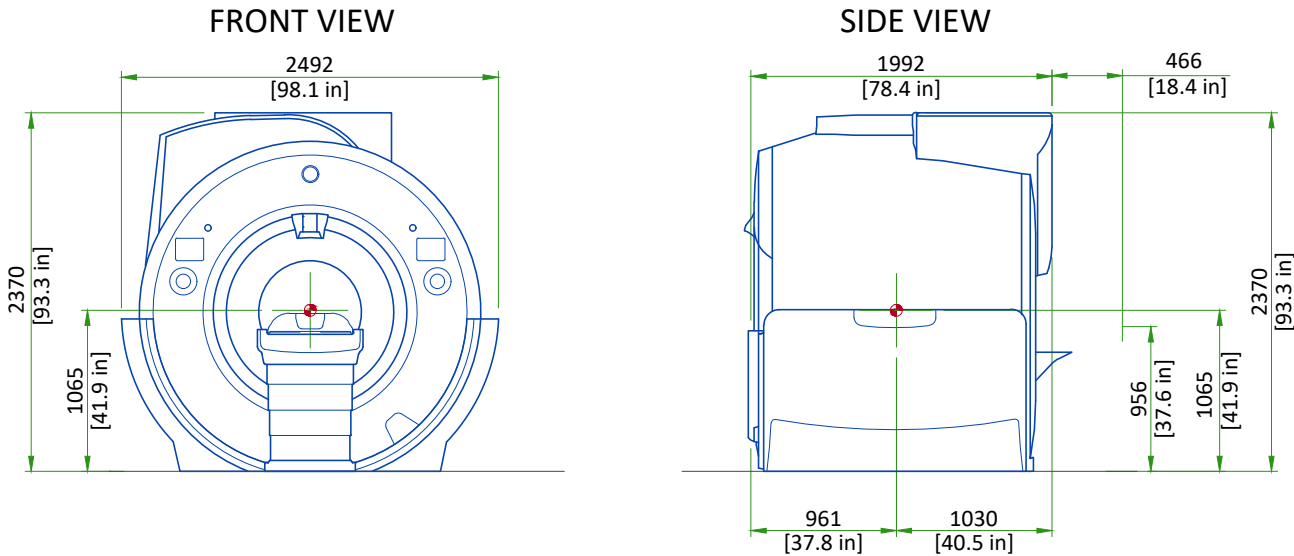


Note:  
Center of gravity is approximate and includes the GE Healthcare supplied VibroAcoustic Dampening Kit, but does not include cryogenics, gradient assembly, side mounted electronics, or enclosures.  
Enclosure dimensions are for reference only, NOT FOR SITE PLANNING USE.  
Center of gravity

PATIENT TABLE (PT)

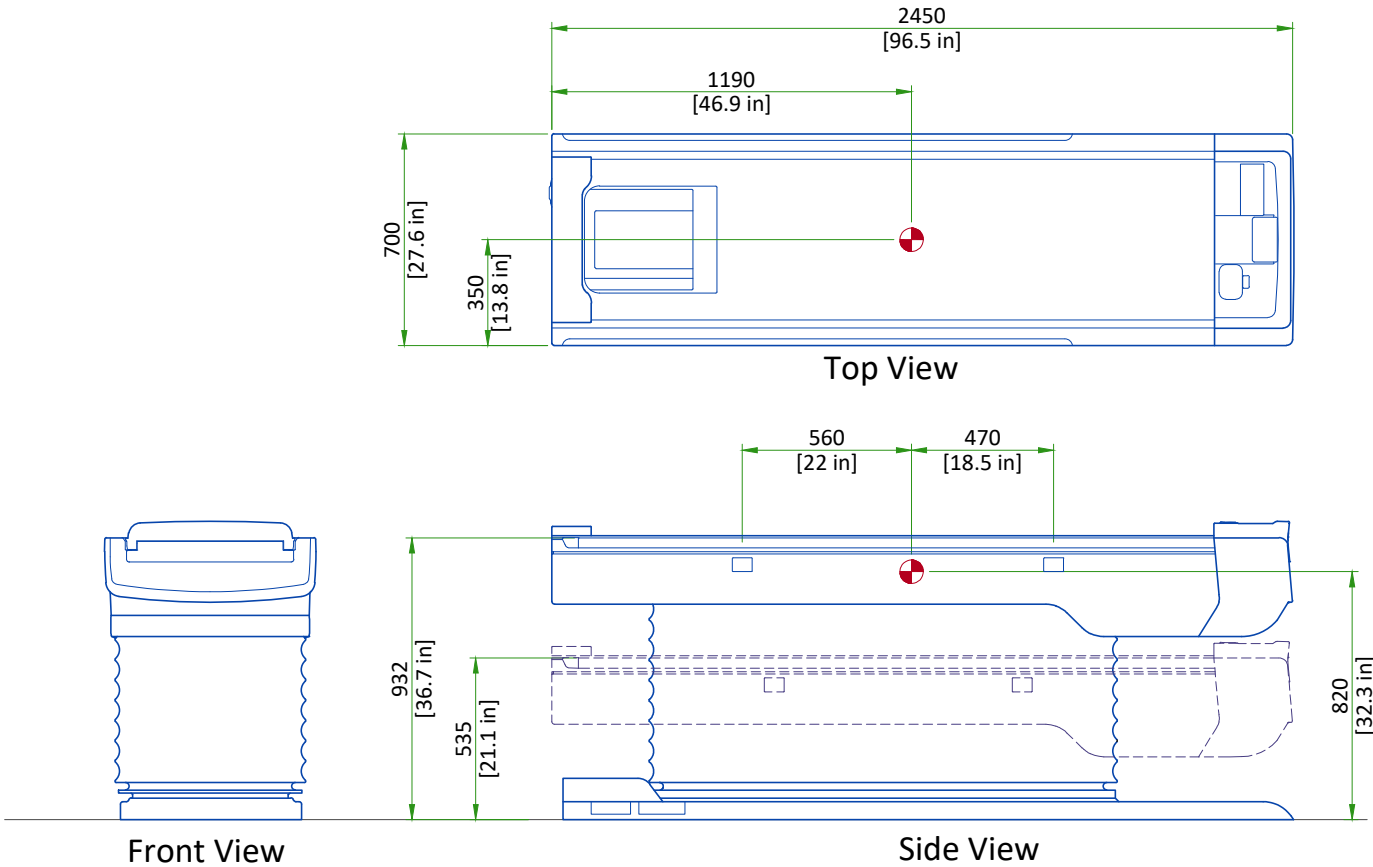


Scale 1:25

INTEGRATED COOLING CABINET

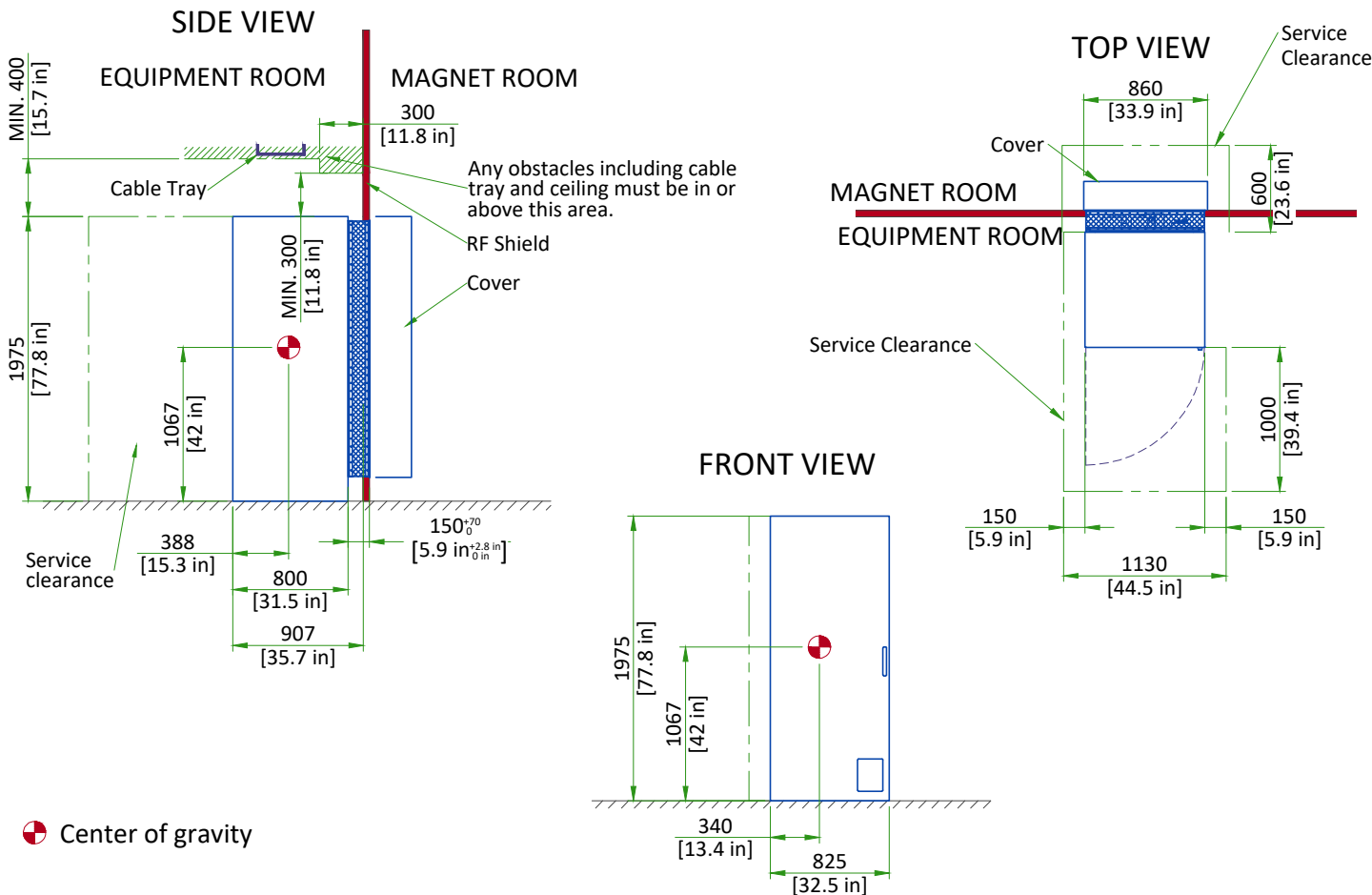


INTEGRATED SYSTEM CABINET

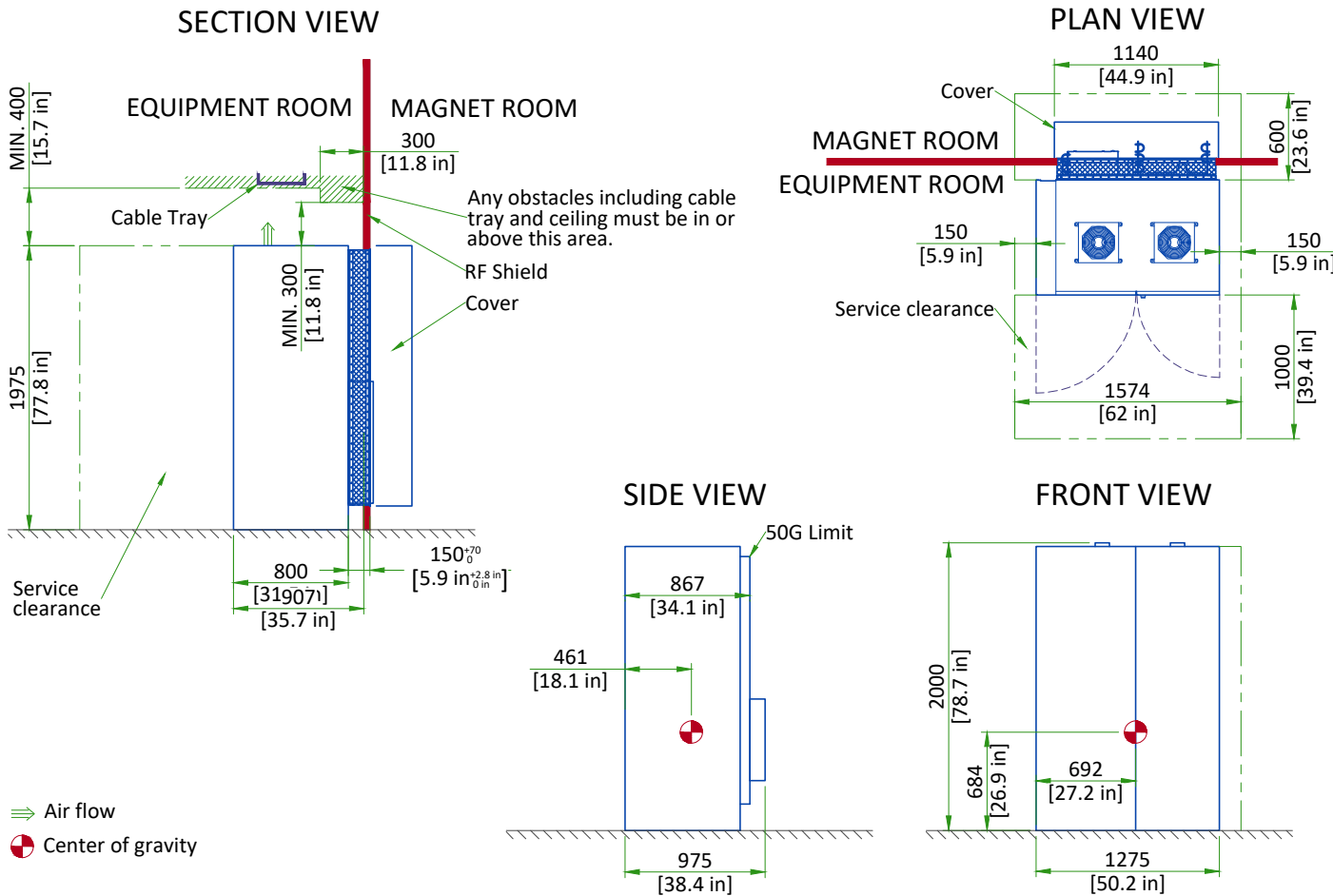




GLOBAL OPERATOR CABINET (GOC)

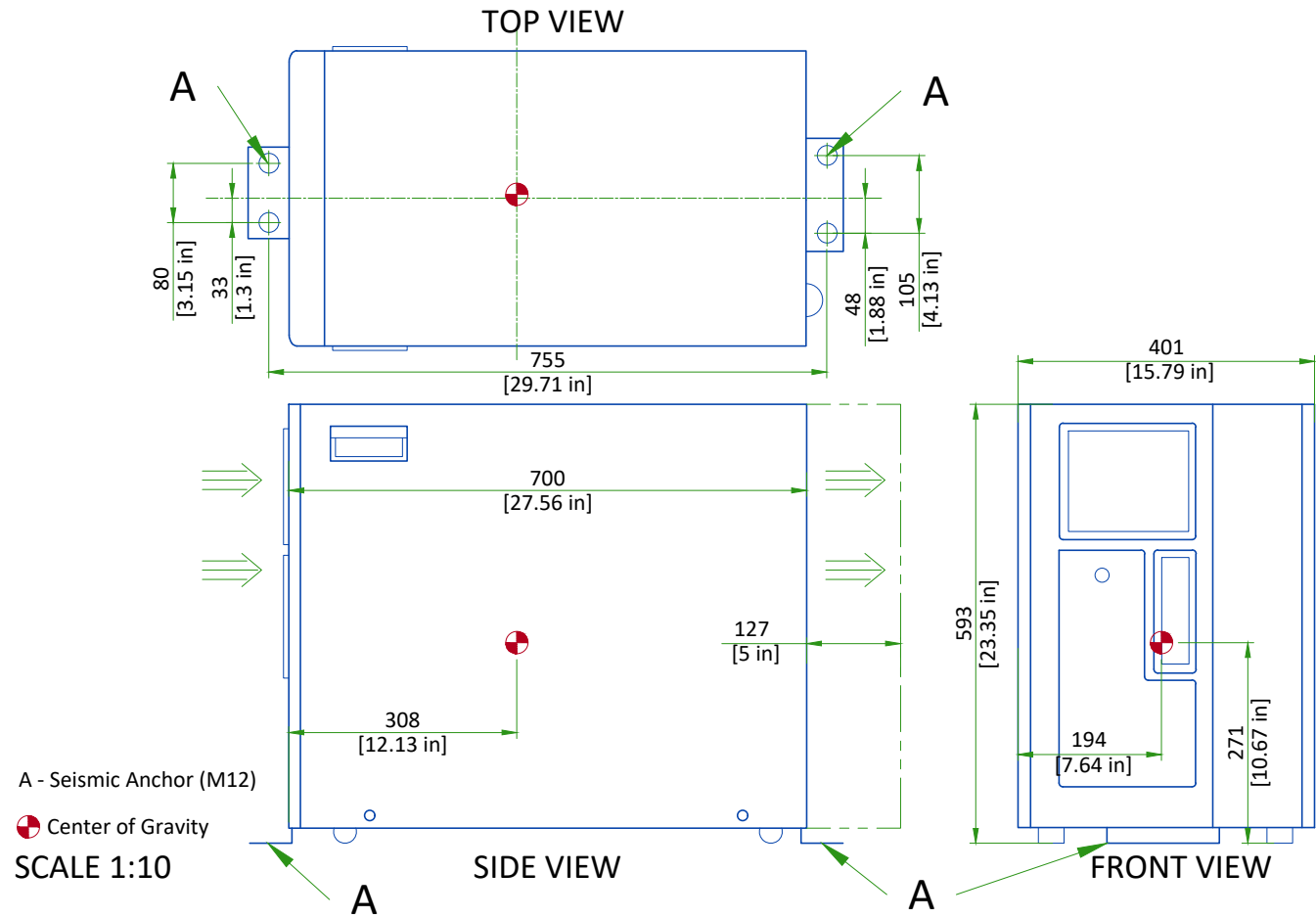

MAIN DISCONNECT PANEL

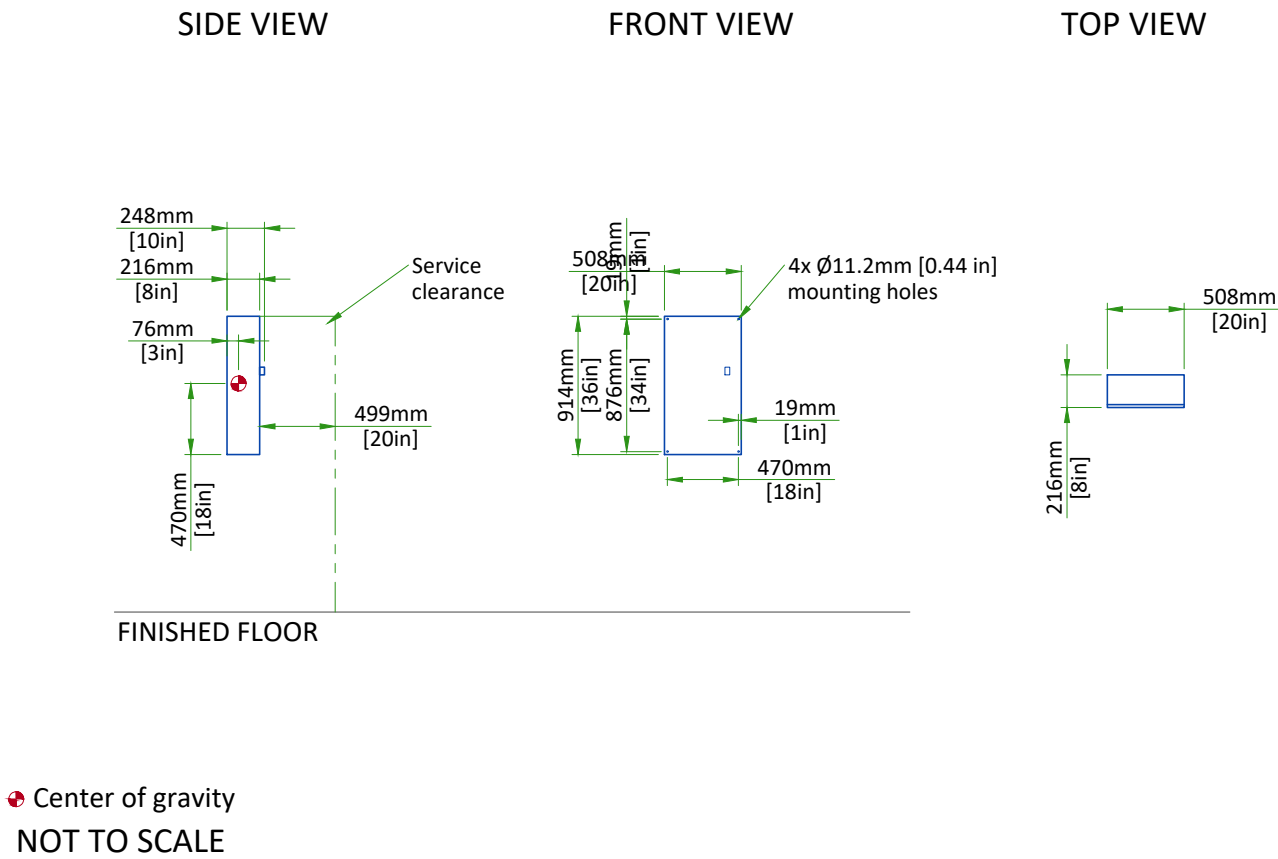


DELIVERY

ROUTING

- The customer is solely liable for routing of components from dock to final site.
- GE must be able to move system components in or out with no need to uncrate or disassemble any of the components. The entire passageway must be cleared, adequately lighted and free from dust.
- The floor and its surfacing must be able to withstand the live load of components and handling equipment.
- Floor surfacing must be continuous.
- The customer must protect any fragile flooring surfaces.

SPECIFICATIONS FOR MAGNET ROUTING

- Floor must be able to withstand a moving load of 7500 kg [16500 lb]
- Recommended opening height: 2.5m [98.5in], width: 2.5m [98.5in]. If recommended dimensions cannot be met refer to pre-installation manual for detailed specifications.
- Maximum slope: 30°

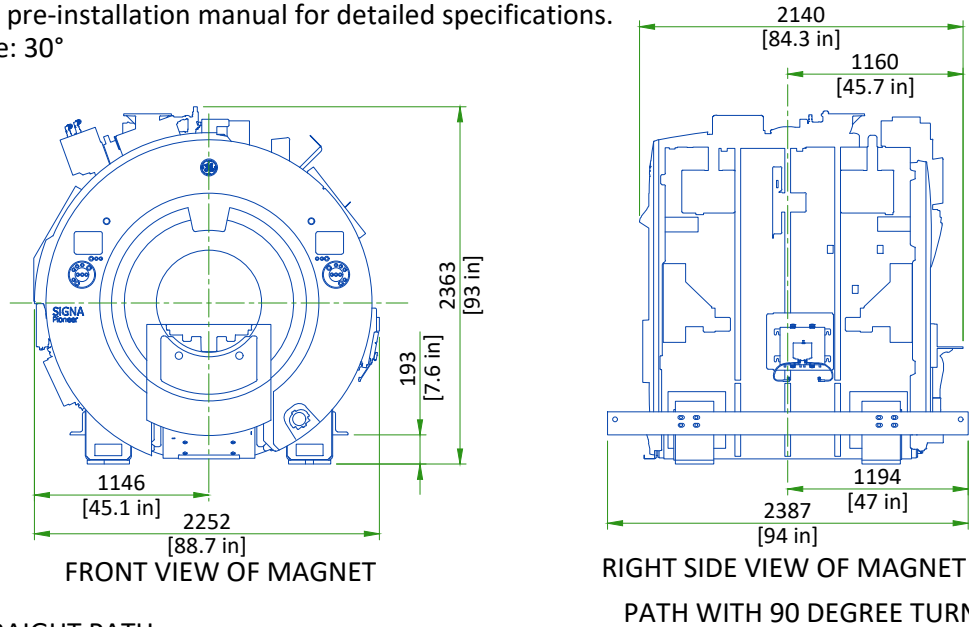

STORAGE CONDITIONS

- System components except the magnet should be stored in a cleaned room.
- Temperature = -30 to 60°C [-22 to 140°F], relative humidity < 90% non condensing.
- Material should not be stored for more than 90 days.
- The magnet will be delivered after GE validation of the site.

INSTALLATION AND DELIVERY ACCEPTANCE


- A survey of the site established by the customer and GE will make the decision for the delivery time.
- This survey of the site (a form is made available by GE) is only to check if the apparent conditions of the site allow the equipment to be delivered.
- If the site is not ready, GE can delay the delivery time.

CRITICAL ITEMS FOR MAGNET DELIVERY

- 24/7 chilled water and 480v power for shield/cryo cooler
- 24/7 120v power for the magnet monitor
- Phone lines for magnet monitoring and emergency use
- Magnet room exhaust fan
- Cryogen venting (if roof hatch, completed within 24 hrs)

This is only a partial list of items required for delivery of the magnet. For a complete checklist refer to the pre-installation manual referenced on cover sheet.

GRADIENT COIL REPLACEMENT

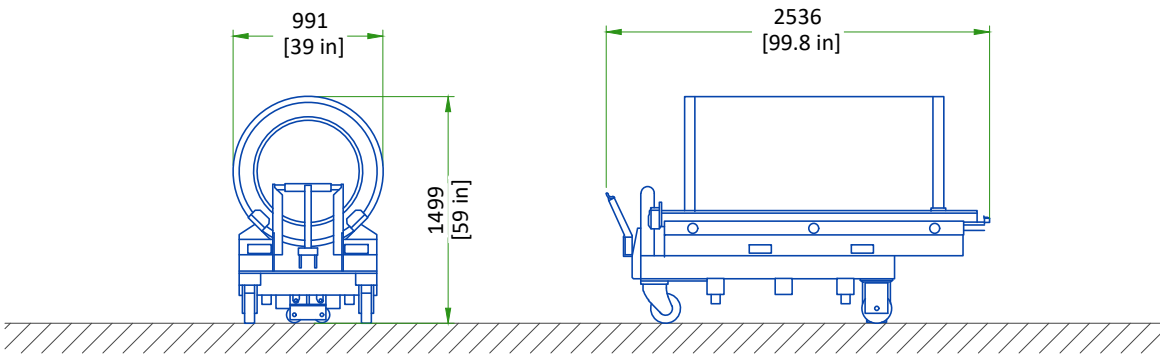

| EQUIPMENT  | DIMENSIONS LxWxH |             | WEIGHT |      | NOTE   |
|--|------------------|-------------|--------|------|--|
|  | mm               | in          | kg     | lbs  |  |
| Replacement XRM gradient coil assembly on a shipping cradle/cart | 991x2536x1499    | 39x99.84x59 | 1449   | 3194 | Initial gradient coil assembly is shipped installed in the magnet. Shipping/installation cart is used to install replacement coil assembly only. |

The weight bearing structure of the site should support any additional weight of the main replacement parts occurring during maintenance of the magnet, throughout the whole lifecycle of the MR.


STRUCTURAL NOTES

- All units that are wall mounted or wall supported are to be provided with supports where necessary. Wall supports are to be supplied and installed by the customer or his contractors.
- Dimensions are to finished surfaces of room.
- Certain MR procedures require an extremely stable environment to achieve high resolution image quality. Vibration is known to introduce field instabilities into the imaging system. The vibration effects on image quality can be minimized during the initial site planning of the mr suite by minimizing the vibration environment. See [PROXIMITY LIMITS](#), [PATIENT TABLE DOCK ANCHOR MOUNTING REQUIREMENTS AND VIBROACOUSTIC DAMPENING KIT](#) details for additional information.
- Standard steel studs, nails, screws, conduit, piping, drains and other hardware are acceptable if properly secured. Any loose steel objects can be violently accelerated into the bore of the magnet. Careful thought should be given to the selection of light fixtures, cabinets, wall decorations, etc. To minimize this potential hazard. For safety, all removable items within the magnet room such as faucet handles, drain covers, switch box cover plates, light fixture components, mounting screws, etc. must be non-magnetic. If you have a specific question about material, bring it to the attention of your GE project manager of installations.
- Floor levelness refer to [MAGNET ROOM FLOOR SPECIFICATIONS DETAIL](#), this floor levelness requirement is important for accurate patient table docking.
- Non-movable steel such as wall studs or hvac components will produce negligible effect on the active shield magnet.
- Customers contractor must provide all penetrations in post tension floors.
- Customers contractor must provide and install any non-standard anchoring. Documents for standard anchoring methods are included with GE equipment drawings for geographic areas that require such documentation.
- Customers contractor must provide and install hardware for "through the floor" anchoring and/or any bracing under access floors. This contractor must also provide floor drilling that cannot be completed because of an obstruction encountered while drilling by the GE installer such as rebar etc.
- Customers contractor to provide and install appropriate supports for the storage of excess cables.
- It is the customer's responsibility to perform any floor or wall penetrations that may be required. The customer is also responsible for ensuring that no subsurface utilities (e.g., electrical or any other form of wiring, conduits, piping, duct work or structural supports (i.e. post tension cables or rebar)) will interfere or come in contact with subsurface penetration operations (e.g. drilling and installation of anchors/screws) performed during the installation process. To ensure worker safety, GE installers will perform surface penetration operations only after the customer's validation and completion of the "GE surface penetration permit"


VIBRATION SPECIFICATIONS


Excessive vibration can affect MR image quality. Vibration testing must be performed early in the site planning process to ensure vibration is minimized. Both steady state vibration (exhaust fans, air conditioners, pumps, etc.) and transient vibrations (traffic, pedestrians, door slamming, etc.) must be assessed. The Magnet cannot be directly isolated from vibration. Any vibration issue must be resolved at the source.

Transient vibration levels above the specified limits in the MR Site Vibration Test Guidelines must be analyzed. Any transient vibration that causes vibration to exceed the steady-state level must be mitigated.

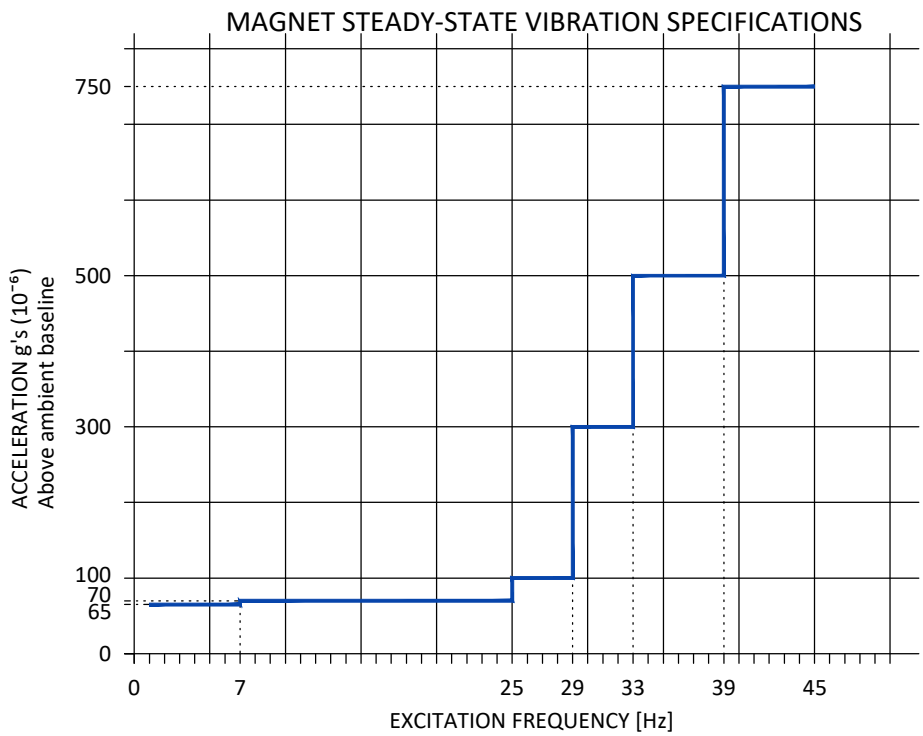

VIBRATION TRANSMITTED THROUGH VIBROACOUSTIC MAT

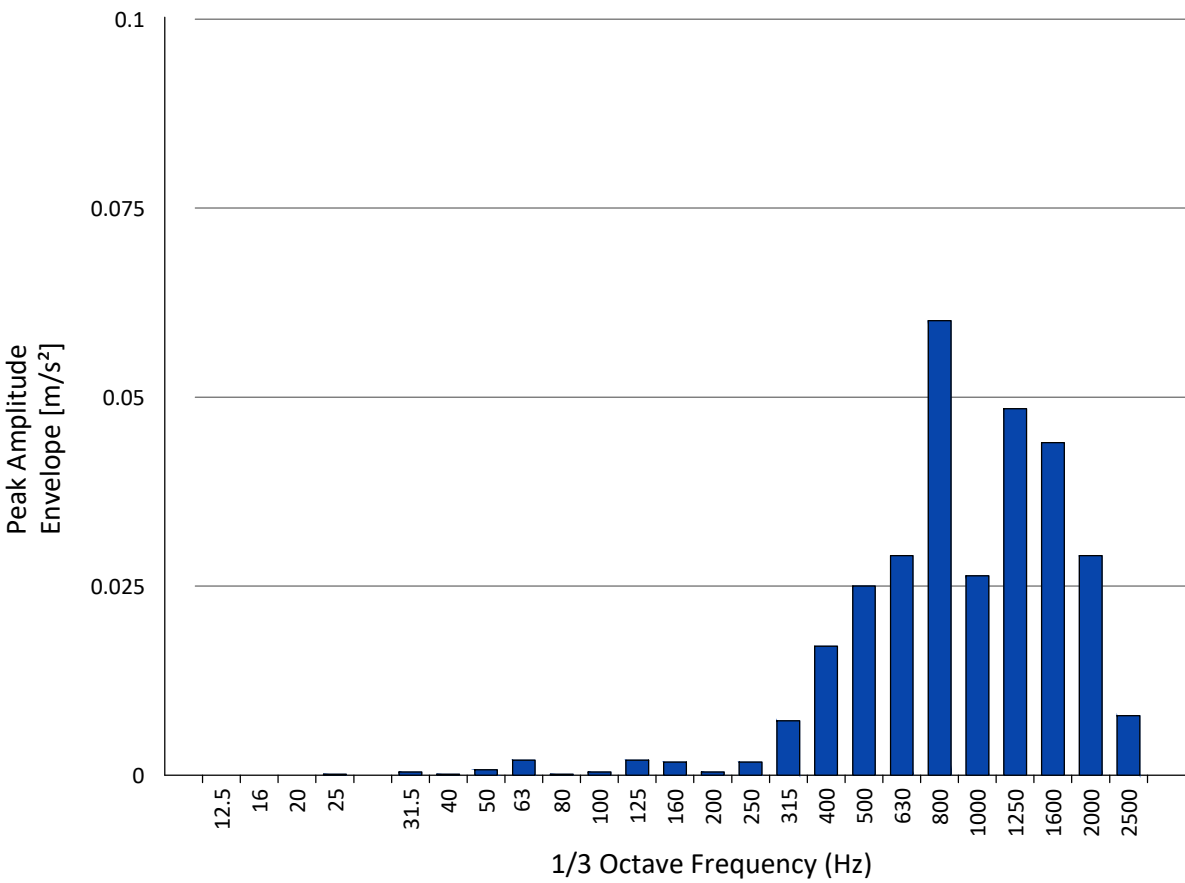


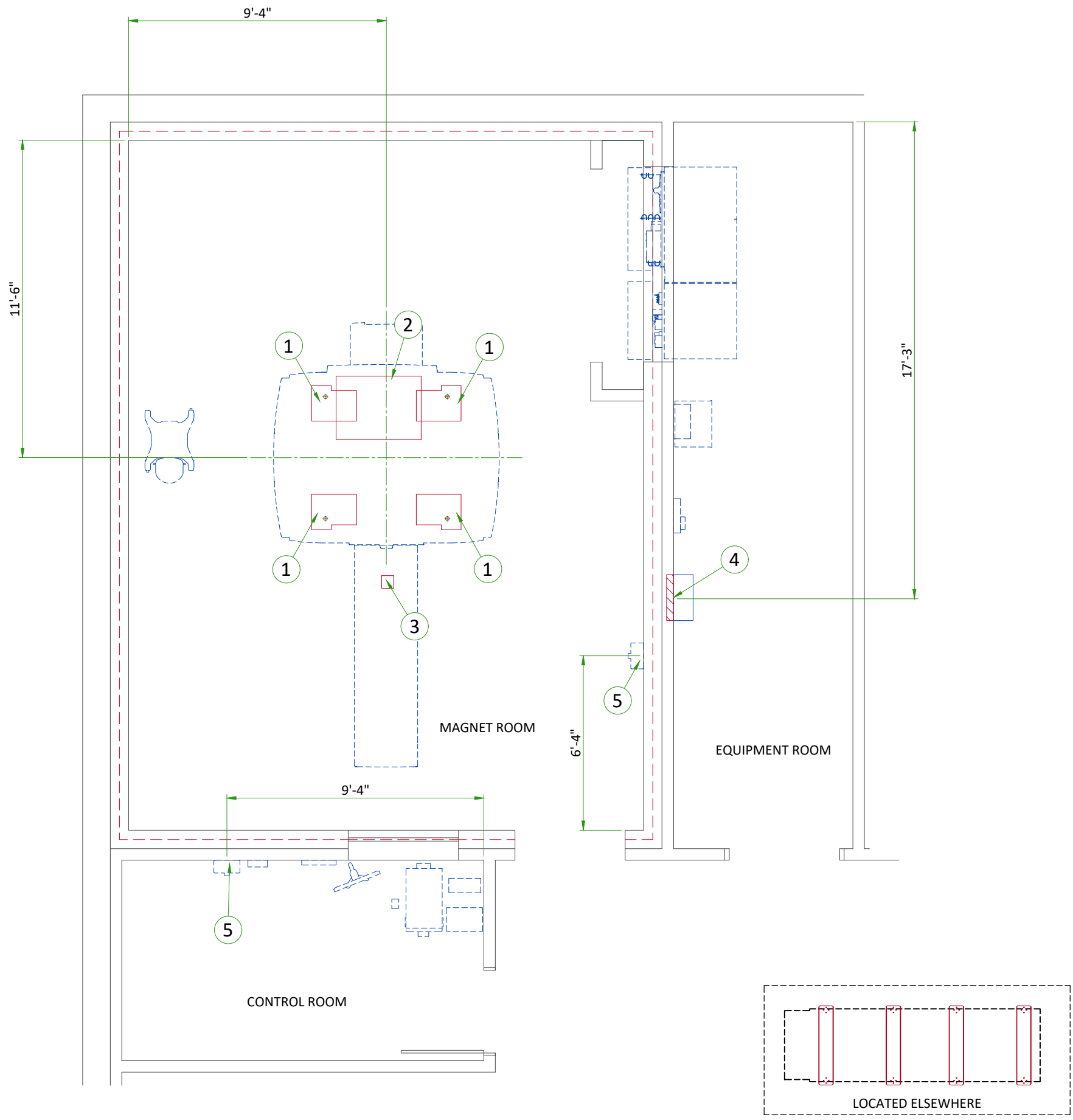

| ITEM                                 | DESCRIPTION   |
|--------------------------------------|---|
| (GE SUPPLIED / CONTRACTOR INSTALLED) |   |
| 1                                    | Vibroacoustic dampening kit (see floor structural detail) |
| 2                                    | Magnet curtain kit  |
| (CONTRACTOR SUPPLIED & INSTALLED)    |   |
| 3                                    | Patient table dock anchoring                              |
| 4                                    | Structural wall backing for Main Disconnect Panel         |
| 5                                    | Structural wall backing for Magnet Rundown Unit           |




MAGNET ON VIBROACOUSTIC DAMPENING KIT "VIBROMAT"

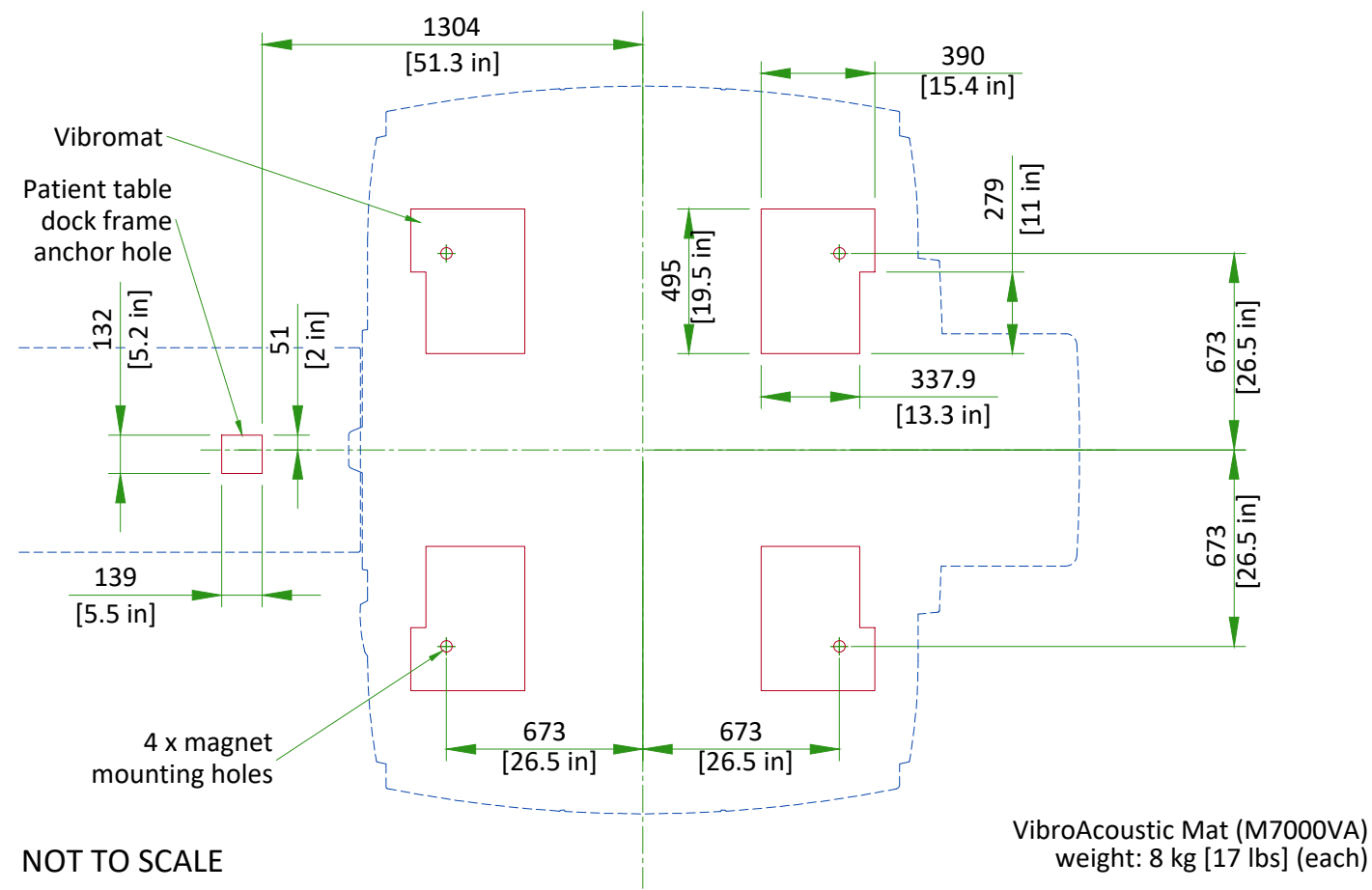


MAGNET ROOM FLOOR SPECIFICATIONS

Floor must be flat and level to **3 mm** [0.12in] between high and low spots over the area shown.

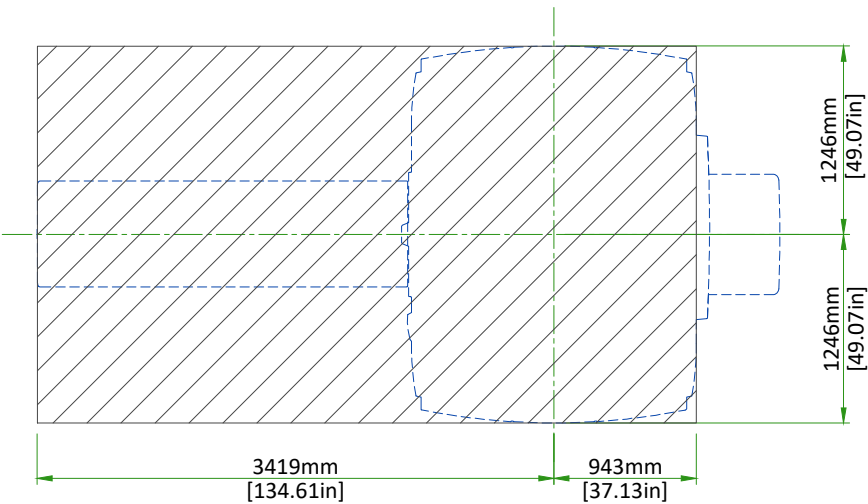

The finished floor must support the weight of all components (e.g., patient table, gradient coil replacement cart) throughout operation and service life.

TABLE FRAME ANCHOR MOUNTING REQUIREMENTS

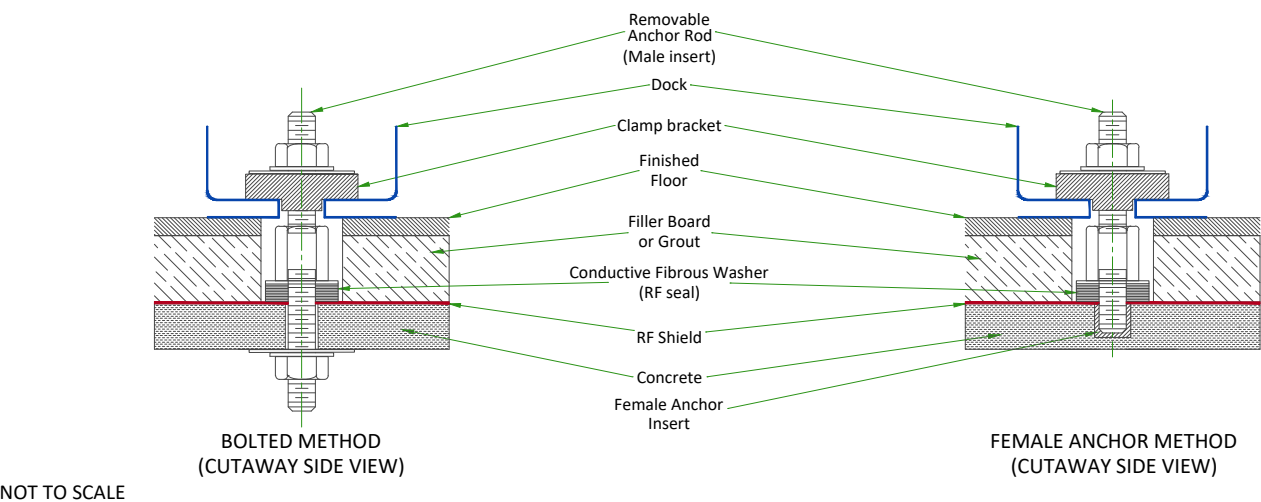


1. The RF Shield vendor must design and install the table frame anchor bolt
2. The table frame anchor hole must be drilled after the Magnet is delivered
3. The table frame anchor must not contact floor rebar or other structural steel
4. The table frame anchor must electrically contact the RF shield at point of entry
5. The table frame anchors must have the following properties:
  - a. Anchors must be two-part assembly (male/female)
  - b. Female side must be expansion- or epoxy-type
  - c. Male side must be a bolt or threaded rod with appropriate-sized nut (bolt or rod must be removable - not epoxied or cemented in place)
  - d. Anchors must be electrically conductive
  - e. Anchors must be non-magnetic
  - f. Anchors must not induce galvanic corrosion with the RF shield
  - g. Anchors must be commercially procured
  - h. The anchor rod hole clearance in the table anchor base is 11 mm (0.43 in). The anchor rod diameter must be sized appropriately
  - i. Anchors must meet the following clamping force: 2669 N (600 lbs)
  - j. The anchor rod must extend 40 mm ± 13 mm (1.57 in ± 0.5 in) above the finished floor
  - k. The anchor rod must be less than 152 mm (6 in) in total length (length above the floor plus embedded length)
6. The RF shield vendor must perform a pull test on the anchor (equal to the clamping force). Results must be provided to the GE Healthcare Project Manager of Installation (PMI).


CABLE CONCEALMENT

- NOTES:
- 1) Ceiling Opening and Cable Concealment Kit Frame Fasten Area are measured from isocenter.
  - 2) Cable Concealment Kit Frame Fasten Area must be enforced to sustain the Kit (Frame + Covers) weight 30 kg [66 lb].

Refer to Pre-Magnet Delivery manual 5728068-1EN for additional information.

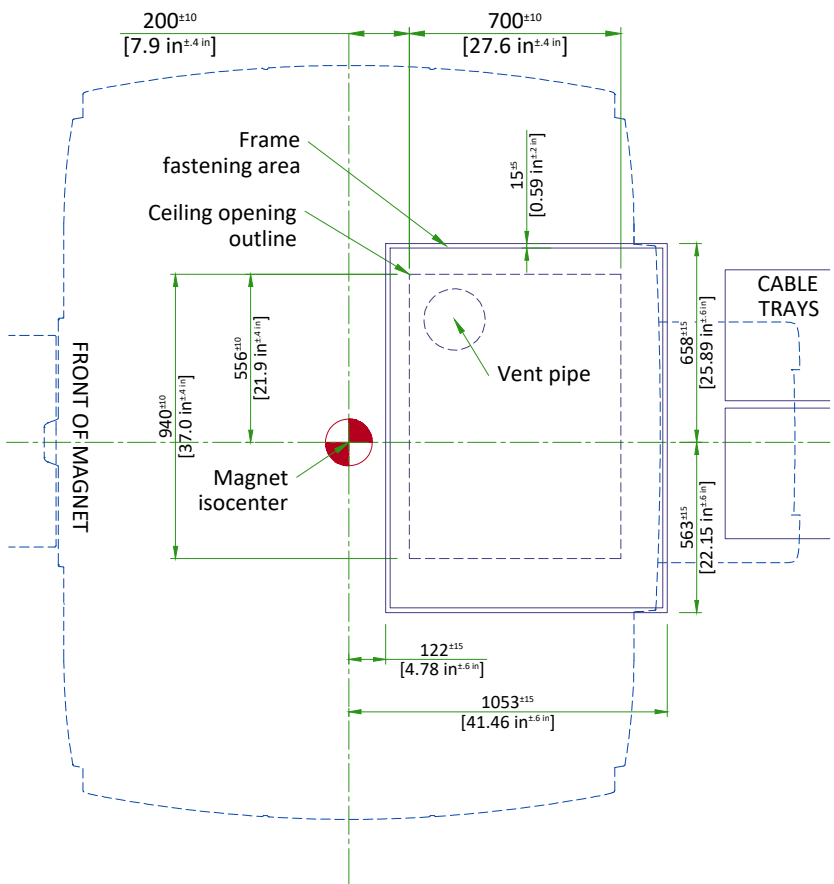

SCALE 1:25

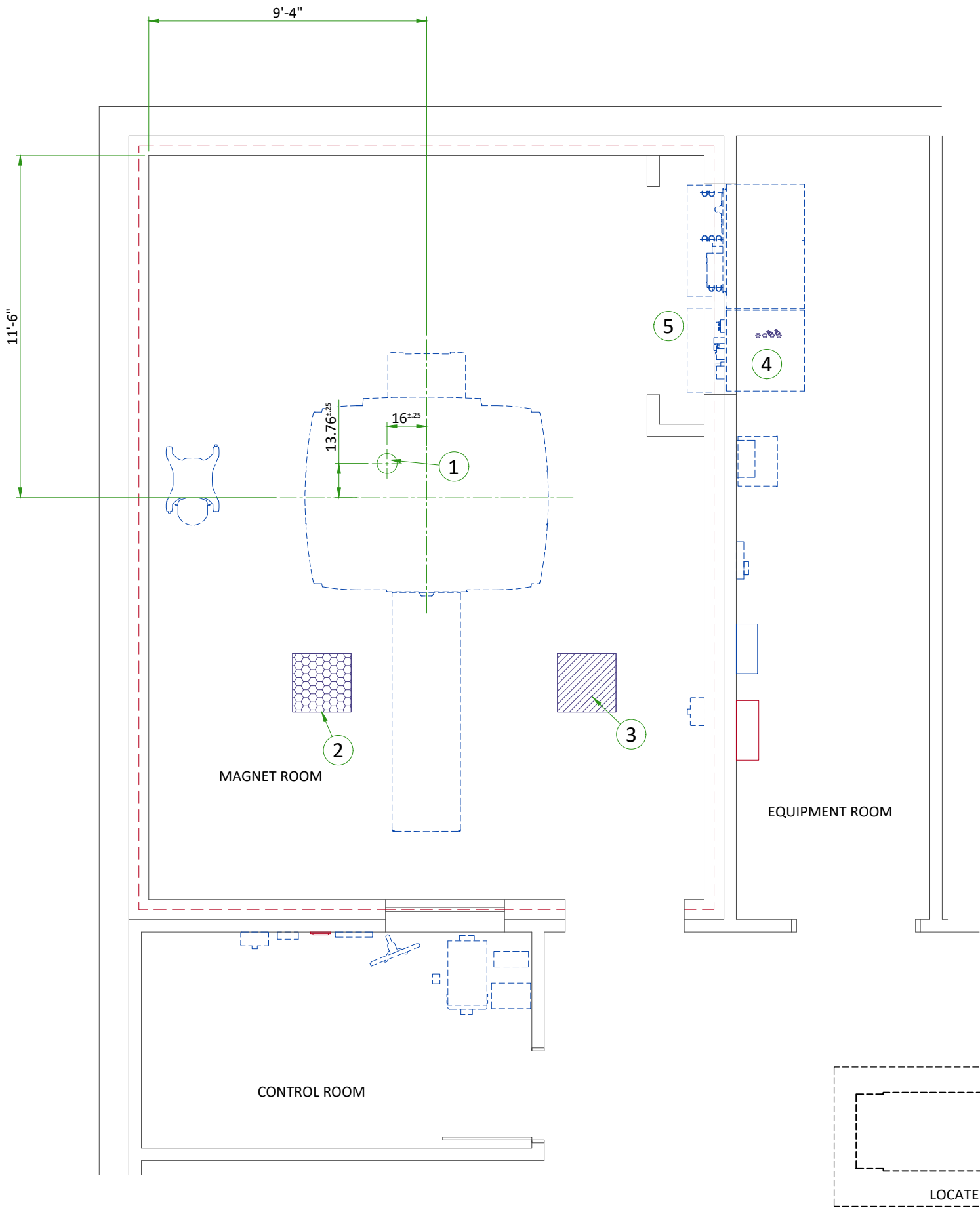

| ITEM | DESCRIPTION   |
|------|---|
| 1    | Cryogen vent (200mm [8"] O.D.)  |
| 2    | Emergency exhaust vent - refer to magnet room vent requirements (position to be defined)                |
| 3    | Pressure equalization vent - refer to magnet room vent requirements (position in ceiling to be defined) |
| 4    | (2) 38mm [1.5"] copper lines (insulated) and (2) shut off valves. refer to chilled water block diagram  |
| 5    | Closet must allow free air exchange of 400 CFM between magnet room and closet                           |
| 6    | (2) 50mm [2"] I.D. High pressure hoses and (2) 50mm [2"] to 38mm [1.5"] Reducers                        |

MECHANICAL/PLUMBING NOTES

- All piping, fittings, supports, hoses, clamps, ventilation systems, etc. are to be supplied and installed by the customer or his contractors.
- For complete design and requirements, specifications and guidelines refer to the pre-installation manual: system cooling, cryogen venting, waveguides and exhaust venting.
- An emergency water cooling back-up supply is recommended for continuous cryogen compressor operation. if using an open loop back-up design, ensure a drain is provided. please refer to the pre-install manual for optional back-up coolant supply requirements

TEMPERATURE AND HUMIDITY SPECIFICATIONS

IN-USE CONDITIONS

|                          | MAGNET ROOM |          |           | CONTROL ROOM |     |         | EQUIPMENT ROOM |           |           |
|--------------------------|-------------|----------|-----------|--------------|-----|---------|----------------|-----------|-----------|
| Temperature              | 15-21°C     |          |           | 15-32°C      |     |         | 15-32°C [3]    |           |           |
|                          | 59-69.8°F   |          |           | 59-89.6°F    |     |         | 59-89.6°F [3]  |           |           |
| Temperature gradient [1] | ≤ 3°C/h     |          |           | ≤ 3°C/h      |     |         | ≤ 3°C/h        |           |           |
|                          | ≤ 5°F/h     |          |           | ≤ 5°F/h      |     |         | ≤ 5°F/h        |           |           |
| Relative humidity        | 30% to 60%  |          |           | 30% to 75%   |     |         | 30% to 75%     |           |           |
| Humidity gradient [2]    | ≤ 5%/h      |          |           | ≤ 5%/h       |     |         | ≤ 5%/h         |           |           |
| System heat dissipation  | Stand by    | Max      | Average   | Stand by     | Max | Average | Stand by       | Max       | Average   |
|                          | 0.56kW      | 2.40kW   | 1.20kW(2) | 1.45kW       |     |         | 1.15kW         | 9.10kW    | 3.50kW(2) |
|                          | 1915 btu    | 8189 btu | 4095 btu  | 4947 btu     |     |         | 3316 btu       | 30456 btu | 11337 btu |

NOTE

1) Operating temperature gradient limits shall be between -3°C/hr (-5°F/hr) and 3°C/hr (5°F/hr), when averaged over 1 hour

2) Operating humidity gradient limits shall be between -5% RH/hr and 5% RH/hr (5°F/hr), when averaged over 1 hour

3) Maximum ambient temperature is derated by 1°C per 300 m above 2000 m (not to exceed 2600 m).

AIR RENEWAL

According to local standards.

NOTE

In case of using air conditioning systems that have a risk of water leakage it is recommended not to install it above electric equipment or to take measures to protect the equipment from dropping water.

HEAT DISSIPATION DETAILS

| DESCRIPTION                         | ROOM      | IDLE |      | AVERAGE |      | MAX  |       |
|-------------------------------------|-----------|------|------|---------|------|------|-------|
|                                     |           | W    | BTU  | W       | BTU  | W    | BTU   |
| Magnet (MAG) and Patient Table (PT) | Magnet    | 561  | 1915 | 1200    | 4095 | 2400 | 8189  |
| Main Disconnect Panel (MDP)         | Equipment | 132  | 450  | 132     | 450  | 264  | 901   |
| Integrated System Cabinet (ISC)     | Equipment | 280  | 955  | 2130    | 7270 | 7100 | 24232 |
| Integrated Cooling Cabinet (ICC)    | Equipment | 0    | 0    | 500     | 1706 | 1000 | 3412  |
| Cryocooler Compressor (CRY)         | Equipment | 500  | 1706 | 500     | 1706 | 500  | 1706  |
| Magnet Monitor (MON)                | Equipment | 240  | 819  | 240     | 819  | 240  | 819   |
| Operator Workspace equipment (OW)   | Control   | 1450 | 4947 | 1450    | 4947 | 1450 | 4947  |
| OPTIONS                             |           |      |      |         |      |      |       |
| MR Elastography (MRE)               | Equipment | 141  | 480  | 141     | 480  | 141  | 480   |

MAGNET ROOM VENTING REQUIREMENTS

HVAC VENT REQUIREMENTS

- HVAC vendor must comply with Magnet room temperature and humidity specifications and RF shielding specifications.
- RF Shield vendor must install open pipe or honeycomb HVAC waveguides.
- All serviceable parts in the Magnet room (e.g.: diffusers) must be non-magnetic.
- Waveguides must be nonmagnetic and electrically isolated.
- Incoming air must contain at least **5% air** from outside the Magnet room (inside or outside the facility) to displace residual helium.


EMERGENCY VENT REQUIREMENT

- Exhaust vent system is supplied by the customer.
- All items within the RF enclosure must be non-magnetic.
- The exhaust vent system must be tested and operational before the magnet is installed.
- The exhaust intake vent must be located near the magnet cryogenic vent at the highest point on the finished or drop ceiling.
- The Magnet room exhaust fan and exhaust intake vent must have a capacity of at least **1200 CFM (34 m³/min)** with a minimum of **12 room air exchanges per hour**.
- The exhaust fan must be placed above RF shielding located outside 10 gauss (1mT) and with appropriate waveguide.
- The system must have a manual exhaust fan switch near the Operator Workspace and in the Magnet room near the door (the switches must be connected in parallel).
- All system components must be accessible for customer inspection, cleaning and maintenance


PRESSURE VENT REQUIREMENT

- A **pressure equalizing vent is required** in the magnet room ceiling or in the wall, at the highest point possible.
  - The **vent minimum size** must be **(610 mm x 610 mm [24 in x 24 in])** or equivalent.
  - The pressure equalization vent must be located so any Helium gas is not vented into occupied areas.
- Note: Location may affect acoustic noise transmission into occupied spaces.

MAGNET ROOM EXHAUST FAN SCHEMATIC

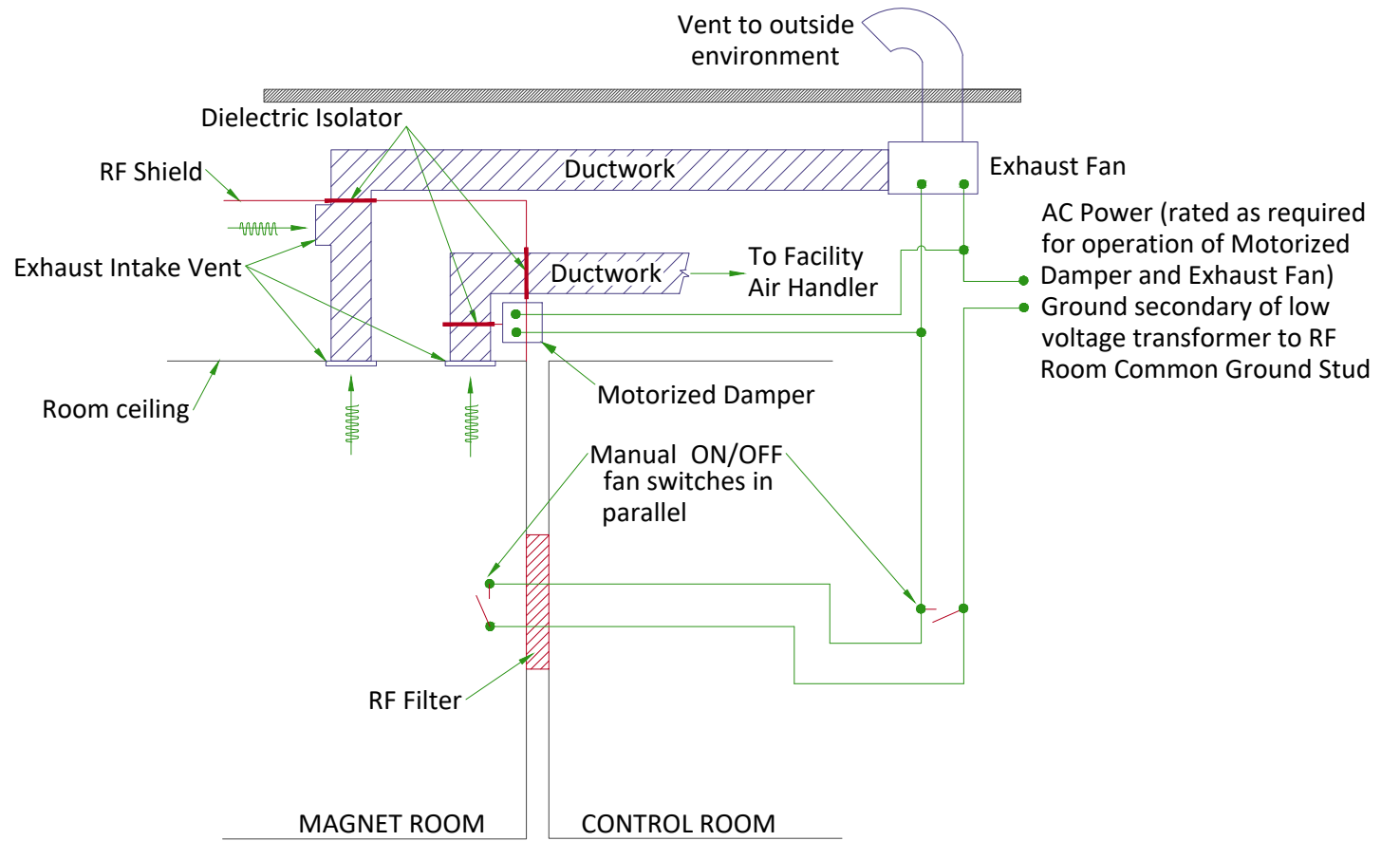

CHILLED WATER BLOCK DIAGRAM

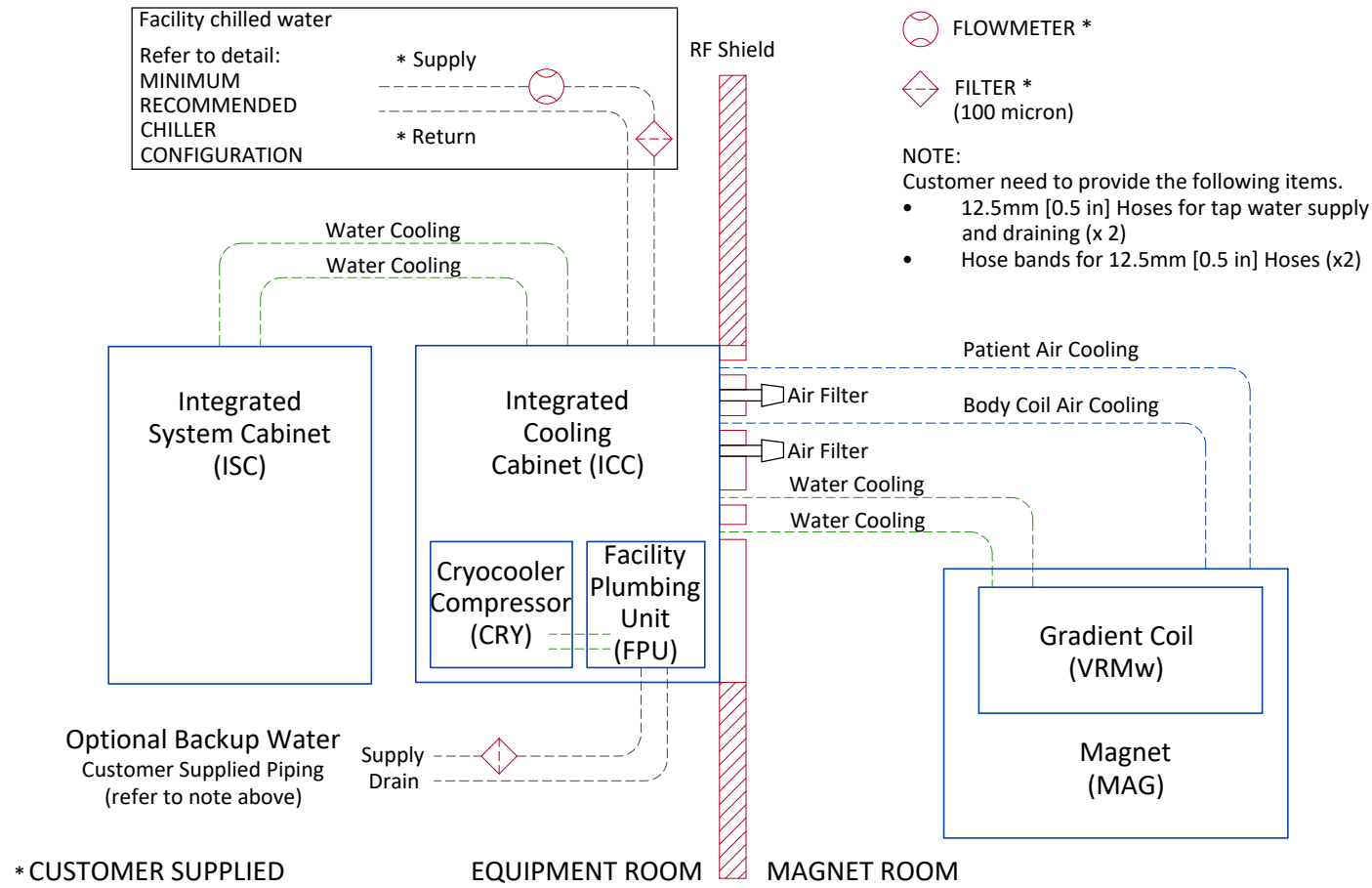


CHILLED WATER SPECIFICATIONS

| PARAMETER                                     |                                    | REQUIREMENTS   |                                     |
|---|------------------------------------|--|-------------------------------------|
| Chiller size                                  |                                    | Minimum 36kW   |                                     |
| Inlet temperature                             |                                    | 5 to 15°C [41 to 59°F] measured at the inlet to the ICC  |                                     |
| Hose connection to ICC (supplied by customer) |                                    | 25.4mm (1 in) minimum hose inside diameter   |                                     |
|   |                                    | MINIMUM FLOW<br>50 l/min (13.2 gpm)  | MAXIMUM FLOW<br>80 l/min (21.1 gpm) |
| PRESSURE DROP<br>IN ICC CABINET               | 40% propylene glycol, 60%<br>water | 1.8 bars (26.1 psi)  | 3.4 bars (49.3 psi)                 |
| Availability                                  |                                    | Continuous   |                                     |
| Antifreeze                                    |                                    | 0-40% propylene glycol   |                                     |
| Maximum inlet pressure to ICC                 |                                    | Maximum 5.52 bar (80 psi)  |                                     |
| Minimum continuous heat load                  |                                    | 7.5 kW   |                                     |
| Condensation protection                       |                                    | Facility plumbing to the ICC must be properly routed and insulated to prevent equipment damage or safety hazards |                                     |
| Water quality                                 |                                    | Refer to pre-installation manual for detailed specifications   |                                     |

- NOTE
- The customer must balance the cost of cryogenes and local controls with the cost of emergency backup facility coolant.
  - For the site without insite connection, the customer must provide and install an in-line thermometer on the supply facility coolant hose. The thermometer must be capable of visually displaying thermometer covering 41 to 59°F (5 to 15°C) and configured for the properties of the cooling fluid in use.


CITY WATER BACKUP SPECIFICATIONS FOR COMPRESSOR

INLET WATER FLOW/TEMPERATURE FOR CRYOCOOLER COMPRESSOR

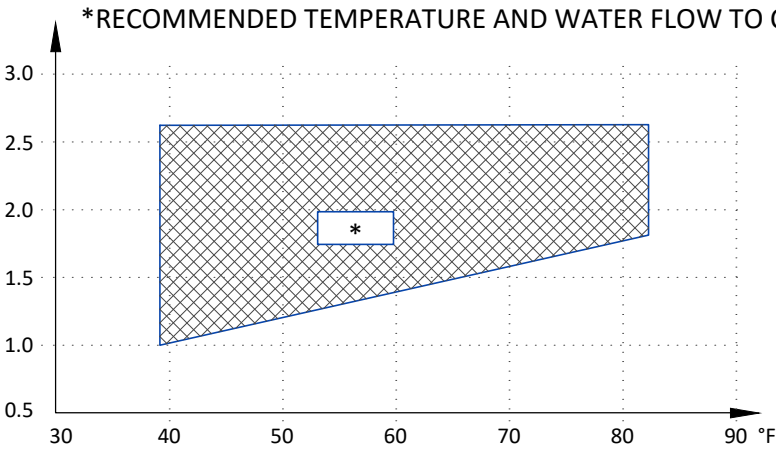


|                       | MIN   | MAX  |
|-----------------------|---|--|
| INLET TEMP            | 39.2°F<br>(4°C)                                     | 82.4°F<br>(28°C)                             |
| INLET FLOW            | 1.0 gpm<br>(4 l/min)                                | 2.6 gpm<br>(10 l/min)                        |
| TEMP RISE             | 89.6°F at 1.0 gpm<br>(32°C at 4 l/min flow)         | 53.6°F at 2.6 gpm<br>(12°C at 10 l/min flow) |
| HEAT DISSIPATION (kW) | 7.2 kW  |  |
| PRESSURE DROP         | 8.7 psi at 2.1 gpm flow<br>(60 kPa at 8 l/min flow) |  |

MINIMUM RECOMMENDED CHILLER CONFIGURATION

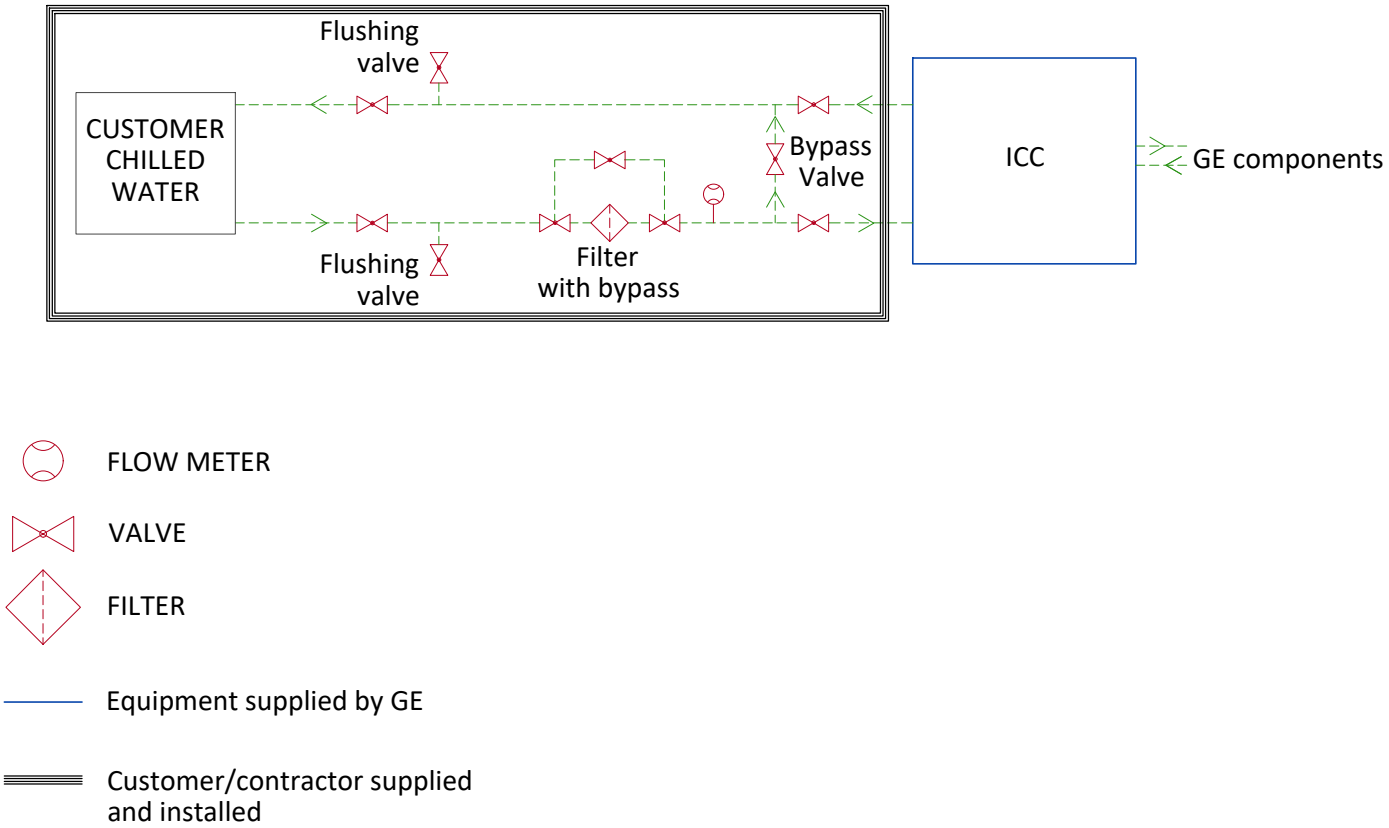



## TYPICAL CRYOGENIC VENT PIPE DETAIL

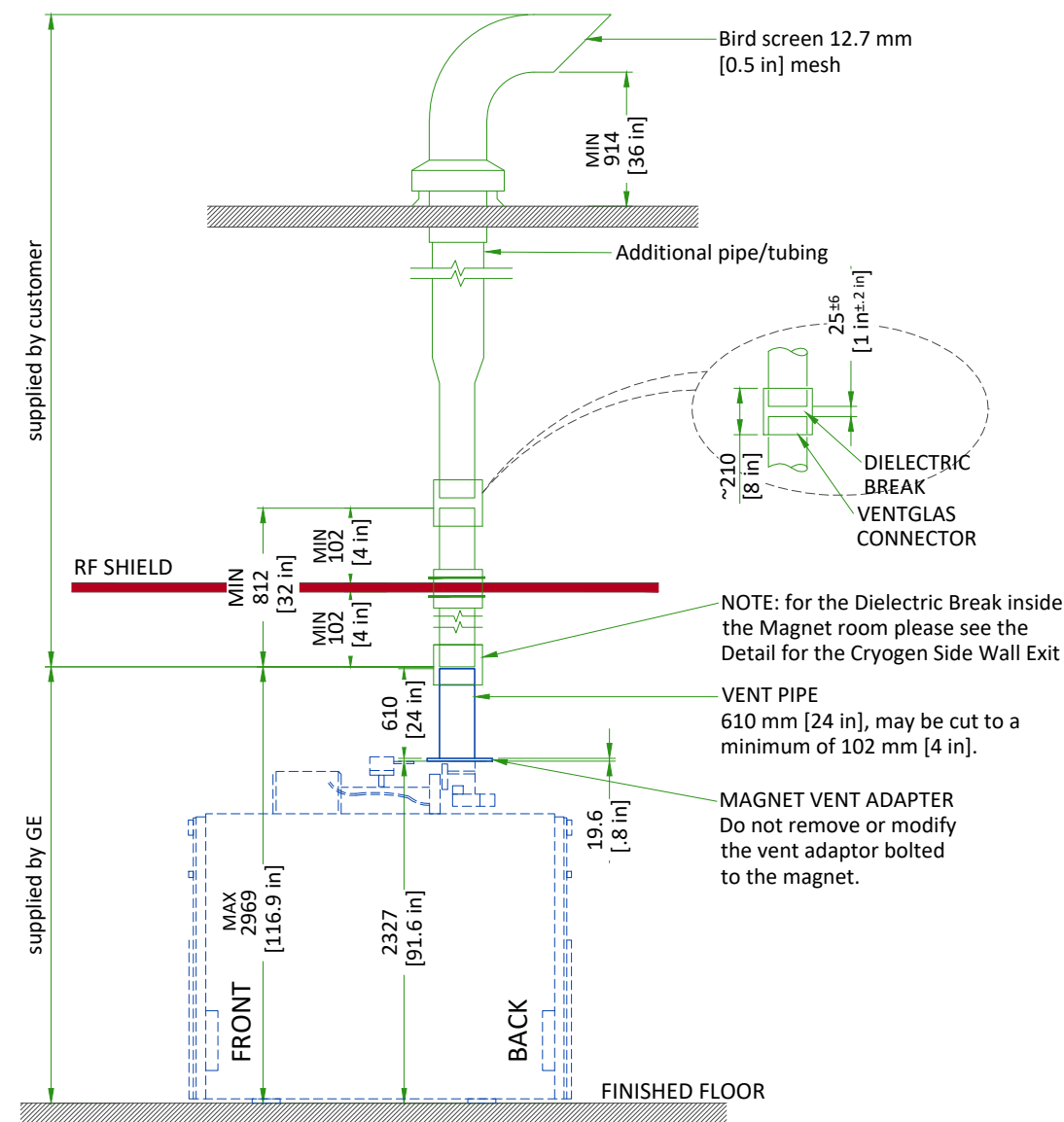

Waveguide is contractor supplied. Minimum 812 mm [32 in]. Must extend at least 100 mm [4 in] on magnet room side of the wall/ceiling and 25±6 mm [1±0.25 in] from the GE supplied pipe below isolation joint. Magnet room end must not be more than 2969 mm [117 in] above finished floor.

- The 203 mm [8 in] OD vent material must be one of the following materials with the wall thickness indicated:
  - SS 304: Minimum 0.89 mm [0.035 in]; Maximum 3.18 mm [0.125 in]
  - AL 6061-T6: Minimum 2.11 mm [0.083 in]; Maximum 3.18 mm [0.125 in]
  - CU DWV, M or L: Minimum 2.11 mm [0.083 in]; Maximum 3.56 mm [0.140 in]
- Either tubes or pipes may be used and must be seamless or have welded seams

### NOTE


All welds on the pipe must be ground down to a smooth 203 mm [8 in] diameter so that it can be clamped to the Ventglas with enough force.

- Corrugated pipe or spiral duct must not be used
- If required, bellows pipe less than 300 mm [12 in] in length may be used as a thermal expansion joint
- The vent pipe must withstand the maximum pressure listed in the Pre-Installation Manual
- Waveguide vent material must match the outside diameter of the magnet flanged vent adapter


## TYPICAL CRYOGEN SIDE WALL EXIT

### KEY COMPONENTS:

- RF waveguide extended from wall to magnet adapter
- Must be all same material and all welded or brazed
- Support system must withstand 11125 N (2500 lbs)
- GE ventglas must be installed in vertical section directly over magnet

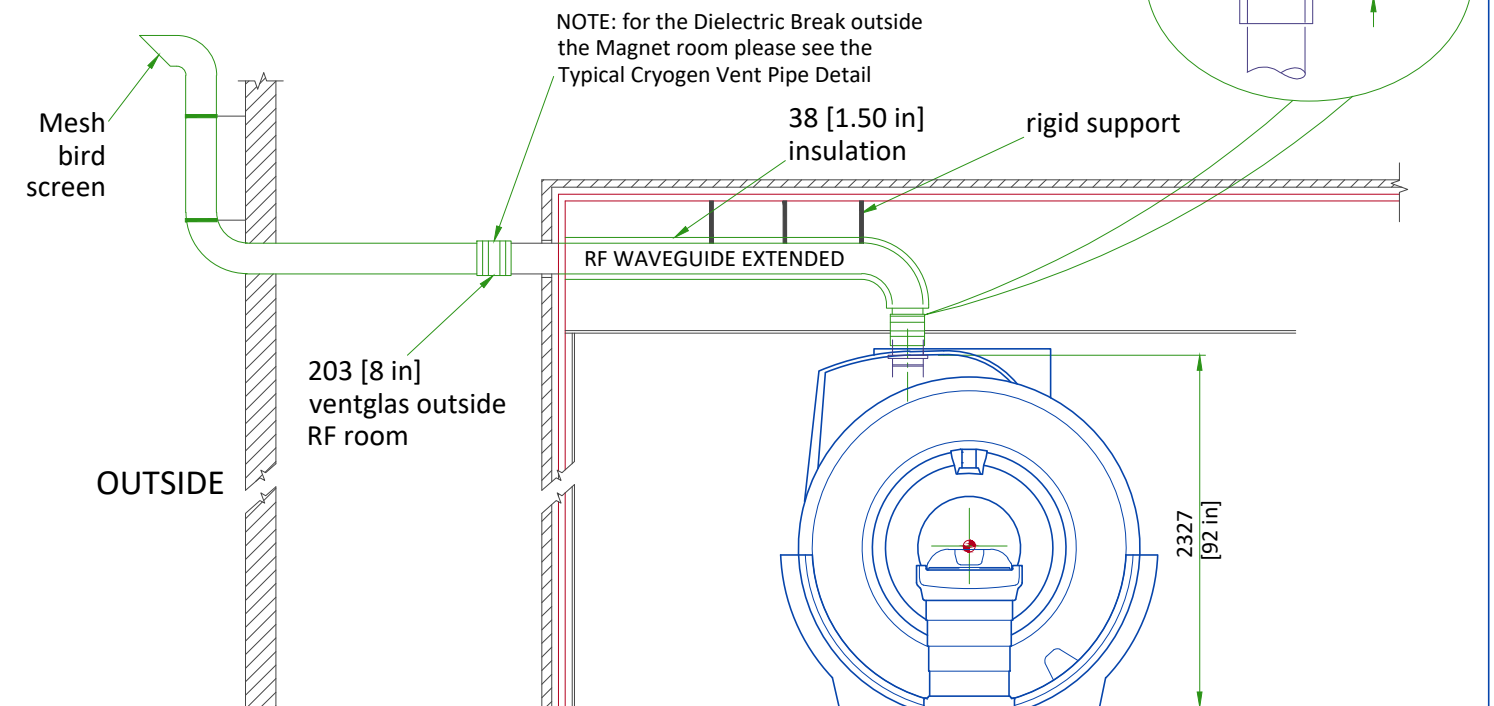

## CRYOGENIC VENTING ( EXTERIOR )

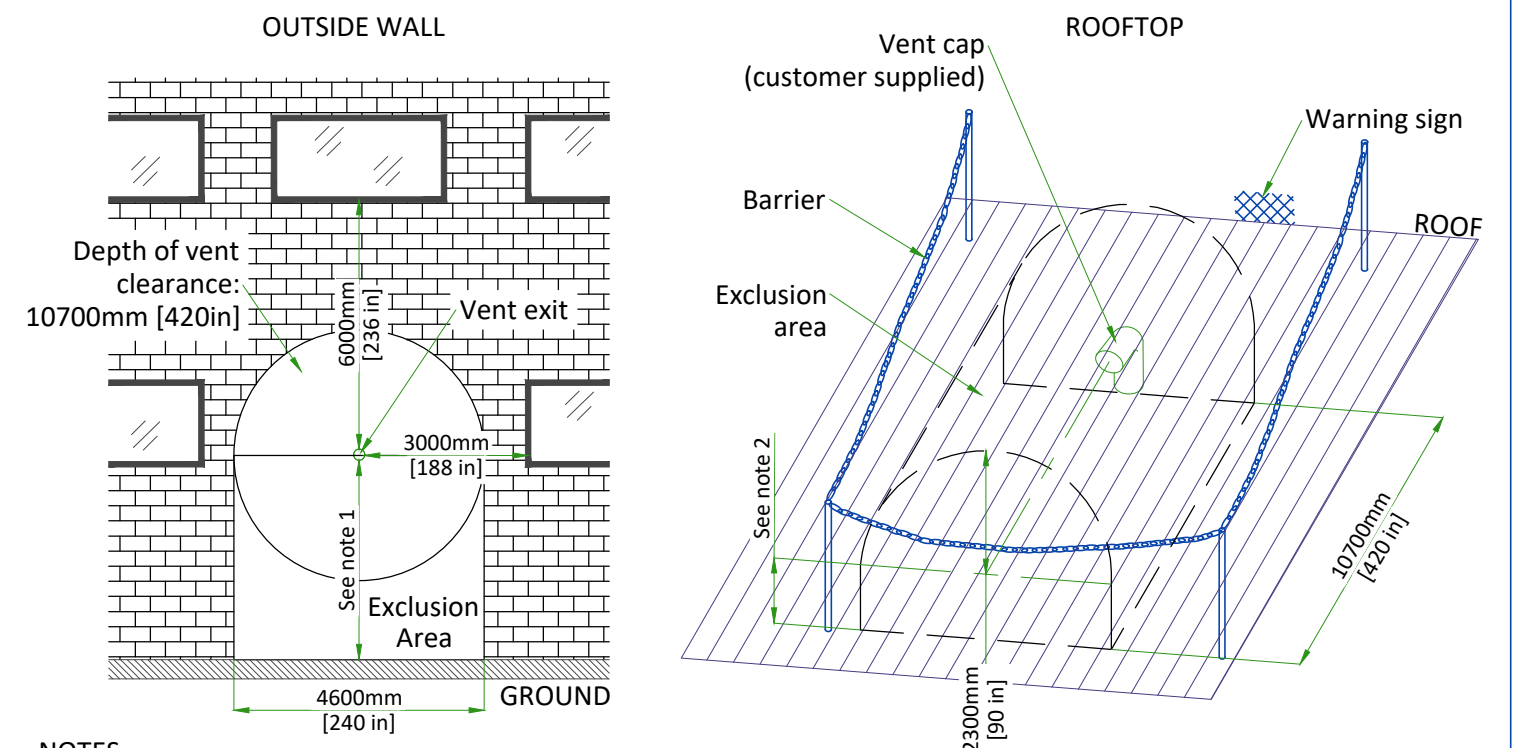

### NOTES

- Restricted area: minimum distance between vent pipe and ground is 3660mm [144 in]. Barriers are required. Public area: barriers are not required if height is > 5000mm [197 in].
- The bottom of the 90° elbow must be at least 914mm [36 in] above the roof deck (or higher if at risk of being blocked by drifting snow, sand, or other potential obstructions.)

NOT TO SCALE

MAGNET CRYOGENIC VENT SYSTEM PRESSURE DROP MATRIX

| Outer dia. of pipe (D) | Distance of vent system component from magnet |             | Pressure drop for straight pipe |       | Std sweep 45°elbow |       | Long sweep 45° elbow |       | Std sweep 90° elbow |       | Long sweep 90° elbow |       | 90° miter bend |        |
|------------------------|---|-------------|---------------------------------|-------|--------------------|-------|----------------------|-------|---------------------|-------|----------------------|-------|----------------|--------|
|                        | ft  | m           | psi/ft                          | kPa/m | psi                | kPa   | psi                  | kPa   | psi                 | kPa   | psi                  | kPa   | psi            | kPa    |
| 8 in. (200mm)          | 0- 10   | 0.00-3.05   | 0.14                            | 3.22  | 1.12               | 7.70  | 0.74                 | 5.13  | 2.09                | 14.43 | 1.40                 | 9.62  | 4.19           | 28.86  |
|                        | 10-20   | 3.05-6.10   | 0.24                            | 5.49  | 1.83               | 12.63 | 1.22                 | 8.42  | 3.43                | 23.67 | 2.29                 | 15.78 | 6.87           | 47.34  |
|                        | 20- 30  | 6.10-9.15   | 0.36                            | 8.23  | 2.49               | 17.20 | 1.66                 | 11.45 | 4.67                | 32.21 | 3.11                 | 21.48 | 9.34           | 64.43  |
|                        | 30- 40  | 9.15-12.2   | 0.47                            | 10.65 | 3.11               | 21.42 | 2.07                 | 14.26 | 5.82                | 40.11 | 3.88                 | 26.74 | 11.64          | 80.23  |
|                        | 40- 50  | 12.20-15.25 | 0.57                            | 12.80 | 3.67               | 25.32 | 2.45                 | 16.86 | 6.88                | 47.42 | 4.58                 | 31.61 | 13.75          | 94.84  |
|                        | 50- 60  | 15.25-18.30 | 0.65                            | 14.68 | 4.20               | 28.93 | 2.79                 | 19.26 | 7.86                | 54.17 | 5.24                 | 36.11 | 15.71          | 108.33 |
|                        | 60- 80  | 18.29-24.39 | 0.77                            | 17.44 | 5.13               | 35.35 | 3.41                 | 23.53 | 9.60                | 66.16 | 6.40                 | 44.11 | 19.19          | 132.33 |
| 10 in. (250mm)         | 0-20  | 0.00-6.10   | 0.06                            | 1.280 | 0.62               | 4.29  | 0.41                 | 2.86  | 1.17                | 8.04  | 0.78                 | 5.36  | 2.33           | 16.07  |
|                        | 20-40   | 6.10-12.22  | 0.12                            | 2.725 | 1.05               | 7.25  | 0.70                 | 4.83  | 1.97                | 13.58 | 1.31                 | 9.05  | 3.94           | 27.16  |
|                        | 40-60   | 12.22-18.29 | 0.17                            | 3.904 | 1.43               | 9.86  | 0.95                 | 6.56  | 2.67                | 18.44 | 1.78                 | 12.29 | 5.35           | 36.88  |
|                        | 60-80   | 18.29-24.39 | 0.21                            | 4.859 | 1.76               | 12.14 | 1.17                 | 8.07  | 3.29                | 22.70 | 2.19                 | 15.13 | 6.58           | 45.40  |
|                        | 80-100  | 24.39-30.49 | 0.25                            | 5.626 | 2.05               | 14.14 | 1.36                 | 9.40  | 3.83                | 26.43 | 2.56                 | 17.62 | 7.67           | 52.86  |
| 12 in. (300mm)         | 0-20  | 0.00-6.10   | 0.020                           | 0.441 | 0.26               | 1.78  | 0.17                 | 1.19  | 0.48                | 3.34  | 0.32                 | 2.22  | 0.97           | 6.67   |
|                        | 20-40   | 6.10-12.22  | 0.041                           | 0.937 | 0.43               | 3.00  | 0.29                 | 1.99  | 0.81                | 5.61  | 0.54                 | 3.74  | 1.63           | 11.22  |
|                        | 40-60   | 12.22-18.29 | 0.060                           | 1.353 | 0.59               | 4.08  | 0.39                 | 2.72  | 1.11                | 7.64  | 0.74                 | 5.09  | 2.22           | 15.27  |
|                        | 60-80   | 18.29-24.39 | 0.075                           | 1.702 | 0.73               | 5.06  | 0.49                 | 3.36  | 1.37                | 9.45  | 0.91                 | 6.30  | 2.74           | 18.89  |
|                        | 80-100  | 24.39-30.49 | 0.088                           | 1.991 | 0.86               | 5.20  | 0.57                 | 3.93  | 1.60                | 11.06 | 1.07                 | 7.37  | 3.21           | 22.12  |
| 14 in. (350mm)         | 0-20  | 0.00-6.10   | 0.008                           | 0.180 | 0.123              | 0.85  | 0.082                | 0.57  | 0.231               | 1.59  | 0.154                | 1.06  | 0.462          | 3.18   |
|                        | 20-40   | 6.10-12.22  | 0.017                           | 0.380 | 0.206              | 1.42  | 0.137                | 0.95  | 0.386               | 2.66  | 0.257                | 1.77  | 0.771          | 5.32   |
|                        | 40-60   | 12.22-18.29 | 0.024                           | 0.552 | 0.281              | 1.94  | 0.187                | 1.29  | 0.525               | 3.62  | 0.350                | 2.42  | 1.051          | 7.25   |
|                        | 60-80   | 18.29-24.39 | 0.031                           | 0.699 | 0.349              | 2.41  | 0.232                | 1.60  | 0.652               | 4.50  | 0.435                | 3.00  | 1.304          | 8.99   |
|                        | 80-100  | 24.39-30.49 | 0.036                           | 0.824 | 0.411              | 2.83  | 0.272                | 1.88  | 0.766               | 5.28  | 0.511                | 3.52  | 1.533          | 10.57  |
| 16 in. (400mm)         | 0-20  | 0.00-6.10   | 0.004                           | 0.083 | 0.065              | 0.45  | 0.043                | 0.30  | 0.122               | 0.84  | 0.081                | 0.56  | 0.244          | 1.68   |
|                        | 20-40   | 6.10-12.22  | 0.008                           | 0.174 | 0.108              | 0.75  | 0.072                | 0.50  | 0.202               | 1.39  | 0.135                | 0.93  | 0.404          | 2.79   |
|                        | 40-60   | 12.22-18.29 | 0.011                           | 0.253 | 0.148              | 1.02  | 0.098                | 0.68  | 0.275               | 1.90  | 0.184                | 1.27  | 0.551          | 3.80   |
|                        | 60-80   | 18.29-24.39 | 0.014                           | 0.323 | 0.184              | 1.27  | 0.122                | 0.84  | 0.342               | 2.36  | 0.228                | 1.57  | 0.685          | 4.72   |
|                        | 80-100  | 24.39-30.49 | 0.017                           | 0.383 | 0.022              | 1.49  | 0.144                | 0.99  | 0.404               | 2.78  | 0.269                | 1.86  | 0.807          | 5.57   |

- Notes
- 1. Elbows with angles greater than 90 deg must not be used
  - 2. Data in Table 2 is based on the following facts and assumptions:
    - a. Initial flow conditions at magnet interface
    - b. EM energy (13MJ) is dumped to He during quench and rises He temperature to 10 Kelvin
    - c. Gas temperature starting at 10 Kelvin and increase with length determined by thermal energy balance
    - d. 90% He is assumed to be evacuated within 30 sec. None left after quench.
    - e. Absolute roughness is assumed to be 0.25 mm.
    - f. R/D = 1.0 for standard sweep elbows, R/D = 1.5 for long sweep elbows where D = outer diameter of pipe; R = radius of bend
  - 3. The total pressure drop of the entire cryogenic vent system must be less than 20 psi (138 kPa). The calculation starts at the magnet vent interface and ends at the termination point outside the building.

LIGHTING REQUIREMENTS

- All lighting fixtures and associated components must meet all RF shielded room and RF grounding requirements (e.g., track lighting is not recommended due to possible RF noise).
- All removable lighting fixtures and associated components must be non-magnetic.
- All lighting must use direct current (the DC must have less than 5% ripple).
- 300 lux must be provided at the front of the magnet for patient access and above the magnet for servicing.
- Fluorescent lighting must not be used in the magnet room.
- Lighting must be adjusted using a discrete switch or a variable DC lighting controller.
- SCR dimmers or rheostats must not be used.
- DC LED lighting may be used if the DC power converter and RF sources are all located outside the magnet room RF shield.

NOTE: LED lighting could cause image quality issues due to RF interference. Make sure a MR-compatible LED lighting solution is chosen.

- Battery chargers (e.g., used for emergency lighting) must be located outside the magnet room.
- Short filament length bulbs are recommended.
- Linear lamps are not recommended due to the high burnout rate.

CONNECTIVITY REQUIREMENTS

Broadband Connections are necessary during the installation process and going forward to ensure full support from the Engineering Teams for the customers system. Maximum performance and availability for the customers system is maintained and closely monitored during the lifetime of the system. Proactive and reactive maintenance is available utilising the wide range of digital tools using the connectivity solutions listed below:


- Site-to-Site VPN/GE Solution
- Site-to-Site VPN/Customer Solution
- Connection through Dedicated Service Network
- Internet Access - connectivity for InSite 2.0

The requirements for these connectivity solutions are explained in the broadband solutions catalogue (separate document).

ELECTRICAL NOTES

1. All wires specified shall be copper stranded, flexible, thermo-plastic, color coded, cut 10 foot long at outlet boxes, duct termination points or stubbed conduit ends. All conductors, power, signal and ground, must be run in a conduit or duct system. Electrical contractor shall ring out and tag all wires at both ends. Wire runs must be continuous copper stranded and free from splices.
  - 1.1. Aluminum or solid wires are not allowed.
2. Wire sizes given are for use of equipment. Larger sizes may be required by local codes.
3. It is recommended that all wires be color coded, as required in accordance with national and local electrical codes.
4. Conduit sizes shall be verified by the architect, electrical engineer or contractor, in accordance with local or national codes.
5. Convenience outlets are not illustrated. Their number and location are to be specified by others. Locate at least one convenience outlet close to the system control, the power distribution unit and one on each wall of the procedure room. Use hospital approved outlet or equivalent.
6. General room illumination is not illustrated. Caution should be taken to avoid excessive heat from overhead spotlights. Damage can occur to ceiling mounting components and wiring if high wattage bulbs are used. Recommend low wattage bulbs no higher than 75 watts and use dimmer controls (except MR). Do not mount lights directly above areas where ceiling mounted accessories will be parked.
7. Routing of cable ductwork, conduits, etc., must run direct as possible otherwise may result in the need for greater than standard cable lengths (refer to the interconnection diagram for maximum usable lengths point to point).
8. Conduit turns to have large, sweeping bends with minimum radius in accordance with national and local electrical codes.
9. A special grounding system is required in all procedure rooms by some national and local codes. It is recommended in areas where patients might be examined or treated under present, future, or emergency conditions. Consult the governing electrical code and confer with appropriate customer administrative personnel to determine the areas requiring this type of grounding system.
10. The maximum point to point distances illustrated on this drawing must not be exceeded.
11. Physical connection of primary power to GE equipment is to be made by customers electrical contractor with the supervision of a GE representative. The GE representative would be required to identify the physical connection location, and insure proper handling of GE equipment.
12. GEHC conducts power audits to verify quality of power being delivered to the system. The customer's electrical contractor is required to be available to support this activity.

- All junction boxes, conduit, duct, duct dividers, switches, circuit breakers, cable tray, etc., are to be supplied and installed by customers electrical contractor.
- Conduit and duct runs shall have sweep radius bends
- Conduits and duct above ceiling or below finished floor must be installed as near to ceiling or floor as possible to reduce run length.
- Ceiling mounted junction boxes illustrated on this plan must be installed flush with finished ceiling.
- All ductwork must meet the following requirements:
  - 1.Ductwork shall be metal with dividers and have removable, accessible covers.
  - 2.Ductwork shall be certified/rated for electrical power purposes.
  - 3.Ductwork shall be electrically and mechanically bonded together in an approved manner.
  - 4.PVC as a substitute must be used in accordance with all local and national codes.
- All openings in raceway and access flooring are to be cut out and finished off with grommet material by the customers contractor.
- General contractor to insert pull cords for all cable run conduits between the equipment room and the operators control room.
- 10 foot pigtails at all junction points.
- Grounding is critical to equipment function and patient safety. Site must conform to wiring specifications shown on this plan.

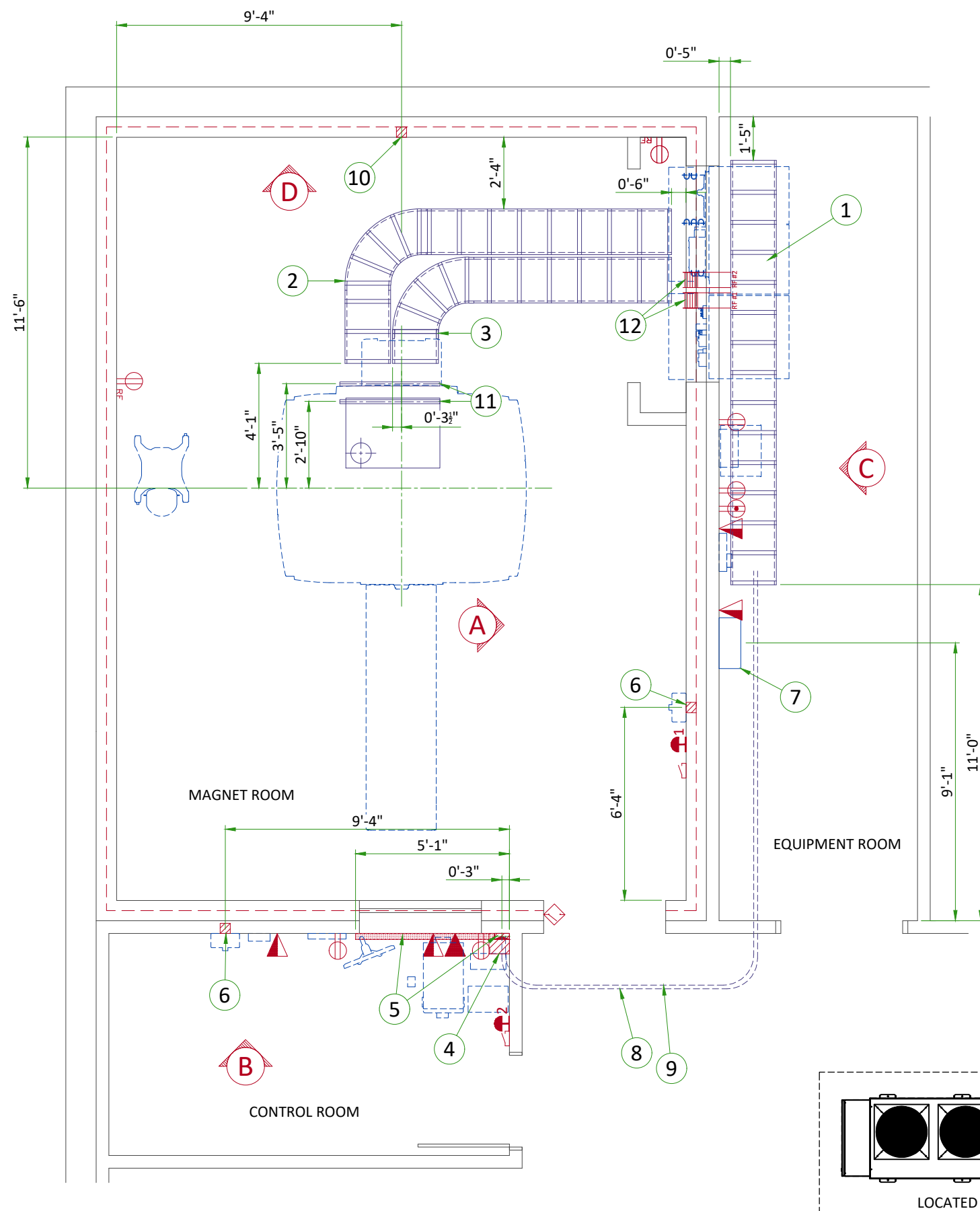


| ITEM | Electrical Layout Item List                                       |
|------|---|
| 1    | Cable ladder 18" x 6" [450 x 150]                                 |
| 2    | Non-ferrous cable ladder 18" x 6" [450 x 150]                     |
| 3    | Non-ferrous cable ladder for gradient cables 18" x 6" [450 x 150] |
| 4    | Box above ceiling size per local code                             |
| 5    | Surface wall duct with minimum 2 dividers 6' x 3 1/2" [150 x 100] |
| 6    | Box in wall 4" x 4" x 2" [100 x 100 x 50]                         |
| 7    | Main disconnect panel   |
| 8    | Conduit above RF screen 2" [50]                                   |
| 9    | Conduit above RF screen 3" [75]                                   |
| 10   | Box in wall 4" x 4" x 4" [100 x 100 x 100]                        |
| 11   | Non-ferrous unistrut cable support                                |
| 12   | RF Filters - grounded to RF shield at Common Ground Stud          |


| ITEM | QTY | Outlet Legend for GE Equipment  |
|------|-----|---|
|      |     | System emergency off (EO1-2), (recommended height 1.2m [48"] above floor)                   |
|      |     | Door interlock switch   |
|      |     | Emergency exhaust fan switch 1.2m [48"] height recommended)                                 |
|      |     | Duplex hospital grade, dedicated wall outlet 120-v, single phase power                      |
|      |     | Network outlet  |
|      |     | Dedicated telephone lines/network connection  |
|      |     | Duplex hospital grade, dedicated outlet 120-v emergency, single phase power, 15a            |
|      |     | Duplex hospital grade, dedicated outlet 120-v, single phase outlet routed through RF filter |

| Additional Conduit Runs<br>(Contractor Supplied and Installed) |                                  |                            |      |          |          |
|--|----------------------------------|----------------------------|------|----------|----------|
| From<br>(Bubble # / Item)                                      | To<br>(Bubble # / Item)          | Qty                        | Size |          |          |
|  |                                  |                            | In.  | mm       |          |
| 7  | Main Disconnect Panel            | Facility Power             | 1    | As req'd | As req'd |
| 7  | Main Disconnect Panel            | Integrated Systems Cabinet | 1    | As req'd | As req'd |
|  |                                  | Integrated Cooling Cabinet | 1    | As req'd | As req'd |
|  |                                  | System emergency off 2     | 1    | 1/2      | 16       |
|  | System emergency off 2           | Integrated Cooling Cabinet | 1    | 1/2      | 16       |
|  | Door switch                      | Integrated Systems Cabinet | 1    | 3/4      | 20       |
|  | System emergency off 1           | Integrated Cooling Cabinet | 1    | 3/4      | 20       |
| 6  | Magnet Rundown Unit              | Magnet                     | 1    | 1        | 25       |
|  |                                  | RF filter #1               | 1    | As req'd | As req'd |
|  | RF filter #1                     | 120-V 1Ø Power             | 1    | As req'd | As req'd |
|  | Room Light                       | RF filter #2               | 1    | As req'd | As req'd |
|  | RF filter #2                     | Facility emergency power   | 1    | As req'd | As req'd |
| 10   | TV Camera                        | Waveguide or RF filter     | 1    | 1        | 25       |
| 4  | TV Monitor                       |                            | 1    | 1        | 25       |
|  | Injector control unit            | Waveguide or RF filter     | 1    | 2 1/2    | 70       |
|  | Injector head                    |                            | 1    | As req'd | As req'd |
|  | Integrated battery charging unit |                            | 1    | As req'd | As req'd |
|  | Dimplex Chiller                  | Remote Graphic Display     | 1    | 3/4      | 20       |
|  |                                  | Facility Power             | 1    | as Req'd |          |




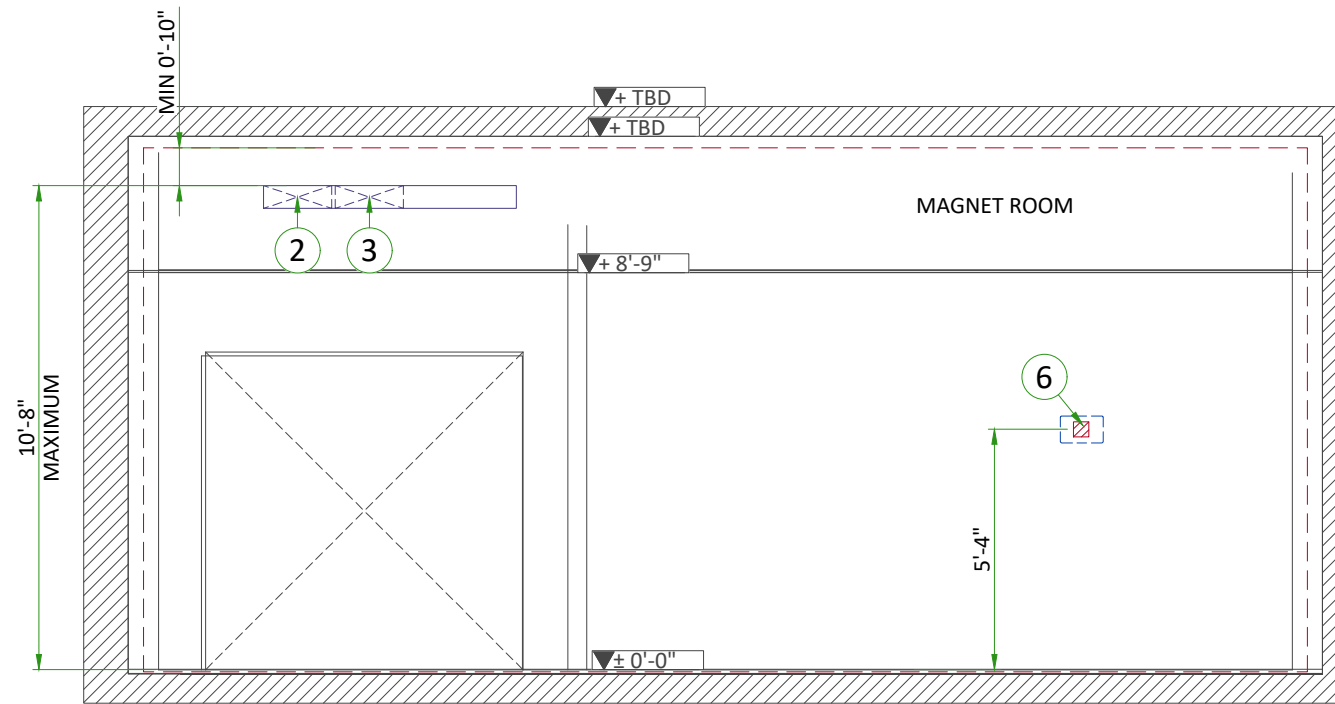


A

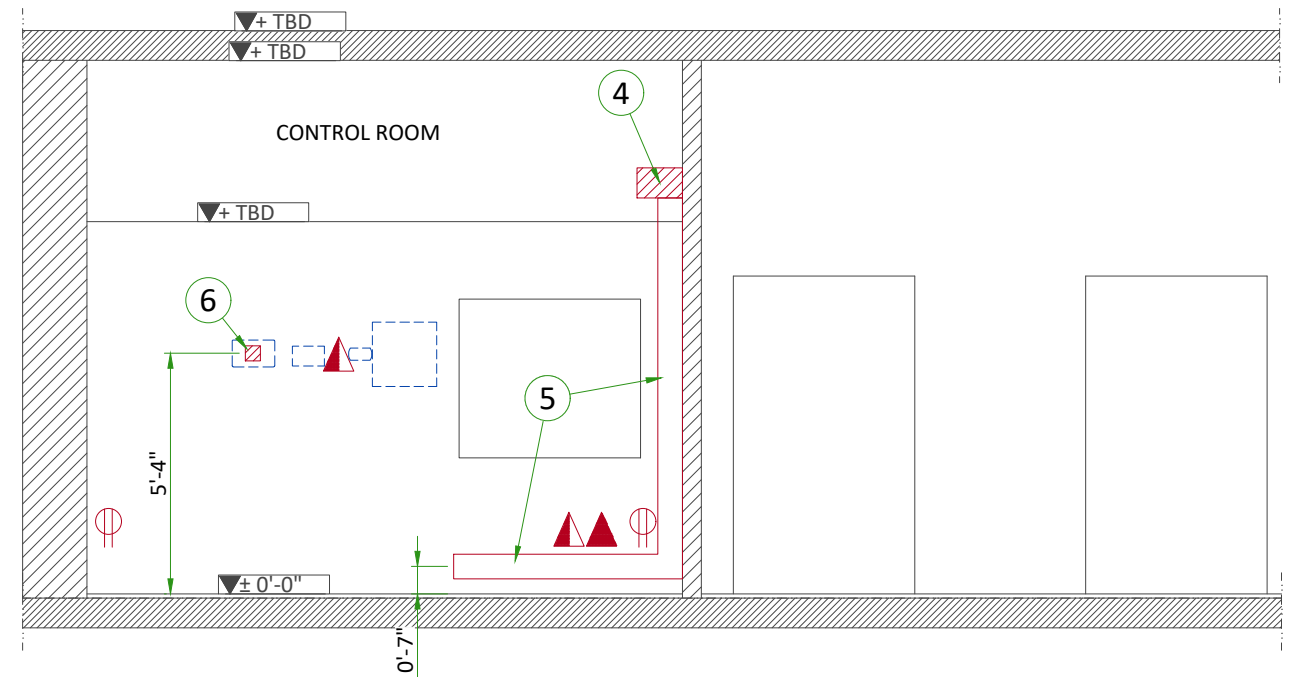

B

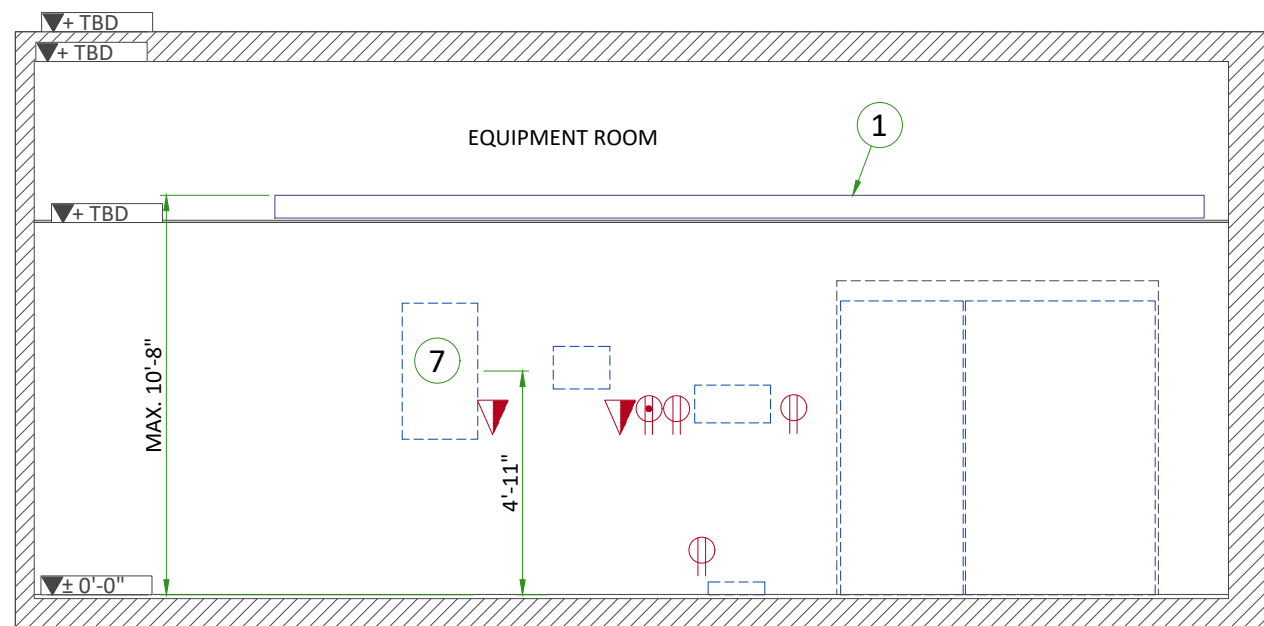


C

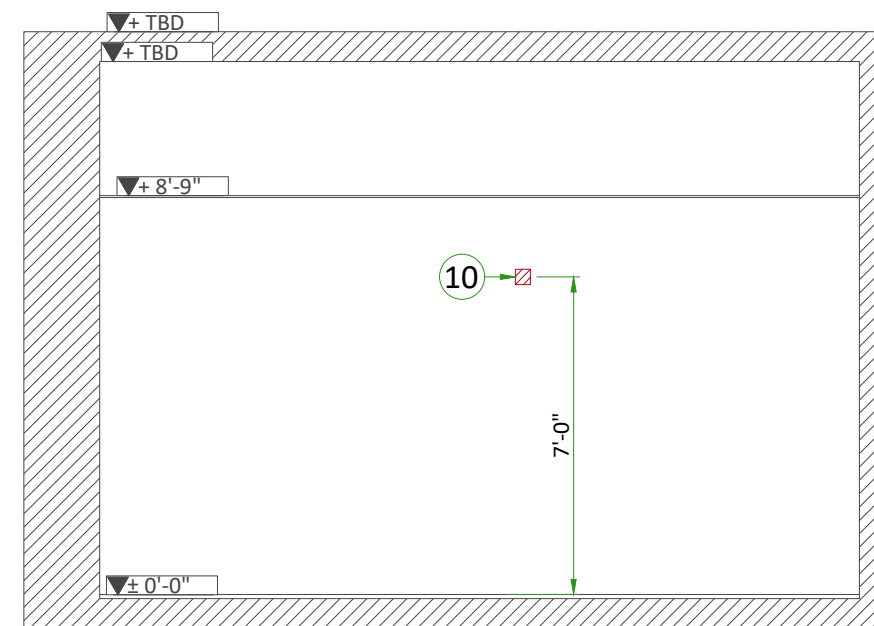


D

CABLE TRAYS IN EQUIPMENT ROOM


CABLE TRAY DETAIL

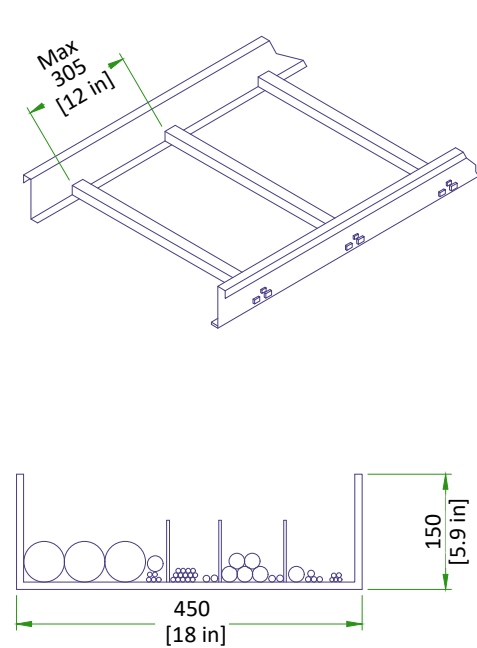

NOT TO SCALE

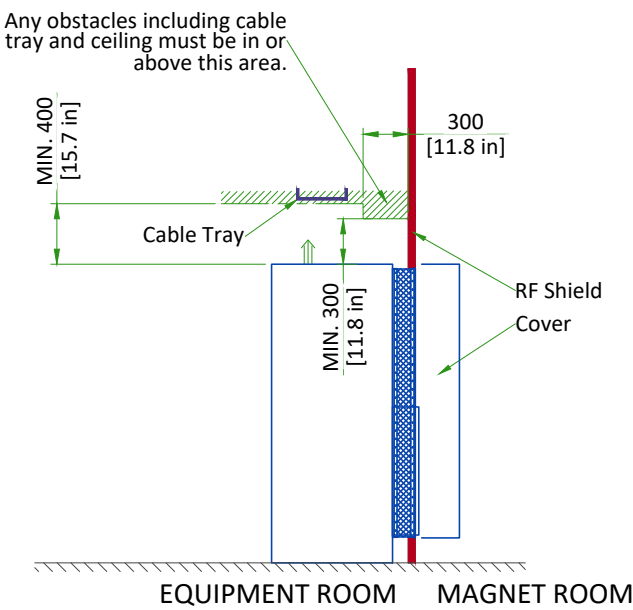

CABLE TRAY REQUIREMENTS IN MAGNET ROOM

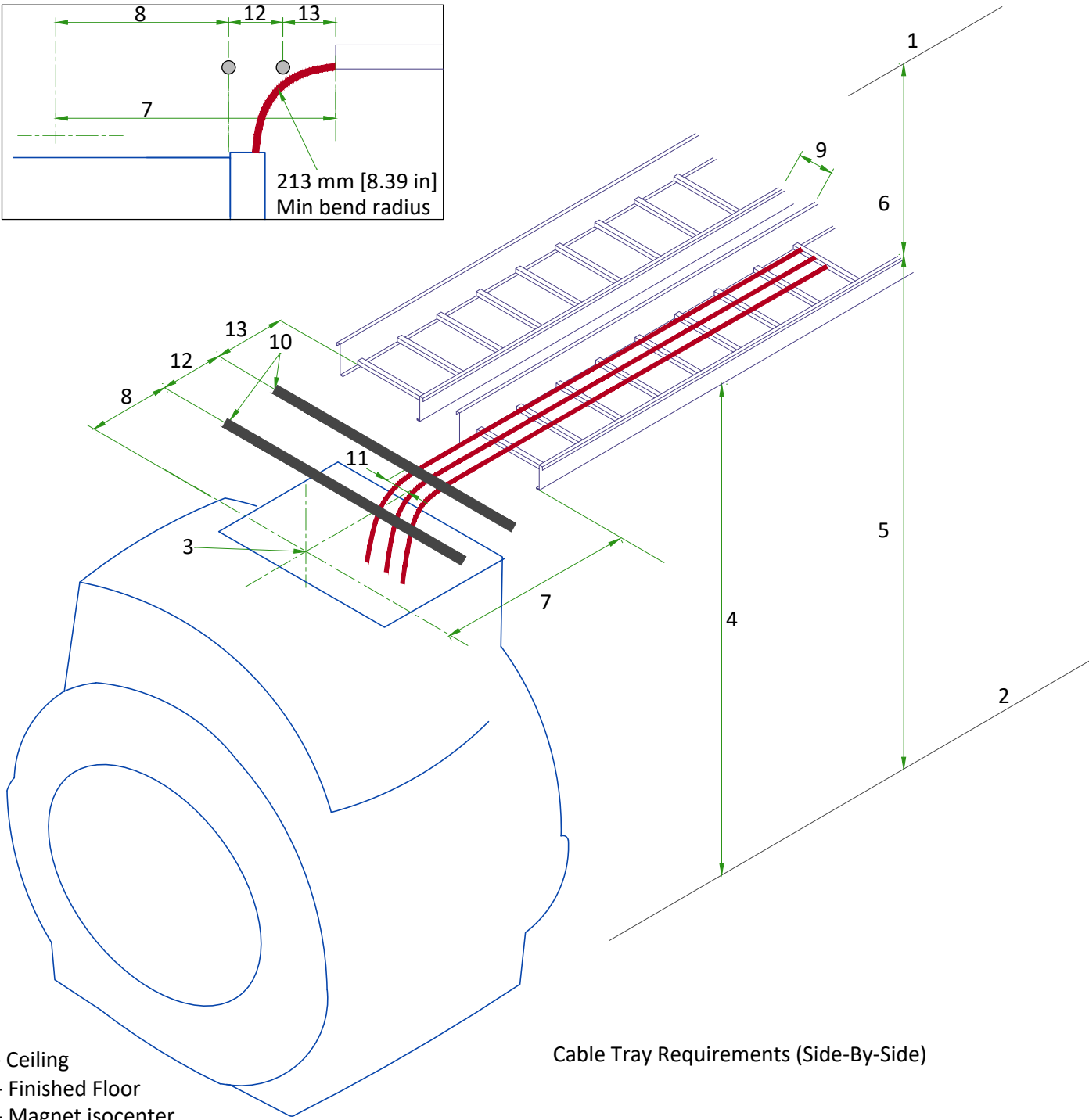

POWER REQUIREMENTS

SPECIFICATIONS OF MAIN POWER INPUT

|                                 |  |
|---------------------------------|--|
| POWER SUPPLY                    | 380/400/415/480V +7.5%/-10%, 3 PHASE + GND |
| FREQUENCIES                     | 50/60Hz ± 3Hz                              |
| POWER FACTOR                    | 0.9  |
| MAXIMUM INPUT POWER (5 sec MAX) | 77kVA                                      |
| CONTINUOUS (AVERAGE) POWER      | 64kVA                                      |
| STAND-BY POWER                  | < 17kVA                                    |

- Power input must be separated from any others which may generate transients (elevators, air conditioning, radiology rooms equipped with high speed film changers...).
- If a neutral conductor is present, it can be terminated on the neutral bus provided in the GE supplied MDP.
- Total harmonic distortion less than 2.5%. Phase imbalance must not exceed 2%.
- Lock-out/Tag-out: The Main Disconnect Panel (MDP) shall provide an external single point lock-out/tag-out feature for the entire system and a means to externally lock-out/tag-out each output breaker independently. Each lock-out/tag-out feature shall accommodate a standard sized lock hasp.

SPECIFICATIONS OF BACK-UP POWER SUPPLY

|  |
|--|
| MAGNET MONITOR REQUIRES A 110/220 VAC, 50/60 HZ, 3.0 A FACILITY SUPPLIED OUTLET. POWER AT THE OUTLET MUST BE CONTINUOUSLY AVAILABLE. |
|--|

|  |  |
|--|--|
| FOR CRYOCOOLER COMPRESSOR  |  |
| POWER INPUT  | 380/400/415/480V, THREE-PHASE + G  |
| POWER REQUIREMENT  | MIN 9kVA   |
| POWER CONSUMPTION  | MAX 7.2kW / STEADY STATE 6.5kW at 50Hz<br>MAX 8.3kW / STEADY STATE 7.5kW at 60Hz |
| FREQUENCY  | 50/60Hz ± 3Hz  |
| Power to Cryocooler Compressor must be removed when emergency off circuit is actuated. |  |

CABLES


- Power and cable installation must comply with the distribution diagram.
- Size of the Main power input cable is determined by the customer, taking its length and admissible voltage drops into consideration.
- All cables must be isolated and flexible, cable color codes must comply with standards for electrical installation.
- The cables from signaling and remote control (Y, Emergency Off Buttons, L...) will go to Main Panel with a pigtail length of 1.5m [60in], and will be connected during installation.
- Each conductor will be identified and isolated (screw connector).

GROUND SYSTEM

- The equipotential link will be by means of an equipotential bar.
- The grounding point of MDP is directly connected to the building's ground by an isolated copper cable.
- The impedance of the earth bar should be less than or equal to 2 ohms.

| GUIDANCE ON SLECTION OF FEEDER AND TRANSFORMER FOR MR SYSTEM   |   |  |
|--|---|--|
|  | Direct feed from facility to MR system    | MR system fed by dedicated facility distribution transformer   |
| Prerequisite Conditions  |   |  |
| MR System Incoming Voltage   | 480V 3-phase                              |  |
| Minimum Source short-circuit kVA   | 2,500 kVA<br>(at source of feeder to MDP) | 2,500 kVA<br>(at input to distribution transformer)  |
| Minimum No-Load Voltage  | 460V                                      | 475V (transformer secondary tapped accordingly)  |
| Feeder and Transformer Recommendations   |   |  |
| Dedicated Distribution Transformer Recommendations   | N/A                                       | Size: 112.5 kVA<br>Impedance (Z): ≤5%<br>K-Factor: ≥ K=20<br>125A overcurrent protection on secondary* |
| Maximum Feeder Length*   | 465 ft                                    | 465 ft   |
| Feeder Size - 3-phase power conductors*  | 1 AWG Cu                                  | 1 AWG Cu   |
| Feeder Size - Ground (USA)*  | 6 AWG Cu (equipment grounding conductor)  | 6 AWG Cu (supply side bonding jumper)<br>6 AWG Cu (equipment grounding conductor)                      |
| Feeder Size - Ground (Canada)*   | 6 AWG Cu (bonding conductor)              | 6 AWG Cu (bonding conductor)   |
| * NOTE: Recommendations shown apply only to cases defined exactly as shown in this table and when not in conflict with local electrical codes. For all other cases, refer to the local codes and the System Voltage Regulation Calculator located on the GE Healthcare Site Planning Website |   |  |

POWER DISTRIBUTION

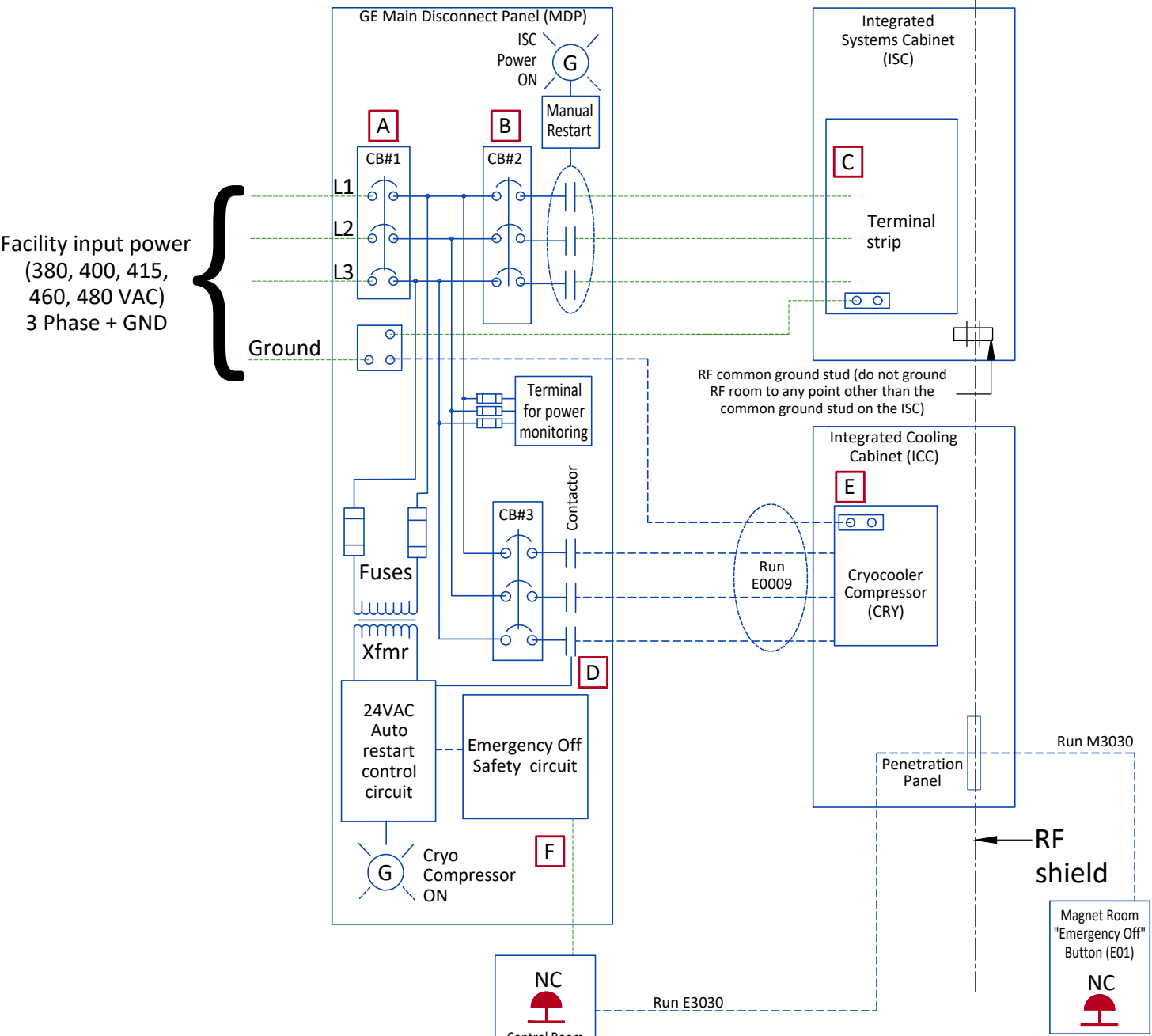

- NOTES:
- Cryocooler Compressor (CRY) must operate 24 hours per day, 7 days per week.
  - Runs E0009, E3030 and M3030 are GE supplied cables. All other wiring is customer supplied and installed.
  - Two remote Emergency Off Maintained Buttons are supplied with the MDP. Emergency Off removes power from all outputs when activated.
  - All MDP output circuits drop out on loss of power. the Cryocooler (CRY) circuit will automatically restart upon restoration of power.
  - MDP Short circuit current rating is 25,000 amperes at 480 VAC.
  - MDP is NRTL labeled.
  - All feeder circuits require dedicated ground wires.


| CB | MDP      |
|----|----------|
| 1  | 125 AMPS |
| 2  | 110 AMPS |
| 3  | 25 AMPS  |

|   |
|---|
| --- Cable SUPPLIED by CUSTOMER                |
| --- Cable SUPPLIED by GE - Customer Installed |
| --- Equipment SUPPLIED by CUSTOMER            |
| --- Equipment SUPPLIED by GE                  |

Accepts following range of standard stranded conductors. All wire types, color and sizing to be selected in accordance with governing electrical code(s).

| GE MDP M70022MC (380V-480V) |           |           |           |          |
|-----------------------------|-----------|-----------|-----------|----------|
| Item                        | Phase     |           | Ground    |          |
|                             | AWG/kcmil | sq mm     | AWG/kcmil | sq mm    |
| A                           | 2 - 3/0   | 35 - 95   | 2 - 250   | 35 - 125 |
| B                           | 8 - 3/0   | 10 - 95   | 2 - 250   | 35 - 125 |
| C                           | 10 - 1    | 6 - 35    | 10 - 1    | 6 - 35   |
| D                           | 14 - 8    | 2.5 - 10  | 6 - 4     | 13 - 21  |
| E                           | 14 - 10   | 2.5 - 6   | 14 - 10   | 2.5 - 6  |
| F                           | 22 - 12   | 0.5 - 2.5 |           |          |

INTERCONNECTIONS

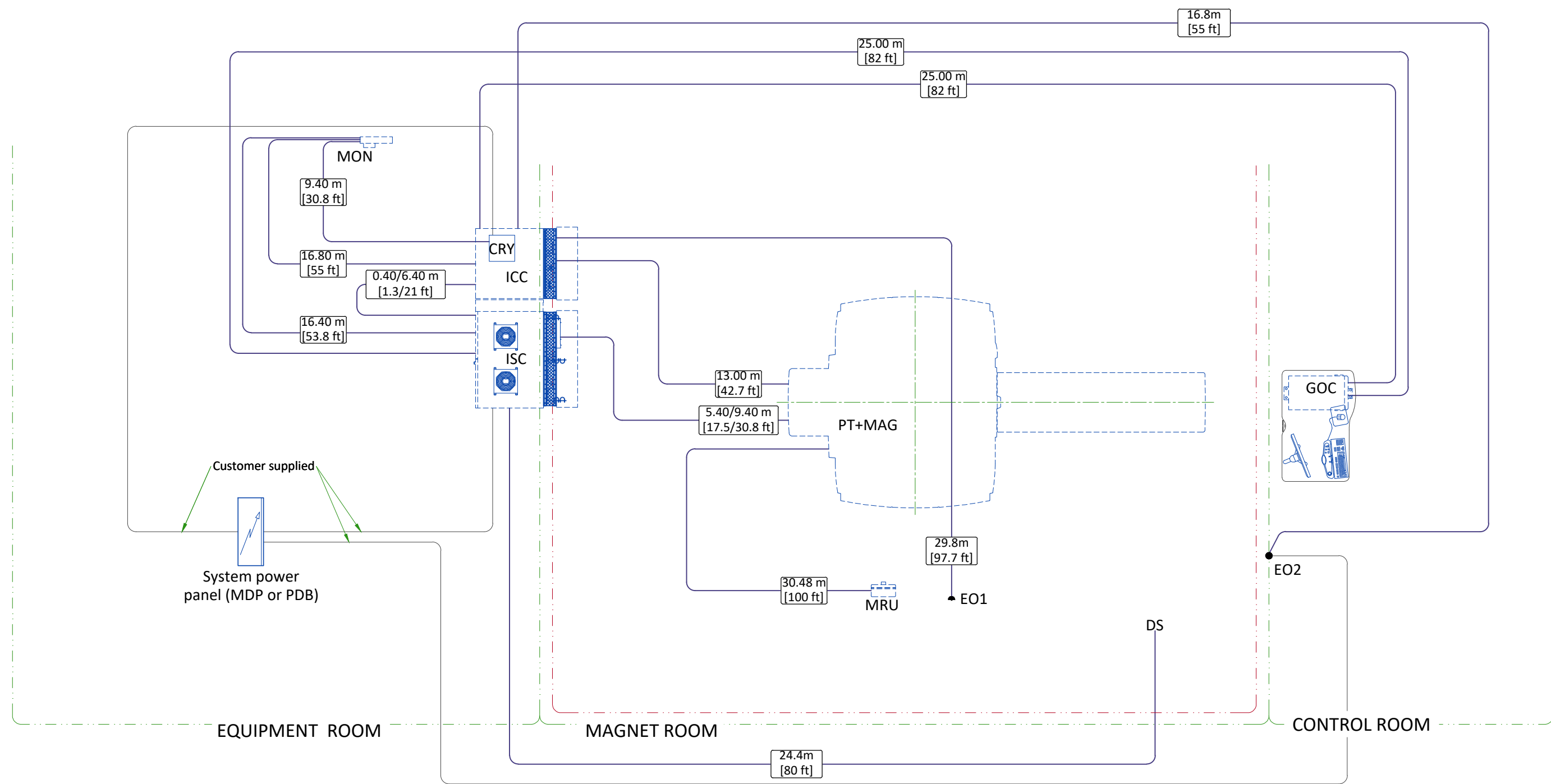

Long / Short cable selection guidance

NOTE: This guidance is based on the assumption that the distance between cable tray and floor is 2.6 m [8.53 ft].

Magnet Room:

- If the Magnet – ISC Distance is less than 5.4 m [17.7 ft], select short cable.
- If the Magnet – ISC Distance is in between 5.4 m [17.7 ft] and 9.4 m [30.8 ft] , select long cable.

Equipment Room:

- If the ICC – ISC Distance is less than 0.4 m [1.3 ft], select short cable.
- If the ICC – ISC Distance is in between 0.4 m [1.3 ft] and 6.4m [21 ft] , select long cable.

CABLES ROUTING FOR OPTIONS

| OPTION     | FROM | TO                       | CABLE LENGTH |       |     |
|------------|------|--------------------------|--------------|-------|-----|
|            |      |                          |              | m     | ft  |
| MRE        | MRE  | Magnet Isocenter         | Nominal      | 7.31  | 24  |
|            |      |                          | Maximum      | 10.06 | 33  |
|            | MRE  | ISC cabinet              |              | 15.24 | 50  |
|            | MRE  | Ethernet Hub in ISC      |              | 15.24 | 50  |
|            | MRE  | Customer Supplied Outlet | 60 Hz        | 6.09  | 20  |
|            |      |                          | 50 Hz        | 7.62  | 25  |
| Remote MRU | ICC  | Remote MRU               | Short        | 1.83  | 6   |
|            |      |                          | Long         | 30.48 | 100 |