




			<div>Typical</div> <div>----</div> <div>----</div>										
REV	DATE	MODIFICATIONS											
C1 - 01 - Cover Sheet C2 - 02 - Disclaimer - Site Readiness A1 - 03 - General Notes A2 - 04 - Equipment Layout A3 - 05 - Radiation Protection A4 - 06 - Equipment Dimensions A5 - 07 - Delivery S1 - 08 - Structural Notes S2 - 09 - Structural Layout		S3 - 10 - Structural Details (1) M1 - 11 - HVAC E1 - 12 - Electrical Notes E2 - 13 - Electrical Layout E3 - 14 - Electrical Elevation E4 - 15 - Power Requirements E5 - 16 - Electrical Details - Interconnections			<div> GE Healthcare</div> <div>----</div> <div>----</div> <div>----</div>								
<div>A mandatory component of this drawing set is the GE Healthcare Pre Installation manual. Failure to reference the Pre Installation manual will result in incomplete documentation required for site design and preparation.</div> <div>Pre Installation documents for GE Healthcare products can be accessed on the web at: www.gehealthcare.com/siteplanning</div>					OPTIMA NM/CT 640 FINAL STUDY								
					Drawn by		Verified by		Concession		S.O. (GON)		PIM Manual
		PMM		CPC		-		-		5426783-1EN		7	
<div>GE does not take responsibility for any damages resulting from changes on drawings made by others. Errors may occur by not referring to the complete set of final issue drawing. GE cannot accept responsibility for any damage due to the partial use of GE final issue drawings, however caused. All dimensions are in millimeters unless otherwise specified. Do not scale from printed pdf files. GE accepts no responsibility or liability for defective work due to scaling from these drawings.</div>		Format		Scale		File Name				Date		Sheet	
		A3		1/4"=1'-0"		EN-NUC-TYP-OPTIMA_NM_640-WEB.DWG				28/Sep/2021		01/16	

DISCLAIMER

GENERAL SPECIFICATIONS

- GE is not responsible for the installation of developers and associated equipment, lighting, cassette trays and protective screens or derivatives not mentioned in the order.
- The final study contains recommendations for the location of GE equipment and associated devices, electrical wiring and room arrangements. When preparing the study, every effort has been made to consider every aspect of the actual equipment expected to be installed.
- The layout of the equipment offered by GE, the dimensions given for the premises, the details provided for the pre-installation work and electrical power supply are given according to the information noted during on-site study and the wishes expressed by the customer.
- The room dimensions used to create the equipment layout may originate from a previous layout and may not be accurate as they may not have been verified on site. GE cannot take any responsibility for errors due to lack of information.
- Dimensions apply to finished surfaces of the room.
- Actual configuration may differ from options presented in some typical views or tables.
- If this set of final drawings has been approved by the customer, any subsequent modification of the site must be subject to further investigation by GE about the feasibility of installing the equipment. Any reservations must be noted.
- The equipment layout indicates the placement and interconnection of the indicated equipment components. There may be local requirements that could impact the placement of these components. It remains the customer's responsibility to ensure that the site and final equipment placement complies with all applicable local requirements.
- All work required to install GE equipment must be carried out in compliance with the building regulations and the safety standards of legal force in the country concerned.
- These drawings are not to be used for actual construction purposes. The company cannot take responsibility for any damage resulting therefrom.

CUSTOMER RESPONSIBILITIES

- It is the responsibility of the customer to prepare the site in accordance with the specifications stated in the final study. A detailed site readiness checklist is provided by GE. It is the responsibility of the customer to ensure all requirements are fulfilled and that the site conforms to all specifications defined in the checklist and final study. The GE Project Manager of Installation (PMI) will work in cooperation with the customer to follow up and ensure that actions in the checklist are complete, and if necessary, will aid in the rescheduling of the delivery and installation date.
- Prior to installation, a structural engineer of record must ensure that the floor and ceiling is designed in such a way that the loads of the installed system can be securely borne and transferred. The layout of additional structural elements, dimensioning and the selection of appropriate installation methods are the sole responsibility of the structural engineer. Execution of load bearing structures supporting equipment on the ceiling, floor or walls are the customer's responsibility.

RADIO-PROTECTION

- Suitable radiological protection must be determined by a qualified radiological physicist in conformation with local regulations. GE does not take responsibility for the specification or provision of radio-protection.

THE UNDERSIGNED, HEREBY CERTIFIES THAT I HAVE READ AND APPROVED THE PLANS IN THIS DOCUMENT.		
DATE	NAME	SIGNATURE


GLOBAL SITE READINESS CHECKLIST (DI)

DOC1809666 Rev. 7

Site Ready Checks at Installation
EHS Site Requirements
Overall access route to the scan room free from obstruction / high hazards.
Enough space to store tools, equipment, parts, install waste and the general area free from obstruction and trip hazards.
Enough necessary facilities for the GE employees available.
No 3rd parties working in the area that may affect the safety of the installation activity.
Area free from any chemical, gas, dust, welding fume exposure and has painting been completed and dry.
All emergency routes identified, signed and clear from obstruction.
Accessible single source lockable panel that LOTO can be applied to for GE equipment installation (MDP and/or PDU).
There are no other conditions or hazards that you have observed or have been made aware of by the customer or contractors on site.
Required for Mechanical Install start
Room dimensions, including ceiling height, for all Exam, Equipment/Technical & Control rooms meets GE specifications.
Ceiling support structure, if indicated on the GE drawing, is in the correct location and at the correct height according to the Original Equipment Manufacturer specifications.
Levelness and spacing has been measured, and is ready for the installation of any GE supplied components.
Overhead support Structure (unistrut) has been confirmed with customer/contractor to meet required GE provided criteria.
Finished ceiling is installed. If applicable ceiling tiles installed per PMI discretion.
Floor levelness/flatness is measured and within tolerance, and there are no visible defects per GEHC specifications.
Entry door threshold meets PIM requirement
Floor Strength and thickness have been discussed with customer/contractor and they have confirmed GE requirements are met.
Rooms that will contain equipment, including staging areas if applicable, are construction debris free. Precautions must be taken to prevent debris from entering rooms containing equipment.
Cable ways (floor/wall/ceiling/Access Flooring) are available for installation of GE cables are of correct length and diameter.
Cable ways routes per GE Final drawings and cable access openings areas installed at a time determined by GEHC PM. Surface floor duct can be installed at time of system installation.
Adequate room illumination installed and working.
Customer supplied countertops where GE equipment will be installed are in place.
Nuclear Medicine systems levelness measurement survey must be provided to GE prior the delivery.
Required for Calibration start
HVAC systems Installed, and the site meets minimum environmental operational system requirements.
System power & grounding (PDB/MDP) is available as per GE specifications.
System power & grounding (PDB/MDP) is installed at point of final connection and ready to use. Lock Out Tag Out is available.
PMI to confirm all feeder wires and breaker are size appropriately. EPO installed if needed.
PMI to confirm with electrician all power and signal cables are well terminated ensuring there are no loose connections.
Network outlets installed.
Computer network available and working.
Site has license for using/importing radioactive sources and a Hot Lab is available. Radioactive Sources should be available for system calibration during installation.
Lead doors and windows complete or scheduled to be installed. If applicable, radiation protection (shielding) finished & radioprotection regulatory approval for installation obtained.

Note: The details shown here are only an extract from DOC1809666. For the complete document please contact your PMI.


CUSTOMER SITE READINESS REQUIREMENTS	ENVIRONMENT
<ul style="list-style-type: none"> Any deviation from these drawings must be communicated in writing to and reviewed by your local GE healthcare installation project manager prior to making changes. Make arrangements for any rigging, special handling, or facility modifications that must be made to deliver the equipment to the installation site. If desired, your local GE healthcare installation project manager can supply a reference list of rigging contractors. New construction requires the following; <ol style="list-style-type: none"> Secure area for equipment, Power for drills and other test equipment, Capability for image analysis, Restrooms. Provide for refuse removal and disposal (e.g. crates, cartons, packing) For CT, MR, PET/CT, and SPECT systems it is required to minimize vibrations within the scan room. It is the customer's responsibility to contract a vibration consultant/engineer to implement site design modifications to meet the GE vibration specification. Refer to the system preinstallation manual for vibration specifications. 	<p>ALTITUDE</p> <ul style="list-style-type: none"> Operating altitude: from -150 m [-492 ft] to 3000 m [9842 ft]. <p>MAGNETIC FIELD SPECIFICATIONS</p> <ul style="list-style-type: none"> In order to avoid interference on the system, the static field limits from the surrounding environment must be less than 1 Gauss in both the scan and the operator rooms. <p>Gantry :</p> <ul style="list-style-type: none"> Ambient static magnetic fields less than 1 Gauss. Ambient AC magnetic fields less than 0.01 Gauss peak. <p>Operator console :</p> <ul style="list-style-type: none"> Ambient static magnetic fields less than 1 Gauss. <ul style="list-style-type: none"> Use static dissipative vinyl. <p>MAXIMUM GANTRY AUDIBLE NOISE LEVEL</p> <ul style="list-style-type: none"> The maximum ambient noise level is produced by the gantry during a CT scan acquisition. It is less than 70 dB when measured at a distance of one meter from the nearest gantry surface, in any direction. <p>BACKGROUND RADIATION</p> <ul style="list-style-type: none"> When the system is calibrated, background radiation from surrounding areas may adversely affect calibration. Therefore all radiation sources must be suitable shielded, including: <ul style="list-style-type: none"> - Waiting/Injection areas - Radionuclide storage and preparation area (sometimes known as "Hot Lab") <p>VIBRATION SPECIFICATIONS</p> <ul style="list-style-type: none"> The system components are sensitive to vibration in the frequency range of 0.5 to 20 Hz, depending on the amplitude of the vibration. It is the customer’s responsibility to contract a vibration consultant or qualified engineer to verify that these specifications are met and implement an appropriate solution. To minimize vibrations, the system must be installed on a solid floor, as far as possible from vibration sources (parking lots, roadways, heliports, elevators, hospital power plants... etc).
<div>Typical</div>	<div> <div>OPTIMA NM/CT 640</div> <div>EN-NUC-TYP-OPTIMA NM 640-WEB.DWG</div> <div>Rev A Date 28/Sep/2021</div> <div>A1 - General Notes</div> <div>03/16</div> </div>

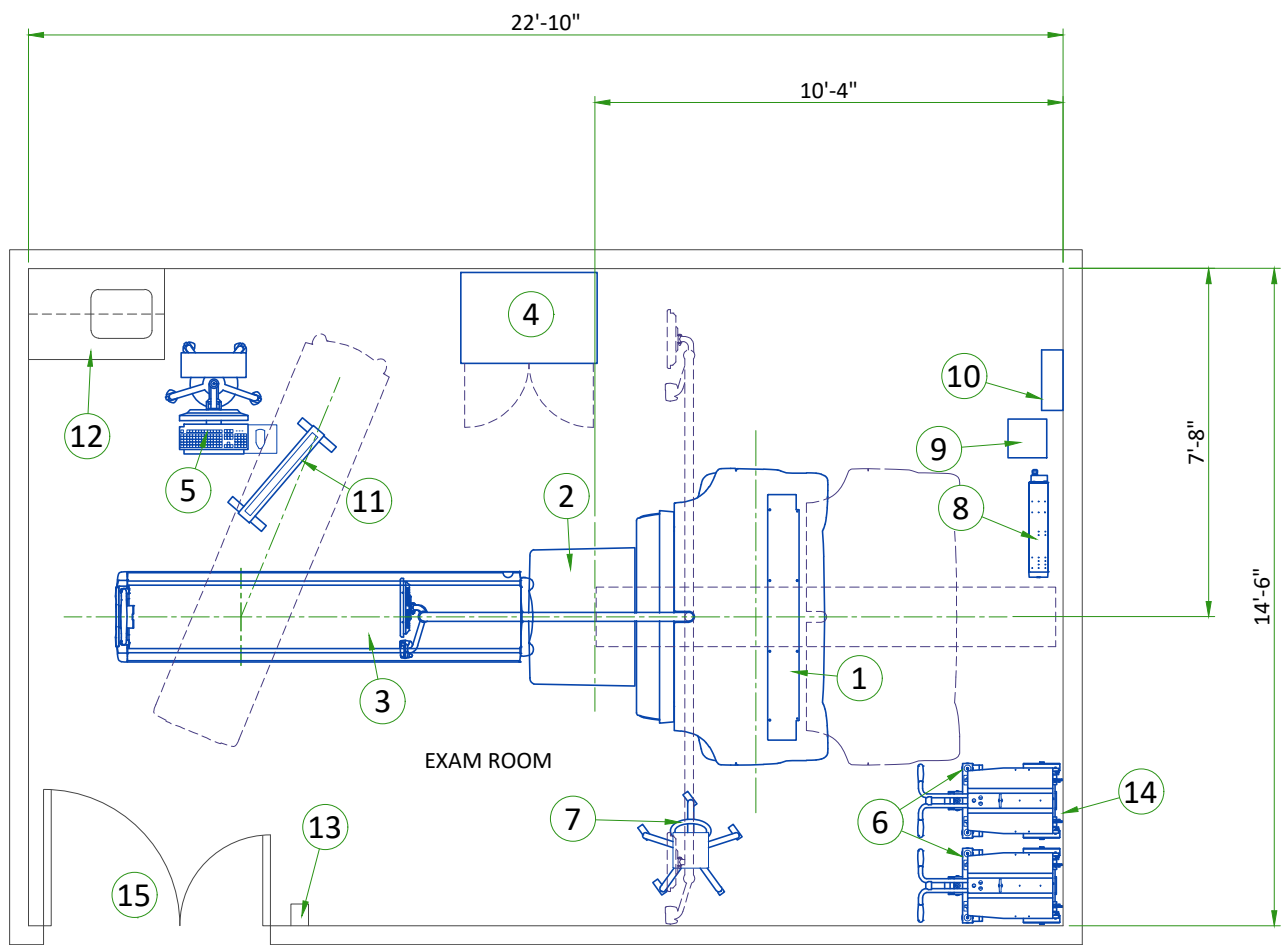

LEGEND						
A	GE Supplied			D	Available from GE	
B	GE Supplied/contractor installed			E	Equipment existing in room	
C	Customer/contractor supplied and installed			*	Item to be reinstalled from another site	
BY	ITEM	DESCRIPTION	MAX HEAT OUTPUT (btu)	WEIGHT (lbs)	MAX HEAT OUTPUT (W)	WEIGHT (kg)
A	1	CT Gantry	3140	1984	920	900
A	2	NM Gantry	4500	4828	1320	2190
A	3	Patient table	682	1228	200	557
A	4	Storage Cabinet	-	287	-	130
A	5	NM Acquisition station	256	25	75	11
A	6	Collimator Cart	-	728	-	330
D	7	ECG Monitor	-	6	-	2.7
D	8	6kVA UPS	1500	106	440	48
D	9	Transformer for 6kVA UPS	1000	77	293	35
B/D	10	Main Disconnect Panel	-	50	-	23
C	11	Mobile radiation shield				
C	12	Counter top with sink, base and wall cabinets				
C	13	Shelf for EMO switch				
C	14	Optional wall protection from collimator cart				
C	15	Minimum opening for equipment delivery is 56 in. w x 91 in. h, contingent on a 96 in. corridor width				

Exam room height	
Finished floor to slab height	TBD
Recommended finished ceiling height	8'-0"

For Accessory Sales: (866) 281-7545 Options 1, 2, 1, 2 or mail to: gehcaccessorysales@ge.com

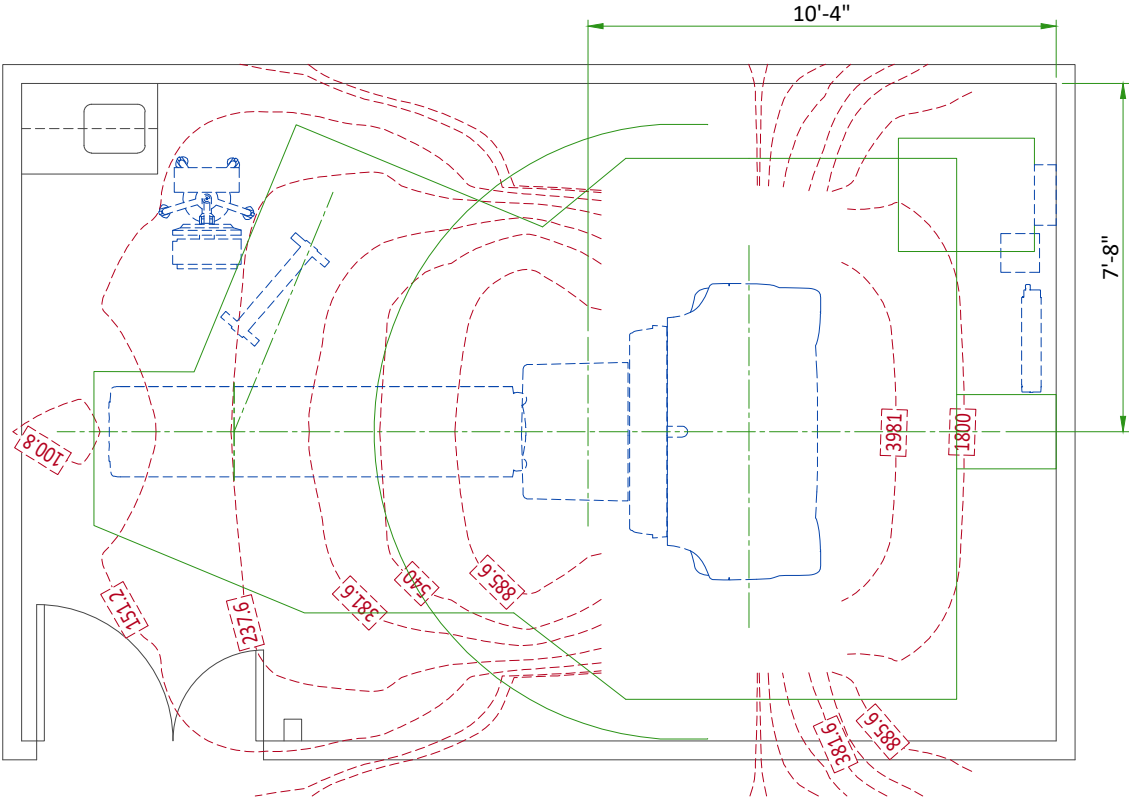

RADIOACTIVE ISOTOPES

USING RADIOACTIVE ISOTOPES


Since the system involves the use of radioactive isotopes, compliance with Nuclear Regulatory Commission regulations, or similar regulatory requirements (depending on the country), must be adhered to and all permissions obtained well in advance. It is recommended that regulatory compliance is arranged early in the site planning process.

It is essential that all preparations are completed so that required source materials can be obtained prior to installation, including calibration sources. Take into consideration that these sources may have fairly long delivery lead times, yet may also have a short half life, so that it may not be advisable to store them over long periods of time.


RADIOACTIVE ISOTOPES FOR SYSTEM CALIBRATION

DESCRIPTION	
Basic calibration	Site has license for Tc ^{99m}
	Tc ^{99m} will be available during installation
Isotopes to be used at site are available for installation. Note: Specify age and strength	Co ⁵⁷ (Rectangular Flood Source)
	Tl ²⁰¹
	I ¹³¹
	I ¹²³
	In ¹¹¹
	Ga ⁶⁷
	Xe ¹³³ (inhalation gas)


GANTRY WITH PATIENT TABLE

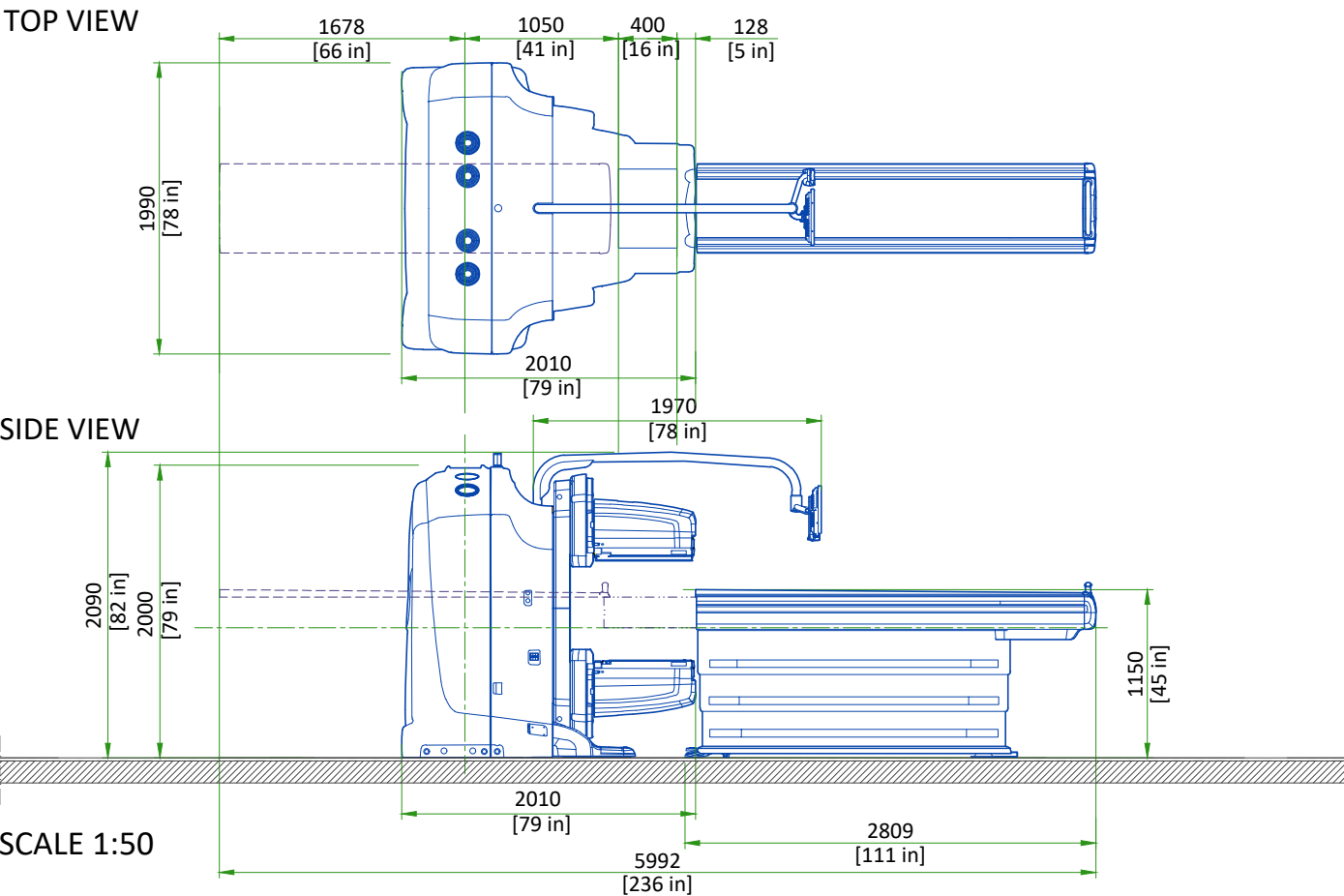

COLLIMATOR CARTS

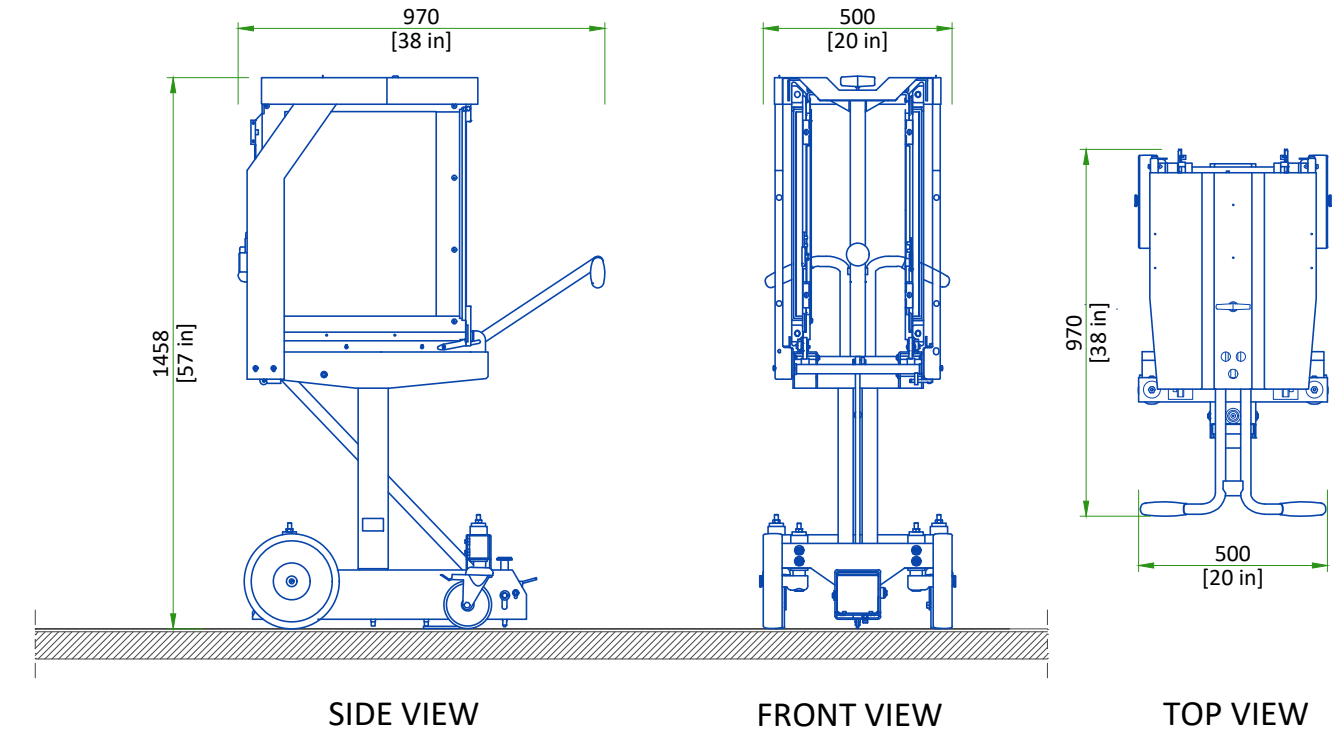

NM ACQUISITION STATION

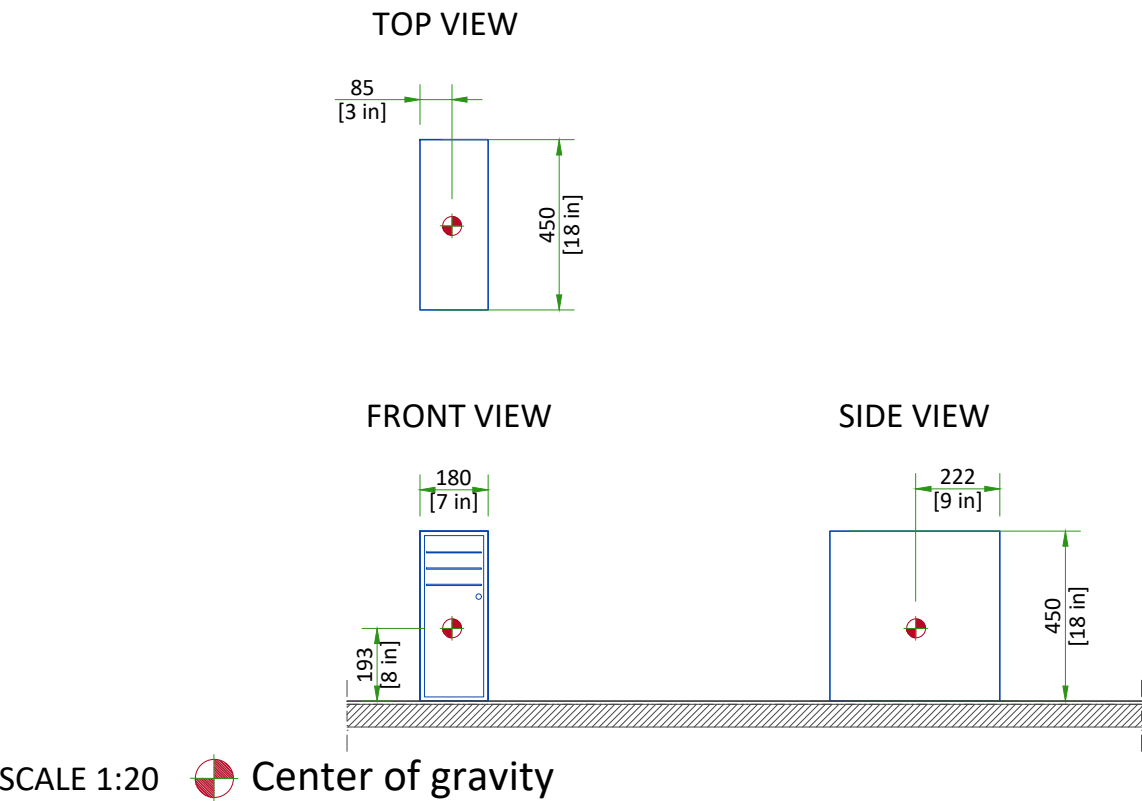


DELIVERY DETAILS

THE CUSTOMER/CONTRACTOR SHOULD:

- Provide an area adjacent to the installation site for delivery and unloading of the GE equipment.
- Ensure that the dimensions of all doors, corridors, ceiling heights are sufficient to accommodate the movement of GE equipment from the delivery area into the definitive installation room.
- Ensure that access routes for equipment will accommodate the weights of the equipment and any transportation, lifting and rigging equipment.
- Ensure that all necessary arrangements for stopping and unloading on public or private property not belonging to the customer have been made.


CT GANTRY:

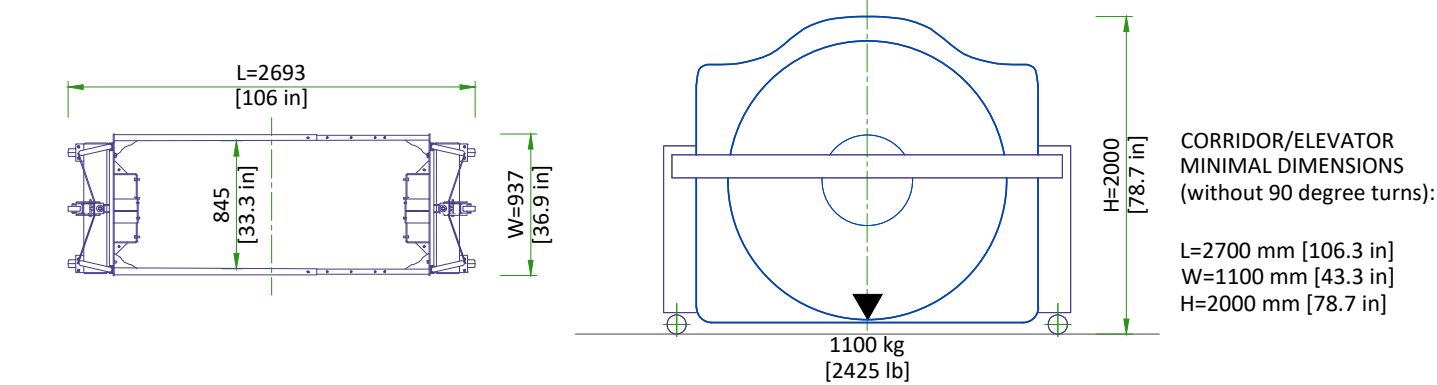

PATIENT TABLE: CORRIDOR/ELEVATOR MINIMAL DIMENSIONS (without 90 degree turns):

L=2809 mm [110.6 in] W=1000 mm [39.4 in] H=1400 mm [55.1 in] Weight: 557 kg [1228 lb]


NM GANTRY:

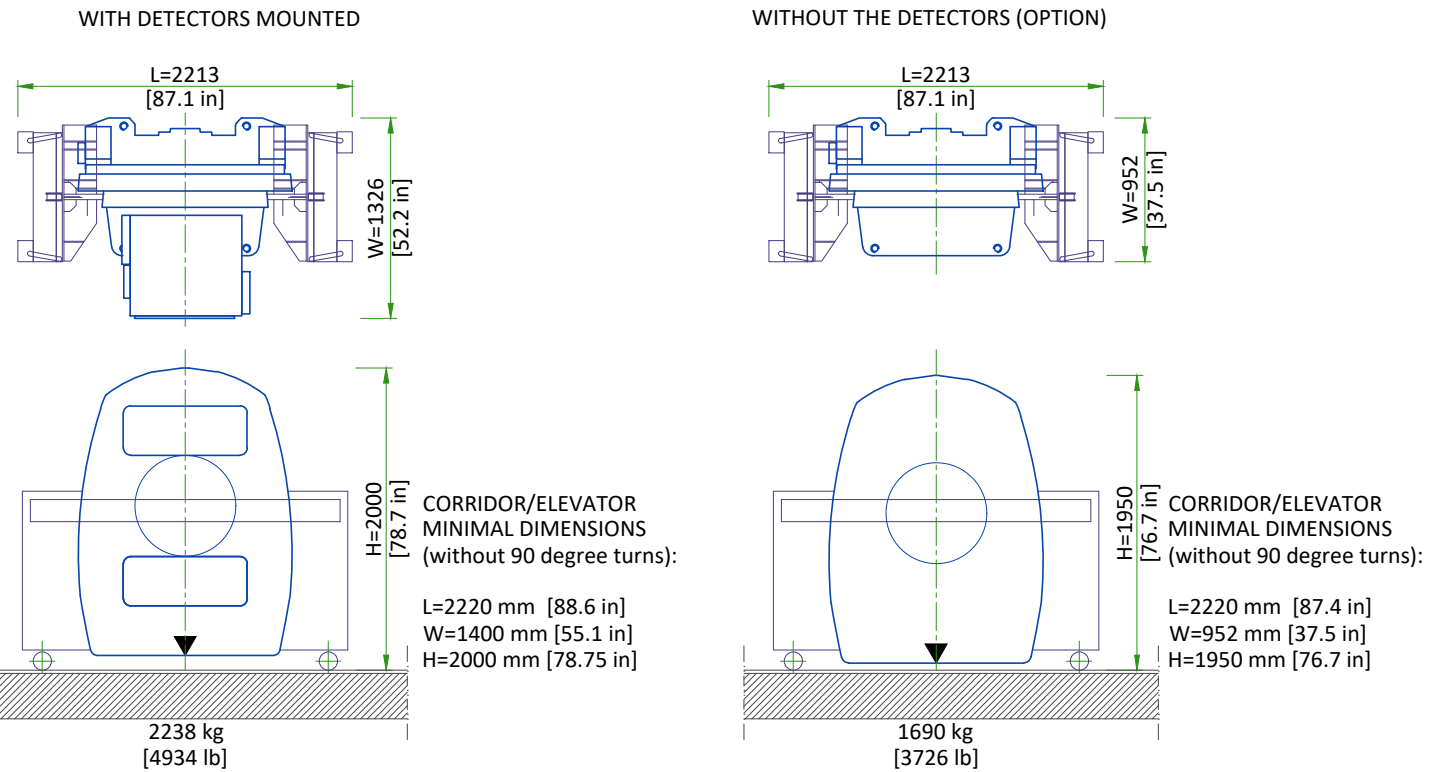

THE GANTRY IS SHIPPED ON A DOLLY EQUIPPED WITH ELEVATING CASTERS (NORMAL SHIPPING CONFIGURATION).


NOT TO SCALE

STRUCTURAL NOTES				
<ul style="list-style-type: none">• All units that are wall mounted or wall supported are to be provided with supports where necessary. Wall supports are to be supplied and installed by the customer or his contractors. See plan and detail sheets for suggested locations and mounting hole locations.• Floor slabs on which equipment is to be installed must be flat and level to specifications.• Dimensions are to finished surfaces of room.• For seismic regions ensure supports span three members.• Customers contractor must provide all penetrations in post tension floors.• Customers contractor must provide and install any non-standard anchoring. Documents for standard anchoring methods are included with GE equipment drawings for geographic areas that require such documentation.• Customers contractor must provide and install hardware for "through the floor" anchoring and/or any bracing under access floors. This contractor must also provide floor drilling that cannot be completed because of an obstruction encountered while drilling by the GE installer such as rebar etc.• It is the customer's responsibility to perform any floor or wall penetrations that may be required. The customer is also responsible for ensuring that no subsurface utilities (e.g., electrical or any other form of wiring, conduits, piping, duct work or structural supports (i.e. post tension cables or rebar)) will interfere or come in contact with subsurface penetration operations (e.g. drilling and installation of anchors/screws) performed during the installation process. To ensure worker safety, GE installers will perform surface penetration operations only after the customer's validation and completion of the "GE surface penetration permit"				
Typical	OPTIMA NM/CT 640	EN-NUC-TYP-OPTIMA NM 640-WEB.DWG	Rev A Date 28/Sep/2021	S1 - Structural Notes 08/16

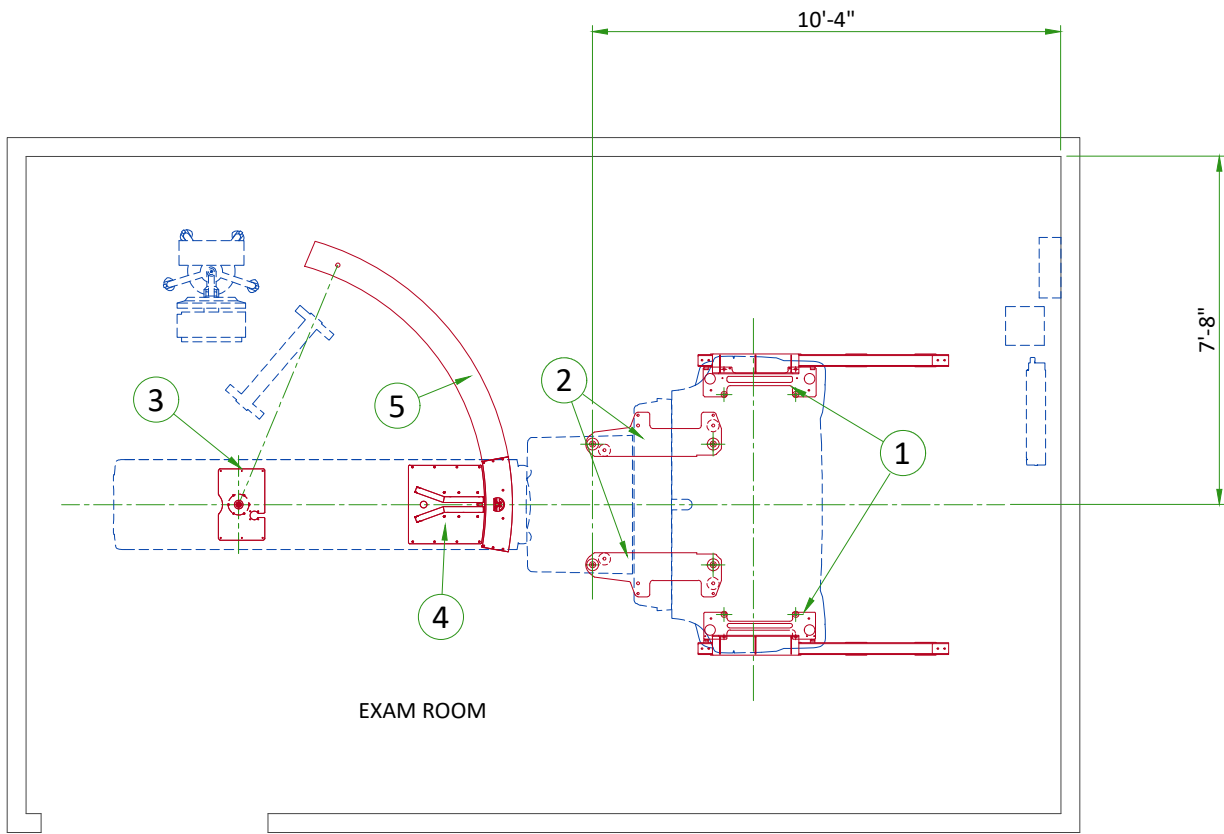


ITEM	DESCRIPTION
(GE SUPPLIED / CONTRACTOR INSTALLED)	
1	CT Gantry baseplate
2	NM Gantry baseplate
3	Table Anchor plate
4	Collimator exchange plate
5	Swing plate

ANCHORING TO THE FLOOR

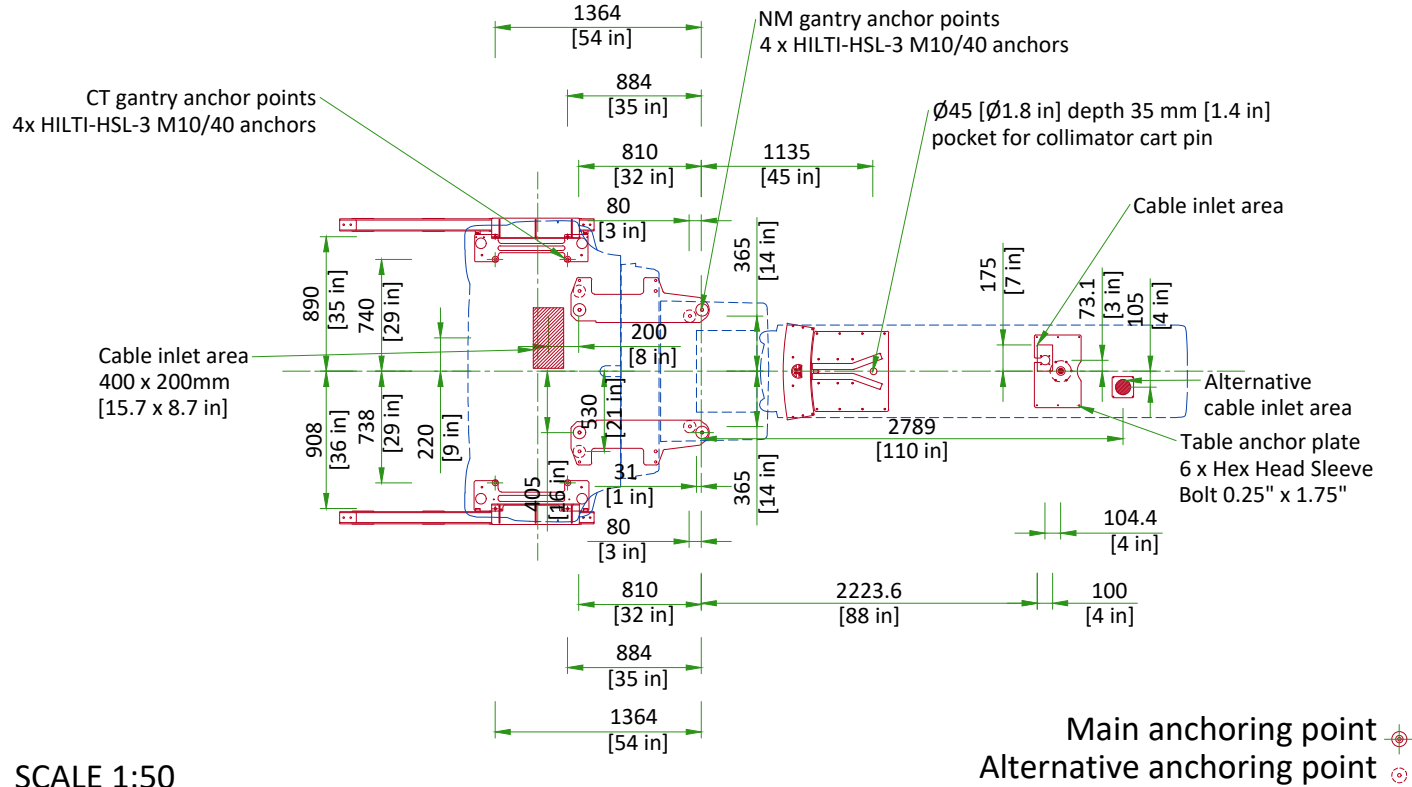


FLOOR SPECIFICATIONS

NM GANTRY MAIN ANCHORING

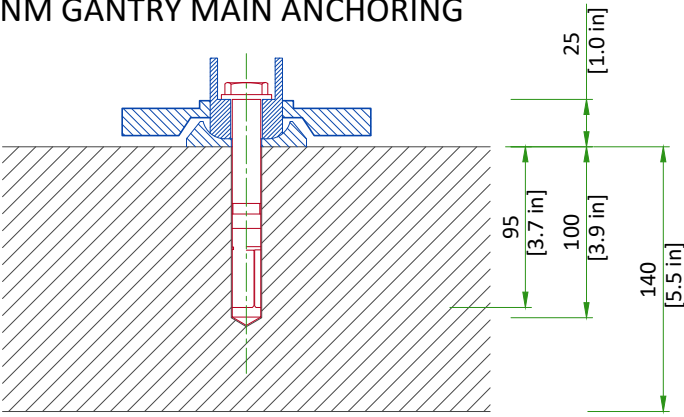


CT TRANSPORTER ANCHORING

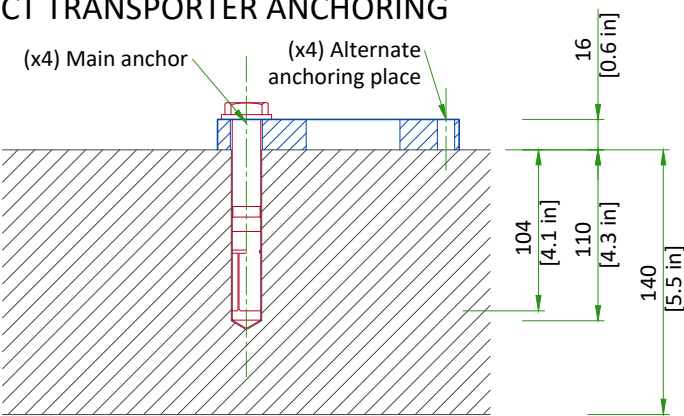


NOT TO SCALE

- Floor leveling area: 576 cm [18.9 ft] x 363 cm [11.9 ft] (covering the entire planned area of table and gantry surface).
- Slope less than 13 mm [0.5 in] over 4300 mm [160 in], if slope is between 13 mm [0.5 in] and 30 mm [1.18 in] refer to PIM for additional requirements.
- Flatness: the surface must be smooth, with deviations of no more than 5 mm [0.195 in] between depressions and high spots in any 1500 mm [59 in] throughout the room or system installation area.
- Floor surface: a single poured surface.
- Floor strength: in order to enable mounting of the system floor anchors, concrete floors must have a minimum cube strength of $f_c=4350$ psi. (30 MPa) at 28 days (curing time) for 25/30 concrete
- Floor thickness: the system's floor anchors are designed for use only on concrete floors that meet the minimal 140 mm [5.5 in] concrete floor requirements
- It is the customer/contractor responsibility to have appropriate tests performed to determine and measure concrete strength.

LOADING DISTRIBUTION

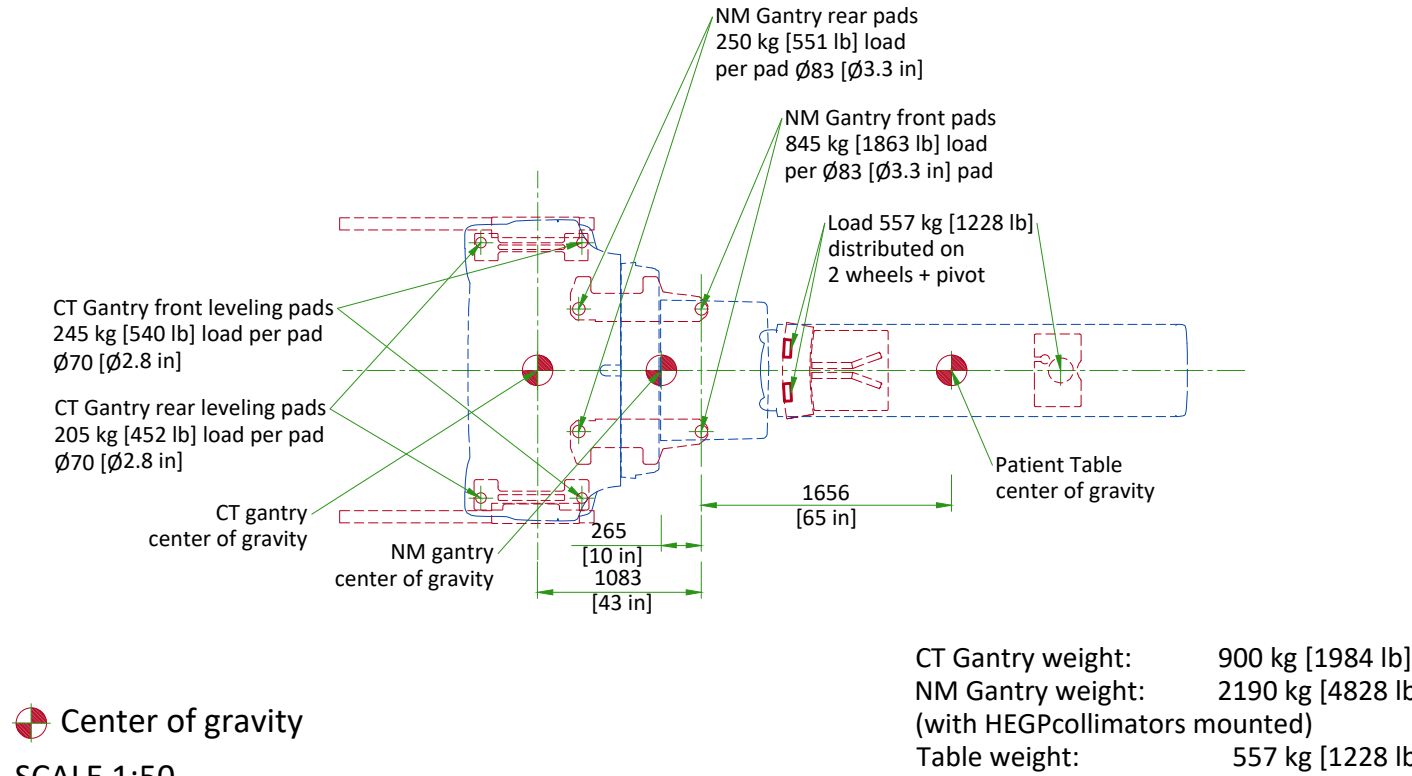

CUSTOMER/CONTRACTOR SUPPLIED MOUNTING PLATE

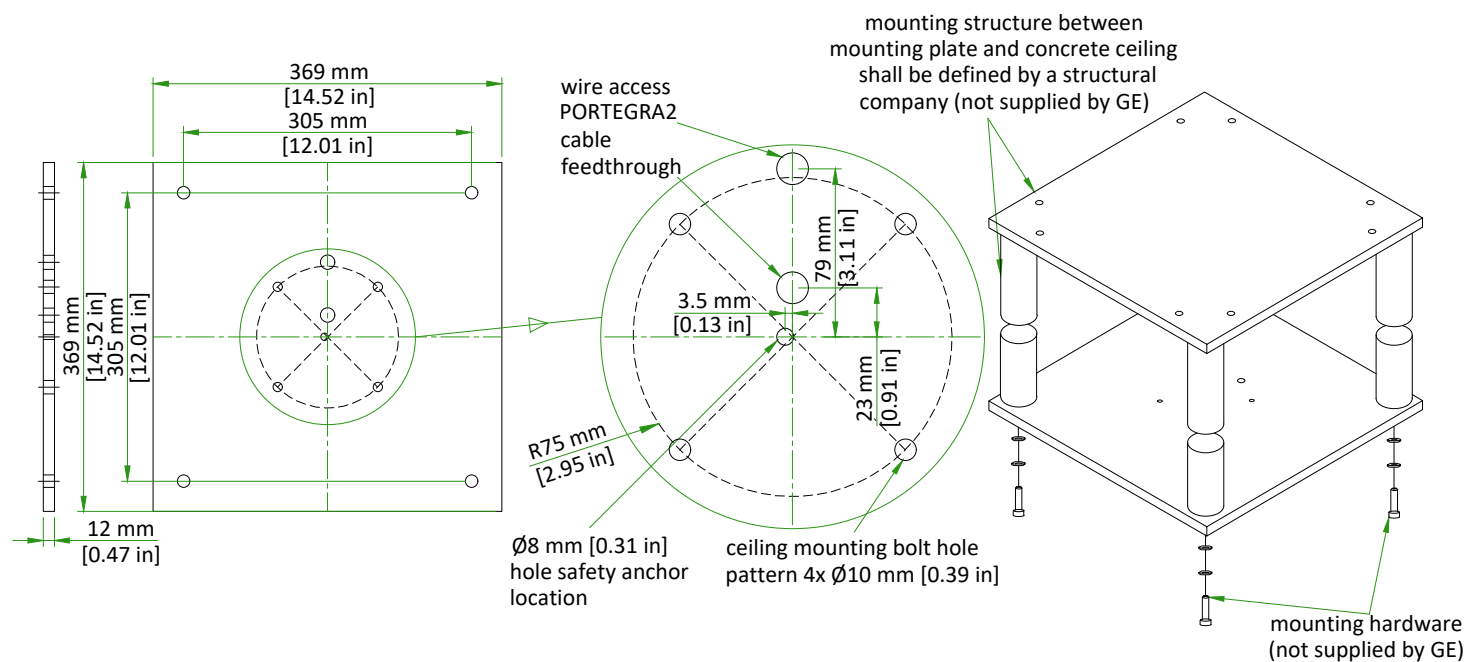

The exact location of all five drill holes for MAVIG column has to be kept, otherwise installation can't be accomplished. Column flange and safety chain fixings to concrete or to structure other than MAVIG anchoring plate or MAVIG bridge shall be defined by a structural company. All design and pre-installation activity must be done in accordance of the MAVIG Installation manual. Contact your GE Project Manager for OEM documentation. Installation of mounting plate performed by GE or a GE sub-contractor.

NOT TO SCALE

CEILING SUSPENSION DISCLAIMER

Safety and precautionary comments:

Only qualified, licensed technicians can perform electrical connections, installation, removal and repair. It is strongly recommended that at least two persons perform the installation.

Installing the system: Prior to installation, a structural engineer must confirm that the mounting structure is strong enough to provide proper support for the entire system and any attached end devices. Installation must be completed according to local building codes.

Determination of required installation hardware and torque values for installation of the ceiling column and ceiling track is the sole responsibility of the structural engineer.

Ceiling mounted systems must be installed properly. Failure to follow the instructions provided may lead to a potentially dangerous and unstable condition of the system.

GE and/or MAVIG is not responsible for unauthorized modifications made to the system or use of the system for unintended purposes. GE and/or MAVIG cannot be held liable for improper operation and modifications. Since improper modifications may impair proper operation, safety or reliability of the system, product modifications require written authorization from MAVIG.


Under GE responsibility or under Customer responsibility, for all pre-installations, whatever is the supporting structure (bridge, chair, Unistrut channel, other channels, direct anchorage in concrete, transversal beam, etc. ...) a certificate must be obtained from a structural engineer.

This certificate shall include the definition of fasteners and of their tightening torque, especially for the non-standard cases described in MAVIG PIM and for which the standard anchoring/screws delivered with product shall not be used but shall be defined (and implemented in most cases) by the structural company.

WARNING:

It is prohibited to alter the length of the ceiling column or remove any securing screws.


SUSPENSION COLUMN LENGTHS AND INSTALLATION DETAILS

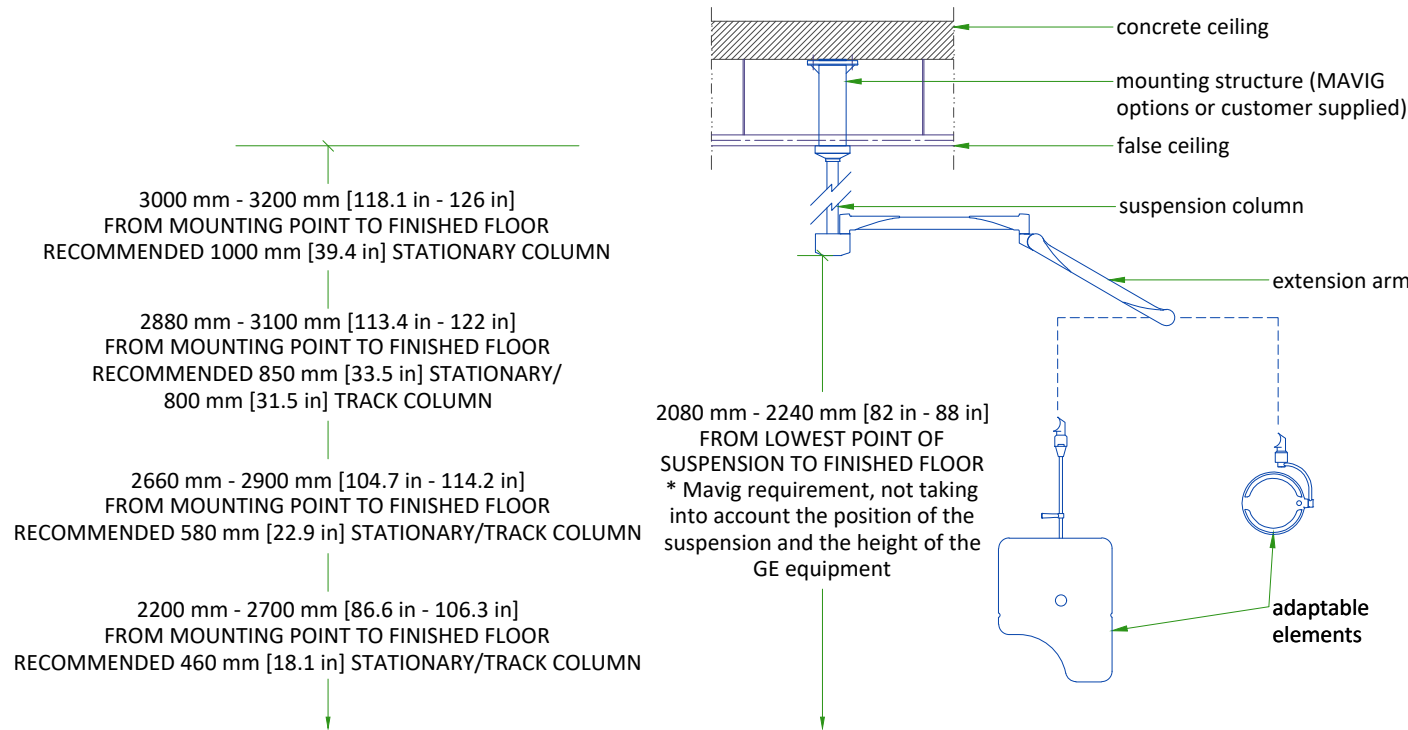

- Available column lengths might differ, please refer to the GE commercial catalog for current selection options
- For rooms with higher mounting point than 3200 mm [126 in], a ceiling construction between structural ceiling and vertical column is suggested which needs to be designed by a structural engineer
- All design and pre-installation activity must be done in accordance of the MAVIG Installation manual
- Contact your GE Project Manager for OEM documentation
- Installation of mounting plate performed by GE or a GE sub-contractor

NOT TO SCALE

PORTEGRA2 COLUMN ASSEMBLY

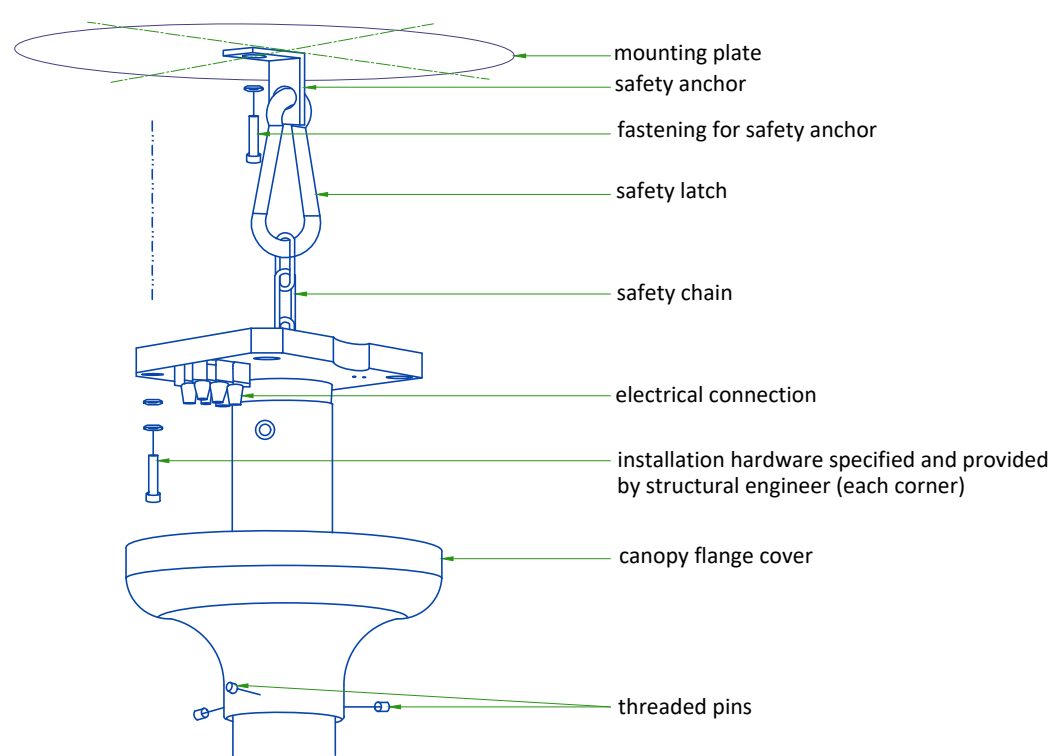


- Safety chain shall be always attached.
- Do not use shims between column and mounting surface.

All design and pre-installation activity must be done in accordance of the MAVIG Installation manual. Contact your GE Project Manager for OEM documentation. Installation of mounting plate performed by GE or a GE sub-contractor.


NOT TO SCALE

MEDRAD MOUNTING DETAILS FOR CEILING INJECTOR

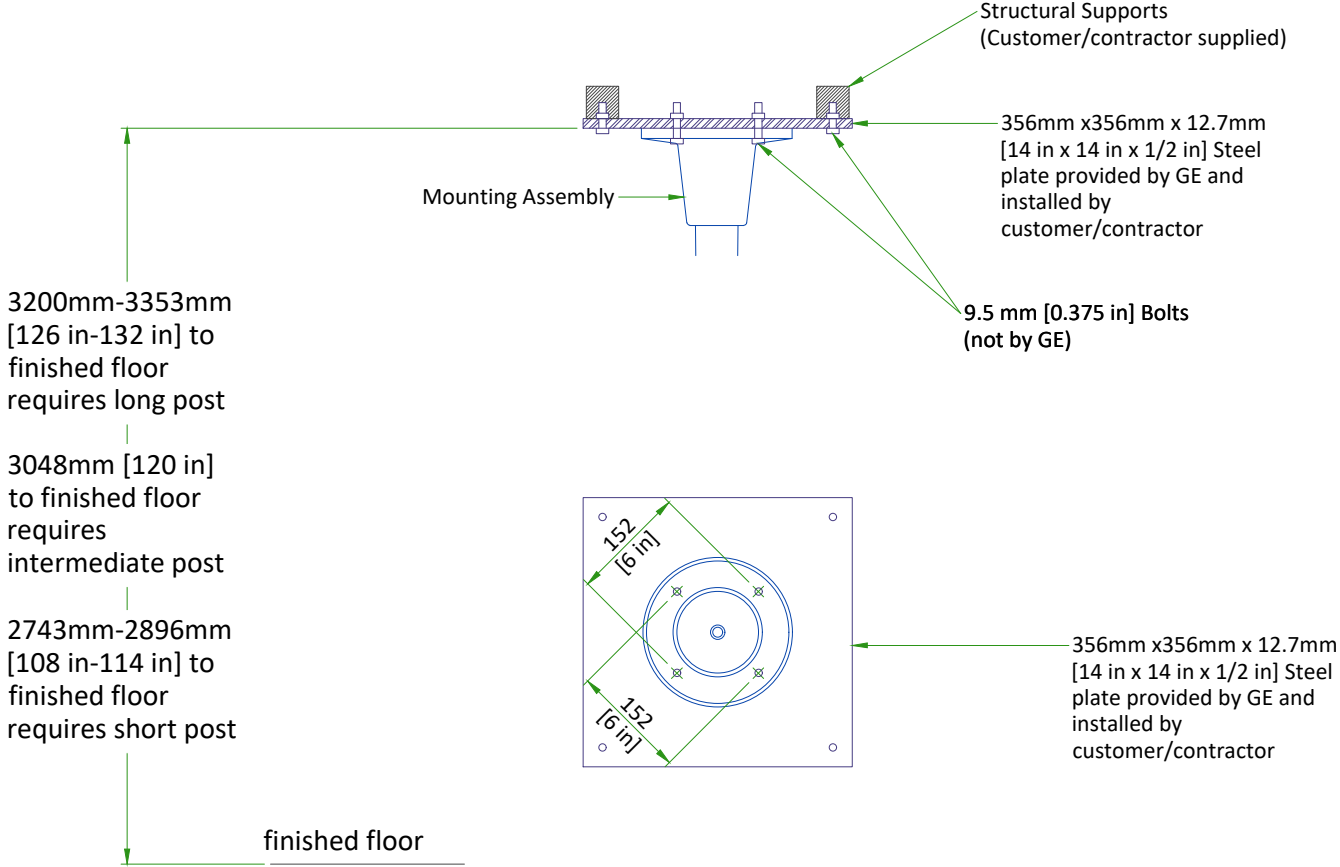

Scale 1:10

MEDRAD MOUNTING DETAILS FOR CEILING INJECTOR

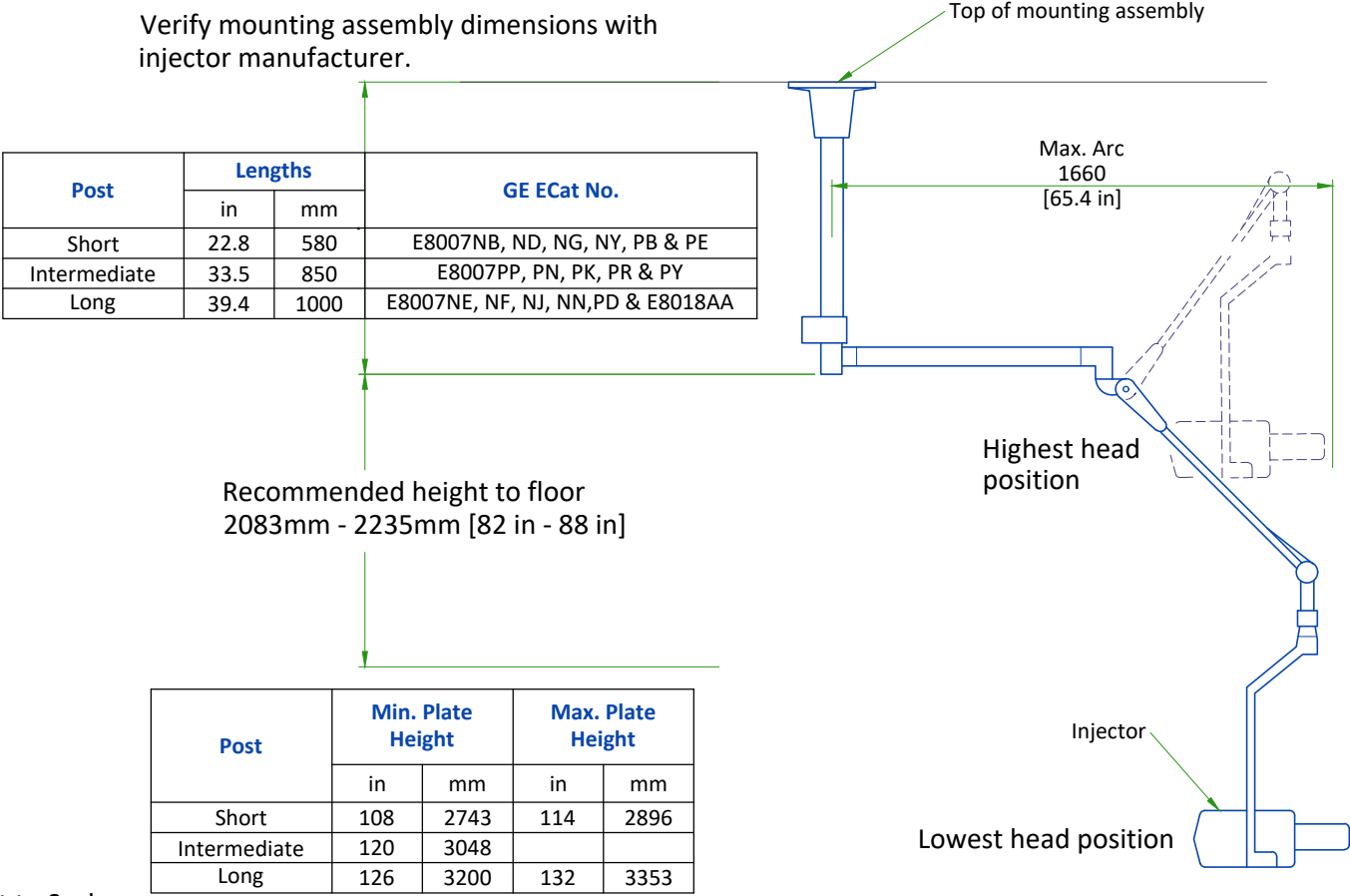

Not to Scale

TEMPERATURE AND HUMIDITY SPECIFICATIONS

IN-USE CONDITIONS

	EXAM/CONTROL ROOM		
	Min	Recommended	Max
Temperature	18°C [64°F]	22°C [72°F]	26°C [79°F]
Temperature gradient	≤ 3°C/h [≤ 5°F/h]		
Relative humidity (1)	30% to 60%		
Humidity gradient	≤ 5%/h		

STORAGE CONDITIONS

Temperature	+4°C [+40°F] to +27°C [+80°F]
Temperature gradient	≤ 3°C/h [≤ 5°F/h]
Relative humidity (1)	30% to 70%
Humidity gradient	≤ 5%/h
Air pressure	700 hPa to 1060 hPa

(1) non condensing

AIR RENEWAL

According to local standards.

NOTE

In case of using air conditioning systems that have a risk of water leakage it is recommended not to install it above electric equipment or to take measures to protect the equipment from dropping water.

HEAT DISSIPATION

ROOM	DESCRIPTION	HEAT DISSIPATION (kW)	HEAT DISSIPATION (BTU/hr)
		MAX	MAX
Exam Room	NM Gantry	1.32	4500
	CT Gantry	0.92	3140
	Patient table	0.20	682
	TOTAL	2.44	8322
Exam/Technical Room*	Eaton 6 kVA UPS	0.44	1500
	Transformer for Eaton 6kVA UPS	0.29	1000
	TOTAL	0.73	2500
Control Room	NM Aquisition station (computer only)	0.08	256
	TOTAL	0.08	256
*Technical Room is not mandatory, the placements of these elements are recommended in the Exam Room.			

CONNECTIVITY REQUIREMENTS


Broadband Connections are necessary during the installation process and going forward to ensure full support from the Engineering Teams for the customers system. Maximum performance and availability for the customers system is maintained and closely monitored during the lifetime of the system. Proactive and reactive maintenance is available utilising the wide range of digital tools using the connectivity solutions listed below:







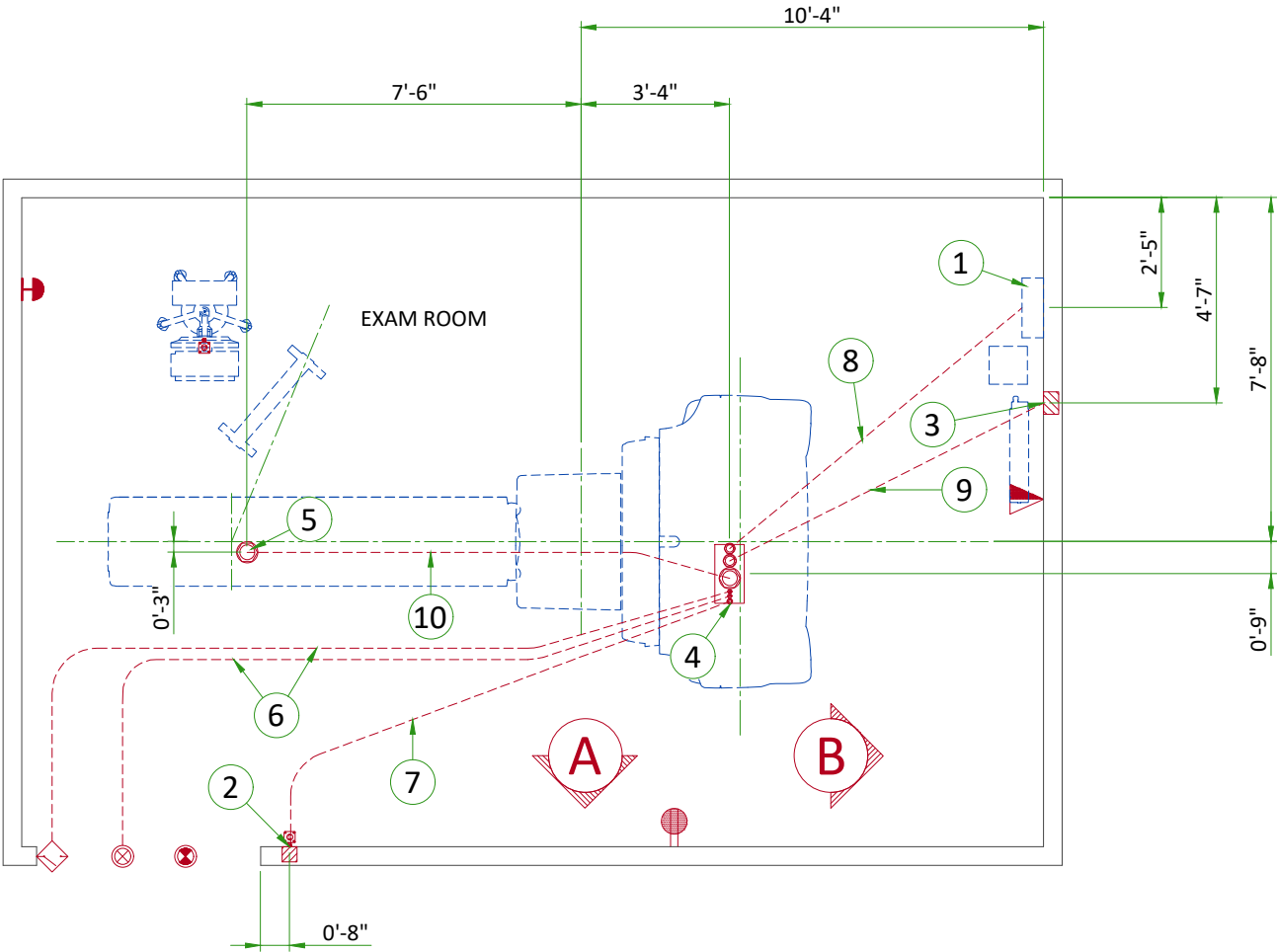

- Site-to-Site VPN/GE Solution
- Site-to-Site VPN/Customer Solution
- Connection through Dedicated Service Network
- Internet Access - connectivity for InSite 2.0

The requirements for these connectivity solutions are explained in the broadband solutions catalogue (separate document).


ELECTRICAL NOTES

1. All wires specified shall be copper stranded, flexible, thermo-plastic, color coded, cut 10 foot long at outlet boxes, duct termination points or stubbed conduit ends. All conductors, power, signal and ground, must be run in a conduit or duct system. Electrical contractor shall ring out and tag all wires at both ends. Wire runs must be continuous copper stranded and free from splices.
 - 1.1. Aluminum or solid wires are not allowed.
 2. Wire sizes given are for use of equipment. Larger sizes may be required by local codes.
 3. It is recommended that all wires be color coded, as required in accordance with national and local electrical codes.
 4. Conduit sizes shall be verified by the architect, electrical engineer or contractor, in accordance with local or national codes.
 5. Convenience outlets are not illustrated. Their number and location are to be specified by others. Locate at least one convenience outlet close to the system control, the power distribution unit and one on each wall of the procedure room. Use hospital approved outlet or equivalent.
 6. General room illumination is not illustrated. Caution should be taken to avoid excessive heat from overhead spotlights. Damage can occur to ceiling mounting components and wiring if high wattage bulbs are used. Recommend low wattage bulbs no higher than 75 watts and use dimmer controls (except MR). Do not mount lights directly above areas where ceiling mounted accessories will be parked.
 7. Routing of cable ductwork, conduits, etc., must run direct as possible otherwise may result in the need for greater than standard cable lengths (refer to the interconnection diagram for maximum usable lengths point to point).
 8. Conduit turns to have large, sweeping bends with minimum radius in accordance with national and local electrical codes.
 9. A special grounding system is required in all procedure rooms by some national and local codes. It is recommended in areas where patients might be examined or treated under present, future, or emergency conditions. Consult the governing electrical code and confer with appropriate customer administrative personnel to determine the areas requiring this type of grounding system.
 10. The maximum point to point distances illustrated on this drawing must not be exceeded.
 11. Physical connection of primary power to GE equipment is to be made by customers electrical contractor with the supervision of a GE representative. The GE representative would be required to identify the physical connection location, and insure proper handling of GE equipment.
 12. GEHC conducts power audits to verify quality of power being delivered to the system. The customer's electrical contractor is required to be available to support this activity.
-
- All junction boxes, conduit, duct, duct dividers, switches, circuit breakers, cable tray, etc., are to be supplied and installed by customers electrical contractor.
 - Conduit and duct runs shall have sweep radius bends
 - Conduits and duct above ceiling or below finished floor must be installed as near to ceiling or floor as possible to reduce run length.
 - Ceiling mounted junction boxes illustrated on this plan must be installed flush with finished ceiling.
 - All ductwork must meet the following requirements:
 - 1.Ductwork shall be metal with dividers and have removable, accessible covers.
 - 2.Ductwork shall be certified/rated for electrical power purposes.
 - 3.Ductwork shall be electrically and mechanically bonded together in an approved manner.
 - 4.PVC as a substitute must be used in accordance with all local and national codes.
 - All openings in raceway and access flooring are to be cut out and finished off with grommet material by the customers contractor.
 - General contractor to insert pull cords for all cable run conduits between the equipment room and the operators control room.
 - 10 foot pigtails at all junction points.
 - Grounding is critical to equipment function and patient safety. Site must conform to wiring specifications shown on this plan.




ITEM	DESCRIPTION	
1	Main disconnect panel	
2	4" x 4" x 4" [100 x 100 x 100] box & coverplate (EMO switch)	
3	6" x 6" x 4" [150 x 150 x 100] box and coverplate (UPS)	
4	Suitable bushings & locknuts (gantry)	
5	Conduit flush with finished floor	
6	1/2" [13] conduit below floor	
7	3/4" [20] conduit below floor	
8	2" [51] conduit below floor	
9	2 1/2" [64] conduit below floor	
10	4" [102] conduit below floor	
		Emergency Off (SEO 1), (recommended height 1.2m [48"] above floor)
		Warning light
		Warning light controller
		Door switch
		Duplex hospital grade, dedicated outlet 120-v, single phase outlet same feeder circuit as Main Disconnect Panel
		Network outlet
		ESTOP switch (locate near acquisition workstation)
		EMO switch

Additional Conduit Runs (Contractor Supplied and Installed)				
From (Bubble # / Item)	To (Bubble # / Item)	Qty	Size	
			In.	mm
3 Phase Power	1 Main Disconnect Panel	1	As req'd	As req'd
Emergency Off (SEO)		1	1/2	13
Warning Light	Warning light controller	1	1/2	13
1 Phase Power		1	As req'd	As req'd

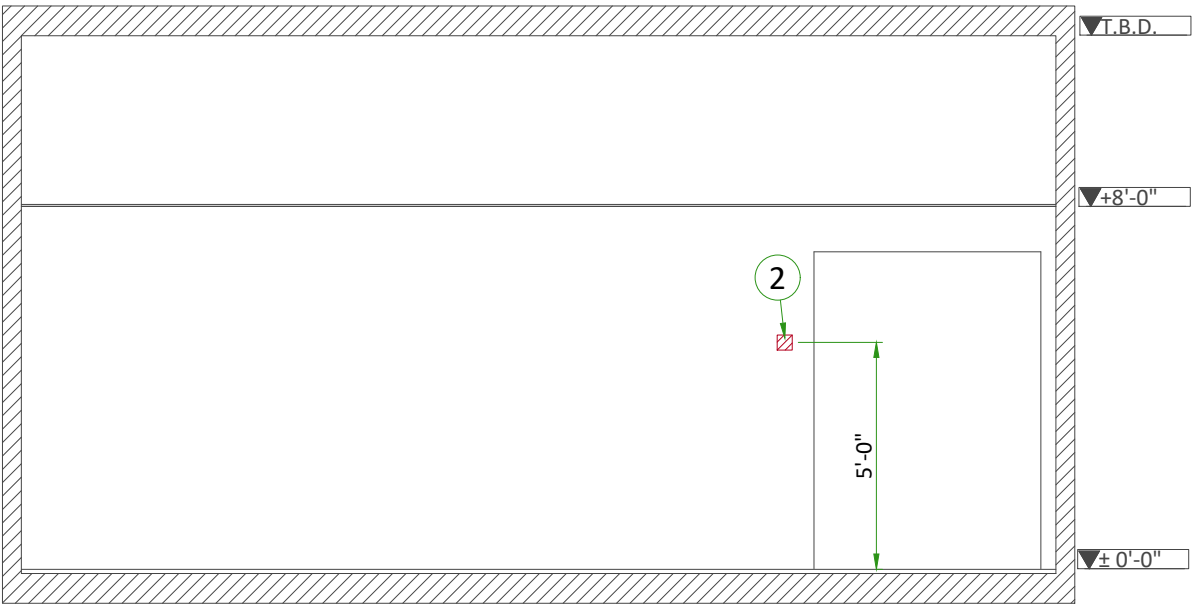

A

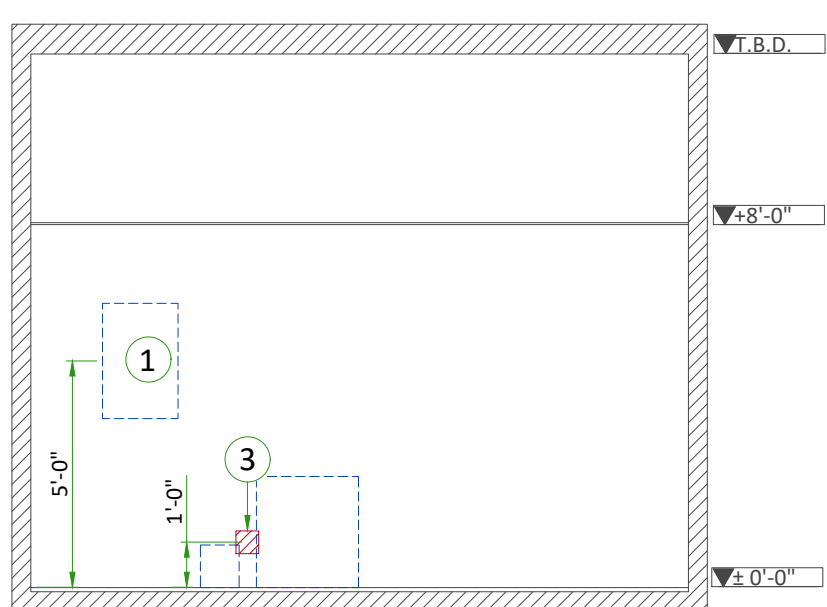

B

POWER REQUIREMENTS

POWER SUPPLY

POWER SUPPLY	3 PHASES+N+G 380 to 480 VAC ± 10%
FREQUENCIES	50/60 Hz ± 3 Hz
MAXIMUM POWER DEMAND	19 kVA
AVERAGE (CONTINUOUS) POWER DEMAND	10 kVA
POWER FACTOR	0.85

- Power supply should come into a Main Disconnect Panel (MDP) containing the protective units and controls.
- The section of the supply cable should be calculated in accordance with its length and the maximum permissible voltage drops.
- There must be discrimination between supply cable protective device at the beginning of the installation (main low-voltage transformer side) and the protective devices in the MDP.

SUPPLY CHARACTERISTICS

- Power input must be separate from any others which may generate transients (elevators, air conditioning, radiology rooms equipped with high speed film changers...).
- All equipment (lighting, power outlets, etc...) installed with GE system components must be powered separately.
- Phase imbalance 2% maximum.
- Maximum voltage regulation at full load = 5% (Including line impedance).
- Transients must be less than 1500 V peak. (on a 400 V line)
- Inrush current can withstand up to 10 times the recommended circuit breaker rating that could be reached during system power up, due to the system main transformer.

GROUND SYSTEM

- System of equipotential grounding.
- Equipotential: The equipotential link will be by means of an equipotential bar. This equipotential bar should be connected to the protective earth conductors in the ducts of the non GE cableways and to additional equipotential connections linking up all the conducting units in the rooms where GE system units are located.
- The impedance of the earth bar should be less than or equal to 10 Ohm.

CABLES


- Power and cable installation must comply with the distribution diagram.
- All cables must be isolated and flexible, cable color codes must comply with standards for electrical installation.
- The cables from signaling and remote control (Y, SEO, L...) will go to MDP with a pigtail length of 1.5 m, and will be connected during installation. Each conductor will be identified and isolated (screw connector).

CABLEWAYS

The general rules for laying cableways should meet the conditions laid down in current standards and regulations, with regard to:

- Protecting cables against water (cableways should be waterproof).
- Protecting cables against abnormal temperatures (proximity to heating pipes or ducts).
- Protecting cables against temperature shocks.
- Replacing cables (cableways should be large enough for cables to be replaced).
- Metal cableways should be grounded.


MAIN DISCONNECT PANEL

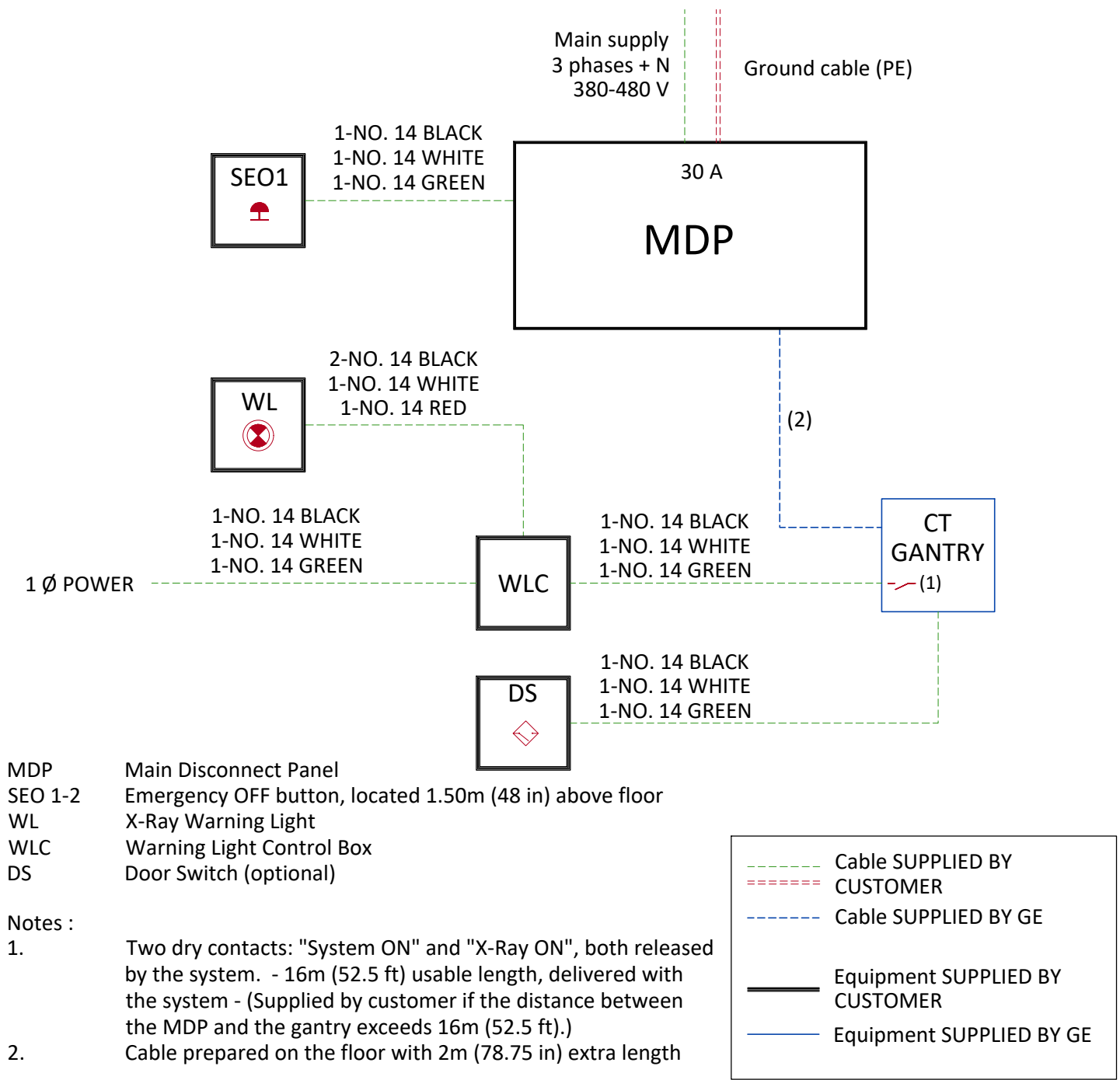

FEEDER TABLE								
MIN. FEEDER WIRE SIZE, AWG OR MCM (mm²)/VAC	MINIMUM FEEDER WIRE LENGTH - ft (m)							
	50 (15)	100 (30.5)	150 (46)					
480 VAC	6 (16)	6 (16)	4 (25)					
GROUND REQ'D	6	6	4					
GENERAL NOTES								
In all cases qualified personnel must verify that the feeder (at the point of take-off) and the run to the GE system meet all the requirements stated in the PIM								
For a single unit installation, the minimum transformer size is 24KV _a , with 2.4% rated regulation at unity power factor. Resultant maximum allowable feeder regulation is 3.4%								
A 6AWG (16mm²) Grounding conductor recommended in all cases, will run from the equipment back to the power source/main grounding point and always travel in the same conduit with the feeders								

TYPICAL CABLE MANAGEMENT

CONDUIT IN THE FLOOR

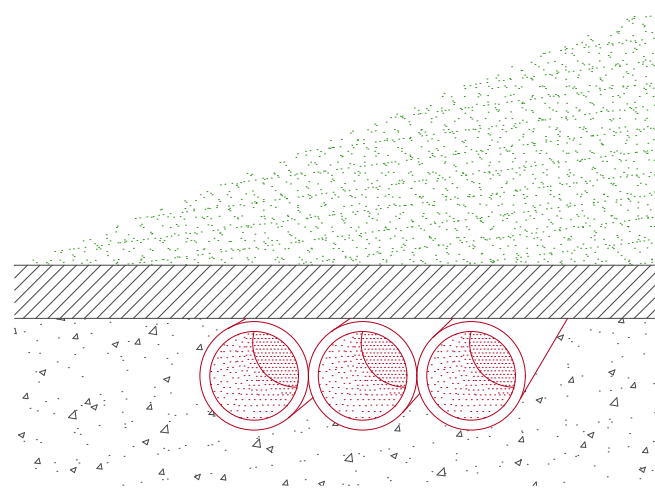

NOT TO SCALE

INTERCONNECTIONS

