





| | | | | | | | | |
|---|-------|---------------------------------|---|-------------|-------------|------------|--|-----|
| | | | <div>SITE NAME</div> <div>CITY</div> <div>COUNTRY</div> | | | | | |
| | | | | | | | | |
| | | | | | | | | |
| | | | | | | | | |
| | | | | | | | | |
| | | | | | | | | |
| REV | DATE | MODIFICATIONS | | | | | | |
| 01 - Cover Sheet 02 - Equipment Layout 03 - Floor - Electrical Layout 04 - Floor Structural Details 05 - Power Requirements 06 - HVAC - Enviroment - Interconnections 07 - Disclaimer - Site Readiness | | | <div> GE Healthcare</div> | | | | <div>GE Contact Name</div> <div>Phone</div> <div>Email</div> | |
| <div>A mandatory component of this drawing set is the GE Healthcare Pre Installation manual. Failure to reference the Pre Installation manual will result in incomplete documentation required for site design and preparation. Pre Installation documents for GE Healthcare products can be accessed on the web at: www.gehealthcare.com/siteplanning</div> <div>GE does not take responsibility for any damages resulting from changes on drawings made by others. Errors may occur by not referring to the complete set of final issue drawing. GE cannot accept responsibility for any damage due to the partial use of GE final issue drawings, however caused. All dimensions are in millimeters unless otherwise specified. Do not scale from printed pdf files. GE accepts no responsibility or liability for defective work due to scaling from these drawings.</div> | | | <div>XR 6000 STARLIGHT</div> <div>FINAL STUDY</div> | | | | | |
| | | | Drawn by | Verified by | Concession | S.O. (GON) | PIM Manual | Rev |
| | | | - | - | - | - | 5549138-2EN | 1 |
| Format | Scale | File Name | | | Date | Sheet | | |
| A3 | 1:50 | EN-RAD-TYP-XR6000_STARLIGHT.DWG | | | 23/MAR/2021 | 01/07 | | |

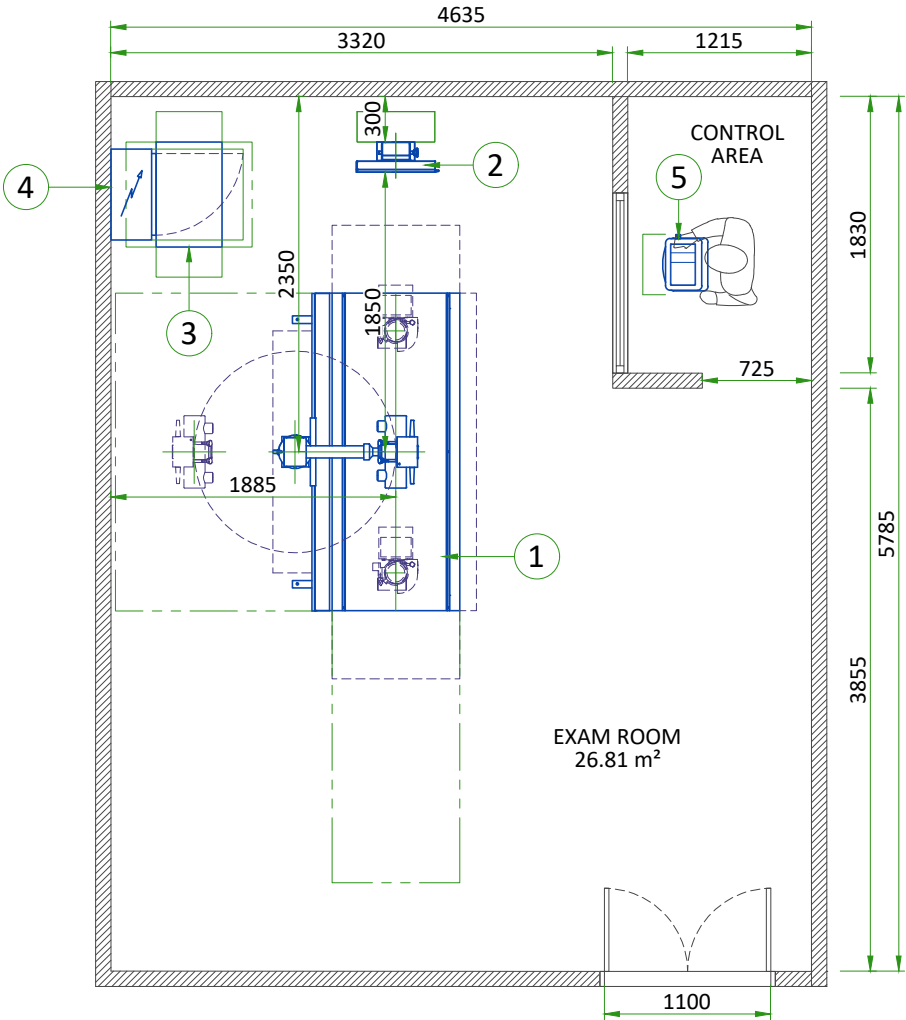

EQUIPMENT LAYOUT

| ITEM | DESCRIPTION | DIMENSIONS LxWxH (mm) | WEIGHT (kg) |
|--------------------------------------|-------------------------------|--------------------------|----------------|
| 1 | STANDARD INTEGRATED TABLE | 2100x1350x2260 | 355 |
| 2 | STANDARD WALL STAND | 655x200x1900 | 75 |
| 3 | POWER DISTRIBUTION UNIT (PDU) | 716x457x589 | 122 |
| 4 | POWER DISTRIBUTION BOX (PDB) | 800x600x300 | 42 |
| 5 | OPERATOR CONSOLE ON PEDESTAL | 330x380x980 | 10.1 |
| WALL - ACCORDING TO RECEIVED DRAWING | | | |
| EXAM ROOM HEIGHT | | | |
| FINISHED FLOOR TO SLAB HEIGHT | | | - |
| FALSE CEILING HEIGHT | | | min. 2.50 m |

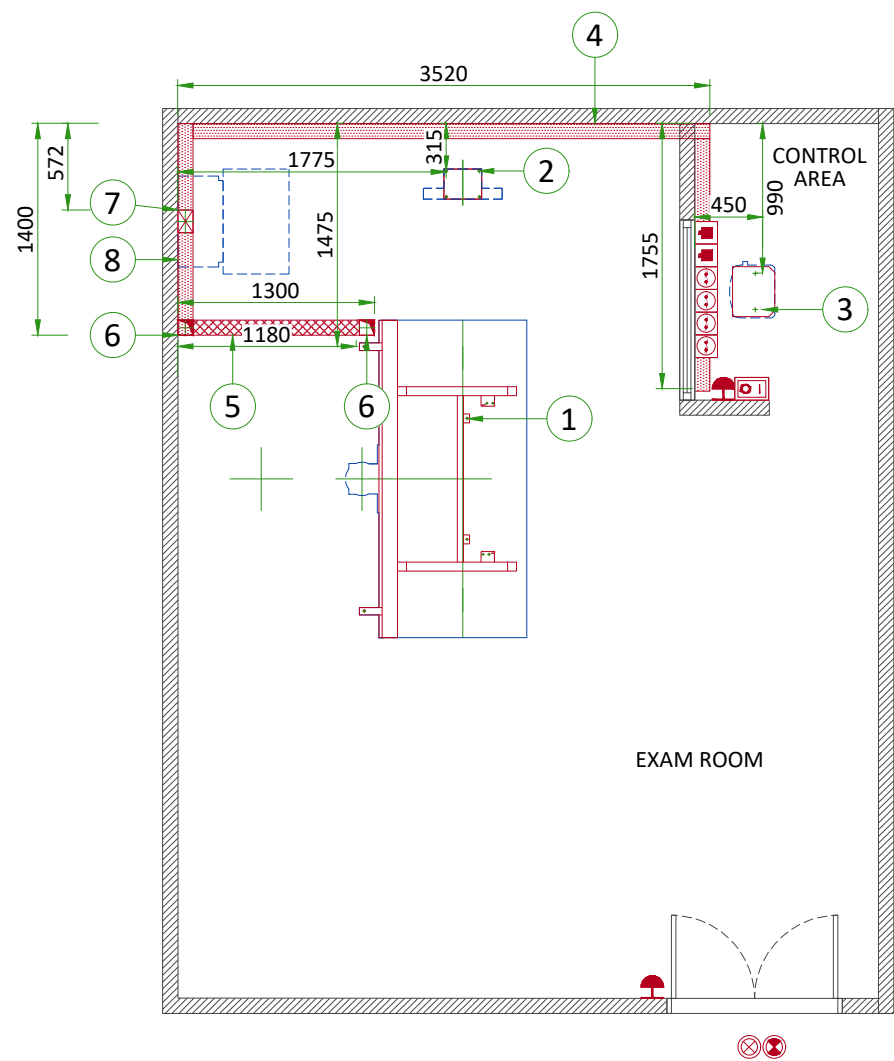

FLOOR - ELECTRICAL LAYOUT

| ITEM | QTY | DESCRIPTION |
|------|-----|---|
| 1 | | Table anchoring (see Floor Structural Details) |
| 2 | | Wall Stand anchoring (see Floor Structural Details) |
| 3 | | Operator console on pedestal anchoring (see Floor Structural Details) |
| 4 | | 150x100 horizontal wall duct |
| 5 | | 100x70 flush floor duct |
| 6 | | 100x100 cable inlet on the floor |
| 7 | | 150x100 vertical duct for PDB cabling |
| 8 | | Power Distribution Box (PDB) |


Basic system

| | | |
|--|---|--|
| | 4 | Additional electrical outlets: 230V 10/16A +G |
| | 2 | RJ45 network socket for the main system console |
| | 2 | System emergency off (SEO), Emergency OFF, 1.50m above floor near access doors |
| | 1 | XR ON lamp (L1) - 24V, located near access doors |

| | |
|--|------------------|
| | Flush floor duct |
| | Wall duct |

FLOOR MOUNTING

TABLE AND WALLSTAND

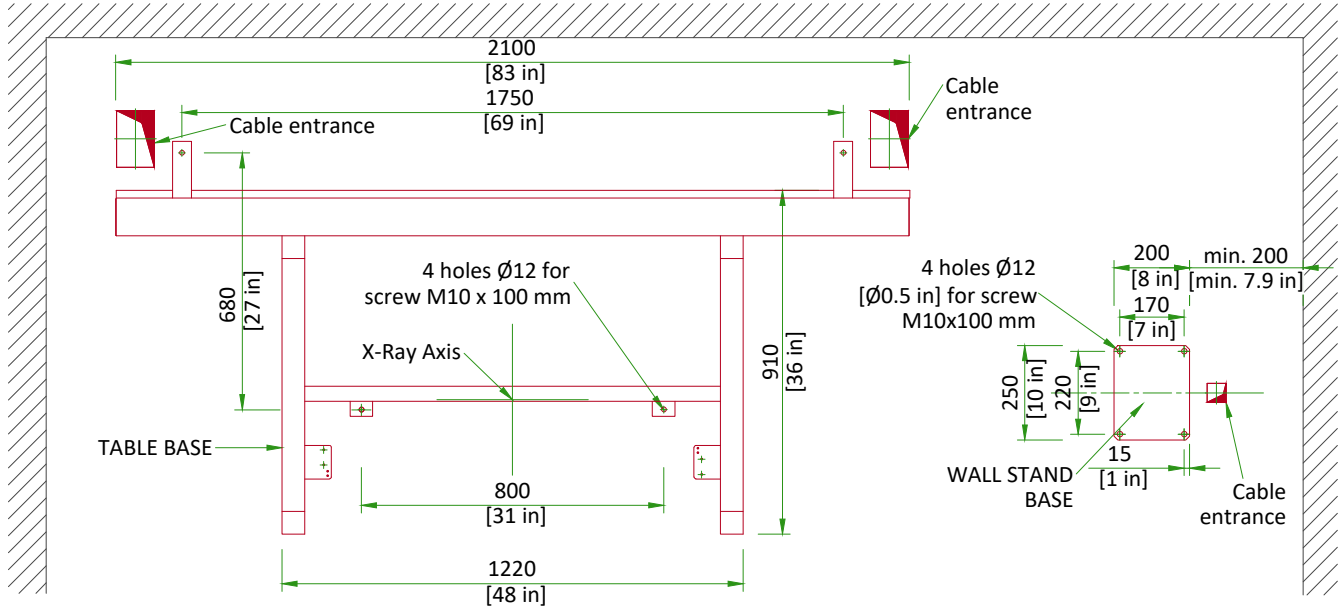


- The supplied anchors require a minimum embedment of 55 mm [2.2 in] into the concrete. If the floor thickness is less than 75 mm [3 in], it is recommended that the unit be secured using a through-bolt method with a reinforcement plate on the back side. The ground surface must be flat and leveled.
- Anchors supplied by GE.

SCALE 1:20

FLOOR MOUNTING

OPERATOR CONSOLE ON PEDESTAL

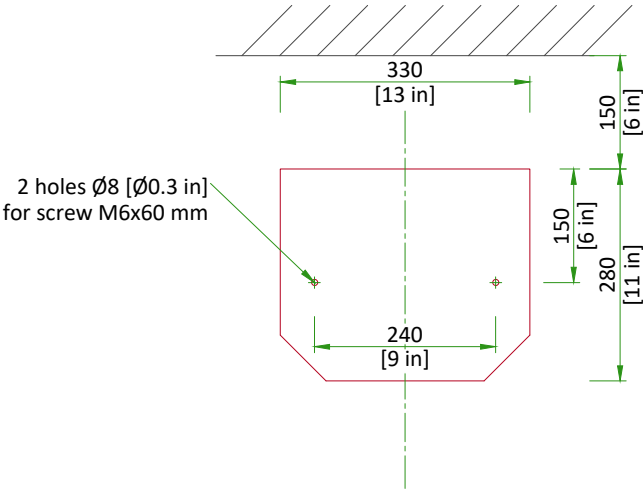


- The supplied anchors require a minimum embedment of 35 mm [1.4 in] into the concrete. If the floor thickness is less than 55 mm [2.2 in], it is recommended that the unit be secured using a through-bolt method with a reinforcement plate on the back side. The ground surface must be flat and leveled.
- Anchors supplied by GE.


SCALE 1:10

TYPICAL CABLE MANAGEMENT

FLUSH FLOOR DUCT

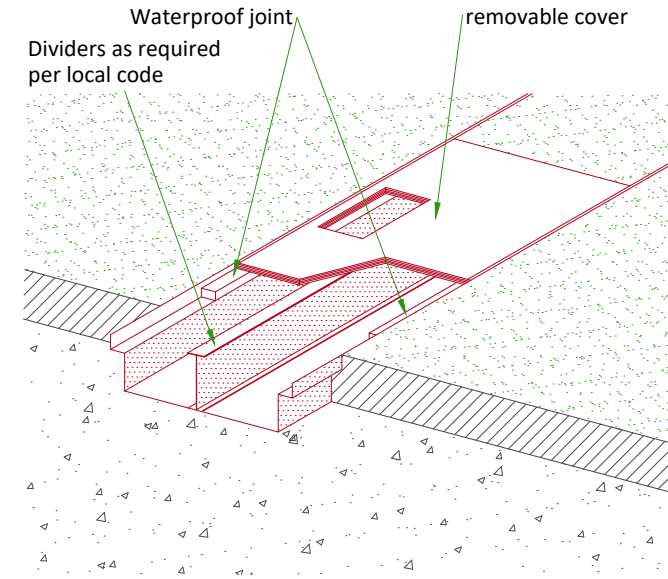

WALL DUCT

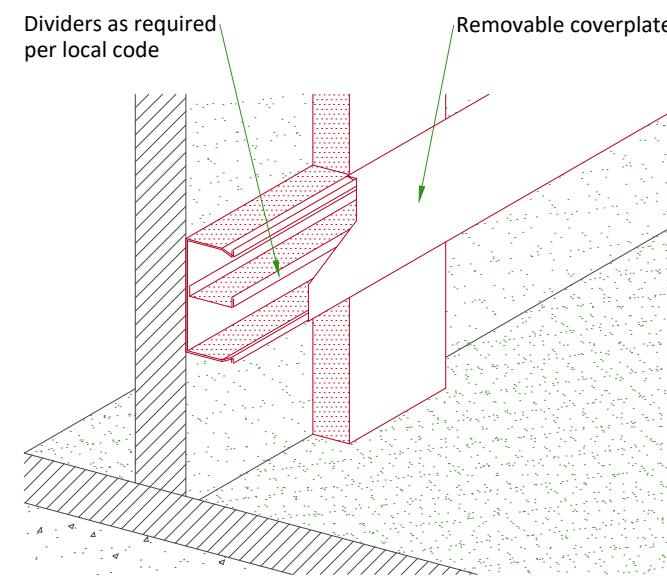

NOT TO SCALE

POWER AND NETWORK REQUIREMENTS

| | |
|----------------|---|
| POWER SUPPLY | 3 PHASES+G 380/400/420/440/460/480 VAC ±10% |
| FREQUENCIES | 50/60 Hz ± 3 Hz |
| POWER DEMAND | 70 kVA |
| LINE IMPEDANCE | 380V : 0.15 Ohm / 400V : 0.16 Ohm / 420V : 0.18 Ohm |
| | 440V : 0.20 Ohm / 460V : 0.22 Ohm / 480V : 0.24 Ohm |

- TNC neutral point connection must not be used.
- Line supply should come into a power distribution box (PDB) containing the protective units and controls.
- The section of the supply cable should be calculated in accordance with its length and the maximum permissible voltage drops.
- There must be discrimination between supply cable protective device at the beginning of the installation (main low-voltage transformer side) and the protective devices in the PDB.

SUPPLY CHARACTERISTICS

- Power input must be separated from any others which may generate transients (elevators, air conditioning, radiology rooms equipped with high speed film changers...)
- All equipment (lighting, power outlets, etc...) installed with GE system components must be powered separately.

GROUND SYSTEM

- Equipotential: the equipotential link will be by means of an equipotential bar. This equipotential bar should be connected to the protective earth conductors in the ducts of the non GE cableways and to additional equipotential connections linking up all the conducting units in the rooms where GE units are located.

CABLES

- Power and cable installation must comply with the distribution diagram below.
- All cables must be isolated and flexible, cable color codes must comply with standards for electrical installation.
- Case PDB furnished by GE: The cables for signals and remote control (Y, SEO, L...) will go to PDB with a pigtail length of 1.5 m, and will be connected during installation. Each conductor will be identified and isolated (screw connector).

CABLEWAYS


The general rules for laying cableways should meet the conditions laid down in current standards and regulations, with regard to:

- Protecting cables against water (cableways should be waterproof)
- Protecting cables against abnormal temperatures (proximity to heating pipes or ducts)
- Protecting cables against temperature shocks
- Replacing cables (cableways should be large enough for cables to be replaced)
- Metal cableways should be grounded.

NETWORK REQUIREMENTS

1 RJ 45 socket is required. The network connection is made at the Operator Console. 100BaseT network connection is preferred. 10BaseT network connection is acceptable.

POWER DISTRIBUTION

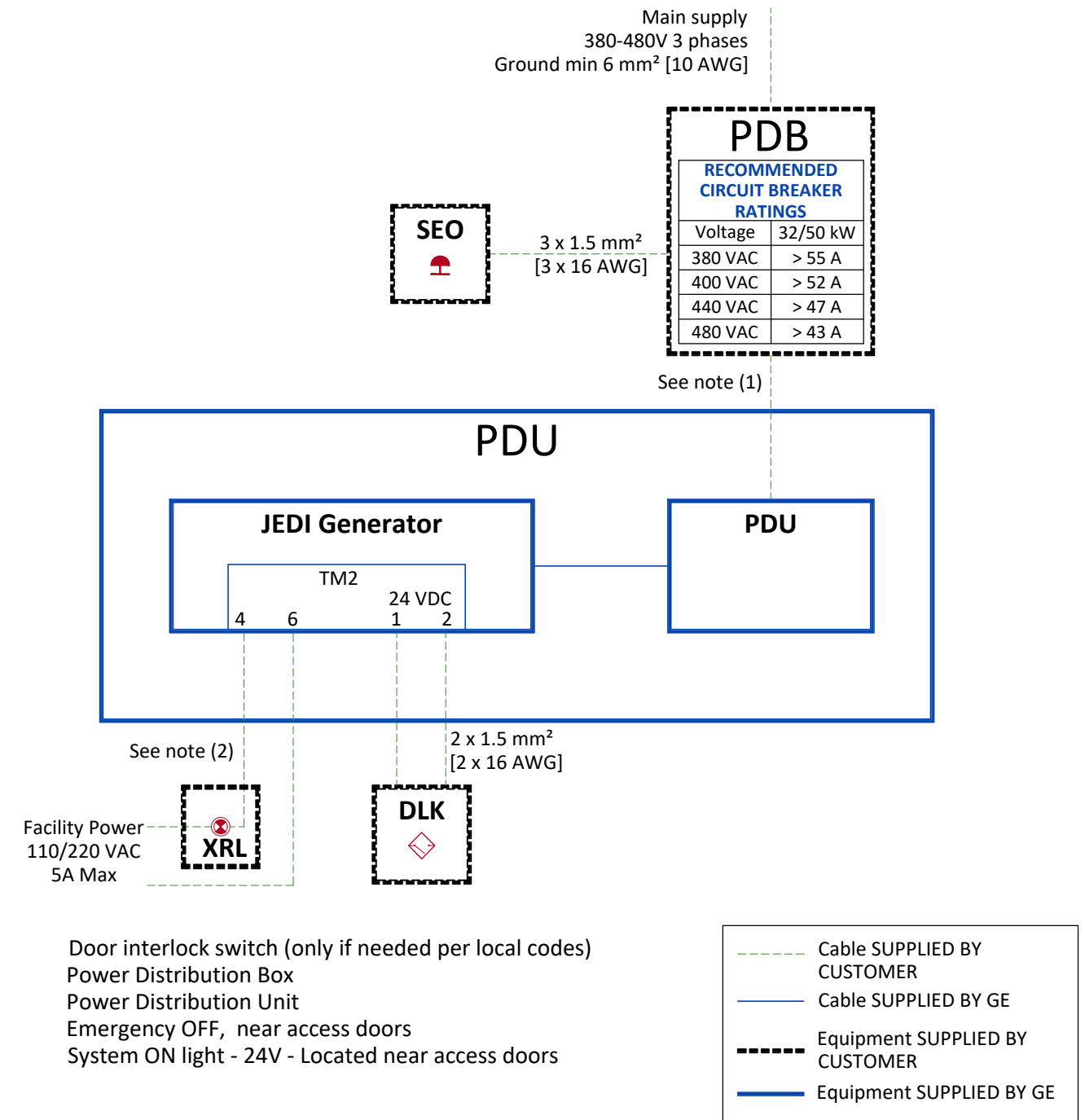

Notes

(1) Use only a multi conductor, shielded, PVC/PVC, UL TYPE CM cable. Alpha Wire. CQA10210637. This wire is found in GE Catalog Item A8091JH as a “bulk” roll of wire (60 Meters). Material consists of two AWG (19/0.0117 strand) conductors. Shields must be grounded at both ends. CHINA ONLY: available via from GE via part number S39222KP.

(2) Warning light grounding not shown, must comply with local codes.

INCOMING FACILITY POWER FEEDER TABLE

| MIN. FEEDER WIRE SIZE, AWG OR MCM (sq. mm)/VAC | MINIMUM FEEDER WIRE LENGTH - ft (m) | | | | | | | |
|--|-------------------------------------|----------|----------|----------|--|--|--|--|
| | 50 (15) | 100 (30) | 150 (46) | 200 (60) | | | | |
| 32 kW 380 VAC | 8 (10) | 6 (16) | 6 (16) | 5 (22) | | | | |
| 50 kW 380-480 VAC | 8 (10) | 6 (16) | 4 (22) | 3 (30) | | | | |
| GENERAL NOTES | | | | | | | | |
| In all cases qualified personnel must verify that the feeder (at the point of take-off) and the run to the GE system meet all the requirements stated in the PIM | | | | | | | | |
| Grounding conductor will be of the same size as the feeder. This ground will run from the equipment back to the power source/main grounding point and always travel in the same conduit with the feeders | | | | | | | | |

TEMPERATURE AND HUMIDITY SPECIFICATIONS

IN-USE CONDITIONS

| | EXAM ROOM | | CONTROL ROOM | |
|-------------------------|-----------------------|----------------|-----------------------|----------------|
| | Min | Max | Min | Max |
| Temperature | 10 °C [50 °F] | 40 °C [104 °F] | 10 °C [50 °F] | 40 °C [104 °F] |
| Temperature gradient | < 10 °C/h [< 50 °F/h] | | < 10 °C/h [< 50 °F/h] | |
| Relative humidity (1) | 30% to 80% | | 30% to 80% | |
| Humidity gradient | < 30%/h | | < 30%/h | |
| System heat dissipation | Idle | Max | Idle | Max |
| | 0.725 kW | 2.480 kW | 0.006 kW | 0.020 kW |
| | 2474 BTU/hr | 8462 BTU/hr | 20 BTU/hr | 68 BTU/hr |

STORAGE CONDITIONS

| | |
|-----------------------|----------------------------------|
| Temperature | -20 °C [-4 °F] to 70 °C [158 °F] |
| Relative humidity (1) | 20% to 90% |
| Temperature gradient | < 20 °C/h [< 68 °F/h] |
| Humidity gradient | < 30%/h |

Material should not be stored for more than 90 days.

(1) non-condensing

AIR RENEWAL


According to local standards.

NOTE

In case of using air conditioning systems that have a risk of water leakage it is recommended not to install it above electric equipment or to take measures to protect the equipment from dropping water.

STORAGE values only refer to equipment that is still in shipping containers. If the equipment is partially or completely installed, refer to IN-USE values.

INTERCONNECTIONS

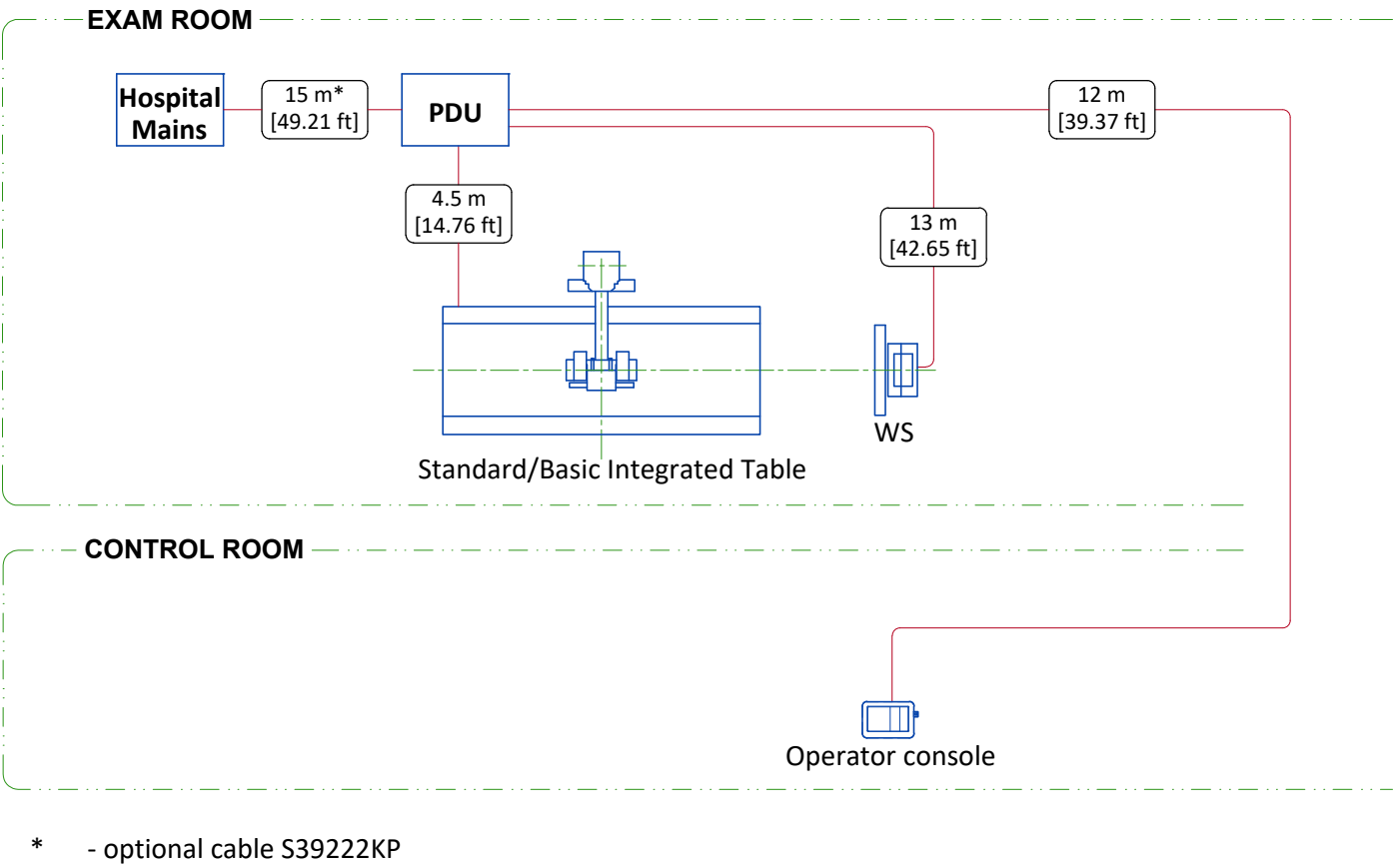

ENVIRONMENTAL SPECIFICATIONS

MAGNETIC INTERFERENCE

To guarantee specified imaging performance:
X-ray tubes and control console equipment must be located in ambient static field of less than 10 Gauss.

ACOUSTIC OUTPUT

Measured 1 m [3.28 ft] from any point in system.

In-use: less than 65 dBA

Stand-by: less than 55 dBA

DELIVERY

THE CUSTOMER/CONTRACTOR SHOULD:

- Provide an area adjacent to the installation site for delivery and unloading of the GE equipment.
- Ensure that the dimensions of all doors, corridors, ceiling heights are sufficient to accommodate the movement of GE equipment from the delivery area into the definitive installation room.
- Ensure that access routes for equipment will accommodate the weights of the equipment and any transportation, lifting and rigging equipment.
- Ensure that all necessary arrangements for stopping and unloading on public or private property not belonging to the customer have been made.

| DIMENSIONS | | | |
|-------------------|--------|---------------------|------------------|
| | SYSTEM | STANDARD WALL STAND | BASIC WALL STAND |
| LENGTH (mm) | 2500 | 2000 | 2000 |
| WIDTH (mm) | 1250 | 700 | 700 |
| HEIGHT (mm) | 1000 | 400 | 400 |
| GROSS WEIGHT (kg) | 682 | 117 | 109 |
| LENGTH (in) | 98.4 | 78.7 | 78.7 |
| WIDTH (in) | 49.2 | 27.6 | 27.6 |
| HEIGHT (in) | 39.4 | 15.7 | 15.7 |
| GROSS WEIGHT (lb) | 1504 | 258 | 240 |

Shipping boxes are included.

DISCLAIMER

GENERAL SPECIFICATIONS

- GE is not responsible for the installation of developers and associated equipment, lighting, cassette trays and protective screens or derivatives not mentioned in the order.
- The final study contains recommendations for the location of GE equipment and associated devices, electrical wiring and room arrangements. When preparing the study, every effort has been made to consider every aspect of the actual equipment expected to be installed.
- The layout of the equipment offered by GE, the dimensions given for the premises, the details provided for the pre-installation work and electrical power supply are given according to the information noted during on-site study and the wishes expressed by the customer.
- The room dimensions used to create the equipment layout may originate from a previous layout and may not be accurate as they may not have been verified on site. GE cannot take any responsibility for errors due to lack of information.
- Dimensions apply to finished surfaces of the room.
- Actual configuration may differ from options presented in some typical views or tables.
- If this set of final drawings has been approved by the customer, any subsequent modification of the site must be subject to further investigation by GE about the feasibility of installing the equipment. Any reservations must be noted.
- The equipment layout indicates the placement and interconnection of the indicated equipment components. There may be local requirements that could impact the placement of these components. It remains the customer's responsibility to ensure that the site and final equipment placement complies with all applicable local requirements.
- All work required to install GE equipment must be carried out in compliance with the building regulations and the safety standards of legal force in the country concerned.
- These drawings are not to be used for actual construction purposes. The company cannot take responsibility for any damage resulting therefrom.

CUSTOMER RESPONSIBILITIES

- It is the responsibility of the customer to prepare the site in accordance with the specifications stated in the final study. A detailed site readiness checklist is provided by GE. It is the responsibility of the customer to ensure all requirements are fulfilled and that the site conforms to all specifications defined in the checklist and final study. The GE Project Manager of Installation (PMI) will work in cooperation with the customer to follow up and ensure that actions in the checklist are complete, and if necessary, will aid in the rescheduling of the delivery and installation date.
- Prior to installation, a structural engineer of record must ensure that the floor and ceiling is designed in such a way that the loads of the installed system can be securely borne and transferred. The layout of additional structural elements, dimensioning and the selection of appropriate installation methods are the sole responsibility of the structural engineer. Execution of load bearing structures supporting equipment on the ceiling, floor or walls are the customer's responsibility.

RADIO-PROTECTION

- Suitable radiological protection must be determined by a qualified radiological physicist in conformation with local regulations. GE does not take responsibility for the specification or provision of radio-protection.

| | | |
|---|------|-----------|
| THE UNDERSIGNED, HEREBY CERTIFIES THAT I HAVE READ AND APPROVED THE PLANS IN THIS DOCUMENT. | | |
| DATE | NAME | SIGNATURE |
| | | |

GLOBAL SITE READINESS CHECKLIST (DI)

DOC1809666 Rev. 7

| Site Ready Checks at Installation |
|---|
| EHS Site Requirements |
| Overall access route to the scan room free from obstruction / high hazards. |
| Enough space to store tools, equipment, parts, install waste and the general area free from obstruction and trip hazards. |
| Enough necessary facilities for the GE employees available. |
| No 3rd parties working in the area that may affect the safety of the installation activity. |
| Area free from any chemical, gas, dust, welding fume exposure and has painting been completed and dry. |
| All emergency routes identified, signed and clear from obstruction. |
| Accessible single source lockable panel that LOTO can be applied to for GE equipment installation (MDP and/or PDU). |
| There are no other conditions or hazards that you have observed or have been made aware of by the customer or contractors on site. |
| Required for Mechanical Install start |
| Room dimensions, including ceiling height, for all Exam, Equipment/Technical & Control rooms meets GE specifications. |
| Ceiling support structure, if indicated on the GE drawing, is in the correct location and at the correct height according to the Original Equipment Manufacturer specifications. |
| Levelness and spacing has been measured, and is ready for the installation of any GE supplied components. |
| Overhead support Structure (unistrut) has been confirmed with customer/contractor to meet required GE provided criteria. |
| Finished ceiling is installed. If applicable ceiling tiles installed per PMI discretion. |
| Floor levelness/flatness is measured and within tolerance, and there are no visible defects per GEHC specifications. |
| Entry door threshold meets PIM requirement. |
| Rooms that will contain equipment, including staging areas if applicable, are construction debris free. Precautions must be taken to prevent debris from entering rooms containing equipment. |
| Cable ways (floor/wall/ceiling/Access Flooring) are available for installation of GE cables are of correct length and diameter. |
| Cable ways routes per GE Final drawings and cable access openings areas installed at a time determined by GEHC PM. Surface floor duct can be installed at time of system installation. |
| Adequate room illumination installed and working. |
| Customer supplied countertops where GE equipment will be installed are in place. |
| Required for Calibration Start |
| HVAC systems Installed, and the site meets minimum environmental operational system requirements. |
| System power & grounding (PDB/MDP) is available as per GE specifications. |
| System power & grounding (PDB/MDP) is installed at point of final connection and ready to use. Lock Out Tag Out is available. |
| PMI to confirm all feeder wires and breaker are size appropriately. EPO installed if needed. |
| PMI to confirm with electrician all power and signal cables are well terminated ensuring there are no loose connections. |
| Network outlets installed. |
| Computer network available and working. |
| Lead doors and windows complete or scheduled to be installed. If applicable, radiation protection (shielding) finished & radioprotection regulatory approval for installation obtained. |

Note: The details shown here are only an extract from DOC1809666. For the complete document please contact your PMI.