

Bone & Metabolic Health

# Prodigy from GE Healthcare

Performance and reliability with one of the largest installed base DXA systems in the world

gehealthcare.com

## Prodigy

High performance, efficient and reliable DXA system with the versatility to offer bone density test and body composition analysis. Prodigy systems provide the option to scale up to a wide-range of clinical applications.



Your practice demands dependable dual-energy X-ray absorptiometry (DXA) assessment, and Prodigy delivers with exceptional precision and low-dose radiation. You can depend on Prodigy to provide precise data on bone and soft tissue composition, including bone-mineral density (BMD), lean- and fat-tissue mass, and percentage of fat. At the same time, Prodigy helps streamline your patient care and practice workflow.

#### **Prodigy software packages - Customized to suit your needs**

#### **Prodigy Primo**



#### Package includes:

- Basic Skeletal
   Assessment
- Basic Body
   Composition

#### **Prodigy Pro**



#### Package includes:

- Essential Skeletal Assessment
- Essential Body
   Composition Assessment
- Pediatric Measurements

#### **Prodigy Advance**



#### Package includes:

- Advanced Skeletal Assessment
- Advanced Body Composition Assessment
- Pediatric Measurements
- Multi-User Database



## Solid performance makes Prodigy chosen worldwide

Prodigy is our proven and dependable DXA product with a large global installed base across 120+ countries.

Clinicians, Researchers and Practitioners have trusted the Prodigy DXA system for more than 20 years making it one of the largest selling DXA systems in the world.

## Prodigy with enCORE v18 software enables you to meet ISCD testing guidelines

#### ISCD indications for Bone Mineral Density (BMD) testing:

- Women 65 and older
- Men 70 and older
- Post-menopausal women with a risk factor:
- Low body weight
- Prior fracture
- High risk medication use

- Adults taking medications associated with low bone mass or bone loss
- Adults with disease or condition associated with low bone mass or bone loss


ISCD guidelines for BMD testing available at ISCD.org



## Prodigy for Bone Health

Prodigy's reliable design and robust technology platform supports a comprehensive portfolio of clinical applications for bone health.

- Bone Mineral Density
- FRAX
- Trabecular Bone Score (TBS)
- DVA (includes LVA, APVA and Lateral BMD)
- Atypical Femur Fracture and more

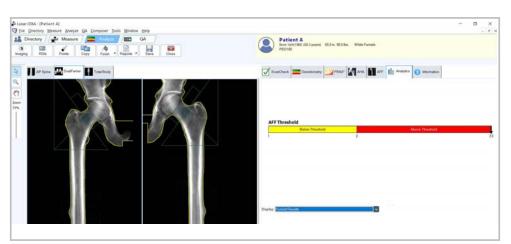


**BMD Insights using AFF User Customizable Threshold** 

## Prodigy for Metabolic Health

Prodigy offers a wide range of clinical applications for metabolic health needs.

- Total Body Composition
- CoreScan
- Sarcopenia
- Fat Color Coding
- Custom Reference Population
- Option to Integrate Hydration Levels from BIA/BIS (TBW, ECW, ICW) to have 5 compartment models (LM, FM, BMC, ECW, ICW) and more

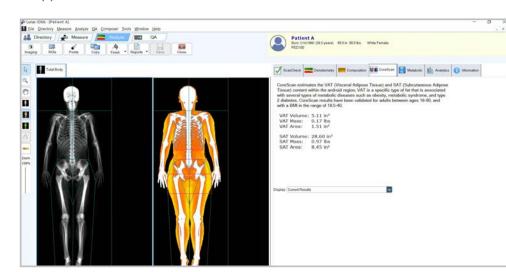


**Newly Available: VAT Area and SAT Results** 

 $_{4}$ 

## A Few of our Newest Applications

#### A wide breadth of applications and features

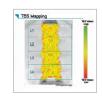

#### **Trabecular Bone Score - Integrated TBS**

Provides TBS score based on assessment of trabecular region of bone, including FRAX-adjusted TBS. Includes TBS license. Complimentary 60-day TBS software trial program available for new TBS customers.




#### **Smaller Body Composition (Regions of** Interest)4

Monitor and report on Regions of Interest (ROI) including upper arm, lower arm, upper leg and lower leg, to study changes in body composition in these regions.



#### DXAVision™

Provides one unified workflow and comprehensive reporting for BMD, AFF, VAT and SAT.¹ Designed to improve operator efficiency with a scan time up to 40% faster.2 Includes Total Body and Smaller Body (ROI) Composition, Total Body Less Head (TBLH) and Neck-to-Knee for Adults.

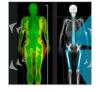

#### **Advanced Analytics**

Provides deep BMD and Body Composition insights with custom equations, metrics and ratios based on 200+ DXA bone and body composition parameters. User-defined classification thresholds, trending and reporting.



#### **Neck-to-Knee for Adults**<sup>3</sup>

Performs a faster scan by omitting head and lower legs, providing an estimate of total body composition.



#### **Customizable Thresholds** (AFF and VAT)5,6,7

Enables setting of custom thresholds to search for correlations: between "beaking" and the probability of AFF, and between VAT and the probability of metabolic disorders.



#### Total Body Less Head (TBLH) for Adults<sup>4</sup>

Including the skull can mask changes occurring in other areas of the skeleton; this tool automatically performs a scan from the neck down. Can also get TBLH results for scans with the head included.



#### **Composer Reporting**

Provides default style sheets, which can be edited using an intuitive WYSIWYG interface to quickly produce customized reports and templates.



#### **CoreScan with VAT and SAT Results**

CoreScan estimates Visceral and Subcutaneous Adipose Tissue (VAT and SAT) mass, volume and area within the android region. Values can be displayed in userdefined statistical formats and trends.



#### **Advanced Cybersecurity**

Advanced cybersecurity features that meet U.S. Department of Defense Risk Management (DoD RMF) requirements.



#### **Sports Athletics Package**

Includes TBLH (Total Body Less Head) for Adults and Smaller Body Comp - ROI to easily scan and report on specific Regions of Interest. Facilitates study of localized changes in body composition.




Multi-User Database with a Secure Platform



#### Advanced security features protect your data.


| Security Feature       | Provided Benefit   |
|------------------------|--|
| IPv6 for DICOM and HL7 | Communication protocol integrating IPSec for better security during data exchange  |
| FIPS 140-2 Encryption  | Federally compliant encryption standard that protects patient exam files using 256-bit encryption  |
| Audit Trails           | Logs information related to:  Software configuration and user access changes, destination IP addresses  Database events including authentication, patient modification/deletion  Events supported by the DICOM Audit Trail Profile |
| TLS for DICOM®         | Provides security at the transport layer of a DICOM transaction by using encryption and node authentication. TLS is an updated, more secure, version of the SSL protocol.  |

## Detailed assessment in just a few clicks

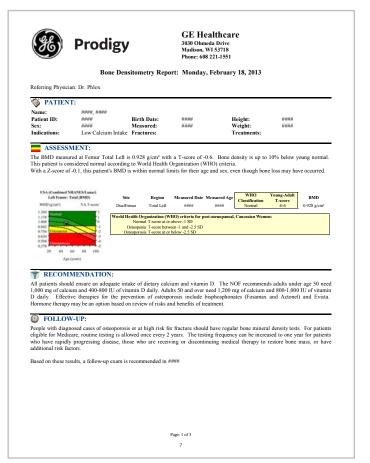

Fully customizable reports can be made as concise or as detailed as needed.

Treatment recommendations designated by the physician are automatically added and can include society guidelines.

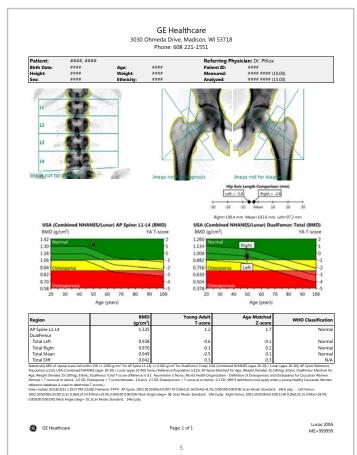

OneScan performs spine and dual femur BMD measurements in a single protocol without repositioning. Results print in a one-page report.



General purpose business reporting tools help you manage your practice. Prodigy will automatically:


- Generate referring physician letters
- Analyze populations and trends
- Export data to tab-delimited text files for use in Microsoft Excel®

### Robust Technology Foundation

#### Third generation DXA technology - with over 30 years of innovations

#### **Patented Narrow Fan Beam Scan**

Combining the features of pencil beams and wide fan beams, Narrow Fan Beam technology offers a shorter scan time with reduced magnification error (inherent to wide-angle fan beam scans).

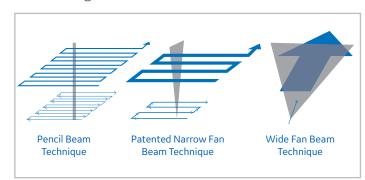

#### **Low-Dose Photon Counting Technology**

Dose-efficient photon counting detector technology more efficiently counts X-ray photons, lowering dosage to the patient.

#### Innovative SmartScan™

Our SmartScan technology reduces scan time and X-ray dosage by identifying bone regions after each transverse sweep and estimating where to begin scanning on the subsequent sweep.

#### K-edge Filter

An exceptional "K-edge filter" that creates a dual energy beam and absorbs the X-rays in the middle energy range and protects the patient against unnecessary exposure.

#### **Multi-View Image Reconstruction (MVIR)**

By performing multiple transverse sweeps across the site of interest, MVIR accurately determines bone-height above the tabletop, minimizes magnification errors and provides excellent precision and accuracy.

#### **Low Scattered Radiation**

Narrow-fan beam technology results in low scatter radiation in comparison to wide-angle fan beam systems.<sup>9</sup>

#### Performance comparison of DXA beam types

|                        | Pencil Beam | Narrow Fan Beam | Wide Fan Beam |
|------------------------|-------------|-----------------|---------------|
| Scan time              | Long        | Short           | Short         |
| Bone height measured   | No          | Yes             | No            |
| Magnification effects  | No          | No              | Yes           |
| Off-center distortions | No          | No              | Yes           |
| Scattered radiation    | Lowest      | Low             | High          |

8

Advance\*

#### × Not Available Optional Standard Primo Pro AP Spine ~ ~ Femur/Dual Femur **/** Forearm/Non-seated Forearm Total Body BMD\* FRAX® Fracture Risk Tool Multi-User Database (1-3) ScanCheck Practice Management Composer Report Tool OneScan OneVision Pediatric - AP Spine Pediatric - Femur Pediatric - TB (Birth to 20 YO)\* Total Body Composition\* DVA (Includes: LVA, APVA, Lateral BMD) CoreScan\* Advanced Body Composition<sup>10\*</sup> × Orthopedic Hip Advanced Hip Analysis X Orthopedic Knee Hand X Multi-User Database (Up to 40) Atypical Femur Fracture X Sarcopenia\*

| Integrated TBS                | • | • | • |
|-------------------------------|---|---|---|
| Advanced Security (DoD RMF)   |   | • | • |
| DXAVision™*                   | × | • | • |
| Sports Athletics Package*     | × | • | • |
| Advanced Analytics Full*      | × | • | • |
| Advanced Analytics Bone*      | × | • | • |
| Advanced Analytics Body Comp* | × | • |   |


X


X


X

## Specifications

#### **Scanner dimensions:**

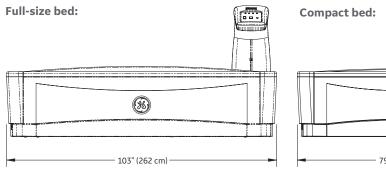


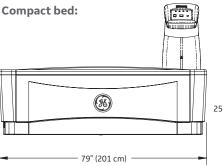

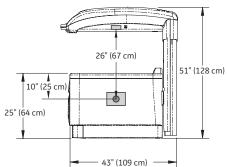


#### Minimum room dimensions<sup>11</sup>:

#### Full-size bed:

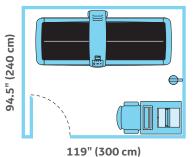

#### Compact bed:

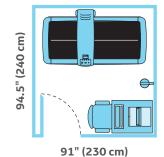

#### Scanner table specifications:

| Scanner size (full-size bed)                              |
|---|
| •   |
| Scanner leakage current meets IEC 60601-1 safety standard |
|   |

#### **Computer specifications:**

- Intel® Core™ i3 Processor
- Windows® 10 IoT Enterprise 64-bit
- RAM 8 GB
- Hard drive 1 TB
- Optical drive DVD-RW
- Monitor 24" SVGA (minimum resolution 1920 x 1080 32-bit color)
- Archive 500 GB USB hard drive
- Adobe® Reader® DC
- Internet Explorer® 11
- Serial port onboard RS-232 115k baud DB
- Windows®-compatible printer

#### **Detector specifications:**

Detector ...... LYSO X-ray counting detector

#### **Environmental specifications:**

| Power               | .100-127 VAC 50/60 Hz 20A dedicated circui |
|---------------------|--|
|                     | 200-240 VAC 50/60 Hz 10A dedicated circuit |
| Consumption         | idling 40VA, scanning 450VA                |
| Distortion          | sinusodal waveform, less than 5% THD       |
| Humidity            |  |
| Room temperature    | 65°F-81°F (18°C-27°C                       |
| Scanner heat output | idling 150 BTU/hr, scanning 1500 BTU/h     |
| Console heat output | approx. 200 BTU/hr with 24" monitor        |
| Ventilation         | all cooling vents must remain unblocked    |
| Dust, fumes, debris | install system in clean, ventilated area   |
|                     |  |

#### **Connectivity:**

- Teledensitometry<sup>12</sup>
- DICOM® interface
- HL7 interface
- SQL Server



\*Not available in Compact size

Small Animal

Quick View (10 second scan)

**APPLICATIO** 

NEW V18
PPLICATIONS

10

#### **References:**

- 1. Requires purchase of AFF application and Corescan (for VAT and SAT) application.
- 2. Data on file with GE Healthcare, April 2019.
- 3. Requires DXAVision.™
- 4. Requires DXAVision™ or Sports Athletics Package.
- 5. Requires Advanced Analytics.
- 6. Customizable Threshold for AFF requires AFF Application.
- 7. Customizable Threshold for VAT requires CoreScan application.
- S.M. Hunt et al, "Changing Bone Densitometers in Clinical Practice: Effect on Precision Error", Presented at the American Society for Bone and Mineral Research Annual Meeting, September 23-27, 2005, Nashville, TN, USA.
- 9. Data on file with GE Healthcare, January 2017.
- Bone-Lean-Tissue Color Coding, Metabolic Results (ICW, ECW, TBW), Resting Metabolic Rate, Composer Style Sheets – Body Sports Medicine Segmental, Body Patient Weight Loss.
- 11. A small room kit with isolation transformer may be required. Please refer to local regulations.
- 12. Additional hardware may be required for fax capabilities.



© 2020 General Electric Company - All rights reserved.

GE Healthcare reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your GE Healthcare representative for the most current information. GE, the GE Monogram, DXAVision and SmartScan are trademarks of General Electric Company. GE Healthcare, a division of General Electric Company. GE Medical Systems, Inc., doing business as GE Healthcare. FRAX is a registered trademark of the World Health Organization Collaborating Centre for Metabolic Bone Diseases, University of Sheffield, UK. DICOM is a trademark of National Electrical Manufacturers Association. Windows and Internet Explorer are a registered trademarks of Microsoft Corporation. All other third party trademarks are the property of their respective owners.