

Technical Publications

**Direction DOC2107162
Revision 4**

**LOGIQ E10 R2.x
LOGIQ E10s R2.x
LOGIQ E11 R2.x
LOGIQ E20 R2.x**

DICOM CONFORMANCE STATEMENT

g

GE Healthcare

Copyright© 2020 by General Electric Co.

Do not duplicate



GE Healthcare

THIS PAGE LEFT INTENTIONALLY BLANK

CONFORMANCE STATEMENT OVERVIEW

LOGIQ scanner is an Ultrasound Scanner acting as an acquisition modality in a DICOM network. The LOGIQ Series compliant with DICOM 3.0.

Table 0.1 provides an overview of the network services supported by LOGIQ scanner.

Table 0.1 – NETWORK SERVICES

SOP Classes	User of Service (SCU)	Provider of Service (SCP)
Transfer		
Ultrasound Image Storage	Yes	Yes
Ultrasound Image Storage (Retired)	Yes	Yes
Ultrasound Multi-frame Image Storage	Yes	Yes
Ultrasound Multi-frame Image Storage (Retired)	Yes	Yes
Enhanced US Volume Storage	Yes	Yes
Secondary Capture Image Storage	Yes	Yes
Comprehensive SR	Yes	No
Key Object Selection Document	Yes	No
Verification SOP Class	Yes	Yes
CT Image Storage	Yes	Yes
MR Image Storage	Yes	Yes
Positron Emission Tomography Image Storage	Yes	Yes
Digital Mammography X-Ray Image Storage – For Presentation	Yes	Yes
Digital Mammography X-Ray Image Storage – For Processing	Yes	Yes
X-Ray Angiographic Image Storage	Yes	Yes
Query/Retrieve		
Study Root Query/Retrieve Information Model – FIND	Yes	No
Study Root Query/Retrieve Information Model – MOVE	Yes	No
Print Management		
Basic Grayscale Print Management Meta SOP Class	Yes	No
Basic Color Print Management Meta SOP Class	Yes	No

SOP Classes	User of Service (SCU)	Provider of Service (SCP)
Basic Annotation Box	Yes	No
Printer SOP Class	Yes	No
Basic Film Session SOP Class	Yes	No
Basic Film Box SOP Class	Yes	No
Basic Grayscale Image Box SOP Class	Yes	No
Basic Color Image Box SOP Class	Yes	No
Workflow Management		
Storage Commitment Push Model SOP Class	Yes	No
Modality Performed Procedure Step SOP Class	Yes	No
Modality Worklist Information Model – FIND SOP Class	Yes	No

Table 0.2 provides an overview of the Media Storage Application Profiles supported by LOGIQ scanner.

Table 0.2 - MEDIA SERVICES

Media Storage Application Profile	Write Files (FSC or FSU)	Read Files (FSR)
Compact Disk - Recordable		
General Purpose CD-R	Yes	Yes
DVD		
General Purpose DVD Interchange with JPEG	Yes	Yes
General Purpose DVD Interchange with JPEG 2000	Yes	Yes
USB		
General Purpose USB Media Interchange with JPEG	Yes	Yes
General Purpose USB Media Interchange with JPEG 2000	Yes	Yes

TABLE OF CONTENTS

1. INTRODUCTION.....	12
1.1 Overview.....	12
1.2 Overall DICOM Conformance Statement Document Structure	13
1.3 Intended Audience	15
1.4 Scope and Field of Application.....	15
1.5 Important Remarks.....	15
1.6 References.....	16
1.7 Definitions.....	16
1.8 Symbols and Abbreviations	18
2. NETWORK CONFORMANCE STATEMENT.....	20
2.1 Introduction	20
2.2 Implementation Model.....	20
2.3 AE Specifications	22
2.4 COMMUNICATION PROFILES	42
2.5 EXTENSIONS / SPECIALIZATIONS / PRIVATIZATIONS	43
2.6 CONFIGURATION	44
2.7 SUPPORT OF EXTENDED CHARACTER SETS	46
2.8 CODES AND CONTROLLED TERMINOLOGY	46
2.9 SECURITY PROFILES.....	46
3. MEDIA STORAGE CONFORMANCE STATEMENT	48
3.1 INTRODUCTION	48
3.2 IMPLEMENTATION MODEL.....	48
3.3 AE SPECIFICATIONS.....	49

3.4	AUGMENTED AND PRIVATE APPLICATION PROFILES	57
3.5	EXTENSIONS, SPECIALIZATIONS, PRIVATIZATIONS OF SOP CLASSES AND TRANSFER SYNTAXES	57
3.6	CONFIGURATION	59
3.7	SUPPORT OF EXTENDED CHARACTER SETS	59
4.	ULTRASOUND IMAGE INFORMATION OBJECT IMPLEMENTATION.....	60
4.1	Introduction	60
4.2	LOGIQ Scanner Mapping of DICOM Entities	60
4.3	IOD Module Table.....	60
4.4	Information Module Definitions.....	61
5.	ULTRASOUND MULTI-FRAME IMAGE INFORMATION OBJECT IMPLEMENTATION	77
5.1	Introduction	77
5.2	LOGIQ Scanner Mapping of DICOM Entities	77
5.3	IOD Module Table.....	77
5.4	Information Module Definitions.....	78
6.	ENHANCED US VOLUME INFORMATION OBJECT IMPLEMENTATION	80
6.1	Introduction	80
6.2	LOGIQ Scanner Mapping of DICOM Entities	80
6.3	IOD Module Table.....	80
6.4	Information Module Definitions.....	82
7.	SECONDARY CAPTURE INFORMATION OBJECT IMPLEMENTATION	92
7.1	Introduction	92
7.2	LOGIQ Scanner Mapping of DICOM Entities	92
7.3	IOD Module Table.....	92

7.4	Information Module Definitions.....	93
8.	COMPREHENSIVE STRUCTURED REPORT INFORMATION OBJECT IMPLEMENTATION.....	95
8.1	Introduction	95
8.2	LOGIQ Scanner Mapping of DICOM Entities	95
8.3	IOD Module Table.....	95
8.4	Information Module Definitions.....	96
8.5	Standard Extended and Private Data Attributes	103
8.6	Standard Extended and Private Context Groups.....	103
8.7	Standard, Standard Extended and Private Templates	103
9.	ENCAPSULATED PDF INFORMATION OBJECT IMPLEMENTATION.....	105
9.1	Introduction	105
9.2	LOGIQ Scanner Mapping of DICOM Entities	105
9.3	IOD Module Table.....	105
9.4	Information Module Definitions.....	106
9.5	Standard Extended and Private Data Attributes	107
10.	KEY OBJECT SELECTION DOCUMENT INFORMATION OBJECT IMPLEMENTATION.....	108
10.1	Introduction	108
10.2	LOGIQ Scanner Mapping of DICOM Entities	108
10.3	IOD Module Table	108
10.4	Information Module Definitions	109
10.5	Standard, Standard Extended and Private Templates	111
11.	MODALITY WORKLIST INFORMATION MODEL DEFINITION.....	112
11.1	Introduction	112

11.2	Modality Worklist Information Model Description.....	112
11.3	Modality Worklist Information Model Entity-Relationship Model.....	112
11.4	Information Model Module Table	114
11.5	Information Model Keys.....	115
12.	MODALITY PERFORMED PROCEDURE STEP IMPLEMENTATION	123
12.1	Introduction	123
12.2	MODALITY PERFORMED PROCEDURE STEP MODULE TABLE.....	123
12.3	MODALITY PERFORMED PROCEDURE STEP MODULE DEFINITIONS	123
12.4	SOP Common Module	123
12.5	Performed Procedure Step Relationship Module	124
12.6	Performed Procedure Step Information Module	126
12.7	Image Acquisition Results Module	127
13.	STORAGE COMMITMENT PUSH MODEL IMPLEMENTATION.....	128
13.1	Introduction	128
13.2	Storage Commitment Push Model Sop Class Definition	128
14.	BASIC DIRECTORY INFORMATION OBJECT IMPLEMENTATION	133
14.1	Introduction	133
14.2	Basic Directory Iod Implementation	133
14.3	Basic Directory Entity-Relationship Model.....	133
14.4	IOD Module Table	135
14.5	Information Module Definitions	135
15.	PRINT MANAGEMENT IMPLEMENTATION.....	145
15.1	INTRODUCTION.....	145
15.2	Basic Print Management Meta Sop Classes.....	145

15.3	Print Management Sop Class Definitions	146
15.4	Print Management Iods	149
15.5	Information Module Definitions	150
16.	STUDY ROOT QUERY/RETRIEVE INFORMATION MODEL	161
16.1	Introduction	161
16.2	Study Root Information Model Description	161
16.3	Study Root Information Model Entity-Relationship Model	161
16.4	Information Model Keys.....	162
16.5	Private Data Dictionary	167
17.	DICOM SR TEMPLATES.....	168
17.1	Vascular Ultrasound Procedure Report	168
17.2	Echocardiography Procedure Report	193
17.3	OB-GYN Ultrasound Procedure Report.....	231
17.4	Breast Imaging Report.....	256
17.5	Soft tissue neck and head Imaging Report.....	264
17.6	Key Object Selection Document Template	273
17.7	USER DEFINED TEMPLATE STRUCTURE	273
17.8	GE Ultrasound M&A Report.....	275
17.9	TID 300 Measurement	277
18.	GRAYSCALE DISPLAY CONSISTENCY	279
19.	AUDIT TRAIL MESSAGE FORMAT PROFILE.....	280
19.1	Supported Audit Message List	280
19.2	Audit Message Descriptiom.....	280

1. INTRODUCTION

1.1 OVERVIEW

This DICOM Conformance Statement is divided into Sections as described below:

Section 1 (Introduction), which describes the overall structure, intent, and references for this Conformance Statement

Section 2 (Network Conformance Statement), which specifies the GEHC equipment compliance to the DICOM requirements for the implementation of Networking features.

Section 3 (Media Storage Conformance Statement), which specifies the GEHC equipment compliance to the DICOM requirements for the implementation of Media Storage features.

Section 4 (Ultrasound Image Information Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of an Ultrasound Image Information Object.

Section 5 (Ultrasound Multi-Frame Image Information Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of an Ultrasound Multi-Frame Image Information.

Section 6 (Enhanced US Volume Information Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of an Enhanced US Volume Information.

Section 7 (Secondary Capture Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of a Secondary Capture Information Object.

Section 8 (Comprehensive Structured Report Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of a Comprehensive Structured Reporting Information Object.

Section 9 (Key Object Selection Document Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of a Key Object Selection Document Object.

Section 10 (Encapsulated PDF Information Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of a Encapsulated PDF Information Object.

Section 11 (Modality Worklist Information Model), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of the Modality Worklist service.

Section 12 (Modality Performed Procedure Step Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of Modality Performed Procedure Step Service.

Section 13 (Storage Commitment Push Model SOP Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of the Storage Commitment Push Model Service.

Section 14 (Basic Directory Information Object Implementation), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of a Basic Directory Information Object.

Section 15 (Print Management), which specifies the GEHC equipment compliance to DICOM requirements for the implementation of Basic Print Meta SOP Classes (Gray and Color).

Section 16 (Study Root Query/Retrieve Information Model), which specifies the GEHC equipment compliance to DICOM requirements for the Study Root Query/Retrieve Information Model.

Section 17 (DICOM SR Templates), which specifies all LOGIQ scanner supported DICOM SR Templates.

Section 18 (Grayscale Display Consistency), mentions about the Grayscale display consistency.

Section 19 (Audit Trail Message Format Profile), which specifies the list of Audit Messages implemented by this product.

1.2 OVERALL DICOM CONFORMANCE STATEMENT DOCUMENT STRUCTURE

The Documentation Structure of the GEHC DICOM Conformance Statements is shown in the Illustration below.

ILLUSTRATION 1–1

DOCUMENTATION STRUCTURE OF THE GEHC DICOM CONFORMANCE STATEMENTS

GEHC DICOM Conformance Statements


@

<https://www.gehealthcare.com/products/interoperability/dicom>

APPLICATION ENTITY SPECIFICATION


(SERVICE CLASSES, INFORMATION OBJECTS, MESSAGE EXCHANGES, ETC.)

**Product
Implementation:**

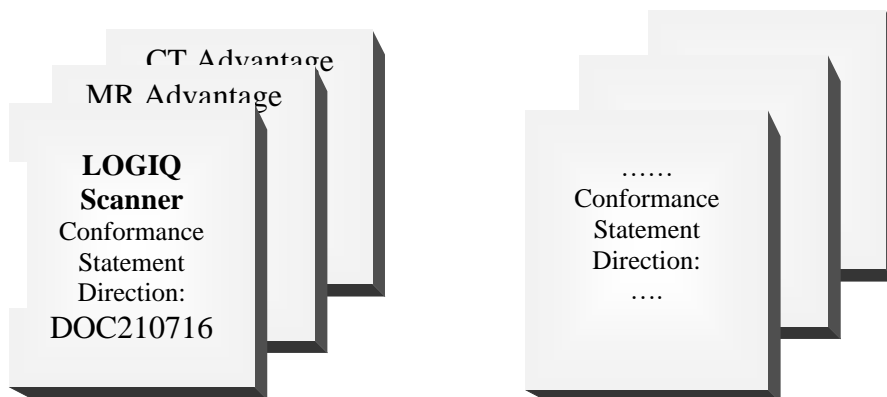

DICOM STANDARD

**Standard
Specification:**

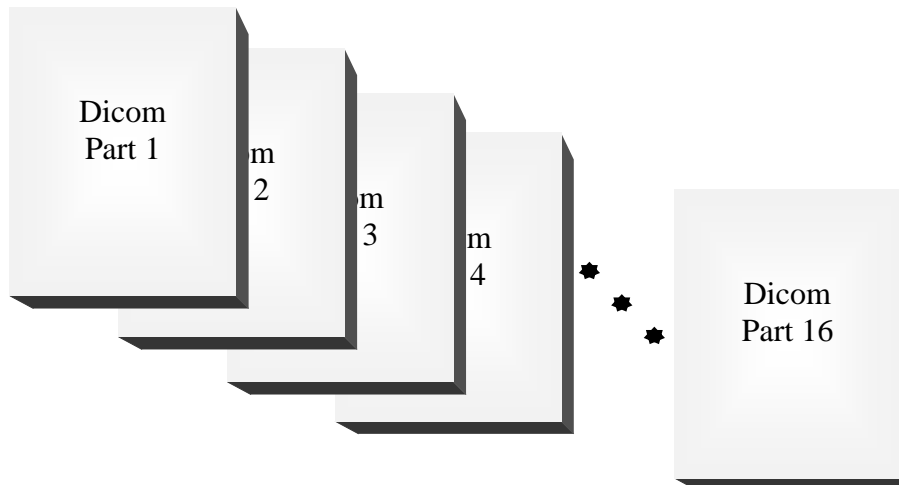

This document specifies the DICOM implementation. It is entitled:

LOGIQ Series
Conformance Statement for DICOM
Direction **DOC2107162**

This DICOM Conformance Statement documents the DICOM Conformance Statement and Technical Specification required to interoperate with the GEHC network interface.

The GEHC Conformance Statement, contained in this document, also specifies the Lower Layer communications which it supports (e.g., TCP/IP). However, the Technical Specifications are defined in the DICOM Part 8 standard.

For more information regarding DICOM, copies of the Standard may be obtained on the Internet at <https://www.dicomstandard.org/>. Comments on the Standard may be addressed to:

DICOM Secretariat
NEMA
1300 N. 17th Street, Suite 900
Arlington, VA 22209
USA

1.3 INTENDED AUDIENCE

The reader of this document is concerned with software design and/or system integration issues. It is assumed that the reader of this document is familiar with the DICOM Standard and with the terminology and concepts which are used in that Standard.

1.4 SCOPE AND FIELD OF APPLICATION

It is the intent of this document to provide an unambiguous specification for GEHC implementations. This specification, called a Conformance Statement, includes a DICOM Conformance Statement and is necessary to ensure proper processing and interpretation of GEHC medical data exchanged using DICOM. The GEHC Conformance Statements are available to the public.

The reader of this DICOM Conformance Statement should be aware that different GEHC devices are capable of using different Information Object Definitions. For example, a GEHC CT Scanner may send images using the CT Information Object, MR Information Object, Secondary Capture Object, etc. Included in this DICOM Conformance Statement are the Module Definitions which define all data elements used by this GEHC implementation. If the user encounters unspecified private data elements while parsing a GEHC Data Set, the user is well advised to ignore those data elements (per the DICOM standard). Unspecified private data element information is subject to change without notice. If, however, the device is acting as a "full fidelity storage device", it should retain and re-transmit all of the private data elements which are sent by GEHC devices.

1.5 IMPORTANT REMARKS

The use of these DICOM Conformance Statements, in conjunction with the DICOM Standards, is intended to facilitate communication with GE imaging equipment. However, **by itself, it is not**

sufficient to ensure that inter-operation will be successful. The **user (or user's agent)** needs to proceed with caution and address at least four issues:

Integration - The integration of any device into an overall system of interconnected devices goes beyond the scope of standards (DICOM v3.0), and of this introduction and associated DICOM Conformance Statements when interoperability with non-GE equipment is desired. The responsibility to analyze the applications requirements and to design a solution that integrates GE imaging equipment with non-GE systems is the **user's** responsibility and should not be underestimated. The **user** is strongly advised to ensure that such an integration analysis is correctly performed.

Validation - Testing the complete range of possible interactions between any GE device and non-GE devices, before the connection is declared operational, should not be overlooked. Therefore, the **user** should ensure that any non-GE provider accepts full responsibility for all validation required for their connection with GE devices. This includes the accuracy of the image data once it has crossed the interface between the GE imaging equipment and the non-GE device and the stability of the image data for the intended applications.

Such a validation is required before any clinical use (diagnosis and/or treatment) is performed. It applies when images acquired on GE imaging equipment are processed/displayed on a non-GE device, as well as when images acquired on non-GE equipment is processed/displayed on a GE console or workstation.

Future Evolution - GE understands that the DICOM Standard will evolve to meet the user's growing requirements. GE is actively involved in the development of the DICOM Standard. DICOM will incorporate new features and technologies and GE may follow the evolution of the Standard. The GEHC protocol is based on DICOM as specified in each DICOM Conformance Statement. Evolution of the Standard may require changes to devices which have implemented DICOM. **In addition, GE reserves the right to discontinue or make changes to the support of communications features (on its products) described by these DICOM Conformance Statements.** The **user** should ensure that any non-GE provider, which connects with GE devices, also plans for the future evolution of the DICOM Standard. Failure to do so will likely result in the loss of function and/or connectivity as the DICOM Standard changes and GE Products are enhanced to support these changes.

Interaction - It is the sole responsibility of the **non-GE provider** to ensure that communication with the interfaced equipment does not cause degradation of GE imaging equipment performance and/or function.

1.6 REFERENCES

NEMA PS3 Digital Imaging and Communications in Medicine (DICOM) Standard, available free at <https://www.dicomstandard.org>

1.7 DEFINITIONS

Informal definitions are provided for the following terms used in this Conformance Statement. The DICOM Standard is the authoritative source for formal definitions of these terms.

Abstract Syntax – the information agreed to be exchanged between applications, generally equivalent to a Service/Object Pair (SOP) Class. Examples: Verification SOP Class, Modality Worklist Information Model Find SOP Class, Computed Radiography Image Storage SOP Class.

Application Entity (AE) – an end point of a DICOM information exchange, including the DICOM network or media interface software; i.e., the software that sends or receives DICOM information objects or messages. A single device may have multiple Application Entities.

Application Entity Title – the externally known name of an *Application Entity*, used to identify a DICOM application to other DICOM applications on the network.

Application Context – the specification of the type of communication used between *Application Entities*. Example: DICOM network protocol.

Association – a network communication channel set up between *Application Entities*.

Attribute – a unit of information in an object definition; a data element identified by a *tag*. The information may be a complex data structure (Sequence), itself composed of lower level data elements. Examples: Patient ID (0010,0020), Accession Number (0008,0050), Photometric Interpretation (0028,0004), Procedure Code Sequence (0008,1032).

Information Object Definition (IOD) – the specified set of *Attributes* that comprise a type of data object; does not represent a specific instance of the data object, but rather a class of similar data objects that have the same properties. The *Attributes* may be specified as Mandatory (Type 1), Required but possibly unknown (Type 2), or Optional (Type 3), and there may be conditions associated with the use of an Attribute (Types 1C and 2C). Examples: MR Image IOD, CT Image IOD, Print Job IOD.

Joint Photographic Experts Group (JPEG) – a set of standardized image compression techniques, available for use by DICOM applications.

Media Application Profile – the specification of DICOM information objects and encoding exchanged on removable media (e.g., CDs)

Module – a set of *Attributes* within an *Information Object Definition* that are logically related to each other. Example: Patient Module includes Patient Name, Patient ID, Patient Birth Date, and Patient Sex.

Negotiation – first phase of *Association* establishment that allows *Application Entities* to agree on the types of data to be exchanged and how that data will be encoded.

Presentation Context – the set of DICOM network services used over an *Association*, as negotiated between *Application Entities*; includes *Abstract Syntaxes* and *Transfer Syntaxes*.

Protocol Data Unit (PDU) – a packet (piece) of a DICOM message sent across the network. Devices must specify the maximum size packet they can receive for DICOM messages.

Security Profile – a set of mechanisms, such as encryption, user authentication, or digital signatures, used by an *Application Entity* to ensure confidentiality, integrity, and/or availability of exchanged DICOM data

Service Class Provider (SCP) – role of an *Application Entity* that provides a DICOM network service; typically, a server that performs operations requested by another *Application Entity* (*Service Class User*). Examples: Picture Archiving and Communication System (image storage SCP, and image query/retrieve SCP), Radiology Information System (modality worklist SCP).

Service Class User (SCU) – role of an *Application Entity* that uses a DICOM network service; typically, a client. Examples: imaging modality (image storage SCU, and modality worklist SCU), imaging workstation (image query/retrieve SCU)

Service/Object Pair (SOP) Class – the specification of the network or media transfer (service) of a particular type of data (object); the fundamental unit of DICOM interoperability specification. Examples: Ultrasound Image Storage Service, Basic Grayscale Print Management.

Service/Object Pair (SOP) Instance – an information object; a specific occurrence of information exchanged in a *SOP Class*. Examples: a specific x-ray image.

Tag – a 32-bit identifier for a data element, represented as a pair of four-digit hexadecimal numbers, the “group” and the “element”. If the “group” number is odd, the tag is for a private (manufacturer-specific) data element. Examples: (0010,0020) [Patient ID], (07FE,0010) [Pixel Data], (0019,0210) [private data element]

Transfer Syntax – the encoding used for exchange of DICOM information objects and messages. Examples: *JPEG* compressed (images), little endian explicit value representation.

Unique Identifier (UID) – a globally unique “dotted decimal” string that identifies a specific object or a class of objects; an ISO-8824 Object Identifier. Examples: Study Instance UID, SOP Class UID, SOP Instance UID.

Value Representation (VR) – the format type of an individual DICOM data element, such as text, an integer, a person’s name, or a code. DICOM information objects can be transmitted with either explicit identification of the type of each data element (Explicit VR), or without explicit identification (Implicit VR); with Implicit VR, the receiving application must use a DICOM data dictionary to look up the format of each data element.

1.8 SYMBOLS AND ABBREVIATIONS

AE	Application Entity
AET	Application Entity Title
CD-R	Compact Disk Recordable
CVIS	Cardio Vascular Information System
CT	Computed Tomography
DHCP	Dynamic Host Configuration Protocol
DICOM	Digital Imaging and Communications in Medicine
DNS	Domain Name System
FSC	File-Set Creator
FSU	File-Set Updater
FSR	File-Set Reader
GSDF	Grayscale Standard Display Function
HIS	Hospital Information System
HL7	Health Level 7 Standard
IHE	Integrating the Healthcare Enterprise
IOD	Information Object Definition
IPv4	Internet Protocol version 4
IPv6	Internet Protocol version 6
ISO	International Organization for Standards
JPEG	Joint Photographic Experts Group
KOS	Key Object Selection
KIN	Key Image Note
LUT	Look-up Table
MPEG	Moving Picture Experts Group
MG	Mammography (X-ray)

GE Healthcare

DIR DOC2107162 REV 4

MPPS	Modality Performed Procedure Step
MR	Magnetic Resonance Imaging
MWL	Modality Worklist
O	Optional (Key Attribute)
OSI	Open Systems Interconnection
PACS	Picture Archiving and Communication System
PET	Positron Emission Tomography
PDU	Protocol Data Unit
R	Required (Key Attribute)
RIS	Radiology Information System
SC	Secondary Capture
SCP	Service Class Provider
SCU	Service Class User
SOP	Service-Object Pair
SPS	Scheduled Procedure Step
SR	Structured Reporting
TCP/IP	Transmission Control Protocol/Internet Protocol
TLS	Transport Layer Security
U	Unique (Key Attribute)
UL	Upper Layer
US	Ultrasound
VR	Value Representation
XA	X-Ray Angiographic Imaging

2. NETWORK CONFORMANCE STATEMENT

2.1 INTRODUCTION

This section of the DICOM Conformance Statement specifies the LOGIQ scanner compliance to DICOM requirements for **Networking** features.

Sending and receiving Echo messages to and from DICOM Verification SCP and client.

Sending DICOM Images to remote DICOM Storage SCP.

Sending measurements as DICOM SR objects to remote DICOM Storage SCP.

Querying and retrieving DICOM Modality Worklist from a Worklist SCP.

Sending start and end of examination to a DICOM Modality Performed Procedure Step SCP.

Sending storage commitment requests (and receiving replies) to a DICOM Storage Commitment SCP.

Printing images to a DICOM Printer.

Querying and retrieving examinations from a DICOM Query/Retrieve SCP.


Receiving DICOM Images from remote DICOM Storage SCU.

2.2 IMPLEMENTATION MODEL

2.2.1 Application Data Flow Diagram

The network application model for the LOGIQ scanner is shown in the following Illustration:

ILLUSTRATION 2–1
LOGIQ NETWORK APPLICATION MODEL AND DATA FLOW DIAGRAM

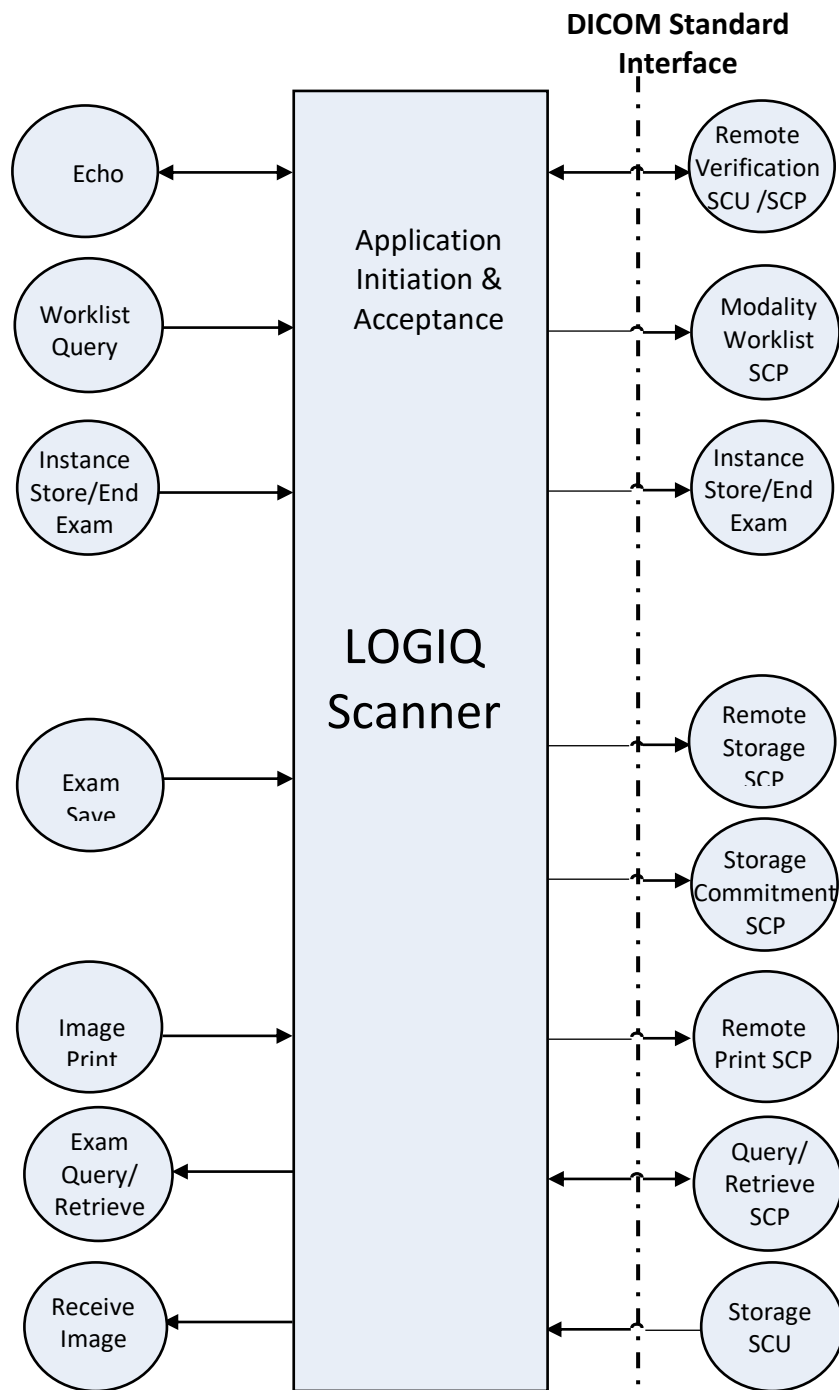

There are seven local real-world activities that occur in LOGIQ Scanner - **Exam Save, Echo, Worklist Query, Image Store/End Exam, Image Print, Exam Query/Retrieve and Receive Image.**

Exam save initiates a connection with the DICOM SCP and transmits images and results to the DICOM SCP. If Storage Commitment is configured a commitment request will be sent for the images and results.

Echo initiates a connection with the DICOM SCP, posts a Verification request and closes the connection. It also responds to incoming Verification requests (for service use).

Worklist Query initiates a connection with the DICOM SCP, performs a query and retrieves the matching entries to the product.

Image Store/End exam: If Modality Performed Procedure Step is configured N-CREATE and N-SET messages will be sent for the exam.

Image Print will send images to a DICOM Print SCP.

Exam Query/Retrieve initiates a connection with the DICOM SCP, performs a query and retrieves selected examination.

Receive Image: Scanner accepts the connection from DICOM SCU, receive images, stores the images in Scanner database and transmit storage results to DICOM SCU.

2.2.2 Functional Definition of AE's

Application Entity LOGIQ Scanner supports the following functions:

Initiates a DICOM association to send images and results.

Initiates a DICOM verification to assist in network diagnostics.

Initiates a DICOM worklist query to receive worklist information.

Initiates a DICOM association to notify start of examination.

Initiates a DICOM association to notify end of examination.

Initiates a DICOM association to request storage commitment of images.

Responds to replies for storage commitment requests of images.

Initiates a DICOM association to print images.

Initiates a DICOM association to query for and retrieve examinations.

Responds to storage requests for images triggered by examination retrieve requests.

Accepts a DICOM association to receive images from remote DICOM nodes.

2.2.3 Sequencing of Real-World Activities

Non-Applicable.

2.3 AE SPECIFICATIONS

This Application Entity provides Standard Conformance to the following DICOM SOP Classes as an SCU and/or as an SCP:

TABLE 2-1

SOP Class Name	SOP Class UID	SCU	SCP
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Yes	Yes
Ultrasound Image Storage (Retired)	1.2.840.10008.5.1.4.1.1.6	Yes	Yes
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	Yes	Yes
Ultrasound Multi-frame Image Storage (Retired)	1.2.840.10008.5.1.4.1.1.3	Yes	Yes
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	Yes	Yes
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Yes	Yes
Comprehensive SR	1.2.840.10008.5.1.4.1.1.88.33	Yes	No
Key Object Selection Document Storage	1.2.840.10008.5.1.4.1.1.88.59	Yes	No
Verification SOP Class	1.2.840.10008.1.1	Yes	Yes
Digital Mammography Image Storage - For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Yes	Yes
Digital Mammography Image Storage - For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Yes	Yes
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Yes	Yes
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Yes	Yes
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Yes	Yes
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Yes	Yes
Study Root Query/Retrieve Information Model - FIND	1.2.840.10008.5.1.4.1.2.2.1	Yes	No
Study Root Query/Retrieve Information Model - MOVE	1.2.840.10008.5.1.4.1.2.2.2	Yes	No
Modality Worklist Information Model - FIND	1.2.840.10008.5.1.4.31	Yes	No
Modality Performed Procedure Step	1.2.840.10008.3.1.2.3.3	Yes	No
Storage Commitment Push Model	1.2.840.10008.1.20.1	Yes	No
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9	Yes	No
Basic Color Print Management Meta SOP Class	1.2.840.10008.5.1.1.18	Yes	No
Basic Annotation Box	1.2.840.10008.5.1.1.15	Yes	No
Printer SOP Class	1.2.840.10008.5.1.1.16	Yes	No
Basic Film Session SOP Class	1.2.840.10008.5.1.1.1	Yes	No
Basic Film Box SOP Class	1.2.840.10008.5.1.1.2	Yes	No
Basic Grayscale Image Box SOP Class	1.2.840.10008.5.1.1.4	Yes	No
Basic Color Image Box SOP Class	1.2.840.10008.5.1.1.4.1	Yes	No

2.3.1 Association Establishment Policies

2.3.1.1 General

The DICOM Application Context Name (ACN), which is always proposed, is:

Application Context Name	1.2.840.10008.3.1.1.1
---------------------------------	------------------------------

The Maximum Length PDU negotiation is included in all association establishment requests. The maximum length PDU for an association initiated by LOGIQ Scanner is:

Maximum Length PDU	32768
---------------------------	--------------

The maximum length PDU is a fixed number - not configurable.

The SOP Class Extended Negotiation is not supported.

The user information Items sent by this product are:

Maximum PDU Length

Implementation UID

Implementation Version Name

2.3.1.2 Number of Associations

The maximum number of simultaneous associations supported by LOGIQ scanner is 1.

2.3.1.3 Asynchronous Nature

Asynchronous mode is not supported. All operations will be performed synchronously.

2.3.1.4 Implementation Identifying Information

The Implementation UIDs for this DICOM Implementation are:

TABLE 2-2

Product Name	Implementation UID	Implementation Version Name
LOGIQ E10	1.2.840.113619.6.455	LOGIQE10
LOGIQ E11	1.2.840.113619.6.455	LOGIQE11
LOGIQ E20	1.2.840.113619.6.455	LOGIQE20
LOGIQ E10s	1.2.840.113619.6.481	LOGIQE10s

2.3.1.5 Association Initiation Policy

The LOGIQ Scanner AE attempts to establish a new association with a remote device due to six Real-World Activities:

Exam save initiated by the operator for images, key object selection document and structured reports and sending request for Storage Commitment.

Verification, which verifies application level communication between peer DICOM AE's for service purposes.

Worklist initiated by the operator for receiving worklist information.

Image Store/End Exam sending messages to Modality Performed Procedure Step.

Print initiated by the operator for a specific image or group of images.

Exam Query/Retrieve initiated by the operator for receiving examination information and selecting examination to retrieve.

2.3.1.5.1 Real-World Activity - 'Exam save' Operation

2.3.1.5.1.1 Associated Real-World Activity

Upon a request by the operator (manual or automatic), images will be sent to a DICOM Storage SCP.

2.3.1.5.1.2 Proposed Presentation Context Tables

The Proposed Presentation Context Table depends on compression according to the following table:

TABLE 2-3

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90	SCU	None
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90	SCU	None
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90	SCU	None
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3	Explicit VR Little Endian Explicit VR Big Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2	SCU	None

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
(retired)		Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90		
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90	SCU	None
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian Run Length Encoding JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossy JPEG 2000 Lossless	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 1.2.840.10008.1.2.5 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.90	SCU	None
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCU	None
Comprehensive Structured Report	1.2.840.10008.5.1.4.1.1.88.33	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
Key Object Selection Document	1.2.840.10008.5.1.4.1.1.88.59	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
Storage Commitment Push Model SOP Class	1.2.840.10008.1.20.1	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.5.1.2.1 SOP Specific Conformance Statement for Image Storage SOP Classes

For these SOP class all status codes with status Refused or Error are treated as failures and terminate the association and operation. On a failure, the request will be put in a holding queue for the user to manually retry the request. All status codes with status Warning or Success are treated as successes.

TABLE 2-4

Service Status	Further Meaning	Status Codes	Application Behavior When receiving Status Codes
Failure	Refused: Out of Resources	A7xx	Terminate association. Mark the job failed.
	Error: Data Set does not match SOP Class	A9xx	
	Error: Cannot understand	Cxxx	
Warning	Coercion of Data Elements	B000	Treat the job success.
	Data Set does not match SOP Class	B007	
	Elements Discarded	B006	
Success		0000	
*	Any other status code	*	Terminate association. Mark the job failed.

C-STORE SCU operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

2.3.1.5.2 Real-World Activity - ‘Echo’ Operation

2.3.1.5.2.1 Associated Real-World Activity

The user may initiate a DICOM Verification Request in the config screen. Associations will be released upon the receipt of each C-ECHO confirmation.

2.3.1.5.2.2 Proposed Presentation Context Table

TABLE 2-5

Presentation Context Table - Proposed				
Abstract Syntax		Transfer Syntax		Role
Name	UID	Name List	UID List	
				Extended Negotiation

Verification SOP Class	1.2.840.10008.1.1	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
------------------------	-------------------	--	--	-----	------

2.3.1.5.2.2.1 SOP Specific DICOM Conformance Statement for Verify SOP Class

The AE provides standard conformance to the Verification SOP Class as an SCU. In the event that the SCP does not respond for some reason, the operation will time out and LOGIQ Scanner will close the association.

C-ECHO SCU operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds and 10secs each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1 to 10 seconds.

2.3.1.5.3 Real-World Activity - ‘Worklist Query’ Operation

2.3.1.5.3.1 Associated Real-World Activity

The user may initiate a DICOM Worklist Query in Search screen, which will send a C-FIND-RQ to the Worklist SCP. Associations will be released upon the receipt of C-FIND-RSP confirmation.

2.3.1.5.3.2 Proposed Presentation Context Tables

TABLE 2-6

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Modality Worklist Information Model – FIND	1.2.840.10008.5.1.4.31	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.5.3.2.1 SOP Specific DICOM Conformance Statement for Worklist SOP Classes

The LOGIQ Scanner includes matching keys in the Modality Worklist queries as described in Section 11. All status codes with status Refused or Error are treated as failures and terminate the association and operation. On a failure, the user will be informed, and the last successful query will be used as Worklist. All status codes with status Warning or Success are treated as successes. Application doesn’t support C-FIND-CANCEL request.

TABLE 2-7

Service Status	Status Code	Further Meaning	Application Behavior When receiving Status Codes
Refused	A700	Out of resources	Terminate the

Service Status	Status Code	Further Meaning	Application Behavior When receiving Status Codes
	0122	SOP Class not Supported	association and operation
Failed	A900	Identifier does not match SOP Class	
	Cxxx	Unable to process	
Success	0000	Matching is complete – No final identifier is supplied	
Pending	FF00	Matches are continuing – Current Match is supplied and any Optional Keys were supported in the same manner	Receiving process of the matches continues.
	FF01	Matches are continuing – Warning that one or more Optional Keys were not supported for existence for this Identifier	
*	*	Any other status code.	Terminate the association and operation

Worklist operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

2.3.1.5.4 Real-World Activity - ‘Image Store/End exam’ Operation

2.3.1.5.4.1 Associated Real-World Activity

The Modality Performed Procedure Step messages are sent when the first image is acquired for the start of an exam and when the exam is ended (for the case where there are no images, the N-CREATE is

sent when the exam is ended). For an exam with saved images or results, the N-SET will be sent with status COMPLETED. For an exam without saved images or results, the N-SET will be sent with status DISCONTINUED.

2.3.1.5.4.2 Proposed Presentation Context Table

TABLE 2-8

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Modality Performed Procedure Step SOP Class	1.2.840.10008.3.1.2.3.3	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.5.4.2.1 SOP Specific DICOM Conformance Statement for Modality Performed Procedure Step Class

LOGIQ Scanner includes attributes in the Modality Performed Procedure Step N-CREATE and N-SET as described in Section 10.2.

LOGIQ Scanner includes attributes in the Modality Performed Procedure Step N-SET as described in Section 12. The mapping from Worklist attributes is described in Section 11.5. LOGIQ Scanner sends N-SET after the exam is ended. The N-SET will include all acquired images, KIN and SR UIDs and the status of COMPLETED or DISCONTINUED.

For this SOP class, all status codes with status Refused or Error are treated as failures and terminate the association and operation. All status codes with status Warning or Success are treated as successes.

Modality Performed Procedure Step N-CREATE response status Handling Behavior

TABLE 2-9

Service Status	Status Code	Further Meaning	Application Behavior When receiving Status Codes
Success	0000	Success	
*	*	Any other status	Treat as failure. Aborts the association.

Modality Performed Procedure Step N-SET response status Handling Behavior

TABLE 2-10

Service Status	Status Code	Further Meaning	Application Behavior When receiving Status Codes
Failure	0110H	Processing Failure. Performed Procedure Step Object may no longer be updated	Aborts the operation.
Success	0000	Success	Completes the operation.
*	*	Any other status	Aborts the operation.

MPPS operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

2.3.1.5.5 Real-World Activity - ‘Image Print’ Operation

2.3.1.5.5.1 Associated Real-World Activity

Upon a request by the operator, print jobs will be sent to a DICOM Print SCP. If an error occurs during the transmission, the current association is released, and a new association initiated. The maximum number of retries is configurable.

2.3.1.5.5.2 Proposed Presentation Context Tables

The following table is used:

TABLE 2-11

Presentation Context Table - Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
Basic Color Print Management Meta SOP Class	1.2.840.10008.5.1.1.18	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
Basic Annotation Box SOP Class	1.2.840.10008.5.1.1.15	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.5.5.2.1 SOP Specific DICOM Conformance Statement for all Print Management SOP Classes

All status codes with status Refused or Error are treated as failures and terminate the association and operation. All status codes with status Warning or Success are treated as successes.

TABLE 2-12

SOP Class	SOP Class UID	DIMSE Service Element	SCU Usage
Basic Film Session	1.2.840.10008.5.1.1.1	N-CREATE	Used
		N-SET	Not Used
		N-DELETE	Not Used
		N-ACTION	Not Used
Basic Film Box	1.2.840.10008.5.1.1.2	N-CREATE	Used
		N-ACTION	Used
		N-DELETE	Used
		N-SET	Not Used
Printer	1.2.840.10008.5.1.1.16	N-EVENT-REPORT	Not Used
		N-GET	Used
Basic Grayscale Image Box	1.2.840.10008.5.1.1.4	N-SET	Used
Basic Color Image Box	1.2.840.10008.5.1.1.4.1	N-SET	Used
Basic Annotation Box	1.2.840.10008.5.1.1.15	N-SET	Used

DICOM Print response status Handling Behavior

TABLE 2-13

Service Status	Status Code	Further Meaning	Application Behavior When receiving Status Codes
Success	0000	Success	
Warning	B600	Warning	Treated as success with warning log.
*	*	Any other status	Treated as failure. Aborts the association.

Dicom Print operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

2.3.1.5.6 Real-World Activity - ‘Exam Query/Retrieve’ Operation

2.3.1.5.6.1 Associated Real-World Activity

The user may initiate a DICOM Exam Query in Search screen, which will send a C-FIND-RQ to the Query/Retrieve SCP.

Associations will be released upon the receipt of C-FIND-RSP confirmation.

The user may then select an examination to be retrieved, using the C-MOVE-RQ command to the Query/Retrieve SCP. The result from the SCP is expected on another association for the retrieved examinations.

2.3.1.5.6.2 Proposed Presentation Context Tables

TABLE 2-14

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Study Root Query/Retrieve Information Model - FIND	1.2.840.10008.5.1.4.1.2.2.1	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None
Study Root Query/Retrieve Information Model - MOVE	1.2.840.10008.5.1.4.1.2.2.2	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.5.6.2.1 SOP Specific DICOM Conformance Statement for Study Root Query/Retrieve Information Model - FIND SOP Classes

Only a single information model, Study Root, is supported.

All queries are initiated at the highest level of the information model (the STUDY level), and then for each response received, recursively repeated at the next lower levels (the SERIES and then IMAGE levels).

C-FIND-CANCEL request is not supported.

All status codes with status Refused or Error are treated as failures and terminate the association and operation. All status codes with status Warning or Success are treated as successes. LOGIQ Scanner will only support hierarchical query.

TABLE 2-15

Service Status	Status Code	Further Meaning	Application Behavior When Receiving Status Code
Failure	A700	Refused: Out of resources	Operation aborted.

Service Status	Status Code	Further Meaning	Application Behavior When Receiving Status Code
	A900	Error: Identifier does not match SOP Class	
	C000-CFFF	Error: Unable to process	
	0122	SOP Class Not Supported	
Success	0000	Matching is complete - No final identifier is supplied	Display the results in UI.
Pending	FF00	Matches are continuing - Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys.	Stores the results internally and waits for more results.
	FF01	Matches are continuing - Warning that one or more Optional Keys were not supported for existence and/or matching for this Identifier	
*	*	Any other status codes.	Operation aborted.

C-FIND SCU operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

LOGIQ Scanner query responses for Key Object Selection documents and Structured Report documents.

Study Level attributes for the Study Root Query/Retrieve Information Model is described in Section 16.2.

Types of Matching:

- Single Value matching (S)
- Universal Matching (U)
- Wild Card Matching (*)
- Range of date, Range of Time (R)

The types of Matching supported by the C-FIND SCU are: 'S' indicates the identifier attribute uses Single Value Matching, an 'R' indicates Range Matching, a "*" indicates wildcard matching, a 'U' indicates Universal Matching, and 'UNIQUE' indicates that this is the Unique Key for that query level, in which case Universal Matching or Single Value Matching is used depending on the query level.

"Filtering is supported" means that matching strings can be controlled from the Search screen at Data Transfer. All other matching fields can be configured in Config screen to be either enabled, enabled with a matching string value or disabled. The constant value will be used as entered by user.

2.3.1.5.6.2.2 SOP Specific DICOM Conformance Statement for Study Root Query/Retrieve Information Model - MOVE SOP Classes

All status codes with status Refused or Error are treated as failures and terminate the association and operation. All status codes with status Warning or Success are treated as successes.

C-MOVE-CANCEL request is not supported.

TABLE 2-16

Service Status	Status Code	Further Meaning	Application Behavior When Receiving Status Code
Failure	A701	Refused: Out of resources - Unable to calculate number of matches	Retrieve operation will be aborted.
	A702	Refused: Out of resources - Unable to perform sub-operations	
	A801	Refused: Move Destination Unknown	
	A900	Error: Identifier does not match SOP Class	
	C000-CFFF	Error: Unable to process	
	0122	SOP Class Not Supported	
Warning	B000	Sub-operations Complete - One or more Failures.	Imports selected patient(s).
Success	0000	Sub-operations Complete - No Failure.	
Pending	FF00	Sub-operations are continuing -	Stores the patients internally until final response.
*	*	Any other status codes.	Retrieve operation will be aborted.

C-MOVE SCU operation supports an “Association Timer” and “Operation Inactivity Timer” with default time out value of 30 seconds each. Operation Inactivity Timer is configurable and different values that can be assigned to it are 1, 2, 3, 4, 5, 10, 20, 30, 60, 120 and 360 seconds.

2.3.1.6 Association Acceptance Policy

The AE accepts an association when it receives a Verification Request from another network device or a store request from a Q/R SCP or a Storage Commitment result from a Storage Commitment SCP.

2.3.1.6.1 Real-World Activity - ‘Echo’ operation

2.3.1.6.1.1 Associated Real-World Activity

An incoming Verification Request will cause the AE to accept the association and respond with a Verification Response.

2.3.1.6.1.2 Accepted Presentation Context Table

TABLE 2-17

Presentation Context Table - Accepted					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Verification SOP Class	1.2.840.10008.1.1	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCP	None

2.3.1.6.1.2.1 SOP Specific DICOM Conformance Statement for Verify SOP Class

The AE provides standard conformance to the Verification SOP Class as an SCP. The port number used is configured on config screen, default is 104.

2.3.1.6.1.3 Presentation Context Acceptance Criterion

No criterion.

2.3.1.6.1.4 Transfer Syntax Selection Policies

The selected transfer syntax is based on the proposed transfer syntax list. The priority order is Explicit VR Little Endian and Implicit VR Little Endian.

2.3.1.6.2 Real-World Activity - ‘Exam Query/Retrieve’ Operation

2.3.1.6.2.1 Associated Real-World Activity

If the user has initiated a retrieve by a C-MOVE-RQ, the AE will accept associations for C-STORE-RQs. The images will be stored locally.

2.3.1.6.2.2 Accepted Presentation Context Table

TABLE 2-18

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Ultrasound Multi-frame Image Storage (retired)	1.2.840.10008.5.1.4.1.1.3	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None

2.3.1.6.2.2.1 SOP Specific DICOM Conformance Statement for Storage SOP Classes

The AE provides standard conformance to the Storage SOP Classes as an SCP. The port number used is 104 (default port number). Below are all possible status codes which LOGIQ Scanner would send upon storage request.

Application doesn't support extended negotiation hence the Storage Level is set to Level 2.

TABLE 2-19

Service Status	Status Code	Conditions
Failed	C000	Processing Error
Success	0000	SOP Instance is stored successfully.

The system supports retrieval of Enhanced US Volume dataset if the dataset is generated by LOGIQ scanner and the LOGIQ scanner generated volume raw data is present.

2.3.1.6.2.3 Presentation Context Acceptance Criterion

No criterion.

2.3.1.6.2.4 Transfer Syntax Selection Policies

The selected transfer syntax is based on the proposed transfer syntax list.

The priority order in case of Ultrasound Image Storage, Ultrasound Multi-frame Image Storage, Ultrasound Image Storage (retired), Ultrasound Multi-frame Image Storage (retired), Enhanced US Volume Storage and Secondary Capture Image Storage is JPEG2000 Lossless, JPEG2000 Lossy, JPEG Lossless Non Hierarchical, JPEG Baseline (Process 1), Run Length Encoding (RLE), Explicit VR Little Endian, Explicit VR Big Endian and Implicit VR Little Endian.

The priority order in case of CT Image Storage, MR Image Storage, Digital Mammography Storage For Presentation, Digital Mammography Storage For Processing, Positron Emission Tomography Image Storage and X-Ray Angiographic Image Storage is Explicit VR Little Endian, Explicit VR Big Endian and Implicit VR Little Endian.

2.3.1.6.3 Real-World Activity - 'Exam save' Operation

2.3.1.6.3.1 Associated Real-World Activity

LOGIQ Scanner will only listen for an N-EVENT-REPORT (Storage Commitment Result) from a Storage Commitment SCP in a new association.

2.3.1.6.3.2 Proposed Presentation Context Tables

The Proposed Presentation Context Table depends on compression according to the following table:

TABLE 2-20

Presentation Context Table – Proposed					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Storage Commitment Push Model SOP Class	1.2.840.10008.1.20.1	Explicit VR Little Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2	SCU	None

2.3.1.6.3.2.1 SOP Specific DICOM Conformance Statement for the Storage Commitment Push Model SOP Class SCU

LOGIQ Scanner will only accept the SCU role (which must be proposed via SCP/SCU Role Selection Negotiation) within a Presentation Context for the Storage Commitment Push Model SOP Class. The result from the SCP is expected on another association for the Storage Commitment result.

The LOGIQ Scanner behavior after receiving an N-EVENT-REPORT (Storage Commitment Result) is described in Section 13.2.4. Below are all possible status codes which LOGIQ scanner would send upon N-EVENT-REPORT request as EVENT-REPORT response.

TABLE 2-21

Service Status	Status Code	Conditions
Failed	0110H	Processing Error
Success	0000	N-Even Report received successfully.

2.3.1.6.4 Real-World Activity - ‘Image Receive’ Operation

2.3.1.6.4.1 Associated Real-World Activity

If a remote dicom node has initiated a C-STORE-RQ, the AE will accept associations for C-STORE-RQs. The images will be stored locally.

2.3.1.6.4.2 Accepted Presentation Context Table

TABLE 2-22

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Ultrasound Multi-frame Image Storage (retired)	1.2.840.10008.5.1.4.1.1.3	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) JPEG 2000 Lossless JPEG 2000 Lossy Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2	SCP	None

2.3.1.6.4.2.1 SOP Specific DICOM Conformance Statement for Storage SOP Classes

The AE provides standard conformance to the Storage SOP Classes as an SCP. The port number used is 104 (default port number). Below are all possible status codes which LOGIQ Scanner would send upon storage request.

Application doesn't support extended negotiation hence the Storage Level is set to Level 2.

TABLE 2-23

Service Status	Status Code	Conditions
Failed	C002	Processing Error
	A700	System is busy.
Success	0000	SOP Instance is stored successfully.

The system supports receiving Enhanced US Volume dataset if the dataset is generated by LOGIQ scanner and the LOGIQ scanner generated volume raw data is present.

If the incoming image(s) already present in the scanner database, the scanner doesn't overwrite the images.

2.3.1.6.4.3 Presentation Context Acceptance Criterion

No criterion.

2.3.1.6.4.4 Transfer Syntax Selection Policies

The selected transfer syntax is based on the proposed transfer syntax list.

The priority order in case of Ultrasound Image Storage, Ultrasound Multi-frame Image Storage, Ultrasound Image Storage (retired), Ultrasound Multi-frame Image Storage (retired), Enhanced US Volume Storage and Secondary Capture Image Storage is JPEG2000 Lossless, JPEG2000 Lossy, JPEG Lossless Non Hierarchical, JPEG Baseline (Process 1), Run Length Encoding (RLE), Explicit VR Little Endian, Explicit VR Big Endian and Implicit VR Little Endian.

The priority order in case of CT Image Storage, MR Image Storage, Digital Mammography Storage For Presentation, Digital Mammography Storage For Processing, Positron Emission Tomography Image Storage and X-Ray Angiographic Image Storage is Explicit VR Little Endian, Explicit VR Big Endian and Implicit VR Little Endian.

2.4 COMMUNICATION PROFILES

2.4.1 Supported Communication Stacks (PS 3.8)

DICOM Upper Layer (PS 3.8) is supported using TCP/IP.

2.4.2 TCP/IP Stack

The TCP/IP stack is inherited from the product's operating system. Please refer to product documentation for more information.

GE Healthcare

DIR DOC2107162 REV 4

2.4.2.1 API

Not applicable to this product.

2.4.3 Additional Protocols

LOGIQ Scanner supports DHCP.

2.4.4 IPv4 and IPv6 support

LOGIQ Scanner supports both IPv4(All LOGIQ versions) and IPv6(Since LOGIQ E10 R1.2).

2.5 EXTENSIONS / SPECIALIZATIONS / PRIVATIZATIONS

If so configured, the product will send ultrasound raw data information in private data elements designated by the Private Creator element:

TABLE 2-24

Element Name	Tag	VR	VM	Description
Private Creator	7FE1,00xx	LO	1	GEMS_Ultrasound_MovieGroup_001

This means that all private tags starting with 7FE1,xx will belong to the GEMS_Ultrasound_MovieGroup_001.

The product will send measurements raw data information in private data elements designated by the Private Creator element:

TABLE 2-25

Element Name	Tag	VR	VM	Description
Private Creator	6005,00xx	LO	1	GE_GROUP

This means that all private tags starting with 6005,xx will belong to the GE_GROUP.

The product will send exam information in private data elements designated by the Private Creator element:

TABLE 2-26

Element Name	Tag	VR	VM	Description
Private Creator	6FF1,00xx	LO	1	GE Contrast Quantification

This means that all private tags starting with 6FF1,xx will belong to the GE Contrast Quantification

The product will send exam information in private data elements designated by the Private Creator element:

TABLE 2-27

Element Name	Tag	VR	VM	Description
Private Creator	6001,00xx	LO	1	GE Generic Data

GE Healthcare

DIR DOC2107162 REV 4

This means that all private tags starting with 6001,xx will belong to the GE Generic Data.

The product will send informatics data in private data elements designated by the Private Creator element:

TABLE 2-28

Element Name	Tag	VR	VM	Description
Private Creator	3003,00xx	LO	1	GE Informatics Data

This means that all private tags starting with 3003,xx will belong to the GE Informatics Data.

The product will send KoiosDS assessment data in private data elements designated by the Private Creator element:

TABLE 2-29

Element Name	Tag	VR	VM	Description
Private Creator	3001,00xx	LO	1	KoiosDS_Assessment

This means that all private tags starting with 3001,xx will belong to the KoiosDS Assessment.

2.6 CONFIGURATION

2.6.1 AE Title/Presentation Address Mapping

The Local AE title is configurable through the Config screen, see below.

2.6.2 Configurable Parameters

Network:

- Local IP address
- Local IP netmask
- Default Gateway
- Local network speed information

Local:

- Local AE Title
- Local port number

Verification:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout

Modality Worklist:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout

GE Healthcare

DIR DOC2107162 REV 4

- Refresh interval - the interval between downloads from the worklist. Searching for a patient within the time of a refresh interval does not perform a new worklist query.
- Disabling/enabling and setting constant values for query fields – This is a way to disable search criteria in the worklist query or setting them to a fixed value.
- Maximum number of downloaded entries
- Validate Incoming Worklists

Storage:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout
- Enable/disable raw data
- Frame rate reduction
- Enable/disable multi-frame
- Compression selections
- Color support
- Association strategies: one association per image or one association per exam
- Include structured report (SR)
- Link Measurements to Images
- Included Key Object Selection Document for Image Rejection (KOS)

Modality Performed Procedure Step:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout

Storage Commitment:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout
- The associated Storage service which triggers the sending of Storage Commitment requests

Print:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout
- Include Annotation Box
- Configuration for each job according to attribute description in Section 12 of this document.

Query/Retrieve:

- The AE Title, IP address and port number of the SCP
- Max retries, Retry interval, Timeout
- Disabling/enabling and setting constant values for query fields
- Maximum number of downloaded entries

2.7 SUPPORT OF EXTENDED CHARACTER SETS

LOGIQ Scanner supports the ISO IR 100 (ISO 8859-1:1987 Latin alphabet N 1. supplementary set). Other character sets won't be displayed correctly if received.

2.8 CODES AND CONTROLLED TERMINOLOGY

2.8.1 Fixed Coded Terminology

The product uses the fixed (non-configurable, non-extensible) coded terminology in SR Document and Key Object Selection attributes, as described in Section: 8 and Section: 10.

2.8.2 Custom Coded Terminology

The product uses the custom (extensible) coded terminology in SR Document Selection attributes, as described in Section: 8.

2.9 SECURITY PROFILES

2.9.1 Environment

Besides several SCU of DICOM SOP Classes (Refer to 2.9.2), the product does not conform to any defined DICOM Security Profiles.

It is assumed that the product is used within a secured environment. It is assumed that a secured environment includes at a minimum:

1. Firewall or router protections to ensure that only approved external hosts have network access to the product.
2. Firewall or router protections to ensure that the product only has network access to approved external hosts and services.
3. Any communications with external hosts and services outside the locally secured environment use appropriate secure network channels (such as a Virtual Private Network (VPN))

All these recommendations must be fulfilled by the hospital.

2.9.2 Secure Transport Connection Profiles

The product supports the Basic TLS Secure Transport Connection Profile and the AES TLS Secure Transport Connection Profile as specified in DICOM PS3.15 2018a, Annex B.1 and Annex B.3, respectively.

The scanner supports TLS 1.0, TLS 1.1 and TLS 1.2 versions. By default, scanner supports fallback mechanism whereby all of TLS 1.2, TLS 1.1, and TLS 1.0 are enabled. The fallback mechanism offers the highest level of interoperability.

The following cipher suites are used by default (in order of preference):

TLS_RSA_WITH_AES_128_CBC_SHA and TLS_RSA_WITH_3DES_EDE_CBC_SHA.

Encrypted DICOM connections are accepted with configured listening port if enabled.

Acting as a service class user, the scanner supports all service classes over TLS, that it supports over unencrypted connections.

The scanner supports import of public certificates and root certificates through configuration interface(.pfx). The product uses this certificate for authentication both when it acts as a TLS server and when it acts as a TLS client.

By default, the TLS option is disabled.

2.9.3 Audit Trail Message Format Profile

The product supports the *Audit Trail Message Format Profile* as specified in DICOM Standard PS3.152018a, Annex A.5.

Please refer section 19 for detailed description.

2.9.4 Audit Trail Message Transmission Profile - SYSLOG-TLS

The product supports the Audit Trail Message Transmission Profile - SYSLOG-TLS as specified in DICOM Standard PS3.152018a, Annex A.6.

2.9.5 Audit Trail Message Transmission Profile - SYSLOG-UDP

The product supports the Audit Trail Message Transmission Profile - SYSLOG-TLS as specified in DICOM Standard PS3.152018a, Annex A.7.

3. MEDIA STORAGE CONFORMANCE STATEMENT

3.1 INTRODUCTION


This section of the conformance statement (CS) specifies the LOGIQ Scanner compliance to DICOM Media Interchange. It details the DICOM Media Storage Application Profiles and roles, which are supported by this product. LOGIQ Scanner can export images, SR documents and encapsulated PDF documents to DICOM media, browse DICOM media or read images, SR documents and encapsulated PDF documents from DICOM media. And this product doesn't support KOS document objects.

3.2 IMPLEMENTATION MODEL

3.2.1 Application Data Flow Diagram

The Basic and Specific Application models for this device are shown in the following Illustration:

ILLUSTRATION 3-1
SPECIFIC AE APPLICATION MODEL

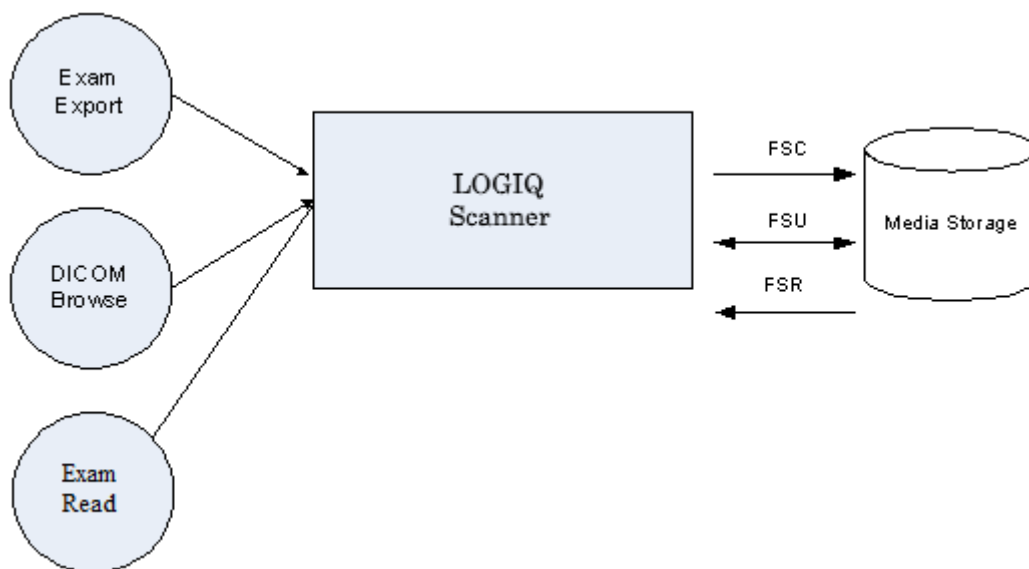

LOGIQ scanner can initialize Media by acting as an FSC to create a new DICOM File-set on CD/DVD media of various sizes. The SOP instances written to media must be one of the instances supported by LOGIQ scanner. A pre-existing File-set will be updated with the information in DICOM files copied to media.

3.2.2 Functional Definition of AE's

LOGIQ scanner can perform these functions:

- Create a new DICOM File-set on media

- Update DICOM File-set by adding new SOP instances to the File-set
- Read information and images, SR documents and encapsulated PDF documents from the existing File-set

3.2.3 Sequencing Requirements

None applicable

3.2.4 File Meta Information Options (See PS3.10)

The File Meta-Information for this implementation is:

TABLE 3-1

Product Name	File Meta-Information Version	Implementation UID	Implementation Version Name
LOGIQ E10	1	1.2.840.113619.6.455	LOGIQE10
LOGIQ E11	1	1.2.840.113619.6.455	LOGIQE11
LOGIQ E20	1	1.2.840.113619.6.455	LOGIQE20
LOGIQ E10s	1	1.2.840.113619.6.481	LOGIQE10s

Note: The Implementation Version Name and may change in the future without modification of this document.

3.3 AE SPECIFICATIONS

3.3.1 LOGIQ scanner AE Specification

The LOGIQ scanner Application Entity provides standard conformance to DICOM Interchange Option of the Media Storage Service Class. The Application Profiles and roles are listed below; the standard profiles are augmented with Secondary Capture images.

TABLE 3-2

Supported Application Profile	Real World Activity	Role	Description
STD-GEN-CD	Exam export	FSC/FSU	Interchange
	Browse	FSR	Interchange
	Exam Read	FSR	Interchange
STD-GEN-DVD-JPEG	Exam export	FSC/ FSU	Interchange
	Browse	FSR	Interchange
	Exam Read	FSR	Interchange
STD-GEN-DVD-J2K	Exam export	FSC/ FSU	Interchange
	Browse	FSR	Interchange
	Exam Read	FSR	Interchange

Supported Application Profile	Real World Activity	Role	Description
STD-GEN-USB-JPEG	Exam export	FSC/ FSU	Interchange
	Browse	FSR	Interchange
	Exam Read	FSR	Interchange
STD-GEN-USB-J2K	Exam export	FSC/ FSU	Interchange
	Browse	FSR	Interchange
	Exam Read	FSR	Interchange

3.3.1.1 File Meta Information for the LOGIQ Scanner Application Entity

The Source Application Entity is set from the LOGIQ scanner local AE title. The local AE is configurable. Following are the default value set in the File Meta Information for this AE Title:

Source Application Entity Title	LOGIQ-000000 (User Configurable)
--	----------------------------------

3.3.1.2 Real-World Activities for the LOGIQ Scanner Application Entity

3.3.1.2.1 Real-World Activity “Exam Export”

“Exam Export” saves a DICOM SOP instances, SR objects and Encapsulated Documents to media and updates DICOMDIR.

3.3.1.2.1.1 Media Storage Application Profile for the Real-World Activity “Exam Export”:

For the list of Application Profiles that invoke this AE for “Exam Export” Real-World Activity, see the Table in Section 3.3 “LOGIQ Scanner AE Specification” where the table describing the profiles and real-world activities is defined.

3.3.1.2.1.1.1 Options

Following are the SOP Classes supported by the Real-World Activity “Exam Export”:

TABLE 3-3

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
-------------------------------	---------------	-----------------	---------------------

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage (retired)	1.2.840.10008.5.1.4.1.1.3	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Comprehensive SR Storage	1.2.840.10008.5.1.4.1.1.88.33	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Encapsulated PDF Storage	1.2.840.10008.5.1.4.1.1.104.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

3.3.1.2.2 Real-World Activity “Exam Import”

“Exam Import” imports DICOM SOP instances from media.

3.3.1.2.2.1 Media Storage Application Profile for the Real-World Activity “Exam Import”:

For the list of Application Profiles that invoke this AE for “Exam Import” Real-World Activity, see the Table in Section 3.3 “LOGIQ Scanner AE Specification” where the table describing the profiles and real-world activities is defined.

3.3.1.2.2.1.1 Options

Following are the SOP Classes supported by the Real-World Activity “Exam Import”:

TABLE 3-4

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
-------------------------------	---------------	-----------------	---------------------

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage (retired)	1.2.840.10008.5.1.4.1.1.3	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Comprehensive SR Storage*	1.2.840.10008.5.1.4.1.1.88.33	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Encapsulated PDF Storage**	1.2.840.10008.5.1.4.1.1.104.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

*supports only GE LOGIQ created SR data.

**supports only GE LOGIQ created encapsulated document.

3.3.1.2.3 Real-World Activity “DICOM Browse”

DICOM Browse is activated when the user searches for an exam in Search screen.

3.3.1.2.3.1 Media Storage Application Profile for the Real-World Activity “DICOM Browse”

For the list of Application Profiles that invoke this AE for Image Read Real-World Activity, see the Table in [Section 3.3.1 “LOGIQ Scanner AE Specification”](#).

3.3.1.2.3.1.1 Options

Following are the SOP Classes supported by the Real-World Activity DICOM Browse:

TABLE 3-5

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
-------------------------------	---------------	-----------------	---------------------

Media Storage Directory Storage	1.2.840.10008.1.3.10	Explicit VR Little Endian	1.2.840.10008.1.2.1
---------------------------------	----------------------	---------------------------	---------------------

3.3.1.2.4 Real-World Activity “Exam read”

“Exam read” reads and displays a DICOM SOP instance from media.

3.3.1.2.4.1 Media Storage Application Profile for the Real-World Activity “Exam read”

For the list of Application Profiles that invoke this AE for Exam read Real-World Activity, see the Table in [Section 3.3.1 “LOGIQ Scanner AE Specification”](#).

3.3.1.2.4.1.1 Options

Following are the SOP Classes supported by the Exam read Real-World Activity:

TABLE 3-6

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Ultrasound Image Storage (retired)	1.2.840.10008.5.1.4.1.1.6	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Ultrasound Multi-frame Image Storage (retired)	1.2.840.10008.5.1.4.1.1.3	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	JPEG 2000 Lossless JPEG 2000 Lossy JPEG Baseline (Process 1) JPEG Lossless Non Hierarchical (Proc 14) Run Length Encoding Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.4.90 1.2.840.10008.1.2.4.91 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.70 1.2.840.10008.1.2.5 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Presentation	1.2.840.10008.5.1.4.1.1.1.2	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Digital Mammography Storage For Processing	1.2.840.10008.5.1.4.1.1.1.2.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
Positron Emission Tomography Image Storage	1.2.840.10008.5.1.4.1.1.128	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian	1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2

3.4 AUGMENTED AND PRIVATE APPLICATION PROFILES

3.4.1 Roles and Service Class Options

3.4.1.1 File Set Creator

File Set Creator shall be able to generate the Basic Directory SOP Class in the DICOMDIR File with all types of Directory Records related to the SOP Classes stored in the File-set.

3.4.1.2 File Set Reader

File Set Readers shall be able to read all the SOP Classes defined for the specific Application Profile for which a Conformance Statement is made using all the defined Transfer Syntaxes.

3.4.2 Augmented Application Profiles

The CD-R, DVD and USB Media Archive Interchange AE does not support any augmented Application Profiles.

3.4.3 Private Application Profiles

The CD-R, DVD and USB Media Interchange AE does not support any private Application Profiles.

3.5 EXTENSIONS, SPECIALIZATIONS, PRIVATIZATIONS OF SOP CLASSES AND TRANSFER SYNTAXES

If so configured, the product will include ultrasound raw data information in private data elements designated by the Private Creator element:

TABLE 3-7

Element Name	Tag	VR	VM	Description
Private Creator	7FE1,00xx	LO	1	GEMS_Ultrasound_MovieGroup_001

This means that all private tags starting with 7FE1,xx will belong to the GEMS_Ultrasound_MovieGroup_001.

The product will include ultrasound preview image in private data elements designated by the Private Creator element:

TABLE 3-8

Element Name	Tag	VR	VM	Description
Private Creator	6003,00xx	LO	1	GEMS_Ultrasound_ImageGroup_001

This means that all private tags starting with 6003,xx will belong to the GEMS_Ultrasound_ImageGroup_001.

The product will include measurements raw data information in private data elements designated by the Private Creator element:

TABLE 3-9

Element Name	Tag	VR	VM	Description
Private Creator	6005,00xx	LO	1	GE_GROUP

This means that all private tags starting with 6005,xx will belong to the GE_GROUP.

The product will include exam information in private data elements designated by the Private Creator element:

TABLE 3-10

Element Name	Tag	VR	VM	Description
Private Creator	6FF1,00xx	LO	1	GE Contrast Quantification

This means that all private tags starting with 6FF1,xx will belong to the GE Contrast Quantification

The product will include exam information in private data elements designated by the Private Creator element:

TABLE 3-11

Element Name	Tag	VR	VM	Description
Private Creator	6001,00xx	LO	1	GE Generic Data

This means that all private tags starting with 6001,xx will belong to the GE Generic Data.

The product will include informatics data in private data elements designated by the Private Creator element:

TABLE 3-12

Element Name	Tag	VR	VM	Description
Private Creator	3003,00xx	LO	1	GE Informatics Data

This means that all private tags starting with 3003,xx will belong to the GE Informatics Data.

The product will include KoiosDS assessment data in private data elements designated by the Private Creator element:

TABLE 3-13

Element Name	Tag	VR	VM	Description
Private Creator	3001,00xx	LO	1	KoiosDS_Assessment

This means that all private tags starting with 3001,xx will belong to the KoiosDS Assessment.

3.6 CONFIGURATION

- Enable/disable raw data
- Convert raw clips to multi-frame
- Convert raw volumes to enhanced US volume
- Compression selections

3.7 SUPPORT OF EXTENDED CHARACTER SETS

LOGIQ Scanner will support the ISO IR 100 (ISO 8859-1:1987 Latin alphabet N 1. supplementary set). Any incoming SOP instance that is encoded using another extended character set will not be read.

4. ULTRASOUND IMAGE INFORMATION OBJECT IMPLEMENTATION

4.1 INTRODUCTION

This section specifies the use of the DICOM US Image IOD to represent the information included in US Images produced by this implementation. Corresponding attributes are conveyed using the module construct.

4.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 4-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Image	Image

4.3 IOD MODULE TABLE

The Ultrasound Image Information Object Definition comprises the modules of the following table. The elements not listed are not used by the application.

TABLE 4-2
US IMAGE IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1
	Patient Study	Used	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	General Series	Used	4.4.3.1
	Clinical Trial Series	Not used	N/A

Entity Name	Module Name	Usage	Reference
Frame of Reference	Frame of Reference	Not used	N/A
	Synchronization	Not used	N/A
Equipment	General Equipment	Used	4.4.4.1
Image	General Image	Used	4.4.5.1
	Image Pixel	Used	4.4.5.2
	Contrast/Bolus	Used Required if contrast media was used in this image	4.4.5.3
	Palette Color Lookup Table	Used Required if photometric Interpretation (0028,0004) has a value of PALETTE COLOR	4.4.5.4
	Device	Not used	N/A
	US Region Calibration	Used	4.4.5.5
	US Image	Used	4.4.5.6
	Overlay Plane	Not used	N/A
	VOI LUT	Used	4.4.5.7
	SOP Common	Used	4.4.5.8

4.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the US Information Object.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

4.4.1 Patient Entity Modules

4.4.1.1 Patient Module

TABLE 4-3
PATIENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Patient's Name	(0010,0010)	2	May be entered from User Interface. Taken from worklist if it is there.
Patient ID	(0010,0020)	2	May be entered from User Interface. Taken from worklist if it is there.
Issuer of Patient ID	(0010,0021)	3	Taken from worklist if it is there.
Patient's Birth Date	(0010,0030)	2	May be entered from User Interface. Taken from worklist if it is there.
Patient's Sex	(0010,0040)	2	May be entered from User Interface. Taken from worklist if it is there.
Referenced Patient Sequence	(0008,1120)	3	Taken from worklist if it is there.
<i>>Include 'SOP Instance Reference Macro'</i>			
Patient's Birth Time	(0010,0032)	3	Taken from worklist if it is there.
Other Patient IDs	(0010,1000)	3	Taken from worklist if it is there.
Other Patient IDs Sequence	(0010,1002)	3	Not supported.
Ethnic Group	(0010,2160)	3	Taken from worklist if it is there.
Patient Comments	(0010,4000)	3	Taken from worklist if it is there.

4.4.2 Study Entity Modules

4.4.2.1 General Study Module

TABLE 4-4
GENERAL STUDY MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Study Instance UID	(0020,000D)	1	Uniquely generated by the equipment. Taken from worklist if it is there.
Study Date	(0008,0020)	2	Is set to examination date
Study Time	(0008,0030)	2	Is set to examination time
Referring Physician's Name	(0008,0090)	2	May be entered from User Interface. Taken from worklist if it is there.
Study ID	(0020,0010)	2	Taken from worklist if it is there (from Requested Procedure ID) or set the study creation date/time in <YYYYMMDD.HHMMSS> format
Accession Number	(0008,0050)	2	May be entered from User Interface. Taken from worklist if it is there.
Study Description	(0008,1030)	3	Taken from worklist if it is there (from Requested Procedure Description).
Physician(s) of Record	(0008,1048)	3	Taken from worklist if it is there (from Names of Intended Recipients of Result)
Referenced Study Sequence	(0008,1110)	3	Taken from worklist if it is there. (Not used in SR Documents.)
>Include 'SOP Instance Reference Macro'			

4.4.2.2 Patient Study Module

TABLE 4-5
PATIENT STUDY MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Patient's Size	(0010,1020)	3	May be entered from User Interface. Taken from worklist if it is there.
Patient's Weight	(0010,1030)	3	May be entered from User Interface. Taken from worklist if it is there.

Attribute Name	Tag	Type	Attribute Description
Additional Patient's History	(0010,21B0)	3	Taken from worklist if it is there.
Admitting Diagnoses Description	(0008,1080)	3	May be entered from User Interface.
Visit Comments	(0038,4000)	3	May be entered from User Interface.

4.4.3 Series Entity Modules

4.4.3.1 General Series Module

TABLE 4-6
GENERAL SERIES MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Modality	(0008,0060)	1	Defined Term "US" used. When reading SC all modalities are accepted.
Series Instance UID	(0020,000E)	1	Uniquely generated by the equipment
Series Number	(0020,0011)	2	Internal number which is incremented for each new series (exam) within a study.
Series Date	(0008,0021)	3	Is set to Series date
Series Time	(0008,0031)	3	Is set to Series time
Performing Physicians' Name	(0008,1050)	3	May be entered from User Interface. Taken from worklist if it is there (from Scheduled Performing Physician's Name)
Protocol Name	(0018,1030)	3	Used in Stress Echo Exam Protocol
Series Description	(0008,103E)	3	Taken from worklist if it is there (from Scheduled Procedure Step Description)
Operators' Name	(0008,1070)	3	May be entered from User Interface. Default is login id.
Request Attributes Sequence	(0040,0275)	3	Used if Modality Worklist is enabled.
>Requested Procedure ID	(0040,1001)	1C	Taken from worklist if it is there.
>Accession Number	(0008,0050)	3	Not supported.
>Study Instance UID	(0020,000D)	3	Not supported.
>Referenced Study Sequence	(0008,1110)	3	Not supported.
>> Include 'SOP Instance Reference Macro'			

Attribute Name	Tag	Type	Attribute Description
>Requested Procedure Description	(0032,1060)	3	Not supported.
>Requested Procedure Code Sequence	(0032,1064)	3	Not supported.
>Scheduled Procedure Step ID	(0040,0009)	1C	Taken from worklist if it is there.
>Scheduled Procedure Step Description	(0040,0007)	3	Taken from worklist if it is there.
>Scheduled Protocol Code Sequence	(0040,0008)	3	Not supported.
Performed Procedure Step ID	(0040,0253)	3	Used if Modality Performed Procedure Step is enabled.
Performed Procedure Step Start Date	(0040,0244)	3	Used if Modality Performed Procedure Step is enabled.
Performed Procedure Step Start Time	(0040,0245)	3	Used if Modality Performed Procedure Step is enabled.
Performed Procedure Step Description	(0040,0254)	3	Used if Modality Performed Procedure Step is enabled.
Performed Protocol Code Sequence	(0040,0260)	3	Taken from worklist if it is there (from Scheduled Protocol Code Sequence) or in case of Stress test
>Include 'Code Sequence Macro'		Please refer Table: Table 4-7	

TABLE 4-7
PERFORMED PROTOCOL CODE SEQUENCE MAPPING

GE Value	DICOM Mapping
Bicycle Normal	(P2-31102, SRT, Stress test using Bicycle Ergometer)
Bicycle Sporty	(P2-31102, SRT, Stress test using Bicycle Ergometer)
Contrast Pharmacological	(P2-31107, SRT, Pharmacologic Stress protocol)
Pharmacological 4x4	(P2-31107, SRT, Pharmacologic Stress protocol)
Pharmacological 8x5	(P2-31107, SRT, Pharmacologic Stress protocol)
Exercise 2x4	(P5-B3050, SRT, Exercise stress echocardiography)
Exercise 2x4 B	(P5-B3050, SRT, Exercise stress echocardiography)
Pharmacological US 4x4	(P2-31107, SRT, Pharmacologic Stress protocol)

4.4.4 Equipment Entity Modules

4.4.4.1 General Equipment Module

TABLE 4-8
GENERAL EQUIPMENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Manufacturer	(0008,0070)	2	Is set to "GE Healthcare"
Institution Name	(0008,0080)	3	Is set to configured Institution Name.
Station Name	(0008,1010)	3	Is set to configured Station Name.
Institutional Department Name	(0008,1040)	3	May be entered from User Interface. Default is configured Department name.
Manufacturer's Model Name	(0008,1090)	3	Is set to "LOGIQE10" or "LOGIQE10s" or "LOGIQE11" or "LOGIQE20" based on the product
Software Versions	(0018,1020)	3	Is set to LOGIQE10 or LOGIQE10s or LOGIQE11 or LOGIQE20 software version based on the product
Device Serial Number	(0018,1000)	3	Is set to LOGIQ Product Serial number.

4.4.5 Image Entity Modules

4.4.5.1 General Image Module

TABLE 4-9
GENERAL IMAGE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Instance Number	(0020,0013)	2	Internal value which is incremented for each captured image within a series. (within an exam)
Patient Orientation	(0020,0020)	2C	Sent with empty value.
Content Date	(0008,0023)	2C	Set from Image date
Content Time	(0008,0033)	2C	Set from Image time
Image Type	(0008,0008)	3	The first two values contain 'ORIGINAL/PRIMARY' or 'DERIVED/ PRIMARY'.

Attribute Name	Tag	Type	Attribute Description																		
			<p>Value 3 is the description of the type of application per the DICOM PS3.3, section C.8.5.6.1.1; Supported enumerated values are: ABDOMINAL GYNECOLOGY OBSTETRICAL PEDIATRIC SMALL PARTS VASCULAR SCROTAL INTRACARDIAC</p> <p>Value 4 is a description of the mode per the DICOM PS3.3, section C.8.5.6.1.1; Supported enumerated values are sum of values in hexadecimal encoded, which based on:</p> <table><tr><th>Value</th><th>Meaning</th></tr><tr><td>0001</td><td>2D Imaging</td></tr><tr><td>0002</td><td>M- Mode</td></tr><tr><td>0004</td><td>CW Doppler</td></tr><tr><td>0008</td><td>PW Doppler</td></tr><tr><td>0010</td><td>Color Doppler</td></tr><tr><td>0020</td><td>Color M-Mode</td></tr><tr><td>0040</td><td>3D Rendering</td></tr><tr><td>0100</td><td>Color Power Mode</td></tr></table> <p>Values 5 and 6 may be used for private data.</p>	Value	Meaning	0001	2D Imaging	0002	M- Mode	0004	CW Doppler	0008	PW Doppler	0010	Color Doppler	0020	Color M-Mode	0040	3D Rendering	0100	Color Power Mode
Value	Meaning																				
0001	2D Imaging																				
0002	M- Mode																				
0004	CW Doppler																				
0008	PW Doppler																				
0010	Color Doppler																				
0020	Color M-Mode																				
0040	3D Rendering																				
0100	Color Power Mode																				
Lossy Image Compression	(0028,2110)	3	Set to 01, if image is lossy compressed.																		

Attribute Name	Tag	Type	Attribute Description
Presentation LUT Shape	(2050,0020)	3	Set with "IDENTITY".
Lossy Image Compression Ratio	(0028,2112)	3	Used if lossy compressed.

4.4.5.2 Image Pixel Module

TABLE 4-10
IMAGE PIXEL MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Samples per Pixel	(0028,0002)	1	Value of '1' if Photometric Interpretation element value has value 'MONOCHROME2', 'PALETTE COLOR' Value of '3' when Photometric Interpretation element value has value 'RGB', 'YBR_FULL' 'YBR_FULL_422' 'YBR_RCT' 'YBR_ICT'
Photometric Interpretation	(0028,0004)	1	Defined Values used: 'MONOCHROME2', (Encoding and Display) 'PALETTE COLOR' (Display only) 'RGB', (Compression sets to JPEG Lossless) 'YBR_FULL' (Compression sets to RLE) 'YBR_FULL_422' (Compression sets to JPEG Lossy) 'YBR_RCT' (Compression sets to JPEG 2000 Lossless) 'YBR_ICT' (Compression sets to JPEG 2000 Lossy)
Rows	(0028,0010)	1	Value depends on scanning mode and configuration setup
Columns	(0028,0011)	1	Value depends on scanning mode and

Attribute Name	Tag	Type	Attribute Description
			configuration setup
Bits Allocated	(0028,0100)	1	Value always = 0008H.
Bits Stored	(0028,0101)	1	Value always = 0008H.
High Bit	(0028,0102)	1	Value always = 0007H.
Pixel Representation	(0028,0103)	1	Defined Value '0' - unsigned integer.
Pixel Data	(7FE0,0010)	1	Pixel Data of image.
Planar Configuration	(0028,0006)	1C	Enumerated Values: 0000H = color-by-pixel, if Photometric Interpretation element value has value RGB, YBR_RCT, YBR_ICT or YBR_FULL_422. 0001H = color by plane, if Photometric Interpretation element has value YBR_FULL.
Pixel Aspect Ratio	(0028,0034)	1C	Not used

4.4.5.3 Contrast/Bolus Module

TABLE 4-11
CONTRAST/BOLUS MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Contrast/Bolus Agent	(0018,0010)	2	Contrast Agent won't be sent if no Contrast Agent was used to acquire images. If contrast agent is used, the value is taken from worklist (Requested Contrast Agent).

4.4.5.4 Palette Color Lookup Table Module

TABLE 4-12
PALETTE COLOR LOOKUP MODULE

Attribute Name	Tag	Type	Use
----------------	-----	------	-----

Red Palette Color Lookup Table Descriptor	(0028,1101)	1	Only used when reading Palette images.
Green Palette Color Lookup Table Descriptor	(0028,1102)	1	Only used when reading Palette images.
Blue Palette Color Lookup Table Descriptor	(0028,1103)	1	Only used when reading Palette images.
Red Palette Color Lookup Table Data	(0028,1201)	1C	Only used when reading Palette images.
Green Palette Color Lookup Table Data	(0028,1202)	1C	Only used when reading Palette images.
Blue Palette Color Lookup Table Data	(0028,1203)	1C	Only used when reading Palette images.

4.4.5.5 US Region Calibration Module

US Region Calibration Module is used to describe multiple regions. Note: if a multi-frame image has been acquired with different calibration, the US Region Calibration Module will not be used.

TABLE 4-13

US REGION CALIBRATION MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Sequence of Ultrasound Regions	(0018,6011)	1	Object contains 1 or more regions
>Region Location Min x_0	(0018,6018)	1	Value is 0
>Region Location Min y_0	(0018,601A)	1	Value is 0
>Region Location Max x_1	(0018,601C)	1	Value is image width-1
>Region Location Max y_1	(0018,601E)	1	Value is image height-1
>Physical Units X Direction	(0018,6024)	1	Enumerated Values supported: 0003H cm 0004H seconds
>Physical Units Y Direction	(0018,6026)	1	Enumerated Values supported: 0003H cm 0004H seconds 0007H cm/sec
>Physical Delta X	(0018,602C)	1	Varies with scanning mode
>Physical Delta Y	(0018,602E)	1	Varies with scanning mode
>Reference Pixel x_0	(0018,6020)	3	Varies with scanning mode
>Reference Pixel y_0	(0018,6022)	3	Varies with scanning mode

Attribute Name	Tag	Type	Use														
>Ref. Pixel Physical Value X	(0018,6028)	3	Varies with scanning mode														
>Ref. Pixel Physical Value Y	(0018,602A)	3	Varies with scanning mode														
>Region Spatial Format	(0018,6012)	1	Supported. The spatial organization of the data within the region. Supported enumerated values are: <table><tr><td>Value</td><td>Meaning</td></tr><tr><td>0000H</td><td>None or not applicable</td></tr><tr><td>0001H</td><td>2D (tissue or flow)</td></tr><tr><td>0002H</td><td>M -Mode (tissue or flow)</td></tr><tr><td>0003H</td><td>Spectral (CW or PW Doppler)</td></tr></table> Per the DICOM PS3.3, section C.8.5.5.1.1	Value	Meaning	0000H	None or not applicable	0001H	2D (tissue or flow)	0002H	M -Mode (tissue or flow)	0003H	Spectral (CW or PW Doppler)				
Value	Meaning																
0000H	None or not applicable																
0001H	2D (tissue or flow)																
0002H	M -Mode (tissue or flow)																
0003H	Spectral (CW or PW Doppler)																
>Region Data Type	(0018,6014)	1	Supported. The type of data within the region. Supported enumerated values are: <table><tr><td>Value</td><td>Meaning</td></tr><tr><td>0000H</td><td>None or not applicable</td></tr><tr><td>0001H</td><td>Tissue</td></tr><tr><td>0002H</td><td>Color Flow</td></tr><tr><td>0003H</td><td>PW Spectral Doppler</td></tr><tr><td>0004H</td><td>CW Spectral Doppler</td></tr><tr><td>0006H</td><td>Doppler Mode Trace</td></tr></table> Per the DICOM PS3.3, section C.8.5.5.1.2	Value	Meaning	0000H	None or not applicable	0001H	Tissue	0002H	Color Flow	0003H	PW Spectral Doppler	0004H	CW Spectral Doppler	0006H	Doppler Mode Trace
Value	Meaning																
0000H	None or not applicable																
0001H	Tissue																
0002H	Color Flow																
0003H	PW Spectral Doppler																
0004H	CW Spectral Doppler																
0006H	Doppler Mode Trace																
>Region Flags	(0018,6016)	1	Bit 0: 0 = Opaque Bit 1: 0 = Not Protected because there may be other regions within the image Bit 2: 0 = Velocity														
>Transducer Frequency	(0018,6030)	3	Supported														
>Pulse Repetition Frequency	(0018,6032)	3	Supported														

4.4.5.6 US Image Module

This section specifies the Attributes that describe ultrasound images.

TABLE 4-14
US IMAGE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Samples Per Pixel	(0028,0002)	1	Value of '1' if Photometric Interpretation element value has value 'MONOCHROME2' or 'PALETTE COLOR' Value of '3' when Photometric Interpretation element value has value 'RGB', (Compression sets to JPEG Lossless) 'YBR_FULL' (Compression sets to RLE) 'YBR_FULL_422' (Compression sets to JPEG Lossy) 'YBR_RCT' (Compression sets to JPEG 2000 Lossless) 'YBR_ICT' (Compression sets to JPEG 2000 Lossy)
Photometric Interpretation	(0028,0004)	1	Defined Values used: 'MONOCHROME2', 'RGB', 'PALETTE COLOR' 'YBR_FULL', 'YBR_FULL_422', 'YBR_RCT', 'YBR_ICT'
Bits Allocated	(0028,0100)	1	Value always = 0008H
Bits Stored	(0028,0101)	1	Value always = 0008H
High Bit	(0028,0102)	1	Value always = 0007H
Planar Configuration	(0028,0006)	1C	Enumerated Values: 0000H = color-by-pixel, if Photometric Interpretation element value has value RGB, YBR_RCT, YBR_ICT or

Attribute Name	Tag	Type	Use
			YBR_FULL_422. 0001H = color by plane, if Photometric Interpretation element has value YBR_FULL.
Pixel Representation	(0028,0103)	1	Always 0000H = unsigned integer.
Frame Increment Pointer	(0028,0009)	1C	Export: Is set to Frame Time (0018,1063) or Frame Time Vector (0018,1065) if the image is multiframe IOD, Not used if the image is a single frame IOD.
Image Type	(0008,0008)	2	<p>The first two values contain 'ORIGINAL/PRIMARY' or 'DERIVED/ PRIMARY'.</p> <p>Value 3 is the description of the type of application per the DICOM PS3.3, section C.8.5.6.1.1; Supported enumerated values are: ABDOMINAL GYNECOLOGY OBSTETRICAL PEDIATRIC SMALL PARTS VASCULAR SCROTAL INTRACARDIAC</p> <p>Value 4 is a description of the mode per the DICOM PS3.3, section C.8.5.6.1.1; Supported enumerated values are sum of values in hexadecimal encoded, which based on:</p>

Attribute Name	Tag	Type	Use
			Value Meaning 0001 2D Imaging 0002 M- Mode 0004 CW Doppler 0008 PW Doppler 0010 Color Doppler 0020 Color M-Mode 0040 3D Rendering 0100 Color Power Mode Values 5 and 6 may be used for private data.
Lossy Image Compression	(0028,2110)	1C	Set to 01 if image is compressed using JPEG Baseline (Process 1) or JPEG 2000 Lossy compression.
Number of Stages	(0008,2124)	2C	Used in case of Stress Echo exams.
Number of Views in Stage	(0008,212A)	2C	Used in case of Stress Echo exams.
Stage Name	(0008,2120)	3	Name of stage of stress test. Sent if image is acquired in a stress test. The name is defined in the User Interface.
Stage Code Sequence	(0040,000A)	3	Used in case of Stress Echo exams. Please refer Table: Table 4-15 for Stage Code Sequence macros.
Stage Number	(0008,2122)	3	Number of stage, starting at one. Sent if image is acquired in a stress test.
View Name	(0008,2127)	3	Name of view of stress test. Sent if image is acquired in a stress test. The name is defined in the User Interface.
View Number	(0008,2128)	3	Number of view, starting at one. Sent if image is acquired in a stress test.
View Code Sequence	(0054,0220)	3	Used in case of Stress Echo exams. Please refer Table: Table 4-16 for View Code Sequence macros.
Acquisition DateTime	(0008,002A)	1C	Not Used.
Heart Rate	(0018,1088)	3	Set to heart rate

Attribute Name	Tag	Type	Use
Transducer Data	(0018,5010)	3	Probe details. Contains 3 values. - Probe Name - Probe Id - Serial Number
Ultrasound Color Data Present	(0028,0014)	3	This element indicates if any ultrasound color data is present in an image. Value Meaning 00 Ultrasound color data not present in image 01 Ultrasound color data is present in image

TABLE 4-15
STAGE CODE SEQUENCE MAPPING

GE Value	DICOM Mapping
Rest	(F-01604, SRT, Resting State)
Peak	(F-05028, SRT, Peak cardiac stress state)
Baseline	(F-01604, SRT, Resting State)
Low dose	(F-05019, SRT, Cardiac stress state)
Peak dose	(F-05028, SRT, Peak cardiac stress state)
Recovery	(F-05018, SRT, Cardiac stress recovery state)

TABLE 4-16
VIEW CODE SEQUENCE MAPPING

GE Value	DICOM Mapping
4CH	(G-A19C, SRT, Apical four chamber)
2CH	(G-A19B, SRT, Apical two chamber)
PLAX	(G-0396, SRT, Parasternal long axis)
APLAX	(G-0395, SRT, Apical long axis)
SAX-PM	(G-039B, SRT, Parasternal short axis at the Papillary Muscle level)

SAX-MV	(G-039A, SRT, Parasternal short axis at the Mitral Valve level)
--------	---

4.4.5.7 VOI LUT module

TABLE 4-17
VOI LUT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Window Center	(0028,1050)	1C	Value set to 127 if Photometric Interpretation has value MONOCHROME2.
Window Width	(0028,1051)	1C	Value set to 256 if Photometric Interpretation has value MONOCHROME2.

4.4.5.8 SOP Common Module

TABLE 4-18
SOP COMMON MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
SOP Class UID	(0008,0016)	1	Set to "1.2.840.10008.5.1.4.1.1.3.1" "1.2.840.10008.5.1.4.1.1.3" "1.2.840.10008.5.1.4.1.1.6.1" "1.2.840.10008.5.1.4.1.1.6" "1.2.840.10008.5.1.4.1.1.7" "1.2.840.10008.5.1.4.1.1.6.2" "1.2.840.10008.5.1.4.1.1.88.59" "1.2.840.10008.5.1.4.1.1.88.33" or "1.2.840.10008.5.1.4.1.1.104.1"
SOP Instance UID	(0008,0018)	1	Uniquely generated by the equipment
Specific Character Set	(0008,0005)	1C	Set to ISO-IR100 if extended characters are used. Image Read: images using other set than ISO-IR 100 are rejected.
Instance Number	(0020,0013)	3	Internal value which is incremented for each captured image within a series. (within an exam)

5. ULTRASOUND MULTI-FRAME IMAGE INFORMATION OBJECT IMPLEMENTATION

5.1 INTRODUCTION

This section specifies the use of the DICOM US Multi-Frame Image IOD to represent the information included in US Multi-Frame Images produced by this implementation. Corresponding attributes are conveyed using the module construct.

5.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 5-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Image	Image

5.3 IOD MODULE TABLE

The Ultrasound Multi-Frame Image Information Object Definition comprises the modules of the following table. The elements not listed are not used by the application.

TABLE 5-2
US MULTI-FRAME IMAGE IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1
	Patient Study	Used	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	General Series	Used	4.4.3.1
	Clinical Trial Series	Not used	N/A

Entity Name	Module Name	Usage	Reference
Frame of Reference	Frame of Reference	Not Used	N/A
	Synchronization	Not Used	N/A
Equipment	General Equipment	Used	4.4.4.1
Image	General Image	Used	4.4.5.1
	Image Pixel	Used	4.4.5.2
	Contrast/Bolus	Used	4.4.5.3
	Cine	Used	5.4.1.1
	Multi-Frame	Used	5.4.1.2
	Frame Pointers	Not used	N/A
	Palette Color Lookup Table	Used	4.4.5.4
	Device	Not used	N/A
	US Region Calibration	Used	4.4.5.5
	US Image	Used	4.4.5.6
	VOI LUT	Used	4.4.5.7
	SOP Common	Used	4.4.5.8

5.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the US Multi-Frame Information Object.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

5.4.1 Image Entity Modules

5.4.1.1 Cine Module

TABLE 5-3
CINE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Preferred Playback Sequencing	(0018,1244)	3	Supported
Frame Time	(0018,1063)	1C	Is set to the interframe time. XOR Frame Time Vector.
Frame Time Vector	(0018,1065)	1C	Supported. XOR Frame Time.
Start Trim	(0008,2142)	3	Supported
Stop Trim	(0008,2143)	3	Supported
Recommended Display Frame Rate	(0008,2144)	3	Supported
Cine Rate	(0018,0040)	3	Supported
Frame Delay	(0018,1066)	3	Supported
Effective Duration	(0018,0072)	3	Supported
Actual Frame Duration	(0018,1242)	3	Supported

5.4.1.2 Multi-Frame Module

TABLE 5-4
MULTI-FRAME MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Number of Frames	(0028,0008)	1	Is set to the number of frames in image
Frame Increment Pointer	(0028,0009)	1	Is set to Frame Time (0018,1063) or Frame Time Vector (0018,1065)

6. ENHANCED US VOLUME INFORMATION OBJECT IMPLEMENTATION

6.1 INTRODUCTION

This section specifies the use of the DICOM Enhanced US Volume IOD to represent the information included in Enhanced US Volume Images produced by this implementation. Corresponding attributes are conveyed using the module construct.

6.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 6-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Image	Image

6.3 IOD MODULE TABLE

The Enhanced US Volume Information Object Definition comprises the modules of the following table. The elements not listed are not used by the application.

TABLE 6-2
ENHANCED US VOLUME IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1
	Patient Study	Used	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	General Series	Used	4.4.3.1
	Enhanced US	Used	6.4.1.1

Entity Name	Module Name	Usage	Reference
	Series		
	Clinical Trial Series	Not used	N/A
Frame of Reference	Frame of Reference	Used	6.4.2.1
	Ultrasound Frame of Reference	Used	6.4.2.1.1
	Synchronization	Used	6.4.2.2
Equipment	General Equipment	Used	4.4.4.1
	Enhanced General Equipment	Used	6.4.3.1
Image	General Image	Used	4.4.5.1
	Image Pixel	Used	4.4.5.2
	Enhanced Contrast/Bolus	Not used	N/A
	Multi-frame Functional Groups	Used	6.4.4.1
	Multi-frame Dimension	Used	6.4.4.2
	Cardiac Synchronization	Not used	N/A
	Respiratory Synchronization	Not used	N/A
	Device	Not used	N/A
	Acquisition Context	Used	6.4.4.3
	Specimen	Not used	N/A
	Enhanced Palette Color Lookup Table	Used	6.4.4.4
	Enhanced US Image	Used	6.4.4.5
	IVUS Image	Not used	N/A
	Excluded Intervals	Not used	N/A
	ICC Profile	Not used	N/A
	SOP Common	Used	4.4.5.8
	Frame Extraction	Not used	N/A

6.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the Enhanced US Volume Information Object.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance as well as what are the expected values when loading such instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

6.4.1 Series Entity Modules

6.4.1.1 Enhanced US Series

TABLE 6-3
ENHANCED US SERIES MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Modality	(0008,0060)	1	Set to US.
Referenced Performed Procedure Step Sequence	(0008,1111)	1C	Used if Modality Performed Procedure Step is enabled.
<i>>Include 'SOP Instance Reference Macro' Table 10-11</i>			
Performed Protocol Code Sequence	(0040,0260)	1C	Not Used.
Performed Protocol Type	(0040,0261)	1C	Used if Modality Performed Procedure Step is enabled.

6.4.2 Frame Of Reference Entity Modules

6.4.2.1 Frame Of Reference Module

TABLE 6-4
FRAME OF REFERENCE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Frame of Reference UID	(0020,0052)	1	Used.
Position Reference Indicator	(0020,1040)	2	Set as empty.

6.4.2.1.1 Ultrasound Frame of Reference

TABLE 6-5
ULTRASOUND FRAME OF REFERENCE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Volume Frame of Reference UID	(0020,9312)	1	Used.
Ultrasound Acquisition Geometry	(0020,9307)	1	Set with defined term "APEX".
Apex Position	(0020,9308)	1C	Used.
Volume to Transducer Mapping Matrix	(0020,9309)	1	Used.
Patient Frame of Reference Source	(0020,930C)	1C	Not used.
Table Frame of Reference UID	(0020,9313)	1C	Not used.
Volume to Table Mapping Matrix	(0020,930A)	1C	Not used.

6.4.2.2 Synchronization Module

TABLE 6-6
SYNCHRONIZATION MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Synchronization Frame of Reference UID	(0020,0200)	1	Used.
Synchronization Trigger	(0018,106A)	1	Enumerated Value "NO TRIGGER" used.
Synchronization Channel	(0018,106C)	1C	Not used.
Acquisition Time Synchronized	(0018,1800)	1	Enumerated Value "N" used.

6.4.3 Equipment Entity Modules

6.4.3.1 Enhanced General Equipment Module

TABLE 6-7
ENHANCED GENERAL EQUIPMENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Manufacturer	(0008,0070)	1	Set with "GE Healthcare".
Manufacturer's Model Name	(0008,1090)	1	Set with "LOGIQE10" or "LOGIQE10s" or "LOGIQE11" or "LOGIQE20".
Device Serial Number	(0018,1000)	1	Set with serial number of device.
Software Versions	(0018,1020)	1	Is set to LOGIQE10 or LOGIQE10s or

			LOGIQE11 or LOGIQE20 software version.
--	--	--	--

6.4.4 Image Entity Modules

6.4.4.1 Multi-frame Functional Groups

TABLE 6-8
MULTI-FRAME FUNCTIONAL GROUPS MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Shared Functional Groups Sequence	(5200,9229)	2	Used.
>US Image Description Sequence	(0018,9806)	1	Used.
>>Frame Type	(0008,9007)	1	The first two values contain 'ORIGINAL/PRIMARY' and 'DERIVED/PRIMARY'.
>>Volumetric Properties	(0008,9206)	1	Set with "VOLUME".
>>Volume Based Calculation Technique	(0008,9207)	1	Set with "NONE".
>Plane Orientation (Volume) Sequence	(0020,930F)	1	Used.
>>Image Orientation (Volume)	(0020,9302)	1	Set with "1\0\0\0\1\0".
>Temporal Position Sequence	(0020,9310)	1	Filled with Temporal position value. Not used in case of multiple volumes (4D).
>>Temporal Position Time Offset	(0020,930D)	1	Used.
> Pixel Measures Sequence	(0028,9110)	1	Used
>>Pixel Spacing	(0028,0030)	1C	Used
>>Slice Thickness	(0018,0050)	1C	Used
>Frame VOI LUT Sequence	(0028,9132)	1	Used.
>>Window Center	(0028,1050)	1	Used.
>>Window Width	(0028,1051)	1	Used.
Per-frame Functional Groups Sequence	(5200,9230)	1	Used.
>Image Data Type Sequence	(0018,9807)	1	Used.
>>Data Type	(0018,9808)	1	Set with "TISSUE_INTENSITY" or

Attribute Name	Tag	Type	Attribute Description
			"FLOW_VELOCITY" or "FLOW_POWER".
>>Aliased Data Type	(0018,980B)	1	Set to "NO".
>Frame Content Sequence	(0020,9111)	1	Used.
>>Frame Reference DateTime	(0018,9151)	1C	Set with the date and time of the acquisition frame created. Used if Frame Type (0008,9007) Value 1 of this frame is ORIGINAL. May be present otherwise.
>>Frame Acquisition DateTime	(0018,9074)	1C	Set with the date and time of the acquisition frame started. Used if Frame Type (0008,9007) Value 1 of this frame is ORIGINAL. May be present otherwise.
>>Frame Acquisition Duration	(0018,9220)	1C	Set with the duration of the time. Used if Frame Type (0008,9007) Value 1 of this frame is ORIGINAL. May be present otherwise.
>>Dimension Index Values	(0020,9157)	1C	Set with 3 values. Each value fills with the dimension specific information. Used if Frame Type (0008,9007) Value 1 of this frame is ORIGINAL. May be present otherwise.
>Plane Position (Volume) Sequence	(0020,930E)	1	Used.
>>Image Position (Volume)	(0020,9301)	1	First value (X) and second value (Y) are set to zero. The third value is set to the plane position in mm.
>Temporal Position Sequence	(0020,9310)	1	Filled with Temporal position value. Not used in case of single volume (3D).
>>Temporal Position Time Offset	(0020,930D)	1	Used.
Instance Number	(0020,0013)	1	Used.
Content Date	(0008,0023)	1	Used.
Content Time	(0008,0033)	1	Used.
Number of Frames	(0028,0008)	1	Used.
Concatenation UID	(0020,9161)	1C	Not used.

6.4.4.2 Multi-frame Dimension

TABLE 6-9
MULTI-FRAME DIMENSION MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Dimension Organization Sequence	(0020,9221)	1	Used.
>Dimension Organization UID	(0020,9164)	1	Used.
Dimension Organization Type	(0020,9311)	3	Set with “3D” in case of Static 3D dataset or “3D_TEMPORAL” in case of 4D dataset.
Dimension Index Sequence	(0020,9222)	1	Three items are sent.
>Dimension Index Pointer	(0020,9165)	1	There are three values. “Temporal Position Time Offset” or “Image Position (Volume)” or “Data Type”.
>Functional Group Pointer	(0020,9167)	1C	There are three values. “Temporal Position Sequence” or “Plane Position Volume Sequence” or “Image Data Type Sequence”.
>Dimension Organization UID	(0020,9164)	1C	Set with unique id.

6.4.4.3 Acquisition Context

TABLE 6-10
ACQUISITION CONTEXT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Acquisition Context Sequence	(0040,0555)	2	Set as empty.

6.4.4.4 Enhanced Palette Color Lookup Table

TABLE 6-11
ENHANCED PALETTE COLOR LOOKUP TABLE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Data Frame Assignment Sequence	(0028,1401)	1	Sequence of items each assigning frames of one particular value of Data Type (0018,9808) to a data path in the Enhanced Blending and Display Pipeline. One, two, or three items shall be included in this sequence.
>Data Type	(0018,9808)	1	Set with "TISSUE_INTENSITY" or "FLOW_VELOCITY" or "FLOW_POWER".
>Data Path Assignment	(0028,1402)	1	Set with following Enumerated values. PRIMARY_SINGLE SECONDARY_SINGLE SECONDARY_HIGH SECONDARY_LOW
>Window Center	(0028,1050)	1	Filled with window center value.
>Window Width	(0028,1051)	1	Filled with window width value.
Blending LUT 1 Sequence	(0028,1404)	1C	Used if there are one or more items of the Data Path Assignment Sequence (0028,1402) other than PRIMARY_PVALUES.
>Blending LUT 1 Transfer Function	(0028,1405)	1	Supported Enumerated values: CONSTANT ALPHA_1 ALPHA_2 TABLE See DICOM PS3.3, section C.7.6.23.4 for details.
>Blending Weight Constant	(0028,1406)	1C	Used if (0028,1405) value is "CONSTANT".
>Blending Lookup Table Descriptor	(0028,1407)	1C	Used if (0028,1405) value is "TABLE".
>Blending Lookup Table Data	(0028,1408)	1C	Used if (0028,1405) value is "TABLE".

Attribute Name	Tag	Type	Attribute Description
Blending LUT 2 Sequence	(0028,140C)	1C	Used if there are one or more items of the Data Path Assignment Sequence (0028,1402) other than PRIMARY_PVALUES.
>Blending LUT 2 Transfer Function	(0028,140D)	1	Supported Enumerated values: CONSTANT ONE_MINUS ALPHA_1 ALPHA_2 TABLE See DICOM PS3.3, C.7.6.23.4 for details.
>Blending Weight Constant	(0028,1406)	1C	Used if (0028,140D) is CONSTANT.
>Blending Lookup Table Descriptor	(0028,1407)	1C	Used if (0028,140D) is TABLE.
>Blending Lookup Table Data	(0028,1408)	1C	Used if (0028,140D) is TABLE.
Enhanced Palette Color Lookup Table Sequence	(0028,140B)	1C	Used if Data Path Assignment (0028,1402) is present with a value other than PRIMARY_PVALUES.
>Data Path ID	(0028,140E)	1	Used below enumerated values. PRIMARY SECONDARY
>RGB LUT Transfer Function	(0028,140F)	1	Set with "TABLE".
>Alpha LUT Transfer Function	(0028,1410)	1	Set with either "NONE" or "TABLE".
>Red Palette Color Lookup Table Descriptor	(0028,1101)	1C	Used.
>Green Palette Color Lookup Table Descriptor	(0028,1102)	1C	Used.
>Blue Palette Color Lookup Table Descriptor	(0028,1103)	1C	Used.
>Alpha Palette Color Lookup Table Descriptor	(0028,1104)	1C	Used.
>Red Palette Color Lookup Table Data	(0028,1201)	1C	Used.
>Green Palette Color Lookup Table Data	(0028,1202)	1C	Used.

Attribute Name	Tag	Type	Attribute Description
>Blue Palette Color Lookup Table Data	(0028,1203)	1C	Used.
>Alpha Palette Color Lookup Table Data	(0028,1204)	1C	Used.
ICC Profile	(0028,2000)	1C	Not used.

6.4.4.5 Enhanced US Image

TABLE 6-12
ENHANCED US IMAGE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
Image Type	(0008,0008)	1	The first two values contain 'ORIGINAL/PRIMARY' and 'DERIVED/ PRIMARY'.
Samples Per Pixel	(0028,0002)	1	Set with 1.
Photometric Interpretation	(0028,0004)	1	Set with MONOCHROME2.
Bits Allocated	(0028,0100)	1	Set with 8.
Bits Stored	(0028,0101)	1	Set with 8.
High Bit	(0028,0102)	1	Set with 7.
Pixel Representation	(0028,0103)	1	Set with 0.
Dimension Organization Type	(0020,9311)	1	Set with “3D” in case of Static 3D dataset or “3D_TEMPORAL” in case of 4D dataset.
Acquisition DateTime	(0008,002A)	1	Used.
Acquisition Duration	(0018,9073)	1	Used.
Pixel Spacing	(0028,0030)	1	Used.
Position Measuring Device Used	(0018,980C)	1C	Set with “RIGID”.
Lossy Image Compression	(0028,2110)	1C	Not used since Lossy compression is not an option.
Lossy Image Compression Ratio	(0028,2112)	1C	Not used since Lossy compression is not an option.
Lossy Image Compression Method	(0028,2114)	1C	Not used since Lossy compression is not an option.
Presentation LUT Shape	(2050,0020)	1	Set with “IDENTITY”.
Rescale Intercept	(0028,1052)	1	Set to 0.

Attribute Name	Tag	Type	Attribute Description
Rescale Slope	(0028,1053)	1	Set to 1.
Source Image Sequence	(0008,2112)	1C	Not used.
Number of Stages	(0008,2124)	1C	Not used.
Stage Number	(0008,2122)	1C	Not used.
Stage Code Sequence	(0040,000A)	1C	Not used.
View Code Sequence	(0054,0220)	1	Used.
>Code Value	(0008,0100)	1	Set to "G-A117".
>Coding Scheme Designator	(0008,0102)	1	Set to "SRT".
>Code Meaning	(0008,0104)	1	Set to "Transverse".
Burned In Annotation	(0028,0301)	1	Set to "NO".
Transducer Scan Pattern Code Sequence	(0018,9809)	1	The scan pattern the transducer is capable of. Only a single item is included in this sequence. See DICOM PS3.3, C.8.24.3.4 for further explanation.
>Code Value	(0008,0100)	1	Set to "125241".
>Coding Scheme Designator	(0008,0102)	1	Set to "DCM".
>Code Meaning	(0008,0104)	1	Set to "Plane scan pattern".
Transducer Geometry Code Sequence	(0018,980D)	1	Geometric structure of the transducer. Only a single item shall be included in this sequence. See DICOM PS3.3, C.8.24.3.4 for further explanation.
>Code Value	(0008,0100)	1	Set to "125254" if the volume acquired using Mechanical 4D probes or "125253" in case of Tru3D probes.
>Coding Scheme Designator	(0008,0102)	1	Set to "DCM".
>Code Meaning	(0008,0104)	1	Set to "Sector ultrasound transducer geometry" if the volume acquired with Mechanical 4D probes or "Curved linear ultrasound transducer geometry" in case of Tru3D probes.

Attribute Name	Tag	Type	Attribute Description
Transducer Beam Steering Code Sequence	(0018,980E)	1	Technique used by the transducer for beam steering. One or more items shall be included in this sequence. If more than one item is present, the order is significant from plane-forming technique to volume-forming technique. See DICOM PS3.3, C.8.24.3.4 for further explanation.
>Code Value	(0008,0100)	1	Set to "125258".
>Coding Scheme Designator	(0008,0102)	1	Set to "DCM".
>Code Meaning	(0008,0104)	1	Set to "Mechanical beam steering".
Transducer Application Code Sequence	(0018,980F)	1	The primary clinical application of the transducer. Only a single Item is included in this sequence. See DICOM PS3.3, C.8.24.3.4 for further explanation.
>Code Value	(0008,0100)	1	Set to "125261".
>Coding Scheme Designator	(0008,0102)	1	Set to "DCM".
>Code Meaning	(0008,0104)	1	Set to "External Transducer".
Mechanical Index	(0018,5022)	1	Used.
Bone Thermal Index	(0018,5024)	1	Used.
Cranial Thermal Index	(0018,5026)	1	Used.
Soft Tissue Thermal Index	(0018,5027)	1	Used.
Depth(s) of Focus	(0018,9801)	1	Used.
Depth of Scan Field	(0018,5050)	1	Used.
Anatomic Region Sequence	(0008,2218)	1	Set to (GEU-1099-1, 99GEMS, "Ultrasound Region")

7. SECONDARY CAPTURE INFORMATION OBJECT IMPLEMENTATION

7.1 INTRODUCTION

This section specifies the use of the DICOM SC Image IOD to represent the information included in SC Images produced by this implementation. Corresponding attributes are conveyed using the module construct.

7.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 7-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Image	Image

7.3 IOD MODULE TABLE

The Secondary Capture Information Object Definition comprises the modules of the following table. The elements not listed are not used by the application.

TABLE 7-2
SC IMAGE IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1
	Patient Study	Used < Need to specify when>	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	General Series	Used	4.4.3.1
	Clinical Trial Series	Not used	N/A
Equipment	General Equipment	Used < Need to specify when>	4.4.4.1
	SC Equipment	Used	7.4.1.1
Image	General Image	Used	4.4.5.1
	Image Pixel	Used	4.4.5.2
	Device	Not used	N/A
	SC Image	Used	7.4.2.1
	Overlay Plane	Not used	N/A
	Modality LUT	Not used	N/A
	VOI LUT	Used < Need to specify when>	4.4.5.7
	SOP Common	Used	4.4.5.8

7.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Standard Part 3 (Information Object Definitions) for a description of each of the entities and modules contained within the SC Information Object.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

7.4.1 Equipment Entity Modules

7.4.1.1 SC Equipment Module

TABLE 7-3
SC EQUIPMENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Conversion Type	(0008,0064)	1	Set to "WSD"
Modality	(0008,0060)	3	Defined Value "US" used. When reading SC all modalities are accepted
Secondary Capture Device ID	(0018,1010)	3	Is set to "LOGIQE10" or "LOGIQE10s" or "LOGIQE11" or "LOGIQE20" based on the product.
Secondary Capture Device Manufacturer	(0018,1016)	3	Implementation defined string "GE Healthcare"
Secondary Capture Device Manufacturer's Model Name	(0018,1018)	3	Is set to "LOGIQE10" or "LOGIQE10s" or "LOGIQE11" or "LOGIQE20" based on the product.
Secondary Capture Device Software Version	(0018,1019)	3	Is set to LOGIQE10 or LOGIQE10s or LOGIQE11 or LOGIQE20 software version based on the product.

7.4.2 Image Entity Modules

7.4.2.1 SC Image Module

TABLE 7-4
SC IMAGE MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Date of Secondary Capture	(0018,1012)	3	Image capture date
Time of Secondary Capture	(0018,1014)	3	Image capture time

8. COMPREHENSIVE STRUCTURED REPORT INFORMATION OBJECT IMPLEMENTATION

8.1 INTRODUCTION

This section specifies the use of the DICOM Comprehensive SR IOD to represent results produced by this implementation. Corresponding attributes are conveyed using the module construct.

8.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 8-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Document	Document

8.3 IOD MODULE TABLE

The Comprehensive Structured Report Information Object Definitions comprise the modules of the following tables, plus Standard Extended and Private attributes. Standard Extended and Private attributes are described in Section 8.5. The elements not listed are not used by the application. The contents of the SR Document Content are constrained by the supported template, as identified in Section 8.4.2.2.1.1. Standard, Standard Extended and Private templates are further described in Section 8.7.

TABLE 8-2
STRUCTURE REPORT IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Specimen Identification	Not used	N/A
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1

Entity Name	Module Name	Usage	Reference
	Patient Study	Used	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	SR Document Series	Used	8.4.1.1
	Clinical Trial Series	Not used	N/A
Equipment	General Equipment	Used	4.4.4.1
Document	SR Document General	Used	8.4.2.1
	SR Document Content	Used	8.4.2.2
	SOP Common	Used	4.4.5.8

8.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the SR Information Objects.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

8.4.1 Series Entity Modules

8.4.1.1 SR Document Series Module

TABLE 8-3
SR DOCUMENT SERIES MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Modality	(0008,0060)	1	Value = SR
Series Instance UID	(0020,000E)	1	Uniquely generated by the equipment
Series Number	(0020,0011)	1	Internal number which is incremented for each new exam within a study.
Referenced Performed Procedure Step Sequence	(0008,1111)	2	Sent as empty.

> 'Referenced SOP Class / Instance
UIDs'

8.4.2 Document Entity Modules

8.4.2.1 SR Document General Module

TABLE 8-4
SR DOCUMENT GENERAL MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Instance Number	(0020,0013)	1	Internal number which is incremented for each new SR document within a study.
Completion Flag	(0040,A491)	1	Defined Term "PARTIAL" used.
Verification Flag	(0040,A493)	1	Defined Term "UNVERIFIED" used.
Content Date	(0008,0023)	1	Date of creation, based upon user action that invoked the use of the object
Content Time	(0008,0033)	1	Time of creation, based upon user action that invoked the use of the object.
Referenced Request Sequence	(0040,A370)	1C	Filled in if the exam is based on a Worklist entry
>Study Instance UID	(0020,000D)	1	Taken from Study Instance UID in General Study Module
>Referenced Study Sequence	(0008,1110)	2	Sent as empty.
>>Include 'SOP Instance Reference Macro'			
>Accession Number	(0008,0050)	2	Taken from Accession Number in General Study Module
>Placer Order Number/Imaging Service Request	(0040,2016)	2	Sent as empty.
>Filler Order Number/Imaging Service Request	(0040,2017)	2	Sent as empty.
>Requested Procedure ID	(0040,1001)	2	Taken from worklist if it is there
>Requested Procedure Description	(0032,1060)	2	Taken from worklist if it is there
>Requested Procedure Code Sequence	(0032,1064)	2	Sent as empty.
>>Include 'Code Sequence Macro'			

Attribute Name	Tag	Type	Use
Performed Procedure Code Sequence	(0040,A372)	2	Sent as empty.
<i>>Include 'Code Sequence Macro'</i>			
Current Requested Procedure Evidence Sequence	(0040,A375)	1	List of all composite SOP Instances of US Single, US MF, Enhanced US Volume or Secondary Capture referenced in the Content Sequence (0040,A730).
<i>>Include 'Hierarchical SOP Instance Reference Macro'</i>			
Author Observer Sequence	(0040,A078)	3	Filled with observer information.

8.4.2.2 SR Document Content Module

TABLE 8-5
SR DOCUMENT CONTENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Observation DateTime	(0040,A032)	1C	Not used

Attribute Name	Tag	Type	Use																						
Content Template Sequence	(0040,A504)	1C	Depending on application, see template																						
			<table><tr><th>Application</th><th>Template ID</th></tr><tr><td>Small Parts/Scrotal</td><td>GEMS_US_0100 Ultrasound M&A document title (99GEMS)</td></tr><tr><td>Pediatrics</td><td>GEMS_US_0100 Ultrasound M&A document title (99GEMS)</td></tr><tr><td>Urology</td><td>GEMS_US_0100 Ultrasound M&A document title (99GEMS)</td></tr><tr><td>Vascular</td><td>5100 Vascular US Proc Rep (DCMR)</td></tr><tr><td>Abdomen</td><td>5100 Vascular US Proc Rep (DCMR)</td></tr><tr><td>OB</td><td>5000 OB-GYN US Proc Rep (DCMR)</td></tr><tr><td>GYN</td><td>5000 OB-GYN US Proc Rep (DCMR)</td></tr><tr><td>Cardiac</td><td>5200 Echocardiography Proc Rep (DCMR)</td></tr><tr><td>Small Parts/Thyroid</td><td>GEMS_US_SP_0100 Soft Tissue Neck and Head Imaging Report (99GEMS)</td></tr><tr><td>Small Parts/Breast</td><td>4200 Breast Imaging Report (DCMR)</td></tr></table>	Application	Template ID	Small Parts/Scrotal	GEMS_US_0100 Ultrasound M&A document title (99GEMS)	Pediatrics	GEMS_US_0100 Ultrasound M&A document title (99GEMS)	Urology	GEMS_US_0100 Ultrasound M&A document title (99GEMS)	Vascular	5100 Vascular US Proc Rep (DCMR)	Abdomen	5100 Vascular US Proc Rep (DCMR)	OB	5000 OB-GYN US Proc Rep (DCMR)	GYN	5000 OB-GYN US Proc Rep (DCMR)	Cardiac	5200 Echocardiography Proc Rep (DCMR)	Small Parts/Thyroid	GEMS_US_SP_0100 Soft Tissue Neck and Head Imaging Report (99GEMS)	Small Parts/Breast	4200 Breast Imaging Report (DCMR)
			Application	Template ID																					
			Small Parts/Scrotal	GEMS_US_0100 Ultrasound M&A document title (99GEMS)																					
			Pediatrics	GEMS_US_0100 Ultrasound M&A document title (99GEMS)																					
			Urology	GEMS_US_0100 Ultrasound M&A document title (99GEMS)																					
			Vascular	5100 Vascular US Proc Rep (DCMR)																					
			Abdomen	5100 Vascular US Proc Rep (DCMR)																					
			OB	5000 OB-GYN US Proc Rep (DCMR)																					
			GYN	5000 OB-GYN US Proc Rep (DCMR)																					
			Cardiac	5200 Echocardiography Proc Rep (DCMR)																					
			Small Parts/Thyroid	GEMS_US_SP_0100 Soft Tissue Neck and Head Imaging Report (99GEMS)																					
			Small Parts/Breast	4200 Breast Imaging Report (DCMR)																					
For flagging images rejected with quality reasons: Template ID: 2010 - Key Object Selection (DCMR)																									
>Mapping Resource	(0008,0105)	1																							
>Template Identifier	(0040,DB00)	1																							
Value Type	(0040,A040)	1	CONTAINER																						
Continuity of Content	(0040,A050)	1C	SEPARATE																						

Attribute Name	Tag	Type	Use
Concept Name Code Sequence	(0040,A043)	1C	<p>Depending on application, see template 'Ultrasound M&A document title' (GEMS_US_0100) 'OB-GYN Ultrasound Procedure Report' 'Vascular Ultrasound Procedure Report' or 'Echocardiography Procedure Report' 'Breast Imaging Report' 'Soft Tissue Neck and Head Imaging Report'</p> <p>For flagging images which are rejected or deleted during direct store (in progress sending): (113001, DCM, 'Rejected for Quality Reasons')</p>

Attribute Name	Tag	Type	Use
>Include 'Code Sequence Macro'			
Insert Concept Value attribute(s)			
Content Sequence	(0040,A730)	1C	Depending on application, see template 'Ultrasound M&A document title' 'OB-GYN Ultrasound Procedure Report' 'Vascular Ultrasound Procedure Report' or 'Echocardiography Procedure Report' 'Breast Imaging Report' 'Soft Tissue Neck and Head Imaging Report' For flagging images see 'Key Object Selection'
> Relationship Type	(0040,A010)	1	Depending on application, see template 'Ultrasound M&A document title', 'OB-GYN Ultrasound Procedure Report', 'Vascular Ultrasound Procedure Report' or 'Echocardiography Procedure Report. 'Breast Imaging Report' 'Soft Tissue Neck and Head Imaging Report' For flagging images see 'Key Object Selection'
> Referenced Content Item Identifier	(0040,DB73)	1C	Not used

Attribute Name	Tag	Type	Use
> <i>Insert SR DocumentContent Module</i>			Depending on application, see template ‘Ultrasound M&A document title’, ‘OB-GYN Ultrasound Procedure Report’, ‘Vascular Ultrasound Procedure Report’ or ‘Echocardiography Procedure Report’. ‘Breast Imaging Report’ ‘Soft Tissue Neck and Head Imaging Report’ For flagging images see ‘Key Object Selection’

8.4.2.2.1 SR Document Content Descriptions

8.4.2.2.1.1 Content Template

The product supports the following root Templates for SR SOP Instances created, processed, or displayed by the product.

TABLE 8-6
SR ROOT TEMPLATES

SOP Class	Template ID	Template Name	Use
Comprehensive SR	GEMS_US_0100	Ultrasound M&A document title	Create
Comprehensive SR	5000	OB-GYN Ultrasound Procedure Report	Create
Comprehensive SR	5100	Vascular Ultrasound Procedure Report	Create
Comprehensive SR	5200	Echocardiography Procedure Report	Create
Comprehensive SR	4200	Breast Imaging Report	Create
Comprehensive SR	GEMS_US_SP_0100	Soft Tissue Neck and Head Imaging Report’	Create
Key Object Selection Document	2010	Key Object Selection Document	Create

Refer to section 8.7 for a detailed description of the supported templates.

8.5 STANDARD EXTENDED AND PRIVATE DATA ATTRIBUTES

The product will send measurements raw data information in private data elements designated by the Private Creator element:

Element Name	Tag	VR	VM	Description
Private Creator	6005,00xx	LO	1	GE_GROUP

This means that all private tags starting with 6005,xx will belong to the GE_GROUP.

8.6 STANDARD EXTENDED AND PRIVATE CONTEXT GROUPS

The Product supports coded terminology using Standard Extended, Private, and Configurable Context Groups defined in the following sections.

- Vascular Ultrasound Procedure Report
- Echocardiography Procedure Report
- OB-GYN Ultrasound Procedure Report
- Breast Imaging Report
- Soft tissue neck and head Imaging Report
- GE Ultrasound M&A Report

8.7 STANDARD, STANDARD EXTENDED AND PRIVATE TEMPLATES

The Product supports the Standard Extended and Private Templates defined in the following sections.

8.7.1 Standard Templates

The Product supports the following standard templates for SOP Instances created by this product.

Application	Template ID
Vascular	5100 Vascular US Proc Rep (DCMR)
Abdomen	5100 Vascular US Proc Rep (DCMR)
OB	5000 OB-GYN US Proc Rep (DCMR)
GYN	5000 OB-GYN US Proc Rep (DCMR)
Cardiac	5200 Echocardiography Proc Rep (DCMR)
Breast	4200 Breast Imaging Report (DCMR)

8.7.2 Standard Extended Templates

Not applicable

8.7.3 Private Templates

The Product supports the following private templates for SOP Instances created by this product.

Application	Template ID
Small Parts/Scrotal	GEMS_US_0100 Ultrasound M&A document title (99GEMS)
Pediatrics	GEMS_US_0100 Ultrasound M&A document title (99GEMS)
Urology	GEMS_US_0100 Ultrasound M&A document title (99GEMS)
Small Parts/Thyroid	GEMS_US_SP_0100 Soft Tissue Neck and Head Imaging Report (99GEMS)

9. ENCAPSULATED PDF INFORMATION OBJECT IMPLEMENTATION

9.1 INTRODUCTION

This section specifies the use of the DICOM Encapsulated PDF IOD to represent results produced by this implementation. Corresponding attributes are conveyed using the module construct.

9.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 9-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Encapsulated Document	Report

9.3 IOD MODULE TABLE

The Encapsulated PDF Information Object Definitions comprise the modules of the following tables, plus Standard Extended and Private attributes. Standard Extended and Private attributes are described in Section 9.5. The elements not listed are not used by the application.

TABLE 9-2
ENCAPSULATED PDF IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Specimen Identification	Not used	N/A
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1
	Patient Study	Used	4.4.2.2

Entity Name	Module Name	Usage	Reference
	Clinical Trial Study	Not used	N/A
Series	Encapsulated Document Series	Used	9.4.1.1
	Clinical Trial Series	Not used	N/A
Equipment	General Equipment	Used	4.4.4.1
	SC Equipment	Not Used	7.4.1.1
Document	Encapsulated Document	Used	9.4.2.1
	SOP Common	Used	4.4.5.8

9.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the SR Information Objects.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

9.4.1 Series Entity Modules

9.4.1.1 Encapsulated Document Series Module

TABLE 9-3
ENCAPSULATED DOCUMENT SERIES MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Modality	(0008,0060)	1	Value = US
Series Instance UID	(0020,000E)	1	Uniquely generated by the equipment
Series Number	(0020,0011)	1	Internal number which is incremented for each new exam within a study.
Referenced Performed Procedure Step Sequence	(0008,1111)	2	Sent as empty.
> 'Referenced SOP Class / Instance UIDs'			

9.4.2 Document Entity Modules

9.4.2.1 Encapsulated Document General Module

TABLE 9-4
ENCAPSULATED DOCUMENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Instance Number	(0020,0013)	1	Internal number which is incremented for each new Encapsulated PDF document within a study.
Content Date	(0008,0023)	1	Date of creation, based upon user action that invoked the use of the object
Content Time	(0008,0033)	1	Time of creation, based upon user action that invoked the use of the object.
Acquisition DateTime	(0008,002A)	2	Set to empty
Burned In Annotation	(0028,0301)	1	Set to "YES".
Document Title	(0042,0010)	2	The title of the document.
MIME Type of Encapsulated Document	(0042,0012)	1	Set to empty.
Encapsulated Document	(0042,0011)	1	Encapsulated Document stream, containing a document.

9.5 STANDARD EXTENDED AND PRIVATE DATA ATTRIBUTES

The product will send measurements raw data information in private data elements designated by the Private Creator element:

TABLE 9-5

Element Name	Tag	VR	VM	Description
Private Creator	6007,00xx	LO	1	GE Report Data

This means that all private tags starting with 6007,xx will belong to the GE Report Data.

10. KEY OBJECT SELECTION DOCUMENT INFORMATION OBJECT IMPLEMENTATION

10.1 INTRODUCTION

This section specifies the use of the DICOM key Object Selection Document IOD to represent results produced by this implementation. Corresponding attributes are conveyed using the module construct. Use of Key Object Selection is limited to identification of images rejected for quality reasons.

Note: This use is in accordance with Image Object Change Management Profile specified by IHE Radiology."

10.2 LOGIQ SCANNER MAPPING OF DICOM ENTITIES

The LOGIQ Scanner maps DICOM Information Entities to local Information Entities in the product's database and user interface.

TABLE 10-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM IE	LOGIQ Scanner Entity
Patient	Patient
Study	Exam
Series	Series
Document	Document

10.3 IOD MODULE TABLE

The Key Object Selection Document Information Object Definitions comprise the modules of the following tables. The elements not listed are not used by the application.

Standard, Standard Extended and Private templates are further described in Section 10.5.

TABLE 10-2
KEY OBJECT SELECTION DOCUMENT IOD MODULES

Entity Name	Module Name	Usage	Reference
Patient	Patient	Used	4.4.1.1
	Specimen Identification	Not used	N/A
	Clinical Trial Subject	Not used	N/A
Study	General Study	Used	4.4.2.1

Entity Name	Module Name	Usage	Reference
	Patient Study	Used	4.4.2.2
	Clinical Trial Study	Not used	N/A
Series	Key Object Document Series	Used	10.4.1.1
	Clinical Trial Series	Not used	N/A
Equipment	General Equipment	Used	4.4.3.1
Document	Key Object Document	Used	10.4.2.1
	SR Document Content	Used	8.4.2.2
	SOP Common	Used	4.4.5.8

10.4 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Part 3 (Information Object Definitions) for a description of each of the entities, modules, and attributes contained within the SR Information Objects.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained from when generating the instance. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions). Also note that Attributes not present in tables are not supported.

10.4.1 Series Entity Modules

10.4.1.1 Key Object Document Series Module

TABLE 10-3
KEY OBJECT DOCUMENT SERIES MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Modality	(0008,0060)	1	Value = KO
Series Instance UID	(0020,000E)	1	Uniquely generated by the equipment
Series Number	(0020,0011)	1	Internal number which is incremented for each new exam within a study.
Series Date	(0008,0021)	3	Set as date of exam (series) created
Series Time	(0008,0031)	3	Set as time of exam (series) created

Attribute Name	Tag	Type	Use
Referenced Performed Procedure Step Sequence	(0008,1111)	2	Sent as empty.
> 'Referenced SOP Class / Instance UIDs'			

10.4.2 Document Entity Modules

10.4.2.1 Key Object Document Module

TABLE 10-4
KEY OBJECT DOCUMENT MODULE ATTRIBUTES

Attribute Name	Tag	Type	Use
Instance Number	(0020,0013)	1	Internal number which is incremented for each new Key Object Selection document within a study.
Content Date	(0008,0023)	1	Date of creation, based upon user action that invoked the use of the object
Content Time	(0008,0033)	1	Time of creation, based upon user action that invoked the use of the object.
Referenced Request Sequence	(0040,A370)	1C	Filled in if the exam is based on a Worklist entry
>Study Instance UID	(0020,000D)	1	Taken from Study Instance UID in General Study Module
>Referenced Study Sequence	(0008,1110)	2	Sent as empty
>>Include 'SOP Instance Reference Macro'			
>Accession Number	(0008,0050)	2	Taken from Accession Number in General Study Module
>Placer Order Number/Imaging Service Request	(0040,2016)	2	Sent as empty
>Filler Order Number/Imaging Service Request	(0040,2017)	2	Sent as empty
>Requested Procedure ID	(0040,1001)	2	Taken from worklist if it is there
>Requested Procedure Description	(0032,1060)	2	Taken from worklist if it is there
>Requested Procedure Code Sequence	(0032,1064)	2	Sent as empty.
>>Include 'Code Sequence Macro'			

Attribute Name	Tag	Type	Use
Current Requested Procedure Evidence Sequence	(0040,A375)	1	List of all composite SOP Instances of US Single, US MF, Enhanced US Volume or Secondary Capture referenced in the Content Sequence (0040,A730).
>Include 'Hierarchical SOP Instance Reference Macro'			
Author Observer Sequence	(0040,A078)	3	Filled with observer information.

10.5 STANDARD, STANDARD EXTENDED AND PRIVATE TEMPLATES

The Product supports the Standard Extended and Private Templates defined in the following sections.

10.5.1 Standard Templates

The Product supports the following standard templates for SOP Instances created by this product.

10.5.1.1 Template ID 2010 Key Object Selection

Please refer following section.

Key Object Selection Document Template

10.5.2 Standard Extended Templates

Not applicable

10.5.3 Private Templates

Not applicable.

11. MODALITY WORKLIST INFORMATION MODEL DEFINITION

11.1 INTRODUCTION

This section specifies the use of the DICOM Modality Worklist Information Model used to organize data and against which a Modality Worklist Query will be performed. The contents of this section are:

- 11.2- Information Model Description
- 11.3- Information Model Entity-Relationship Model
- 11.4- Information Model Module Table
- 11.5- Information Model Keys

11.2 MODALITY WORKLIST INFORMATION MODEL DESCRIPTION


This section defines the implementation of Modality Worklist Information Model.

11.3 MODALITY WORKLIST INFORMATION MODEL ENTITY-RELATIONSHIP MODEL

The Entity-Relationship diagram for the Modality Worklist Information Model schema is shown in Illustration 10.3-1. It represents the information that composes a Worklist Item. In this figure, the following diagrammatic convention is established to represent the information organization:

- Each entity is represented by a rectangular box.
- Each relationship is represented by a diamond shaped box.
- The fact that a relationship exists between two entities is depicted by lines connecting the corresponding entity boxes to the relationship boxes.

ILLUSTRATION 10.3-1
MODALITY WORKLIST INFORMATION MODEL E/R DIAGRAM

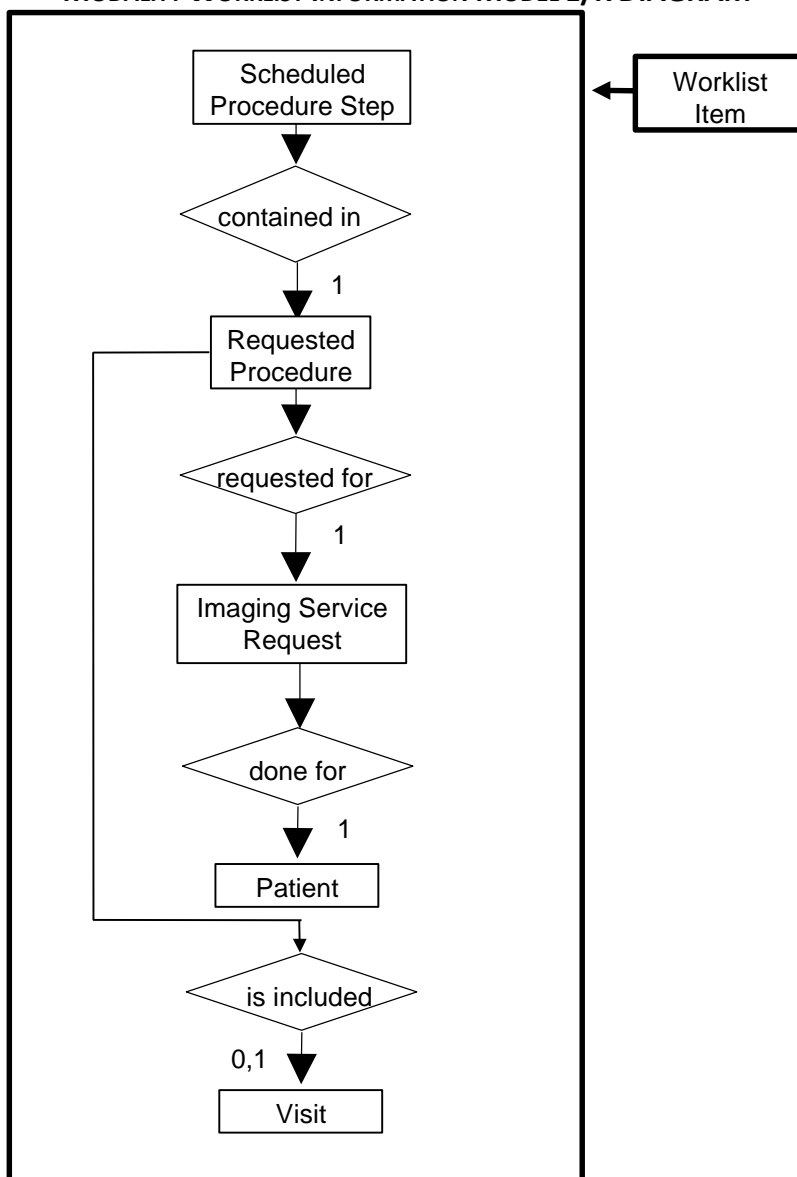

11.3.1 Entity Descriptions

Please refer to DICOM Standard PS 3.3. (Information Object Definitions) and PS 3.4 (Service Class Specifications) for a description of each of the Entities contained within the Modality Worklist Information Model.

11.3.1.1 Scheduled Procedure Step

Schedule Procedure Step is implemented in a basic form to allow for the user to retrieve a subset of attributes.

11.3.1.2 Requested Procedure Entity Description

Requested Procedure Step is implemented in a basic form to allow for the user to retrieve a subset of attributes.

11.3.1.3 Imaging Service Request Entity Description

Image Service is implemented in a basic form to allow for the user to retrieve a subset of attributes.

11.3.1.4 Visit Entity Description

Visit Entity is implemented in a basic form to allow for the user to retrieve a subset of attributes.

11.3.1.5 Patient Entity Description

Patient Entity Description is implemented in a basic form to allow for the user to retrieve a subset of attributes.

11.3.2 LOGIQ Scanner Mapping of DICOM entities

TABLE 11-1 MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM	LOGIQ Scanner Entity
Scheduled Procedure Step	Worklist entry
Requested Procedure	Exam
Imaging Service Request	Exam
Visit	Not Applicable
Patient	Patient

11.4 INFORMATION MODEL MODULE TABLE

Within an entity of the DICOM Modality Worklist Information Model, attributes are grouped into related set of attributes. A set of related attributes is termed a module. A module facilitates the understanding of the semantics concerning the attributes and how the attributes are related with each other. A module grouping does not infer any encoding of information into datasets.

Table 11-2 identifies the defined modules within the entities that comprise the DICOM Modality Worklist Information Model. Modules are identified by Module Name.

See DICOM PS 3.3 and PS 3.4 for a complete definition of the entities, modules, and attributes.

TABLE 11-2 MODALITY WORKLIST INFORMATION MODEL MODULES

Entity Name	Module Name	Reference
Scheduled Procedure Step	SOP Common	11.5.2.1
	Scheduled Procedure Step	11.5.2.2
Requested Procedure	Requested Procedure	11.5.3.1
Imaging Service Request	Imaging Service Request	11.5.4.1
Visit	Visit Identification	11.5.5.1
	Visit Status	11.5.5.2
	Visit Relationship	11.5.5.3
	Visit Admission	Not Used
Patient	Patient Relationship	Not Used
	Patient Identification	11.5.6.1
	Patient Demographic	11.5.6.2
	Patient Medical	11.5.6.3

11.5 INFORMATION MODEL KEYS

Please refer to DICOM Standard PS 3.3. (Information Object Definitions) and PS 3.4 (Service Class Specifications) for a description of each of the Entities contained within the Modality Worklist Information Model.

The following Module descriptions are included to specify what data elements are supported and what type of matching can be applied. It should be noted that they are the same ones as defined in the DICOM Standard PS 3.4 (Service Class Specifications).

The term Instance is used for Images and Results in examinations that are based on Worklist entries. Please note that tags that are not defined for SR documents will not be mapped (see Section: 8).

11.5.1 Supported Matching

Following are the types of matching that can be requested by the implementation:

- Single Value Matching.
- Wild Card Matching.
- Range of date.

Fields with “Filtering is supported” in the Matching column can be controlled from the Search screen. This means that the user can filter the downloaded C-FIND result, to view a limited set of the result.

All non-required matching fields can be configured in Config screen to be either enabled, enabled with a constant value or disabled. The constant value will be used as entered by user.

Wild Card Matching is only used for Patient's Name (0010,0010).

11.5.2 Scheduled Procedure Step Entity

11.5.2.1 SOP Common Module

TABLE 11-3 SOP COMMON MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPP S	Matching
Specific Character Set	(0008,0005)	O	1C	Yes/Yes	Attribute is supported if the query contains matching keys in other than the default character repertoire. ISO-IR 100 is supported in responses.

11.5.2.2 Scheduled Procedure Step Module

TABLE 11-4 SCHEDULED PROCEDURE STEP MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MP PS	Matching
Scheduled Procedure Step Sequence	(0040,0100)	R	1	Yes/Yes	Matching is supported.
>Scheduled Station AE Title	(0040,0001)	R	1	No/No	Matching is supported. Filtering is supported.
>Scheduled Procedure Step Start Date	(0040,0002)	R	1	No/No	Matching is supported. Filtering is supported.
>Scheduled Procedure Step Start Time	(0040,0003)	R	1	No/No	Matching is supported. Filtering is supported.
>Modality	(0008,0060)	R	1	Yes/Yes (but always "US")	Matching is supported. Filtering is supported.
>Scheduled Performing Physician's Name	(0040,0006)	R	2	Yes/Yes (to Performing Physician's Name)	Matching is supported. Filtering is supported.

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MP PS	Matching
>Scheduled Procedure Step Description	(0040,0007)	O	1C	Yes/Yes (to Series Description)	Matching is supported. Filtering is supported.
>Scheduled Station Name	(0040,0010)	O	2	No/No	Matching is supported. Filtering is supported.
>Scheduled Procedure Step Location	(0040,0011)	O	2	No/No	Matching is supported.
>Scheduled Procedure Step ID	(0040,0009)	O	1	Yes/Yes	Matching is supported. Filtering is supported.
>Scheduled Protocol Code Sequence	(0040,0008)	O	1C	Yes/Yes	Matching is supported. Filtering is supported.

11.5.3 Requested Procedure Entity

11.5.3.1 Requested Procedure Module

TABLE 11-5 REQUESTED PROCEDURE MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPPS	Matching
Requested Procedure ID	(0040,1001)	O	1	Yes/Yes (to Requested Procedure ID and Study ID)	Matching is supported. Filtering is supported.
Requested Procedure Description	(0032,1060)	O	1C	Yes/Yes (to Study Description)	Matching is supported. Filtering is supported.
Requested Procedure Code Sequence	(0032,1064)	O	1C	Yes/Yes (to Procedure Code Sequence)	Matching is supported. Filtering is supported.
Requested Procedure Comments	(0040,1400)	O	3	No/No	Matching is supported. Filtering is supported.
Study Instance UID	(0020,000D)	O	1	Yes/Yes	Matching is supported. Filtering is supported.

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPPS	Matching
Referenced Study Sequence	(0008,1110)	O	2	Yes/Yes	Matching is supported. Filtering is supported.
>Referenced SOP Class UID	(0008,1150)	O	1C	Yes/Yes	Matching is supported.
>Referenced SOP Instance UID	(0008,1155)	O	1C	Yes/Yes	Matching is supported.
Names of Intended Recipients of Results	(0040,1010)	O	3	No/No (to Physician(s) of Record)	Matching is supported. Filtering is supported.

11.5.4 Imaging Service Request Entity

11.5.4.1 Imaging Service Request Module

TABLE 11-6 IMAGING SERVICE REQUEST MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MP PS	Matching
Accession Number	(0008,0050)	O	2	Yes/Yes	Matching is supported. Filtering is supported.
Referring Physician's Name	(0008,0090)	O	2	Yes/No	Matching is supported. Filtering is supported.
Imaging Service Request Comments	(0040,2400)	O	3	No/No	Matching is supported. Filtering is supported.
Requesting Physician	(0032,1032)	O	2	No/No	Matching is supported. Filtering is supported.
Requesting Service	(0032,1033)	O	3	No/No	Matching is supported. Filtering is supported.

11.5.5 Visit Entity

11.5.5.1 Visit Identification

TABLE 11-7 VISIT IDENTIFICATION MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPP S	Matching
Admission ID	(0038,0010)	O	2	No/Yes	Matching is supported. Filtering is supported.

11.5.5.2 Visit Status

TABLE 11-8 VISIT STATUS MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPP S	Matching
Current Patient Location	(0038,0300)	O	2	No/No	Matching is supported. Filtering is supported.

11.5.5.3 Visit Relationship

TABLE 11-9 VISIT RELATIONSHIP MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MP PS	Matching
Referenced Patient Sequence	(0008,1120)	O	2	Yes/Yes	Matching is supported.
>Referenced SOP Class UID	(0008,1150)	O	2	Yes/Yes	Matching is supported.
>Referenced SOP Instance UID	(0008,1155)	O	2	Yes/Yes	Matching is supported.

11.5.6 Patient Entity

11.5.6.1 Patient Identification

TABLE 11-10 PATIENT IDENTIFICATION MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/M PPS	Matching
Patient's Name	(0010,0010)	R	1	Yes/Yes	<p>Matching is supported. Filtering is supported.</p> <p>Matching is supported as follows: either no Patient's Name is supplied (universal matching), the patient's last (family) name or first name specified on the worklist screen is supplied, for matching; this is user selectable. Wild card matching is implemented for leading and trailing characters.</p> <p>Patient Name is displayed on the "Worklist Schedule" screen.</p> <p>Application does support matching for Patient Last name and First name component groups. The caret characters (^) will be automatically inserted between the supported component groups.</p>
Patient ID	(0010,0020)	R	1	Yes/Yes	Matching is supported. Filtering is supported.
Issuer Of Patient ID	(0010,0021)	O	3	Yes/Yes	Matching is supported. Filtering is supported.
Other Patient IDs Sequence	(0010,1002)	O	3	No/No	Not used.

11.5.6.2 Patient Demographic

TABLE 11-11 PATIENT DEMOGRAPHIC MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/M PPS	Matching
Patients Birth Date	(0010,0030)	O	2	Yes/Yes	Matching is supported. Filtering is supported.
Patients Birth Time	(0010,0032)	O	3	Yes/No	Matching is supported. Filtering is supported.
Patient's Sex	(0010,0040)	O	2	Yes/Yes	Matching is supported. Filtering is supported.
Patient's Size	(0010,1020)	O	3	Yes/No	Matching is supported. Filtering is supported.
Patient's Weight	(0010,1030)	O	2	Yes/No	Matching is supported. Filtering is supported.
Patient's Address	(0010,1040)	O	3	No/No	Matching is supported.
Ethnic Group	(0010,2160)	O	3	Yes/No	Matching is supported. Filtering is supported.
Patient Comments	(0010,4000)	O	3	Yes/No	Matching is supported. Filtering is supported.

11.5.6.3 Patient Medical

TABLE 11-12 PATIENT MEDICAL MODULE ATTRIBUTES

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/M PPS	Matching
Additional Patient History	(0010,21B0)	O	3	Yes/No	Matching is supported. Filtering is supported.
Contrast Allergies	(0010,2110)	O	2	No/No	Matching is supported. Filtering is supported.

Attribute Name	Tag	Expected Matching Key Type	Expected Returned Key Type	Mapped into Instance/MPPS	Matching
Medical Alerts	(0010,2000)	O	2	No/No	Matching is supported. Filtering is supported.
Pregnancy Status	(0010,21C0)	O	2	No/No	Matching is supported. Filtering is supported.

12. MODALITY PERFORMED PROCEDURE STEP IMPLEMENTATION

12.1 INTRODUCTION

This section specifies the use of the DICOM Modality Performed Procedure Step information to be communicated to the Hospital/Radiology information system.

This feature works in conjunction with DICOM Modality Worklist feature, if installed. However, the conformance of this feature is independent of Modality Worklist feature. For information on conformance of Modality Worklist feature to DICOM standard please refer to the appropriate section in this document.

12.1.1 RELATIONSHIP BETWEEN SCHEDULED AND PERFORMED PROCEDURE STEPS

The scanner supports a one-to-one relationship between Scheduled Procedure Step and PPS and a zero-to-one relationship (aka Unscheduled Case or Acquisition without MWL Data).

12.2 MODALITY PERFORMED PROCEDURE STEP MODULE TABLE

See DICOM PS 3.3 and PS 3.4 for a complete definition of the entities, modules, and attributes.

TABLE 12-1 MODALITY PERFORMED PROCEDURE STEP MODULES

Module Name	Reference
SOP Common	12.4
Performed Procedure Step Relationship	0
Performed Procedure Step Information	12.6
Image Acquisition Results	12.7
Radiation Dose	Not Used
Billing and Material Management Codes	Not Used

12.3 MODALITY PERFORMED PROCEDURE STEP MODULE DEFINITIONS

Please refer to DICOM Standard PS 3.3. (Information Object Definitions) for a description of each of the attributes contained within the Modality Performed Procedure Step Information Object Definition.

12.4 SOP COMMON MODULE

TABLE 12-2 SOP COMMON MODULE ATTRIBUTES

Attribute Name	Tag	Type for SCU N-CREATE	Type for SCU N-SET	Use
Specific Character Set	(0008,0005)	1C	1C	Not used.

12.5 PERFORMED PROCEDURE STEP RELATIONSHIP MODULE

TABLE 12-3 PERFORMED PROCEDURE STEP RELATIONSHIP MODULE ATTRIBUTES

Attribute Name	Tag	Type for SCU - N-CREATE	
		Acquisition with MWL Entry	Acquisition without MWL Entry
Scheduled Step Attributes Sequence	(0040,0270)	1, One item sent	1, One item sent
>Study Instance UID	(0020,000D)	1, Copied from SPS	1, Created by scanner
>Referenced Study Sequence	(0008,1110)	2, Copied from SPS. Single item sent.	2, Empty
>>Referenced SOP Class UID	(0008,1150)	1, Copied from SPS.	1, Not sent
>>Referenced SOP Instance UID	(0008,1155)	1, Copied from SPS.	1, Not sent
>Accession Number	(0008,0050)	2, Copied from SPS.	2, Empty
>Placer Order Number/Imaging Service Request	(0040,2016)	3, Not sent.	3, Not sent.
>Filler Order Number/Imaging Service Request	(0040,2017)	3, Not sent.	3, Not sent.
>Requested Procedure ID	(0040,1001)	2, Copied from SPS.	2, Empty
>Requested Procedure Code Sequence	(0032,1064)	3, Not sent.	3, Not sent.
>>Code Value	(0008,0100)	1, Not sent.	1, Not sent.
>>Coding Scheme Designator	(0008,0102)	1, Not sent.	1, Not sent.
>>Code Meaning	(0008,0104)	1, Not sent.	1, Not sent.
>Requested Procedure Description	(0032,1060)	2, Empty	2, Empty
>Scheduled Procedure Step ID	(0040,0009)	2, Copied from SPS.	2, Sent empty.
>Scheduled Procedure Step Description	(0040,0007)	2, Copied from SPS.	2, Sent Empty
>Scheduled Protocol Code Sequence	(0040,0008)	2, Copied from SPS. One item sent.	2, Sent Empty.
>>Code Value	(0008,0100)	1, Copied from SPS.	1, Not sent
>>Coding Scheme Designator	(0008,0102)	1, Copied from SPS.	1, Not sent
>>Coding Scheme Version	(0008,0103)	3, Copied from SPS.	3, Not sent
>>Code Meaning	(0008,0104)	3, Copied from SPS.	3, Not sent
Patient's Name	(0010,0010)	2, Copied from worklist.	2 Created by scanner.

Attribute Name	Tag	Type for SCU - N-CREATE	
		Acquisition with MWL Entry	Acquisition without MWL Entry
Patient ID	(0010,0020)	2, Copied from worklist.	2, Created by scanner.
Issuer of Patient ID	(0010,0021)	3, Copied from worklist.	3, Not sent
Patient's Birth Date	(0010,0030)	2, Copied from worklist.	2, Created by scanner.
Patient's Sex	(0010,0040)	2, Copied from worklist.	2, Created by scanner.
Referenced Patient Sequence	(0008,1120)	2, Copied from worklist.	2, Empty
>Referenced SOP Class UID	(0008,1150)	1, Copied from worklist.	1, Not sent
>Referenced SOP Instance UID	(0008,1155)	1, Copied from worklist.	1, Not sent
Admission ID	(0038,0010)	3, Not sent	3, Not sent
Issuer of Admission ID	(0038,0011)	3, Not sent	3, Not sent
Service Episode ID	(0038,0060)	3, <Not sent	3, Not sent
Issuer of Service Episode ID	(0038,0061)	3, Not sent	3, Not sent
Service Episode Description	(0038,0062)	3, Not sent	3, Not sent

12.6 PERFORMED PROCEDURE STEP INFORMATION MODULE

TABLE 12-4 PERFORMED PROCEDURE STEP INFORMATION MODULE ATTRIBUTES

Attribute Name	Tag	Type for SCU N-CREATE	Type for SCU N-SET	Use
Performed Procedure Step ID	(0040,0253)	1	-	Supported.
Performed Station AE Title	(0040,0241)	1	-	Supported.
Performed Station Name	(0040,0242)	2	-	Supported.
Performed Location	(0040,0243)	2	-	Supported.
Performed Procedure Step Start Date	(0040,0244)	1	-	Supported.
Performed Procedure Step Start Time	(0040,0245)	1	-	Supported.
Performed Procedure Step Status	(0040,0252)	1	3	Supported.
Performed Procedure Step Description	(0040,0254)	2	3	Supported.
Performed Procedure Type Description	(0040,0255)	2	3	Supported.
Procedure Code Sequence	(0008,1032)	2	3	Supported.
>Code Value	(0008,0100)	1	1	Supported.
>Coding Scheme Designator	(0008,0102)	1	1	Supported.
>Coding Scheme Version	(0008,0103)	3	3	Supported.
>Code Meaning	(0008,0104)	3	3	Supported.
Performed Procedure Step End Date	(0040,0250)	2	3	Supported.
Performed Procedure Step End Time	(0040,0251)	2	3	Supported.
Comments on the Performed Procedure Step	(0040,0280)	3	3	Not sent.
Performed Procedure Step Discontinuation Reason Code Sequence	(0040,0281)	3	3	Not sent.
>Code Value	(0008,0100)	1	1	Not sent.
>Coding Scheme Designator	(0008,0102)	1	1	Not sent.
>Coding Scheme Version	(0008,0103)	3	3	Not sent.
>Code Meaning	(0008,0104)	3	3	Not sent.

12.7 IMAGE ACQUISITION RESULTS MODULE

TABLE 12-5 IMAGE ACQUISITION RESULTS MODULE ATTRIBUTES

Attribute Name	Tag	Type for SCU N-CREATE	Type for SCU N-SET	Use
Modality	(0008,0060)	1	-	Supported.
Study ID	(0020,0010)	2	-	Supported.
Performed Protocol Code Sequence	(0040,0260)	2	3	Supported.
>Code Value	(0008,0100)	1	1	Supported.
>Coding Scheme Designator	(0008,0102)	1	1	Supported.
>Coding Scheme Version	(0008,0103)	3	3	Supported.
>Code Meaning	(0008,0104)	3	3	Supported.
Performed Series Sequence	(0040,0340)	2	3	Supported.
>Performing Physician's Name	(0008,1050)	2	2	Sent empty.
>Protocol Name	(0018,1030)	1	1	Supported.
>Operator's Name	(0008,1070)	2	2	Supported.
>Series Instance UID	(0020,000E)	1	1	Supported.
>Series Description	(0008,103E)	2	2	Sent empty.
>Retrieve AE Title	(0008,0054)	2	2	Sent empty.
> Archive Requested	(0040,A494)	3	3	Not sent.
>Referenced Image Sequence	(0008,1140)	2	2	Supported.
>>Referenced SOP Class UID	(0008,1150)	1	1	Supported.
>>Referenced SOP Instance UID	(0008,1155)	1	1	Supported.
>Referenced Non-Image Composite SOP Instance Sequence	(0040,0220)	2	2	Sent empty.
>>Referenced SOP Class UID	(0008,1150)	1	1	Not sent.
>>Referenced SOP Instance UID	(0008,1155)	1	1	Not sent.

13. STORAGE COMMITMENT PUSH MODEL IMPLEMENTATION

13.1 INTRODUCTION

This section of the DICOM Conformance Statement specifies the Storage Commitment Push Model SOP Class, the optional attributes and service elements supported, the valid range of values for mandatory and optional attributes, and the status code behavior.

13.2 STORAGE COMMITMENT PUSH MODEL SOP CLASS DEFINITION

13.2.1 IOD Description

13.2.1.1 STORAGE COMMITMENT MODULE

TABLE 13-1 STORAGE COMMITMENT MODULE

Attribute Name	Tag	Attribute Description
Transaction UID	(0008,1195)	Uniquely generated by the equipment
Retrieve AE Title	(0008,0054)	Not used
Storage Media File-Set ID	(0088,0130)	Not used
Storage Media File-Set UID	(0088,0140)	Not used
Referenced SOP Sequence	(0008,1199)	Supported
>Referenced SOP Class UID	(0008,1150)	Supported
>Referenced SOP Instance UID	(0008,1155)	Supported
>Retrieve AE Title	(0008,0054)	Not used
>Storage Media File-Set ID	(0088,0130)	Not used
>Storage Media File-Set UID	(0088,0140)	Not used
Failed SOP Sequence	(0008,1198)	Supported
>Referenced SOP Class UID	(0008,1150)	Supported

Attribute Name	Tag	Attribute Description
>Referenced SOP Instance UID	(0008,1155)	Supported
>Failure Reason	(0008,1197)	Supported

13.2.2 DIMSE Service Group

TABLE 13-2

DIMSE Service Element	Usage SCU/SCP
N-EVENT-REPORT	M/M
N-ACTION	M/M

13.2.3 Operations

13.2.3.1 Action Information

TABLE 13-3 STORAGE COMMITMENT REQUEST - ACTION INFORMATION

Action Type Name	Action Type ID	Attribute	Tag	Requirement Type SCU/SCP
Request Storage Commitment	1	Transaction UID	(0008,1195)	1/1
		Storage Media File-Set ID	(0088,0130)	Not used
		Storage Media File-Set UID	(0088,0140)	Not used
		Referenced SOP Sequence	(0008,1199)	1/1
		>Referenced SOP Class UID	(0008,1150)	1/1
		>Referenced SOP Instance UID	(0008,1155)	1/1
		>Storage Media File-Set ID	(0088,0130)	Not used
		>Storage Media File-Set UID	(0088,0140)	Not used

13.2.3.2 Service Class User Behavior

LOGIQ Scanner sends the N-ACTION primitive (Storage Commitment Request) after successful exam save to a DICOM Storage SCP.

LOGIQ Scanner may request storage commitment for all generated SOP Class UIDs:

TABLE 13-4

SOP Class Name	SOP Class UID
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1
Ultrasound Multi-frame Image Storage (Retired)	1.2.840.10008.5.1.4.1.1.3
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1
Ultrasound Image Storage (Retired)	1.2.840.10008.5.1.4.1.1.6
Enhanced US Volume Storage	1.2.840.10008.5.1.4.1.1.6.2
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7
Comprehensive Structured Report	1.2.840.10008.5.1.4.1.1.88.33
Key Object Selection Document	1.2.840.10008.5.1.4.1.1.88.59

The association for the N-ACTION is disconnected after processing the response. Thus, the N-EVENT-REPORT must be sent on a separate association.

The Referenced Study Component Sequence Attribute is not supported.

The Transaction UID is valid for two days. If no answer is received, the request will be removed without warning the user.

The optional Storage Media File-Set ID & UID Attributes in the N-ACTION are not supported.

13.2.4 Notifications

LOGIQ Scanner will only listen for an N-EVENT-REPORT from the SCP in a new association on the listen port for Verification and Storage Commitment.

Role Negotiation is supported and expected in the new association requested.

13.2.4.1 Event Information

TABLE 13-5
STORAGE COMMITMENT RESULT - EVENT INFORMATION

Event Type Name	Event Type ID	Attribute	Tag	Requirement Type SCU/SCP
Storage Commitment Request Successful	1	Transaction UID	(0008,1195)	-/1
		Retrieve AE Title	(0008,0054)	Not used
		Storage Media File-Set ID	(0088,0130)	Not used
		Storage Media File-Set UID	(0088,0140)	Not used
		Referenced SOP Sequence	(0008,1199)	-/1
		>Referenced SOP Class UID	(0008,1150)	-/1
		>Referenced SOP Instance UID	(0008,1155)	-/1
		>Retrieve AE Title	(0008,0054)	Not used
		>Storage Media File-Set ID	(0088,0130)	Not used
		>Storage Media File-Set UID	(0088,0140)	Not used
Storage Commitment Request Complete - Failures Exist	2	Transaction UID	(0008,1195)	-/1
		Retrieve AE Title	(0008,0054)	Not used
		Storage Media File-Set ID	(0088,0130)	Not used
		Storage Media File-Set UID	(0088,0140)	Not used
		Referenced SOP Sequence	(0008,1199)	-/1C
		>Referenced SOP Class UID	(0008,1150)	-/1
		>Referenced SOP Instance UID	(0008,1155)	-/1
		>Retrieve AE Title	(0008,0054)	Not used
		>Storage Media File-Set ID	(0088,0130)	Not used
		>Storage Media File-Set UID	(0088,0140)	Not used
		Failed SOP Sequence	(0008,1198)	-/1

Event Type Name	Event Type ID	Attribute	Tag	Requirement Type SCU/SCP
		>Referenced SOP Class UID	(0008,1150)	-/1
		>Referenced SOP Instance UID	(0008,1155)	-/1
		>Failure Reason	(0008,1197)	Not used.

13.2.4.2 Service Class User Behavior

Upon receipt of N-EVENT-REPORT request with valid Transaction UID, the request will be removed without warning the user, otherwise the job will be left in the holding queue.

If no N-EVENT-REPORT request is received, the job will be removed without warning the user after two days.

14. BASIC DIRECTORY INFORMATION OBJECT IMPLEMENTATION

14.1 INTRODUCTION

This section specifies the use of the DICOM Basic Directory IOD to represent the information included in directories produced by this implementation. Corresponding attributes are conveyed using the module construct. The contents of this section are:

- 14.2 - IOD Implementation
- 14.3 - IOD Entity-Relationship Model
- 14.4- IOD Module Table
- 14.5 - IOD Module Definition

14.2 BASIC DIRECTORY IOD IMPLEMENTATION

This section defines the implementation of Basic Directory information object.

14.3 BASIC DIRECTORY ENTITY-RELATIONSHIP MODEL

The Entity-Relationship diagram for the Basic Directory interoperability schema is shown in

Illustration **14.3-1**. In this figure, the following diagrammatic convention is established to represent the information organization:


- each entity is represented by a rectangular box
- each relationship is represented by a diamond shaped box
- the fact that a relationship exists between two entities is depicted by lines connecting the corresponding entity boxes to the relationship boxes.

14.3.1 LOGIQ Scanner Mapping of DICOM entities

TABLE 14-1 MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM	LOGIQ Scanner
Patient	Patient
Study	Exam
Series	Exam
Image or SR Document or Encapsulated PDF	Image or Results

**Illustration 14.3-1
BASIC DIRECTORY ENTITY RELATIONSHIP DIAGRAM**

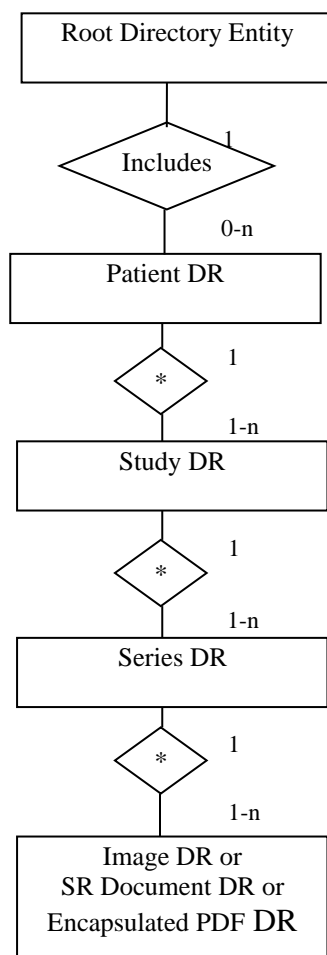

14.4 IOD MODULE TABLE

Within an entity of the Basic Directory IOD, attributes are grouped into related set of attributes. A set of related attributes is termed a module. A module facilitates the understanding of the semantics concerning the attributes and how the attributes are related with each other. A module grouping does not infer any encoding of information into datasets.

Table 14-2 identifies the defined modules within the entities, which comprise the Basic Directory IOD. Modules are identified by Module Name.

See DICOM Part 3 for a complete definition of the entities, modules, and attributes.

TABLE 14-2 BASIC DIRECTORY IOD MODULES

Entity Name	Module Name	Reference
File Set Identification	File Set Identification	14.5.1.1
Directory Information	Directory Information	14.5.2.1

The Directory Information Module is created when initializing the media. If it already exists, the existing information is not changed regarding patient, study, series or image/result data.

An existing Directory Information Module may have been obtained from application entities using removable media. These instances are external to this conformance claim and the origin of the SOP instances is outside the scope of this claim.

14.5 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Standard Part 3 (Information Object Definitions) for a description of each of the entities and modules contained within the Basic Directory Information Object.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported. Type 1 & Type 2 Attributes are also included for completeness and to define what values they may take and where these values are obtained. It should be noted that they are the same ones as defined in the DICOM Standard Part 3 (Information Object Definitions).

14.5.1 Common File Set Identification Modules

14.5.1.1 File Set identification Module

TABLE 14-3 FILE-SET IDENTIFICATION MODULE

Attribute Name	Tag	Type	Attribute Description
File-set ID	(0004,1130)	2	Set to empty value.
File-set Descriptor File ID	(0004,1141)	3	Not used
Specific Character Set of File-set Descriptor File	(0004,1142)	1C	Not used

14.5.2 Common Directory Information Modules

14.5.2.1 Directory Information Module

TABLE 14-4 DIRECTORY INFORMATION MODULE

Attribute Name	Tag	Type	Attribute Description
Offset of the First Directory Record of the Root Directory Entity	(0004,1200)	1	Is set
Offset of the Last Directory Record of the Root Directory Entity	(0004,1202)	1	Is set
File-set Consistency Flag	(0004,1212)	1	FSC/FSU/FSR: Has the value 0000H: no known inconsistencies, ignored when reading.
Directory Record Sequence	(0004,1220)	2	Is created by FSC or updated by FSU
>Offset of the Next Directory Record	(0004,1400)	1C	Is set
>Record In-use Flag	(0004,1410)	1C	FSC/FSU: Is set to FFFFH FSR: A value of 0000H: imply skipping this record
>Offset of Referenced Lower-Level Directory Entity	(0004,1420)	1C	Is set
>Directory Record Type	(0004,1430)	1C	The values support by FSC and FSU are PATIENT STUDY SERIES IMAGE SR DOCUMENT ENCAP DOC
>Private Record UID	(0004,1432)	1C	Not used
>Referenced File ID	(0004,1500)	1C	Contains the file path consisting of 5 elements: 1. "GEMS_IMG" (if IMAGE) 2. Month of exam 3. Day of exam 4. Patient initials and time of exam 5. Time stamp
>Referenced SOP Class UID in File	(0004,1510)	1C	Is set to the SOP class UID in File if Directory Record Type is IMAGE
>Referenced SOP Instance UID in File	(0004,1511)	1C	Is set to the SOP instance UID in File if Directory Record Type is IMAGE
>Referenced Transfer Syntax UID in File	(0004,1512)	1C	Is set to the Transfer Syntax UID in File if Directory Record Type is IMAGE
>Record Selection Keys			See 14.5.3.

14.5.3 Definition of Specific Directory Records

14.5.3.1 Patient Directory Record Definition

TABLE 14-5 PATIENT KEYS

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Name	(0010,0010)	2	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient ID	(0010,0020)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, a Patient Id is created by the equipment.
Patient's Birth Date	(0010,0030)	2	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Sex	(0010,0040)	2	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Referenced Patient Sequence	(0008,1120)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
>Referenced SOP Class UID	(0008,1150)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
>Referenced SOP Instance UID	(0008,1155)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Birth Time	(0010,0032)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Other Patient Ids	(0010,1000)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Other Patient Names	(0010,1001)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

Key	Tag	Type	Attribute Description
Ethnic Group	(0010,2160)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient Comments	(0010,4000)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

14.5.3.2 Study Directory Record Definition

TABLE 14-6 STUDY KEYS

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Study Instance UID	(0020,000D)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Study Date	(0008,0020)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, a Study Date is created by the equipment.
Study Time	(0008,0030)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, a Study Time is created by the equipment.
Referring Physician's Name	(0008,0090)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Study ID	(0020,0010)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, a Study Id is created by the equipment.
Accession Number	(0008,0050)	2	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Study Description	(0008,1030)	2	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

Key	Tag	Type	Attribute Description
Physician(s) of Record	(0008,1048)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Name of Physician(s) Reading Study	(0008,1060)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Referenced Study Sequence	(0008,1110)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
>Referenced SOP Class UID	(0008,1150)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
>Referenced SOP Instance UID	(0008,1155)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Admitting Diagnoses Description	(0008,1080)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Age	(0010,1010)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Size	(0010,1020)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Patient's Weight	(0010,1030)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Occupation	(0010,2180)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Additional Patient's History	(0010,21B0)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

14.5.3.3 Series Directory Record Definition

TABLE 14-7 SERIES KEYS

Key	Tag	Type	Attribute Description
-----	-----	------	-----------------------

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC or FSU as contained in the image or SR document message, if one of the tags contains extended characters
Modality	(0008,0060)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Series Instance UID	(0020,000E)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Series Number	(0020,0011)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, a Series Number is created by the equipment.
Icon Image Sequence	(0088,0200)	3	Not used.
Series Date	(0008,0021)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Series Time	(0008,0031)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Performing Physicians' Name	(0008,1050)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Protocol Name	(0018,1030)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Series Description	(0008,103E)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.

Key	Tag	Type	Attribute Description
Operator's Name	(0008,1070)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Request Attributes Sequence	(0040,0275)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
>Requested Procedure ID	(0040,1001)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
>Scheduled Procedure Step ID	(0040,0009)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
>Scheduled Procedure Step Description	(0040,0007)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is IMAGE.
Manufacturer	(0008,0070)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Institution Name	(0008,0080)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Station Name	(0008,1010)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Institutional Department Name	(0008,1040)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Manufacturer's Model Name	(0008,1090)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Software Versions	(0018,1020)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

Key	Tag	Type	Attribute Description
Related Series Sequence	(0008,1250)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is SRDOC or ENCAP DOC.
>Study Instance UID	(0020,000D)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is SRDOC or ENCAP DOC.
>Series Instance UID	(0020,000E)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7 if instance is SRDOC or ENCAP DOC.

14.5.3.4 Image Directory Record Definition

TABLE 14-8 IMAGE KEYS

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Instance Number	(0020,0013)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, an Instance Number is created by the equipment.
Icon Image Sequence	(0088,0200)	3	Not used
Content Date	(0008,0023)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Content Time	(0008,0033)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Image Type	(0008,0008)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Rows	(0028,0010)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Columns	(0028,0011)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

Number Of Frames	(0028,0008)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Photometric Interpretation	(0028,0004)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Contrast/Bolus Agent	(0018,0010)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Lossy Image Compression	(0028,2110)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Lossy Image Compression Ratio	(0028,2112)	3	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

14.5.3.5 SR Document Directory Record Definition

TABLE 14-9
SR DOCUMENT KEYS

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.
Instance Number	(0020,0013)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, an Instance Number is created by the equipment.
Completion Flag	(0040,A491)	1	Set to "PARTIAL".
Verification Flag	(0040,A493)	1	Set to "UNVERIFIED".
Content Date	(0008,0023)	1	Set to SR creation date.
Content Time	(0008,0033)	1	Set to SR creation time.

14.5.3.6 Encapsulated PDF Document Directory Record Definition

TABLE 14-10
ENCAPSULATED PDF DOCUMENT KEYS

Key	Tag	Type	Attribute Description
Specific Character Set	(0008,0005)	1C	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7.

Instance Number	(0020,0013)	1	Is filled in by FSC and FSU as in chapter 4, 5, 6 and 7. If empty, an Instance Number is created by the equipment.
Content Date	(0008,0023)	1	Set to Encapsulated document creation date.
Content Time	(0008,0033)	1	Set to Encapsulated document creation time.

14.5.3.7 Private Directory Record Definition

Not used.

15. PRINT MANAGEMENT IMPLEMENTATION

15.1 INTRODUCTION

This section of the DICOM Conformance Statement specifies the supported Print Management SOP and Meta SOP Classes, the optional attributes and service elements supported, the valid range of values for mandatory and optional attributes, and the status code behavior.

15.2 - Basic Print Management Meta SOP Classes

15.3 - Print Management SOP Class Definitions

15.4 - Print Management IODs

15.5 - IOD Module Definition

15.2 BASIC PRINT MANAGEMENT META SOP CLASSES

The Basic Print Management Meta SOP Classes correspond with the minimum functionality that an implementation of the Print Management Service Class shall support.

LOGIQ Scanner supports the Basic Grayscale Print Management Meta SOP Class and the Basic Color Print Management Meta SOP Class. These are defined in Table 15-1 and Table 15-2.

15.2.1 Basic Grayscale Print Management Meta SOP Class

The Basic Grayscale Print Management Meta SOP Class is defined by the following set of supported SOP Classes.

TABLE 15-1
BASIC GRAYSCALE PRINT MANAGEMENT META SOP CLASS

SOP Class Name	Usage SCU	Reference
Basic Film Session SOP Class	M	see 15.3.1
Basic Film Box SOP Class	M	see 15.3.2
Basic Grayscale Image Box SOP Class	M	see 15.3.3.1
Printer SOP Class	M	see 15.3.4
Basic Annotation Box SOP Class	U	See 15.3.5

15.2.2 Basic Color Print Management Meta SOP Class

The Basic Color Print Management Meta SOP Class is defined by the following set of supported SOP Classes

TABLE 15-2
BASIC COLOR PRINT MANAGEMENT META SOP CLASS

SOP Class Name	Usage	Reference
Basic Film Session SOP Class	M	see 15.3.1
Basic Film Box SOP Class	M	see 15.3.2
Basic Color Image Box SOP Class	M	see 15.3.3.2
Printer SOP Class	M	see 15.3.4
Basic Annotation Box SOP Class	U	See 15.3.5

15.3 PRINT MANAGEMENT SOP CLASS DEFINITIONS

15.3.1 Basic Film Session SOP Class

The Basic Film Session IOD describes the presentation parameters, which are common for all the films of a film session. The DIMSE services that are applicable to the IOD are shown in below table

TABLE 15-3
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
N-CREATE	M	see 15.3.1.1.1
N-SET	U	see 15.3.1.1.2
N-DELETE	U	see 15.3.1.1.3
N-ACTION	U	see 15.3.1.1.4

15.3.1.1 DIMSE Service Group

15.3.1.1.1 N-CREATE

The N-CREATE DIMSE Service is used by LOGIQ Scanner to request that the SCP (printer) create a Film Session SOP Instance. Table 15-15 defines the Basic Film Session Presentation Module attributes used in this request.

15.3.1.1.2 N-SET

Not used in this implementation.

15.3.1.1.3 N-DELETE

Not used in this implementation.

15.3.1.1.4 N-ACTION

Not used in this implementation

15.3.2 Basic Film Box SOP Class

The Basic Film Box IOD is an abstraction of the presentation of one film of the film session. The DIMSE services that are applicable to the IOD are shown in below table

TABLE 15-4
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
N-CREATE	M	see 15.3.2.1.1
N-ACTION	M	see 15.3.2.1.2
N-DELETE	U	see 15.3.2.1.3
N-SET	U	see 15.3.2.1.4

15.3.2.1 DIMSE Service Group

15.3.2.1.1 N-CREATE

The N-CREATE DIMSE Service is used by LOGIQ Scanner to request that the SCP create a Film Box SOP Instance. Table 15-17 defines the Basic Film Box Presentation Module attributes used in this request. The Table 15-18 defines the Basic Film Box Relationship module attributes.

15.3.2.1.2 N-ACTION

The N-ACTION DIMSE Service is used by LOGIQ Scanner to request the SCP (printer) to print the number of copies configured by the user to a film of the film session.

15.3.2.1.3 N-DELETE

The N-DELETE DIMSE Service is used by LOGIQ Scanner to request the SCP (printer) to delete the complete Film Box. The root Film Box Instance UID is sent to the SCP to accomplish this.

15.3.2.1.4 N-SET

Not used in this implementation.

15.3.3 Image Box SOP Class

15.3.3.1 Basic Grayscale Image Box SOP Class

The Basic Grayscale Image Box IOD is an abstraction of the presentation of an image and image related data in the image area of a film. The DIMSE services that are applicable to the IOD are shown in below table.

TABLE 15-5
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
-----------------------	-----------	-----------

N-SET	M	see 15.3.3.1.1
-------	---	----------------

15.3.3.1.1 DIMSE Service Group (N-SET)

The N-SET DIMSE Service is used by LOGIQ Scanner to update the Basic Grayscale Image Box SOP Instance.

Table 15-19 defines the Basic Image Box Pixel Presentation Module attributes used.

15.3.3.2 Basic Color Image Box SOP Class

The Basic Color Image Box IOD is an abstraction of the presentation of an image and image related data in the image area of a film. The DIMSE services that are applicable to the IOD are shown in below table

TABLE 15-6
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
N-SET	M	see 15.3.3.2.1

15.3.3.2.1 DIMSE Service Group (N-SET)

The N-SET DIMSE Service is used by LOGIQ Scanner to update the Basic Color Image Box SOP Instance. The Table 14-19B defines the Basic Image Box Pixel Presentation Module attributes used.

15.3.4 Printer SOP Class

The Printer IOD is an abstraction of the hard copy printer and is the basic Information Entity to monitor the status of the printer. The DIMSE services that are applicable to the IOD are shown in below table

15.3.4.1 DIMSE Service Group

TABLE 15-7
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
N-SET	M	see 15.3.3.2.1

15.3.4.1.1 N-EVENT_REPORT

LOGIQ Scanner confirms the N-EVENT-REPORT initiated by the SCP (printer).

15.3.4.1.2 N-GET

Used by LOGIQ Scanner to request the SCP to get a Printer SOP Instance.

Table 15-20 defines the Printer Module attributes.

15.3.5 Basic Annotation Box SOP Class

The Basic Annotation Box IOD is an abstraction to create an annotation box to print server. The DIMSE services that are applicable to the IOD are shown in below table.

15.3.5.1 DIMSE Service Group

TABLE 15-8
DIMSE SERVICE GROUP

DIMSE Service Element	Usage SCU	Reference
N-SET	M	see 15.3.5.1.1

15.3.5.1.1 DIMSE Service Group (N-SET)

The N-SET DIMSE Service is used by LOGIQ Scanner to update the Basic Annotation Box SOP Instance. Table 15-21 defines Basic Annotation Presentation Module Attributes used.

15.4 PRINT MANAGEMENT IODS

Within an entity of a DICOM Print Management, attributes are grouped into a related set of attributes. A set of related attributes is termed a module. A module facilitates the understanding of the semantics concerning the attributes and how the attributes are related with each other. A module grouping does not infer any encoding of information into datasets.

Table 15-9, Table 15-10, Table 15-11, Table 15-12 and Table 15-13 identify the defined modules within the entities which comprise the DICOM Print Management Service IODs. Modules are identified by Module Name.

See DICOM for a complete definition of the entities, modules and attributes.

15.4.1 Film Session IOD Module

TABLE 15-9 FILM SESSION IOD MODULES

Module Name	Reference
SOP Common Module	15.5.1.1
Basic Film Session Presentation Module	15.5.2.1
Basic Film Session Relationship Module	15.5.2.2

15.4.2 Basic Film Box IOD Module Table

TABLE 15-10 BASIC FILM BOX IOD MODULES

Module Name	Reference
SOP Common Module	15.5.1.1
Basic Film Box Presentation Module	15.5.2.3
Basic Film Box Relationship Module	15.5.2.4

15.4.3 Basic Image Box IOD Module Table

TABLE 15-11 BASIC IMAGE BOX IOD MODULES

Module Name	Reference
SOP Common Module	15.5.1.1
Image Box Pixel Presentation Module	15.5.2.5

15.4.4 Printer IOD Module Table

TABLE 15-12 PRINTER IOD MODULES

Module Name	Reference
SOP Common Module	15.5.1.1
Printer Module	15.5.2.6

15.4.5 Basic Annotation Box IOD Module Table

TABLE 15-13 BASIC ANNOTATION IOD MODULES

Module Name	Reference
SOP Common Module	15.5.1.1
Basic Annotation Presentation Module	15.5.2.7

15.5 INFORMATION MODULE DEFINITIONS

Please refer to DICOM Standard Part 3 (Information Object Definitions) for a description of each of the entities and modules that comprise the Print Management.

The following modules are included to convey Enumerated Values, Defined Terms, and Optional Attributes supported.

15.5.1 General Modules

15.5.1.1 SOP Common Module

This section defines the attributes that are required for proper functioning and identification of the associated SOP Instances. They do not specify any semantics about the Real-World Object represented by the IOD.

TABLE 15-14 SOP COMMON MODULE ATTRIBUTES

Attribute Name	Tag	Type	Attribute Description
SOP Class UID	(0008,0016)	1	Varies with Module Instance and DIMSE Service being used. 1.2.840.100011.5.1.1.1 (Film Session) 1.2.840.100011.5.1.1.2 (Film Box) 1.2.840.100011.5.1.1.4 (Image Box) 1.2.840.100011.5.1.1.15 (Basic Annotation Box) 1.2.840.100011.5.1.1.14 (Print Job)
SOP Instance UID	(0008,0018)	1	Provided by SCP (printer).
Specific Character Set	(0008,0005)	1C	Not used as expanded or replacement character sets not used.

15.5.2 Print Management Modules

For all user configurable tags with no default, no value will be sent if the tag is not configured.

15.5.2.1 Basic Film Session Presentation Module

This section defines the attributes that are common for all films of a film session. The attributes described in below table apply when the N-CREATE DIMSE service is used.

TABLE 15-15 BASIC FILM SESSION PRESENTATION MODULE ATTRIBUTES

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Number of Copies	(2000,0010)	U	Defined Terms used (user configurable): Default is 1. Max is 99.
Print Priority	(2000,0020)	U	Defined Terms used (user configurable): HIGH, MED, LOW. Default is HIGH.
Medium Type	(2000,0030)	U	Defined Terms used (user configurable): PAPER BLUE FILM CLEAR FILM Default is CLEAR FILM.
Film Destination	(2000,0040)	U	Defined Terms used (user configurable): MAGAZINE - default PROCESSOR
Film Session Label	(2000,0050)	U	User configurable. No default.
Memory Allocation	(2000,0060)	U	Not Used
Owner Id	(2100,0160)	U	Not Used

15.5.2.2 Basic Film Session Relationship Module

TABLE 15-16 BASIC FILM SESSION RELATIONSHIP MODULE ATTRIBUTES

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Referenced Film Box Sequence	(2000,0500)	U	Not used
>Referenced SOP Class UID	(0008,1150)	U	
>Referenced SOP Instance UID	(0008,1155)	U	

15.5.2.3 Basic Film Box Presentation Module

The attributes described in below table apply when the N-CREATE DIMSE service is used.

TABLE 15-17 BASIC FILM BOX PRESENTATION MODULE ATTRIBUTES

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Image Display Format	(2010,0010)	M	Enumerated values used (user configurable): STANDARD\X,Y, where X and Y can take values from 1 to 5. Default is STANDARD\1,1.
Annotation Display Format ID	(2010,0030)	U	Not used.
Film Orientation	(2010,0040)	U	Defined Terms used (user configurable): PORTRAIT - default LANDSCAPE
Film Size ID	(2000,0050)	U	Defined Terms used (user configurable): 8INX10IN - default 10INX12IN 10INX14IN 11INX14IN 14INX14IN 14INX17IN 24CMX24CM 24CMX30CM
Magnification Type	(2010,0060)	U	Defined Terms Used (user configurable): REPLICATE - default BILINEAR CUBIC NONE

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Smoothing Type	(2010,0080)	U	Free form text entry field (user configurable) and only sent if Magnification Type is CUBIC. No default
Border Density	(2010,0100)	U	Defined Terms Used (user configurable): BLACK WHITE Default is BLACK.
Empty Image Density	(2010,0110)	U	Defined Terms Used (user configurable): BLACK WHITE Default is WHITE.
Min Density	(2010,0120)	U	User configurable. Set to 0 as default.
Max Density	(2010,0130)	U	User configurable. Set to 300 as default.
Trim	(2010,0140)	U	Enumerated Values Used (user configurable): YES NO Default is NO.
Configuration Information	(2010,0150)	U	User configurable. No default.

15.5.2.4 Basic Film Box Relationship Module

This section defines the attributes that describe the common parameters, which apply for all images on a given sheet of film.

TABLE 15-18 BASIC FILM BOX RELATIONSHIP MODULE ATTRIBUTES

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Referenced Film Session Sequence	(2010,0500)	M	
>Referenced SOP Class UID	(0008,1150)	M	1.2.840.10008.5.1.1.1
>Referenced SOP Instance UID	(0008,1155)	M	Provided by SCP (printer)
Referenced Image Box Sequence	(2010,0510)	U	Used for the subsequent handling of Image Boxes
>Referenced SOP Class UID	(0008,1150)	U	In case of Basic Color ImageBox, SOP class UID 1.2.840.10008.5.1.1.4.1 is used. In case of Basic Grayscale Image Box, SOP class UID 1.2.840.10008.5.1.1.4 is used.
>Referenced SOP Instance UID	(0008,1155)	U	
Referenced Basic Annotation Sequence	(2010,0520)	U	Used. Provided by Printer SCP.
>Referenced SOP Class UID	(0008,1150)	U	Set with Basic Annotation Box SOP Class UID.
>Referenced SOP Instance UID	(0008,1155)	U	Set with Basic Annotation Box SOP Instance UID.

15.5.2.5 Image Box Pixel Presentation Module

The attributes described in below table apply when the DIMSE Service N-SET is used.

The first attributes in the table are used for both grayscale and color printing. The attributes within the sequences are used for each type of printing respectively.

TABLE 15-19 IMAGE BOX PIXEL PRESENTATION MODULE ATTRIBUTES (GRAY SCALE)

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Image Position	(2020,0010)	M	Based on the image display format.
Polarity	(2020,0020)	U	Defined term, NORMAL
Requested Image Size	(2020,0030)	U	Not sent
Basic Grayscale Image Sequence	(2020,0110)	M	
>Samples Per Pixel	(0028,0002)	M	Value = '1'
>Photometric Interpretation	(0028,0004)	M	Defined Term MONOCHROME2 used
>Rows	(0028,0010)	M	Value depends on scanning mode and configuration setup.
>Columns	(0028,0011)	M	Value depends on scanning mode and configuration setup.
>Pixel Aspect Ratio	(0028,0034)	MC	Not used
>Bits Allocated	(0028,0100)	M	Value always = 0008H
>Bits Stored	(0028,0101)	M	Value always = 0008H
>High Bit	(0028,0102)	M	Value always = 0007H
>Pixel Representation	(0028,0103)	M	Defined Value '0' - unsigned integer
>Pixel Data	(7FE0,0010)	M	

TABLE 15-19B IMAGE BOX PIXEL PRESENTATION MODULE ATTRIBUTES (COLOR)

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Image Position	(2020,0010)	M	Based on the image display format.
Polarity	(2020,0020)	U	Defined term, NORMAL
Requested Image Size	(2020,0030)	U	Not sent
Basic Color Image Sequence	(2020,0111)	M	
>Samples Per Pixel	(0028,0002)	M	Value = '3'
>Photometric Interpretation	(0028,0004)	M	Defined Term RGB used
>Rows	(0028,0010)	M	Value depends on scanning mode and configuration setup.
>Columns	(0028,0011)	M	Value depends on scanning mode and configuration setup.
>Pixel Aspect Ratio	(0028,0034)	MC	Not used
>Bits Allocated	(0028,0100)	M	Value always = 0008H
>Bits Stored	(0028,0101)	M	Value always = 0008H
>High Bit	(0028,0102)	M	Value always = 0007H
>Pixel Representation	(0028,0103)	M	Defined Value '0' - unsigned integer
>Pixel Data	(7FE0,0010)	M	
>Planar Configuration	(0028, 0006)	M	0001H, color-by-plane, when Basic Color Image Sequence is set

15.5.2.6 Printer Module

This section defines the attributes that are used to monitor the status of the printer. The attributes described in below table apply when the DIMSE Service N-GET is used.

TABLE 15-20 PRINTER MODULE ATTRIBUTES

Attribute Name	Tag	USAG E (SCU)	Attribute Description
Printer Status	(2110,0010)	U	Used to check the status of the printer before a print operation is started. If the printer status is FAILURE, the print operation is aborted, a message is displayed, and the print files reside in the print buffer.
Printer Status Info	(2110,0020)	U	If the "Printer Status" is "FAILURE" then this status information is displayed, and the print files resides in the print buffer.
Printer Name	(2110,0030)	U	Requested, but not used
Manufacturer	(0008,0070)	U	Requested, but not used
Manufacturer Model Name	(0008,1090)	U	Requested, but not used
Device Serial Number	(0018,1000)	U	Requested, but not used
Software Versions	(0018,1020)	U	Requested, but not used
Date Last Calibration	(0018,1200)	U	Requested, but not used
Last Calibration	(0018,1201)	U	Requested, but not used

15.5.2.7 Basic Annotation Presentation Module

This section defines the attributes that are used to set Basic Annotation Box. The attributes described in below table apply when the DIMSE Service N-SET is used.

TABLE 15-21 BASIC ANNOTATION PRESENTATION MODULE ATTRIBUTES

Attribute Name	Tag	USAGE (SCU)	Attribute Description
Annotation Position	(2030,0010)	U	Supported.
Text String	(2030,0020)	U	Text string containing Patient Id and Patient Name.

16. STUDY ROOT QUERY/RETRIEVE INFORMATION MODEL

16.1 INTRODUCTION

This section specifies the use of the DICOM Study Root Query/Retrieve Model used to organize data and against which a Query/Retrieve will be performed. The contents of this section are:

- 16.2 - Information Model Description
- 16.3 - Information Model Entity-Relationship Model
- 16.4 - Information Model Keys

16.2 STUDY ROOT INFORMATION MODEL DESCRIPTION

This section defines the implementation of Study Root Query/Retrieve Information Model.


16.3 STUDY ROOT INFORMATION MODEL ENTITY-RELATIONSHIP MODEL

The Entity-Relationship diagram for the Study Root Information Model schema is shown in Illustration 16.3-1. In this figure, the following diagrammatic convention is established to represent the information organization:

- each entity is represented by a rectangular box
- each relationship is represented by a diamond shaped box.
- the fact that a relationship exists between two entities is depicted by lines connecting the corresponding entity boxes to the relationship boxes.

The relationships are fully defined with the maximum number of possible entities in the relationship shown. In other words, the relationship between Series and Image can have up to n Images per Series.

ILLUSTRATION 16.3-1
STUDY ROOT QUERY/RETRIEVE INFORMATION MODEL E/R DIAGRAM

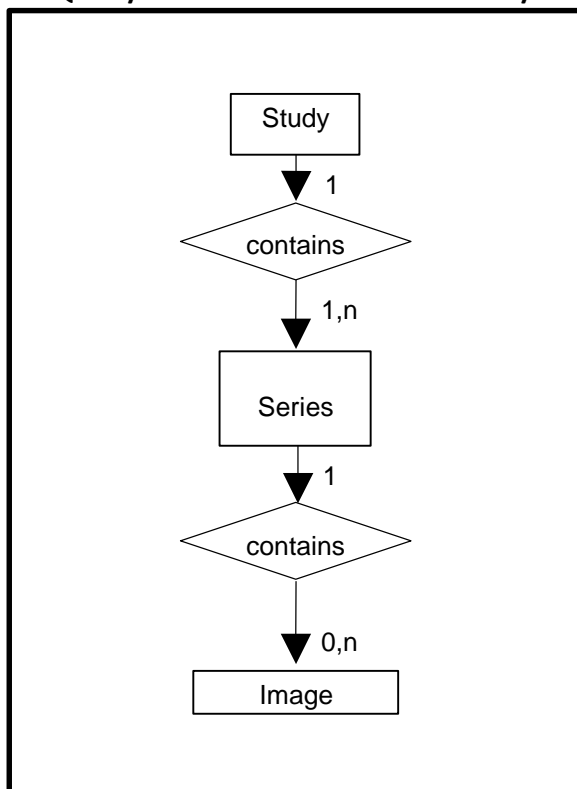

16.3.1 Entity Descriptions

Please refer to DICOM Standard PS 3.4 (Service Class Specifications) for a description of each of the levels contained within the Study Root Query/Retrieve Information Model.

16.3.2 LOGIQ Scanner Mapping of DICOM entities

TABLE 16-1
MAPPING OF DICOM ENTITIES TO LOGIQ SCANNER ENTITIES

DICOM	LOGIQ Scanner Entity
Study	Exam
Series	Exam
Image	Image

16.4 INFORMATION MODEL KEYS

Please refer to DICOM Standard PS 3.4 (Service Class Specifications) for a description of each of the levels contained within the Study Root Query/Retrieve Information Model.

The following Level descriptions are included to specify what data elements are supported and what type of matching can be applied. It should be noted that they are the same ones as defined in the DICOM Standard PS 3.4 (Service Class Specifications).

16.4.1 Supported Matching

Following are the types of matching that can be requested by the implementation:

- Single Value matching (S)
- List of UID matching
- Universal Matching (U)
- Wild Card Matching (*)
- Range of date, Range of Time (R)
- Sequence Matching

Fields with “Filtering is supported” in the Matching column can be controlled from the Search screen. This means that the user can filter the downloaded C-FIND result, to view a limited set of the result.

All non-required matching fields can be configured in Config screen to be either enabled, enabled with a constant value or disabled. The constant value will be used as entered by user.

16.4.2 Study Level

This section defines the keys at the Study Level of the Study Root Query/Retrieve Information Model that are supported by this implementation.

TABLE 16-2
STUDY LEVEL ATTRIBUTES FOR THE STUDY ROOT
QUERY/RETRIEVE INFORMATION MODEL

Attribute Name	Tag	Type	Type of Matching
Study Date	(0008,0020)	R	R (Filter is supported)
Study Time	(0008,0030)	R	R
Accession Number	(0008,0050)	R	*, S, U (Filter is supported)
Patient's Name	(0010,0010)	R	*, U (Filter is supported)
Patient ID	(0010,0020)	R	*, U (Filter is supported)
Study ID	(0020,0010)	R	S, *, U
Study Instance UID	(0020,000D)	U	UNIQUE
Modalities in Study	(0008,0061)	O	S, *, U
Referring Physician's Name	(0008,0090)	O	S, *, U

Attribute Name	Tag	Type	Type of Matching
Study Description	(0008,1030)	O	S, *, U (Filter is supported)
Procedure Code Sequence	(0008,1032)	O	U
Name of Physician(s) Reading Study	(0008,1060)	O	U
Admitting Diagnoses Description	(0008,1080)	O	U
Referenced Study Sequence	(0008,1110)	O	U
Referenced Patient Sequence	(0008,1120)	O	U
Patient's Birth Date	(0010,0030)	O	R
Patient's Birth Time	(0010,0032)	O	R
Patient's Sex	(0010,0040)	O	S, U
Other Patient Names	(0010,1001)	O	U
Patient's Age	(0010,1010)	O	U
Patient's Size	(0010,1020)	O	U
Patient's Weight	(0010,1030)	O	U
Ethnic Group	(0010,2160)	O	U
Occupation	(0010,2180)	O	U
Additional Patient History	(0010,2180)	O	U
Patient Comments	(0010,4000)	O	U
Other Study Numbers	(0020,1070)	O	U
Number of Patient Related Studies	(0020,1200)	O	U
Number of Patient Related Series	(0020,1202)	O	U
Number of Patient Related Instances	(0020,1204)	O	U
Number of Study Related Series	(0020,1206)	O	U

Attribute Name	Tag	Type	Type of Matching
Number of Study Related Instances	(0020,1208)	O	U
Interpretation Author	(4008,010C)	O	U

TABLE 16-3

Q/R STUDY LEVEL AND LOCATION FOR RETRIEVE ATTRIBUTES

Attribute Name	Tag	Type	Note
Query Retrieve Level	(0008,0052)	-	Value = STUDY

TABLE 16-4

Q/R SPECIFIC CHARACTER SET ATTRIBUTES

Attribute Name	Tag	Type	Note
Specific Character Set	(0008,0005)	-	Set to "ISO-IR 100" if extended characters are used in query. ISO-IR 100 is supported in responses.

16.4.3 Series Level

This section defines the keys at the Series Level of the Study Root Query/Retrieve Information Model that are supported by this implementation.

TABLE 16-5

**SERIES LEVEL ATTRIBUTES FOR THE STUDY ROOT
 QUERY/RETRIEVE INFORMATION MODEL**

Attribute Name	Tag	Type	Type of Matching
Modality	(0008,0060)	R	U
Series Number	(0020,0011)	R	U
Series Instance UID	(0020,000E)	U	UNIQUE
Number of Series Related Instances	(0020,1209)	O	U
Series Date	(0008,0021)	O	R
Series Time	(0008,0031)	O	R
Performing Physicians' Name	(0008,1050)	O	U
Protocol Name	(0018,1030)	O	U
Series Description	(0008,103E)	O	U

Attribute Name	Tag	Type	Type of Matching
Operator's Name	(0008,1070)	O	U
Institutional Department Name	(0008,1040)	O	U
Software Versions	(0018,1020)	O	U
Performed Procedure Step Start Date	(0040,0244)	O	R
Performed Procedure Step Start Time	(0040,0245)	O	R

TABLE 16-6

Q/R SERIES LEVEL AND LOCATION FOR RETRIEVE ATTRIBUTES

Attribute Name	Tag	Type	Note
Query Retrieve Level	(0008,0052)	-	Value = SERIES

TABLE 16-7

Q/R SPECIFIC CHARACTER SET ATTRIBUTES

Attribute Name	Tag	Type	Note
Specific Character Set	(0008,0005)	-	Set to "ISO-IR 100" if extended characters are used in query. ISO-IR 100 is supported in responses.

16.4.4 Image Level

This section defines the keys at the Image Level of the Study Root Query/Retrieve Information Model that are supported by this implementation.

TABLE 16-8

**IMAGE LEVEL ATTRIBUTES FOR THE STUDY ROOT
 QUERY/RETRIEVE INFORMATION MODEL**

Attribute Name	Tag	Type	Type of Matching
Instance Number	(0020,0013)	R	U
SOP Instance UID	(0008,0018)	U	UNIQUE
Contrast/Bolus Agent	(0018,0010)	O	U

TABLE 16-9
Q/R IMAGE LEVEL AND LOCATION FOR RETRIEVE ATTRIBUTES

Attribute Name	Tag	Type	Note
Query Retrieve Level	(0008,0052)	-	Value = IMAGE

TABLE 16-10
Q/R SPECIFIC CHARACTER SET ATTRIBUTES

Attribute Name	Tag	Type	Note
Specific Character Set	(0008,0005)	-	Set to "ISO-IR 100" if extended characters are used in query. ISO-IR 100 is supported in responses.

16.5 PRIVATE DATA DICTIONARY


No private data dictionary is defined.

17. DICOM SR TEMPLATES

17.1 VASCULAR ULTRASOUND PROCEDURE REPORT

This section describes the contents of the Vascular Ultrasound Procedure Report (TID 5100) SR.

17.1.1 Vascular Template Structure

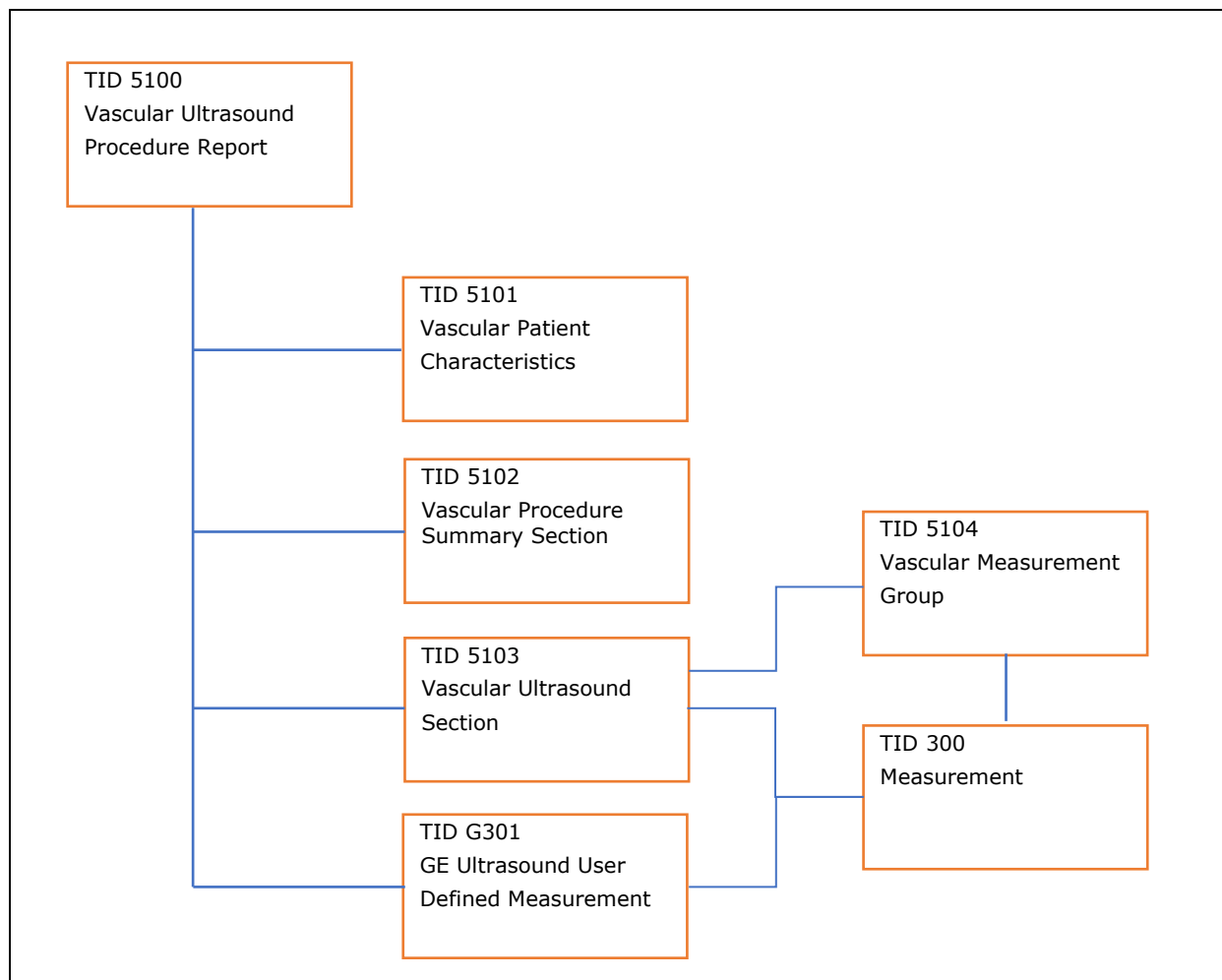

17.1.2 Usage and Extension of TID 5100 Vascular Ultrasound Report

LOGIQ Scanner supports the following template TID 5100 for SOP Instances created by this product when exam types selected are Vascular and Abdomen.

TABLE 17-1 TID 5100

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (125100, DCM, "Vascular Ultrasound Procedure Report")	1	M		
2	>	CONTAINS	INCLUDE	DTID (5101) Vascular Patient Characteristics	1	U		
3	>	CONTAINS	INCLUDE	DTID (5103) Vascular Ultrasound Section	1-n	U		
4	>	CONTAINS	INCLUDE	DTID (G301) GE Ultrasound User Defined Measurement	1-n	UC	When User creates user defined measurements via Measurement Configuration	

17.1.3 TID 5101 Vascular Patient Characteristics

TABLE 17-2 TID 5101

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (121118, DCM, "Patient Characteristics")	1	M		
2	>	CONTAINS	NUM	EV (121033, DCM, "Subject Age")	1	U		Units = DCID (7456) Units of Measure for Age
3	>	CONTAINS	CODE	EV (121032, DCM, "Subject Sex")	1	U		DCID (7455) Sex
4	>	CONTAINS	NUM	EV (8867-4, LN, "Heart Rate")	1	U		
5	>	CONTAINS	NUM	EV (F-008EC, SRT, "Systolic Blood Pressure")	1	U		
6	>	CONTAINS	NUM	EV (F-008ED, SRT, "Diastolic Blood Pressure")	1	U		

17.1.4 TID 5102 Vascular Procedure Summary Section

TABLE 17-3 TID 5103

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121111, DCM, "Summary")	1	M		
2	>	CONTAINS	TEXT	DCID 12101 "Vascular Summary"	1-n	M		May contain comments May contain indications

17.1.5 TID 5103 Vascular Ultrasound Section

TABLE 17-4 TID 5103

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121070, DCM, "Findings")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	M		See Table 17-6 GEU Applications and Extensions- \$SectionScope
3	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	MC	IFF Laterality is defined	See Table 17-9 GE Ultrasound Sidedness
4	>	HAS CONCEPT MOD	CODE	EV (G-0373, SRT, "Image Mode")	1	U		See Table 17-8 GE Ultrasound Modes
5	>	CONTAINS	INCLUDE	DTID (5104) Vascular Measurement Group	1-n	M		See Table 17-5 TID 5104
6	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement = \$AnatomyRatio Anatomy ratio is from Table 16-5

17.1.6 TID 5104 Vascular Ultrasound Measurement Group (extended)

TABLE 17-5 TID 5104

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	\$Anatomy GEU Parameters	1	M		See Table 17-6 GEU Applications and Extensions
2	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Proximal Anastomosis')	1	U	IFF Proximal Anastomosis is defined	See Table 17-10 GE Ultrasound Vessel Location
3	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Distal Anastomosis')	1	U	IFF Distal Anastomosis is defined	See Table 17-10 GE Ultrasound Vessel Location
4	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Proximal Stent Margin')	1	U	IFF Proximal Stent Margin is defined	See Table 17-10 GE Ultrasound Vessel Location
5	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Distal Stent Margin')	1	U	IFF Distal Stent Margin is defined	See Table 17-10 GE Ultrasound Vessel Location
6	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Stent Laterality')	1	U	IFF Stent Laterality is defined	See Table 17-9 GE Ultrasound Sidedness

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
7	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Stenosis Modifier')	1	U	IFF Stenosis Modifier is defined	See Table 17-10 GE Ultrasound Vessel Location
8	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Anatomic Modifier')	1	U	IFF Anatomic Modifier is defined	See Table 17-10 GE Ultrasound Vessel Location
9	>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, 'Topographical Modifier')	1	U	IFF Topographical Modifier is defined	See Table 17-10 GE Ultrasound Vessel Location
10	>	HAS CONCEPT MOD	TEXT	EV (125101, DCM, 'Vessel Branch')	1	UC	When user name vessel more specific	Vessel branch defined by user.
11	>	HAS CONCEPT MOD	TEXT	EV (GEU-1005-7, 99GEMS, 'Anatomy Label')	1	UC	When user inserts user defined study measurements	Text Value of Vessel Name or Folder
12	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement = See 17.1.11 \$Derivation = See DCID (3627) Measurement Type Anatomy ratio is from Table 16-5

17.1.7 GEU Applications and Extensions

This section specifies for each type of vascular study the anatomy for which the application can create measurements, and the mapping from the user interface abbreviations to the SNOMED codes.

TABLE 17-6 GEU APPLICATIONS AND EXTENSIONS

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name	
DT (121070, DCM, 'Findings')	EV (G-C171, SRT, "Laterality")				
(T-40501, SRT, 'Blood Vessel of Head')	(G-A101, SRT, Left) for Left, (G-A100, SRT, Right) for Right. Or (G-A103, SRT, Unilateral)	DCID 12105 Intracranial Cerebral Vessels Or DCID 12106 Intracranial Cerebral Vessels (Unilateral)		Anatomy GEU parameter	Code and Description
				ICA	(T-45300, SRT, 'Internal Carotid Artery')
				MCA	(T-45600, SRT, 'Middle Cerebral Artery')
				ACA	(T-45540, SRT, 'Anterior Cerebral Artery')
				PCA	(T-45900, SRT, 'Posterior Cerebral Artery')
				PComA	(T-45320, SRT, 'Posterior Communicating Artery')
				AComA	(T-45530, SRT, 'Anterior Communicating Artery')
				BA	(T-45800, SRT, 'Basilar Artery')
				VERT	(T-45700, SRT, 'Vertebral Artery')
				Basilar	(T-45800, SRT, 'Basilar Artery')
				Ves	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy'); (GEU-1005-7, 99GEMS, 'Anatomy Label') : Ves
				SVD	(125107,DCM,"Sample Volume Depth")
				AC	(125106,DCM,"Doppler Angle")
				Others	(GEU-1005-3, 99GEMS, General Anatomy)
Table v.1 TCD Study Folder Code MAP					
(T-45005, SRT, 'Artery of neck)	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID (12104) Extracranial Arteries	DCID (12123) Carotid Ratios	Anatomy GEU parameter	Code and Description
				VERT	(T-45700, SRT, 'Vertebral Artery')
				CCA	(T-45100, SRT, 'Common Carotid Artery')
				ICA	(T-45300, SRT, 'Internal Carotid Artery')
				BULB	(T-45170, SRT, 'Carotid Bulb')
				ECA	(T-45200, SRT, 'External Carotid Artery')
				SUBC	(T-46100, SRT, 'Subclavian Artery')
				BIF	(T-45160, SRT, 'Carotid Bifurcation')
				STA	(T-45270, SRT, 'Superficial Temporal Artery')
				Frontal Branch	(T-45270, SRT, 'Superficial Temporal Artery'); (125101, DCM, 'Vessel branch'): 'Frontal'
				ParietalBranch	(T-45270, SRT, 'Superficial Temporal Artery'); (125101, DCM, 'Vessel branch'): 'Parietal'
				Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')
				Stent	(A-25500, SRT, 'Stent')
				Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')
				2D/Stenosis	(T-45300, SRT, Internal Carotid Artery)
				Ves	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy'); (GEU-1005-7, 99GEMS, 'Anatomy Label') : Ves
				Others	(GEU-1005-3, 99GEMS, General Anatomy)
Table v.2 Carotid Study FOLDER Code MAPS					

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name	
DT (121070, DCM, 'Findings')	EV (G-C171, SRT, "Laterality")				
(T-47040, SRT, 'Artery of Lower Extremity')	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID (12109) Lower Extremity Arteries		Anatomy GEU parameter	Code and Description
				ComIliac	(T-46710, SRT, 'Common Iliac Artery')
				ExtIliac	(T-46910, SRT, 'External Iliac Artery')
				ComFemoral	(T-47400, SRT, 'Common Femoral Artery')
				SupFemoral	(T-47403, SRT, 'Superficial Femoral Artery')
				Popliteal	(T-47500, SRT, 'Popliteal Artery')
				AntTibial	(T-47700, SRT, 'Anterior Tibial Artery')
				PostTibial	(T-47600, SRT, 'Posterior Tibial Artery')
				Peroneal	(T-47630, SRT, 'Peroneal Artery')
				DorsPedis	(T-47741, SRT, 'Dorsalis Pedis Artery')
				DeepFemoral	(T-47440, SRT, 'Profunda Femoris Artery')
				Profunda	(T-47440, SRT, 'Profunda Femoris Artery')
				Pseudo	(M-32390, SRT, 'Pseudo Aneurysm')
				AVF	(M-39390, SRT, 'AV Fistula')
				Graft	(T-D000F, SRT, 'Vascular Graft')
				Stent	(A-25500, SRT, 'Stent')
				Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')
				Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')
				Ves	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy'); (GEU-1005-7, 99GEMS, 'Anatomy Label') : Ves
				2D/Stenosis	(T-47403, SRT, 'Superficial Femoral Artery')
				Others	(GEU-1005-3, 99GEMS, General Anatomy)
Table V.3 LEA STUDY Folder CODE MAPS					

Table V.3 LEA STUDY Folder CODE MAPS

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name	
DT (121070, DCM, 'Findings')	EV (G-C171, SRT, "Laterality")				
(T-49403, SRT, 'Vein of Lower Extremity')	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID (12110) Lower Extremity of Veins		Anatomy GEU parameter	Code and Description
				Popliteal	(T-49640, SRT, 'Popliteal Vein')
				LSaphenous	(T-49550, SRT, 'Lesser Saphenous Vein')
				AntTibial	(T-49630, SRT, 'Anterior Tibial Vein')
				PostTibial	(T-49620, SRT, 'Posterior Tibial Vein')
				Peroneal	(T-49650, SRT, 'Peroneal Vein')
				Profunda	(T-49660, SRT, 'Profunda Femoris Vein')
				Gsaphenous	(T-49530, SRT, 'Great Saphenous Vein')
				GreatSaphThigh	(R-10259, SRT, 'Great Saphenous Vein of Thigh')
				GreatSaphCalf	(R-1025A, SRT, 'Great Saphenous Vein of Calf')
				GreatSaphKnee	(T-49530, SRT, 'Great Saphenous Vein') with finding side (T-D9200, SRT, 'Knee')
				GreatSaphAnkle	(T-49530, SRT, 'Great Saphenous Vein') with finding side (T-15750, SRT, 'Ankle')
				SaphPopJunc	(T-4941A ,SRT, 'Saphenopopliteal junction')
				SmallSaphCalf	(T-49550, SRT, 'Lesser Saphenous Vein') w/i finding side (T-D9440, SRT, 'Calf of Leg')
				SmallSaphAnkle	(T-49550, SRT, 'Lesser Saphenous Vein') w/l finding side (T-15750, SRT, 'Ankle')
				SmallSaphCranialExt	(GEU-1004-45 ,99GEMS, 'Lesser Saphenous Vein Flows to Cranial Ext')
				SaphFemJunc	(T-D930A, SRT, 'Saphenofemoral Junction')
				Femoral	(G-035B, SRT, 'Femoral Vein')
				ComFemoral	(G-035B, SRT, 'Common Femoral Vein')
				ComIliac	(T-48920, SRT, 'Common Iliac Vein')
				ExtIliac	(T-48930, SRT, 'External Iliac Vein')
				IVC	(T-48710, SRT, 'Inferior Vena Cava')
				Ves	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy'); (GEU-1005-7, 99GEMS, 'Anatomy Label') : Ves
				Others	(GEU-1005-3, 99GEMS, General Anatomy)

Table V.4 LEV STUDY FOLDER CODE MAPS

Table V.4 LEV STUDY FOLDER CODE MAPS

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name																																
DT (121070, DCM, ‘Findings’)	EV (G-C171, SRT, "Laterality")																																			
(T-47020, SRT, ‘Artery of Upper Extremity’)	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID (12107) Upper Extremity Arteries		<div>Section Artery of Upper Extremity</div> <table><tr><th>Anatomy GEU parameter</th><th>Code and Description</th></tr><tr><td>SUBC</td><td>(T-46100, SRT, ‘Subclavian artery’)</td></tr><tr><td>Axill</td><td>(T-47100, SRT, ‘Axillary artery’)</td></tr><tr><td>BrachialA</td><td>(T-47160, SRT, ‘Brachial artery’)</td></tr><tr><td>RadialA</td><td>(T-47300, SRT, ‘Radial artery’)</td></tr><tr><td>UlnarA</td><td>(T-47200, SRT, ‘Ulnar artery’)</td></tr><tr><td>Palmar</td><td>(T-47340, SRT, ‘Deep Palmar Arch of Radial Artery’)</td></tr><tr><td>Innominate</td><td>(T-46010, SRT, ‘Innominate Artery’)</td></tr><tr><td>Pseudo</td><td>(M-32390, SRT, 'Pseudo Aneurysm')</td></tr><tr><td>AVF</td><td>(M-39390, SRT, 'AV Fistula')</td></tr><tr><td>Ves</td><td>(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves</td></tr><tr><td>Stent</td><td>(A-25500, SRT, 'Stent')</td></tr><tr><td>Pre-Stent</td><td>(GEU-1004-71, 99GEMS, 'Pre-Stent')</td></tr><tr><td>Post-Stent</td><td>(GEU-1004-72, 99GEMS, 'Post-Stent')</td></tr><tr><td>2D/Stenosis</td><td>(T-46100, SRT, ‘Subclavian artery’)</td></tr><tr><td>Others</td><td>(GEU-1005-3, 99GEMS, General Anatomy)</td></tr></table>	Anatomy GEU parameter	Code and Description	SUBC	(T-46100, SRT, ‘Subclavian artery’)	Axill	(T-47100, SRT, ‘Axillary artery’)	BrachialA	(T-47160, SRT, ‘Brachial artery’)	RadialA	(T-47300, SRT, ‘Radial artery’)	UlnarA	(T-47200, SRT, ‘Ulnar artery’)	Palmar	(T-47340, SRT, ‘Deep Palmar Arch of Radial Artery’)	Innominate	(T-46010, SRT, ‘Innominate Artery’)	Pseudo	(M-32390, SRT, 'Pseudo Aneurysm')	AVF	(M-39390, SRT, 'AV Fistula')	Ves	(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves	Stent	(A-25500, SRT, 'Stent')	Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')	Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')	2D/Stenosis	(T-46100, SRT, ‘Subclavian artery’)	Others	(GEU-1005-3, 99GEMS, General Anatomy)
Anatomy GEU parameter	Code and Description																																			
SUBC	(T-46100, SRT, ‘Subclavian artery’)																																			
Axill	(T-47100, SRT, ‘Axillary artery’)																																			
BrachialA	(T-47160, SRT, ‘Brachial artery’)																																			
RadialA	(T-47300, SRT, ‘Radial artery’)																																			
UlnarA	(T-47200, SRT, ‘Ulnar artery’)																																			
Palmar	(T-47340, SRT, ‘Deep Palmar Arch of Radial Artery’)																																			
Innominate	(T-46010, SRT, ‘Innominate Artery’)																																			
Pseudo	(M-32390, SRT, 'Pseudo Aneurysm')																																			
AVF	(M-39390, SRT, 'AV Fistula')																																			
Ves	(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves																																			
Stent	(A-25500, SRT, 'Stent')																																			
Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')																																			
Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')																																			
2D/Stenosis	(T-46100, SRT, ‘Subclavian artery’)																																			
Others	(GEU-1005-3, 99GEMS, General Anatomy)																																			

Table V.5 UEA Study Folder CODE MAPS

Table V.5 UEA Study Folder CODE MAPS

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name	
DT (121070, DCM, ‘Findings’)	EV (G-C171, SRT, "Laterality")				
(T-49103, SRT, ‘Vein of Upper Extremity’)	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID (12108) Upper Extremity Veins		Anatomy GEU parameter	Code and Description
				JugularV	(T-48170, SRT, ‘Internal Jugular vein’)
				InnoV	(T-48620, SRT, ‘Innominate vein’)
				SUBCV	(T-48330, SRT, ‘Subclavian vein’)
				AxillV	(T-49110, SRT, ‘Axillary vein’)
				CephV	(T-49240, SRT, ‘Cephalic vein’)
				BasilV	(T-48052, SRT, ‘Basilic vein’)
				BracV	(T-49350, SRT, ‘Brachial vein’)
				McubV	(T-49250, SRT, ‘Median Cubital vein’)
				RadialV	(T-49340, SRT, ‘Radial vein’)
				UlnarV	(T-49330, SRT, ‘Ulnar vein’)
				CephArm	(T-49240, SRT, ‘Cephalic vein’); finding side (T-D8200, SRT, ‘Arm’)
				CephAntecub	(T-49240, SRT, ‘Cephalic vein’); finding side (T-49215, SRT, ‘Antecubital Vein’)
				CephForearm	(T-49240, SRT, ‘Cephalic vein’); finding side (T-12402, SRT, ‘Forearm’)
				CephWrist	(T-49240, SRT, ‘Cephalic vein’); finding side (T-D8600, SRT, ‘Wrist’)
				BasilicArm	(T-48052, SRT, ‘Basilic vein’); finding side (T-D8200, SRT, ‘Arm’)
				BasilicAntecub	(T-48052, SRT, ‘Basilic vein’); finding side (T-49215, SRT, ‘Antecubital Vein’)
				BasilicForearm	(T-48052, SRT, ‘Basilic vein’); finding side (T-12402, SRT, ‘Forearm’)
				BasilicWrist	(T-48052, SRT, ‘Basilic vein’); finding side (T-D8600, SRT, ‘Wrist’)
				Pseudo	(M-32390, SRT, 'Pseudo Aneurysm')
				AVF	(M-39390, SRT, 'AV Fistula')
				Ves	(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves
				Graft	(T-D000F, SRT, 'Vascular Graft')
				Others	(GEU-1005-3, 99GEMS, General Anatomy)

TABLE V.6 UEV STUDY FOLDER CODE MAPS

TABLE V.6 UEV STUDY FOLDER CODE MAPS

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name																																							
DT (121070, DCM, ‘Findings’)	EV (G-C171, SRT, "Laterality")																																										
(T-71019, SRT, ‘Vascular Structure of Kidney’)	(G-A101, SRT, Left) for Left, or (G-A100, SRT, Right) for Right.	DCID(12115) Renal Vessels	DCID(12124) Renal Ratios	<table><tr><th>Anatomy GEU parameter</th><th>Code and Description</th></tr><tr><td>MRenalA</td><td>(T-46600, SRT, ‘Renal Artery’)</td></tr><tr><td>RenalV</td><td>(T-48740, SRT, ‘Renal Vein’)</td></tr><tr><td>SegmentalA</td><td>(T-46659, SRT, ‘Segmental Artery’)</td></tr><tr><td>InterlobarA</td><td>(T-4667D, SRT, ‘Interlobar Artery of Kidney’)</td></tr><tr><td>ArcuateA</td><td>(T-4668A, SRT, ‘Arcuate Artery of the Kidney’)</td></tr><tr><td>Aorta</td><td>(T-42000, SRT, ‘Aorta’)</td></tr><tr><td>Renal Volume</td><td>(T-71000, SRT, ‘Kidney’)</td></tr><tr><td>Spleen</td><td>(T-C3000, SRT, ‘Spleen’)</td></tr><tr><td>AAA</td><td>(D3-83305, SRT,"Abdominal Aortic Aneurysm")</td></tr><tr><td>Bladder</td><td>(T-74000, SRT, ‘Bladder’)</td></tr><tr><td>PostBladder</td><td>(GEU-1004-62, 99GEMS, ‘Post Bladder’)</td></tr><tr><td>Iliac</td><td>(T-46710, SRT, ‘Common Iliac Artery’)</td></tr><tr><td>RenalLength</td><td>(T-71000, SRT, ‘Kidney’)</td></tr><tr><td>NewBladder</td><td>(T-74000, SRT, ‘Bladder’)</td></tr><tr><td>NewPostBladder</td><td>(GEU-1004-62, 99GEMS, ‘Post Bladder’)</td></tr><tr><td>RAOrigin</td><td>(GEU-1005-4, 99GEMS, ‘Renal Artery Origin’)</td></tr><tr><td>Ves</td><td>(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves</td></tr><tr><td>Others</td><td>(GEU-1005-3, 99GEMS, General Anatomy)</td></tr></table>		Anatomy GEU parameter	Code and Description	MRenalA	(T-46600, SRT, ‘Renal Artery’)	RenalV	(T-48740, SRT, ‘Renal Vein’)	SegmentalA	(T-46659, SRT, ‘Segmental Artery’)	InterlobarA	(T-4667D, SRT, ‘Interlobar Artery of Kidney’)	ArcuateA	(T-4668A, SRT, ‘Arcuate Artery of the Kidney’)	Aorta	(T-42000, SRT, ‘Aorta’)	Renal Volume	(T-71000, SRT, ‘Kidney’)	Spleen	(T-C3000, SRT, ‘Spleen’)	AAA	(D3-83305, SRT,"Abdominal Aortic Aneurysm")	Bladder	(T-74000, SRT, ‘Bladder’)	PostBladder	(GEU-1004-62, 99GEMS, ‘Post Bladder’)	Iliac	(T-46710, SRT, ‘Common Iliac Artery’)	RenalLength	(T-71000, SRT, ‘Kidney’)	NewBladder	(T-74000, SRT, ‘Bladder’)	NewPostBladder	(GEU-1004-62, 99GEMS, ‘Post Bladder’)	RAOrigin	(GEU-1005-4, 99GEMS, ‘Renal Artery Origin’)	Ves	(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves	Others	(GEU-1005-3, 99GEMS, General Anatomy)
Anatomy GEU parameter	Code and Description																																										
MRenalA	(T-46600, SRT, ‘Renal Artery’)																																										
RenalV	(T-48740, SRT, ‘Renal Vein’)																																										
SegmentalA	(T-46659, SRT, ‘Segmental Artery’)																																										
InterlobarA	(T-4667D, SRT, ‘Interlobar Artery of Kidney’)																																										
ArcuateA	(T-4668A, SRT, ‘Arcuate Artery of the Kidney’)																																										
Aorta	(T-42000, SRT, ‘Aorta’)																																										
Renal Volume	(T-71000, SRT, ‘Kidney’)																																										
Spleen	(T-C3000, SRT, ‘Spleen’)																																										
AAA	(D3-83305, SRT,"Abdominal Aortic Aneurysm")																																										
Bladder	(T-74000, SRT, ‘Bladder’)																																										
PostBladder	(GEU-1004-62, 99GEMS, ‘Post Bladder’)																																										
Iliac	(T-46710, SRT, ‘Common Iliac Artery’)																																										
RenalLength	(T-71000, SRT, ‘Kidney’)																																										
NewBladder	(T-74000, SRT, ‘Bladder’)																																										
NewPostBladder	(GEU-1004-62, 99GEMS, ‘Post Bladder’)																																										
RAOrigin	(GEU-1005-4, 99GEMS, ‘Renal Artery Origin’)																																										
Ves	(GEU-1005-6, 99GEMS, ‘User Vessel Anatomy’); (GEU-1005-7, 99GEMS, ‘Anatomy Label’) : Ves																																										
Others	(GEU-1005-3, 99GEMS, General Anatomy)																																										

TABLE V.7-1. RENAL STUDY CODE MAPS

TABLE V.7-1. RENAL STUDY CODE MAPS

Section Scope	Section Laterality	Anatomy	Anatom y Ratio	GEU Parameters Base Measurement Concept Name		
DT (121070, DCM, ‘Findings’)	EV (G-C171, SRT, "Laterality")					
(T-46002, SRT, ‘Artery of Abdomen’),	(G-A101, SRT, Left) for Left, (G-A100, SRT, Right) for Right. Or (G-A103, SRT, Unilateral)	DCID 12111 or (12112) Abdominal Arteries (unilateral)		Anatomy GEU parameter	Code and Description	
				Aorta	(T-42000, SRT, ‘Aorta’)	
				Renal Length	(T-71000, SRT, ‘Kidney’)	
				Spleen	(T-C3000, SRT, ‘Spleen’)	
				Celiac	(T-46400, SRT, ‘Celiac Axis’)	
				Splenic A	(T-46460, SRT, ‘Splenic Artery’)	
				SMA	(T-46510, SRT, ‘Superior Mesenteric Artery’)	
				IMA	(T-46520, SRT, ‘Inferior Mesenteric Artery’)	
				CIA	(T-46710, SRT, ‘Common Iliac Artery’)	
				Stent	(A-25500, SRT, 'Stent')	
				Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')	
				Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')	
				HepaticA	(T-46420, SRT, ‘Hepatic Artery’)	
				IIA	(T-46740, SRT, ‘Internal Iliac Artery’)	
Table V.8-1 Abdomen Study CODE MaPs (Page 1)						
				Anatomy GEU parameter	Code and Description	
				Gall Bladder	(T-63000, SRT, ‘Gall Bladder’)	
				CBD	(T-60610, SRT, ‘Bile Duct’)	
				Liver	(T-62002, SRT, ‘Liver’)	
				Pancreas	(T-D4034, SRT, ‘Pancreas’)	
				AutoContourROI	(F-03FCD, SRT, ‘Lesion Finding’)	
				Others	(GEU-1005-3, 99GEMS, General Anatomy)	
TABLE V.8-2 ABDOMEN STUDY CODE MAPS (B-MODE)						
				Shearwave GEU parameter	Code and Description	Remarks
				Shearwave Stiffness	(GEU-1010-01, 99GEMS, ‘Shearwave Stiffness’)	Refer Table 17-14 GE Abdominal Shear wave MeasurementsError! Reference source not found.
				Shearwave Velocity	(GEU-1010-02, 99GEMS, ‘Shearwave Velocity’)	
				Shearwave Stiffness Velocity	(GEU-1010-03, 99GEMS, ‘Shearwave Stiffness Velocity’)	
TABLE V.8-3 ABDOMEN STUDY SHEARWAVE MEASUREMENT CODE						
				Attenuation GEU parameter	Code and Description	Remarks
				UGAPC	(GEU-1012-01, 99GEMS, ‘Attenuation Coefficient’)	Table 17-15 GE Abdominal Attenuation
				UGAPR	(GEU-1012-02, 99GEMS, ‘Attenuation Rate’)	Measurements

Section Scope	Section Laterality	Anatomy	Anatomy Ratio	GEU Parameters Base Measurement Concept Name																																							
DT (121070, DCM, ‘Findings’)	EV (G-C171, SRT, "Laterality")																																										
(T-487A0, SRT, ‘Vein of Abdomen)	(G-A101, SRT, Left) for Left, (G-A100, SRT, Right) for Right. Or (G-A103, SRT, Unilateral)	DCID 12113 or (12114) Abdominal Veins lateral (unilateral)		<table><tr><th>Anatomy GEU parameter</th><th>Code and Description</th></tr><tr><td>IIV</td><td>(T-48940, SRT, ‘Internal Iliac Vein’)</td></tr><tr><td>PortalVein</td><td>(T-48810, SRT, ‘Portal Vein’)</td></tr><tr><td>SMV</td><td>(T-48840, SRT, ‘Superior Mesenteric Vein’)</td></tr><tr><td>PortalV</td><td>(T-48810 , SRT, ‘Portal Vein’)</td></tr><tr><td>SplenicV</td><td>(T-48890, SRT, ‘Splenic Vein’)</td></tr><tr><td>IMV</td><td>(T-48910, SRT, ‘Inferior Mesenteric Vein’)</td></tr><tr><td>HepaticV</td><td>(T-48720, SRT, ‘Hepatic Vein’)</td></tr><tr><td>PortalV</td><td>(T-4882A, SRT, ‘Right Main Branch of Portal Vein’)</td></tr><tr><td>PortalV</td><td>(T-4881F, SRT, ‘Left Main Branch of Portal Vein’)</td></tr><tr><td>HepaticA</td><td>(T-46420, SRT, ‘Hepatic Artery’)</td></tr><tr><td>CHA</td><td>(T-46421, SRT, ‘Common Hepatic Artery’)</td></tr><tr><td>Stent</td><td>(A-25500, SRT, 'Stent')</td></tr><tr><td>Pre-Stent</td><td>(GEU-1004-71, 99GEMS, 'Pre-Stent')</td></tr><tr><td>Post-Stent</td><td>(GEU-1004-72, 99GEMS, 'Post-Stent')</td></tr></table>	Anatomy GEU parameter	Code and Description	IIV	(T-48940, SRT, ‘Internal Iliac Vein’)	PortalVein	(T-48810, SRT, ‘Portal Vein’)	SMV	(T-48840, SRT, ‘Superior Mesenteric Vein’)	PortalV	(T-48810 , SRT, ‘Portal Vein’)	SplenicV	(T-48890, SRT, ‘Splenic Vein’)	IMV	(T-48910, SRT, ‘Inferior Mesenteric Vein’)	HepaticV	(T-48720, SRT, ‘Hepatic Vein’)	PortalV	(T-4882A, SRT, ‘Right Main Branch of Portal Vein’)	PortalV	(T-4881F, SRT, ‘Left Main Branch of Portal Vein’)	HepaticA	(T-46420, SRT, ‘Hepatic Artery’)	CHA	(T-46421, SRT, ‘Common Hepatic Artery’)	Stent	(A-25500, SRT, 'Stent')	Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')	Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')									
				Anatomy GEU parameter	Code and Description																																						
				IIV	(T-48940, SRT, ‘Internal Iliac Vein’)																																						
				PortalVein	(T-48810, SRT, ‘Portal Vein’)																																						
				SMV	(T-48840, SRT, ‘Superior Mesenteric Vein’)																																						
				PortalV	(T-48810 , SRT, ‘Portal Vein’)																																						
				SplenicV	(T-48890, SRT, ‘Splenic Vein’)																																						
				IMV	(T-48910, SRT, ‘Inferior Mesenteric Vein’)																																						
				HepaticV	(T-48720, SRT, ‘Hepatic Vein’)																																						
				PortalV	(T-4882A, SRT, ‘Right Main Branch of Portal Vein’)																																						
				PortalV	(T-4881F, SRT, ‘Left Main Branch of Portal Vein’)																																						
				HepaticA	(T-46420, SRT, ‘Hepatic Artery’)																																						
				CHA	(T-46421, SRT, ‘Common Hepatic Artery’)																																						
				Stent	(A-25500, SRT, 'Stent')																																						
				Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')																																						
				Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')																																						
V – 9.2 Abdomen Vein Study Folder Code Maps																																											
(T-D000F, SRT, 'Vascular Graft')	(G-A101, SRT, Left) for Left, (G-A100, SRT, Right) for Right. Or (G-A103, SRT, Unilateral)	DCID 12113 or (12114) Abdominal Veins lateral (unilateral)		<table><tr><th>Anatomy GEU parameter</th><th>Code and Description</th></tr><tr><td>Limb</td><td>(GEU-1004-46, 99GEMS, 'Limb')</td></tr><tr><td>Inflow</td><td>(GEU-1004-63, 99GEMS, 'Inflow')</td></tr><tr><td>Anast</td><td>(M-18200, SRT, 'Anastomosis')</td></tr><tr><td>Outflow</td><td>(GEU-1004-44, 99GEMS, 'Outflow')</td></tr><tr><td>Thigh</td><td>(T-D9100, SRT, 'Thigh')</td></tr><tr><td>Knee</td><td>(T-D9200, SRT, 'Knee')</td></tr><tr><td>Calf</td><td>(T-D9440, SRT, 'Calf of Leg')</td></tr><tr><td>Ankle</td><td>(T-15750, SRT, 'Ankle')</td></tr><tr><td>RunOff</td><td>(GEU-1004-43, 99GEMS, 'RunOff')</td></tr><tr><td>VGraft</td><td>(T-D000F, SRT, 'Vascular Graft')</td></tr><tr><td>Arm</td><td>(T-D8200, SRT, 'Arm')</td></tr><tr><td>Ante Cub</td><td>(T-49215, SRT, 'Antecubital Vein')</td></tr><tr><td>Forearm</td><td>(T-12402, SRT, 'Forearm')</td></tr><tr><td>Wrist</td><td>(T-15460, SRT, 'Wrist joint')</td></tr><tr><td>UserVessel</td><td>(GEU-1005-6, 99GEMS, 'User Vessel Anatomy')</td></tr><tr><td>Pre-Stent</td><td>(GEU-1004-71, 99GEMS, 'Pre-Stent')</td></tr><tr><td>Stent</td><td>(A-25500, SRT, 'Stent')</td></tr><tr><td>Post-Stent</td><td>(GEU-1004-72, 99GEMS, 'Post-Stent')</td></tr></table>	Anatomy GEU parameter	Code and Description	Limb	(GEU-1004-46, 99GEMS, 'Limb')	Inflow	(GEU-1004-63, 99GEMS, 'Inflow')	Anast	(M-18200, SRT, 'Anastomosis')	Outflow	(GEU-1004-44, 99GEMS, 'Outflow')	Thigh	(T-D9100, SRT, 'Thigh')	Knee	(T-D9200, SRT, 'Knee')	Calf	(T-D9440, SRT, 'Calf of Leg')	Ankle	(T-15750, SRT, 'Ankle')	RunOff	(GEU-1004-43, 99GEMS, 'RunOff')	VGraft	(T-D000F, SRT, 'Vascular Graft')	Arm	(T-D8200, SRT, 'Arm')	Ante Cub	(T-49215, SRT, 'Antecubital Vein')	Forearm	(T-12402, SRT, 'Forearm')	Wrist	(T-15460, SRT, 'Wrist joint')	UserVessel	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy')	Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')	Stent	(A-25500, SRT, 'Stent')	Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')	
				Anatomy GEU parameter	Code and Description																																						
				Limb	(GEU-1004-46, 99GEMS, 'Limb')																																						
				Inflow	(GEU-1004-63, 99GEMS, 'Inflow')																																						
				Anast	(M-18200, SRT, 'Anastomosis')																																						
				Outflow	(GEU-1004-44, 99GEMS, 'Outflow')																																						
				Thigh	(T-D9100, SRT, 'Thigh')																																						
				Knee	(T-D9200, SRT, 'Knee')																																						
				Calf	(T-D9440, SRT, 'Calf of Leg')																																						
				Ankle	(T-15750, SRT, 'Ankle')																																						
				RunOff	(GEU-1004-43, 99GEMS, 'RunOff')																																						
				VGraft	(T-D000F, SRT, 'Vascular Graft')																																						
				Arm	(T-D8200, SRT, 'Arm')																																						
				Ante Cub	(T-49215, SRT, 'Antecubital Vein')																																						
				Forearm	(T-12402, SRT, 'Forearm')																																						
				Wrist	(T-15460, SRT, 'Wrist joint')																																						
UserVessel	(GEU-1005-6, 99GEMS, 'User Vessel Anatomy')																																										
Pre-Stent	(GEU-1004-71, 99GEMS, 'Pre-Stent')																																										
Stent	(A-25500, SRT, 'Stent')																																										
Post-Stent	(GEU-1004-72, 99GEMS, 'Post-Stent')																																										

17.1.8 TID 300 Measurement (extended for Vascular)

TABLE 17-7 GEU APPLICATIONS AND EXTENSIONS

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			NUM	\$Measurement	1	M		Units = \$Units
2	>	HAS CONCEPT MOD	TEXT	EV(GEU-1005-5, 99GEMS, Measurement Label)	1	UC	When Measurement is created by user (User Defined)	Measurement Label created by User (\$Label)
3	>	HAS CONCEPT MOD	CODE	EV(GEU-1005-5, 99GEMS, Measurement Label)	1	UC	Only for AP or Trans	- AP (122675, Anterior-Posterior, DCM,) - Trans (G-A117. Transverse, SRT)
4	>	HAS CONCEPT MOD	CODE	EV(G-A1F8, SRT, 'Topographical modifier')	1	U		See Table 17-10 GE Ultrasound Vessel Location
5	>	HAS CONCEPT MOD	CODE	EV(121401, DCM, 'Derivation')	1	U		\$Derivation
6	>	HAS PROPERTIES	CODE	EV(121404, DCM, 'Selection Status')	1	U		\$Selection Status

17.1.9 GE Ultrasound Modes

TABLE 17-8 GE ULTRASOUND MODES

GE Ultrasound Modes	Code Value
2D	(G-03A2, SRT, "2D mode")
CF	(R-409E2, SRT, "Doppler Color Flow")
PW	(R-409E4, SRT, "Doppler Pulsed")
MM	(G-0394, SRT, "M mode")
CW	(R-409E3, SRT, "Doppler Continuous Wave")

17.1.10 GE Ultrasound Sidedness and Vessel Location

TABLE 17-9 GE ULTRASOUND SIDEDNESS

Side	Code Value
Rt	(G-A100, SRT, "Right")
Lt	(G-A101, SRT, "Left")

Note (*) when there is no Sidedness or Locations, the SR nodes are not populated

TABLE 17-10 GE ULTRASOUND VESSEL LOCATION

Vessel Location	Code Value
Prox	(G-A118, SRT, "Proximal")
Mid	(G-A188, SRT, "Mid-longitudinal")
Dist	(G-A119, SRT, "Distal")
Pre	(GEU-1002-68, GEU, "Pre")
At	(GEU-1002-70, GEU, "At")
Post	(GEU-1002-69, GEU, "Post")
Upper	(GEU-1007-29, GEU, "Upper")
Lower	(GEU-1007-31, GEU, "Lower")
Inflow	(GEU-1004-63, GEU, "Inflow")
Outflow	(GEU-1004-44, GEU, "Outflow")

TABLE 17-11 GE ULTRASOUND STENOSIS LOCATION

Vessel Location	Code Value
Pre-Stenosis	(GEU-1002-65, GEU, "Pre-Stenosis")
At-Stenosis	(GEU-1002-66, GEU, "At-Stenosis")
Post-Stenosis	(GEU-1002-67, GEU, "Post-Stenosis")

TABLE 17-12 GE ULTRASOUND ANASTOMOSIS AND MARGIN LOCATION

Vessel Location	Code Value
CCA	(T-45100, SRT, 'Common Carotid Artery')
ICA	(T-45300, SRT, 'Internal Carotid Artery'E
ECA	(T-45200, SRT, 'External Carotid Artery')
BULB	(T-45170, SRT, 'Carotid Bulb')
Innominate	(T-46010, SRT, 'Innominate Artery')
VERT	(T-45700, SRT, 'Vertebral Artery')
SUBC	(T-46100, SRT, 'Subclavian artery')
STA	(T-45270, SRT, 'Superficial Temporal Artery')
EIA	(T-46910, SRT, "External Iliac Artery")
CFA	(T-47400, SRT, "Common Femoral Artery")
DFA	(T-47440, SRT, "Profunda Femoris Artery")
SFA	(T-47403, SRT, "Superficial Femoral Artery")
PopA	(T-47500, SRT, "Popliteal Artery")
ATA	(T-47700, SRT, "Anterior Tibial Artery")
PTA	(T-47600, SRT, "Posterior Tibial Artery")

Vessel Location	Code Value
PerA	(T-47630, SRT, "Peroneal Artery")
DPA	(T-47741, SRT, "Dorsalis Pedis Artery")
MCA	(T-45600, SRT, "Middle Cerebral Artery")
ACA	(T-45540, SRT, "Anterior Cerebral Artery")
PCA	(T-45900, SRT, "Posterior Cerebral Artery")
ACoA	(T-45530, SRT, "Anterior Communicating Artery")
PCoA	(T-45320, GEU, "Posterior Communicating Artery")
Basi	(T-45800, SRT, "Basilar Artery")
Ves	(GEU-1005-6, SRT, "User Vessel Anatomy")
AxiA	(T-47100, SRT, "Axillary Artery")
BraA	(T-47160, SRT, "Brachial Artery")
RadA	(T-47300, SRT, "Radial Artery")
UlnA	(T-47200, SRT, "Ulnar Artery")
Pseudo	(M-32390, SRT, "Pseudo Aneurysm")
AVF	(M-39390, SRT, "AV Fistula")
SplA	(T-46460, SRT, "Splenic Artery")
RenA	(T-46600, SRT, "Renal Artery")
IIA	(T-46740, SRT, "Internal Iliac Artery")
Ao	(T-42000, SRT, "Aorta")
CeIA	(T-46400, SRT, "Celiac Axis")
SMA	(T-46510, SRT, "Superior Mesenteric Artery")
CIA	(T-46710, SRT, "Common Iliac Artery")
CHA	(T-46421, SRT, "Common Hepatic Artery")
IMA	(T-46520, SRT, "Inferior Mesenteric Artery")

TABLE 17-13 GE ULTRASOUND ANATOMIC LOCATION

Vessel Location	Code Value
Anastomosis	(M-18200, SRT, "Anastomosis")
Jump Graft Anastomosis	(GEU-D876, GEU, "Jump Graft Anastomosis")
Stent	(A-25500, SRT, "Stent")
Spine	(T-D0146, SRT, "Spine")
Hip	(T-15710, SRT, "Hip")
Knee	(T-D9200, SRT, "Knee")
Ankle	(T-15750, SRT, "Ankle")
ICA	(T-45300, SRT, "Internal Carotid Artery")
CAS	(GEU-D877, GEU, "Carotid Artery Stenting")
MCA	(T-45600, SRT, "Middle Cerebral Artery")

Vessel Location	Code Value
Circle of Willis	(T-45520, SRT, "Circle of Willis")
Axilla	(T-D8104, SRT, "Axilla")
Elbow	(T-15430, SRT, "Elbow")
Wrist	(T-15460, SRT, "Wrist")
Kidney	(T-71000, SRT, "Kidney")
Renal Artery	(T-46600, SRT, "Renal Artery")
Renal Transplant	(GEU-D878, GEU, "Renal Transplant")
Spleen	(T-C3000, SRT, "Spleen")
Liver	(T-62000, SRT, "Liver")
Pancreas	(T-D4034, SRT, "Pancreas")
Portal Vein	(T-48810, SRT, "Portal Vein")
Tip	(GEU-1002-72, GEU, "Tip")
SMA	(T-46510, SRT, "Superior Mesenteric Artery")
IMA	(T-46520, SRT, "Inferior Mesenteric Artery")

TABLE 17-14 GE ABDOMINAL SHEAR WAVE MEASUREMENTS

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (GEU-1010-01, 99GEMS, "Shearwave Stiffness")	1	M		
2	>	CONTAINS	NUM	EV(GEU-1003-9, 99GEMS, "Pressure")	1-12	M		
3	>	HAS CONCEPT MOD	TEXT	EV(GEU-1005-5, 99GEMS, 'Measurement Label')	1-12	M		"E1" to "E12"
4			CONTAINER	EV (GEU-1010-02, 99GEMS, "Shearwave Velocity")	1	M		
5	>	CONTAINS	NUM	EV (G-D784, SRT, "Velocity")	1-12	M		
6	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1-12	M		"V1" to "V12"
7			CONTAINER	EV (GEU-1010-03, 99GEMS, "Shearwave Stiffness Velocity")	1	M		
9	>	CONTAINS	NUM	EV(GEU-1003-9, 99GEMS, "Pressure")	1-12	M		
11	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1-12	M		"E1" to "E12"
10	>	CONTAINS	NUM	EV (G-D784, SRT, "Velocity")	1-12	M		
11	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1-12	M		"V1" to "V12"

TABLE 17-15 GE ABDOMINAL ATTENUATION MEASUREMENTS

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (GEU-1012-01, 99GEMS, "Attenuation Coefficient")	1	M		
2	>	HAS CONCEPT MOD	NUM	EV(GEU-1002-71, 99GEMS, 'Lesion Number')	1-12	M		1 to 12
3	>	CONTAINS	TEXT	EV (GEU-1004-74, 99GEMS, 'Measurement Property')	1	UC		Site name defined by user.
4	>>	HAS CONCEPT MOD	TEXT	EV (GEU-1005-5, 99GEMS, 'Measurement Label')	1	MC		Name
5	>	CONTAINS	NUM	\$Measurement	1	M		Refer table Table 17-16 Attenuation Coefficient Measurements Parameters
6			CONTAINER	EV (GEU-1012-02, 99GEMS, "Attenuation Rate")	1	M		
7	>	HAS CONCEPT MOD	TEXT	EV(GEU-1002-71, 99GEMS, 'Lesion Number')	1-12	M		1 to 12
8	>	CONTAINS	TEXT	EV (GEU-1004-74, 99GEMS, 'Measurement Property')	1	UC		Site name defined by user.
9	>>	HAS CONCEPT MOD	TEXT	EV (GEU-1005-5, 99GEMS, 'Measurement Label')	1	MC		Name
10	>	CONTAINS	NUM	\$Measurement	1	M		Refer table Table 17-17 Attenuation Rate Measurements Parameters

TABLE 17-16 ATTENUATION COEFFICIENT MEASUREMENTS PARAMETERS

Name of GEU parameter	Base Measurement Concept Name
UGAPCMean	(GEU-1004-76, 99GEMS, "UGAPCMean")
UGAPCMedian	(GEU-1004-77, 99GEMS, "UGAPCMedian")
UGAPCStd	(GEU-1004-78, 99GEMS, "UGAPCStd")
UGAPCIqr	(GEU-1004-79, 99GEMS, "UGAPCIqr")
UGAPCIqrMedianRatio	(GEU-1004-80, 99GEMS, "UGAPCIqrMedianRatio")
UGAPCCnt	(GEU-1004-81, 99GEMS, "UGAPCCnt")
UGAPC1	(GEU-1004-82, 99GEMS, "UGAPC1")
UGAPC2	(GEU-1004-83, 99GEMS, "UGAPC2")
UGAPC3	(GEU-1004-84, 99GEMS, "UGAPC3")
UGAPC4	(GEU-1004-85, 99GEMS, "UGAPC4")
UGAPC5	(GEU-1004-86, 99GEMS, "UGAPC5")
UGAPC6	(GEU-1004-87, 99GEMS, "UGAPC6")
UGAPC7	(GEU-1004-88, 99GEMS, "UGAPC7")
UGAPC8	(GEU-1004-89, 99GEMS, "UGAPC8")
UGAPC9	(GEU-1004-90, 99GEMS, "UGAPC9")
UGAPC10	(GEU-1004-91, 99GEMS, "UGAPC10")

UGAPC11	(GEU-1004-92, 99GEMS, "UGAPC11")
UGAPC12	(GEU-1004-93, 99GEMS, "UGAPC12")

TABLE 17-17 ATTENUATION RATE MEASUREMENTS PARAMETERS

Name of GEU parameter	Base Measurement Concept Name
UGAPRMean	(GEU-1004-94, 99GEMS, "UGAPRMean")
UGAPRMedian	(GEU-1004-94, 99GEMS, "UGAPRMedian")
UGAPRStd	(GEU-1004-96, 99GEMS, "UGAPRStd")
UGAPRIqr	(GEU-1004-97, 99GEMS, "UGAPRIqr")
UGAPRIqrMedianRatio	(GEU-1004-98, 99GEMS, "UGAPRIqrMedianRatio")
UGAPRCnt	(GEU-1004-99, 99GEMS, "UGAPRCnt")
UGAPR1	(GEU-1004-100, 99GEMS, "UGAPR1")
UGAPR2	(GEU-1004-101, 99GEMS, "UGAPR2")
UGAPR3	(GEU-1004-102, 99GEMS, "UGAPR3")
UGAPR4	(GEU-1004-103, 99GEMS, "UGAPR4")
UGAPR5	(GEU-1004-104, 99GEMS, "UGAPR5")
UGAPR6	(GEU-1004-105, 99GEMS, "UGAPR6")
UGAPR7	(GEU-1004-106, 99GEMS, "UGAPR7")
UGAPR8	(GEU-1004-107, 99GEMS, "UGAPR8")
UGAPR9	(GEU-1004-108, 99GEMS, "UGAPR9")
UGAPR10	(GEU-1004-109, 99GEMS, "UGAPR10")
UGAPR11	(GEU-1004-110, 99GEMS, "UGAPR11")
UGAPR12	(GEU-1004-111, 99GEMS, "UGAPR12")

17.1.11 SR Mapping Table for Vascular Base Measurement Concept

17.1.11.1 PWD-Mode MEASUREMENTS

TABLE 17-18 PWD-MODE MEASUREMENTS

GEU Measurement Parameter	Standard Measurement Concept Name
PS, Ps_velocitySD, Ps_velocityDS	(11726-7, LN, 'Peak Systolic Velocity')
ED, Ved	(11653-3, LN, 'End Diastolic Velocity')
MD, Md_velocitySD, Md_velocityDS	(11665-7, LN, 'Minimum Diastolic Velocity')
Tamax, TamaxM	(11692-1, LN, 'Time averaged peak velocity')
PI	(12008-9, LN, 'Pulsatility Index')
RI, PI/RI	(12023-8, LN, 'Resistivity Index')
SD Ratio, Ratio S/D	(12144-2, LN, 'Systolic to Diastolic Velocity Ratio')
DS Ratio, Ratio D/S	(122218, DCM, 'Diastolic to Systolic Velocity Ratio')
Accel	(20167-3, LN, 'Acceleration Index')
AT	(20168-1, LN, 'Acceleration Time')

GEU Measurement Parameter	Standard Measurement Concept Name
TAMEAN	(20352-1, LN, 'Time averaged mean velocity')
VOLFLOW	(33878-0, LN, 'Volume flow')
ICACCA Ratio (PS)	(33868-1, LN, 'ICA/CCA velocity')
HR, HR Cycle	(8867-4, LN, 'Heart Rate')
RAR	(33869-9, LN, 'Renal Artery/Aorta velocity Ratio')
AC	(GEU-1004-9, 99GEMS, 'Angular Correction')
IVRatio	(GEU-1004-11, 99GEMS, 'Intra Vessel Ratio')
PS/Hz	(GEU-1004-13, 99GEMS, 'Peak Systolic Frequency')
ED/Hz	(GEU-1004-14, 99GEMS, 'End Diastolic Frequency')
PV/Hz	(GEU-1004-15, 99GEMS, 'Peak Velocity Frequency')
MD/Hz	(GEU-1004-16, 99GEMS, 'Minimum Diastolic Frequency')
Vmax	(GEU-1004-24, 99GEMS, 'Maximum Velocity')
Vmin	(GEU-1004-25, 99GEMS, 'Minimum Velocity')
MaxPG	(GEU-1004-26, 99GEMS, 'Maximum Pressure Gradient')
MeanPG	(GEU-1004-27, 99GEMS, 'Average Pressure Gradient')
GreatSaphAccess	(GEU-1004-73, 99GEMS, 'Great Saphenous Vein of Accessory')

17.1.11.2 Vascular B- MODE Measurements

TABLE 17-19 VASCULAR B-MODE MEASUREMENTS

GEU Measurement Parameter	Standard Measurement Concept Name
DiamStenD1, DiamStenD2	(G-0364, SRT, 'Vessel Lumen Diameter')
AreaStenA1, AreaStenA2	(G-0366, SRT, 'Vessel Lumen Cross-Sectional Area')
StenosisD, Stenosis	(R-101BB, SRT, 'Lumen Diameter Stenosis')
StenosisA	(R-101BA, SRT, 'Lumen Area Stenosis')
AP	(M-02550, SRT, 'Diameter') with Measurement Labels (122675, DCM, 'Anterior-Posterior')
Trans	(M-02550, SRT, 'Diameter') with Measurement Labels (G-A117, SRT, 'Transverse')
Angle	(GEU-1004-18, 99GEMS, 'Angle')
Ellipse	(125226, DCM, 'Single Plane Ellipse')
Area, AreaRatioA1, AreaRatioA2	(121056, DCM, 'Area Outline')
SplenicJPNL	(GEU-1004-56, 99GEMS, 'Splenic D1')
SplenicJPNH	(GEU-1004-57, 99GEMS, 'Splenic D2')
SplenicIndexJPN	(GEU-1004-58, 99GEMS, 'Splenic Index-JP')
IMT Ant Avg	(GEU-1005-20, 99GEMS, 'IMT Anterior Average')
IMT Ant Max	(GEU-1005-21, 99GEMS, 'IMT Anterior Max')

GEU Measurement Parameter	Standard Measurement Concept Name
IMT Ant Min	(GEU-1005-22, 99GEMS, 'IMT Anterior Min')
IMT Ant SD	(GEU-1005-23, 99GEMS, 'IMT Anterior SD')
IMT Ant nMeas	(GEU-1005-24, 99GEMS, 'IMT Anterior nMeas')
IMT Ant Dist	(GEU-1005-25, 99GEMS, 'IMT Anterior Dist')
IMT Post Avg	(GEU-1005-26, 99GEMS, 'IMT Posterior Average')
IMT Post Max	(GEU-1005-27, 99GEMS, 'IMT Posterior Max')
IMT Post Min	(GEU-1005-28, 99GEMS, 'IMT Posterior Min')
IMT Post SD	(GEU-1005-29, 99GEMS, 'IMT Posterior SD')
IMT Post nMeas	(GEU-1005-30, 99GEMS, 'IMT Posterior nMeas')
IMT Post Dist	(GEU-1005-31, 99GEMS, 'IMT Posterior Dist')
DiamRatio	(GEU-1004-20, 99GEMS, 'Diameter Ratio')
VFDiam	(GEU-1004-49, 99GEMS, 'Volume Flow Diameter')
VolD1	(GEU-1004-52, 99GEMS, 'Volume Diameter 1')
VolD2	(GEU-1004-53, 99GEMS, 'Volume Diameter 2')
VolD3	(GEU-1004-54, 99GEMS, 'Volume Diameter 3')
ABDiamRatio	(GEU-1004-55, 99GEMS, 'A/B Diameter Ratio')
ABAreaRatio	(GEU-1004-66, 99GEMS, 'A/B Area Ratio')

17.1.11.3 Abdomen B- MODE Measurements

TABLE 17-20 ABDOMEN B-MODE MEASUREMENTS

GEU Measurement Parameter	Standard Measurement Concept Name
SpleenDistL, SpleenDistH, SpleenDistW, SpleenVolume	(Length, SRT,G-A22A), (Height, DCM, 121207), (Width, SRT, G-A220), (Volume, SRT, G-D705) inside (T-C3000, SRT, Spleen) container
RenalDistL, RenalDistH, RenalDistW, RenalVolume	(Length, SRT,G-A22A), (Height, DCM, 121207), (Width, SRT, G-A220), (Volume, SRT, G-D705) inside (T-71000, SRT, Kidney) container
BladderL, BladderH, BladderW, BladderVolume	(Length, SRT,G-A22A), (Height, DCM, 121207), (Width, SRT, G-A220), (Volume, SRT, G-D705) Inside (T-74000, SRT, Bladder) container
LiverL, LiverW	(Length, SRT,G-A22A), (Width, SRT, G-A220) Inside (T-62002, SRT, Liver) container
CBDporta, CBDpanc	(GEU-1004-67, 99GEMS, CBD Porta), (GEU-1004-68, 99GEMS, CBD Panc) (Length, SRT,G-A22A)

GEU Measurement Parameter	Standard Measurement Concept Name
	inside (T-60610, SRT, Bile Duct)
CBD	(Length, SRT,G-A22A) inside (T-60610, SRT, Bile Duct)
PancHead, PancBody, PancDuct	(GEU-1004-59, 99GEMS, Pancreas Head), (GEU-1004-60, 99GEMS, Pancreas Body), (GEU-1004-61, 99GEMS, Pancreas Duct)
AAAL	(Length, SRT,G-A22A) inside (R-002CD, SRT, Aneurysm of Aortic Root)
GBW	(GEU-1004-38,99GEMS, Gall Bladder Wall)
AoProxAP, AoProxTrans	AP = (Diameter,SRT, M-02550), with Measurement Label (Anterior-Posterior, DCM, 122675), Trans = (Diameter,SRT, M-02550), with Measurement Label (Transverse , SRT, G-A117) inside (T-42000, SRT, Aorta) with (Proximal, SRT,G-A118) as Topographical Modifier
AoMidAP, AoMidTrans	AP = (Diameter,SRT, M-02550), with Measurement Label (Anterior-Posterior, DCM, 122675), Trans = (Diameter,SRT, M-02550), with Measurement Label (Transverse , SRT, G-A117) inside (T-42000, SRT, Aorta) with (Mid-longitudinal, SRT,G-A188) as Topographical Modifier
AoDistAP, AoDistTrans	AP = (Diameter,SRT, M-02550), with Measurement Label (Anterior-Posterior, DCM, 122675), Trans = (Diameter,SRT, M-02550), with Measurement Label (Transverse , SRT, G-A117) inside (T-42000, SRT, Aorta) with (Distal, SRT,G-A119) as Topographical Modifier
IliacAP, IliacTrans	AP = (Diameter,SRT, M-02550), with Measurement Label (Anterior-Posterior, DCM, 122675), Trans = (Diameter,SRT, M-02550), with Measurement Label (Transverse, SRT, G-A117) inside (T-46710, SRT, Common Iliac Artery)
AutoContourL, AutoContourW, AutoContourH, ACArea	(Length, SRT, G-A22A), (Width, SRT, G-A220), (Height, DCM, 121207), (GEU-1006-11, 99GEMS, "Auto Contour Area") inside (F-03FCD, SRT, 'Lesion Finding')

17.1.12 TID G301 GE Ultrasound User Defined Measurement - Type: Extensible

The General Ultrasound User Defined Measurement Template provides a CONTAINER with a structure for reporting user-defined measurements and calculations.

The difference between this from TID 5103, TID 300 or 1414 for general measurement is the template uses labels that specified by users at configuration time rather assigning specific codes to the individual measurements.

Note: In practice, to assign specific codes to user defined measurements are impractical and cumbersome since the system/organization has to maintain database/map of codes and labels with public/private code designator schemes.

TABLE 17-21 TID G301

	NL	Relation with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (121070, DCM, 'Findings')	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, 'Laterality')	1	MC	IFF anatomy has laterality	DCID (244) Laterality
3	>	HAS CONCEPT MOD	CODE	EV (G-0373, SRT, 'Image Mode')	1	M		See Table 17-8 GE Ultrasound Modes
4	>	HAS CONCEPT MOD	CONTAINER	EV (GEU-1005-6, 99GEMS, 'User Vessel Anatomy') or (GEU-1005-3, 99GEMS, 'General Anatomy')	1	MC	IFF there exists user defined measurements	Note: User Vessel Anatomy mainly used to defined vessels and using Insert Study/Vessel template. General Anatomy mainly used in PWD and B-Mode and when user added individual folders and measurements.
5	>>	HAS CONCEPT MOD	TEXT	DT (GEU-1005-7, 99GEMS, 'Anatomy Label')	1	MC	IFF The container uses Finding as Concept Name	Labels from user which defined at configuration
6	>>	HAS CONCEPT MOD	CODE	EV (G-A1F8, SRT, Topographical modifier)	1	MC	IFF topographical defined	See Table 17-10 GE Ultrasound Vessel Location
7	>>	CONTAIN	INCLUDE	DTID (300) Measurement – see section: 17.1.8	1-n	M		\$Measurement = When 'Add Folder' insert study measurements by pre-populated study – Please refer to section 17.1.6. Vascular Ultrasound Measurement Group. When individually added 'Add Measurement' please refer to DCID (G5001) GE Defined

	NL	Relation with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
								<p>Measurements since new measurement codes are determined by its measurement units</p> <p>\$Units = DCID (82) Units of Measurement</p> <p>\$Derivation = DCID (3627) Measurement Type</p> <p>\$Selection = DCID (224) Selection Method</p> <p>\$Label = Measurement Label created by User</p>

17.1.13 Context ID G5001 GE Ultrasound Defined Measurement List

TABLE 17-22 CID G5001

distance-cm	cm	(121206, DCM, "Distance")
distance-mm	mm	(121206, DCM, "Distance")
volume-cm3	cm3	(G-D705, SRT, "Volume")
volume-mm3	mm3	(G-D705, SRT, "Volume")
frequency	kHz	(R-407E7, SRT, "Frequency")
frequency-h	Hz	(R-407E7, SRT, "Frequency")
area-cm2	cm2	(G-A166, SRT, "Area")
area-mm2	mm2	(G-A166, SRT, "Area")
time-s	s	(G-D7FF, SRT, "Time")
time-ms	ms	(G-D7FF, SRT, "Time")
time-min	min	(G-D7FF, SRT, "Time")
volume-flow	ml/min	(33878-0, LN, "Volume Flow")
acceleration-index	cm/s2	(20167-3, LN, "Acceleration Index")
calculated-value	no units	(GEU-1004-7, 99GEMS, "Calculated Value")
angle	deg	(GEU-1004-18, 99GEMS, "Angle")
volume-gram	g	(G-D705, SRT, "Volume")
volume-ml	ml	(G-D705, SRT, "Volume")
velocity	cm/s	(G-D784, SRT, "Velocity")
weekday	WeekDay	(GEU-1004-6, 99GEMS, "WeekDay")
date	Date	(GEU-1004-5, 99GEMS, "Date")
percent	%	(GEU-1004-4, 99GEMS, "Percent")
diameter-cm-m2	cm/m2	(GEU-1004-39, 99GEMS, "Diameter")

volume-index	ml/m2	(GEU-1004-40, 99GEMS, "Volume Index")
volume-flow-ml	ml/s	(33878-0, LN, "Volume Flow")
volume-flow-l	l/s	(33878-0, LN, "Volume Flow")
volume-flow-l-min	l/minm2	(33878-0, LN, "Volume Flow")
mass-flux	g/m2	(GEU-1004-42, 99GEMS, "Mass Flux")

17.2 ECHOCARDIOGRAPHY PROCEDURE REPORT

This section describes the contents of the Echocardiography Procedure Report (TID 5200) SR.

17.2.1 Echocardiography Structure

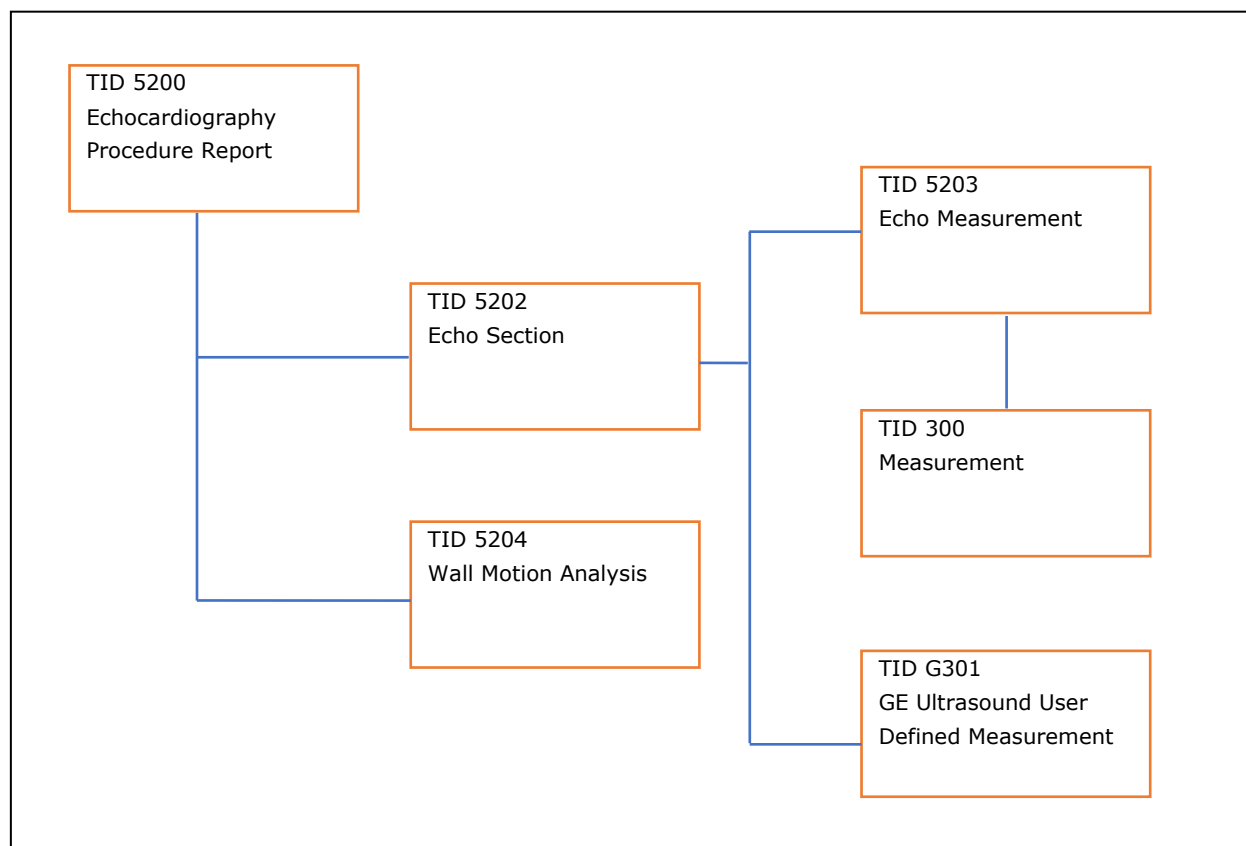


TABLE 17-23 TID 5200 – ECHOCARDIOGRAPHY PROCEDURE REPORT

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (125200, DCM, “Adult Echocardiography Procedure Report”)	1	M		
2		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-32600, SRT, “Left Ventricle”)
3		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-32500, SRT, “Right Ventricle”)
4		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-32300, SRT, “Left Atrium”)
5		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject =

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
								EV (T-32200, SRT, "Right Atrium")
6		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-35400, SRT, "Aortic Valve")
7		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-35300, SRT, "Mitral Valve")
8		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-35200, SRT, "Pulmonic Valve")
9		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-35100, SRT, "Tricuspid Valve")
10		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-42000, SRT, "Aorta")
11		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-44000, SRT, "Pulmonary artery")
12		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-48600, SRT, "Vena Cava")
13		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (T-48581, SRT, "Pulmonary Venous Structure")
14		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (P5-30031, SRT, "Cardiac Shunt Study")
15		CONTAINS	INCLUDE	DTID (5202) Echo Section	1	U		\$SectionSubject = EV (D4-30000, SRT, "Congenital Anomaly of Cardiovascular System")
16		CONTAINS	INCLUDE	DTID (5204) Wall Motion Analysis	1-n	U		\$Procedure = DT (P5-B3121, SRT, "Echocardiography")

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
								for Determining Ventricular Contraction")

Table 17- 24 TID 5202. Echo Section

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (121070, DCM, "Findings")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	M		\$SectionSubject
3	>	CONTAINS	CONTAINER	DT (125007, DCM, "Measurement Group")	1-n	M		
4	>>	HAS CONCEPT MOD	CODE	EV (G-0373, SRT, "Image Mode")	1	U		BCID 12224 "Ultrasound Image Modes"
5	>>	HAS CONCEPT MOD	CODE	DT (125203, DCM, "Acquisition Protocol")	1	U		
6	>>	HAS CONCEPT MOD	TEXT	DT (125203, DCM, "Acquisition Protocol")	1	U		
7	>>	HAS ACQ CONTEXT	CODE	EV (18139-6, LN, "Stage")	1	U		BCID 12002 "Ultrasound Protocol Stage Types"
8	>>	CONTAINS	INCLUDE	DTID 5203 "Echo Measurement"	1-n	M		\$Measurement = \$MeasType \$Method = CID 12227 "Echocardiography Measurement Method"
9	>	CONTAINS	INCLUDE	DTID G301 "User Defined Measurement"	1	M		

Table 17- 25 TID 5203. Echo Measurement

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			INCLUDE	DTID 300 "Measurement"	1	M		\$Measurement = \$Measurement \$Method = \$Method \$TargetSite = BCID 12236 "Echo Anatomic Sites" \$TargetSiteMod = BCID 12237 "Echocardiography Anatomic Site Modifiers"
2	>	HAS CONCEPT MOD	CODE	EV (G-C048, SRT, "Flow Direction")	1	U		BCID 12221 "Flow Direction"
3	>	HAS CONCEPT MOD	CODE	EV (R-40899, SRT, "Respiratory Cycle Point")	1	U		DCID 12234 "Respiration State"
4	>	HAS CONCEPT MOD	CODE	EV (R-4089A, SRT, "Cardiac Cycle Point")	1	U		DCID 12233 "Cardiac Phase"

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
5	>	HAS ACQ CONTEXT	CODE	EV (G-0373, SRT, "Image Mode")	1	U		DCID 12224 "Ultrasound Image Modes"
6	>	HAS ACQ CONTEXT	CODE	EV (111031, DCM, "Image View")	1	U		BCID 12226 "Echocardiography Image View"
7	>	HAS ACQ CONTEXT	CODE	EV (18139-6, LN, "Stage")	1	U		BCID 12002 "Ultrasound Protocol Stage Types"

TABLE 17-26 TID 5204. WALL MOTION ANALYSIS

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (121070, DCM, "Findings")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (121058, DCM, "Procedure reported")	1	M		\$Procedure
3	>	HAS ACQ CONTEXT	CODE	EV (18139-6, LN, "Stage")	1	U		BCID 3207 "Stress Test Procedure Phases"
4	>	CONTAINS	IMAGE	EV (125201, DCM, "Illustration of Finding")	1	U		
5	>	CONTAINS	TEXT	EV (18118-0, LN, "LV Wall Motion Segmental Findings")	1	U		
6	>	CONTAINS	NUM	DT (125202, DCM, "LV Wall Motion Score Index")	1	U		
7	>>	HAS CONCEPT MOD	CODE	EV (G-E048, SRT, "Assessment Scale")	1	M		BCID 12238 "Wall Motion Scoring Schemes"
8	>	CONTAINS	CONTAINER	EV (121070, DCM, "Findings")	1	U		
9	>>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	M		DT (T-D0772, SRT, "Myocardial Wall")
10	>>	CONTAINS	CODE	EV (18179-2, LN, "Wall Segment")	1-n	M		BCID 3717 "Myocardial Wall Segments"
11	>>>	HAS PROPERTIES	CODE	EV (F-32050, SRT, "Cardiac Wall Motion")	1	MC	IF row 12 is absent	DCID 3703 "Wall Motion"
12	>>>	HAS PROPERTIES	CODE	EV (G-C504, SRT, "Associated Morphology")	1	MC	IF row 11 is absent	DCID 3704 "Myocardium Wall Morphology Findings"
13	>>>	HAS PROPERTIES	NUM	DT (G-C1E3, SRT, "Score")	1	U		

GE Healthcare

DIR DOC2107162 REV 4

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
14	>>>	HAS PROPERTIES	NUM	EV (122624, DCM, "Wall Thickness Ratio end-systolic to end-diastolic")	1	U		UNITS = DT (% , UCUM, "%")

17.2.2 Aorta

TABLE 17-27 AORTA

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/Ao Diam SVals	(GEU-1005-45, 99GEMS, "Ao Diam SVals")	
2D/Ao Diam STub	(GEU-1005-46, 99GEMS, "Ao Diam STub")	
MM/LAAo/Ao Root Diam	(18015-8, LN, "Aortic Root Diameter")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
2D/Ao Root Diam	(18015-8, LN, "Aortic Root Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao Asc Diam	(18012-5, LN, "Ascending Aortic Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao st junct	(GEU-1005-40, 99GEMS, "Ao st junct")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao Arch Diam	(18011-7, LN, "Aortic Arch Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao Desc Diam	(18013-3, LN, "Descending Aortic Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao Isthmus	(18014-1, LN, "Aortic Isthmus Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Ao st junct/Ao	(GEU-1005-41, 99GEMS, "Ao st junct/Ao")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/Trans AoD diastole	(18015-8, LN, "Aortic Root Diameter")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/Trans AoD systole	(18015-8, LN, "Aortic Root Diameter")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/Trans AoD diastole	(18015-8, LN, "Aortic Root Diameter")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/Trans AoD systole	(18015-8, LN, "Aortic Root Diameter")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
MM/Ao Root Diam	(18015-8, LN, "Aortic Root Diameter")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
Asc Ao Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-42100, SRT, "Ascending aorta")
Asc Ao maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-42100, SRT, "Ascending aorta")
Dsc Ao Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-42400, SRT, "Descending aorta")
Dsc Ao maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-42400, SRT, "Descending aorta")

17.2.3 Aortic Valve

TABLE 17-28 AORTIC VALVE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/AVA/AV Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/AVA/AV Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
AV Dec Time	(20217-6, LN, "Deceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV Dec Slope	(20216-8, LN, "Deceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PISA/AR/RF	(G-0390, SRT, "Regurgitant Fraction")	(G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
CFM/AR Signal Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-0373, SRT, "Image Mode") = (R-409E2, SRT, "Doppler Color Flow")
MM/AV Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
2D/AV Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/AV Cusp	(17996-0, LN, "Aortic Valve Cusp Separation")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/Trans AVA diastole	(G-038E, SRT, "Cardiovascular Orifice Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/Trans AVA systole	(G-038E, SRT, "Cardiovascular Orifice Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/Trans AVA diastole	(G-038E, SRT, "Cardiovascular Orifice Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/Trans AVA systole	(G-038E, SRT, "Cardiovascular Orifice Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/AVA Planimetry	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125220, DCM, "Planimetry")
2D/AV Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
MM/AV Cusp	(17996-0, LN, "Aortic Valve Cusp Separation")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
AV Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV Vmax P	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
AV maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV Acc Time	(20168-1, LN, "Acceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AV VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA (VTI)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125215, DCM, "Continuity Equation by Velocity Time Integral")
AVA (Vmax)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA (Vmax)2	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA (Vmax)P	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA (Vmax)P2	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AV Env.Ti	(GEU-1005-34, 99GEMS, "AV Env.Ti")	
AV SV	(F-32120, SRT, "Stroke Volume")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AV HR	(8867-4, LN, "Heart rate")	
AV SI	(F-00078, SRT, "Stroke Index")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AV CO	(F-32100, SRT, "Cardiac Output")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AV CI	(F-32110, SRT, "Cardiac Index")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AVA/AV Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA/AV Vmax P	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA/AV maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA/AV Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA/AV meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
AVA/AV VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
AVA/AVA (VTI)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125215, DCM, "Continuity Equation by Velocity Time Integral")
AVA/AVA (Vmax)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA/AVA (Vmax)2	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA/AVA (Vmax)P	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA/AVA (Vmax)P2	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
AVA/AV Env.Ti	(GEU-1005-35, 99GEMS, "AVA/AV Env.Ti")	
AVA/AV SV	(F-32120, SRT, "Stroke Volume")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AVA/AV HR	(8867-4, LN, "Heart rate")	
AVA/AV SI	(F-00078, SRT, "Stroke Index")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AVA/AV CO	(F-32100, SRT, "Cardiac Output")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AVA/AV CI	(F-32110, SRT, "Cardiac Index")	(G-C0E3, SRT, "Finding Site") = (T-42000, SRT, "Aorta")
AV Acc Slope	(20174-9, LN, "Aortic Valve Acceleration Slope")	
AVET	(18041-4, LN, "Aortic Valve Ejection Time")	
AV Acc Time/ET Ratio	(G-0382, SRT, "Ratio of Aortic Valve Acceleration Time to Ejection Time")	
AV dp/dt	(GEU-1005-156, 99GEMS, "AV dp/dt")	
AV Time To Peak	(59121-4, LN, "Time to Peak by US")	
AR PHT	(20280-4, LN, "Pressure Half-Time")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR Dec Time	(20217-6, LN, "Deceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR Dec Slope	(20216-8, LN, "Deceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
ARend Vmax	(11653-3, LN, "End Diastolic Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
ARend maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (R-4089A, SRT, "Cardiac Cycle Point") = (F-32011, SRT, "End Diastole")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
AR Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR Env.Ti	(59108-1, LN, "Envelope Duration")	
AR VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
AR dp/dt	(GEU-1005-155, 99GEMS, "AR dp/dt")	
AP VTI	(20354-7, LN, "Velocity Time Integral")	
AP Area	(GEU-1005-131, 99GEMS, "AP Area")	
PISA/AR/Flow	(34141-2, LN, "Peak Instantaneous Flow Rate")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/AR/Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/AR/VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/AR/ERO	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/AR/RV	(33878-0, LN, "Volume Flow")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
AR HR	(8867-4, LN, "Heart rate")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
AV EOA I (VTI)	(GEU-1005-188, 99GEMS, "AV EOA I (VTI)")	
AV EOA I (Vmax)	(GEU-1005-189, 99GEMS, "AV EOA I Vmax Tr")	
AV EOA I (Vmax)2	(GEU-1005-190, 99GEMS, "AV EOA I Vmax Pt")	
AV EOA I (Vmax)P	(GEU-1005-191, 99GEMS, "AV EOA I Vmax Pt")	
AV EOA I (Vmax)P2	(GEU-1005-192, 99GEMS, "AV EOA I Vmax Tr")	
AVAI (VTI)	(GEU-1005-193, 99GEMS, "AVAI (VTI)")	
AVAI (Vmax)	(GEU-1005-194, 99GEMS, "AVAI Vmax Tr")	
AVAI (Vmax)2	(GEU-1005-195, 99GEMS, "AVAI (Vmax) Pt")	
AVAI (Vmax)P	(GEU-1005-196, 99GEMS, "AVAI (Vmax) Pt")	
AVAI (Vmax)P2	(GEU-1005-197, 99GEMS, "AVAI Vmax Tr")	

17.2.4 Congenital Anomaly of Cardiovascular System

TABLE 17- 29 CONGENITAL ANOMALY OF CARDIOVASCULAR SYSTEM

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/PDA Diam		(G-C0E3, SRT, "Finding Site") = (D4-32012, SRT, "Patent ductus

	(M-02550, SRT, "Diameter")	arteriosus") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/ASD Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (D4-31220, SRT, "Atrial Septal Defect")
2D/VSD Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (D4-31150, SRT, "Ventricular Septal Defect")
2D/PFO Diam	(M-02550, SRT, "Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (D4-31012, SRT, "Patent foramen ovale")
2D/Pre Ductal	(GEU-1005-84, 99GEMS, "Pre Ductal")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") See *1
2D/Post Ductal	(GEU-1005-83, 99GEMS, "Post Ductal")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") See *2
VSD Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (D4-31150, SRT, "Ventricular Septal Defect")
VSD maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (D4-31150, SRT, "Ventricular Septal Defect")
ASD Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (D4-31220, SRT, "Atrial Septal Defect")
ASD maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (D4-31220, SRT, "Atrial Septal Defect")
PDA Systolic	(GEU-1005-100, 99GEMS, "PDA Systolic")	(G-C0E3, SRT, "Finding Site") = (D4-32012, SRT, "Patent ductus arteriosus")
PDA Systolic PG	(GEU-1005-101, 99GEMS, "PDA Systolic PG")	(G-C0E3, SRT, "Finding Site") = (D4-32012, SRT, "Patent ductus arteriosus")
PDA Diastolic	(GEU-1005-102, 99GEMS, "PDA Diastolic")	(G-C0E3, SRT, "Finding Site") = (D4-32012, SRT, "Patent ductus arteriosus")
PDA Diastolic PG	(GEU-1005-103, 99GEMS, "PDA Diastolic PG")	(G-C0E3, SRT, "Finding Site") = (D4-32012, SRT, "Patent ductus arteriosus")
Coarctation/Pre-Ductal	(GEU-1005-98, 99GEMS, "Coarctation/Pre-Ductal")	(G-C0E3, SRT, "Finding Site") = (D4-32014, SRT, "Coarctation of aorta")
Coarctation/Pre-Ductal PG	(GEU-1005-99, 99GEMS, "Coarctation/Pre-Ductal PG")	(G-C0E3, SRT, "Finding Site") = (D4-32014, SRT, "Coarctation of aorta")
Coarctation/Post-Ductal	(GEU-1005-96, 99GEMS, "Coarctation/Post-Ductal")	(G-C0E3, SRT, "Finding Site") = (D4-32014, SRT, "Coarctation of aorta")
Coarctation/Post-Ductal PG	(GEU-1005-97, 99GEMS, "Coarctation/Post-Ductal PG")	(G-C0E3, SRT, "Finding Site") = (D4-32014, SRT, "Coarctation of aorta")

*1: Pre Ductal means the narrowing is proximal to the ductus arteriosus.

*2: Post Ductal means the narrowing is distal to the insertion of the ductus arteriosus.

17.2.5 Cardiac Shunt Study

TABLE 17- 30 CARDIAC SHUNT STUDY

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
Systemic HR	(GEU-1005-152, 99GEMS, "Systemic HR")	Left Ventricular HR
Systemic SV	(GEU-1005-95, 99GEMS, "Systemic SV")	Left Ventricular SV
Systemic CO	(GEU-1005-94, 99GEMS, "Systemic CO")	Left Ventricular CO
Pulmonic HR	(GEU-1005-151, 99GEMS, "Pulmonic HR")	Right Ventricular HR
Pulmonic SV	(GEU-1005-93, 99GEMS, "Pulmonic SV")	Right Ventricular SV
Pulmonic CO	(GEU-1005-92, 99GEMS, "Pulmonic CO")	Right Ventricular CO
2D/Pulmonic Diam	(GEU-1005-150, 99GEMS, "Pulmonic Diam")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/Systemic Diam	(GEU-1005-149, 99GEMS, "Systemic Diam")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")

Qp/Qs	(29462-9, LN, "Pulmonary-to-Systemic Shunt Flow Ratio")	
Systemic Vmax	(GEU-1005-120, 99GEMS, "Systemic Vmax")	Left Ventricular Vmax
Systemic Vmean	(GEU-1005-121, 99GEMS, "Systemic Vmean")	Left Ventricular Vmean
Systemic maxPG	(GEU-1005-117, 99GEMS, "Systemic maxPG")	Left Ventricular maxPG
Systemic meanPG	(GEU-1005-118, 99GEMS, "Systemic meanPG")	Left Ventricular meanPG
Systemic Env.Ti	(GEU-1005-116, 99GEMS, "Systemic Env.Ti")	Left Ventricular Env,Ti
Systemic VTI	(GEU-1005-44, 99GEMS, "Systemic VTI")	Left Ventricular VTI
Pulmonic Vmax	(GEU-1005-112, 99GEMS, "Pulmonic Vmax")	Right Ventricular Vmax
Pulmonic Vmean	(GEU-1005-113, 99GEMS, "Pulmonic Vmean")	Right Ventricular Vmean
Pulmonic maxPG	(GEU-1005-110, 99GEMS, "Pulmonic maxPG")	Right Ventricular maxPG
Pulmonic meanPG	(GEU-1005-111, 99GEMS, "Pulmonic meanPG")	Right Ventricular meanPG
Pulmonic Env.Ti	(GEU-1005-109, 99GEMS, "Pulmonic Env.Ti")	Right Ventricular Env,Ti
Pulmonic VTI	(GEU-1005-43, 99GEMS, "Pulmonic VTI")	Right Ventricular VTI

17.2.6 Left Atrium

TABLE 17- 31 LEFT ATRIUM

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/Ao/LA	(17985-3, LN, "Left Atrium to Aortic Root Ratio")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LA D1	(GEU-1005-59, 99GEMS, "LA D1")	
2D/LA D2	(GEU-1005-60, 99GEMS, "LA D2")	
2D/LA D3	(GEU-1005-61, 99GEMS, "LA D3")	
2D/LA Volume	(GEU-1005-62, 99GEMS, "LA Volume")	
MM/Ao/LA	(17985-3, LN, "Left Atrium to Aortic Root Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LA	(29469-4, LN, "Left Atrium Antero-posterior Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LAAo/LA	(29469-4, LN, "Left Atrium Antero-posterior Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LAAo/LA/Ao	(17985-3, LN, "Left Atrium to Aortic Root Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LAAo/Ao/LA	(GEU-1008-01, 99GEMS, "Aortic Root to Left Atrium Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
SD/LAappendix Vmax	(11726-7, LN, "Peak Velocity")	
2D/LA Volume Index	(GEU-1005-63, 99GEMS, "LA Volume Index")	
2D/LA Major	(G-A193, SRT, "Major Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LA Minor	(G-A194, SRT, "Minor Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LA	(29469-4, LN, "Left Atrium Antero-posterior Systolic Dimension")	
2D/LA/Ao	(17985-3, LN, "Left Atrium to Aortic Root Ratio")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAA diastole	(GEU-1005-69, 99GEMS, "LAA diastole")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAA systole	(17977-0, LN, "Left Atrium Systolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/LA Area	(G-A166, SRT, "Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
LALd(A4C)	(29467-8, LN, "Left Atrium Superior-Inferior Dimension, 4-chamber view")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole")
LAAd(A4C)	(17977-0, LN, "Left Atrium Area A4C view")	
LAEDV(A-L A4C)	(GEU-1005-54, 99GEMS, "LAEDV(A-L A4C)")	
LAESV(A-L A4C)	(G-0383, SRT, "Left Atrium Systolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LAEDV(MOD A4C)	(GEU-1005-56, 99GEMS, "LAEDV(MOD A4C)")	
LALs(A4C)	(29467-8, LN, "Left Atrium Superior-Inferior Dimension, 4-chamber view")	
LAAs(A4C)	(GEU-1005-51, 99GEMS, "LAAs(A4C)")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
LAESV(MOD A4C)	(G-0383, SRT, "Left Atrium Systolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LALd(A2C)	(GEU-1005-48, 99GEMS, "LALd(A2C)")	
LAAd(A2C)	(GEU-1005-47, 99GEMS, "LAAd(A2C)")	
LAEDV(A-L A2C)	(GEU-1005-53, 99GEMS, "LAEDV(A-L A2C)")	
LAEDV(MOD A2C)	(GEU-1005-55, 99GEMS, "LAEDV(MOD A2C)")	
LALs(A2C)	(GEU-1005-52, 99GEMS, "LALs(A2C)")	
LAAs(A2C)	(GEU-1005-50, 99GEMS, "LAAs(A2C)")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber")
LAESV(A-L A2C)	(G-0383, SRT, "Left Atrium Systolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LAESV(MOD A2C)	(G-0383, SRT, "Left Atrium Systolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
MM/LA/Ao	(17985-3, LN, "Left Atrium to Aortic Root Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
2D/LAEDV(A-L)	(122407, DCM, "Left Atrial End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125204, DCM, "Area-Length Biplane")
2D/LAESV(A-L)	(G-0383, SRT, "Left Atrium Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125204, DCM, "Area-Length Biplane")
LAEDV(MOD BP)	(122407, DCM, "Left Atrial End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
LAEDV Index(MOD BP)	(GEU-1005-170, 99GEMS, "Left Atrial End Diastolic Volume Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
LAESV(MOD BP)	(G-0383, SRT, "Left Atrium Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
LAESV Index(MOD BP)	(GEU-1005-171, 99GEMS, "Left Atrial End Systolic Volume Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")

17.2.7 Left Ventricle

TABLE 17- 32 LEFT VENTRICLE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
CO(A-L)	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(A-L A4C)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L A4C)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(MOD A4C)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CI(MOD A4C)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CO(A-L A2C)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L A2C)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(MOD A2C)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CI(MOD A2C)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CO(A-L LAX)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L LAX)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(MOD LAX)/AutoHR	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
CI(MOD LAX)/AutoHR	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVEDV(MOD BP)_03	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
LVESV(MOD BP)_03	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
EF(Biplane)_03	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
SV(Biplane)_03	(F-32120, SRT, "Stroke Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
SI(Biplane)_03	(F-00078, SRT, "Stroke Index")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
CO(Biplane)_03	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
CI(Biplane)_03	(F-32110, SRT, "Cardiac Index")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
ECG/HeartRate/Auto	(8867-4, LN, "Heart rate")	
2D/LV Major	(G-A193, SRT, "Major Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LV Minor	(G-A194, SRT, "Minor Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/IVSd	(18154-5, LN, "Interventricular Septum Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVIDd	(29436-3, LN, "Left Ventricle Internal End Diastolic Dimension")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVIDd Index	(GEU-1005-139, 99GEMS, "LVIDd Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVIDs	(29438-9, LN, "Left Ventricle Internal Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVIDs Index	(GEU-106-0029, 99GEMS, "Left Ventricle Internal Systolic Dimension Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVPWd	(18152-9, LN, "Left Ventricle Posterior Wall Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVPWs	(18156-0, LN, "Left Ventricle Posterior Wall Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/IVSs	(18158-6, LN, "Interventricular Septum Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVOT Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
2D/AVA/LVOT Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
2D/AVA/LVOT Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
2D/EDV(Teich)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/ESV(Teich)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/EF(Teich)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/EDV(Cube)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/ESV(Cube)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/EF(Cube)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/%FS	(18051-3, LN, "Left Ventricular Fractional Shortening")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/%IVS Thck	(18054-7, LN, "Interventricular Septum % Thickening")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/%LVPW Thck	(18053-9, LN, "Left Ventricle Posterior Wall % Thickening")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SV(Teich)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/SI(Teich)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/CO(Teich)	(F-32100, SRT, "Cardiac Output")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/CI(Teich)	(F-32110, SRT, "Cardiac Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
2D/SV(Cube)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/SI(Cube)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/CO(Cube)	(F-32100, SRT, "Cardiac Output")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/CI(Cube)	(F-32110, SRT, "Cardiac Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
2D/LVd Mass	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVd Mass Index	(GEU-1005-135, 99GEMS, "LVd Mass Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVs Mass	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVs Mass Index	(GEU-1005-137, 99GEMS, "LVs Mass Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVd Mass/ASE	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125221, DCM, "Left Ventricle Mass by M-mode")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/LVd Mass Index/ASE	(GEU-1005-136, 99GEMS, "LVd Mass Index/ASE")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVs Mass/ASE	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125221, DCM, "Left Ventricle Mass by M-mode")
2D/LVs Mass Index/ASE	(GEU-1005-138, 99GEMS, "LVs Mass Index/ASE")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVA diastole	(G-0375, SRT, "Left Ventricular Diastolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LVA systole	(G-0374, SRT, "Left Ventricular Systolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/LVA diastole	(G-0375, SRT, "Left Ventricular Diastolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/SAX/LVA systole	(G-0374, SRT, "Left Ventricular Systolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/SAX/LVAepi diastole	(59093-5, LN, "Epicardial Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/SAX/LVAepi systole	(59093-5, LN, "Epicardial Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/SAX/LVAend diastole	(59094-3, LN, "Endocardial Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/SAX/LVAend systole	(59094-3, LN, "Endocardial Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/LVOT Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
2D/EDV(A-L)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
2D/EDV(MOD)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
2D/ESV(A-L)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
2D/ESV(MOD)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
2D/EF(A-L)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/SV(A-L)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
2D/SI(A-L)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
2D/EF(MOD)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
2D/SV(MOD)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
2D/SI(MOD)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLd(A4C)	(18074-5, LN, "Left Ventricular Major Axis Diastolic Dimension, 4-chamber view")	
LVAd(A4C)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
LVEDV(A-L A4C)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
LVEDV(MOD A4C)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLs(A4C)	(18075-2, LN, "Left Ventricular Major Axis Systolic Dimension, 4-chamber view")	
LVAs(A4C)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
LVESV(A-L A4C)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
LVESV(MOD A4C)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
EF(A-L A4C)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
SV(A-L A4C)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")
SI(A-L A4C)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area- Length Single Plane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
CO(A-L A4C)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L A4C)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
EF(MOD A4C)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SV(MOD A4C)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SI(MOD A4C)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CO(MOD A4C)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CI(MOD A4C)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLd(A2C)	(18072-9, LN, "Left Ventricular Major Axis Diastolic Dimension, 2-chamber view")	
LVAd(A2C)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber")
LVEDV(A-L A2C)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LVEDV(MOD A2C)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLs(A2C)	(18073-7, LN, "Left Ventricular Major Axis Systolic Dimension, 2-chamber view")	
LVA(A2C)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber")
LVESV(A-L A2C)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LVESV(MOD A2C)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
EF(A-L A2C)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
SV(A-L A2C)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
SI(A-L A2C)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(A-L A2C)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L A2C)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
EF(MOD A2C)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SV(MOD A2C)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SI(MOD A2C)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CO(MOD A2C)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CI(MOD A2C)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-A19B, SRT, "Apical two chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLd(LAX)	(GEU-1005-57, 99GEMS, "LVLd(LAX)")	
LVAAd(LAX)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis")
LVEDV(A-L LAX)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LVEDV(MOD LAX)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVLs(LAX)	(GEU-1005-58, 99GEMS, "LVLs(LAX)")	
LVAAs(LAX)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
LVESV(A-L LAX)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LVESV(MOD LAX)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
EF(A-L LAX)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
SV(A-L LAX)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
SI(A-L LAX)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CO(A-L LAX)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
CI(A-L LAX)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
EF(MOD LAX)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SV(MOD LAX)	(F-32120, SRT, "Stroke Volume")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
SI(MOD LAX)	(F-00078, SRT, "Stroke Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CO(MOD LAX)	(F-32100, SRT, "Cardiac Output")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
CI(MOD LAX)	(F-32110, SRT, "Cardiac Index")	(111031, DCM, "Image View") = (G-0395, SRT, "Apical long axis") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
LVEDV(MOD BP)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
LVESV(MOD BP)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
EF(Biplane)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
SV(Biplane)	(F-32120, SRT, "Stroke Volume")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
SI(Biplane)	(F-00078, SRT, "Stroke Index")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
CO(Biplane)	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
CI(Biplane)	(F-32110, SRT, "Cardiac Index")	(G-C036, SRT, "Measurement Method") = (125207, DCM, "Method of Disks, Biplane")
LVLd(apical)	(18077-8, LN, "Left Ventricle diastolic major axis")	

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
LVLs(apical)	(18076-0, LN, "Left Ventricle systolic major axis")	
LVAd(sax MV)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-039A, SRT, "Parasternal short axis at the Mitral Valve level")
LVA(sax MV)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-039A, SRT, "Parasternal short axis at the Mitral Valve level")
LVAd(sax PM)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-039B, SRT, "Parasternal short axis at the Papillary Muscle level")
LVA(sax PM)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-039B, SRT, "Parasternal short axis at the Papillary Muscle level")
LVAd(sax epi)	(G-0379, SRT, "Left Ventricle Epicardial Diastolic Area, psax pap view")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
LVA(sax epi)	(59093-5, LN, "Epicardial Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-039B, SRT, "Parasternal short axis at the Papillary Muscle level")
LVAd(sax)	(G-0375, SRT, "Left Ventricular Diastolic Area")	(111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
LVA(sax)	(G-0374, SRT, "Left Ventricular Systolic Area")	(111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
EDV(mod sim)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
ESV(mod sim)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
EF(mod sim)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
SV(mod sim)	(F-32120, SRT, "Stroke Volume")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
SI(mod sim)	(F-00078, SRT, "Stroke Index")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
CO(mod sim)	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
CI(mod sim)	(F-32110, SRT, "Cardiac Index")	(G-C036, SRT, "Measurement Method") = (125227, DCM, "Modified Simpson")
EDV(bullet)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
ESV(bullet)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
EF(bullet)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
SV(bullet)	(F-32120, SRT, "Stroke Volume")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
SI(bullet)	(F-00078, SRT, "Stroke Index")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
CO(bullet)	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
CI(bullet)	(F-32110, SRT, "Cardiac Index")	(G-C036, SRT, "Measurement Method") = (125228, DCM, "Bullet Method")
EDV(bp el)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
ESV(bp el)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
EF(bp el)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
SV(bp el)	(F-32120, SRT, "Stroke Volume")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
SI(bp el)	(F-00078, SRT, "Stroke Index")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
CO(bp el)	(F-32100, SRT, "Cardiac Output")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
CI(bp el)	(F-32110, SRT, "Cardiac Index")	(G-C036, SRT, "Measurement Method") = (125211, DCM, "Biplane Ellipse")
LVd Mass(A-L)	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
LVs Mass(A-L)	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
MM/IVSd	(18154-5, LN, "Interventricular Septum Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/IVSs	(18158-6, LN, "Interventricular Septum Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVIDd	(29436-3, LN, "Left Ventricle Internal End Diastolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVIDd Index	(GEU-1005-88, 99GEMS, "LVIDd Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVIDs	(29438-9, LN, "Left Ventricle Internal Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVIDs Index	(GEU-1005-89, 99GEMS, "LVIDs Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVPWd	(18152-9, LN, "Left Ventricle Posterior Wall Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVPWs	(18156-0, LN, "Left Ventricle Posterior Wall Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/EDV(Teich)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/ESV(Teich)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/EF(Teich)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/SV(Teich)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/SI(Teich)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/CO(Teich)	(F-32100, SRT, "Cardiac Output")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")
MM/CI(Teich)	(F-32110, SRT, "Cardiac Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125209, DCM, "Teichholz")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MM/EDV(Cube)	(18026-5, LN, "Left Ventricular End Diastolic Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/ESV(Cube)	(18148-7, LN, "Left Ventricular End Systolic Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/EF(Cube)	(18043-0, LN, "Left Ventricular Ejection Fraction")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/SV(Cube)	(F-32120, SRT, "Stroke Volume")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/SI(Cube)	(F-00078, SRT, "Stroke Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/CO(Cube)	(F-32100, SRT, "Cardiac Output")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/CI(Cube)	(F-32110, SRT, "Cardiac Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125206, DCM, "Cube Method")
MM/%FS	(18051-3, LN, "Left Ventricular Fractional Shortening")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/IVSd/LVPWd	(18155-2, LN, "Interventricular Septum to Posterior Wall Thickness Ratio")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/%IVS Thck	(18054-7, LN, "Interventricular Septum % Thickening")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/%LVPW Thck	(18053-9, LN, "Left Ventricle Posterior Wall % Thickening")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVd Mass	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVs Mass	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVd Mass Index	(GEU-1005-140, 99GEMS, "LVd Mass Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVs Mass Index	(GEU-1005-142, 99GEMS, "LVs Mass Index")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVd Mass/ASE	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125221, DCM, "Left Ventricle Mass by M-mode")
MM/LVs Mass/ASE	(18087-7, LN, "Left Ventricle Mass")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C036, SRT, "Measurement Method") = (125221, DCM, "Left Ventricle Mass by M-mode")
MM/LVd Mass Index/ASE	(GEU-1005-141, 99GEMS, "LVd Mass Index/ASE")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVs Mass Index/ASE	(GEU-1005-143, 99GEMS, "LVs Mass Index/ASE")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/HeartRate	(8867-4, LN, "Heart rate")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVPEP	(18068-7, LN, "Left Ventricle Pre Ejection Period")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/LVET	(20222-6, LN, "Ejection Time")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MM/LVPEP/ET Ratio	(59088-5, LN, "Pre-Ejection Period/Ejection Time Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/Vcf mean	(59117-2, LN, "Mean Velocity of Circumferential Fiber Shortening (Mean VcFv)")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/Vcf mean (corr)	(59118-0, LN, "HR-Corrected Mean Velocity of Circumferential Fiber Shortening")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
IVCT	(G-037E, SRT, "Left Ventricular Isovolumic Contraction Time")	
IVRT	(18071-1, LN, "Left Ventricular Isovolumic Relaxation Time")	
MP/LVOT Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
MP/LVOT VTI	(20354-7, LN, "Velocity Time Integral")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT Vmax P	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT Vmean	(20352-1, LN, "Mean Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT meanPG	(20256-4, LN, "Mean Gradient")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT VTI	(20354-7, LN, "Velocity Time Integral")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT Env.Ti	(GEU-1005-32, 99GEMS, "LVOT Env.Ti")	
LVOT HR	(8867-4, LN, "Heart rate")	
LVOT SV	(F-32120, SRT, "Stroke Volume")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT SI	(F-00078, SRT, "Stroke Index")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT CO	(F-32100, SRT, "Cardiac Output")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVOT CI	(F-32110, SRT, "Cardiac Index")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT Vmax P	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT Vmean	(20352-1, LN, "Mean Velocity")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT meanPG	(20256-4, LN, "Mean Gradient")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT VTI	(20354-7, LN, "Velocity Time Integral")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT Env.Ti	(GEU-1005-33, 99GEMS, "AVA/LVOT Env.Ti")	
AVA/LVOT HR	(8867-4, LN, "Heart rate")	

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
AVA/LVOT SV	(F-32120, SRT, "Stroke Volume")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT SI	(F-00078, SRT, "Stroke Index")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT CO	(F-32100, SRT, "Cardiac Output")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AVA/LVOT CI	(F-32110, SRT, "Cardiac Index")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
LVET	(20222-6, LN, "Ejection Time")	
LVPEP	(18068-7, LN, "Left Ventricle Pre Ejection Period")	
LVPEP/ET Ratio	(GEU-1005-130, 99GEMS, "LVPEP/ET Ratio")	
LIMP	(G-037F, SRT, "Left Ventricular Index of Myocardial Performance")	
AP/LVOT Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
AP/LVOT VTI	(20354-7, LN, "Velocity Time Integral")	(G-C0E3, SRT, "Finding Site") = (T-32650, SRT, "Left Ventricular Outflow Tract")
ECG/R-R interval	(122182, DCM, "R-R interval")	
ECG/HeartRate	(8867-4, LN, "Heart rate")	
IMP/LVOT	(GEU-1005-147, 99GEMS, "IMP/LVOT")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")

17.2.8 Mitral Valve

TABLE 17- 33 MITRAL VALVE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MM/MV lft sep	(59109-9, LN, "Leaflet Separation")	
MM/MV D-E Slope	(59127-1, LN, "D-E Slope")	
MV Eann Velocity	(18037-2, LN, "Mitral Valve E-Wave Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-35313, SRT, "Mitral Annulus")
MR Acc Slope	(20269-7, LN, "Mitral Valve Acceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MR dp/dt	(18035-6, LN, "Mitral Regurgitation dp/dt derived from Mitral Regurgitation velocity")	
MVET	(20222-6, LN, "Ejection Time")	
PISA/MR/RF	(G-0390, SRT, "Regurgitant Fraction")	(G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
CFM/MR Signal Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-0373, SRT, "Image Mode") = (R-409E2, SRT, "Doppler Color Flow")
2D/MV Annulus Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-35313, SRT, "Mitral Annulus")
2D/EPSS	(18036-4, LN, "Mitral Valve EPSS, E wave")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/MVL Thck	(59110-7, LN, "Leaflet Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/SAX/MVA	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (111031, DCM, "Image View") = (G-0397, SRT, "Parasternal short axis")
2D/MVA Planimetry	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125220, DCM, "Planimetry")
2D/MV Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
MM/EPSS	(18036-4, LN, "Mitral Valve EPSS, E wave")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/MV D-E Excursion	(17997-8, LN, "Mitral Valve D-E Excursion")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/MV E-F Slope	(18040-6, LN, "Mitral Valve E-F Slope by M-Mode")	
MM/MV CE Dist	(59122-2, LN, "C-E Distance")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/MV E/A Ratio	(18038-0, LN, "Mitral Valve E to A Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MV Acc Time	(20168-1, LN, "Acceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV Acc Slope	(20269-7, LN, "Mitral Valve Acceleration Slope")	
MV A Dur	(G-0385, SRT, "Mitral Valve A-Wave Duration")	
MV E Velocity	(18037-2, LN, "Mitral Valve E-Wave Peak Velocity")	
MV E VTI	(GEU-1005-165, 99GEMS, "MV E VTI")	(R-4089A, SRT, "Cardiac Cycle Point") = (R-40B1B, SRT, "Early Diastole")
MV E Env.Ti	(GEU-1005-160, 99GEMS, "MV E Env.Ti")	
MV A Velocity	(17978-8, LN, "Mitral Valve A-Wave Peak Velocity")	
MV A VTI	(GEU-1005-164, 99GEMS, "MV A VTI")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32030, SRT, "Atrial Systole")
MV A Env.Ti	(GEU-1005-159, 99GEMS, "MV A Env.Ti")	
MV Dec Time	(20217-6, LN, "Deceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV PHT	(20280-4, LN, "Pressure Half-Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV Dec Slope	(20216-8, LN, "Deceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MVA (PHT)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125210, DCM, "Area by Pressure Half-Time")
MVA (VTI)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125215, DCM, "Continuity Equation by Velocity Time Integral")
MV meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MV Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV Env.Ti	(GEU-1005-157, 99GEMS, "MV Env.Ti")	
MV VTI	(GEU-1005-162, 99GEMS, "MV VTI")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
MV HR	(8867-4, LN, "Heart rate")	
MV SV	(F-32120, SRT, "Stroke Volume")	
MV SI	(F-00078, SRT, "Stroke Index")	
MV CO	(F-32100, SRT, "Cardiac Output")	
MV CI	(F-32110, SRT, "Cardiac Index")	
MV E/A Ratio	(18038-0, LN, "Mitral Valve E to A Ratio")	
MV E/A Ratio/Calc	(18038-0, LN, "Mitral Valve E to A Ratio")	
MV Acc Time/MV Dec Time	(G-0386, SRT, "Mitral Valve AT/DT Ratio")	
MV dp/dt	(59120-6, LN, "dP/dt by US")	
MV Time To Peak	(59121-4, LN, "Time to Peak by US")	
MR Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MR meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MR Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MR maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MR Env.Ti	(GEU-1005-158, 99GEMS, "MR Env.Ti")	
MR VTI	(GEU-1005-161, 99GEMS, "MR VTI")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
MP VTI	(GEU-1005-163, 99GEMS, "MP VTI")	
MP Area	(GEU-1005-132, 99GEMS, "MP Area")	
MCO	(G-0387, SRT, "Mitral Valve Closure to Opening Time")	
PISA/MR/Flow	(34141-2, LN, "Peak Instantaneous Flow Rate")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/MR/Radius	(GEU-1005-36, 99GEMS, "PISA/MR/Radius")	
PISA/MR/Velocit	(GEU-1005-37, 99GEMS, "PISA/MR/Velocit")	
PISA/MR/Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/MR/VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
PISA/MR/ERO	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/MR/RV	(33878-0, LN, "Volume Flow")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
MV Eprime Velocity	(59133-9, LN, "Peak Tissue Velocity")	(R-4089A, SRT, "Cardiac Cycle Point") = (R-40B1B, SRT, "Early Diastole") (G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed") (G-C0E3, SRT, "Finding Site") = (T-35313, SRT, "Mitral Annulus")
MV E/Eprime Ratio/Calc	(59111-5, LN, "E Velocity to Annulus E Velocity Ratio")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Aprime Velocity	(GEU-1005-133, 99GEMS, "MV Aprime Velocity")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
IMP/MCO	(G-0387, SRT, "Mitral Valve Closure to Opening Time")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Aprime Septal Velocity	(GEU-1005-174, 99GEMS, "TVD- A' Septal velocity(Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/A Septal Ratio	(GEU-1005-175, 99GEMS, "TVD- E'/A' septal ratio(Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Eprime Lateral Velocity	(GEU-1005-176, 99GEMS, "TVD- E' lateral velocity (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Aprime Lateral Velocity	(GEU-1005-177, 99GEMS, "TVD- A' lateral velocity (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/A Lateral Ratio	(GEU-1005-178, 99GEMS, "TVD- E'/A' lateral ratio (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Eprime Medial Velocity	(GEU-1005-179, 99GEMS, "TVD- E' medial velocity (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Aprime Medial Velocity	(GEU-1005-180, 99GEMS, "TVD- A' medial velocity (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/A Medial Ratio	(GEU-1005-181, 99GEMS, "TVD- E'/A' medial ratio (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV Eprime Average Velocity	(GEU-1005-182, 99GEMS, "TVD- E' Average Velocity(Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/Eprime Septal Ratio/Calc	(GEU-1005-183, 99GEMS, "TVD- Mitral Valve E velocity/E' Septal ratio(Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/Eprime Lateral Ratio/Calc	(GEU-1005-184, 99GEMS, "TVD- Mitral Valve E velocity/E' Lateral ratio (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/Eprime Average Ratio/Calc	(GEU-1005-185, 99GEMS, "TVD- Mitral Valve E velocity/E' Average ratio (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MV E/Eprime Medial Ratio/Calc	(GEU-1005-186, 99GEMS, "TVD- Mitral Valve E velocity/E' Medial ratio (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
MM/MAPSE	(GEU-1005-187, 99GEMS, "Mitral Annular Plane Systolic Excursion (MAPSE)")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")

17.2.9 Pulmonary Artery

TABLE 17- 34 PULMONARY ARTERY

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
PA Vmax	(11726-7, LN, "Peak Systolic Velocity")	
PA maxPG	(20247-3, LN, "Peak Gradient")	
2D/MPA	(18020-8, LN, "Main Pulmonary Artery Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RPA	(18021-6, LN, "Right Pulmonary Artery Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LPA	(18019-0, LN, "Left Pulmonary Artery Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/RPA area	(GEU-1005-73, 99GEMS, "LAX/RPA area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/LAX/LPA area	(GEU-1005-74, 99GEMS, "LAX/LPA area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/RPA area	(GEU-1005-76, 99GEMS, "SAX/RPA area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/SAX/LPA area	(GEU-1005-75, 99GEMS, "SAX/LPA area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
RPA Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-44200, SRT, "Right Pulmonary Artery")
RPA maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-44200, SRT, "Right Pulmonary Artery")
LPA Vmax	11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-44400, SRT, "Left Pulmonary Artery")
LPA maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-44400, SRT, "Left Pulmonary Artery")
MPA Vmax	(G-038A, SRT, "Main Pulmonary Artery Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (T-44000, SRT, " Pulmonary Artery")
MPA maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (T-44000, SRT, " Pulmonary Artery")
PAEDP	(GEU-1005-42, 99GEMS, "PAEDP")	

17.2.10 Pericardial cavity

TABLE 17- 35 PERICARDIAL CAVITY

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/PEs	(121206, DCM, "Distance")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (D3-90008, SRT, "Pericardial effusion")
2D/PEd	(121206, DCM, "Distance")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (D3-90008, SRT, "Pericardial effusion")
MM/PEd	(121206, DCM, "Distance")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C0E3, SRT, "Finding Site") = (D3-90008, SRT, "Pericardial effusion")

17.2.11 Pulmonic Valve

TABLE 17- 36 PULMONIC VALVE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
-----------------------	-------------------------------	---

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
CFM/PR Signal Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-0373, SRT, "Image Mode") = (R-409E2, SRT, "Doppler Color Flow")
2D/PV Annulus Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/PV Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
MM/Q-to-PV close	(20295-2, LN, "Time from Q wave to Pulmonic Valve Closes")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
PV Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV Vmax P	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV Acc Time	(20168-1, LN, "Acceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PV VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PVA (VTI)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125215, DCM, "Continuity Equation by Velocity Time Integral")
PVA (Vmax)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
PVA (Vmax)P	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
PV Env.Ti	(GEU-1005-39, 99GEMS, "PV Env.Ti")	
PV HR	(8867-4, LN, "Heart rate")	
PV SV	(GEU-1005-107, 99GEMS, "PV SV")	
PV SI	(GEU-1005-106, 99GEMS, "PV SI")	
PV CO	(GEU-1005-105, 99GEMS, "PV CO")	
PV CI	(GEU-1005-104, 99GEMS, "PV CI")	
PV Acc Slope	(20167-3, LN, "Acceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
PVET	(18042-2, LN, "Pulmonic Valve Ejection Time")	
SD/Q-to-PV close	(20295-2, LN, "Time from Q wave to Pulmonic Valve Closes")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
PV Acc Time/ET Ratio	(G-0388, SRT, "Ratio of Pulmonic Valve Acceleration Time to Ejection Time")	
PV dp/dt	(GEU-1005-153, 99GEMS, "PV dp/dt")	
PV Time To Peak	(GEU-1005-38, 99GEMS, "PV Time To Peak")	
PR PHT	(20280-4, LN, "Pressure Half-Time")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR Dec Time	(20217-6, LN, "Deceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR Dec Slope	(20216-8, LN, "Deceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PR Env.Ti	(59108-1, LN, "Envelope Duration")	
PR VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PRend Vmax	(18026-5, LN, "End Diastolic Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
PRend maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (R-4089A, SRT, "Cardiac Cycle Point") = (F-32011, SRT, "End Diastole")
PR dp/dt	(GEU-1005-154, 99GEMS, "PR dp/dt")	
PV A Dur-MV A Dur	(GEU-1005-124, 99GEMS, "PV A Dur-MV A Dur")	
PV A Dur/MV VTI	(GEU-1005-122, 99GEMS, "PV A Dur/MV VTI")	
PV A Dur/MV A Dur	(GEU-1005-123, 99GEMS, "PV A Dur/MV A Dur")	
PISA/PR/Flow	(34141-2, LN, "Peak Instantaneous Flow Rate")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/PR/Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/PR/VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/PR/ERO	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
PISA/PR/RV	(33878-0, LN, "Volume Flow")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PR HR	(8867-4, LN, "Heart rate")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")

17.2.12 Pulmonary Venous Structure

TABLE 17- 37 PULMONARY VENOUS STRUCTURE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
P_Vein S	(29450-4, LN, "Pulmonary Vein Systolic Peak Velocity")	
P_Vein D	(29451-2, LN, "Pulmonary Vein Diastolic Peak Velocity")	
P_Vein A	(29453-8, LN, "Pulmonary Vein Atrial Contraction Reversal Peak Velocity")	
P_Vein A Dur	(G-038B, SRT, "Pulmonary Vein A-Wave Duration")	
P_Vein S/D Ratio	(29452-0, LN, "Pulmonary Vein Systolic to Diastolic Ratio")	
P_Vein S VTI	(G-038C, SRT, "Pulmonary Vein S-Wave VTI")	
P_Vein S Env.Ti	(59108-1, LN, "Envelope Duration")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole")
P_Vein D VTI	(G-038D, SRT, "Pulmonary Vein D-Wave VTI")	
P_Vein D Env.Ti	(59108-1, LN, "Envelope Duration")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole")

17.2.13 Right Atrium

TABLE 17- 38 RIGHT ATRIUM

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/RA	(M-02550, SRT, "Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RAD Major	(G-A193, SRT, "Major Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RAD Minor	(G-A194, SRT, "Minor Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RAA diastole	(GEU-1005-70, 99GEMS, "RAA diastole")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RAA systole	(17988-7, LN, "Right Atrium Systolic Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RA Area	(GEU-1005-134", 99GEMS, "RA Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
RALd(A4C)	(29466-0, LN, "Right Atrium Superior-Inferior Dimension, 4-chamber view")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole")
RAAd(A4C)	(17988-7, LN, "Right Atrium Area A4C view")	
RAEDV(A-L A4C)	(GEU-1005-65, 99GEMS, "RAEDV(A-L A4C)")	
RAEDV(MOD A4C)	(GEU-1005-66, 99GEMS, "RAEDV(MOD A4C)")	
RALs(A4C)	(29466-0, LN, "Right Atrium Superior-Inferior Dimension, 4-chamber view")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole")
RAAs(A4C)	(17988-7, LN, "Right Atrium Systolic Area")	
RAESV(A-L A4C)	(GEU-1005-67, 99GEMS, "RAESV(A-L A4C)")	
RAESV(MOD A4C)	(GEU-1005-68, 99GEMS, "RAESV(MOD A4C)")	

RAP	(18070-3, LN, "Right Atrium Systolic Pressure")	
-----	---	--

17.2.14 Right Ventricle

TABLE 17- 39 RIGHT VENTRICLE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MM/RVOT	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode") (G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
2D/RVOT Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
2D/RVAWd	(18153-7, LN, "Right Ventricular Anterior Wall Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVAWs	(18157-8, LN, "Right Ventricular Anterior Wall Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVIDd	(20304-2, LN, "Right Ventricular Internal Diastolic Dimension")	
2D/RVIDs	(20305-9, LN, "Right Ventricular Internal Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVD Major	(G-A193, SRT, "Major Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVD Minor	(G-A194, SRT, "Minor Axis")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVA diastole	(GEU-1005-71, 99GEMS, "RVA diastole")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVA systole	(GEU-1005-72, 99GEMS, "RVA systole")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
2D/RVOT Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVld(A4C)	(18078-6, LN, "Right Ventricular Major Axis Diastolic Dimension, 4-chamber view")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
RVAd(A4C)	(G-A166, SRT, "Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32010, SRT, "Diastole") (111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
RVEDV(A-L A4C)	(8822-3, LN, "Right Ventricular ED Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")
RVEDV(MOD A4C)	(8822-3, LN, "Right Ventricular ED Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
RVls(A4C)	(18079-4, LN, "Right Ventricular Major Axis Systolic Dimension, 4-chamber view")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
RVAs(A4C)	(G-A166, SRT, "Area")	(R-4089A, SRT, "Cardiac Cycle Point") = (F-32020, SRT, "Systole") (111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber")
RVESV(A-L A4C)	(8824-5, LN, "Right Ventricular ES Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125205, DCM, "Area-Length Single Plane")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
RVESV(MOD A4C)	(8824-5, LN, "Right Ventricular ES Volume")	(111031, DCM, "Image View") = (G-A19C, SRT, "Apical four chamber") (G-C036, SRT, "Measurement Method") = (125208, DCM, "Method of Disks, Single Plane")
MM/RVIDd	(20304-2, LN, "Right Ventricular Internal Diastolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVIDs	(20305-9, LN, "Right Ventricular Internal Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVIDs	(20305-9, LN, "Right Ventricular Internal Systolic Dimension")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVAWd	(18153-7, LN, "Right Ventricular Anterior Wall Diastolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVAWs	(18157-8, LN, "Right Ventricular Anterior Wall Systolic Thickness")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVPEP	(20301-8, LN, "Right Ventricle Pre Ejection Period")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVET	(20222-6, LN, "Ejection Time")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
MM/RVPEP/ET Ratio	(59088-5, LN, "Pre-Ejection Period/Ejection Time Ratio")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
Est RVSP	(G-0380, SRT, "Right Ventricular Peak Systolic Pressure")	
RVOT Vmax	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT Vmax P	(11726-7, LN, "Peak Velocity")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT maxPG	(20247-3, LN, "Peak Gradient")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT Vmean	(20352-1, LN, "Mean Velocity")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT meanPG	(20256-4, LN, "Mean Gradient")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT VTI	(20354-7, LN, "Velocity Time Integral")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT Env.Ti	(59108-1, LN, "Envelope Duration")	
RVOT HR	(8867-4, LN, "Heart rate")	
RVOT SV	(F-32120, SRT, "Stroke Volume")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT SI	(F-00078, SRT, "Stroke Index")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT CO	(F-32100, SRT, "Cardiac Output")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVOT CI	(F-32110, SRT, "Cardiac Index")	(G-C0E3, SRT, "Finding Site") = (G-039D, SRT, "Right Ventricular Outflow Tract")
RVPEP	(20301-8, LN, "Right Ventricle Pre Ejection Period")	
RVET	(20222-6, LN, "Ejection Time")	
RVPEP/ET Ratio	(GEU-1005-129, 99GEMS, "RVPEP/ET Ratio")	
RIMP	(G-0381, SRT, "Right Ventricular Index of Myocardial Performance")	
IMP/RVOT	(GEU-1005-148, 99GEMS, "IMP/RVOT")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")

17.2.15 Tricuspid Valve

TABLE 17- 40 TRICUSPID VALVE

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
CFM/TR Signal Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-0373, SRT, "Image Mode") = (R-409E2, SRT, "Doppler Color Flow")
2D/TV Annulus Diam	(G-038F, SRT, "Cardiovascular Orifice Diameter")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-35111, SRT, "Tricuspid Annulus")
2D/TV Area	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")
MM/Q-to-TV open	(20296-0, LN, "Time from Q wave to Tricuspid Valve Opens")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
TV Acc Time	(20168-1, LN, "Acceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV Acc Slope	(20343-0, LN, "Tricuspid Valve Acceleration Slope")	
TV E Velocity	(18031-5, LN, "Tricuspid Valve E Wave Peak Velocity")	
TV A Velocity	(18030-7, LN, "Tricuspid Valve A Wave Peak Velocity")	
TV Dec Time	(20217-6, LN, "Deceleration Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV Dec Slope	(20216-8, LN, "Deceleration Slope")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV PHT	(20280-4, LN, "Pressure Half-Time")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TVA	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV Vmax P	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TV VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow")
TVA (VTI)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125215, DCM, "Continuity Equation by Velocity Time Integral")
TVA (Vmax)	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
TVA (Vmax)P	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-C036, SRT, "Measurement Method") = (125214, DCM, "Continuity Equation by Peak Velocity")
TV HR	(8867-4, LN, "Heart rate")	
TV E/A Ratio	(18039-8, LN, "Tricuspid Valve E to A Ratio")	
TV Acc Time/TV Dec Time	(GEU-1005-144, 99GEMS, "TV Acc Time/TV Dec Time")	
SD/Q-to-TV open	(20296-0, LN, "Time from Q wave to Tricuspid Valve Opens")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
TR meanPG	(20256-4, LN, "Mean Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
TR Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
TR Vmean	(20352-1, LN, "Mean Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
TR maxPG	(20247-3, LN, "Peak Gradient")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
TR VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow")
TCO	(G-0389, SRT, "Tricuspid Valve Closure to Opening Time")	
PISA/TR/Flow	(34141-2, LN, "Peak Instantaneous Flow Rate")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/TR/Vmax	(11726-7, LN, "Peak Velocity")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/TR/VTI	(20354-7, LN, "Velocity Time Integral")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/TR/ERO	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
PISA/TR/RV	(33878-0, LN, "Volume Flow")	(G-C048, SRT, "Direction of Flow") = (R-42E61, SRT, "Regurgitant Flow") (G-C036, SRT, "Measurement Method") = (125216, DCM, "Proximal Isovelocity Surface Area")
2D/TVA Planimetry	(G-038E, SRT, "Cardiovascular Orifice Area")	(G-C048, SRT, "Direction of Flow") = (R-42047, SRT, "Antegrade Flow") (G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C036, SRT, "Measurement Method") = (125220, DCM, "Planimetry")
TV A Dur	(59105-7, LN, "A-Wave Duration")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
IMP/TCO	(G-0389, SRT, "Tricuspid Valve Closure to Opening Time")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")
RV Sprime Velocity	(GEU-1005-172, 99GEMS, "TVD- S' velocity (Tissue Velocity Doppler)")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed")

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
MM/TAPSE	(77903-3, LN, "Tricuspid Annular Plane Systolic Excursion (TAPSE)")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")

17.2.16 Vena Cava

TABLE 17- 41 VENA CAVA

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/IVC Diam Ins	(18006-7, LN, "Inferior Vena Cava Diameter")	(R-40899, SRT, "Respiratory Cycle Point") = (F-20010, SRT, "Inspiration")
2D/IVC Diam Exp	(18006-7, LN, "Inferior Vena Cava Diameter")	(R-40899, SRT, "Respiratory Cycle Point") = (F-20020, SRT, "Expiration")
2D/SVC Diam Ins	(18007-5, LN, "Superior Vena Cava Diameter")	(R-40899, SRT, "Respiratory Cycle Point") = (F-20010, SRT, "Inspiration")
2D/SVC Diam Exp	(18007-5, LN, "Superior Vena Cava Diameter")	(R-40899, SRT, "Respiratory Cycle Point") = (F-20020, SRT, "Expiration")
2D/IVC	(18006-7, LN, "Inferior Vena Cava Diameter")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode")

17.2.17 General Anatomy


TABLE 17- 42 GENERAL ANATOMY

Name of GEU parameter	Base Measurement Concept Name	Concept or Acquisition Context Modifier
2D/LCA	(121206, DCM, "Distance")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-43107, SRT, " Left Main Coronary Artery")
2D/RCA	(121206, DCM, "Distance")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-43203, SRT, " Right Coronary Artery")
2D/LCX	(121206, DCM, "Distance")	(G-0373, SRT, "Image Mode") = (G-03A2, SRT, "2D mode") (G-C0E3, SRT, "Finding Site") = (T-43120, SRT, " Circumflex Coronary Artery")
MM/R-R interval	(122182, DCM, "R-R interval")	(G-0373, SRT, "Image Mode") = (G-0394, SRT, "M mode")
IMP/LIMP	(GEU-1005-145, 99GEMS, "IMP/LIMP")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed") Left, Interaction Between Intramyocardial Pressure
IMP/RIMP	(GEU-1005-146, 99GEMS, "IMP/RIMP")	(G-0373, SRT, "Image Mode") = (R-409E4, SRT, "Doppler Pulsed") Right, Interaction Between Intramyocardial Pressure

17.3 OB-GYN ULTRASOUND PROCEDURE REPORT

This section describes the contents of the OB-GYN Ultrasound Procedure Report (TID 5000) SR.

17.3.1 OB-GYN Template Structure

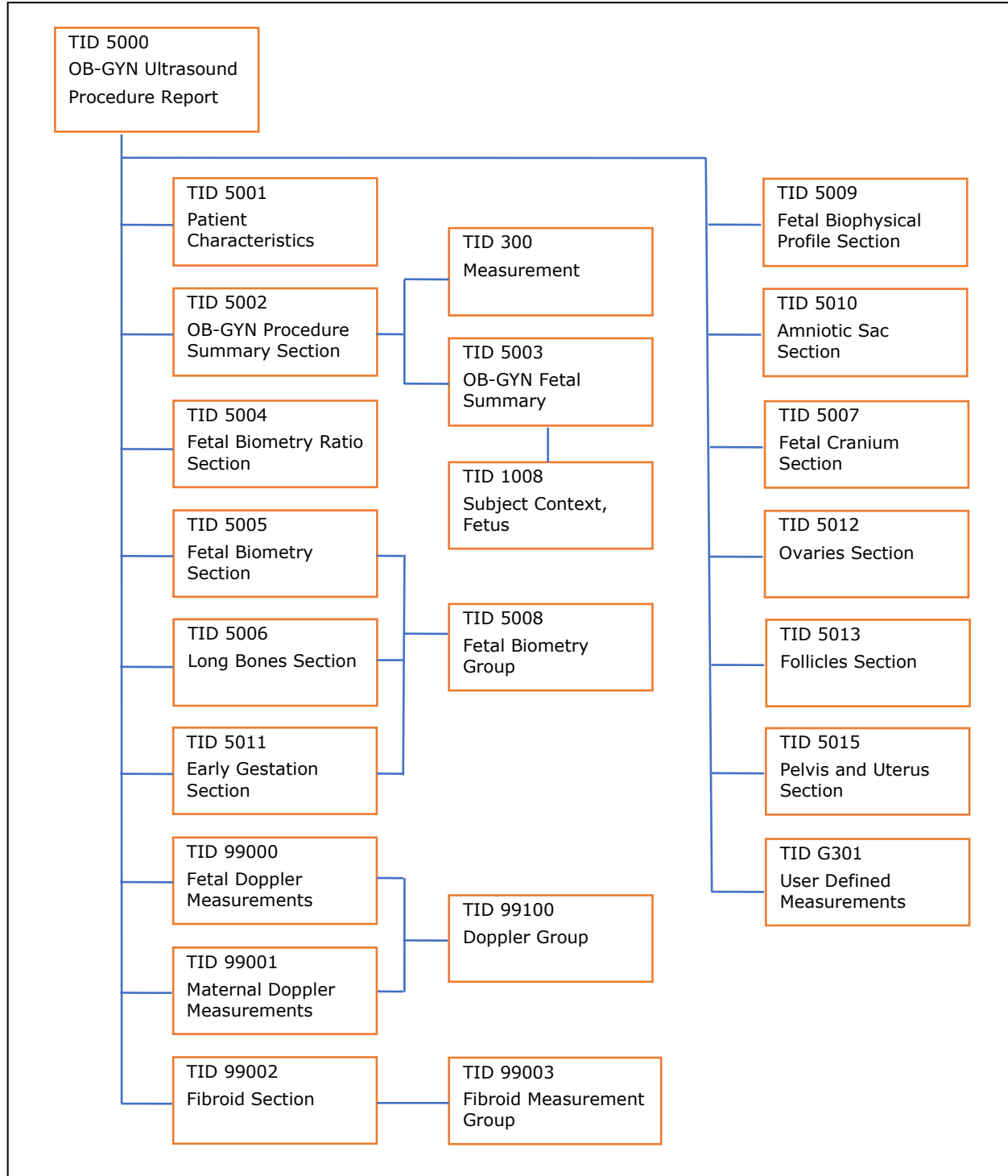

17.3.2 TID 5000 Template

LOGIQ Scanner supports the following template TID 5000 for SOP Instances created by this product when exam types selected are OB and GYN.

TABLE 17- 43 TID 5000

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (125000, DCM, 'OB-GYN Ultrasound Procedure Report')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1001) Observation Context	1	M		
3	>	CONTAINS	INCLUDE	DTID (5001) Patient Characteristics	1	U		
4	>	CONTAINS	INCLUDE	DTID (5002) OB-GYN Procedure Summary Section	1	U		
5	>	CONTAINS	INCLUDE	DTID (5004) Fetal Biometry Ratio Section	1-n	U		
6	>	CONTAINS	INCLUDE	DTID (5005) Fetal Biometry Section	1-n	U		
7	>	CONTAINS	INCLUDE	DTID (5006) Long Bones Section	1-n	U		
8	>	CONTAINS	INCLUDE	DTID (5007) Fetal Cranium Section	1-n	U		
9	>	CONTAINS	INCLUDE	DTID (5009) Fetal Biophysical Profile Section	1-n	U		
10	>	CONTAINS	INCLUDE	DTID (5011) Early Gestation Section	1-n	U		
11	>	CONTAINS	INCLUDE	DTID (5010) Amniotic Sac Section	1-n	U		
12	>	CONTAINS	INCLUDE	DTID (5015) Pelvis and Uterus Section	1-n	U		
13	>	CONTAINS	INCLUDE	DTID (5012) Ovaries Section	1-n	U		
14	>	CONTAINS	INCLUDE	DTID (5013) Follicles Section	1-n	U		\$Laterality = EV (G-A101, SRT, 'Left') \$Number = EV (11879-4, LN, 'Number of follicles in left ovary')
14	>	CONTAINS	INCLUDE	DTID (5013) Follicles Section	1-n	U		\$Laterality = EV (G-A100, SRT, 'Right') \$Number = EV (11879-4, LN, 'Number of follicles in left ovary')
15	>	CONTAINS	INCLUDE	DTID (99000) Fetal Doppler Measurements	1-n	U		\$Laterality \$FindingSite
16	>	CONTAINS	INCLUDE	DTID (99001) Maternal Doppler Measurement	1-n	U		\$Laterality \$FindingSite
17	>	CONTAINS	INCLUDE	DTID (99002) Fibroid Section	1	U		\$Number = EV (GEU-99009-97, 99GEMS, "Number of fibroids in ovary")

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
18	>	CONTAINS	INCLUDE	DTID (5013) SonoAVC Follicle Section	1-n			\$Laterality = EV (G- A101, SRT, 'Left')
19	>	CONTAINS	INCLUDE	DTID (5013) SonoAVC Follicle Section	1-n			\$Laterality = EV (G- A100, SRT, 'Right')
20	>	CONTAINS	INCLUDE	DTID (99060) Fetal Anatomy Section	1-n	U		
21	>	CONTAINS	INCLUDE	DTID (G301) GE Ultrasound User Defined Measurement	1-n	UC		

17.3.3 TID 5001 OB-GYN Patient Characteristics

TABLE 17- 44 TID 5001

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition / Notes
1			CONTAINER	EV (12118, DCM, 'Patient Characteristics')		M	
2	>	CONTAINS	TEXT	EV (121106, DCM,'Comment')	1	U	
3	>	CONTAINS	NUM	EV (8302-2, LN, 'Patient Height')	1	U	
4	>	CONTAINS	NUM	EV (11996-6, LN, 'Gravida')	1	U	See GEU SR Mapping Table xvi
5	>	CONTAINS	NUM	EV (11977-6, LN,'Para')	1	U	See GEU SR Mapping Table xvi
6	>	CONTAINS	NUM	EV (11612-9,LN,'Aborta')	1	U	See GEU SR Mapping Table xvi
7	>	CONTAINS	NUM	EV (33065-4, LN, 'Ectopic Pregnancies')	1	U	See GEU SR Mapping Table xvi

17.3.4 GEU SR Mapping Table for Patient Characteristics:

TABLE 17- 45 GEU SR MAPPING TABLE FOR PATIENT CHARACTERISTICS

Name of GEU parameter	Base Measurement Concept Name
Gravida	(11996-6, LN, 'Gravida')
Para	(11977-6, LN, 'Para')
Abortion	(11612-9, LN, 'Aborta')
Ectopic	(33065-4, LN, 'Ectopic Pregnancies')

17.3.5 TID 5002 OB-GYN Procedure Summary Section

TABLE 17- 46 TID 5002

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT(121111, DCM, 'Summary')	1	M		
2	>	CONTAINS	DATE	DCID (12003) OB-GYN	1-n	U		

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
				Dates				
3	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement=BCID (12018) OB-GYN Summary
4	>	CONTAINS	TEXT	EV (121106, DCM, 'Comment')	1-n	U		
5	>	CONTAINS	TEXT	EV (GEU-1001-31, GEU, 'Indications')	1-n	UC	No more than 1 inclusion per fetus	
6	>	CONTAINS	INCLUDE	BTID (5003) OB-GYN Fetus Summary	1-n	UC	No more than 1 inclusion per fetus	

17.3.6 Usage of TID 5003 OB-GYN Fetus Summary

TABLE 17- 47 TID 5003

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125008, DCM, 'Fetus Summary')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	To describe n fetus, invokes this template n times	
3	>	CONTAINS	TEXT	EV (121106, DCM, 'Comment')	1-n	U		
4	>	CONTAINS	INCLUDE	DTID (300) Measurements	1-n	U		\$Measurement = DCID (12002) OB-GYN Fetus Summary \$Equation = DCID (12012) OB Equations and Tables

17.3.7 GEU Parameters Map from Context ID 12019 OB-GYN Fetus Summary

TABLE 17- 48 CONTEXT ID 12019

Name of GEU parameter	Base Measurement Concept Name
OB/CUA	(11888-5, LN, Composite Ultrasound Age)
OB/AUA	(11884-4, LN, Average Ultrasound Age)
EFW(AC,BPD)	(11727-5, LN, 'Estimated Weight')
EFW(Hadlock)-GP	(GEU-1009-10, 99GEMS, "EFW(Hadlock)-GP")
EFW(Brenner)-GP	(GEU-1009-11, 99GEMS, "EFW(Brenner)-GP")
EFW(Williams)-GP	(GEU-1009-12,99GEMS, "EFW(Williams)-GP")
EFW, JSUM 2001	(GEU-1009-15,99GEMS, " EFW, JSUM 2001")
2 Sigma Upper Value of population	(GEU-1009- 16,99GEMS, "2 Sigma Upper Value of population")
2 Sigma Lower Value of population	(GEU-1009-17,99GEMS, "2 Sigma Lower Value of population")
BPD, JSUM 2001	(GEU-1009-18,99GEMS, " BPD, JSUM 2001")
2 Sigma Upper Value of population	(GEU-1009-19,99GEMS, "2 Sigma Upper Value of population")

Name of GEU parameter	Base Measurement Concept Name
2 Sigma Lower Value of population	(GEU-1009-20,99GEMS, "2 Sigma Lower Value of population")
AC, JSUM 2001	(GEU-1009-21,99GEMS, "AC, JSUM 2001")
2 Sigma Upper Value of population	(GEU-1009-22,99GEMS, "2 Sigma Upper Value of population")
2 Sigma Lower Value of population	(GEU-1009-23,99GEMS, "2 Sigma Lower Value of population")
AC, JSUM 2009	(GEU-1009-24,99GEMS, "AC, JSUM 2009")
2 Sigma Upper Value of population	(GEU-1009-25,99GEMS, "2 Sigma Upper Value of population")
2 Sigma Lower Value of population	(GEU-1009-26,99GEMS, "2 Sigma Lower Value of population")
FL, JSUM 2001	(GEU-1009-27,99GEMS, "FL, JSUM 2001")
2 Sigma Upper Value of population	(GEU-1009-28,99GEMS, "2 Sigma Upper Value of population")
2 Sigma Lower Value of population	(GEU-1009-29,99GEMS, "2 Sigma Lower Value of population")
CRL, JSUM 2001	(GEU-1009-30,99GEMS, "CRL, JSUM 2001")
FL, ASUM 2000	(GEU-1009-31,99GEMS, "FL, ASUM 2000")
1 Sigma Upper Value of population	(GEU-1009-32,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-33,99GEMS, "1 Sigma Lower Value of population")
1 Sigma Upper Value of population	(GEU-1009-34,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-35,99GEMS, "1 Sigma Lower Value of population")
1 Sigma Upper Value of population	(GEU-1009-36,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-37,99GEMS, "1 Sigma Lower Value of population")
1 Sigma Upper Value of population	(GEU-1009-38,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-39,99GEMS, "1 Sigma Lower Value of population")
Binocular Distance	(GEU-1009-40,99GEMS, "Binocular Distance")
BD, Jeanty 1982	(GEU-1009-41,99GEMS, "BD, Jeanty 1982")
FTA, Osaka 1989	(GEU-1009-42,99GEMS, "FTA, Osaka 1989")
1 Sigma Upper Value of population	(GEU-1009-43,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-44,99GEMS, "1 Sigma Lower Value of population")
EFW, Osaka 1989	(GEU-1009-45,99GEMS, "EFW, Osaka 1989")
1 Sigma Upper Value of population	(GEU-1009-46,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-47,99GEMS, "1 Sigma Lower Value of population")
SL, Tokyo 1986	(GEU-1009-48,99GEMS, "SL, Tokyo 1986")
1 Sigma Upper Value of population	(GEU-1009-49,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-50,99GEMS, "1 Sigma Lower Value of population")
APTDxTTD	(GEU-1009-51,99GEMS, "APTDxTTD")
AxT, Shinozuka 1996	(GEU-1009-52,99GEMS, "AxT, Shinozuka 1996")
1 Sigma Upper Value of population	(GEU-1009-53,99GEMS, "1 Sigma Upper Value of population")
1 Sigma Lower Value of population	(GEU-1009-54,99GEMS, "1 Sigma Lower Value of population")
Ultrasound EDD(CUA)	(GEU-1009-04,99GEMS,"EDD from composite ultrasound age")
Ultrasound EDD(AUA)	(11781-2, LN, "EDD from average ultrasound age")
Clinical Age(LMP)	(11885-1, LN, "Gestational Age by LMP")
Clinical Age(EDD)	(GEU-1009-05,99GEMS,"Gestational Age by EDD")
OB/Shepard/EFW(AC,BPD)	(11739-0, LN, "EFW by AC, BPD, Shepard 1982")

Name of GEU parameter	Base Measurement Concept Name
OB/Hadlock/EFW(AC,FL)	(11750-7, LN, "EFW by AC, FL, Hadlock 1984")
OB/Hadlock/EFW(AC,BPD,FL)	(11735-8, LN, "EFW by AC, BPD, FL, Hadlock 1985")
OB/Hadlock/EFW(AC,FL,HC)	(11746-5, LN, "EFW by AC, FL, HC, Hadlock 1985")
OB/Hadlock/EFW(AC,BPD,FL,HC)	(11732-5, LN, "EFW by AC, BPD, FL, HC, Hadlock 1985")
OB/Hadlock/EFW(AC,HC)	(11754-9, LN, "EFW by AC, HC, Hadlock 1984")

17.3.8 Usage of TID 5004 Fetus Biometry Ratio Section

TABLE 17- 49 TID 5004

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125001, DCM, 'Fetus Biometry Ratios')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	NUM	DCID (12004) Fetal Biometry Ratios	1-n	M		
4	>>	R-INFERRED FROM	NUM		2	U		
5	>>	HAS PROPERTIES	INCLUDE	DTID (312) Normal Range Properties	1	U		

17.3.9 Usage of TID 5005 Fetus Biometry Section

TABLE 17- 50 TID 5005

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125002, DCM, 'Fetus Biometry')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	INCLUDE	DTID (5008) Fetal Biometry Group	1-n	M		\$BiometryType = MemberOf {DCID (12005) Fetal Biometry Measurements}

17.3.10 GEU Parameter Map from Context ID 12005 Fetal Biometry Measurements

TABLE 17- 51 CONTEXT ID 12005

Name of GEU parameter	Base Measurement Concept Name
BPD	(11820-8, LN, 'Biparietal Diameter')
AC	(11979-2, LN, 'Abdominal Circumference')
OFD	(11851-3, LN, 'Occipital-Frontal Diameter')
FL	(11963-6, LN, 'Femur Length')
HC	(11984-2, LN, 'Head Circumference')
TAD	(11862-0, LN, 'Transverse Abdominal Diameter')
TC	(11863-8, LN, 'Trans Cerebellar Diameter')
TTD	(11864-6, LN, 'Transverse Thoracic Diameter')
Ft	(11965-1, LN, 'Foot length')
FTA	(33068-8, LN, 'Thoracic Area')
APTD	(11819-0, LN, 'Anterior-Posterior Trunk Diameter')
CM	(11860-4, LN, 'Cisterna Magna length')
NT	(33069-6, LN, 'Nuchal Translucency')
NFT	(12146-7, LN, 'Nuchal Fold Thickness')
APAD	(11818-2, LN, 'Anterior-Posterior Abdominal Diameter')
TC	(11988-3, LN, 'Thoracic Circumference')
BD	(GEU-1009-40, 99GEMS, 'Binocular Distance')
AxT	(GEU-1009-51, 99GEMS, 'APTDxTTD')
BOD	(GEU-1009-65, 99GEMS, 'Biorbital Diameter')
NBL	(GEU-1009-73, 99GEMS, 'Nasal Bone Length')

17.3.11 Usage of TID 5006 Long Bone Section

TABLE 17- 52 TID 5006

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125003, DCM, 'Fetus Long Bones)	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	INCLUDE	DTID (5008) Fetal Biometry Group	1-n	M		\$BiometryType = MemberOf {DCID (12006) Fetal Long Bones Biometry Measurements}

17.3.12 GEU Parameter Map from Context ID 12006 Fetal Long Bones Measurement

TABLE 17- 53 CONTEXT ID 12006

Name of GEU parameter	Base Measurement Concept Name
FL	(11963-6, LN, 'Femur Length')

Name of GEU parameter	Base Measurement Concept Name
HL	(11966-9, LN, 'Humerus length')
ULNA	(11969-3, LN, 'Ulna length')
TIB	(11968-5, LN, 'Tibia length')
FIB	(11964-4, LN, 'Fibula length')
Radius	(11967-7, LN, 'Radius length')
CLA	(11962-8, LN, 'Clavicle length')

17.3.13 Usage of TID 5007 Fetus Cranium Section

TABLE 17- 54 TID 5007

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125004, DCM, 'Fetus Cranium')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	INCLUDE	DTID (5008) Fetal Biometry Group	1-n	M		\$BiometryType = MemberOf {DCID (12007) Fetal Cranium}

17.3.14 Usage of TID 5008 Fetus Biometry Group

TABLE 17- 55 TID 5008

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125005, DCM, 'Biometry Group')	1	M		
2	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	MC	At least one of row 2 and 3 shall be present	\$Measurement - \$Biometry Type \$Derivation = DCID (3627) Measurement Type
3	>	CONTAINS	NUM	EV (18185-9, LN, "Gestation Age")	1	MC	At least one of row 2 and 3 shall be present	Units = EV (d, UCUM, days) Note: If configured, the Gestational Age will be sent in WeekDay format.
4	>>	INFERRED FROM	CODE	DCID (228) Equation or Table	1	U		DCID (12013) Gestation Age Equation and Tables
5	>>	R-INFERRED	NUM		1-n	U		

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
		FROM						
6	>>	HAS PROPERTIES	NUM	DCID (226) Population Statistical Descriptors	1-n	U		
7	>	CONTAINS	NUM	DTID (12017) Growth Distribution Rank	1	U		
8	>>	INFERRED FROM	CODE	DCID (228) Equation or Table	1	U		\$Equation = DCID (12015) Fetal Growth Equations and Tables

17.3.15 GEU Parameter Map from Context ID 12004 Fetal Biometry Ratios

TABLE 17- 56 CONTEXT ID 12004

Name of GEU parameter	Base Measurement Concept Name
Ratio HC/AC	(11947-9, LN, 'HC/AC')
Ratio FL/BPD	(11872-9, LN, 'FL/BPD')
Ratio FL/AC	(11871-1, LN, 'FL/AC')
Ratio FL/HC	(11873-7, LN, 'FL/HC')
CI	(11823-2, LN, 'Cephalic Index')
OB/Hadlock/EFW-GP	(GEU-1009-10, 99GEMS, "EFW(Hadlock)-GP")
OB/Brenner/EFW-GP	(GEU-1009-11, 99GEMS, "EFW(Brenner)-GP")
OB/Williams/EFW-GP	(GEU-1009-12, 99GEMS, "EFW(Williams)-GP")
OB/Kramer(f)/EFW-GP	(GEU-1009-13, 99GEMS, "EFW(Kramer(f))-GP")
OB/Kramer(m)/EFW-GP	(GEU-1009-14, 99GEMS, "EFW(Kramer(m))-GP")
OB/WHO/EFW-GP	(GEU-1009-55, 99GEMS, "EFW(WHO)-GP")
OB/Alexander/EFW-GP	(GEU-1009-62, 99GEMS, "EFW(Alexander)-GP")
OB/Doubilet/EFW-GP	(GEU-1009-64, 99GEMS, "EFW(Doubilet)-GP")
OB/Yarkoni/EFW-GP	(GEU-1009-72, 99GEMS, "EFW(Yarkoni)-GP")

17.3.16 Usage of TID 5009 Fetal Biophysical Profile Section

TABLE 17- 57 TID 5009

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125006, DCM, 'Biophysical Profile')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	NUM	EV (11631-9, LN, "Gross Body Movement")	1	MC	At least on of row 3-7 shall be present	Units = DT ("(0,2)", UCUM, "range 0:2")
4	>	CONTAINS	NUM	EV (11632-7, LN, "Fetal Breathing")	1	MC	At least on of row 3-7 shall be present	Units = DT ("(0,2)", UCUM, "range 0:2")

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
5	>	CONTAINS	NUM	EV (11635-0, LN, "Fetal Tone")	1	MC	At least on of row 3-7 shall be present	Units = DT ("(0,2)", UCUM, "range 0:2")
6	>	CONTAINS	NUM	EV (11635-5, LN, "Fetal Heart Reactivity")	1	MC	At least on of row 3-7 shall be present	Units = DT ("(0,2)", UCUM, "range 0:2")
7	>	CONTAINS	NUM	EV (11630-1, LN, "Amniotic Fluid Volume")	1	MC	At least on of row 3-7 shall be present	Units = DT ("(0,2)", UCUM, "range 0:2")
8	>	CONTAINS	NUM	DT (11634-3, LN, "Biophysical Profile Sum Score")	1	U		

17.3.17 GEU Parameters Map from TID 5009 Fetal Biophysical Profile Section

TABLE 17- 58 GEU PARAMETERS MAP FROM TID 5009

Name of GEU parameter	Base Measurement Concept Name
OB/Biophysical/movement	(11631-9, LN, 'Gross Body Movement')
OB/Biophysical/tone	(11635-0, LN, 'Fetal Tone')
OB/Biophysical/breathing	(11632-7, LN, 'Fetal Breathing')
OB/Biophysical/fluid	(11630-1, LN, 'Amniotic Fluid Volume')
OB/Biophysical/reactiveNst	(11635-5, LN, 'Fetal Heart Reactivity')
OB/Biophysical/score	(11634-3, LN, 'Biophysical Profile Sum Score')

17.3.18 Usage of TID 5010 Amniotic Sac Section

TABLE 17- 59 TID 5010

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121070, DCM, 'Findings')	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, 'Finding Site')	1	M		DT (T-F13DD, SRT, 'Amniotic Sac')
3	>	CONTAINS	INCLUDE	DTID (300) Measurement	1	M		\$Measurement = DT (11627-7, LN, "Amniotic Fluid Index")
4	>	CONTAINS	INCLUDE	DTID (300) Measurement	4	U		DT (T-F13DD, SRT, 'Amniotic Sac')

17.3.19 GEU Parameter Map from Context ID 12008 OB-GYN Amniotic Sac

TABLE 17- 60 CONTEXT ID 12008

Name of GEU parameter	Measurement Concept Name
AFI1	(11624-4, LN, 'First Quadrant Diameter')
AFI2	(11626-9, LN, 'Second Quadrant Diameter')
AFI3	(11625-1, LN, 'Third Quadrant Diameter')
AFI4	(11623-6, LN, 'Fourth Quadrant Diameter')
AFI Sum	(11627-7, LN, 'Amniotic Fluid Index')

17.3.20 Usage of TID 5011 Early Gestation Section

TABLE 17- 61 TID 5011

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125009, DCM, 'Early Gestation')	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID (1008) Subject Context, Fetus	1	MC	IF this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	INCLUDE	DTID (5008) Fetal Biometry Group	1-n	M		\$BiometryType = DCID (12009) Early Gestation Biometry Measurements

17.3.21 GEU Parameter Map from Context ID 12009 Early Gestation Biometry

TABLE 17- 62 CONTEXT ID 12009

Name of GEU parameter	Standard Concept Name
CRL	(11957-8, LN, 'Crown Rump Length')
GS	(11850-5, LN, 'Gestational Sac Diameter')
NT	(33069-6, LN, 'Nuchal Translucency')
YolkSac	(11816-6, LN, 'Yolk Sac Length')

17.3.22 Usage of TID 5012 Ovaries Section

TABLE 17- 63 TID 5012

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121070, DCM, 'Findings')	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-COE3, SRT, "Finding Site")	1	M		DT (T-87000, SRT, "Ovary")
3	>	CONTAINS	INCLUDE	DTID (5016) LWH Volume Group	1	U		\$GroupName = EV (T-87000, SRT, "Ovary") \$Width = EV (11829-9, LN, "Left Ovary Width") \$Length = EV (11840-6, LN, "Left Ovary Length") \$Height = EV (11857-0, LN, "Left Ovary Height") \$Volume = EV (12164-0, LN, "Left Ovary Volume")
4	>	CONTAINS	INCLUDE	DTID (5016) LWH Volume Group	1	M		\$GroupName = EV (T-87000, SRT, "Ovary") \$Width = EV (11830-7, LN, "Right Ovary Width") \$Length = EV (11841-4, LN, "Right Ovary Length") \$Height = EV (11858-8, LN, "Right Ovary Height")

								\$Volume = EV (12165-7, LN, "Right Ovary Volume")
--	--	--	--	--	--	--	--	---

17.3.23 Usage of TID 5013 Follicles Section

TABLE 17- 64 TID 5013

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121070, DCM, 'Findings')	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-COE3, SRT, "Finding Site")	1	M		DT (T-87600, SRT, "Ovarian Follicle")
3	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		\$Laterality
4	>	CONTAINS	NUM	\$Number	1	U		
5	>	CONTAINS	INCLUDE	DTID (5014) Follicle Measurement Group	1-n	U		

17.3.24 Usage of TID 5015 Pelvis and Uterus Section

TABLE 17- 65 TID 5007

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (125004, DCM, 'Pelvis and Uterus')	1	M		
2	>	CONTAINS	INCLUDE	DTID (5016) LWH Volume Group	1	U		\$GroupName = EV (T-87000, SRT, "Ovary") \$Width = EV (11829-9, LN, "Left Ovary Width") \$Length = EV (11840-6, LN, "Left Ovary Length") \$Height = EV (11857-0, LN, "Left Ovary Height") \$Volume = EV (12164-0, LN, "Left Ovary Volume")
3	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement = DCID (12011) Ultrasound Pelvic and Uterus \$Derivation = DCID (3627) Measurement Type

17.3.25 GEU Parameter Map from Context ID 12011 Ultrasound Pelvis and Uterus

TABLE 17- 66 CONTEXT ID 12011

Name of GEU parameter	Base Measurement Concept Name
Gyn/2D/UtL	(11842-2, LN, Uterus Length)
Gyn/2D/UtH	(11859-6, LN, Uterus Height)
Gyn/2D/UtW	(11865-3, LN, Uterus Width)
OB/2D/Cervix Trace	(11961-0, LN, Cervix Length)
OB/2D/Cervix Length	(11961-0, LN, Cervix Length)
Gyn/2D/Cervix Trace	(11961-0, LN, Cervix Length)
Gyn/2D/Cervix Length	(11961-0, LN, Cervix Length)
Gyn/2D/UtVolume	(33192-6, LN, Uterus Volume)

Name of GEU parameter	Base Measurement Concept Name
Gyn/2D/Endo	(12145-9, LN, 'Endometrium Thickness')
Ob/2D/OvL with Side=Rt	(11841-4, LN, 'Right Ovary Length')
Ob/2D/OvH with Side=Rt	(11858-8, LN, 'Right Ovary Height')
Ob/2D/OvW with Side=Rt	(11830-7, LN, 'Right Ovary Width')
Ob/2D/OvVolume with Side=Rt	(12165-7, LN, 'Right Ovary Volume')
Ob/2D/OvL with Side=Lt	(11840-6, LN, 'Left Ovary Length')
Ob/2D/OvH with Side=Lt	(11857-0, LN, 'Left Ovary Height')
Ob/2D/OvW with Side=Lt	(11829-9, LN, 'Left Ovary Width')
Ob/2D/OvVolume with Side=Lt	(12164-0, LN, 'Left Ovary Volume')
Gyn/2D/OvFoVolume/Volume	(G-D705, SRT, 'Volume')

17.3.26 GEU Parameter Map from Context ID 12013 Gestational Age Equations and Tables

TABLE 17- 67 CONTEXT ID 12013

Name of GEU parameter	Base Measurement Concept Name
OB/Campbell/Ratio HC/AC-Min	(33182-7, LN, 'HC/AC by GA, Campbell 1977')
OB/Campbell/Ratio HC/AC-Max	(33182-7, LN, 'HC/AC by GA, Campbell 1977')
OB/Jeanty/BPD-GP	(33153-8, LN, 'BPD by GA, Jeanty 1982')
OB/Jeanty/HC-GP	(33175-1, LN, 'HC by GA, Jeanty 1982')
OB/Jeanty/FL-GP	(33168-6, LN, 'FL by GA, Jeanty 1982')
OB/Hadlock/BPD-GP	(33198-3, LN, 'BPD by GA, Hadlock 1984')
OB/Hadlock/AC-GP	(33146-2, LN, 'AC by GA, Hadlock 1984')
OB/Hadlock/HC-GP	(33173-6, LN, 'HC by GA, Hadlock 1984')
OB/Hadlock/FL-GP	(33166-0, LN, 'FL by GA, Hadlock 1984')
OB/ASUM/BPD-GA	(33079-5, LN, 'BPD, ASUM 1989')
OB/ASUM/CRL-GA	(33089-4, LN, 'CRL, ASUM 1991')
OB/ASUM/AC-GA	(33072-0, LN, 'AC, ASUM 2000')
OB/ASUM2000/BPD-GA	(33151-2, LN, "BPD, ASUM 2000")
Eriksen/TAD-GA	(33128-0, LN, 'TAD, Eriksen 1985')
OB/Goldstein/TCD-GA	(33133-0, LN, 'TCD, Goldstein 1987')
OB/Hohler/FL-GA	(11922-2, LN, 'FL, Hohler 1982')
OB/Jeanty/CRL-GA	(33092-8, LN, 'CRL, Jeanty 1982')
OB/Jeanty/BPD-GA	(33539-8, LN, 'BPD, Jeanty 1982')
OB/Jeanty/AC-GA	(33537-2, LN, 'AC, Jeanty 1982')
OB/Jeanty/HC-GA	(33113-2, LN, 'HC, Jeanty 1982')
OB/Jeanty/FL-GA	(33099-3, LN, 'FL, Jeanty 1982')
OB/Jeanty/ULNA-GA	(11944-6, LN, 'Ulna, Jeanty 1984')
OB/Jeanty/HL-GA	(11936-2, LN, 'Humerus, Jeanty 1984')
OB/Jeanty/TIB-GA	(11941-2, LN, 'Tibia, Jeanty 1984')

Name of GEU parameter	Base Measurement Concept Name
OB/Jeanty/FIB-GA	(33097-7, LN, 'Fibula, Jeanty 1983')
OB/Jeanty/Radius-GA	(33126-4, LN, 'Radius, Jeanty 1983')
OB/Merz/BPD-GA	(33081-1, LN, 'BPD, Merz 1988')
OB/Merz/AC-GA	(33075-3, LN, 'AC, Merz 1988')
OB/Merz/HC-GA	(33115-7, LN, 'HC Merz, 1988')
OB/Merz/FL-GA	(33542-2, LN, 'FL, Merz 1988')
OB/Hadlock/BPD-GA	(11902-4, LN, 'BPD, Hadlock 1984')
OB/Hadlock/AC-GA	(11892-7, LN, 'AC, Hadlock 1984')
OB/Hadlock/HC-GA	(11932-1, LN, 'HC, Hadlock 1984')
OB/Hadlock/FL-GA	(11920-6, LN, 'FL, Hadlock 1984')
OB/Hadlock/CRL-GA	(11910-7, LN, 'CRL, Hadlock 1992')
OB/Hansmann/GS-GA	(33105-8, LN, 'GS, Hansmann 1979')
OB/Hansmann/BPD-GA	(11903-2, LN, 'BPD, Hansmann 1985')
OB/Hansmann/CRL-GA	(11911-5, LN, 'CRL, Hansmann 1985')
OB/Hansmann/OFD-GA	(33544-8, LN, 'OFD, Hansmann 1985')
OB/Hansmann/HC-GA	(33112-4, LN, 'HC, Hansmann 1985')
OB/Hansmann/TAD-GA	(33129-8, LN, 'TAD Hansmann, 1979')
OB/Hansmann/ThD-GA	(33136-3, LN, 'Transverse Thoracic Diameter, Hansmann 1985')
OB/Hansmann/FL-GA	(11921-4, LN, 'FL, Hansmann 1985')
OB/Hansmann/AC-GA	(33073-8, LN, 'AC, Hansmann 1985')
OB/Hellman/GS-GA	(11928-9, LN, 'GS, Hellman 1969')
OB/Rempen/GS-GA	(11929-7, LN, 'GS, Rempen 1991')
OB/Rempen/CRL-GA	(33094-4, LN, 'CRL, Rempen 1991')
OB/Rempen/BPD-GA	(33083-7, LN, 'BPD, Rempen 1991')
OB/Hill/TCD-GA	(33134-8, LN, 'TCD, Hill 1990')
OB/Mercer/Ft-GA	(11926-3, LN, 'Foot Length, Mercer 1987')
OB/Kurtz/BPD-GA	(11906-5, LN, 'BPD, Kurtz 1980')
OB/Nelson/CRL-GA	(11913-1, LN, 'CRL, Nelson 1981')
OB/Robinson/CRL-GA	(11914-9, LN, 'CRL, Robinson 1975')
OB/Mayden/OOD-GA	(33124-9, LN, 'OOD, Mayden, 1982')
OB/Mayden/IOD-GA	(33122-3, LN, 'IOD, Mayden 1982')
OB/Yarkoni/CLA-GA	(33088-6, LN, 'Clavical length,Yarkoni 1985')
OB/Osaka/CRL-GA	(33093-6, LN, 'CRL, Osaka 1989')
OB/Osaka/BPD-GA	(33082-9, LN, 'BPD, Osaka 1989')
OB/Osaka/FL-GA	(33101-7, LN, 'FL, Osaka 1989')
OB/Osaka/HL-GA	(33117-3, LN, 'Humerus Length, Osaka 1989')
OB/Tokyo/GS-GA	(33108-2, LN, 'GS, Tokyo 1986')
OB/Tokyo/CRL-GA	(33096-9, LN, 'CRL, Tokyo 1986')
OB/Tokyo/BPD-GA	(33085-2, LN, 'BPD, Tokyo 1986')
OB/Tokyo/FL-GA	(33103-3, LN, 'FL, Tokyo 1986')

Name of GEU parameter	Base Measurement Concept Name
OB/Shinozuka/BPD-GA	(33084-5, LN, 'BPD, Shinozuka 1996')
OB/Shinozuka/AC-GA	(33076-1, LN, 'AC, Shinozuka 1996')
OB/Shinozuka/FL-GA	(33102-5, LN, 'FL, Shinozuka 1996')
OB/Shinozuka/CRL-GA	(33095-1, LN, 'CRL, Shinozuka 1996')
average	(11884-4, LN, 'Average Ultrasound Age')
OB/WHO/AC-GP	(33148-8, LN, "AC by GA, WHO 2017")
OB/WHO/BPD-GP	(33154-6, LN, "BPD by GA, WHO 2017")
OB/WHO/FL-GP	(33169-4, LN, "FL by GA, WHO 2017")
OB/WHO/HC-GP	(33176-9, LN, "HC by GA, WHO 2017")
OB/WHO/HL-GP	(33169-4, LN, "HL by GA, WHO 2017")
OB/Jeanty1984/FL-GA	(11923-0, LN, "FL, Jeanty 1984")
OB/Daya/CRL-GA	(33091-0, LN, "CRL, Daya 1993")
OB/JSUM/EFW-GA	(GEU-1009-15, 99GEMS, "EFW, JSUM 2001")
OB/JSUM/BPD-GA	(GEU-1009-18, 99GEMS, "BPD, JSUM 2001")
OB/JSUM/AC-GA	(GEU-1009-21, 99GEMS, "AC, JSUM 2001")
OB/JSUM2009/AC-GA	(GEU-1009-24, 99GEMS, "AC, JSUM 2009")
OB/JSUM/FL-GA	(GEU-1009-27, 99GEMS, "FL, JSUM 2001")
OB/JSUM/CRL-GA	(GEU-1009-30, 99GEMS, "CRL, JSUM 2001")
OB/Jeanty/BD-GA	(GEU-1009-41, 99GEMS, "BD, Jeanty 1982")
OB/Osaka/FTA-GA	(GEU-1009-42, 99GEMS, "FTA, Osaka 1989")
OB/Osaka/EFW-GA	(GEU-1009-45, 99GEMS, "EFW, Osaka 1989")
OB/Tokyo/LV-GA	(GEU-1009-48, 99GEMS, "SL, Tokyo 1986")
OB/Shinozuka/AxT-GA	(GEU-1009-52, 99GEMS, "AxT, Shinozuka 1996")
OB/Rempen2004/CRL-GA	(GEU-1009-56, 99GEMS, "CRL, Rempen 2004")
OB/Pexsters/CRL-GA	(GEU-1009-70, 99GEMS, "CRL, Pexsters 2010")

17.3.27 GEU Parameters Map to Context ID 226 Population Statistical Descriptors

TABLE 17- 68 CONTEXT ID 226

Name of GEU parameter	Base Measurement Concept Name
OB/Eriksen/TAD-Min OB/Jeanty/CRL-Min OB/Jeanty/AC-Min OB/Hadlock/CRL-Min OB/Hansmann/TAD-Min OB/Hansmann/AC-Min OB/Hellman/GS-Min OB/Hill/TCD-Min OB/Kurtz/BPD-Min OB/Robinson/CRL-Min OB/Osaka/CRL-Min OB/Osaka/BPD-Min OB/Osaka/FL-Min OB/Osaka/HL-Min OB/Tokyo/GS-Min	(R-00347, SRT, '1 Sigma Lower Value of population')

Name of GEU parameter	Base Measurement Concept Name
OB/Tokyo/CRL-Min OB/Tokyo/BPD-Min OB/Tokyo/FL-Min OB/Shinozuka/BPD-Min OB/Shinozuka/AC-Min OB/Shinozuka/FL-Min OB/Shinozuka/CRL-Min	
OB/ASUM2000/BPD-Min OB/ASUM2000/AC-Min	(R-00388, SRT, '1 Sigma Lower Value of population')
OB/Eriksen/TAD-Max OB/Jeanty/CRL-Max OB/Jeanty/AC-Max OB/Hadlock/CRL-Max OB/Hansmann/TAD-Max OB/Hansmann/AC-Max OB/Hellman/GS-Max OB/Hill/TCD-Max OB/Kurtz/BPD-Max OB/Robinson/CRL-Max OB/Osaka/CRL-Max OB/Osaka/BPD-Max OB/Osaka/FL-Max OB/Osaka/HL-Max OB/Tokyo/GS-Max OB/Tokyo/CRL-Max OB/Tokyo/BPD-Max OB/Tokyo/FL-Max OB/Shinozuka/BPD-Max OB/Shinozuka/AC-Max OB/Shinozuka/FL-Max OB/Shinozuka/CRL-Max	(R-00346, SRT, '1 Sigma Upper Value of population')
OB/ASUM2000/BPD-Max OB/ASUM2000/AC-Max	(R-00387, SRT, '1 Sigma Upper Value of population')
OB/ASUM/BPD-Max OB/ASUM/AC-Max OB/Jeanty1984/FL-Max OB/Hadlock/BPD-Max OB/Hadlock/AC-Max OB/Hadlock/HC-Max OB/Hadlock/FL-Max OB/Hansmann/BPD-Max OB/Hansmann/CRL-Max OB/Hansmann/OFD-Max OB/Hansmann/HC-Max OB/Hansmann/ThD-Max OB/Hansmann/FL-Max OB/Rempen/GS-Max OB/Rempen/CRL-Max OB/Rempen/BPD-Max OB/Mercer/Ft-Max	(R-00387, SRT, "2 Sigma Upper Value of population")
OB/ASUM/BPD-Min OB/ASUM/AC-Min OB/Jeanty1984/FL-Min OB/Hadlock/BPD-Min	(R-00388, SRT, "2 Sigma Lower Value of population")

Name of GEU parameter	Base Measurement Concept Name
OB/Hadlock/AC-Min OB/Hadlock/HC-Min OB/Hadlock/FL-Min OB/Hansmann/BPD-Min OB/Hansmann/CRL-Min OB/Hansmann/OFD-Min OB/Hansmann/HC-Min OB/Hansmann/ThD-Min OB/Hansmann/FL-Min OB/Rempen/GS-Min OB/Rempen/CRL-Min OB/Rempen/BPD-Min OB/Mercer/Ft-Min	
OB/Jeanty/BPD-Min OB/Jeanty/HC-Min OB/Jeanty/FL-Min OB/Jeanty/ULNA-Min OB/Jeanty1984/ULNA-Min OB/Jeanty/HL-Min OB/Jeanty/TIB-Min OB/Merz/BPD-Min OB/Merz/AC-Min OB/Merz/HC-Min OB/Merz/FL-Min OB/Rempen2004/CRL-Min	(R-00397, SRT, '5th Percentile Value of population')
OB/Jeanty/BPD-Max OB/Jeanty/HC-Max OB/Jeanty/FL-Max OB/Jeanty/ULNA-Max OB/Jeanty1984/ULNA-Max OB/Jeanty/HL-Max OB/Jeanty/TIB-Max OB/Merz/BPD-Max OB/Merz/AC-Max OB/Merz/HC-Max OB/Merz/FL-Max OB/Rempen2004/CRL-Max	(R-00337, SRT, '95th Percentile Value of population')
standard deviation	(121414, DCM, "Standard deviation of population")
OB/JSUM/EFW-Max	(GEU-1009-16, 99GEMS, "2 Sigma Upper Value of population")
OB/JSUM/EFW-Min	(GEU-1009-17, 99GEMS, "2 Sigma Lower Value of population")
OB/JSUM/BPD-Max	(GEU-1009-19, 99GEMS, "2 Sigma Upper Value of population")
OB/JSUM/BPD-Min	(GEU-1009-20, 99GEMS, "2 Sigma Lower Value of population")
OB/JSUM/AC-Max	(GEU-1009-22, 99GEMS, "2 Sigma Upper Value of population")
OB/JSUM/AC-Min	(GEU-1009-23, 99GEMS, "2 Sigma Lower Value of population")
OB/JSUM2009/AC-Max	(GEU-1009-25, 99GEMS, "2 Sigma Upper Value of population")
OB/JSUM2009/AC-Min	(GEU-1009-26, 99GEMS, "2 Sigma Lower Value of population")
OB/JSUM/FL-Max	(GEU-1009-28, 99GEMS, "2 Sigma Upper Value of population")
OB/JSUM/FL-Min	(GEU-1009-29, 99GEMS, "2 Sigma Lower Value of population")
OB/ASUM2000/OFD-Max	(GEU-1009-32, 99GEMS, "1 Sigma Upper Value of population")
OB/ASUM2000/OFD-Min	(GEU-1009-33, 99GEMS, "1 Sigma Lower Value of population")

Name of GEU parameter	Base Measurement Concept Name
OB/ASUM2000/FL-Max	(GEU-1009-34, 99GEMS, "1 Sigma Upper Value of population")
OB/ASUM2000/FL-Min	(GEU-1009-35, 99GEMS, "1 Sigma Lower Value of population")
OB/ASUM2000/HC-Max	(GEU-1009-36, 99GEMS, "1 Sigma Upper Value of population")
OB/ASUM2000/HC-Min	(GEU-1009-37, 99GEMS, "1 Sigma Lower Value of population")
OB/ASUM2000/HL-Max	(GEU-1009-38, 99GEMS, "1 Sigma Upper Value of population")
OB/ASUM2000/HL-Min	(GEU-1009-39, 99GEMS, "1 Sigma Lower Value of population")
OB/Osaka/FTA-Max	(GEU-1009-43, 99GEMS, "1 Sigma Upper Value of population")
OB/Osaka/FTA-Min	(GEU-1009-44, 99GEMS, "1 Sigma Lower Value of population")
OB/Osaka/EFW-Max	(GEU-1009-46, 99GEMS, "1 Sigma Upper Value of population")
OB/Osaka/EFW-Min	(GEU-1009-47, 99GEMS, "1 Sigma Lower Value of population")
OB/Tokyo/LV-Max	(GEU-1009-49, 99GEMS, "1 Sigma Upper Value of population")
OB/Tokyo/LV-Min	(GEU-1009-50, 99GEMS, "1 Sigma Lower Value of population")
OB/Shinozuka/AxT-Max	(GEU-1009-53, 99GEMS, "1 Sigma Upper Value of population")
OB/Shinozuka/AxT-Min	(GEU-1009-54, 99GEMS, "1 Sigma Lower Value of population")

17.3.28 GEU Parameters Map to Context ID 12015 Fetal Growth Equations and Tables

TABLE 17- 69 CONTEXT ID 12015

Name of GEU parameter	Base Measurement Concept Name
OB/Jeanty/FIB-GP	(33164-5, LN, 'Fibula by GA, Jeanty 1983')
OB/Jeanty/Radius-GP	(33180-1, LN, 'OB/Jeanty/Radius-GP')
OB/Merz/BPD-GP	(33154-6, LN, 'BPD by GA, Merz 1988')
OB/Merz/AC-GP	(33148-8, LN, 'AC by GA, Merz 1988')
OB/Merz/HC-GP	(33176-9, LN, 'HC by GA, Merz 1988')
OB/Merz/FL-GP	(33169-4, LN, 'FL by GA, Merz 1988')
OB/Hansmann/TTD-GA	(33136-3, LN, 'Transverse Thoracic Diameter, Hansmann 1985')
OB/Rempen/GS-GP	(33171-0, LN, 'GS by GA, Rempen 1991')
OB/Rempen/CRL-GP	(33160-3, LN, 'CRL by GA, Rempen1991')
OB/Rempen/BPD-GP	(33155-3, LN, 'BPD by GA, Rempen 1991')
OB/ASUM2000/OFD-GA	(33119-9, LN, 'OFD, ASUM 2000')
OB/ASUM2000/CRL-GA	(33090-2, LN, 'CRL, ASUM 2000')
OB/ASUM2000/HC-GA	(33109-0, LN, 'HC, ASUM 2000')
OB/ASUM2000/AC-GA	(33072-0, LN, 'AC, ASUM 2000')
OB/ASUM2000/HL-GA	(33116-5, LN, 'Humerus Length, ASUM 2000')
OB/ASUM2000/FL-GA	(GEU-1009-31, 99GEMS, 'FL, ASUM 2000')
OB/Chervenak/HC-GP	(GEU-1009-63, 99GEMS, 'HC by GA, Chervenak 1984')
OB/Jeanty1984/FL-EDD	(11778-8, LN, 'Estimated Date of Delivery')
OB/Jeanty1986/FIB-GA	(GEU-1009-58, 99GEMS, 'Fibula, Jeanty 1986')
OB/Jeanty1986/FIB-GP	(GEU-1009-59, 99GEMS, 'Fibula by GA, Jeanty 1986')
OB/Jeanty1986/FIB-EDD	(11778-8, LN, 'Estimated Date of Delivery')
OB/Jeanty/BOD-GA	(GEU-1009-66, 99GEMS, 'BOD, Jeanty 1984')

OB/Jeanty/BOD-GP	(GEU-1009-67, 99GEMS, 'BOD by GA, Jeanty 1984')
OB/Jeanty1984/ULNA-EDD	(11778-8, LN, 'Estimated Date of Delivery')
OB/Jeanty1983/Radius-GP	(33180-1, LN, 'Radius by GA, Jeanty 1983')
OB/Jeanty/OFD-GA	(GEU-1009-68, 99GEMS, 'OFD, Jeanty 1984')
OB/Jeanty/OFD-GP	(GEU-1009-69, 99GEMS, 'OFD by GA, Jeanty 1984')
OB/Hansmann1986/GS-GA	(GEU-1009-61, 99GEMS, 'GS, Hansmann 1986')
OB/Hansmann1986/GS-EDD	(11778-8, LN, 'Estimated Date of Delivery')
OB/Pexsters/CRL-GP	(GEU-1009-71, 99GEMS, 'CRL by GA, Pexsters 2010')
OB/Sonek/NBL-GP	(GEU-1009-74, 99GEMS, 'NBL by GA, Sonek 2003')

17.3.29 GEU Parameters Map to Context ID 225 Measurement Uncertainty Concepts

TABLE 17- 70 CONTEXT ID 225

Name of GEU parameter	Base Measurement Concept Name
+/- , range of measurement uncertainty	(R-00363, SRT, "+/- , Range of measurement uncertainty")
+ , range of upper measurement uncertainty	(R-00364, SRT, "+ , Range of upper measurement uncertainty")
- , range of lower measurement uncertainty	(R-00362, SRT, "- , Range of lower measurement uncertainty")

17.3.30 Context ID 3627 Extended Measurement Types

TABLE 17- 71 CONTEXT ID 3627

GEU parameter attribute	Concept Modifier or Has Properties
Aver	(121401, DCM, 'Derivation') = (R-00317, SRT, 'Mean')
Max	(121401, DCM, 'Derivation') = (G-A437, SRT, 'Maximum')
Min	(121401, DCM, 'Derivation') = (R-404FB, SRT, 'Minimum')
Last	(121404, DCM, 'Selection Status') = (121411, DCM, 'Most recent value chosen')
'C'	(121401, DCM, 'Derivation') = (121428, DCM, 'Calculated')

17.3.31 Miscellaneous GEU Parameters Map to SR Coding Standards

TABLE 17- 72 MISCELLANEOUS GEU PARAMETERS MAP

Name of GEU parameter	Base Measurement Concept Name
GA	(18185-9, LN, Gestational Age)
GP	(125012, DCM, Growth Percentile Rank)
OOR	(114009, DCM, Value out of range)
Side=Rt	(G-A100, SRT, Right)
Side=Lt	(G-A101, SRT, Left)

17.3.32 TID 99000 Fetal Doppler Measurements

TABLE 17-73 TID 99000 FETAL DOPPLER MEASUREMENTS

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value set
1			CONTAINER	DT(99000, DCM, "Fetal Doppler"	1	M		
2	>	HAS OBS CONTEXT	INCLUDE	DTID(1008) Subject Context, Fetus ID	1	MC	If this template is invoked more than once to describe more than one fetus	
3	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-45510, SRT "Cerebral artery"); \$Laterality= EV(G-A101, SRT, "Left"); \$TargetSiteMod = EV(G-A113, SNM3, "Internal") \$MeasType = MemberOf DCID(9900)
4	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-45510, SRT "Cerebral artery"); \$Laterality=EV(G-A100, SRT, "Right"); \$TargetSiteMod =

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value set
								EV(G-A113, SNM3, "Internal") \$MeasType = MemberOf DCID(9900)
5	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-46420, SRT, "Hepatic Artery"); \$MeasType = MemberOf DCID(9900)
6	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-48720, SNM3, "Hepatic Vein"); \$MeasType = MemberOf DCID(9900)
7	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-48710, SNM3, "Inferior Vena Cava"); \$MeasType = MemberOf DCID(9900)
8	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-46600, SNM3, "Renal artery"); \$Laterality = EV(G-A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
9	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-46600, SNM3, "Renal artery"); \$Laterality = EV(G-A101, SRT, "Left"); \$MeasType = MemberOf DCID(9900)
10	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-46460, SNM3, "Splenic artery"); \$MeasType = MemberOf DCID(9900)
11	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-42070, SNM3, "Thoracic aorta"); \$MeasType = MemberOf DCID(9900)
12	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-F1810, SNM3, "Umbilical artery"); \$MeasType = MemberOf DCID(9900)
13	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-48817, SNM3, "Umbilical vein"); \$MeasType = MemberOf DCID(9900)
14	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(VP-0001, 99VP, "Ductus venosus vein"); \$MeasType = MemberOf DCID(9900)
15	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-45010, SNM3, "Carotid artery"); \$Laterality = EV(G-A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value set
16	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$TargetSite = EV(T-45010, SNM3, "Carotid artery"); \$Laterality = EV(G-A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
17	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$TargetSite = EV(T-F1412, SRT, "Vitelline Artery of Placenta"); \$MeasType = MemberOf DCID(9900)
18	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-45600, SRT, "Middle Cerebral Artery"); \$Laterality = EV(G-A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
19	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-45600, SRT, "Middle Cerebral Artery"); \$Laterality = EV(G-A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
20	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-42000, SRT, "Aorta"); \$MeasType = MemberOf DCID(9900)
21	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T-F680F, SRT, "Ductus Venosus"); \$MeasType = MemberOf DCID(9900)

17.3.33 TID 99001 Maternal Doppler Measurements

TABLE 17-74 TID 99001 MATERNAL DOPPLER MEASUREMENTS

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT(99001, DCM, "Maternal Doppler Measurements"	1	M		
2	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T- 46820, SRT, "Uterine artery"); \$Laterality = EV(G- A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
3	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T- 46820, SRT, "Uterine artery"); \$Laterality = EV(G- A101, SRT, "Left");

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
								\$MeasType = MemberOf DCID(9900)
4	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T- 46980, SRT, "Ovarian artery"); \$Laterality = EV(G- A100, SRT, "Right"); \$MeasType = MemberOf DCID(9900)
5	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T- 46980, SRT, "Ovarian artery"); \$Laterality = EV(G- A101, SRT, "Left"); \$MeasType = MemberOf DCID(9900)
6	>	CONTAINS	INCLUDE	DTID(99100) Doppler Group	1	U		\$FindingSite = EV(T- 40003, SRT, "Entire Vessel"); \$MeasType = MemberOf DCID(9900)

17.3.34 TID 99100 Doppler Group

TABLE 17-75 TID 99100 DOPPLER GROUP

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT(99100, DCM, "Doppler Group")	1	M		
2	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	M		\$Measurement = \$MeasType

17.3.35 TID 99002 Fibroid Section

TABLE 17-16 TID 99002 FIBROID SECTION

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT(121070, DCM, "Findings")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV(G-C0E3, SRT, "Finding Site")	1	M		DT(GEU-1009-06, 99GEMS, "Fibroid")
3	>	CONTAINS	NUM	\$Number	1	M		Number of Fibroids
4	>	CONTAINS	INCLUDE	DTID (99003) Fibroid Measurement Group	1-n	U		

17.3.36 TID 99003 Fibroid Measurement Group

TABLE 17-77 TID 99003 FIBROID MEASUREMENT GROUP

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
--	--------	-------------------------	------------	--------------	----	-------------	-----------	----------------------

1			CONTAINER	EV(125007, DCM, "Measurement Group")	1	M		
2	>	HAS OBS CONTEXT	TEXT	EV(12510, DCM, "Identifier")	1	U		Unique among all groups of same laterality
3	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement = EV (G-D705, SRT, "Volume")
4	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	M		\$Measurement = EV(GEU-1009-09, 99GEMS, "Fibroid Diameter") \$Derivation = DCID (3627) Measurement Type

17.3.37 TID 99060 Fetal Anatomy Section

TABLE 17-78 TID 99060 FETAL ANATOMY SECTION

	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV(GEU-1009-01, 99GEMS, "Fetal Anatomy")	1	M		
2	>	HAS OBS CONTEXT	TEXT	EV(121030, DCM, "Subject ID")	1	U		
3	>	HAS OBS CONTEXT	NUM	EV(11878-6, LN, "Number of Fetuses")	1	U		
4	>	CONTAINS	TEXT	EV(GEU-1009-02, 99GEMS, "Fetus Position")	1	U		
5	>	CONTAINS	TEXT	EV(GEU-1009-03, 99GEMS, "Placenta Location")	1	U		

17.3.38 Usage of TID 5013 SonoAVC Follicle Section

TABLE 17-79 TID 5013

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	DT (121070, DCM, 'Findings')	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	M		DT (GEU-1011-01, 99GEMS, "SonoAVC Follicle")
3	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		\$Laterality
4	>	CONTAINS	INCLUDE	DTID (5014) SonoAVC Measurement Group	1-n	U		

17.3.39 TID 5014 SonoAVC Measurement Group

TABLE 17-80 TID 5014 SONOAVC MEASUREMENT GROUP


	N L	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV(125007, DCM,	1	M		

				"Measurement Group"				
2	>	HAS OBS CONTEXT	TEXT	EV(12510, DCM, "Identifier")	1	U		Unique among all groups of same laterality
3	>	CONTAINS	INCLUDE	DTID (300) Measurement	1-n	U		\$Measurement = EV (GEU-1011-05, 99GEMS, "Manual dx") \$Measurement = EV (GEU-1011-06, 99GEMS, "Manual dy") \$Measurement = EV (GEU-1011-07, 99GEMS, "Manual meand") \$Measurement = EV (GEU-1011-08, 99GEMS, "Auto dv") \$Measurement = EV (GEU-1011-09, 99GEMS, "Auto dx") \$Measurement = EV (GEU-1011-10, 99GEMS, "Auto dy") \$Measurement = EV (GEU-1011-11, 99GEMS, "Auto dz") \$Measurement = EV (GEU-1011-12, 99GEMS, "Auto meand") \$Measurement = EV (GEU-1011-13, 99GEMS, "Auto volume")

17.4 BREAST IMAGING REPORT

This section describes the contents of the Breast Imaging Report (TID 4200) SR.

17.4.1 Breast Imaging Report Template Structure supported by LOGIQ Scanner

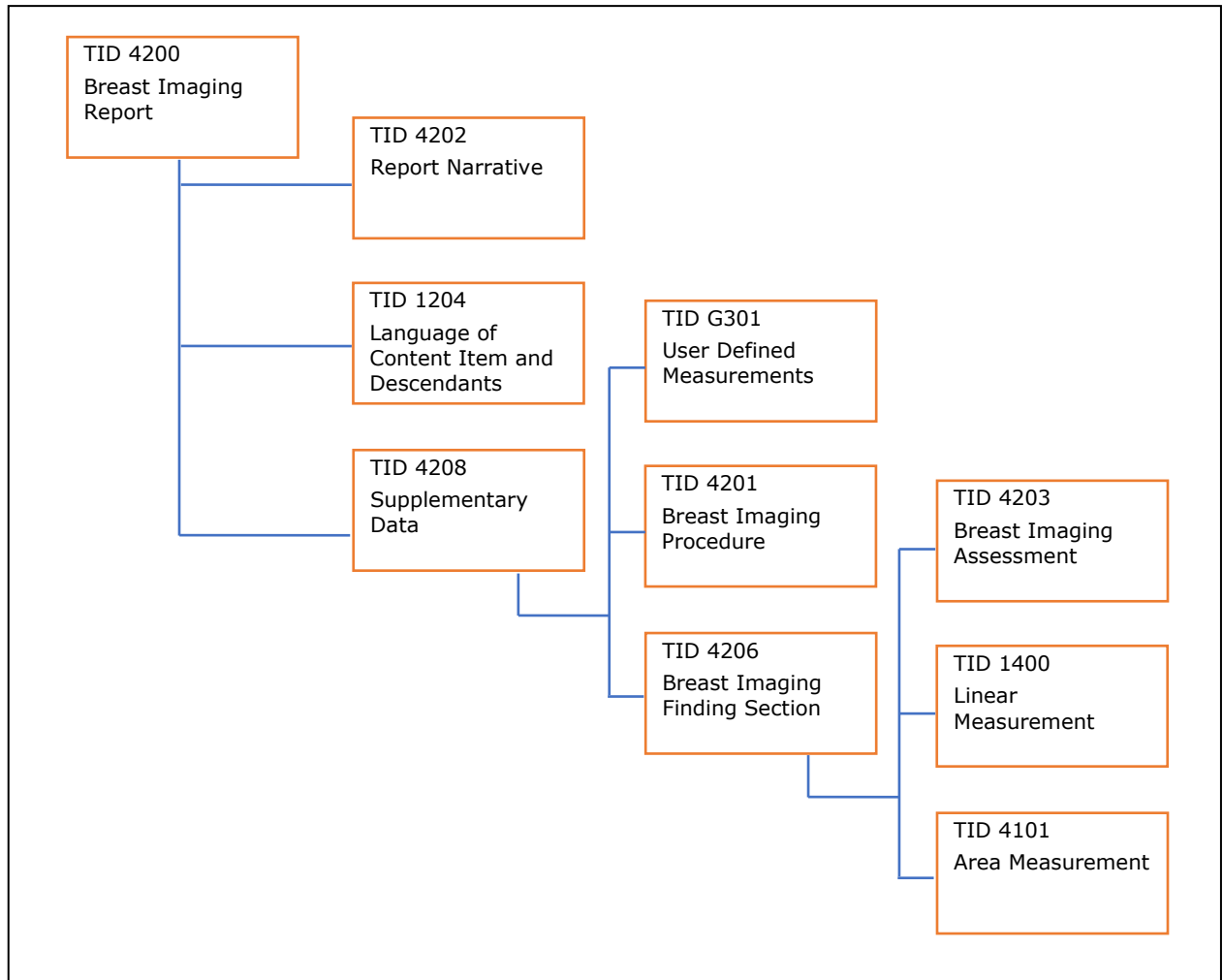

17.4.2 TID 4200 Breast Imaging Report

TABLE 17-81 TID 4200

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (111400, DCM, "Breast Imaging Report")	1	M		
2	>	HAS CONCEPT MOD	INCLUDE	DTID (1204) Language of Content Item and Descendants	1	M		

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
3	>	CONTAINS	INCLUDE	DTID (4202) Breast Imaging Report Narrative	1	M		
4	>	CONTAINS	INCLUDE	DTID (4208) Breast Imaging Report Supplementary Data	1	U		

17.4.3 TID 4201 Breast Imaging Procedure Reported

TABLE 17-82 TID 4201

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
1			CODE	EV (121058, DCM, "Procedure reported")	1	M		(P5-B8500, SRT, "Ultrasonography of breast")	
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		GE Value	DICOM Map
								Lt	(T-04030, SRT, "Left breast")
								Rt	(T-04020, SRT, "Right breast")

17.4.4 TID 4202 Breast Imaging Report Narrative

TABLE 17-83 TID 4202

	NL	Rel with Parent	VT	Concept Name	V M	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (111412, DCM, "Narrative Summary"	1	M		
2	>	CONTAINS	CONTAINER	(121058, DCM, "Procedure reported")	1- n	M		
3	>>	CONTAINS	TEXT	EV (121058, DCM, "Procedure reported")	1	M		"Ultrasonography of breast"

17.4.5 TID 4203 Breast Imaging Assessment

TABLE 17-84 TID 4203

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CODE	EV (111005, DCM, "Assessment Category")	1	M		DCID (6027) Mammography Assessment

17.4.6 TID 4206 Breast Imaging Report Finding Section

TABLE 17-85 TID 4206

	NL	Rel with Parent	VT	Concept Name		VM	Re q Ty pe	Co ndi tio n	Value Set Constraint	
1			CONTAINER	EV (121070, DCM, “Findings”)		1	M			
2	>	CONTAINS	INCLUDE	DTID (4201) Breast Imaging Procedure Reported		1	M			
3	>	CONTAINS	CODE	EV (121071, DCM, “Finding”)		1-n	M		GE Value for Composition	
									DICOM Map	
									-	
									a. Homogeneous background echotexture-fat	
									b. Homogeneous background echotexture-fibroglandular	
c. Heterogenous background echotexture										
(GEU-1006-01, 99GEMS, “Lesion”)										
(111351, DCM, “Homogeneous fat echotexture”)										
(111352, DCM, “Homogeneous fibroglandular echotexture”)										
(111353, DCM, “Heterogeneous echotexture”)										
4	>>	HAS PROPERTIES	INCLUDE	DTID (4203) Breast Imaging Assessment		1	U			
5	>>	HAS PROPERTIES	TEXT	(GEU-1006-03, 99GEMS, “Assessment Comment”)		1	U			
6	>>	HAS PROPERTIES	CODE	GE Value		1	U		GE Value	
				DICOM Map					DICOM Map	
				Position					1 OClock	
				(111014, DCM, “Clockface or region”)					(F-01781, SRT, 1 o’clock position)	
									2 OClock	
									(F-01782, SRT, 2 o’clock position)	
									3 OClock	
									(F-01783, SRT, 3 o’clock position)	
									4 OClock	
									(F-01784, SRT, 4 o’clock position)	
									5 OClock	
									(F-01785, SRT, 5 o’clock position)	
									6 OClock	
									(F-01786, SRT, 6 o’clock position)	
									7 OClock	
		(F-01787, SRT, 7 o’clock position)								
		8 OClock								
		(F-01788, SRT, 8 o’clock position)								
		9 OClock								
		(F-01789, SRT, 9 o’clock position)								
		10 OClock								
		(F-0178A, SRT, 10 o’clock position)								
		11 OClock								
		(F-0178B, SRT, 11 o’clock position)								
		12 OClock								
		(F-0178C, SRT, 12 o’clock position)								
		Areolar								
		(F-0178F, SRT, Central region of breast)								
		SubAreolar								
		(F-0178D, SRT, Subareolar region)								
		Axillary								
		(F-01794, SRT, Axilla region)								
7	>>	HAS PROPERTIES	INCLUDE	DTID (1400) Linear Measurement		1-n	U			

LOGIQ E10 R2.X / LOGIQ E10s R2.X
LOGIQ E11 R2.X / LOGIQ E20 R2.X
DICOM CONFORMANCE STATEMENT

GE Healthcare

DIR DOC2107162 REV 4

	NL	Rel with Parent	VT	Concept Name		VM	Re q Ty pe	Co ndi tio n	Value Set Constraint	
8	>>	HAS PROPERTIES	INCLUDE	DTID (1401) Area Measurement		1-n	U			
9	>>	HAS PROPERTIES	NUM	(GEU-1006-07, 99GEMS, “A/B Ratio(area)”)			U			
10	>>	HAS PROPERTIES	NUM	(GEU-1006-10, 99GEMS, “A/B Ratio(Diam)”)			U			
11	>>	HAS PROPERTIES	CODE	GE Value	DICOM Map	1	U		GE Value	DICOM Map
				Segme nt	(111020, DCM, “Depth”)				A	(G-A105, SRT, Anterior)
									B	(G-A109, SRT, Middle)
									C	(G-A106, SRT, Posterior)
12	>>	HAS PROPERTIES	CODE	EV (M-020F9, SRT, “Shape”)		1-n	U		GE Value	DICOM Code
									Oval	(M-02120, SRT, "Ovoid shape (Oval)")
									Round	(M-02100, SRT, "Round shape")
									Irregular	(G-A402, SRT, "Irregular")
13	>>	HAS PROPERTIES	CODE	EV (111037, DCM, “Margins”)		1-n	U		GE Value	DICOM Map
									"Circumscribed"	(F-01741, SRT, "Circumscribed lesion")
									"Indistinct"	(F-01744, SRT, "Indistinct lesion")
									"Angular"	(111343, DCM, "Angular margins")
									"Microlobulated"	(F-01742, SRT, "Microlobulated lesion")
"Spiculated"	(F-01745, SRT, "Spiculated lesion")									
14	>>	HAS PROPERTIES	CODE	GE Value	DICOM Map	1-n	U		GE Value	DICOM Map
				Calcifica tions	(111009, CM, “Calcification Type”)				"Intraductal calcifications"	(GEU-1006-22, 99GEMS, “Intraductal calcifications”)
									"Calcifications in a mass"	(111346, DCM, “Calcifications within a mass”)
									"Calcifications outside of mass"	(111347, DCM, “Calcifications outside of a mass”)

**LOGIQ E10 R2.X / LOGIQ E10s R2.X
LOGIQ E11 R2.X / LOGIQ E20 R2.X
DICOM CONFORMANCE STATEMENT**

GE Healthcare

DIR DOC2107162 REV 4

	NL	Rel with Parent	VT	Concept Name		VM	Re q Ty pe	Co ndi tio n	Value Set Constraint	
15	>>	HAS PROPERTIES	CODE	GE Value	DICOM Map	1-n	U		GE Value	DICOM Map
				Special Cases	(G-C189, SRT, “Associated Finding”)				“Simple Cyst”	(D7-90035, SRT, “Cyst of breast”)
									"Clustered microcysts"	(111129, DCM, “Clustered microcysts”)
									"Complicated cysts"	(111130, DCM, “Complicated cyst”)
									"Mass in or on skin"	(D7-90382, SRT, "Sebaceous cyst of skin of breast”)
									"Foreign body"	(M-30400, SRT, “Foreign body”)
									“Vascular abnormalities AVMs”	(GEU-1006-24, 99GEMS, “Foreign body”)
									“Vascular abnormalities Mondor disease”	(D3-87780, SRT, “Thrombophlebitis of breast(Mondor's disease)”)
									“Postsurgical fluid collection”	(GEU-1006-25, 99GEMS, “Postsurgical fluid collection”)
									“Fat Necrosis”	(D7-90434, SRT, “Fat necrosis of breast”)
16	>>	HAS PROPERTIES	CODE	EV (111354, DCM, “Orientation”)		1	U		DCID (6152) Orientation	
18	>>	HAS PROPERTIES	CODE	EV (111360, DCM, “Echo pattern”)		1	U		DCID (6154) Echo pattern	
19	>>	HAS PROPERTIES	CODE	EV (111366, DCM, “Posterior acoustic features”)		1	U		GE Value	DICOM Map
									"No posterior features"	(111367, DCM, “No osterior acoustic features”)
									"Enhancement"	(111368, DCM, “Posterior enhancement”)
									“Shadowing”	(111369, DCM, "Posterior shadowing")
									“Combined Pattern”	(111370, DCM, "Combined posterior enhancement and shadowing")

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
20	>>	HAS PROPERTIES	CODE	EV (111371, DCM, "Identifiable effect on surrounding tissues")	1-n	U		GE Value	DICOM Map
								"Duct changes"	(GEU-1006-02, 99GEMS, "Duct changes")
								"Edema"	(F-01743, SRT, "Obscured lesion")
								"Architectural distortion"	(F-01795, SRT, "Architectural distortion of breast")
								"Skin thickening"	(F-0179A, SRT, "Skin thickening of breast")
								"Skin retraction"	(F-01799, SRT, "Skin retraction of breast")
								"Absent"	(R-4089B, SRT, "Absent")
								"Internal vascularity"	(GEU-1006-18, 99GEMS, "Internal vascularity")
								"Vessels in rim"	(GEU-1006-19, 99GEMS, "Vessels in rim")
								"Soft"	(GEU-1006-20, 99GEMS, "Soft")
								"Hard"	(GEU-1006-21, 99GEMS, "Hard")
								"Intermediate"	(G-A114, SRT, "Intermediate")
21	>>	HAS PROPERTIES	TEXT	(GEU-1006-12, 99GEMS, "Transaction UID")	1	U		<unique identifier>	
22	>>	HAS PROPERTIES	TEXT	(GEU-1006-13, 99GEMS, "Koios DS Version")	1	U		<version info>	
23	>>	HAS PROPERTIES	TEXT	(GEU-1006-14, 99GEMS, "Koios DS Assessment Score")	1	U		<score info>	
24	>>	HAS PROPERTIES	TEXT	(GEU-1006-15, 99GEMS, "Koios DS Assessment")	1	U		<assessment info>	

17.4.7 TID 4208 Breast Imaging Report Supplementary Data

TABLE 17-86 TID 4208

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (111414, DCM, "Supplementary Data")	1	M		
2	>	CONTAINS	INCLUDE	DTID (4201) Breast Imaging Procedure Reported	1-n	M		
3	>	CONTAINS	INCLUDE	DTID (4206) Breast Imaging Report Finding Section	1-n	U		
4	>	CONTAINS	INCLUDE	DTID (G301) GE Ultrasound User Defined Measurement	1-n	UC		

17.4.8 TID 1400 Linear Measurement Template

TABLE 17-87 TID 1400

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
1			NUM	GE Measurement	DICOM Map	1	M		UNITS = DCID(7460) "Units of Linear
				Length	(G-A22A, SRT, "Length")				

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
				Width	(G-A220, SRT, "Width")				Measurement"
				Height	(121207, DCM, "Height")				
				SP/2D/DiamRatioD1	(GEU-1006-08, 99GEMS, "A/B Ratio(Diam) D1")				
				SP/2D/DiamRatioD2	(GEU-1006-09, 99GEMS, "A/B Ratio(Diam) D2")				
				SP/2D/LesionDistToNipple	(121242, DCM, "Distance from nipple")				

17.4.9 TID 1401 Area Measurement Template

TABLE 17-88 TID 1401

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
1			NUM	GE Measurement	DICOM Map	1	M		Value shall be > 0 UNITS = DCID(7461) "Units of Area Measurement"
				SP/2D/AreaRatioA1	(GEU-1006-05, 99GEMS, "A/B Ratio(area) A1")				
				SP/2D/AreaRatioA2	(GEU-1006-06, 99GEMS, "A/B Ratio(area) A2")				
				SP/2D/ACArea	(GEU-1006-11, 99GEMS, "Auto Contour Area")				

17.4.10 TID 1204 Language of Content Item and Descendants

TABLE 17-89 TID 1204

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1		HAS CONCEPT MOD	CODE	(121049, DCM, "Language of Content Item and Descendants")	1	M		(en, RFC3066, "English")

TABLE 17-90 SHEAR WAVE MEASUREMENTS

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1		CONTAINS	CONTAINER	EV (GEU-1010-02, 99GEMS, "Shearwave Velocity")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		\$SectionLaterality
3	>	CONTAINS	NUM	EV (G-D784, SRT, "Velocity")	1	M		
4	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1	M		"V"
5	>	CONTAINS	NUM	EV (G-D784, SRT, "Velocity")	1	M		
6	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1	M		"V1"
7	>	CONTAINS	NUM	EV (G-D784, SRT, "Velocity")	1	M		
8	>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1	M		"V2"

17.4.11 TID 1402 Volume Measurement Template

TABLE 17-91 TID 1402

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
--	----	-----------------	----	--------------	----	----------	-----------	----------------------

GE Healthcare


DIR DOC2107162 REV 4

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
1			NUM	GE Measurement	DICOM Map	1	M		Value shall be > 0 UNITS = DCID(7462) "Units of Volume Measurement"
				SP/2D/LesionVolume	(G-D705, SRT, "Volume")				

17.5 SOFT TISSUE NECK AND HEAD IMAGING REPORT

This section describes the contents of the Soft Tissue Neck and Head Imaging Report (TID GEMS_US_SP_0100) SR.

17.5.1 Thyroid Imaging Report Template Structure supported by LOGIQ scanner

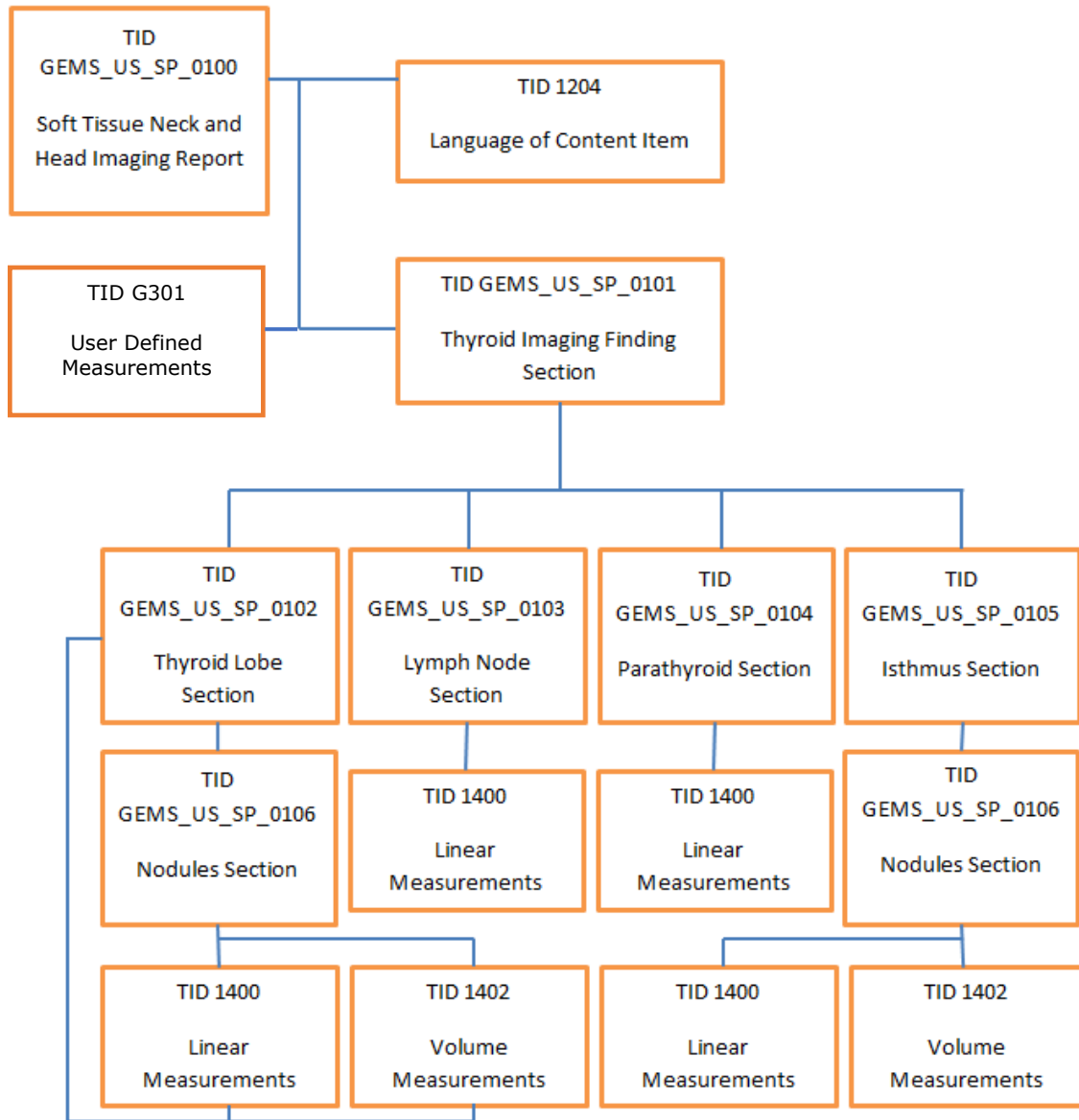

17.5.2 TID GEMS_US_SP_0100 Soft Tissue Neck and Head Imaging Report

TABLE 17-92 GEMS_US_SP_0100

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	(GEU-1007-01, 99GEMS, "Soft Tissue Neck and Head	1	M		

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
				Imaging Report")				
2	>	HAS CONCEPT MOD	INCLUDE	DTID (1204) Language of Content Item and Descendants	1	M		
3	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0101) Thyroid Imaging Finding Section	1	M		
4	>	CONTAINS	INCLUDE	DTID (G301) GE Ultrasound User Defined Measurement	1-n	M		

17.5.3 TID GEMS_US_SP_0101 Thyroid Imaging Finding Section

TABLE 17-93 GEMS_US_SP_0101

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
1			CONTAINER	EV (121070, DCM, "Findings")	1	M			
2	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	M		EV (T-D1600, SRT, "Neck")	
3	>	HAS CONCEPT MOD	CODE	EV (G-0373, SRT, "Image Mode")	1	M		EV (G-03A2, SRT, "2D mode")	
4	>	CONTAINS	TEXT	EV (121106, DCM, "Comments")	1	U			
5	>	CONTAINS	CODE	EV(G-C102, SRT, Extent of Resection)	1	U		GE Value	DICOM Map
								"Totally"	(R-40507, SRT, Total)
								"Partially"	(R-404FE, SRT, Partial)
6	>	CONTAINS	CODE	EV(GEU-1007-04, 99GEMS, "Appearance")	1	U		GE Value	DICOM Map
								"Within normal limits"	(GEU-1007-75, 99GEMS, "Within normal limits")
								"Abnormal"	(R-42037, SRT, "Abnormal")
								"Symmetric"	(GEU-1007-05, 99GEMS, "Symmetric")
								"Asymmetric R>L"	(GEU-1007-76, 99GEMS, "Asymmetric R>L")
								"Asymmetric L>R"	(GEU-1007-77, 99GEMS, "Asymmetric L>R")
7	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0102) Thyroid Lobe Section	1	U		\$SectionLaterality = EV (G-A101, SRT, "Left")	
8	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0102) Thyroid Lobe Section	1	U		\$SectionLaterality = EV (G-A100, SRT, "Right")	
9	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0103) Lymph Node Section	1	U		\$SectionLaterality = EV (G-A101, SRT, "Left")	
10	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0103) Lymph Node Section	1	U		\$SectionLaterality = EV (G-A100, SRT, "Right")	

	N L	Rel with Parent	VT	Concept Name	V M	Req Type	Condit ion	Value Set Constraint
11	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0103) Lymph Node Section	1	U		\$SectionLaterality = EV (GEU-1007-47, 99GEMS, "Midline")
12	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0104) Parathyroid Section	1	U		\$SectionLaterality = EV (G-A101, SRT, "Left")
13	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0104) Parathyroid Section	1	U		\$SectionLaterality = EV (G-A100, SRT, "Right")
14	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0105) Isthmus Section	1	U		
15	>	CONTAINS	INCLUDE	Table 17-90 Shear wave Measurements	1	M		\$SectionLaterality = EV (G-A101, SRT, "Left")
16	>	CONTAINS	INCLUDE	Table 17-90 Shear wave Measurements	1	M		\$SectionLaterality = EV (G-A100, SRT, "Right")

TABLE 17-94 GEMS_US_SP_0102

Parameter Name	Parameter Usage
\$SectionLaterality	The laterality (if any) of the anatomy in this section heading

17.5.4 TID GEMS_US_SP_0102 Thyroid Lobe Section

TABLE 17-95

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
1			CONTAINER	EV (T-B6000, SRT, “Thyroid”)	1	M			
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, “Laterality”)	1	M		\$SectionLaterality	
3	>	CONTAINS	CODE	EV (G-C102, SRT, Extent of Resection)	1	U		<div>GE Value</div> <div>“Totally”</div> <div>“Partially”</div>	<div>DICOM Map</div> <div>EV (GEU-1007-21, 99GEMS, “Totally”)</div> <div>EV (GEU-1007-22, 99GEMS, “Partially”)</div>
4	>	CONTAINS	CODE	EV (110849, DCM, "Echogenicity")	1	U		<div>GE Value</div> <div>“Homogeneous”</div> <div>“Coarse”</div> <div>“Heterogeneous”</div> <div>“HashimotoClassic”</div> <div>"HashimotoProbable"</div>	<div>DICOM Map</div> <div>(GEU-1007-23, 99GEMS, “Homogeneous”)</div> <div>(GEU-1007-24, 99GEMS, “Coarse”)</div> <div>(GEU-1007-63, 99GEMS, “Heterogeneous”)</div> <div>(GEU-1007-73, 99GEMS, “HashimotoClassic”)</div> <div>(GEU-1007-74, 99GEMS, “HashimotoProbable”)</div>
5	>	CONTAINS	CODE	EV (111372, DCM, “Vascularity”)	1	U		<div>GE Value</div> <div>“Normal”</div> <div>“Increased”</div>	<div>DICOM Map</div> <div>(GEU-1007-54, 99GEMS, “Normal”)</div> <div>EV (111377, DCM, "Diffusely increased</div>

NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
								vascularity in surrounding tissue")
							"Decreased"	(GEU-1007-64, 99GEMS, "Decreased")
6	>	CONTAINS	CODE	EV (112025, DCM, "Size Descriptor")	1	U		
							GE Value	DICOM Map
							"Normal"	EV (R-404A9, SRT, "Medium")
							"Enlarged"	EV (R-40750, SRT, "Enlarged")
							"Small"	EV (R-404A8, SRT, "Small")
7	>	CONTAINS	TEXT	EV (121106, DCM, "Comments")	1	U		
8	>	CONTAINS	INCLUDE	DTID (1400) Linear Measurement	1-n	U		
9	>	CONTAINS	INCLUDE	DTID (1402) Volume Measurement	1	U		
10	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0106) Nodules Section	1	U		

17.5.5 TID GEMS_US_SP_0103 Lymph Node Section

TABLE 17-6 GEMS_US_SP_0103

Parameter Name	Parameter Usage
\$SectionLaterality	The laterality (if any) of the anatomy in this section heading

NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
1		CONTAINER	EV (T-C4000, SRT, "Lymph Node")	1	M			
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		\$SectionLaterality
3	>	CONTAINS	CODE	EV (121071, DCM, "Finding")	1-n	M		
							GE Value for Composition	DICOM Map
							Solid	(111462, DCM, "Solid mass")
							Cystic	(GEU-1007-56, 99GEMS, "Cystic")
							Complex	(GEU-1007-61, 99GEMS, "Complex")
4	>>	HAS CONCEPT MOD	CODE	EV(GEU-1007-48, 99GEMS, "Location")	1	U		
							GE Value	DICOM Map
							"Submental"	(GEU-1007-38, 99GEMS, "Submental")
							"Submandibular"	(GEU-1007-39, 99GEMS, "Submandibular")
							"Parotid"	(GEU-1007-40, 99GEMS, "Parotid")
							"Upper cervical"	(GEU-1007-41, 99GEMS, "Upper cervical")

NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
							"Middle cervical"	(GEU-1007-42, 99GEMS, "Middle cervical")
							"Lower cervical"	(GEU-1007-43, 99GEMS, "Lower cervical")
							"Supraclavicular fossa"	(GEU-1007-44, 99GEMS, "Supraclavicular fossa")
							"Posterior triangle"	(GEU-1007-45, 99GEMS, "Posterior triangle")
5 >>	HAS PROPERTIES	CODE	EV(GEU-1007-04, 99GEMS, "Appearance")	1	U		GE Value	DICOM Map
							"Within normal limits"	(GEU-1007-75, 99GEMS, "Within normal limits")
							"Pathologic"	(GEU-1007-07, 99GEMS, "Pathologic")
							"Suspicious"	(GEU-1007-70, 99GEMS, "Suspicious")
6 >>	HAS PROPERTIES	CODE	EV (111372, DCM, "Vascularity")	1	U		GE Value	DICOM Map
							"Normal"	EV (GEU-1007-54, 99GEMS, "Normal")
							"Increased hilar"	(GEU-1007-71, 99GEMS, "Increased hilar")
							"Increased non-hilar"	(GEU-1007-72, 99GEMS, "Increased non-hilar")
7 >>	HAS PROPERTIES	TEXT	EV (121106, DCM, "Comments")	1	U			
8 >>	HAS PROPERTIES	INCLUDE	DTID (1400) Linear Measurement	1-n	U			
9 >>	HAS PROPERTIES	INCLUDE	DTID (1401) Area Measurement	1-n	U			
10 >>	HAS PROPERTIES	NUM	EV(GEU-1007-13, 99GEMS, "Cortical Thickness")	1	U		UNITS = DCID(7460) "Units of Linear Measurement"	

17.5.6 TID GEMS_US_SP_0104 Parathyroid Section

TABLE 17-97 GEMS_US_SP_0104

Parameter Name	Parameter Usage
\$SectionLaterality	The laterality (if any) of the anatomy in this section heading

NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1		CONTAINER	EV (T-B7000, SRT, "Parathyroid")	1	M		
2 >	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	M		\$SectionLaterality

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint			
3	>	CONTAINS	CODE	EV (121071, DCM, “Finding”)	1-n	M		EV (T-B7000, SRT, “Parathyroid”)			
4	>>	HAS PROPERTIES	CODE	EV(GEU-1007-51, 99GEMS,“Visibility”)	1	U			GE Value	DICOM Map	
									"Visualized"	(GEU-1007-10, 99GEMS,“Visualized”)	
									"Not Visualized"	(GEU-1007-55, 99GEMS,“Not Visualized”)	
5	>>	HAS CONCEPT MOD	CODE	EV(GEU-1007-48, 99GEMS,“Location”)	1	U			GE Value	DICOM Map	
									"Upper Gland"	(GEU-1007-49, 99GEMS,“Upper Gland”“”)	
									"Lower Gland"	(GEU-1007-50, 99GEMS,“Lower Gland”)	
6	>>	HAS PROPERTIES	TEXT	EV (121106, DCM, “Comments”)	1	U					
7	>>	HAS PROPERTIES	INCLUDE	DTID (1400) Linear Measurement	1-n	U					

17.5.7 Table 17-98 TID GEMS_US_SP_0105 Isthmus Section

TABLE 17-98 GEMS_US_SP_0105

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (T-B6300, SRT, "thyroid isthmus")	1	M		
2	>	CONTAINS	TEXT	EV (121106, DCM, "Comments")	1	U		
3	>	CONTAINS	NUM	EV(GEU-1007-12, 99GEMS, "Isthmus AP")	1	U		UNITS = DCID(7460) "Units of Linear Measurement"
4	>	CONTAINS	INCLUDE	DTID (GEMS_US_SP_0106) Nodules Section	1	U		

17.5.8 TID GEMS_US_SP_0106 Nodules Section

TABLE 17-179 GEMS_US_SP_0106

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint											
1			CONTAINER	EV (M-03010, SRT, "Nodule")	1	M													
2	>	CONTAINS	CODE	EV (121071, DCM, "Finding")	1-n	M		<table><tr><th>GE Value for Composition</th><th>DICOM Map</th></tr><tr><td>Solid</td><td>(111462, DCM, "Solid mass")</td></tr><tr><td>Cystic</td><td>(GEU-1007-56, 99GEMS, "Cystic")</td></tr><tr><td>Mixed cystic and Solid</td><td>(GEU-1007-20, 99GEMS, "Mixed cystic and solid")</td></tr><tr><td>Spongiform</td><td>(GEU-1007-81, 99GEMS,</td></tr></table>	GE Value for Composition	DICOM Map	Solid	(111462, DCM, "Solid mass")	Cystic	(GEU-1007-56, 99GEMS, "Cystic")	Mixed cystic and Solid	(GEU-1007-20, 99GEMS, "Mixed cystic and solid")	Spongiform	(GEU-1007-81, 99GEMS,	
GE Value for Composition	DICOM Map																		
Solid	(111462, DCM, "Solid mass")																		
Cystic	(GEU-1007-56, 99GEMS, "Cystic")																		
Mixed cystic and Solid	(GEU-1007-20, 99GEMS, "Mixed cystic and solid")																		
Spongiform	(GEU-1007-81, 99GEMS,																		

**LOGIQ E10 R2.X / LOGIQ E10s R2.X
LOGIQ E11 R2.X / LOGIQ E20 R2.X
DICOM CONFORMANCE STATEMENT**

GE Healthcare

DIR DOC2107162 REV 4

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint		
3	>>	HAS CONCEPT MOD	CODE	EV (GEU-1007-95, 99GEMS, "Nodule Number")	1	M				
4	>>	HAS CONCEPT MOD	CODE	EV (GEU-1007-28, 99GEMS, "Location A")	1	U		GE Value	DICOM Code	
								"Upper"	(GEU-1007-29, 99GEMS, "Upper")	
								"Mid"	(GEU-1007-30, 99GEMS, "Mid")	
								"Lower"	(GEU-1007-31, 99GEMS, "Lower")	
5	>>	HAS CONCEPT MOD	CODE	EV (GEU-1007-32, 99GEMS, "Location B")	1	U		GE Value	DICOM Code	
								"Lateral"	(GEU-1007-33, 99GEMS, "Lateral")	
								"Midline"	(GEU-1007-35, 99GEMS, "Midline")	
								"Medial"	(GEU-1007-34, 99GEMS, "Medial")	
6	>>	HAS PROPERTIES	CODE	EV (M-020F9, SRT, "Shape")	1	U		GE Value	DICOM Code	
								Wider-than-tall	(GEU-1007-84, 99GEMS, "Wider-than-tall")	
								Taller-than-wide	(GEU-1007-85, 99GEMS, "Taller-than-wide")	
7	>>	HAS PROPERTIES	CODE	EV (111037, DCM, "Margins")	1	U		GE Value	DICOM Code	
								Smooth	(G-A545, SRT, "Smooth")	
								Ill-defined	(GEU-1007-86, 99GEMS, "Ill-defined")	
								Lobulated	(112135, DCM, "Lobulated")	
								Irregular	(G-A402, SRT, "Irregular")	
								Extra-thyroidal extension	(GEU-1007-87, 99GEMS, "Extra-thyroidal extension")	
8	>>	HAS PROPERTIES	CODE	EV (111372, DCM, "Vascularity")	1	U		GE Value	DICOM Map	
								"Normal"	EV (GEU-1007-54, 99GEMS, "Normal")	
								"Increased"	EV (111377, DCM, "Diffusely increased vascularity in surrounding tissue")	
								"Decreased"	(GEU-1007-64, 99GEMS, "Decreased")	
								"Central vasc avascular"	(GEU-1007-65, 99GEMS, "Central vasc avascular")	
								"Central vasc hypovascular"	(GEU-1007-66, 99GEMS, "Central vasc hypovascular")	
								"Central vasc isovascular"	(GEU-1007-67, 99GEMS, "Central vasc isovascular")	
								"Central vasc	(GEU-1007-68, 99GEMS,	

NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint	
							hypervascular"	"Central vasc hypervascular")
							"Central vasc severely hypervascular"	(GEU-1007-69, 99GEMS, "Central vasc severely hypervascular")
9	>>	HAS PROPERTIES	CODE	EV (111009, DCM, "Calcification Type")			GE Value	DICOM Map
							"No calcification"	(GEU-1007-78, 99GEMS, "No calcification")
							"Coarse central"	EV (GEU-1007-15, 99GEMS, "Coarse central")
							"Coarse rim"	EV (GEU-1007-16, 99GEMS, "Coarse rim")
							"Punctate scattered"	EV (GEU-1007-79, 99GEMS, "Punctate scattered")
							"Punctate clumped"	EV (GEU-1007-80, 99GEMS, "Punctate clumped")
							"Colloid"	EV (GEU-1007-19, 99GEMS, "Colloid")
							"Mixed"	EV (GEU-1007-20, 99GEMS, "Mixed")
10	>>	HAS PROPERTIES	CODE	EV (GEU-1007-91, 99GEMS, "Echogenic Foci")			GE Value	DICOM Map
							None	(R-40775, SRT, "None")
							Comet	EV (GEU-1007-88, 99GEMS, "Comet")
							Macrocalcifications	EV (111345, DCM, "Macrocalcifications")
							Peripheral calcifications	EV (GEU-1007-89, 99GEMS, "Peripheral calcifications")
							Punctate echogenic foci	EV (GEU-1007-90, 99GEMS, "Punctate echogenic foci")
11	>>	HAS PROPERTIES	TEXT	(GEU-1007-93, 99GEMS, "Total Points")	1	U		
12	>>	HAS PROPERTIES	TEXT	(GEU-1007-92, 99GEMS, "TI-RADS Level")	1	U		
13	>>	HAS PROPERTIES	TEXT	(GEU-1007-94, 99GEMS, "TI-RADS source")	1	U		
14	>>	HAS PROPERTIES	TEXT	(GEU-1006-12, 99GEMS, "Transaction UID")	1	U	<unique identifier>	
15	>>	HAS PROPERTIES	TEXT	(GEU-1006-13, 99GEMS, "Koios DS Version")	1	U	<version info>	
16	>>	HAS PROPERTIES	TEXT	(GEU-1006-14, 99GEMS, "Koios DS Assessment Score")	1	U	<score info>	
17	>>	HAS PROPERTIES	TEXT	(GEU-1006-15, 99GEMS, "Koios DS Assessment")	1	U	<assessment info>	
18	>>	HAS PROPERTIES	TEXT	EV (121106, DCM, "Comments")	1	U		
19	>>	HAS PROPERTIES	INCLUDE	DTID (1400) Linear Measurement	1-n	U		

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
20	>>	HAS PROPERTIES	INCLUDE	DTID (1402) Volume Measurement	1	U		

17.5.9 TID 1400 Linear Measurement Template

TABLE 17-100 TID 1400

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
1			NUM	GE Measurement	DICOM Map	1	M		UNITS = DCID(7460) "Units of Linear Measurement"
				Length	(G-A22A, SRT, "Length")				
				Width	(G-A220, SRT, "Width")				
				Height	(121207, DCM, "Height")				

17.5.10 TID 1402 Volume Measurement Template

TABLE 17-101 TID 1402

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			NUM	EV(G-D705, SRT, "Volume")	1	M		Value shall be > 0 UNITS = DCID(7462) "Units of Volume Measurement"

17.5.11 ID 1204 Language of Content Item and Descendants

TABLE 17-102 TID 1204

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1		HAS CONCEPT MOD	CODE	(121049,DCM,"Language of Content Item and Descendants")	1	M		(en, RFC3066, "English")

17.5.12 TID 1401 Area Measurement Template

TABLE 17-103 TID 1401

	NL	Rel with Parent	VT	Concept Name		VM	Req Type	Condition	Value Set Constraint
1			NUM	GE Measurement	DICOM Map	1	M		Value shall be > 0 UNITS = DCID(7461) "Units of Area Measurement"
				SP/2D/ACArea	(GEU-1006-11, 99GEMS, "Auto Contour Area")				

17.6 KEY OBJECT SELECTION DOCUMENT TEMPLATE

This section describes the contents of the Key Object Selection Document Template (TID 2010) SR.

17.6.1 TID 2010 Template

TABLE 17-104 TID 2010 KEY OBJECT SELECTION DOCUMENT

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	(DCM, 113001, "Rejected for Quality Reasons")	1	M		
2	>	HAS CONCEPT MOD	INCLUDE	DTID(1204) Language of Content Item and Descendants	1	U		(en, RFC3066, "English")
3	>	HAS OBS CONTEXT	INCLUDE	DTID(1002) Observer Context	1-n	U		
4	>	CONTAINS	TEXT	EV(113012, DCM, "Key Object Description")	1	U		"List of Rejected Images for Quality"
5	>	CONTAINS	IMAGE	Purpose of Reference shall not be present	1-n	MC		List of image instance UIDs that are marked for deletion

17.7 USER DEFINED TEMPLATE STRUCTURE

The General Ultrasound User Defined Measurement Template provides a CONTAINER with a structure for reporting user-defined measurements and calculations.

The difference between this from TID 5103, TID 300 or 1414 for general measurement is the template uses labels that specified by users at configuration time rather assigning specific codes to the individual measurements.

Note: In practice, to assign specific codes to user defined measurements are impractical and cumbersome since the system/organization has to maintain database/map of codes and labels with public/private code designator schemes.

17.7.1 TID G301 Template

TABLE 17-105 TID G301 USER DEFINED TEMPLATE

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CONTAINER	EV (121070, DCM, "Findings")	1	M		
2	>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	MC	IFF anatomy has laterality	DCID (244) Laterality
3	>	HAS CONCEPT MOD	CODE	EV (G-0373, SRT, "Image Mode")	1	M		See Table 16-7
4	>	HAS CONCEPT MODE	CONTAINER	EV (GEU-1005-6, 99GEMS, 'User Vessel Anatomy') or (GEU-1005-3, 99GEMS, 'General Anatomy' or (125007, DCM, 'Measurement Group') or	1	MC	IFF there exists user defined measurements	Note: User Vessel Anatomy is mainly used to define vessels and for inserting the Study/Vessel template. General Anatomy is mainly used in PWD and B-Mode and when the user has added individual folders and

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
				(125005, DCM, 'Biometry Group') or (99100, DCM, 'Doppler Group')				measurements.
5	>>	HAS CONCEPT MOD	TEXT	DT (GEU-1005-7, 99GEMS, 'Anatomy Label') or DT (GEU-1002-58, 99GEMS, 'Biometry Group Name') or DT (GEU-1002-60, 99GEMS, 'Doppler Group Name')	1	MC	IFF the container uses Finding as Concept Name	Labels from user which defined at configuration
6	>>	CONTAINS	INCLUDE	DTID (300) Measurement – see section: 16.1.7	1-n	M	IFF there exists user defined measurements	\$Measurement = When 'Add Folder' insert study measurements by prepopulated study – Please refer to section 16.1.5 Vascular Ultrasound Measurement Group \$Units = DCID (82) Units of Measurement \$Derivation = DCID (3627) Measurement Type \$Selection = DCID (224) Selection Method
7	>>>	HAS CONCEPT MOD	TEXT	(GEU-1005-5, 99GEMS, 'Measurement Label')	1	MC	IFF the container uses Finding as Concept Name	Labels from user which defined at configuration

17.8 GE ULTRASOUND M&A REPORT

LOGIQ Scanner supports the following private templates GEMS_US_0100 for SOP Instances created by this product when exam types selected are Small Parts, Urology and Pediatrics.

17.8.1 TID GEMS_US_0100 Template


TABLE 17-106 GEMS_US_0100

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
			CONTAINER	EV (43126, 99GEMS, 'ultrasound M&A Document')	1	M		
	>	CONTAINS	UIDREF	EV (43140, 99GEMS, 'Series Instance UID')	1	U		
	>	CONTAINS	TEXT	EV(29463-7, LN, 'Patient Weight')	1	U		
	>	CONTAINS	TEXT	EV(8302-2, LN, 'Patient Height')	1	U		
	>	CONTAINS	TEXT	EV (43148, 99GEMS, 'Ultrasound Category')	1	U		
	>	CONTAINS	TEXT	EV (11878-6, LN, 'Number of Fetuses')	1	U		
	>	CONTAINS	TEXT	EV(43139, 99GEMS, 'EDD method')	1	U		
	>	CONTAINS	CONTAINER	(43127, 99GEMS, "ultrasound parameter")	1-n	U		
	>>	CONTAINS	TEXT	(43127, 99GEMS, "ultrasound parameter averaging method")	1	M		
	>>	CONTAINS	TEXT	(43143, 99GEMS, "ultrasound display mode")	1	M		
	>>	CONTAINS	TEXT	(43132, 99GEMS, "ultrasound parameter preferred display unit")	1	M		
	>>	CONTAINS	TEXT	(43137, 99GEMS, "ultrasound parameter value, manually edited")	1	M		
	>>	CONTAINS	TEXT	(43135, 99GEMS, "ultrasound parameter, exclude from averaging")	1	M		
	>>	CONTAINS	TEXT	(43136, 99GEMS, "ultrasound parameter, exclude from calculation")	1	M		
	>>	CONTAINS	TEXT	(43148, 99GEMS, "ultrasound category")	1	M		
	>>	CONTAINS	TEXT	(43141, 99GEMS, "ultrasound parameter, mark deleted")	1	M		

	NL	Relation with Parent	Value Type	Concept Name	VM	Req Type	Condition	Value Set Constraint
	>>	CONTAINS	TEXT	(43149, 99GEMS, "ultrasound parameter, srStudy text")	1	M		
	>>	CONTAINS	TEXT	(43146, 99GEMS, "ultrasound parameter, measurement tool")	1	M		
	>>	CONTAINS	TEXT	(43144, 99GEMS, "ultrasound image mode")	1	M		
	>>	CONTAINS	TEXT	(43131, 99GEMS, "ultrasound parameter ID")	1	M		
	>>	CONTAINS	TEXT	(43133, 99GEMS, "ultrasound parameter preferred display name")	1	M		
	>>	CONTAINS	TEXT	(43134, 99GEMS, "ultrasound parameter value source")	1	M		
	>>	CONTAINS	TEXT	(43142, 99GEMS, "Result Number")	1	M		
	>>	CONTAINS	TEXT	(43145, 99GEMS, "ultrasound study type")	1	M		
	>>	CONTAINS	TEXT	(43129, 99GEMS, "ultrasound parameter textual value")	1	M		
	>>	CONTAINS	NUM	(43128, 99GEMS, "ultrasound parameter numeric value")	1	M		

17.9 TID 300 MEASUREMENT

This Template provides a general structure for a numeric measurement, together with evaluations of its normality and/or significance, and the inference source(s) for its value. This structure is instantiated by inclusion of this Template with specific contextual parameters from a parent Template. This follows the Measurement Templates section found within part 16 of the DICOM Standard.

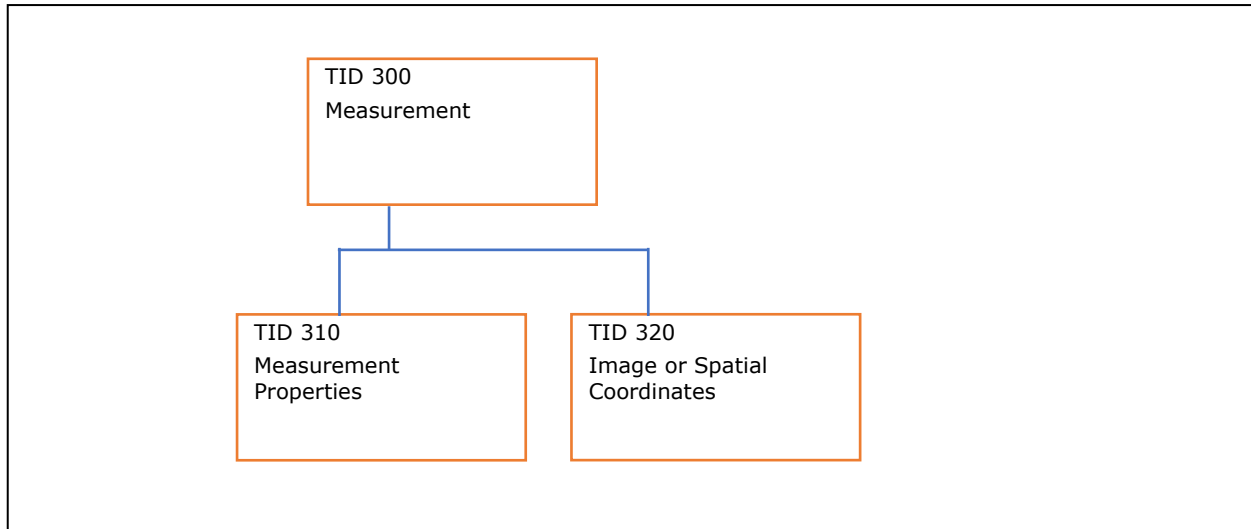

17.9.1 TID 300 Measurement

TABLE 17-107 TID 300 MEASUREMENT

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			NUM	\$Measurement	1	M		\$UNITS = \$Units
2	>	HAS CONCEPT MOD	CODE	\$ModType	1-n	U		\$ModValue
3	>	HAS CONCEPT MOD	CODE	EV (G-036, SRT, "Measurement Method")	1	U		\$Method
4	>	HAS CONCEPT MODE	CODE	EV (121401, DCM, "Derivation")	1	U		\$Derivation
5	>	HAS CONCEPT MOD	CODE	EV (G-C0E3, SRT, "Finding Site")	1	U		\$TargetSite
6	>>	HAS CONCEPT MOD	CODE	EV (G-C171, SRT, "Laterality")	1	U		DCID 244 "Laterality"
7	>>	HAS CONCEPT MOD	CODE	DT (G-A1F8, SRT, "Stenosis Modifier")	1	U	IFF Stenosis Modifier is defined	See Table 17-10 GE Ultrasound Vessel Location
8	>>	HAS CONCEPT MOD	CODE	DT (G-A1F8, SRT, "Anatomic Modifier")	1	U	IFF Anatomic Modifier is defined	
9	>>	HAS CONCEPT MOD	CODE	DT (G-A1F8, SRT, "Topographical Modifier")	1	U		\$TargetSiteMod

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
10	>	HAS CONCEPT MOD	INCLUDE	DTID 310 "Measurement Properties"	1	U		Only applicable to OB exams. \$RefAuthority = \$RefAuthority \$RangeAuthority = \$RangeAuthority
11	>	INFERRED FROM	NUM	\$DerivationParameter	1-n	UC	XOR Row 10	\$UNITS = \$DerivationParameterUnits
12	>	R-INFERRED FROM	NUM	\$DerivationParameter	1-n	UC	XOR Row 9	\$UNITS = \$DerivationParameterUnits
13	>	INFERRED FROM	INCLUDE	DTID 315 "Equation or Table"	1	UC	XOR Row 12	\$Equation = \$Equation
14	>	INFERRED FROM	TEXT	DCID 228 "Equation of Spatial Coordinates"	1-n	U		
15	>		INCLUDE	DTID 320 "Image or Spatial Coordinates"	1-n	U		\$Purpose = \$ImagePurpose
16	>	HAS OBS CONTEXT	INCLUDE	DTID 4108 "Tracking Identifier"	1	U		

17.9.2 TID 310 Measurement

TABLE 17-1808 TID 310 MEASUREMENT PROPERTIES

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1			CODE	EV (121402, DCM, "Normality")	1	U		DCID 222 "Normality Codes"
2			INCLUDE	DTID 311 "Measurement Statistical Properties"	1	U		\$RefAuthority = \$RefAuthority
3			INCLUDE	DTID 312 "Normal Range Properties"	1	U		\$RangeAuthority = \$RangeAuthority
4			NUM	DCID 225 "Measurement Uncertainty Concepts"	1-n	U		
5			CODE	EV (121404, DCM, "Selection Status")	1	U		DCID 224 "Selection Method"

17.9.3 TID 320 Image or Spatial Coordinates

TABLE 17-109 TID 320 IMAGE OR SPATIAL COORDINATES

	NL	Rel with Parent	VT	Concept Name	VM	Req Type	Condition	Value Set Constraint
1		INFERRED FROM	IMAGE	\$Purpose	1	MC		IFF image is present and functionality is enabled.

18. GRAYSCALE DISPLAY CONSISTENCY

The high-resolution display monitor attached to the LOGIQ E10 scanner is adjusted to meet the Grayscale Standard Display Function (GSDF). The implementation follows the image viewing pipeline as defined by the DICOM standard. The displayed/saved images are applied the Modality LUT Transformation, Window/Level Transformation and Presentation LUT Transformation. Provided luminance test patterns can be used together with a luminance meter to measure the performance of the display system. See the service manual for details on how to perform these measurements. When the Grayscale Standard Display Function (GSDF) is being applied to an image the Presentation LUT Shape tag (2050,0020) within the General Image Module is set to IDENTITY. This tag is also a part of the Enhanced US Image Module.

19. AUDIT TRAIL MESSAGE FORMAT PROFILE

19.1 Supported Audit Message List

This section described the list of Audit Messages implemented by this product to conform to the DICOM Audit Profile.

TABLE 19-1

Audit Message	Usage
Application Activity	Used
Audit Log Used	Not Used
Begin Transferring DICOM Instances	Used
Data Export	Used
Data Import	Used
DICOM Instances Accessed	Used
DICOM Instances Transferred	Used
DICOM Study Deleted	Used
Network Entry	Not Used
Query	Used
Security Alert	Used
User Authentication	Used
Order Record	Not Used
Patient Record	Not Used
Procedure Record	Not Used

19.2 Audit Message Description

The following subsections define message details and specializations used by this product as part of the DICOM Audit Trail Profile.

19.2.1 Application Activity

This audit message describes the event of an application starting or stopping.

TABLE 19-2
APPLICATION ACTIVITY MESSAGE

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110100, DCM, "Application Activity")
	EventActionCode	M	Enumerated Value E = Execute
	EventDateTime	M	System Date/Time, e.g. 2019-04-18T20:15:20.1540000-06:00

Real World Entities	Field Name	Opt.	Value Constraints
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: Application started (1)	UserID	M	Empty
	AlternativeUserID	MC	LOGIQE10
	UsersRequestor	M	true
	RoleIDCode	M	EV (110150, DCM, "Application")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device)

19.2.2 Begin Transferring DICOM Instances

This message describes the event of a system beginning to transfer a set of DICOM instances from one node to another node within control of the system's security domain. This message only includes information about a single patient.

TABLE 19-3
AUDIT MESSAGE FOR BEGIN TRANSFERRING DICOM INSTANCES

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110102, DCM, "Begin Transferring DICOM Instances")
	EventActionCode	M	Enumerated Value E = Execute
	EventDateTime	M	System Date/Time, e.g.2019-04-18T20:53:57.7320000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: Process Sending the Data (1)	UserID	M	Empty
	AlternativeUserID	U	Empty
	UsersRequestor	M	false
	RoleIDCode	M	EV (110153, DCM, "Source Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Active Participant: Process receiving the data (1)	UserID	M	Empty
	AlternativeUserID	U	Empty
	UsersRequestor	M	false
	RoleIDCode	M	EV (110152, DCM, "Destination Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address.

Real World Entities	Field Name	Opt.	Value Constraints
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device")
Participating Object:	ParticipantObjectTypeCode	M	Enumerated Value: 2 = system
Studies being transferred (1..N)	ParticipantObjectTypeCodeRole	M	Enumerated Value: 3 = report
	ParticipantObjectIDTypeCode	M	EV (110180, DCM, "Study Instance UID")
	ParticipantObjectID	M	The Study Instance UID
Participating Object: Patient (1)	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = person
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.3 Data Export

This message describes the event of exporting data from the system, meaning that the data is leaving control of the system's security domain. Example of exporting includes printing to paper, conversion to another format of storage in an EHR, writing to removeable media or sending via e-mail. A single patient is described in one event message.

TABLE 19-4
AUDIT MESSAGE FOR DATA EXPORT

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110106, DCM, "Export")
	EventActionCode	M	Enumerated Value: R = Read
	EventDateTime	M	System Date/Time, e.g.2019-04-22T11:19:42.7020000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: User or Process Exporting the data(1..2)	UserID	M	2
	UserIsRequestor	M	true
	RoleIDCode	M	EV (110153, DCM, "Source Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Active Participant:	UserID	M	2
	UserIsRequestor	M	false

Real World Entities	Field Name	Opt.	Value Constraints
Media (1)	RoleIDCode	M	EV (110154, DCM, "Destination Media")
	MediaIdentifier	M	Empty
	MediaType	M	EV (110030, DCM, "USB Disk Emulation EV (110033, DCM, "DVD")
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device)
Participating Object: Patients (1..N)	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = patient
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.4 Data Import

This message describes the event of importing data into an organization, implying that the data now entering the system was not under the control of the security domain of this organization. An example of importing is creating new local instances from data on removable media. A single patient is described in one event message. A single user (either local or remote) shall be identified as the requestor, i.e., UserIsRequestor with a value of TRUE. This accommodates both push and pull transfer models for media.

TABLE 19-5
Audit Message for Data Import

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110107, DCM, "Import")
	EventActionCode	M	Shall be: C = Create
	EventDateTime	M	System Date/Time, e.g.2019-04-22T11:42:56.3850000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: User or Process Importing the data (1..n)	UserID	M	2
	UserIsRequestor	M	true
	RoleIDCode	M	EV (110152, DCM, "Destination Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Active Participant:	UserID	M	2
	UserIsRequestor	M	false

Real World Entities	Field Name	Opt.	Value Constraints
Source Media (1)	RoleIDCode	M	EV (110155, DCM, "Source Media")
	MediaIdentifier	M	Empty
	MediaType	M	EV (-1, DCM, "")
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device)
Participating Object: Patients (1..N)	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = patient
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.5 DICOM Instances Accessed

This message describes the event of DICOM SOP Instances being viewed, utilized, updated, or deleted. This message shall only include information about a single patient and can be used to summarize all activity for several studies for that patient. This message records the studies to which the instances belong, not the individual instances. If all instances within a study are deleted, then the EV(110105, DCM, "DICOM Study Deleted") event is used, see Section 19.2.7 DICOM Study Deleted.

TABLE 19-6
Audit Message for DICOM Instances Accessed

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110103, DCM, "DICOM Instances Accessed")
	EventActionCode	M	Enumerated value:
			C = create
			R = read
			U = update
			D = delete
	EventDateTime	M	System Date/Time, e.g.2019-04-18T22:05:58.8130000-06:00
Active Participant: Person and or Process manipulating the data	EventOutcomeIndicator	M	2
	EventTypeCode	U	"0" Nominal Success "8" Serious failure
	UserID	M	2
	UserIsRequestor	M	true
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address

Real World Entities	Field Name	Opt.	Value Constraints
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device")
Participating Object: Studies (1..N)	ParticipantObjectTypeCode	M	Enumerated Value: 2 = system
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 3 = report
	ParticipantObjectIDTypeCode	M	EV (110180, DCM, "Study Instance UID")
	ParticipantObjectID	M	Filled with Study Instance UID
Participating Object: Patient (1)	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = person
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.6 DICOM Instances Transferred

This message describes the event of the completion of transferring DICOM SOP Instances between two Application Entities. This message only includes information about a single patient.

TABLE 19-7
Audit Message for DICOM Instances Transferred

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110104, DCM, "DICOM Instances Transferred")
	EventActionCode	M	Enumerated Value: R = Read.
	EventDateTime	M	System Date/Time, e.g.2019-04-18T20:53:59.6540000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: Process that sent the data (1)	UserID	M	Empty
	UserIsRequestor	M	false
	RoleIDCode	M	EV (110153, DCM, "Source Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Active Participant: The process that received the	UserID	M	Empty
	UserIsRequestor	M	false
	RoleIDCode	M	EV (110152, DCM, "Destination Role ID")
	NetworkAccessPointTypeCode	U	2

Real World Entities	Field Name	Opt.	Value Constraints
data. (1)	NetworkAccessPointID	U	Local IP Address.
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device)
Participating Object:	ParticipantObjectTypeCode	M	Enumerated Value: 2 = system
Studies being transferred (1..N)	ParticipantObjectTypeCodeRole	M	Enumerated Value: 3 = report
	ParticipantObjectIDTypeCode	M	EV (110180, DCM, "Study Instance UID")
	ParticipantObjectID	M	The Study Instance UID
Participating Object:	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
Patient (1)	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = person
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.7 DICOM Study Deleted

This message describes the event of deletion of `jSelect` one or more studies and all associated SOP Instances of a single Study and its associated SOP Instances in a single action. This message only includes information about a single patient.

TABLE 19-8
Audit Message for DICOM Study Deleted

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110105, DCM, "DICOM Study Deleted")
	EventActionCode	M	Enumerated Value D = Delete
	EventDateTime	M	System Date/Time, e.g. 2019-04-18T22:05:58.7370000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: the person or process deleting the study (1..2)	UserID	M	2
	UserIsRequestor	M	true
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address

Real World Entities	Field Name	Opt.	Value Constraints
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device")
Participating Object: Studies being transferred (1..N)	ParticipantObjectTypeCode	M	Enumerated Value: 2 = system
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 3 = report
	ParticipantObjectIDTypeCode	M	EV (110180, DCM, "Study Instance UID")
	ParticipantObjectID	M	The Study Instance UID
Participating Object: Patient (1)	ParticipantObjectTypeCode	M	Enumerated Value: 1 = person
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 1 = person
	ParticipantObjectIDTypeCode	M	EV (2, RFC-3881, "Patient Number")
	ParticipantObjectID	M	The patient ID

19.2.8 Query

This message describes the event of a Query being issued or received. The message does not record the response to the query, but merely records the fact that a query was issued:

1. Composite Instance Query.

TABLE 19-9
Audit Message for Query

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110112, DCM, "Query")
	EventActionCode	M	Enumerated Value E = Execute
	EventDateTime	M	System Date/Time, e.g. 2019-04-22T10:57:53.0880000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
Active Participant: Process Issuing the Query (1)	UserID	M	Empty
	UserIsRequestor	M	false
	RoleIDCode	M	EV (110153, DCM, "Source Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Active Participant: The process that will respond to	UserID	M	Empty
	UserIsRequestor	M	false
	RoleIDCode	M	EV (110152, DCM, "Destination Role ID")
	NetworkAccessPointTypeCode	U	2

Real World Entities	Field Name	Opt.	Value Constraints
the query (1)	NetworkAccessPointID	U	Local IP Address
Active Participant: Other Participants that are known, especially third parties that requested the query (0..N)	UserID	M	Empty
	UserIsRequestor	M	true
	RoleIDCode	U	EV (110153, DCM, "Source Role ID")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device")
Participating Object: SOP Queried and the Query (1)	ParticipantObjectTypeCode	M	Enumerated Value: 2 = system
	ParticipantObjectTypeCodeRole	M	Enumerated Value: 3 = report
	ParticipantObjectIDTypeCode	M	DT (110181, DCM, "SOP Class UID")
	ParticipantObjectID	M	If the ParticipantObjectIDTypeCode is (110181, DCM, "SOP Class UID"), then this field shall hold the UID of the SOP Class being queried
	ParticipantObjectQuery	M	If the ParticipantObjectIDTypeCode is (110181, DCM, "SOP Class UID"), then this field shall hold the Dataset of the DICOM query, xs:base64Binary encoded. Otherwise, it shall be the query in the format of the protocol used.
	ParticipantObjectDetail	MC	Used if the ParticipantObjectIDTypeCode is (110181, DCM, "SOP Class UID") A ParticipantObjectDetail element with the XML attribute "TransferSyntax" is always present. In this case, the value of the Transfer Syntax attribute is the UID of the transfer syntax of the query and therefore is a DICOM Transfer Syntax. The element content is an xs:base64Binary encoding.

19.2.9 Security Alert

This message describes any event for which a node needs to report a security alert, e.g., a node authentication failure when establishing a secure communications channel.

TABLE 19-10

Audit Message for User Authentication

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110113, DCM, "Security Alert")
	EventActionCode	M	Enumerated Value: E = Execute
	EventDateTime	M	System Date/Time, e.g. 2019-04-18T22:44:10.8080000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
	EventTypeCode	M	EV (110128, DCM, "Network Configuration") or EV (110129, DCM, "Security Configuration") or EV (110130, DCM, "Hardware Configuration") or EV (110131, DCM, "Software Configuration") or EV (110132, DCM, "Use of Restricted Function") or EV (110137, DCM, "User Security Attributes Changed")
Active Participant: Reporting Person and/or Process (1..2)	UserID	M	2
	UserIsRequestor	M	true
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device")

19.2.10 User Authentication

This message describes the event that a user has attempted to log on or log off. This report is made regardless of whether the attempt was successful or not. Note The user usually has UserIsRequestor TRUE, but in the case of a logout timer, the Node can be the UserIsRequestor.

TABLE 19-11

Audit Message for User Authentication

Real World Entities	Field Name	Opt.	Value Constraints
Event	EventID	M	EV (110114, DCM, "User Authentication")
	EventActionCode	M	Enumerated Value: E = Execute
	EventDateTime	M	System Date/Time, e.g. 2019-04-22T11:53:28.4490000-06:00
	EventOutcomeIndicator	M	"0" Nominal Success "8" Serious failure
	EventTypeCode	M	Defined Terms: EV (110123, DCM, "Logout")

GE Healthcare

DIR DOC2107162 REV 4

Real World Entities	Field Name	Opt.	Value Constraints
Active Participant:Person Authenticated or claimed(1)	UserID	M	4
	UsersRequestor	M	true
Active Participant:Node or System performing authentication (0..1)	UserID	M	Empty
	UsersRequestor	M	true
	RoleIDCode	U	EV (110150, DCM, "Application")
	NetworkAccessPointTypeCode	U	2
	NetworkAccessPointID	U	Local IP Address
Audit Source	AuditEnterpriseSiteID	U	Empty
	AuditSourceID	M	LOGIQE10
	AuditSourceTypeCode	U	EV (1, DCM, "End-user display device, diagnostic device)

End of Document