




INNOVA 2100 ELECTROPHYSIOLOGY (EP) LAB

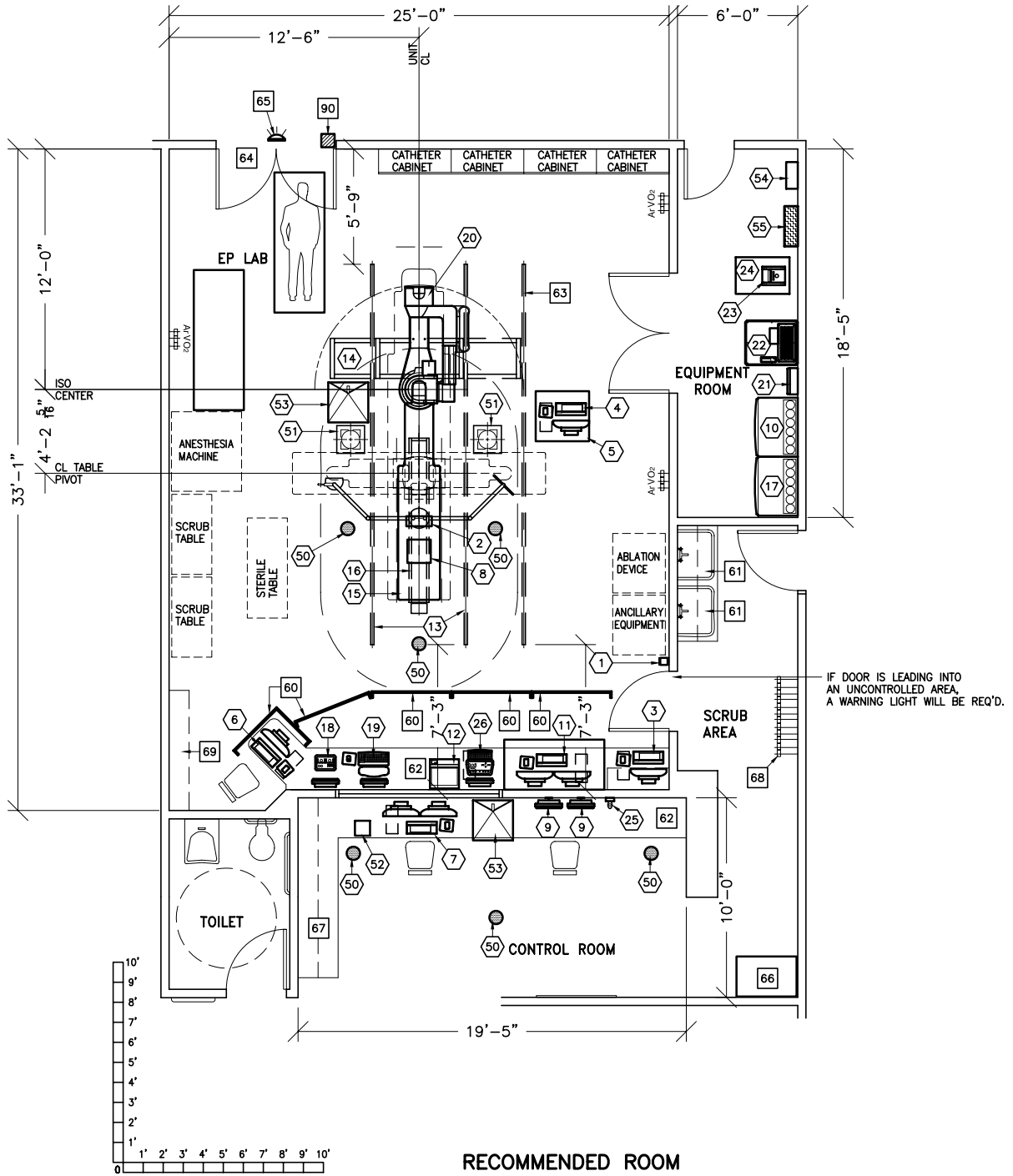

RECOMMENDED ROOM DIMENSIONS
39'-4" x 24'-0"

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
ABSOLUTE MINIMUM:	22'-6" x 13'-9" [6.89m x 4.23m]	9'-6" [2.92m]

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 5-86

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	XR BUZZER <LOCATED ABOVE CEILING>	2 lbs	
②	1	TRAM NET RACK	8 lbs	
③	1	MAPPING WORKSTATION	46 lbs	682 btu
④	1	NURSE WORKSTATION	46 lbs	682 btu
⑤	1	WORKSTATION CART		
⑥	1	CLINICAL WORKSTATION	46 lbs	682 btu
⑦	1	REMOTE WORKSTATION, INCLUDES MONITORS AND KEYBOARD	181 lbs	2935 btu
⑧	1	CLAB 2 PLUS AMPLIFIER	24 lbs	204 btu
⑨	2	18 in. MONITOR ON WALL SUPPORT	26 lbs	204 btu
⑩	1	ATLAS CABINET <C1>	998 lbs	3389 btu
⑪	1	COMBO LAB CONSOLE, INCLUDES MONITORS AND KEYBOARD	203 lbs	2935 btu
⑫	1	COLOR PRINTER		1054 btu
⑬	2	LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	68 lbs	
⑭	1	SIX LCD MONITOR SUSPENSION ON 9 ft. 6 in. XT INBOARD BRIDGE	557 lbs	1228 btu
⑮	1	OMEGA IV/V TABLE WITH ROTATING TOP	1750 lbs	614 btu
⑯	1	COUNTERBALANCED EYE AND THYROID SHIELD WITH LAMP	143 lbs	
⑰	1	ATLAS CABINET <C2>	568 lbs	1825 btu
⑱	1	CONTROL ROOM MONITOR WITH DL KEYPAD	19 lbs	204 btu
⑲	1	OPERATORS CONSOLE	22 lbs	546 btu
⑳	1	INNOVA POSITIONER <REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION>	1653 lbs	2416 btu
㉑	1	UPS INTERFACE BOX		
㉒	1	UPS CABINET	1170 lbs	4050 btu
㉓	1	DETECTOR CHILLER	33 lbs	706 btu
㉔	1	WATER CHILLER	449 lbs	18716 btu
㉕	1	BOLUS CHASE HANDSWITCH	2 lbs	
㉖	1	IVUS VOLCANO S51 CONSOLE, INCLUDES FLAT PANEL MONITOR AND KEYBOARD <DESK MOUNTED>	68 lbs	1631 btu

ENVIRONMENT

Ambient Operation Temperature: 55-75° (F)
 Allowable Temperature Change: 15° (F) per hour
 Humidity: 30%-70% non condensing
 Allowable Humidity Change: 10% per hour
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE 5-86

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
50	6	VITALING SPEAKER		
51	2	SKYTRON LIGHTING UNIT	50 lbs	341 btu
52	1	VITALING CONSOLE		
53	2	VITALING MICROPHONE		
54	1	FILTER ENCLOSURE	90 lbs	
55	1	INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.	275 lbs	1532 btu
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		MOBILE RADIATION SHIELD		
61		SCRUB SINK		
62		COUNTER TOP FOR EQUIPMENT-SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
63		CABLE DRAPE RAIL.		
64		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
65		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
66		CUSTOMER SUPPLIED STORAGE CABINET		
67		COUNTERTOP W BASE AND WALL CABINETS		
68		LEAD APRON RACK		
69		UPPER CASEWORK		
90		X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -CAT. NO. E4502SS FOR WARNING LIGHT & ROOM LIGHT CONTROL.		