



ADVANTX VMP/INNOVA 2000/OMEGA IV/GEMNET

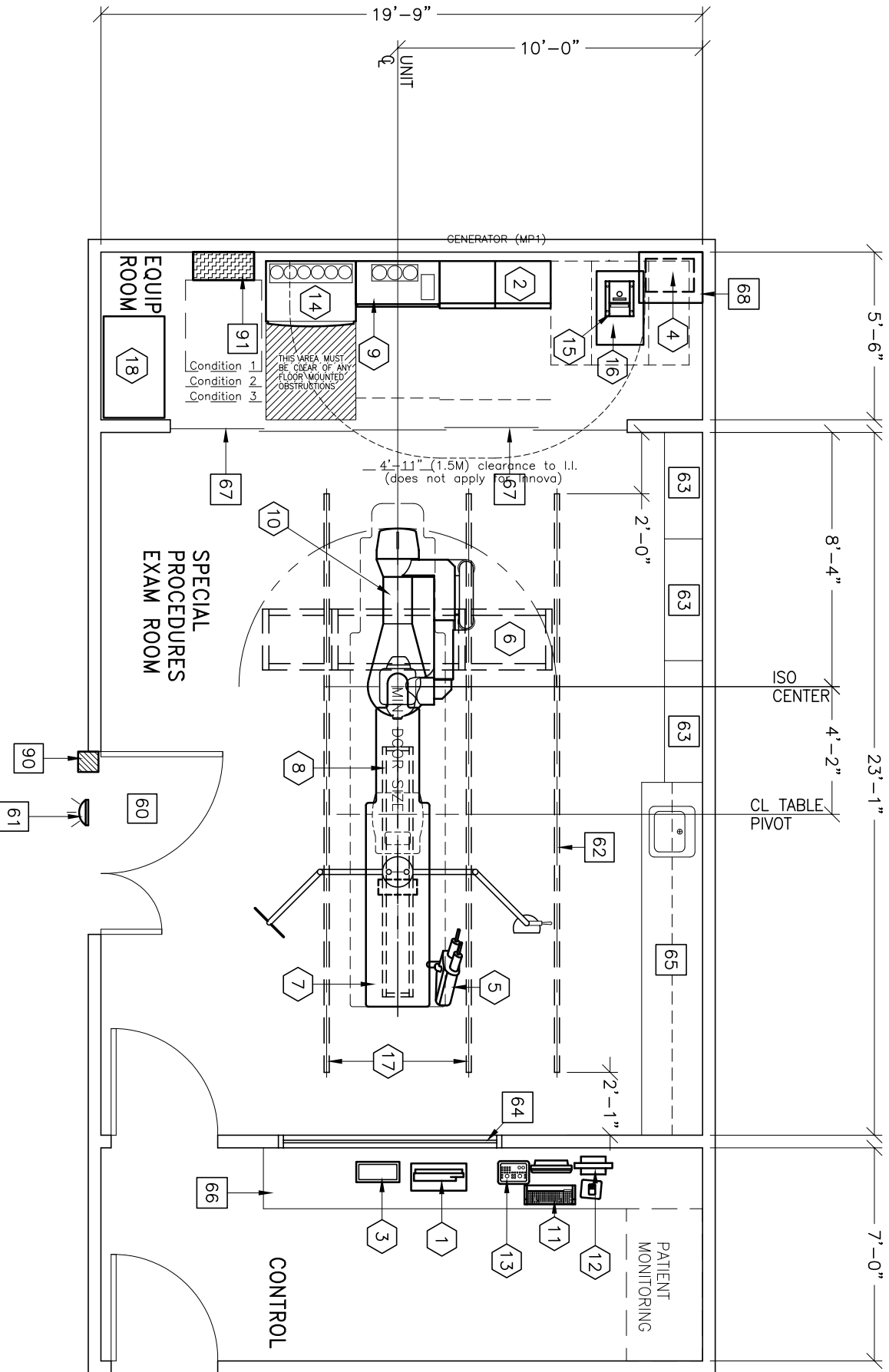

MINIMUM:

26'-0" x 20'-6" [7.92m x 6.24m]

9'-6" [2.89m]

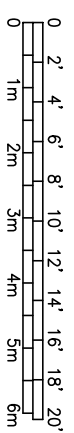

5-80

①	1 OPERATORS CONSOLE	15 lbs	194 btu
②	1 ADVANTX VMP 100 GENERATOR	1358 lbs	5119 btu
③	1 REMOTE CONTROL FOR INJECTOR <OPTION>	4 lbs	
④	1 INJECTOR ELECTRONICS <OPTION>	37 lbs	320 btu
⑤	1 INJECTOR HEAD ON TABLE RAIL <OPTION>	15 lbs	
⑥	1 FOUR LCD MONITOR SUSPENSION ON 9 ft. 6 in. XT INBOARD BRIDGE	485 lbs	1157 btu
⑦	1 OMEGA IV/V TABLE WITH ROTATING TOP	1300 lbs	614 btu
⑧	1 COUNTERBALANCED EYE AND THYROID SHIELD WITH LAMP <OPTION>	167 lbs	150 btu
⑨	1 POSITIONER CABINET	663 lbs	11126 btu
⑩	1 INNOVA 2000 POSITIONER <REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION>	1543 lbs	1167 btu
⑪	1 DLX DIGITAL CONSOLE AND KEYBOARD	17 lbs	870 btu
⑫	1 DLX DIALOG DISPLAY	15 lbs	
⑬	1 DLX KEYPAD	4 lbs	
⑭	1 ATLAS CABINET	947 lbs	9556 btu
⑮	1 DETECTOR CHILLER	33 lbs	706 btu
⑯	1 WATER CHILLER	229 lbs	6143 btu
⑰	2 LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	68 lbs	
⑱	1 JEDI GENERATOR CART	50 lbs	

60°-75° (F)
 15° (F) per hour
 30%-70% non condensing
 10% per hour
 8000 ft. above sea level

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

- 60 MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS
44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT
ON A 96 IN. [2438mm] CORRIDOR WIDTH
- 61 X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY
CALL: 800-200-9760
GE CAT. NO. WXIABWW-OF-XIU
- 62 CABLE DRAPE RAIL, CAT. NO. CPGE55 OR EQUIVALENT.
CONTACT UNISTRUT WISCONSIN: 262-796-8710
- 63 CATHETER CABINETS
- 64 CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING
WINDOW.
- 65 COUNTER TOP WITH SINK, BASE AND
WALL CABINETS
- 66 COUNTER TOP FOR EQUIPMENT-
SHELVING MAY BE REQUIRED
PROVIDE GROMMETED OPENINGS AS
REQUIRED TO ROUTE INTERCONNECT
CABLES TO RACEWAY BELOW COUNTERTOP.
- 67 SLIDING EQUIPMENT ROOM DOORS
- 68 SHELF FOR EQUIPMENT
- 90 X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL
REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR
DETAILED DESCRIPTION -CAT. NO. E4500SS FOR WARNING
LIGHT & ROOM LIGHT CONTROL.
- 91 INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A'
ON SHEET E1 FOR DETAILED DESCRIPTION. CAT NO. E4502K
<24 W x 36 H x 11.2 in. DEEP>