



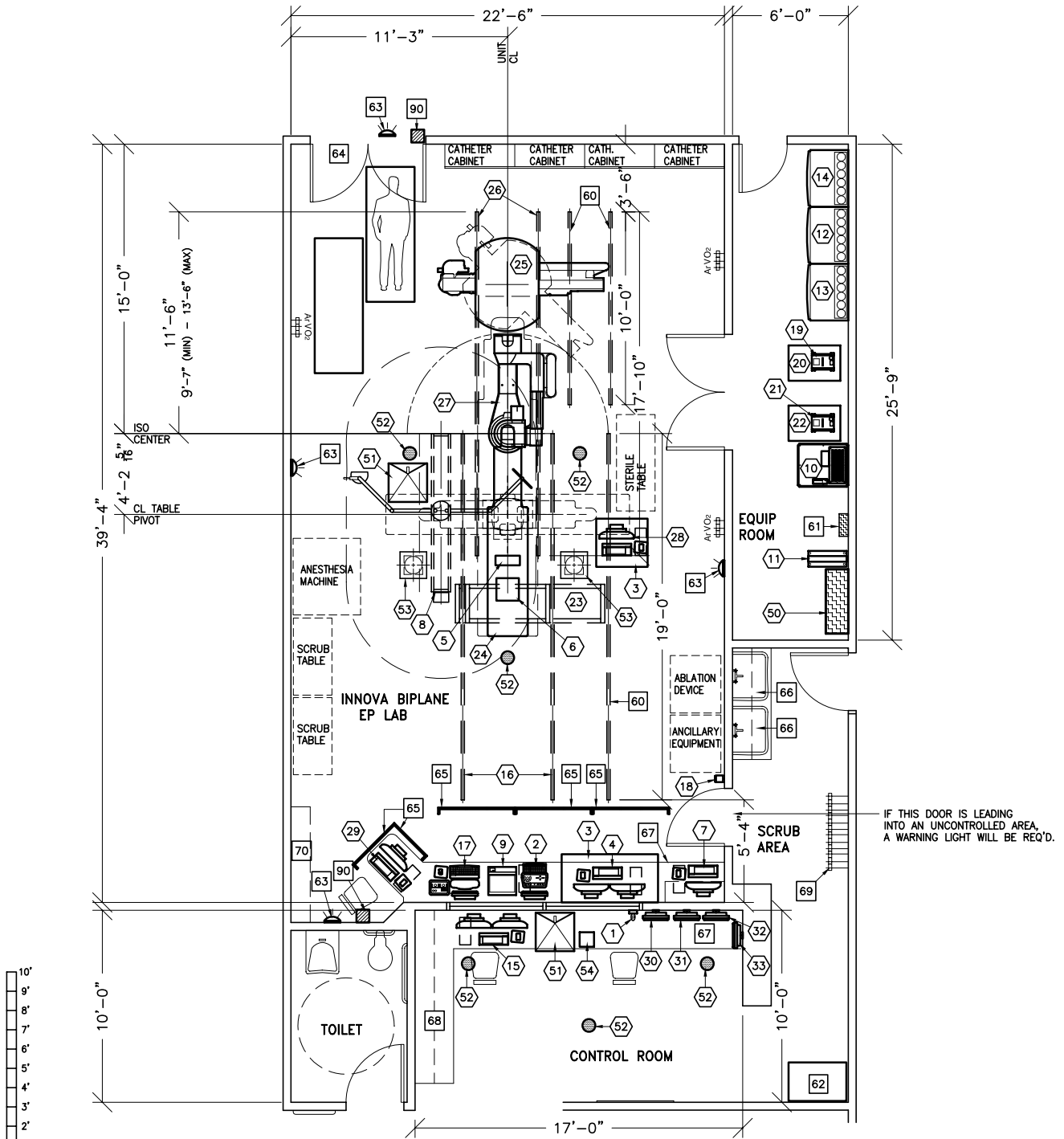

INNOVA 2121/3131 BIPLANE ELECTROPHYSIOLOGY (EP) LAB

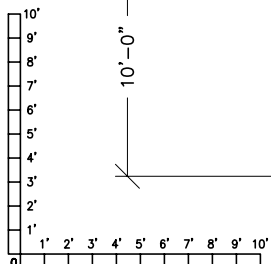

RECOMMENDED ROOM DIMENSIONS
32'-0" x 19'-7"



ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
ABSOLUTE MINIMUM:	22'-6" x 13'-9" [6.89m x 4.23m]	9'-4" ±0.2 [2.84m]

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 5-88

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	BOLUS CHASE HANDSWITCH	2 lbs	
②	1	IVUS VOLCANO S5; CONSOLE, INCLUDES FLAT PANEL MONITOR AND KEYBOARD (DESK MOUNTED)	68 lbs	1631 btu
③	1	WORKSTATION CART		
④	1	COMBO LAB CONSOLE, INCLUDES MONITORS AND KEYBOARD	203 lbs	2935 btu
⑤	1	TRAM NET RACK	8 lbs	
⑥	1	CLAB 2 PLUS AMPLIFIER	24 lbs	204 btu
⑦	1	MAPPING WORKSTATION	46 lbs	682 btu
⑧	1	COUNTERBALANCED EYE AND THYROID SHIELD WITH LAMP	143 lbs	
⑨	1	IVUS VOLCANO COLOR PRINTER		
⑩	1	UPS CABINET	1170 lbs	4050 btu
⑪	1	3 KVA UPS CABINET	81 lbs	546 btu
⑫	1	AP FRONTAL CABINET (C1)	890 lbs	4413 btu
⑬	1	LC/LP CABINET (C2)	630 lbs	4570 btu
⑭	1	LATERAL CABINET (C3)	705 lbs	2945 btu
⑮	1	REMOTE WORKSTATION, INCLUDES MONITORS AND KEYBOARD	181 lbs	2935 btu
⑯	2	LONGITUDINAL STATIONARY RAIL FOR LCD MONITOR SUSPENSION	68 lbs	
⑰	1	OPERATORS CONSOLE	22 lbs	546 btu
⑱	1	XR BUZZER (LOCATED ABOVE CEILING)	2 lbs	
⑲	1	LATERAL DETECTOR CHILLER	33 lbs	709 btu
⑳	1	LATERAL WATER CHILLER	447 lbs	16320 btu
㉑	1	AP DETECTOR CHILLER	33 lbs	709 btu
㉒	1	AP TUBE CHILLER	447 lbs	18723 btu
㉓	1	EIGHT LCD MONITOR SUSPENSION ON 7 ft. 9 in. XT INBOARD BRIDGE	630 lbs	1638 btu
㉔	1	OMEGA IV/V TABLE WITH ROTATING TOP	1750 lbs	614 btu
㉕	1	LATERAL POSITIONER BRIDGE MOUNT ASSEMBLY MOUNTED FROM CEILING SUPPORTS	1679 lbs	4126 btu
㉖	2	LONGITUDINAL STATIONARY RAIL FOR LATERAL GANTRY INNOVA POSITIONER	68 lbs	
㉗	1	INNOVA POSITIONER (REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION)	1653 lbs	2416 btu
㉘	1	NURSE WORKSTATION	46 lbs	682 btu

ENVIRONMENT

Ambient Operation Temperature: 55-75° (F)
 Allowable Temperature Change: 15° (F) per hour
 Humidity: 30%-70% non condensing
 Allowable Humidity Change: 10% per hour
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE 5-88

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
29	1	CLINICAL WORKSTATION	46 lbs	682 btu
30	1	18 in. MONITOR ON WALL SUPPORT <LATERAL - REFERENCE>	26 lbs	204 btu
31	1	18 in. MONITOR ON WALL SUPPORT <LATERAL - LIVE>	26 lbs	204 btu
32	1	18 in. MONITOR ON WALL SUPPORT <AP - REFERENCE>	26 lbs	204 btu
33	1	18 in. MONITOR ON WALL SUPPORT <AP - REFERENCE>	26 lbs	204 btu
50	1	INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.	899 lbs	2215 btu
51	2	VITALINQ MICROPHONE		
52	6	VITALINQ SPEAKER		
53	2	SKYTRON LIGHTING UNIT	50 lbs	341 btu
54	1	VITALINQ CONSOLE		
<p>***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****</p>				
60		CABLE DRAPE RAIL.		
61		150-AMP DISCONNECT BREAKER <FOR LOCK-OUT/TAG-OUT CAPABILITY>		
62		CUSTOMER SUPPLIED STORAGE CABINET		
63		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU		
64		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
65		MOBILE RADIATION SHIELD		
66		SCRUB SINK		
67		COUNTER TOP FOR EQUIPMENT- SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
68		COUNTERTOP W/BASE AND WALL CABINETS		
69		LEAD APRON RACK		
70		UPPER CASEWORK		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4500AK FOR WARNING LIGHT CONTROL ONLY.		