



## trophon Printer

The trophon Printer records each HLD cycle and generates an adhesive label detailing the results of each disinfection cycle for documentation and traceability to link the probe and the HLD cycle date and time with the patient and the procedure.

Up to 4 labels can be selected and printed after each cycle is completed. The results of each cycle (pass or fail) are presented in a clear, illustrative format on the label, minimizing the risk of misinterpretation. The user can also add further information such as probe serial number and operator details. Each high quality, direct thermal adhesive label roll can generate up to 900 labels. The Printer label has seven years of print stability.

Benefits of trophon Printer:

- Simplifies record maintenance
- Minimizes the risk of error due to automated process



**trophon Printer** generates an adhesive label detailing the results of each disinfection cycle for documentation and traceability – helping to eliminate missing information.

1 trophon

2 15/05/2014 15:24

3 SN: 18333-024

4 Disinfection: PASS

5 Cycle#: 15000

6 Indicator: PASS  FAIL

Operator: \_\_\_\_\_

Probe: \_\_\_\_\_

Notes: \_\_\_\_\_

trophon

15/05/2014 15:24 SN: 18333-024

Disinfection: FAIL Cycle#: 15000

7 Error: XXXX

8 Operator: \_\_\_\_\_

9 Probe: \_\_\_\_\_

10 Notes: \_\_\_\_\_

### trophon sample labels.

- |                               |  |
|-------------------------------|--|
| 1. trophon logo               | 7. Error code                                |
| 2. Date and time              | 8. Operator name ID (manual entry field)     |
| 3. trophon EPR serial number  | 9. Probe information ID (manual entry field) |
| 4. Disinfection result (PASS) | 10. Notes (manual entry field)               |
| 5. trophon cycle number       |  |
| 6. Chemical indicator result  |  |

<sup>1</sup> Electronically Accessed: The Joint Commission Quick Safety Alert: Most Challenging Requirements in 2013; Originally accessed September 7, 2014.

### Learn more.

To learn more about the trophon EPR traceability solution, contact your local GE representative or visit us at [gehealthcare.com/trophon](http://gehealthcare.com/trophon).

© 2014 General Electric Company – All rights reserved.

General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your GE representative for the most current information.

GE and GE monogram are trademarks of General Electric Company or one of its subsidiaries.

trophon is a registered trademark of Nanosonics, Ltd. Windows is a trademark of the Microsoft Corporation. Third party trademarks of their respective owners.

General Electric Company, doing business as GE Healthcare.

GE Healthcare  
9900 Innovation Drive  
Wauwatosa, WI 53226  
U.S.A.  
[www.gehealthcare.com](http://www.gehealthcare.com)

