


| ROOM DIMENSIONS | LENGTH & WIDTH | CEILING HEIGHT |
|-----------------|---------------------------------|----------------|
| TYPICAL: | 20'-8" x 13'-2" [6.30M x 4.00M] | 9'-0" [2.74M] |

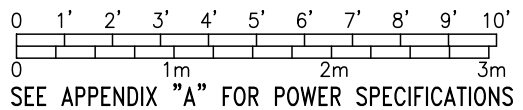

ROOM PLAN LOCATED ON PAGE 6-92

| ITEM | QUANTITY | ITEM DESCRIPTION | WEIGHT | HEAT OUTPUT |
|---|----------|---|--------|-------------|
| ① | 1 | CONSOLE CABINET | 80 kg | 840 W |
| ② | 1 | OPTIMA TABLE | 57 kg | |
| ③ | 1 | OPERATOR'S CHAIR | | |
| ④ | 1 | POWER DISTRIBUTION UNIT | 300 kg | 700 W |
| ⑤ | 1 | BRIVO 385 GANTRY | 890 kg | 4430 W |
| ⑥ | 1 | PATIENT TABLE WITH EXTENDED TABLE TOP. WITH 396 LB (180kg) PATIENT | 520 kg | 120 W |
| ⑦ | 1 | OPTIONAL FREEDOM WORKSPACE SMALL TABLE | 44 kg | |
| ***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS ***** | | | | |
| 60 | | MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 33.5 IN. W x 74.8 IN. H [850mm x 1900mm], CONTINGENT ON A 90 IN. [2300mm] CORRIDOR WIDTH | | |
| 61 | | X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU | | |
| 62 | | DOOR LIMIT SWITCH (REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES) | | |
| 63 | | COUNTER TOP FOR EQUIPMENT PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP. | | |
| 64 | | LEAD GLASS WINDOW | | |
| 90 | | MAIN DISCONNECT CONTROL PANEL GEMS CAT. NO. E4502AB | | |
| 91 | | X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY. | | |

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F) 18-26° C
 Allowable Temperature Change: 5° F [3° C] per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 7875 [2400M] ft. above sea level