

ROOM PLAN LOCATED ON PAGE 7-26

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	XELERIS WORKSTATION	55 lbs	255 btu
②	1	COLOR PRINTER		
③	1	UPS SYSTEM	33 lbs	4436 btu
④	1	MG/MC IMAGING SYSTEM	3300 lbs	4095 btu
⑤	1	PATIENT TABLE	363 lbs	
⑥	1	LIMIT OF TABLE TRAVEL		
⑦	2	COLLIMATOR STORAGE CART	551 lbs	
⑧	1	R-WAVE TRIGGER UNIT	19 lbs	170 btu
⑨	1	ACQUISITION MOBILE CART INCLUSIVE OF MONITOR AND KEYBOARD	105 lbs	34 btu
⑩	1	ACQUISITION COMPUTER	33 lbs	600 btu
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		TABLE		
61		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 37.5 IN. W x 80 IN. H [953mm x 2083mm], CONTINGENT ON A 74 IN. [1880mm] CORRIDOR WIDTH		
62		OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED. FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE		
63		MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.		
90		OPERATORS CHAIR		

ENVIRONMENT

Ambient Operation Temperature: 55 to 75 [15 to 28° C]
 Allowable Temperature Change: 5° F [3° C] per hour
 Humidity: 20-80 percent non condensing
 Allowable Humidity Change: 10 percent per hour
 Max - Altitude: 8000 FT. [2438 M]