




PRECISION 500D with SATURN V4 WIRELESS DR IMAGING OPTION

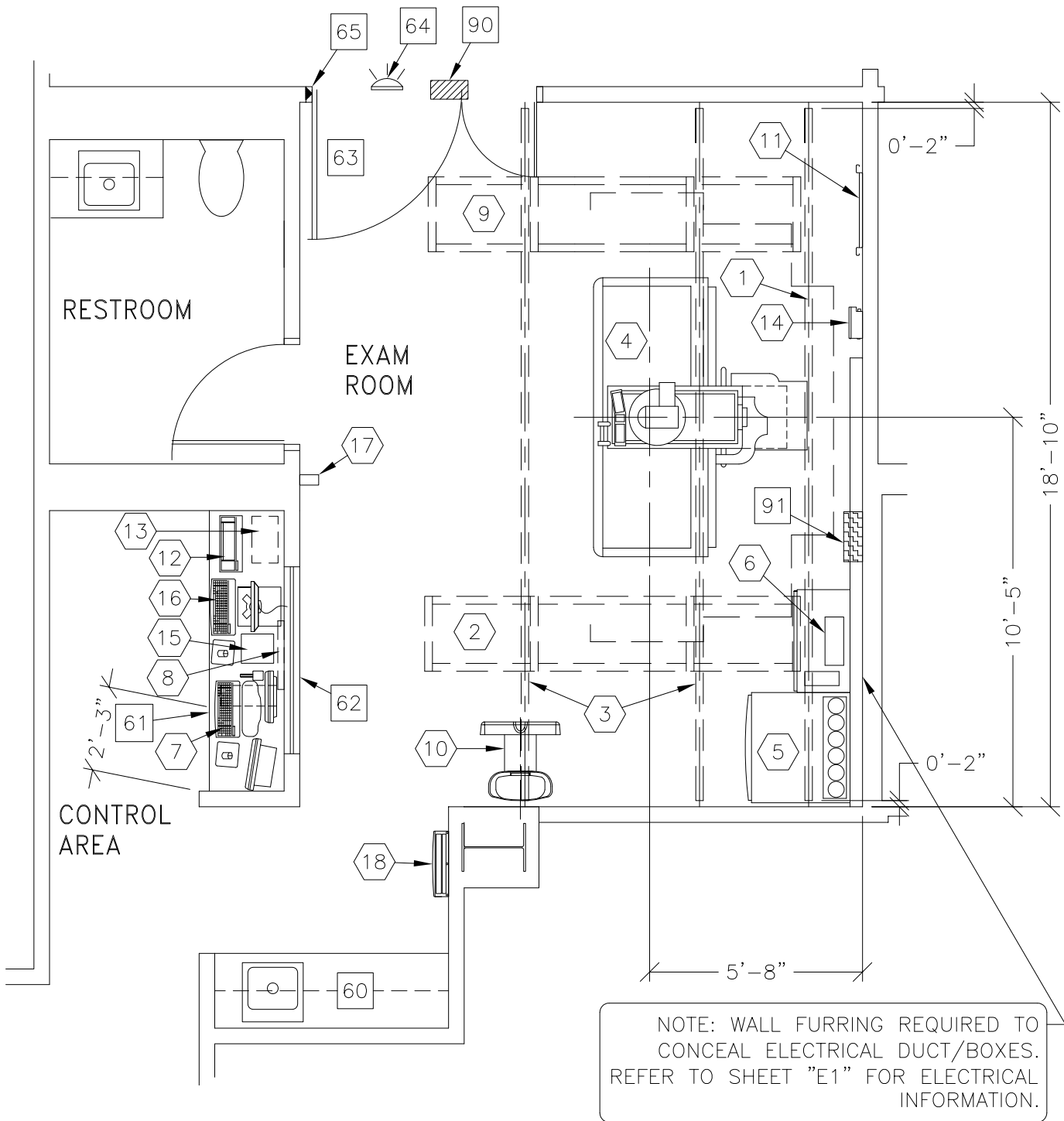

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	16'-6" x 12'-6" [5.03m x 3.81m]	9'-0" [2.74"]

ROOM PLAN LOCATED ON PAGE 2-60

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	CABLE DRAPE RAIL.	108 lbs	
②	1	XT RADIOGRAPHIC SUSPENSION WITH INBOARD MOUNTING.	623 lbs	501 btu
③	2	LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	110 lbs	
④	1	PRECISION R&F TABLE WITH INTELLIGENT DIGITAL DEVICE	3443 lbs	426 btu
⑤	1	SYSTEMS CABINET	881 lbs	2457 btu
⑥	1	POSITIONER CABINET	685 lbs	3412 btu
⑦	1	SATURN OPERATORS CONSOLE	85 lbs	781 btu
⑧	1	IUI ACCESSORY ASSEMBLY	15 lbs	
OPTIONS:				
⑨	1	ONE LCD MONITOR SUSPENSION ON XT INBOARD BRIDGE	282 lbs	68 btu
⑩	1	SG-80 CHEST UNIT WITH KNEE EXTENSION	427 lbs	
⑪	1	TABLE ACCESSORY RACK	90 lbs	
⑫	1	DVD RECORDER KEYBOARD	2 lbs	
⑬	1	DVD RECORDER	13 lbs	
WIRELESS DR IMAGING (OPTION PKG):				
⑭	1	TETHER INTERFACE BOX	15 lbs	10 btu
⑮	1	DR IMAGING CABINET	37 lbs	477 btu
⑯	1	DR IMAGING CONSOLE	28 lbs	525 btu
⑰	1	DONGLE	4 lbs	
⑱	1	DETECTOR BIN	33 lbs	

ENVIRONMENT

Ambient Operation Temperature: 55-75° (F)
 Allowable Temperature Change: 15° (F) per hour
 Humidity: 20-80 percent non condensing
 Allowable Humidity Change: 10 percent per hour
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE

ITEM ◻	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
60		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
61		COUNTER TOP FOR EQUIPMENT- MINIMUM DEPTH 24 IN. AND ADDITIONAL SHELVING MAY BE REQUIRED BELOW COUNTER TOP FOR PC TOWER. PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE CABLES.		
62		CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING WINDOW.		
63		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
64		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
65		DOOR LIMIT SWITCH <NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
90		X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -CAT. NO. E4502SS FOR WARNING LIGHT & ROOM LIGHT CONTROL.		
91		MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1. <16" W x 24" H x 6" D>		