



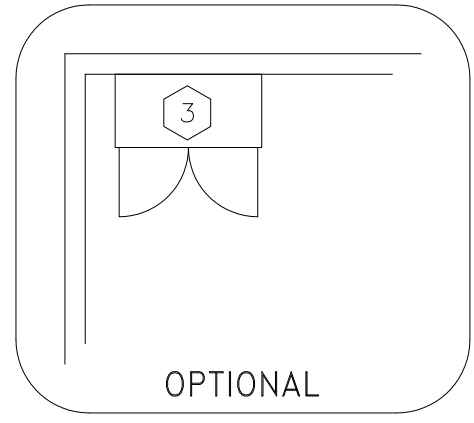


## Discovery CT750 HD with 1700 Table

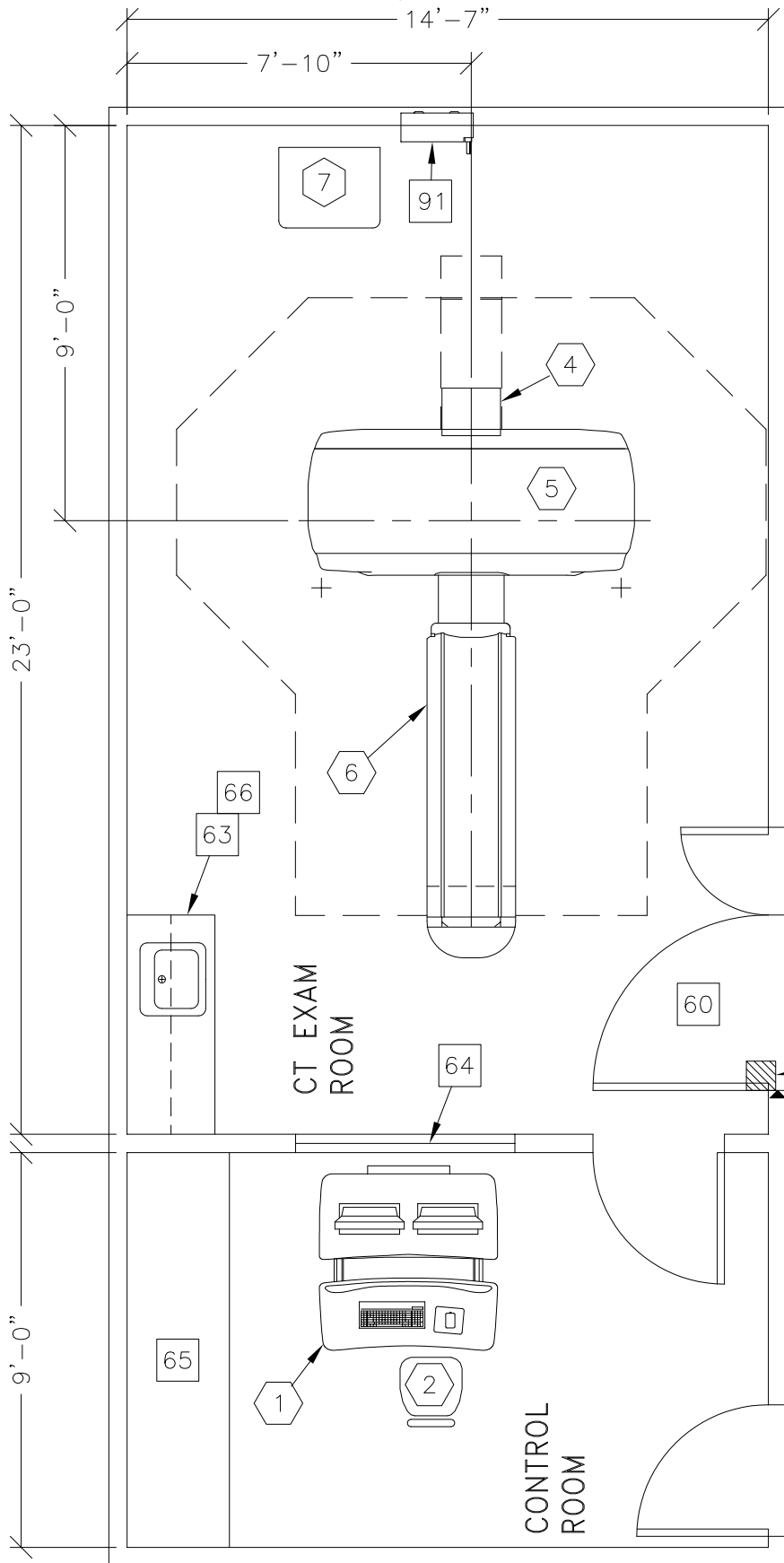

NOTE: THIS IS A SUGGESTED LAYOUT BASED OFF OF GE HEALTHCARE PRE-INSTALL MANUAL. PLEASE CONTACT LOCAL PROJECT MANAGER OF INSTALLATION IF YOUR ROOM LAYOUT OR DIMENSIONS ARE DIFFERENT THAN THE LAYOUT SHOWN.

RECOMMENDED  
SCAN ROOM  
CEILING HEIGHT  
9'-0" [2743mm]

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL:	20'-0" x 13'-6" [6.1M x 4.1M]	8'-0" [2.43M]

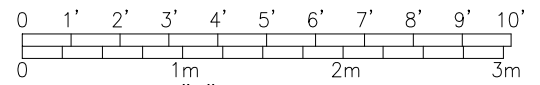

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 6-68

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	OPERATOR'S CONSOLE / COMPUTER	568 lbs	6000 btu
②	1	OPERATOR'S CHAIR		
③	1	STORAGE CABINET (EMPTY CABINET WEIGHT)	99 lbs	
④	1	REAR CABLE COVER		
⑤	1	DISCOVERY CT750 HD GANTRY (BTU'S INCLUDE GANTRY, TABLE & PDU)	4100 lbs	35007 btu
⑥	1	GT1700 PATIENT TABLE WITH EXTENDED TABLE TOP.	881 lbs	
⑦	1	POWER DISTRIBUTION UNIT	798 lbs	
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
61		DOOR LIMIT SWITCH (REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)		
62		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU		
63		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
64		COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
65		LEAD GLASS WINDOW		
66		CASEWORK REQUIRED WITH A MINIMUM OF 16CF TO STORE ALL SERVICE MATERIALS		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1 FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		
91		MAIN DISCONNECT CONTROL		

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)  
 Allowable Temperature Change: 5° F [3° C] per hour  
 Humidity: 30-60 percent non condensing  
 Allowable Humidity Change: 5 percent per hour  
 Max - Altitude: 7875 [2400M] ft. above sea level