



MILLENNIUM VG WITH HAWKEYE OPTION

RECOMMENDED CEILING HEIGHT
8'-0" [2438mm]

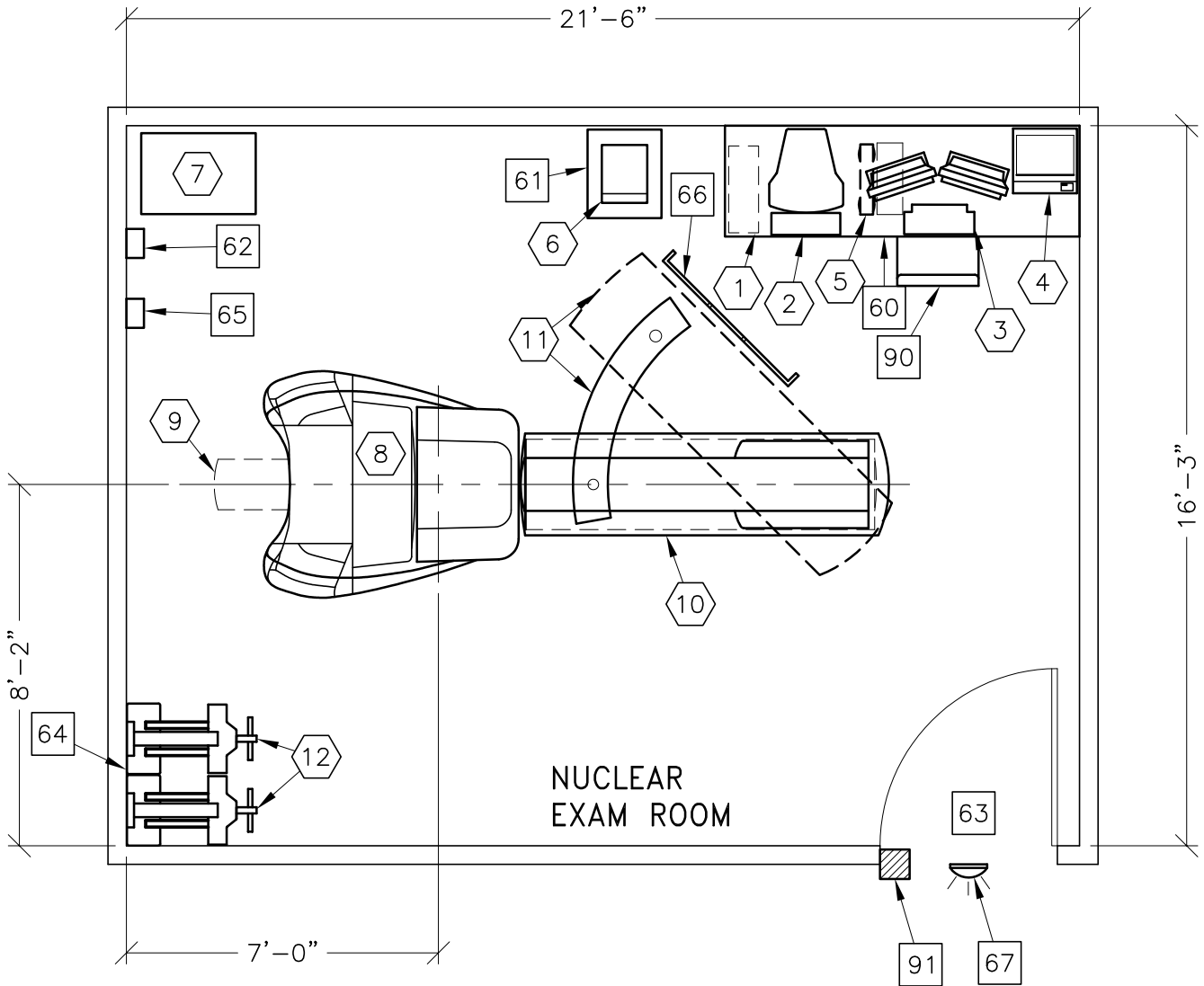

MINIMUM: 21'-6" x 13'-6" [6.55m x 4.11m] 7'-6" [2.29m]

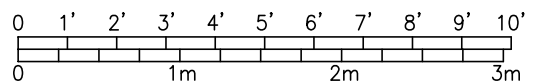

7-40

①	1 ACQUISITION COMPUTER	68 lbs	1194 btu
②	1 ACQUISITION MONITOR/KEYBOARD	81 lbs	409 btu
③	1 XELERIS WORKSTATION	55 lbs	
④	1 COLOR PRINTER		
⑤	1 UPS SYSTEM	33 lbs	4436 btu
⑥	1 R-WAVE TRIGGER UNIT	19 lbs	170 btu
⑦	1 UPS SYSTEM	965 lbs	5099 btu
⑧	1 VG IMAGING SYSTEM GANTRY	6459 lbs	6484 btu
⑨	1 LIMIT OF TABLE TRAVEL		
⑩	1 IMAGING TABLE	881 lbs	
⑪	1 TABLE SWING PATH AND PLATE FOR COLLIMATOR EXCHANGE		
⑫	2 COLLIMATOR STORAGE CART	1058 lbs	

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

60	TABLE
61	CART
62	DISCONNECT
63	MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 48 IN. W x 80 IN. H [1219mm x 2032mm], CONTINGENT ON A 78 IN. [1981mm] CORRIDOR WIDTH
64	OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.
65	MAIN DISCONNECT CONTROL, GE CAT. NO. E4502SN
66	MOBILE RADIATION SHIELD
67	X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760, GE CAT. NO. WXIABWW-OF-XIU
90	OPERATORS CHAIR
91	X-RAY ROOM WARNING LIGHT CONTROL PANEL. REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION-E4502RL FOR WARNING LIGHT CONTROL ONLY.

59-81° F [15-27° C]
 5° F [3° C] per hour
 20-80 percent non condensing
 5 percent per hour
 N/A