





LightSpeed RT Xtra with 1700 Table

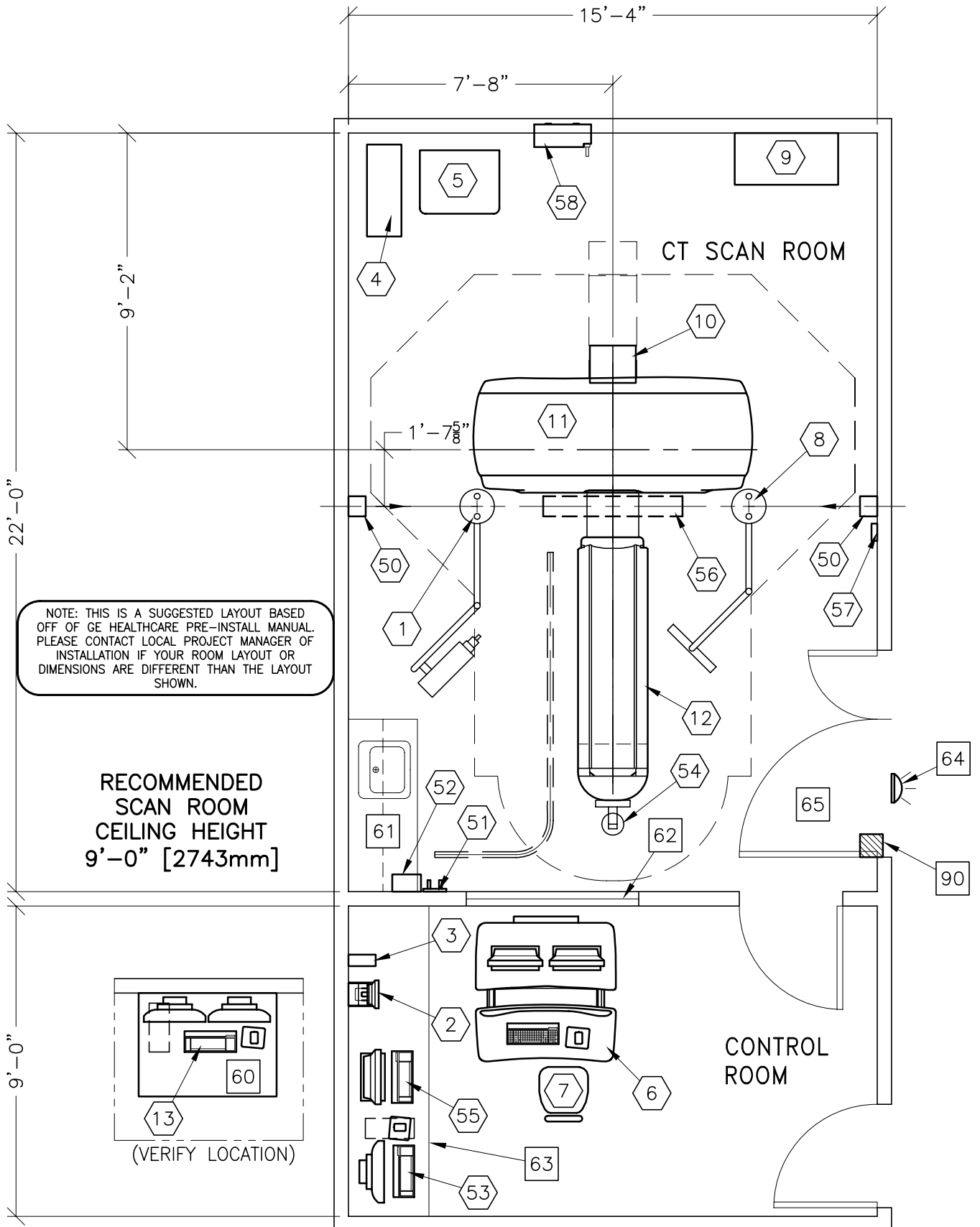

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL	20-0" x 14'-0" [6.0M x 4.2M]	8'-0" [2.43M]

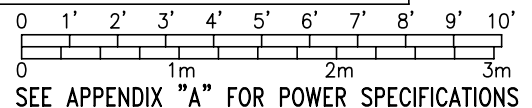

ROOM PLAN LOCATED ON PAGE 6-64

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	INJECTOR HEAD ON OVERHEAD COUNTERPOISED SUSPENSION.	22 lbs	
②	1	INJECTOR CONTROL CONSOLE	8 lbs	
③	1	MAIN UNIT	13 lbs	
④	1	UPS SYSTEM	350 lbs	
⑤	1	POWER DISTRIBUTION UNIT	701 lbs	3402 btu
⑥	1	OPERATOR'S CONSOLE / COMPUTER	493 lbs	8126 btu
⑦	1	OPERATOR'S CHAIR		
⑧	1	LCD MONITOR	61 lbs	300 btu
⑨	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	90 lbs	
⑩	1	REAR CABLE COVER		
⑪	1	LIGHTSPEED RT16/XTRA	3891 lbs	43010 btu
⑫	1	GT1700 PATIENT TABLE WITH EXTENDED TABLE TOP.	881 lbs	703 btu
⑬	1	ADVANTAGE WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑤①	2	PATIENT POSITIONING LASER LIGHT	66 lbs	
⑤②	1	VARIAN CAMERA HOLDER <WHEN CAMERA IS IN STANDBY POSITION>		
⑤③	1	VARIAN POWER SUPPLY MODULE <GE PROJ MGR; PREINSTALL ITEM, PRE-ORDER FROM VARIAN>	13 lbs	
⑤④	1	VARIAN OPERATORS WORKSTATION	48 lbs	819 btu
⑤⑤	1	VARIAN CAMERA AND BRACKET	8 lbs	341 btu
⑤⑥	1	WORKSTATION COMPUTER		
⑤⑦	1	PATIENT POSITIONING LASER LIGHT	66 lbs	
⑤⑧	1	KEYPAD	2 lbs	
⑤⑨	1	MAIN DISCONNECT CONTROL	132 lbs	

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)
 Allowable Temperature Change: 5° F [3° C] per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 7875 [2400M] ft. above sea level

ROOM PLAN LOCATED ON PAGE

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
		***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****		
60		WORKSTATION TABLE		
61		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
62		LEAD GLASS WINDOW		
63		COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
64		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-0F-XIU		
65		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		