
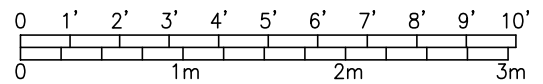


MILLENNIUM MYOSIGHT

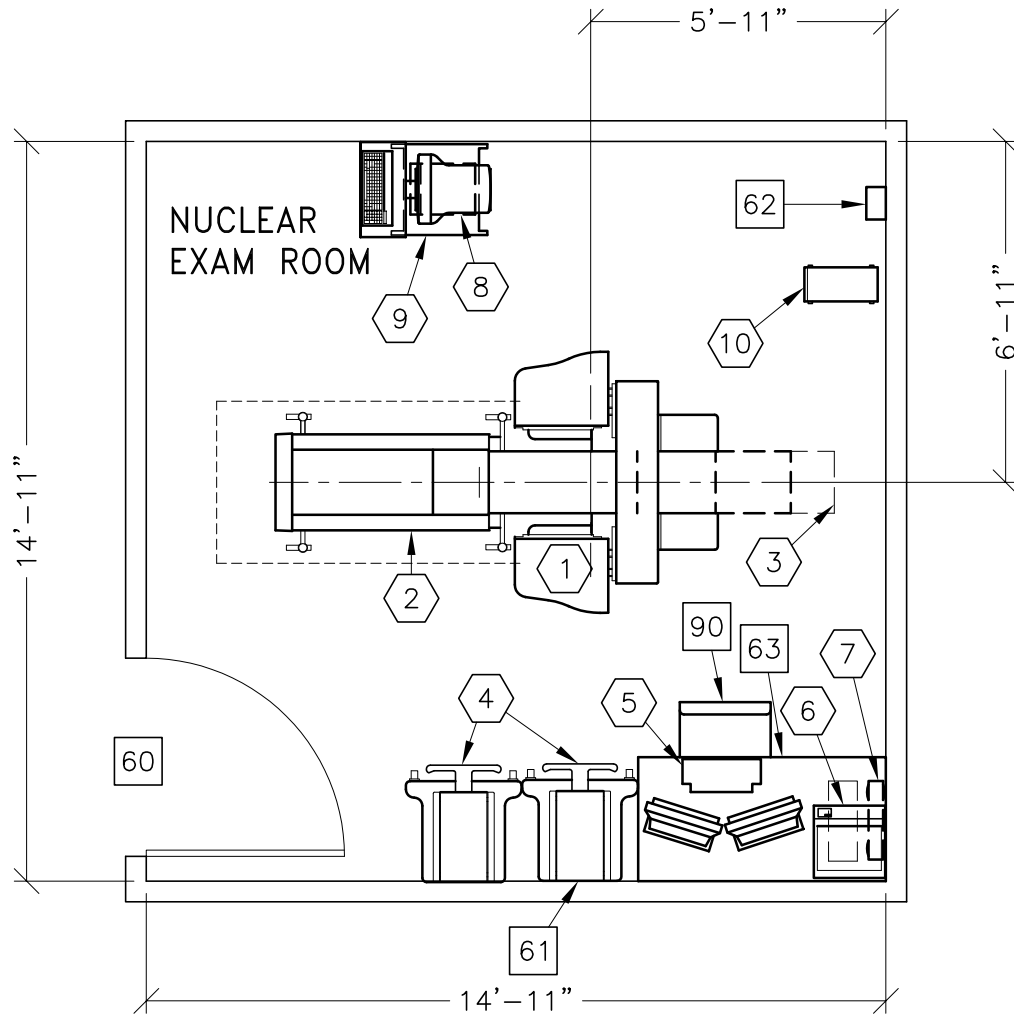

MINIMUM: 14'-6" x 10'-0" [4.42m x 3.05m] 8'-0" [2.44m]

7-54

①	1 MYOSIGHT IMAGING SYSTEM	3300 lbs	4095 btu
②	1 PATIENT TABLE	720 lbs	
③	1 LIMIT OF TABLE TRAVEL		
④	2 COLLIMATOR STORAGE CART	551 lbs	
⑤	1 XELERIS WORKSTATION	55 lbs	
⑥	1 COLOR PRINTER		
⑦	1 UPS SYSTEM	33 lbs	4436 btu
⑧	1 R-WAVE TRIGGER UNIT	19 lbs	170 btu
⑨	1 ACQUISITION MOBILE CART INCLUSIVE OF MONITOR AND KEYBOARD	105 lbs	34 btu
⑩	1 ACQUISITION COMPUTER	33 lbs	600 btu

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

- ⑥0 MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS
37.5 IN. W x 80 IN. H [953mm x 2083mm], CONTINGENT ON
A 74 IN. [1880mm] CORRIDOR WIDTH
- ⑥1 OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO,
FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING
MOVEMENT AND BEING PARKED FOR A LONG PERIOD.
SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.
- ⑥2 MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON
SHEET E1 FOR DETAILED DESCRIPTION.
- ⑥3 TABLE
- ⑨0 OPERATORS CHAIR

59-81° F [15-27° C]
 5° F [3° C] per hour
 20-80 percent non condensing
 5 percent per hour
 N/A