



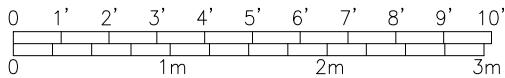

BRIVO XR385/DR-F

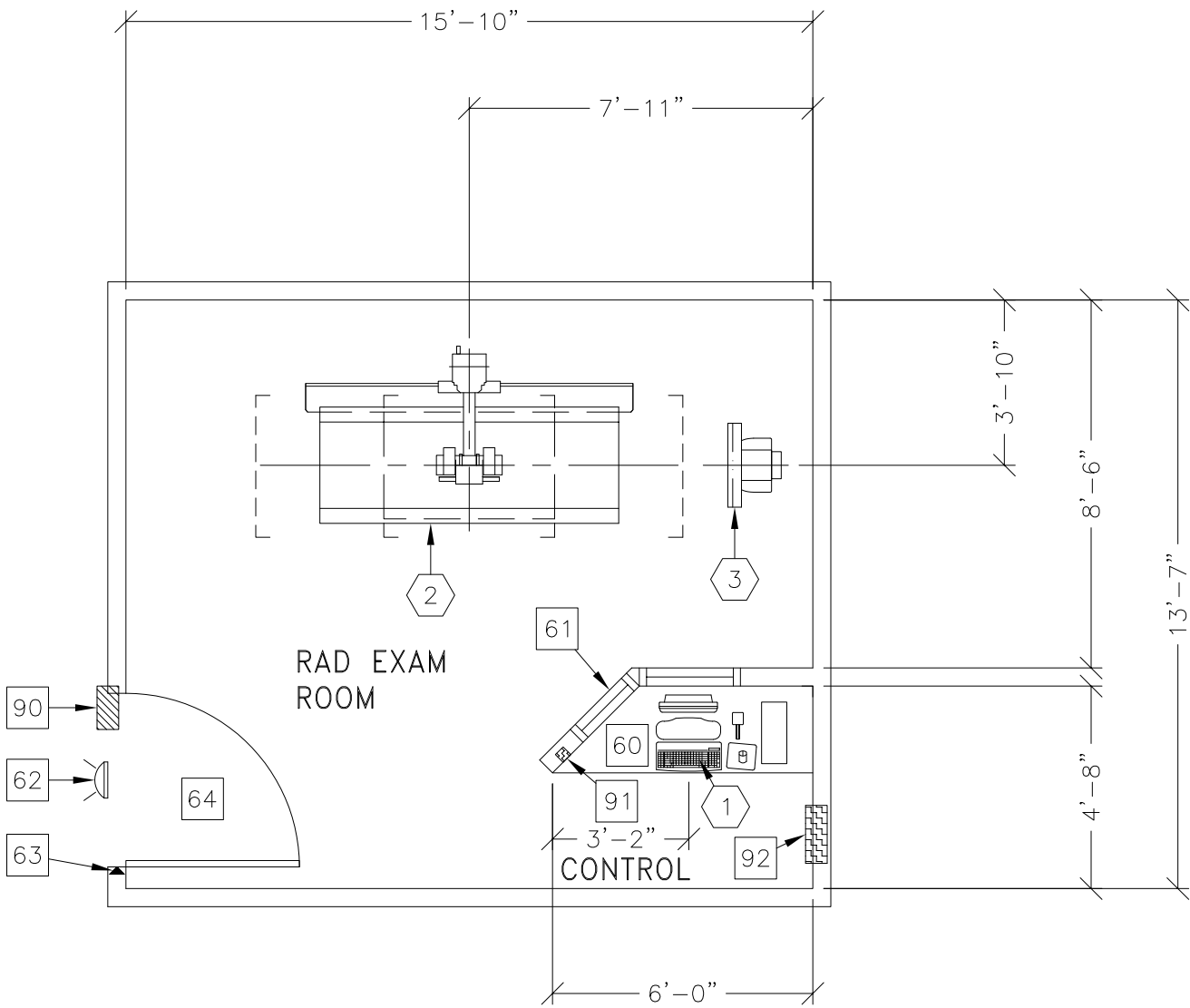

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	VARIABLES: BASED ON EQUIPMENT AND APPLICATIONS	7'-10 1/2" [2.4M]

ROOM PLAN LOCATED ON PAGE 1-146

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	BRIVO XR385 CONTROL CONSOLE	52 lbs	426 btu
②	1	BRIVO XR385 WALLSTAND	407 lbs	
③	1	BRIVO XR385 TABLE	1393 lbs	5119 btu
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		COUNTER TOP FOR EQUIPMENT- MINIMUM DEPTH 24 IN. AND ADDITIONAL SHELVING MAY BE REQUIRED BELOW COUNTER TOP FOR PC TOWER. PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE CABLES.		
61		CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING WINDOW.		
62		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 39 IN. W x 88.5 IN. H [1000mm x 2250mm], CONTINGENT ON A 92 IN. [2500mm] CORRIDOR WIDTH		
63		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
64		DOOR LIMIT SWITCH <NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		
91		EMERGENCY OFF SWITCH <NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
92		MAIN DISCONNECT CAT. NO. E4502ST OR WITH AUTO RESTART E4502RP. <20 W x 48 H x 6.68 in. D>		

ENVIRONMENT

Ambient Operation Temperature: 50-86° (F) [10-30° (C)]
 Allowable Temperature Change: 50° (F) [10° (C)] per hour
 Humidity: 30-80 percent non condensing
 Allowable Humidity Change: <30 percent per hour
 Max - Altitude: no restriction listed in the PIM