



ASSIGNED BY THE HOSPITAL NET ADMIN IF CONNECTING TO THE HOSPITAL LAN	HOSTNAME	IP	AE TITLE	DICOM PORT
ACQUISITION HOST				
PROCESSING HOST				
HARDCOPY HOST				
LAN NET MASK				
GATEWAY TO OTHER NETWORKS				
OTHER				
HUB OR SWITCH				

- PREPARE ADEQUATE NETWORK SOCKETS IN THE PROPER LOCATIONS TO SUPPORT ALL ACQUISITION, LOCAL AND REMOTE WORKSTATION.
- IT DEPARTMENT MUST ASSIGN DEDICATED IP ADDRESSES (NOT DHCP) NOTE THE ADDRESSES BELOW FOR THE ACQUISITION, LOCAL AND REMOTE WORKSTATIONS.
- PREPARE BROADBAND CONNECTIVITY LINE AND DEDICATED IP ADDRESSES FOR INSITE CONNECTIVITY.
- REFER TO TABLE ON A1 PAGE

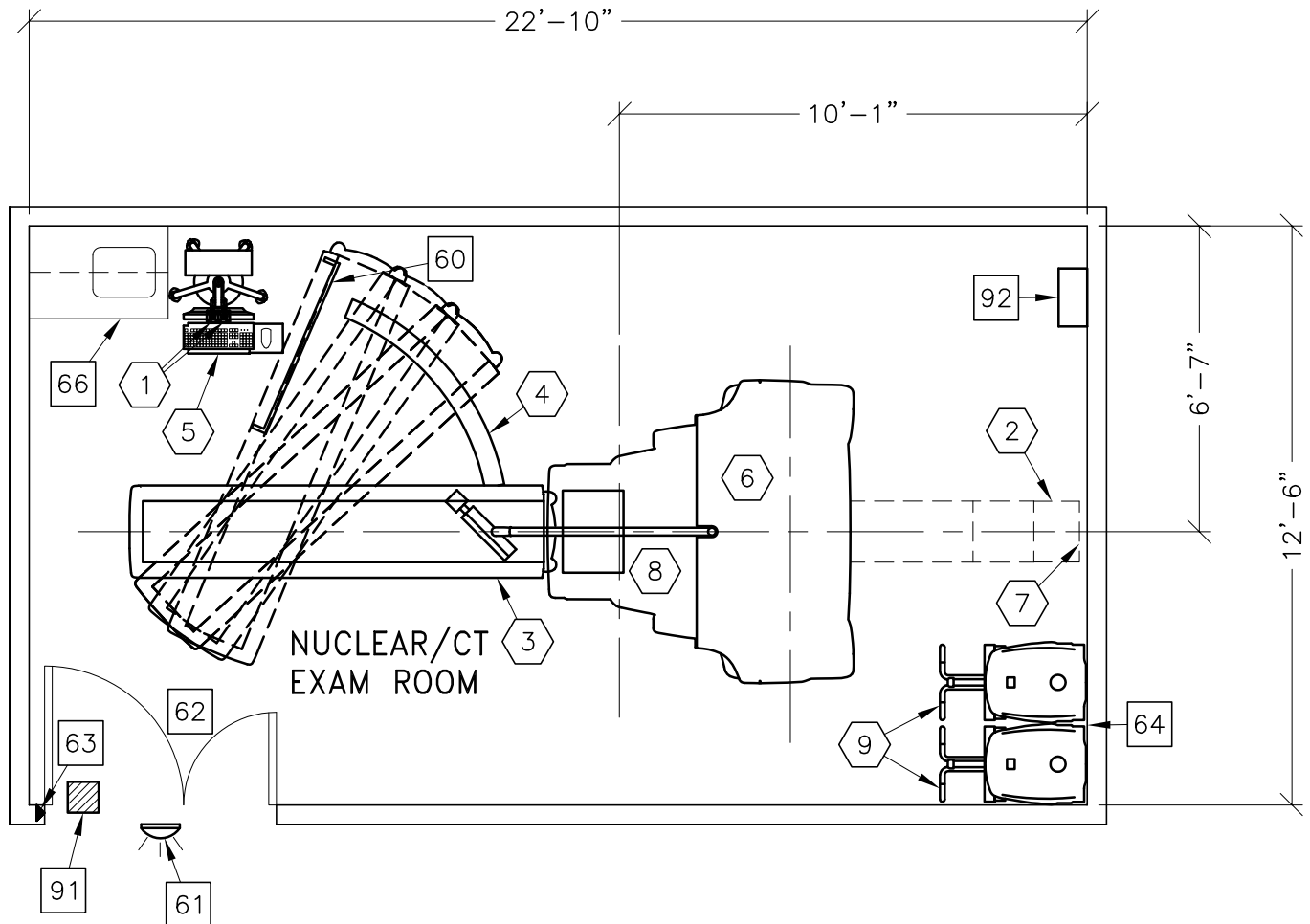

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	Refer to Pre-Installation Manual	

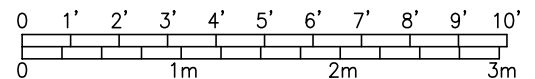

ROOM PLAN LOCATED ON PAGE 7-84

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	2	EMO PUSHBUTTON		
②	1	TABLE EXTENDER		
③	1	OPTIMA NM/CT 640 TABLE	1245 lbs	682 btu
④	1	TABLE SWING PLATE FOR COLLIMATOR EXCHANGE		
⑤	1	OPTIMA NM/CT 640 AQUISITION WORKSTATION ON MOBILE CART	44 lbs	255 btu
⑥	1	OPTIMA NM/CT 640 GANTRY <CT>	1984 lbs	3139 btu
⑦	1	LIMIT OF TABLE TRAVEL		
⑧	1	OPTIMA NM/CT 640 GANTRY <NUCLEAR>	4828 lbs	4501 btu
⑨	2	OPTIMA NM/CT 640 COLLIMATOR CART	727 lbs	
⑩	1	XELERIS WORKSTATION	55 lbs	255 btu
***** <b>CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS</b> *****				
60		MOBILE RADIATION SHIELD		
61		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760, GE CAT. NO. WXIABWW-OF-XIU		
62		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 55.1 IN. W x 86.8 IN. H [1399mm x 2204mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
63		DOOR LIMIT SWITCH <REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
64		OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.		
65		TABLE		
66		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
90		OPERATORS CHAIR		
91		X-RAY ROOM WARNING LIGHT CONTROL PANEL, REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION-E4502RL FOR WARNING LIGHT CONTROL ONLY		
92		MAIN DISCONNECT CONTROL, GE CAT. NO. GEX640NM25A480V, CALL: 800-200-9760		

ENVIRONMENT

Ambient Operation Temperature: 64-79° F [18-26° C]  
 Allowable Temperature Change: 5° F [3° C] per hour  
 Humidity: 30-60 percent non condensing  
 Allowable Humidity Change: 5 percent per hour  
 Max - Altitude: 7875 FT. [2400 M]