



## FEEDER TABLE

### FEEDER TABLE

REV. DATE: 03/25/08

- CALCULATIONS BASED UPON NOMINAL VOLTAGE, WIRE SIZE IN AWG.
- RECOMMENDED FEEDER SIZES FROM DIST. TRANS. TO ROOM DISCONNECT. CALCULATIONS ARE AT NOMINAL VOLTAGE BASED UPON 1/0 WIRE SIZE FROM ROOM DISCONNECT TO POWER CABINET WITH A MAXIMUM RUN OF 25 FT.
- NEUTRAL MUST BE TERMINATED INSIDE THE MAIN DISCONNECT PANEL AND NOT AT ANY GE CABINET.
- THE GROUNDING CONDUCTOR ( ) WILL BE A 2 AWG MINIMUM. THIS GROUND WILL RUN FROM THE EQUIPMENT BACK TO THE POWER SOURCE/MAIN GROUNDING POINT AND ALWAYS TRAVEL IN THE SAME CONDUIT WITH THE FEEDERS AND NEUTRAL.
- \* MINIMUM WIRE SIZE FOR CIRCUIT BREAKER, BASED ON RECOMMENDED OVERCURRENT PROTECTION.
- FOR A FULL SYSTEM UPS, REFER TO ELECTRICAL DETAILS FOR UPS FEEDER WIRES.
- IF THE FEEDER IS BIGGER THAN 3/0, THE HOSPITAL MUST PROVIDE AND INSTALL A REDUCTION BOX.

RUN LENGTH IN FEET	POWER SUPPLY VOLTAGE													
	324-396 360		342-418 380		360-440 400		378-462 420		396-484 440		414-506 460		432-528 480	
	FEEDER	GROUND	FEEDER	GROUND	FEEDER	GROUND	FEEDER	GROUND	FEEDER	GROUND	FEEDER	GROUND	FEEDER	GROUND
50	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)
100	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)	*1/0	(2)
150	3/0	(2)	2/0	(2)	2/0	(2)	1/0	(2)	1/0	(2)	1/0	(2)	*1/0	(2)
200	4/0	(2)	4/0	(2)	3/0	(2)	3/0	(2)	2/0	(2)	2/0	(2)	1/0	(2)
250	300M	(2)	300M	(2)	250M	(2)	4/0	(2)	3/0	(2)	3/0	(2)	3/0	(2)
300	400M	(2)	350M	(2)	300M	(2)	250M	(2)	4/0	(2)	4/0	(2)	4/0	(2)
350	600M	(2)	500M	(2)	400M	(2)	350M	(2)	300M	(2)	250M	(2)	4/0	(2)
400	700M	(2)	600M	(2)	500M	(2)	400M	(2)	350M	(2)	300M	(2)	300M	(2)

# POWER SPECIFICATIONS

## INNOVA SYSTEMS

REV. DATE: 01/04/07

VOLTAGE PRIMARY SOURCE IS REQUIRED FOR ALL INSTALLATIONS.  
 RANGE OF LINE VOLTAGES :  
 NOMINAL LINE VOLTAGE OF 360 TO 480, 3 PHASE, 50 OR 60 Hz

REQUIRED POWER SUPPLY: WYE DISTRIBUTION

MAXIMUM DAILY VOLTAGE VARIATION MUST FALL WITHIN ONE OF THE RANGES IN TABLE A.

TABLE A  
 ALLOWABLE  
 INPUT  
 VOLTAGES/  
 CURRENT  
 DEMAND

NOMINAL VOLTAGE	NORMAL RANGE ±10 PERCENT	CURRENT (AMPS)	
		MAX. MOMENTARY	CONTINUOUS
360	324-396	304	32
380	342-418	289	31
400	360-440	274	29
420	378-462	264	28
440	396-484	249	26
460	414-506	238	25
480	432-528	228	24

ALL CALCULATIONS BASED UPON NOMINAL VOLTAGE

NOTE LOW LINE CONDITIONS MAY INHIBIT SOME HIGH kVp TECHNIQUES. THE GENERATOR AUTOMATICALLY ESTABLISHES THESE INHIBITS BASED ON ACTUAL LINE CONDITIONS AND SYSTEM REGULATION.

PHASE-BALANCE. PHASE-TO-PHASE VOLTAGES MUST BE WITHIN +2 PERCENT OF THE LOWEST PHASE-TO-PHASE VOLTAGE. MAXIMUM ALLOWABLE TRANSIENT VOLTAGE EXCURSIONS ARE 2.5 PERCENT OF RATED LINE VOLTAGE AT A MAXIMUM DURATION OF 5 CYCLES AND FREQUENCY OF 10 TIMES PER HOUR.

POWER DEMAND CONTINUOUS POWER DEMAND = 20KVA. (MAX DEMAND = 171 KVA)

TABLE B  
 MAXIMUM  
 MOMENTARY  
 POWER  
 DEMAND.

DEMAND	ADVANTX 100
kVa * POWER FACTOR AT	171 0.9
mA	1250
kVp	80

\* DEMAND INCLUDES POWER FOR ENTIRE ADVANTX SYSTEM. LINE VOLTAGE REGULATION AT MAXIMUM POWER DEMAND MUST BE LESS THAN OR EQUAL TO 6 PERCENT.

DISTRI-BUTION TRANS-FORMER FOR A SINGLE UNIT INSTALLATION, THE MINIMUM TRANSFORMER SIZE IS 225 KVA.