



MILLENNIUM MPS w/ECT TABLE

RECOMMENDED CEILING HEIGHT
8'-0" [2438mm]

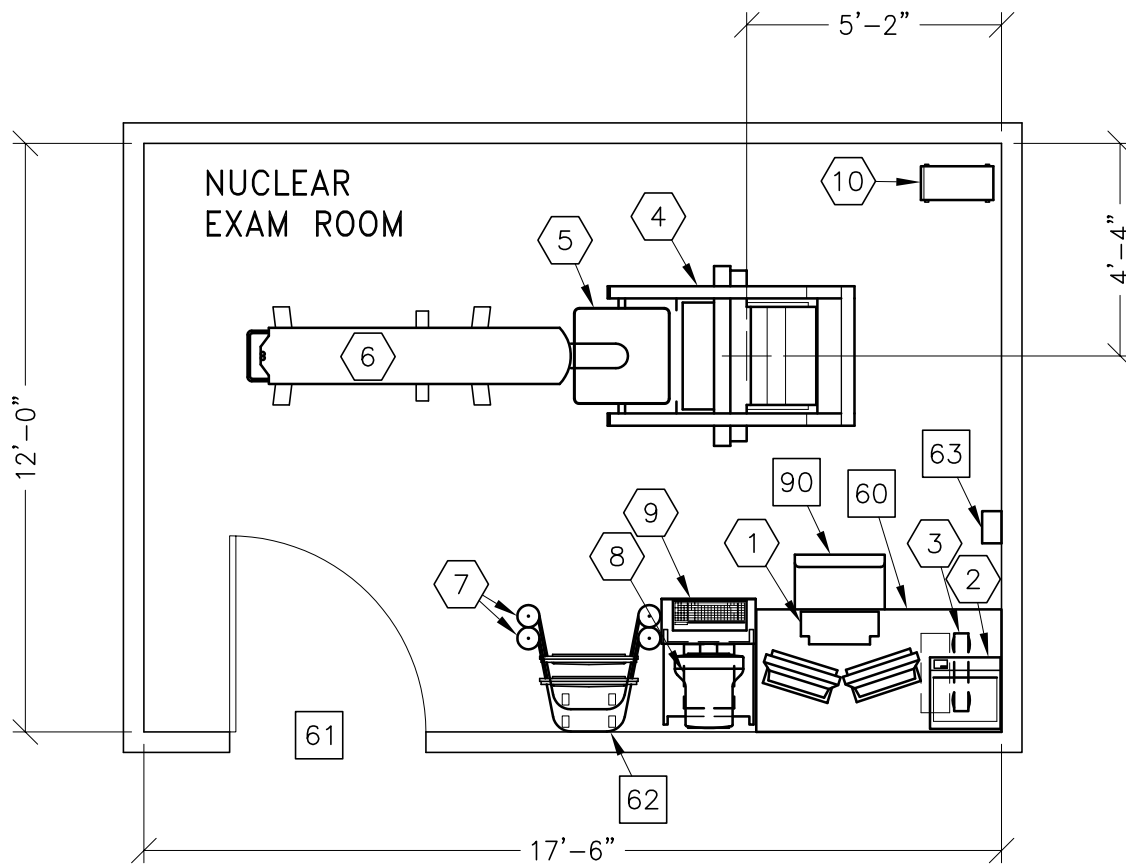

MINIMUM: 17'-4" x 10'-0" [4.87m x 3.05m] 8'-0" [2.44m]

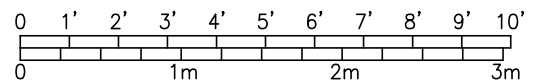

7-52

①	1 XELERIS WORKSTATION	55 lbs	
②	1 COLOR PRINTER		
③	1 UPS SYSTEM	33 lbs	4436 btu
④	1 MILLENNIUM GANTRY WITH IPS	1895 lbs	2047 btu
⑤	1 SQUARE HEAD DETECTOR	423 lbs	194 btu
⑥	1 ECT TABLE	275 lbs	
⑦	2 MILLENNIUM COLLIMATOR STORAGE CART	346 lbs	
⑧	1 R-WAVE TRIGGER UNIT	19 lbs	170 btu
⑨	1 ACQUISITION MOBILE CART INCLUSIVE OF MONITOR AND KEYBOARD	105 lbs	34 btu
⑩	1 ACQUISITION COMPUTER	33 lbs	600 btu

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

60	TABLE
61	MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 45 IN. W x 80 IN. H [1143mm x 2083mm], CONTINGENT ON A 84 IN. [2134mm] CORRIDOR WIDTH
62	OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.
63	MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.
90	OPERATORS CHAIR

59-81° F [15-27° C]
 5° F [3° C] per hour
 20-80 percent non condensing
 5 percent per hour
 N/A