



INFINIA SYSTEM

**RECOMMENDED CEILING HEIGHT
8'-0" [2438mm]**

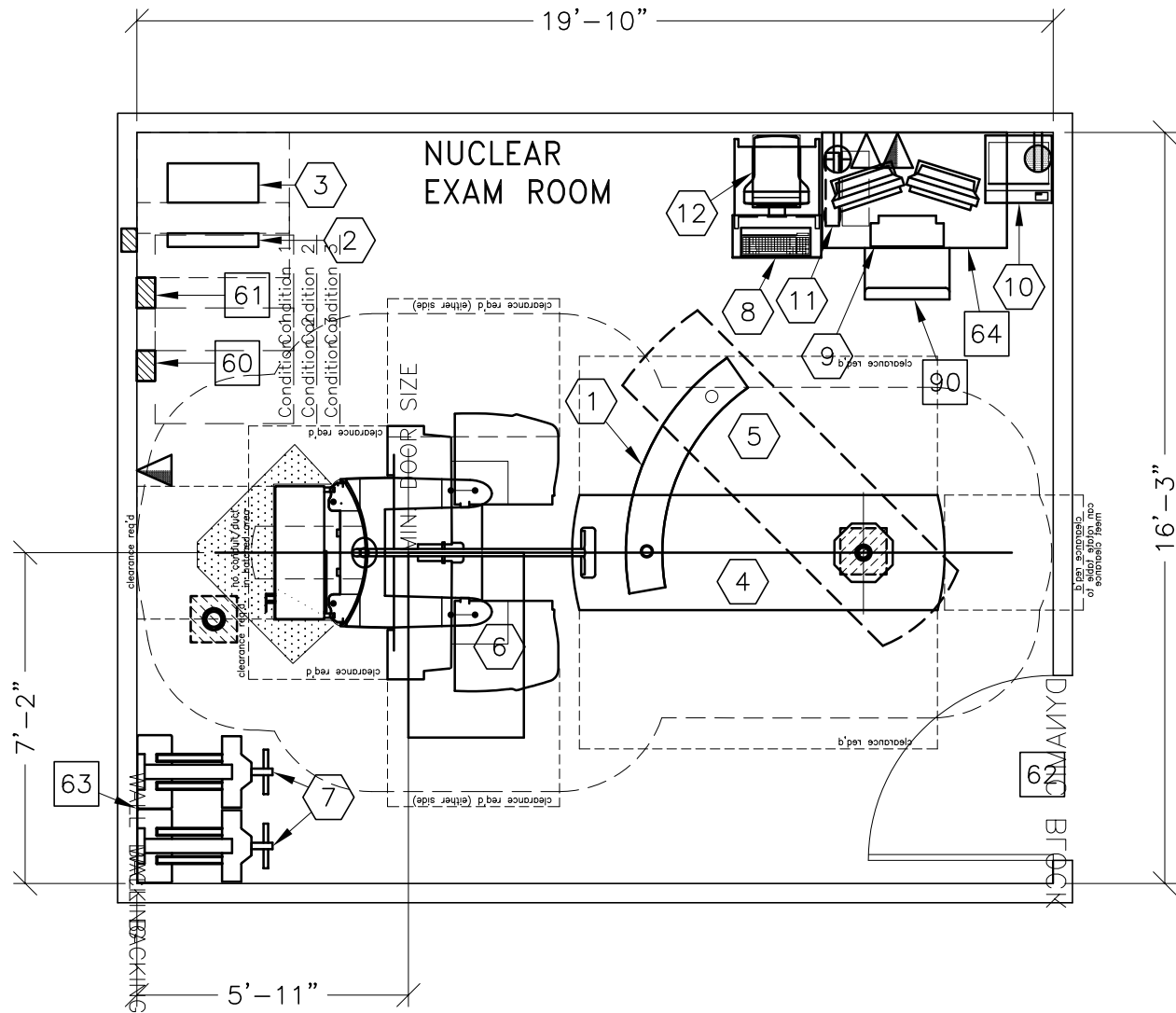

MINIMUM: 15'-0" x 16'-6" [4.57m x 5.03m] (GANTRY AT ANGLE IN ROOM)
7'-6" [2.29m] (HUNG CEILING)

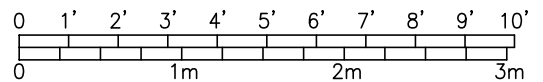

7-56

①	1 TABLE SWING PLATE FOR COLLIMATOR EXCHANGE		
②	1 UPS CONTROL CABINET	39 lbs	1740 btu
③	1 UPS BATTERY CABINET	198 lbs	
④	1 IMAGING TABLE	881 lbs	341 btu
⑤	1 TABLE SWING FOR COLLIMATOR EXCHANGE		
⑥	1 INFINIA I IMAGING SYSTEM GANTRY	6172 lbs	7167 btu
⑦	2 COLLIMATOR STORAGE CART	1058 lbs	
⑧	1 ACQUISITION MOBILE CART INCLUSIVE OF MONITOR AND KEYBOARD	180 lbs	1604 btu
⑨	1 XELERIS WORKSTATION	55 lbs	
⑩	1 COLOR PRINTER		
⑪	1 UPS SYSTEM	33 lbs	4436 btu
⑫	1 R-WAVE TRIGGER UNIT	19 lbs	170 btu

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

- ⑥0 MAIN DISCONNECT CONTROL, GE CAT. NO. E4502SN
- ⑥1 DISCONNECT
- ⑥2 MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 45 IN. W x 80 IN. H [1143mm x 2083mm], CONTINGENT ON A 84 IN. [2134mm] CORRIDOR WIDTH
- ⑥3 OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.
- ⑥4 TABLE
- ⑨0 OPERATORS CHAIR

68-75° F [20-24° C]
 5° F [3° C] per hour
 40-60 percent non condensing
 5 percent per hour
 N/A