





REVOLUTION DISCOVERY GSI/HD

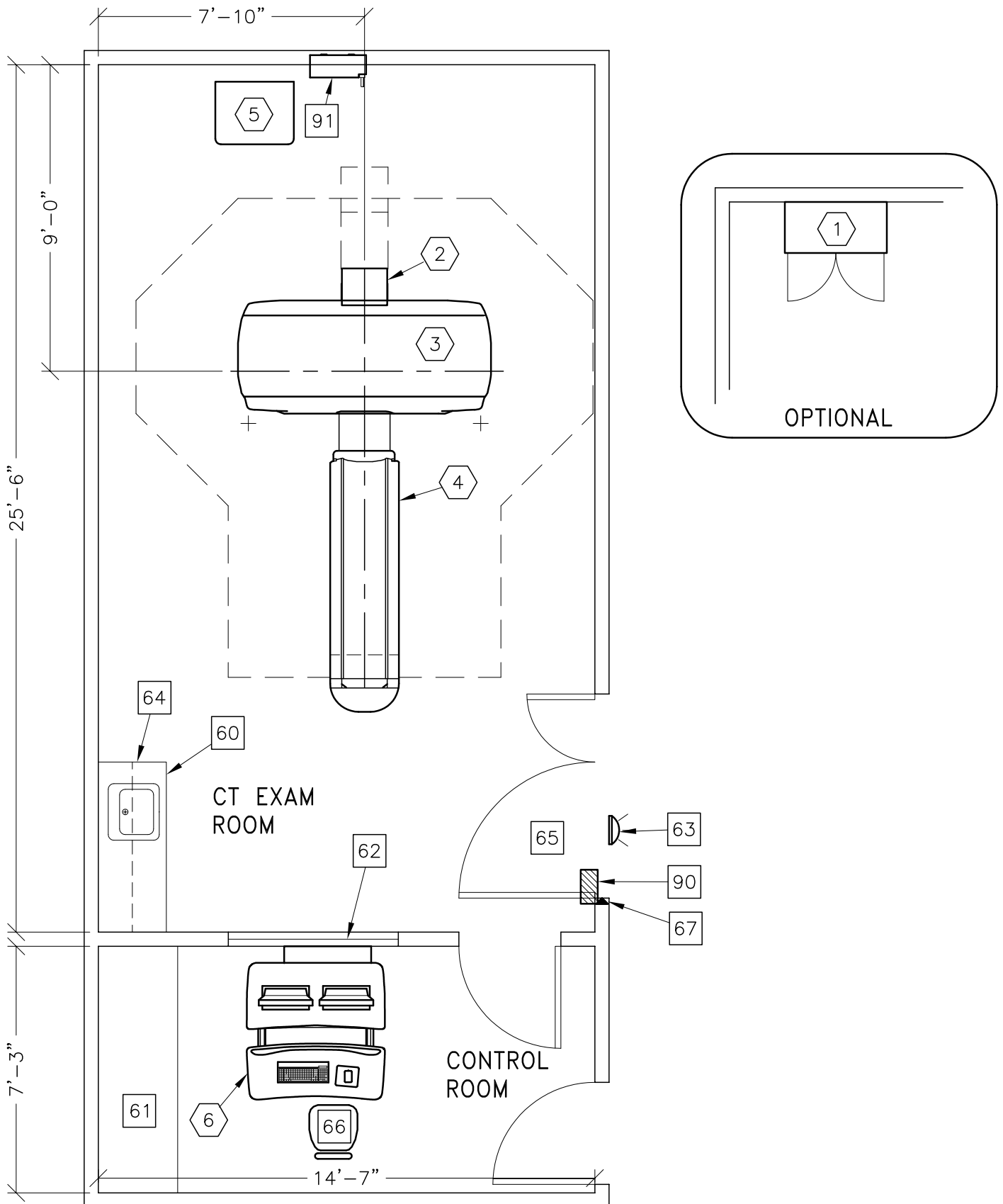

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL:	22'-0" x 14'-0" [6.71M x 4.27M]	9'-0" [2.74M]

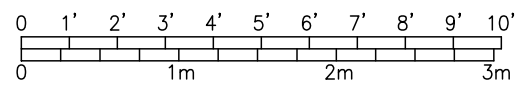

ROOM PLAN LOCATED ON PAGE 6-102

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	99 lbs	
②	1	REAR CABLE COVER		
③	1	REVOLUTION DISCOVERY CT/ DISCOVERY CT GANTRY <BTU'S INCLUDE GANTRY & TABLE>	3884 lbs	33297 btu
④	1	VT1700 PATIENT TABLE WITH EXTENDED TABLE TOP <W/O PATIENT>	1047 lbs	
⑤	1	POWER DISTRIBUTION UNIT	798 lbs	1706 btu
⑥	1	OPERATOR'S CONSOLE / COMPUTER WITH MONITORS	568 lbs	6000 btu
***** <b>CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS</b> *****				
⑥0		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
⑥1		COUNTER TOP WITH BASE CABINETS		
⑥2		LEAD GLASS WINDOW		
⑥3		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
⑥4		CASEWORK REQUIRED WITH A MINIMUM OF 16 CUBIC FEET TO STORE ALL SERVICE MATERIALS		
⑥5		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
⑥6		OPERATOR'S CHAIR		
⑥7		DOOR LIMIT SWITCH <REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		
91		MAIN DISCONNECT CONTROL GEMS CAT. NO. E4502AE 125 lbs., SEE DETAIL E4502AE.		

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F) 18-26° C  
 Allowable Temperature Change: 5° F [3° C] per hour  
 Humidity: 30-60 percent non condensing  
 Allowable Humidity Change: 5 percent per hour  
 Max - Altitude: 7875 [2400M] ft. above sea level