



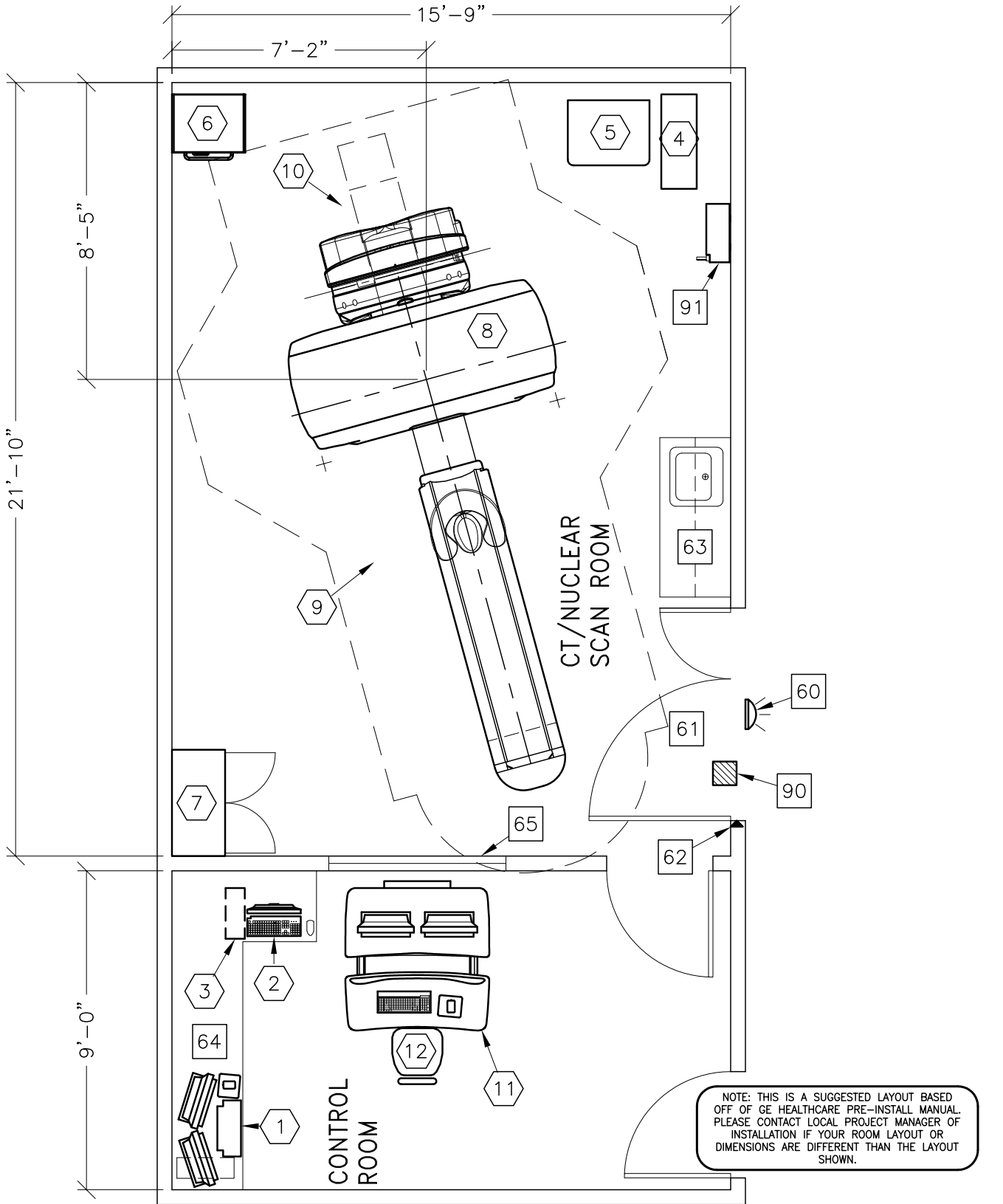

## DISCOVERY NMCT570

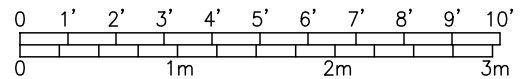

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	Refer to Pre-Installation Manual	



SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 7-72

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	XELERIS WORKSTATION	55 lbs	255 btu
②	1	OPERATORS CONSOLE-DESK MOUNTED	99 lbs	273 btu
③	1	ACQUISITION COMPUTER	33 lbs	508 btu
④	1	UPS SYSTEM	619 lbs	5122 btu
⑤	1	POWER DISTRIBUTION UNIT	701 lbs	3402 btu
⑥	1	INTEGRATED POWER SUPPLY	171 lbs	204 btu
⑦	1	STORAGE CABINET <EMPTY CABINET WEIGHT>	99 lbs	
⑧	1	GANTRY <CT>	4047 lbs	43010 btu
⑨	1	GT2000 PATIENT TABLE WITH EXTENDED TABLE TOP	1250 lbs	703 btu
⑩	1	GANTRY <NUCLEAR>	1430 lbs	3170 btu
⑪	1	OPERATOR'S CONSOLE / COMPUTER	493 lbs	8826 btu
⑫	1	OPERATOR'S CHAIR		
***** <b>CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS</b> *****				
60		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760, GE CAT. NO. WXIABWW-0F-XIU		
61		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
62		DOOR LIMIT SWITCH <REQUIRED IN, SOUTH CAROLINA OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES>		
63		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
64		COUNTER TOP FOR EQUIPMENT-PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP		
65		CONTROL WALL TO CEILING WITH LEAD GLASS WINDOW		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL, REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION-E4502RL FOR WARNING LIGHT CONTROL ONLY		
91		MAIN DISCONNECT CONTROL, GEMS CAT. NO. E4502AE, 125 lbs., SEE DETAIL E4502AE.		

ENVIRONMENT

Ambient Operation Temperature: 64-79° F [18-26° C]  
 Allowable Temperature Change: 5° F [3° C] per hour  
 Humidity: 40-60 percent non condensing  
 Allowable Humidity Change: 10 percent per hour  
 Max - Altitude: 8000 FT. [2438 M]