

Venue 50 R4

Site Preparation


Physical Dimensions and Weight :

Please Note: The picture may vary according to order configuration.



VENUE 50 ;



Docking Cart :

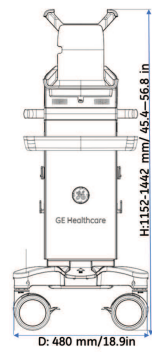

Docking Station :

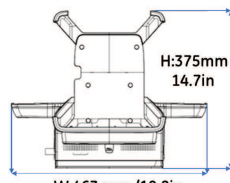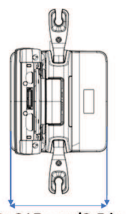

D:243mm/9.5 in


External Inputs and Outputs :



Environmental Requirements

Operating Temperature	3 to 40 C (37 to 104 F)
Operating Humidity	30 to 80 % rH, non-condensing , Pressure : 700 -1060 hPa
Heat Dissipation	3800 BTU/hour + 300 Btu for each person. Without lights, other equipment etc.
Storage Temperature	-5 to 50 C (23 to 122 F) ,
Storage Humidity	10 % - 90 % rH, non-condensing , Pressure : 700 -1060 hPa
Lighting	Combination lighting system (dim/bright) is recommended. Keep in mind that lighting controls and dimmers can be a source of EMI which could degrade image quality.

Recommended Ultrasound Room Layout 4.27 x 5.18 m (14 x 17 foot)

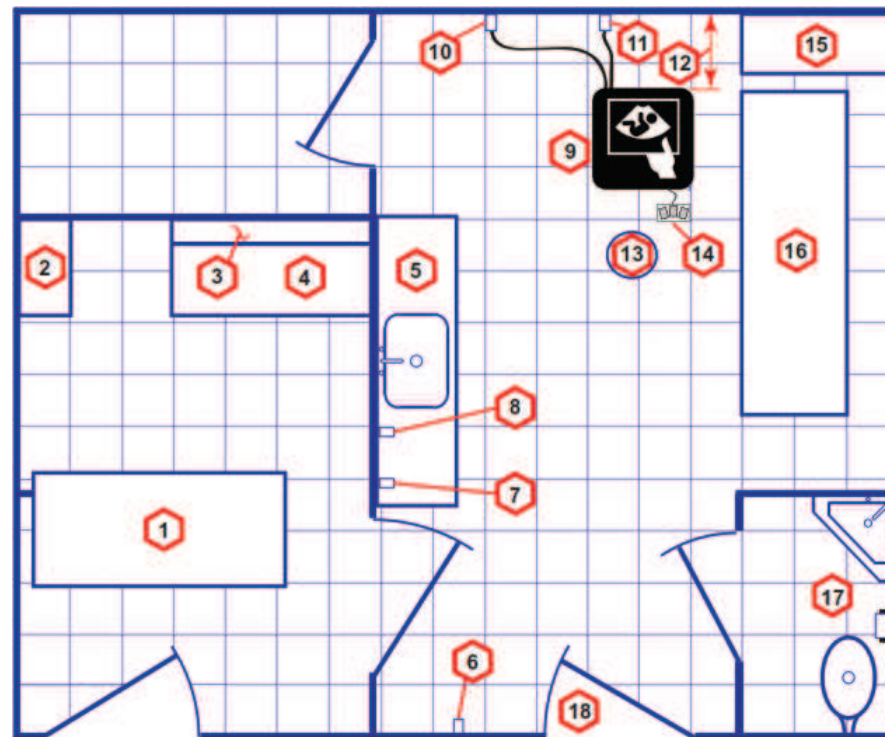

1. Secretaries or Doctors Desk
2. File Cabinet
3. Film Viewer
4. Counter Top
5. Counter Top and Sink with hot and cold water
6. Overhead Lights Dimmer - Dual Level Lighting (bright and dim)
7. Emergency Oxygen
8. Suction Line
9. Ultrasound system
10. Dedicated Power Outlet - Circuit Breaker protected and easily accessible
11. Network Interface
12. 457 mm (18 inches) distance of Ultrasound system from wall or objects
13. Stool
14. Footswitch
15. Storage for Linens and Equipment
16. Examination Table - 1930 x 610 mm (76 x 24 inches)
17. Lavatory and Dressing Room
18. Door - at least 762 mm (30 inches)

Electrical Specifications

Voltage	Venue 50 : 100-240 VAC ±10% 50/60 Hz , 1.6A Max Docking Cart: 100-240 VAC ±10% 50/60 Hz , 2.7A Max
Power Consumption	Venue 50 : Max 180 VA Docking Cart : Max 380 VA
Site Power Outlets	A dedicated AC power outlet must be within reach of the unit without extension cords
Power outage may occur	Requires a dedicated single branch circuit. To avoid circuit overload and possible loss of critical care equipment, make sure you DO NOT have any other equipment operating on the same circuit.
Acoustic Noise output	Less than 55 dB(A)

Please Note: For more information please refer to the product's service & user manuals.