



Mac-Lab/CardioLab Software Compatibility Matrix

Product Group: Interventional Invasive Products
Products: Mac-Lab IT/XT/XTi, CardioLab IT/XT/XTi, SpecialsLab and ComboLab IT/XT/XTi Recording Systems
Subject: Supported Software Versions
Date: May 10, 2017

Summary

This document details the latest software versions which are compatible with Mac-Lab/CardioLab. Where there are differences between the product manual and this document, this document supersedes information provided in the manual.

Software Family	Software Name	Software Versions	Mac-Lab/CardioLab Versions							
			6.5.4	6.5.6	6.8	6.8.1	6.9	6.9.5	6.9.6	
CVIS	DMS	3.02	X							
		4.01	X	X	X	X				
		4.1	X	X	X	X	X	X		
		4.2	X	X	X	X	X	X		
		4.2.10								X
		4.2.11								X
		4.2.13								X
		4.2.16	X	X	X	X	X	X	X	X
		4.3	X	X	X	X	X	X	X	X
		4.3.1*1	X	X	X	X	X	X	X	X
		4.3.2*2	X	X	X	X	X	X	X	X
	4.3.3	X	X	X	X	X	X	X	X	
	Carddas	2.02	X	X	X	X	X	X	X	
		2.03	X	X	X	X	X	X	X	
		2.04		X	X	X	X	X	X	
		2.12		X	X	X	X	X	X	
		2.13		X	X	X	X	X	X	
		2.2		X	X	X	X	X	X	
		2.3	X	X	X	X	X	X	X	
		2.5		X	X	X	X	X	X	
		2.6						X	X	
		3.1		X	X	X	X	X	X	
	3.5		X	X	X	X	X	X		
CCW	4.0		X	X	X					



Software Family	Software Name	Software Versions	Mac-Lab/CardioLab Versions						
			6.5.4	6.5.6	6.8	6.8.1	6.9	6.9.5	6.9.6
		5.0		X	X	X	X	X	
		5.0.6							X
		5.1							X
		5.1.2		X	X	X	X	X	X
		5.1.2.1		X	X	X	X	X	X
		6.0 SP5							X
Backup Software	Symantec Backup Exec for Windows Servers	12.5		X	X	X			
		2010					X		
		2010 R3						X	
		2012 SP3							X
		15 (2015)						X	
	Veritas Backup Exec	16							X
	CA ARCserve® Backup for Windows	r12			X				
		r12.5		X		X			
		r15					X		
		r16						X	
		16.5 SP1							X
		r17.5							X
	IBM Tivoli Storage Manager	5.5.1.0			X				
		6.1.0		X		X			
		6.2.0.0					X		
		6.3.0						X	
		6.4.0.1							X
		7.1.6.4							X
	EMC NetWorker	7.4 SP3			X				
		7.5 SP1		X		X			
		7.6					X		
		7.6 SP3						X	
		8.1							X
9.1								X	

*1: The ICD registry usage requires DMS 4.3.1 or higher

*2: The AFib registry usage requires DMS 4.3.2 or higher

Note: If installation of these software versions fail, follow Pre-Installation and Post Installation steps below:

Pre-Installation

1. Applicable only for 6.9.6 R2 INW server (G9 hardware), login as Administrator
2. Go to Run > regedit.exe.



3. Navigate to "HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing", set the registry key State to Hexadecimal value 23c00.

Post installation

1. Applicable only for 6.9.6 R2 INW server (G9 hardware), login as Administrator
2. Go to Run > regedit.exe.
3. Navigate to "HKEY_CURRENT_USER\Software\Microsoft\Windows\CurrentVersion\WinTrust\Trust Providers\Software Publishing", set the registry key State to Hexadecimal value 10000.