



MILLENNIUM MPR

RECOMMENDED CEILING HEIGHT
8'-0" [2438mm]

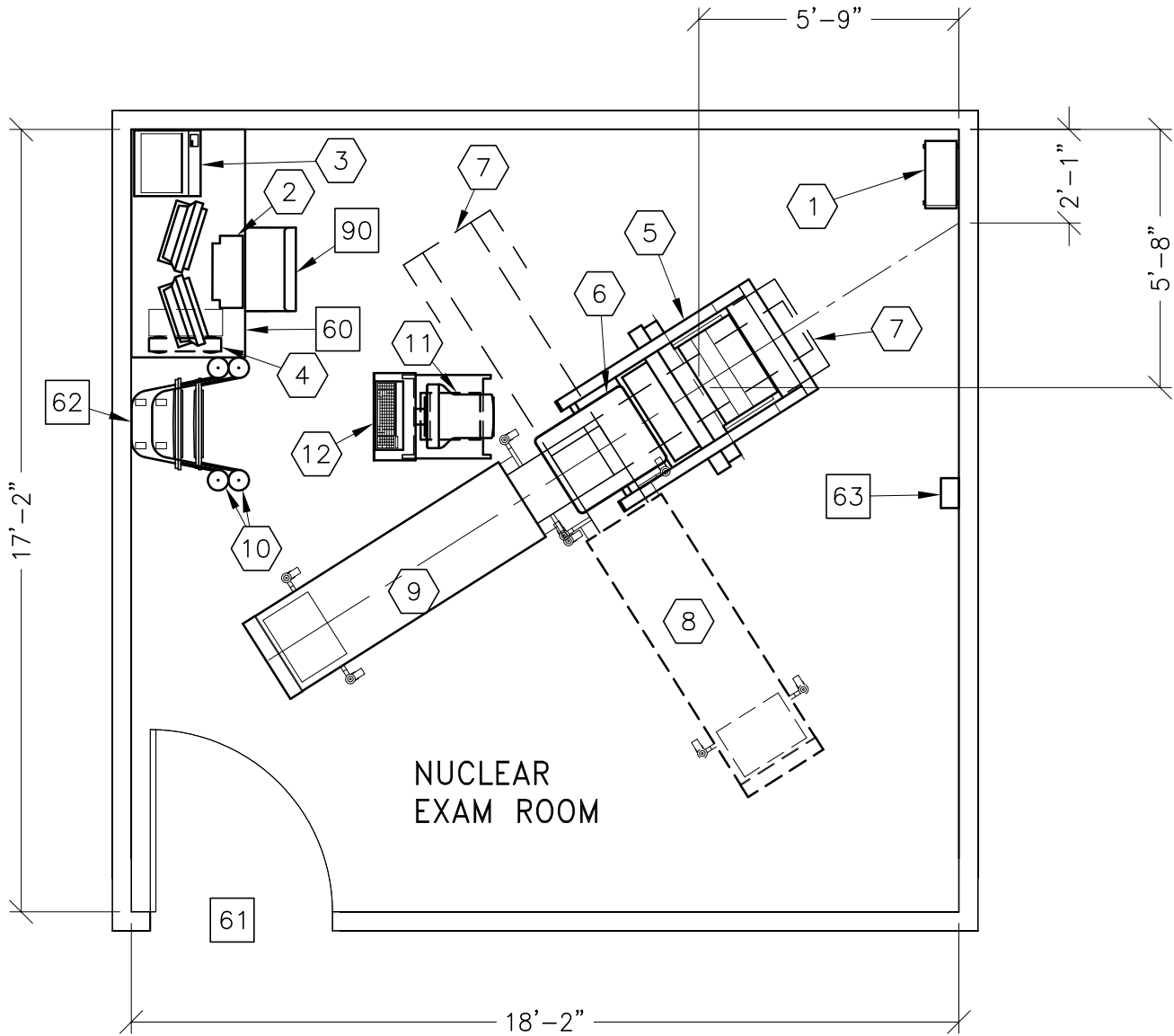

MINIMUM: 16'-0" x 16'-0" [4.87m x 4.87m] 8'-0" [2.44m]

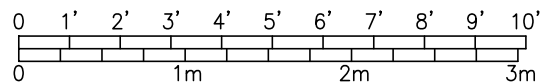

7-20

①	1 ACQUISITION COMPUTER	33 lbs	600 btu
②	1 XELERIS WORKSTATION	55 lbs	
③	1 COLOR PRINTER		
④	1 UPS SYSTEM	33 lbs	4436 btu
⑤	1 MILLENNIUM GANTRY WITH IPS	1895 lbs	2047 btu
⑥	1 RECTANGULAR HEAD DETECTOR	524 lbs	194 btu
⑦	1 LIMIT OF TABLE TRAVEL		
⑧	1 IMAGING TABLE (POSITION FOR WHOLE BODY SCAN)	804 lbs	
⑨	1 IMAGING TABLE (POSITION FOR TOMOGRAPHIC SCAN)	804 lbs	
⑩	2 MILLENNIUM COLLIMATOR STORAGE CART	346 lbs	
⑪	1 R-WAVE TRIGGER UNIT	19 lbs	170 btu
⑫	1 ACQUISITION MOBILE CART INCLUSIVE OF MONITOR AND KEYBOARD	105 lbs	34 btu

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

- ⑥0 TABLE
- ⑥1 MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 45 IN. W x 80 IN. H [1143mm x 2083mm], CONTINGENT ON A 84 IN. [2134mm] CORRIDOR WIDTH
- ⑥2 OPTIONAL WALL PROTECTION FROM COLLIMATOR CART. ALSO, FINISHED FLOORING COULD BE SUBJECT TO DAMAGE DURING MOVEMENT AND BEING PARKED FOR A LONG PERIOD. SUFFICIENT FLOORING MUST BE USED TO PREVENT DAMAGE.
- ⑥3 MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.
- ⑥0 OPERATORS CHAIR

59-81° F [15-27° C]
 5° F [3° C] per hour
 20-80 percent non condensing
 5 percent per hour
 N/A