


# INNOVA 3100-4100/OMEGA V/MACLAB

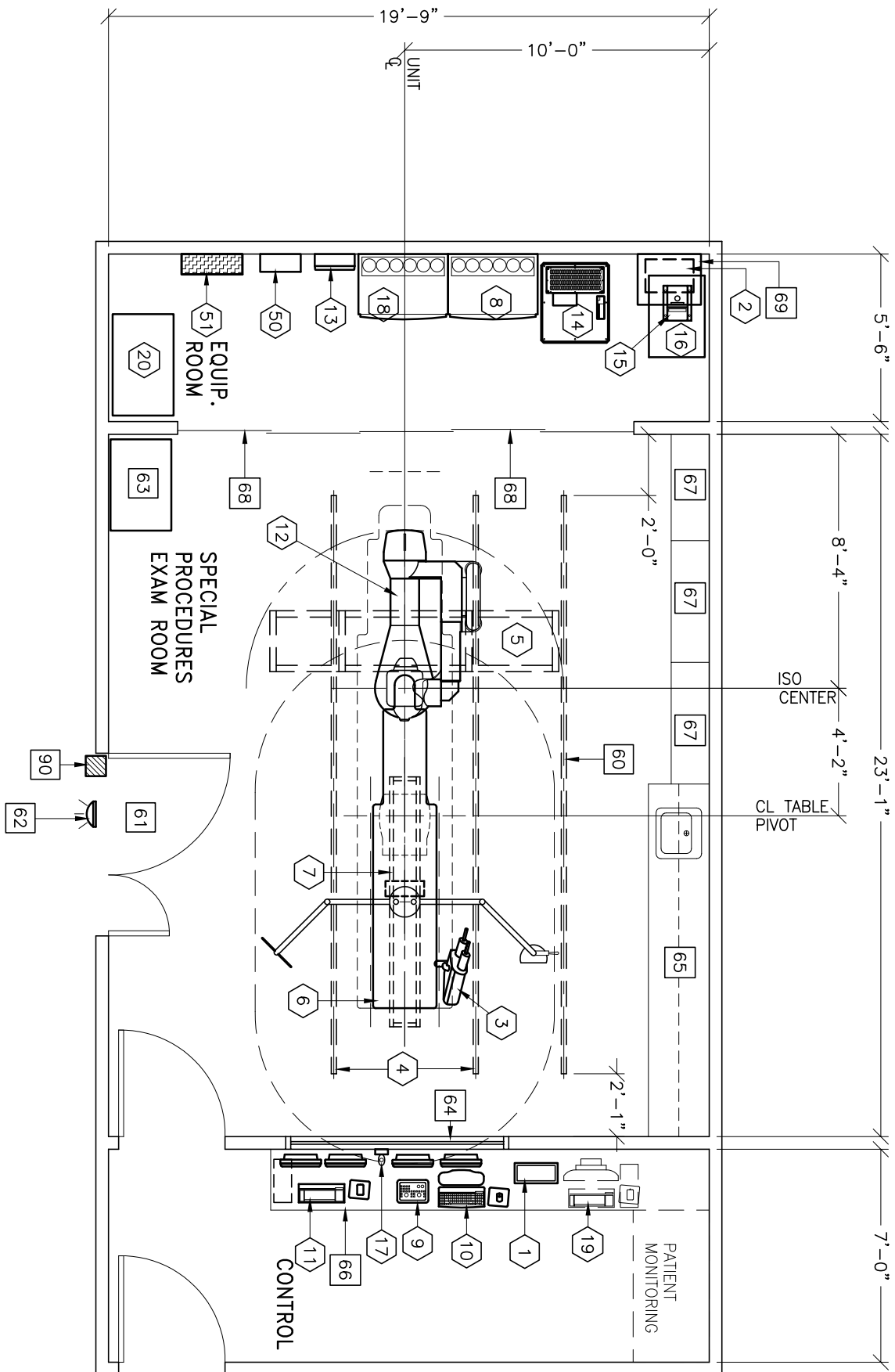

MINIMUM: 26'-0" x 20'-6" [7.92m x 6.24m] 9'-6" [2.89m]



## 4-58

①	1 REMOTE CONTROL FOR INJECTOR <OPTION>	4 lbs	
②	1 INJECTOR ELECTRONICS <OPTION>	37 lbs	320 btu
③	1 INJECTOR HEAD ON TABLE RAIL <OPTION>	15 lbs	
④	2 LONGITUDINAL STATIONARY RAIL FOR XT SUSPENSION	68 lbs	
⑤	1 SIX LCD MONITOR SUSPENSION ON 9 ft. 6 in. XT INBOARD BRIDGE	557 lbs	1228 btu
⑥	1 OMEGA IV/V TABLE WITH ROTATING TOP	1300 lbs	600 btu
⑦	1 COUNTERBALANCED EYE AND THYROID SHIELD WITH LAMP <OPTION>	167 lbs	150 btu
⑧	1 ATLAS CABINET <C2>	568 lbs	1825 btu
⑨	1 CONTROL ROOM MONITOR WITH DL KEYPAD	19 lbs	204 btu
⑩	1 OPERATORS CONSOLE	19 lbs	399 btu
⑪	1 AW WORKSTATION	81 lbs	1201 btu
⑫	1 INNOVA POSITIONER <REFERENCE TABLE BASE-PLATE DETAIL FOR FLOOR MOUNTING INFORMATION>	1653 lbs	2416 btu
⑬	1 UPS INTERFACE BOX		
⑭	1 UPS CABINET	1170 lbs	6750 btu
⑮	1 DETECTOR CHILLER	33 lbs	706 btu
⑯	1 WATER CHILLER	449 lbs	18716 btu
⑰	1 BOLUS CHASE HANDSWITCH <OPTION>	2 lbs	
⑱	1 ATLAS CABINET <C1>	998 lbs	3389 btu
⑲	1 IVUS VOLCANO READY KIT		
⑳	1 JEDI GENERATOR CART	50 lbs	
㉑	1 FILTER ENCLOSURE	90 lbs	
㉒	1 INNOVA MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION.	275 lbs	1532 btu

55-75° (F)  
15° (F) per hour  
30%-70% non condensing  
10% per hour  
8000 ft. above sea level

\*\*\*\*\*  
CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS  
\*\*\*\*\*

- 60 CABLE DRAPE RAIL, CAT. NO. CPGE55 OR EQUIVALENT.  
CONTACT UNISTRUT WISCONSIN: 262-796-8710
- 61 MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS  
44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT  
ON A 96 IN. [2438mm] CORRIDOR WIDTH
- 62 X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY  
CALL: 800-200-9760  
GE CAT. NO. WXIABWW-OF-XIU
- 63 CUSTOMER SUPPLIED  
STORAGE CABINET
- 64 CONTROL WALL TO CEILING WITH LEAD GLASS VIEWING  
WINDOW.
- 65 COUNTER TOP WITH SINK, BASE AND  
WALL CABINETS
- 66 COUNTER TOP FOR EQUIPMENT-  
SHELVING MAY BE REQUIRED  
PROVIDE GROMMETED OPENINGS AS  
REQUIRED TO ROUTE INTERCONNECT  
CABLES TO RACEWAY BELOW COUNTERTOP.
- 67 CATHETER CABINETS
- 68 SLIDING EQUIPMENT ROOM DOORS
- 69 SHELF FOR EQUIPMENT
- 90 X-RAY ROOM WARNING LIGHT/ROOM LIGHTING CONTROL PANEL  
REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR  
DETAILED DESCRIPTION -CAT. NO. E4500SS FOR WARNING  
LIGHT & ROOM LIGHT CONTROL.