





SENOGRAPHE DMR/SENOVISION/DATAFLASH

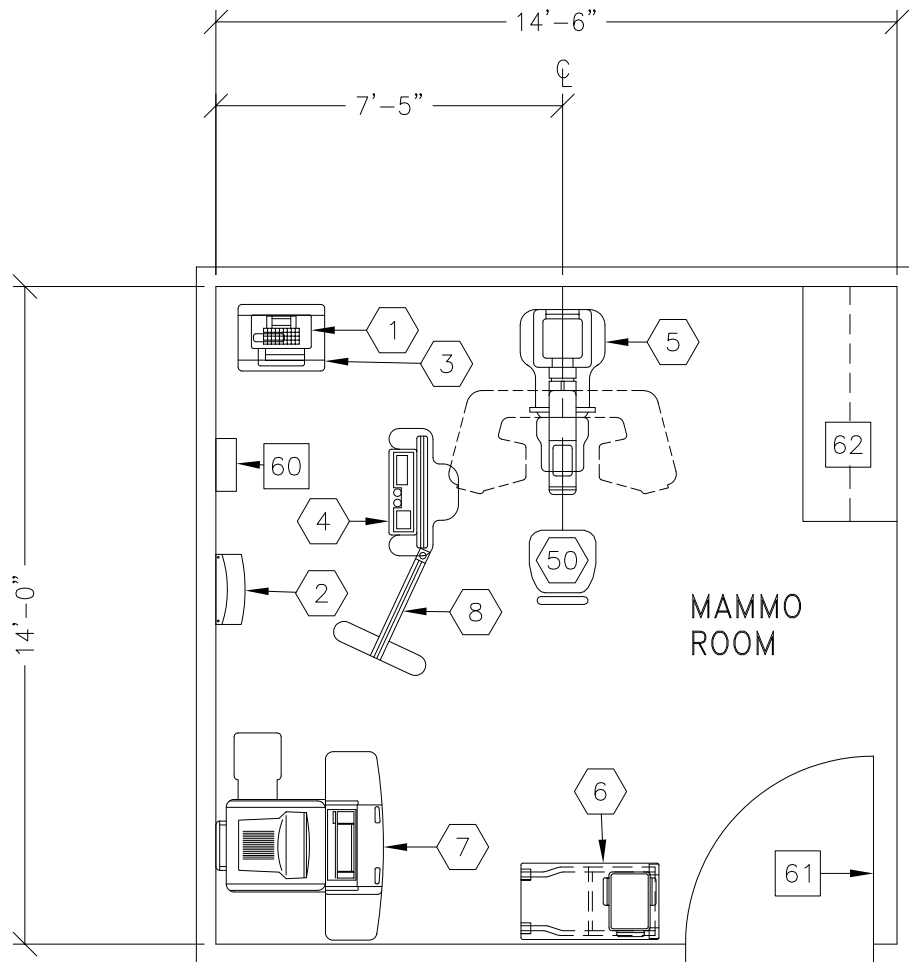

MINIMUM: 10'-0" x 12'-0" [3.5m x 3.7m] 7'-8" [2.3m]

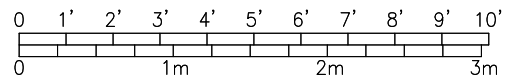

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT

SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 9-22

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	DATAFLASH RECORDER	24 lbs	
②	1	SENOVISION WALL BOX	33 lbs	
③	1	SENOGRAPHE DMR GENERATOR CABINET	231 lbs	
④	1	SENOGRAPHE DMR CONTROL CONSOLE WITH RAD SHIELD SCREEN	73 lbs	
⑤	1	SENOGRAPHE DMR MAMMOGRAPHY GANTRY	635 lbs	4952 btu
⑥	1	DMR ACCESSORY STORAGE RACK	110 lbs	
⑦	1	SENOVISION CART	485 lbs	1700 btu
⑧	1	SENOGRAPHE DMR ACCESSORY RADIATION SHIELD SCREEN	82 lbs	
⑤0	1	HYDRAULIC CHAIR		
<p>*****                      CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS                      *****</p>				
⑥0		MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET E1 FOR DETAILED DESCRIPTION		
⑥1		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 30 IN. W X 77 IN. H [762MM X 1956MM], CONTINGENT ON A 60 IN. [1524MM] CORRIDOR WIDTH		
⑥2		COUNTER TOP WITH BASE AND WALL CABINETS		

ENVIRONMENT

Ambient Operation Temperature: 50-75° F (10-24° C)  
 Allowable Temperature Change: 15° F (10° C) per hour  
 Humidity: 30-75 percent non condensing  
 Allowable Humidity Change: 10 percent per hour  
 Max - Altitude: 9500 ft. (29,000 M) above sea level