





LightSpeed RT Xtra with 2000 Table

NOTE: THIS IS A SUGGESTED LAYOUT BASED OFF OF GE HEALTHCARE PRE-INSTALL MANUAL. PLEASE CONTACT LOCAL PROJECT MANAGER OF INSTALLATION IF YOUR ROOM LAYOUT OR DIMENSIONS ARE DIFFERENT THAN THE LAYOUT SHOWN.

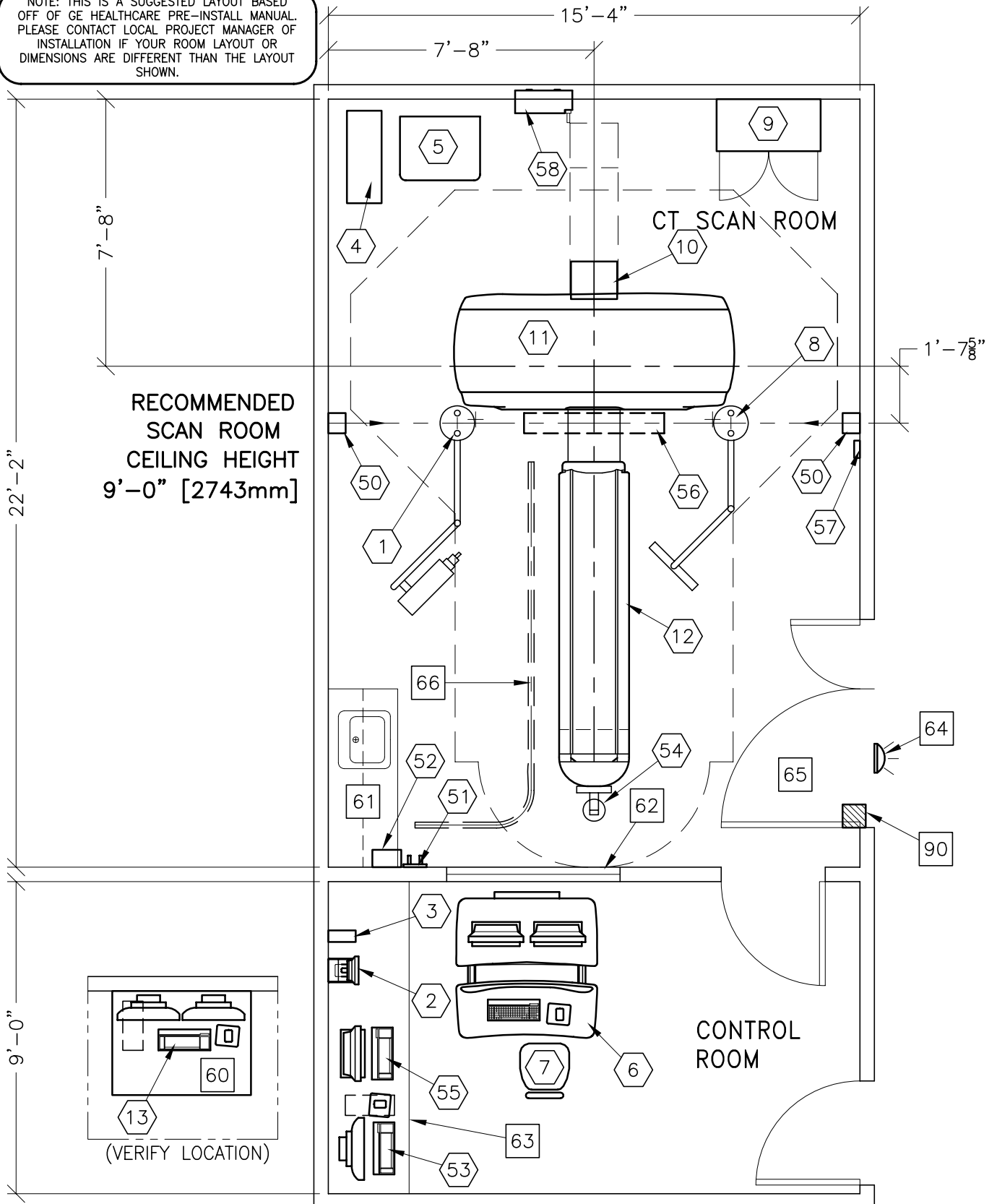

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	22-0" x 14'-0" [6.7M x 4.2M]	8'-0" [2.43M]

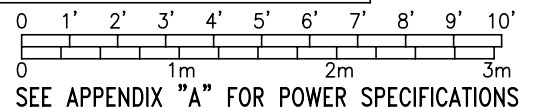


ROOM PLAN LOCATED ON PAGE 6-66

ITEM ①	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	INJECTOR HEAD ON OVERHEAD COUNTERPOISED SUSPENSION.	22 lbs	
②	1	INJECTOR CONTROL CONSOLE	8 lbs	
③	1	MAIN UNIT	13 lbs	
④	1	UPS SYSTEM	350 lbs	
⑤	1	POWER DISTRIBUTION UNIT	701 lbs	3402 btu
⑥	1	OPERATOR'S CONSOLE / COMPUTER	493 lbs	8126 btu
⑦	1	OPERATOR'S CHAIR		
⑧	1	LCD MONITOR	61 lbs	300 btu
⑨	1	STORAGE CABINET (EMPTY CABINET WEIGHT)	90 lbs	
⑩	1	REAR CABLE COVER		
⑪	1	LIGHTSPEED RT16/XTRA	3891 lbs	43010 btu
⑫	1	GT2000 PATIENT TABLE WITH EXTENDED TABLE TOP.	1029 lbs	703 btu
⑬	1	ADVANTAGE WORKSTATION WITH TWO LCD MONITORS	81 lbs	1109 btu
⑤①	2	PATIENT POSITIONING LASER LIGHT	66 lbs	
⑤②	1	VARIAN CAMERA HOLDER (WHEN CAMERA IS IN STANDBY POSITION)		
⑤③	1	VARIAN POWER SUPPLY MODULE (GE PROJ MGR: PREINSTALL ITEM, PRE-ORDER FROM VARIAN)	13 lbs	
⑤④	1	VARIAN OPERATORS WORKSTATION	48 lbs	819 btu
⑤⑤	1	VARIAN CAMERA AND BRACKET	8 lbs	341 btu
⑤⑥	1	WORKSTATION COMPUTER		
⑤⑦	1	PATIENT POSITIONING LASER LIGHT	66 lbs	
⑤⑧	1	KEYPAD	2 lbs	
⑤⑨	1	MAIN DISCONNECT CONTROL	132 lbs	

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F)
 Allowable Temperature Change: 5° F [3° C] per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 8000 ft. above sea level

ROOM PLAN LOCATED ON PAGE

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
				
<p>*****</p>				
<p>CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS</p>				
<p>*****</p>				
<p>[60]</p>		<p>WORKSTATION TABLE</p>		
<p>[61]</p>		<p>COUNTER TOP WITH SINK, BASE AND WALL CABINETS</p>		
<p>[62]</p>		<p>LEAD GLASS WINDOW</p>		
<p>[63]</p>		<p>COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.</p>		
<p>[64]</p>		<p>X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABW-0F-XIU</p>		
<p>[65]</p>		<p>MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH</p>		
<p>[66]</p>		<p>CURTAIN RAIL SUSPENDED FROM FALSE CEILING FOR VARIAN CABLE</p>		
<p>[90]</p>		<p>X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.</p>		