

Connect for a smoother workflow and greater efficiency

Centricity Perinatal – Connect

Reviewing data from different hospital systems or paper records to make critical care decisions for a mother and baby can be both inefficient and inconvenient.

Centricity Perinatal – Connect lets you view multiple fetal monitoring strips and have access to other hospital systems—all at the same time.



- 1 Dock for accessing Centricity Perinatal – Connect
- 2 Expanded view of patient's record
- 3 Historical shade can display a condensed view of entire pregnancy
- 4 Time Continuum for viewing annotations
- 5 Compact views of current fetal monitoring strips
- 6 Patient Directory button to select which patients' data to view

healthymagination



imagination at work

An add-on module to Centricity Perinatal, Connect aggregates and correlates data from your Hospital Information System (HIS) and other sources onto the fetal strip. Everything you need to know is conveniently displayed on one screen. You can make fast, more informed clinical decisions, enhancing your workflow and patient care.

- Review multiple fetal monitoring strips without obstructing other clinical data
- Fully integrates with major Hospital Information Systems
- Log into multiple products simultaneously with user context sharing
- Optimize usability with a solution that leverages clinical input and evidence-based research

Key Features

- The Centricity Perinatal – Connect Dock lets you select patients from the Patient Directory and displays compact views of each patient's fetal monitor strip.
- Pin or unpin the application to customize your view. Pinning Connect fixes the Dock to your desktop and resizes open applications in the remaining space. Unpinning Connect displays just a small application tab.
- Tool tips show you patient names and event times so you know exactly when an event happened and to whom.
- Historical Shade initially displays up to 72 hours of patient data and can display a condensed view of the entire pregnancy. Live data continues to stream in the expanded view.
- Annotations are displayed in the same minute in which they are displayed on the patient record. Time Continuum aggregates and displays data from multiple sources on a rolling timeline for correlation with the fetal waveform.

Applications

- Monitor your patients, even when you are performing other tasks.
- See information in one place at one time—without searching other hospital systems or paper files.
- Access information as you need it—view the whole strip or focus on just one short time segment.
- Review annotations charted in other systems on the fetal strip, whether or not a fetal monitor is connected.
- See trends by reviewing compressed data of the current and historical fetal strip on one display.



GE Healthcare
540 West Northwest Highway
Barrington, IL 60010
U.S.A.
www.gehealthcare.com



imagination at work

©2011 General Electric Company – All rights reserved.

General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation.

Centricity Perinatal does not replace clinical observation and evaluation of the patient at regular intervals, by a qualified care provider, who will make diagnoses and decide on treatments or interventions. Features of the Centricity Perinatal system are intended to support clinical decision making and should be used in combination with other clinical inputs, such as real time patient observation and information contained within other systems or recording tools. Not intended to be used as a primary monitoring device.

*GE, the GE Monogram, Centricity, and imagination at work are trademarks of General Electric Company.

GE Healthcare, a division of General Electric Company.